

Posey incontinence products are designed for use with patients/clients who are at risk of incontinence episodes. The quilted polyester laminate multi-layer construction features a white non-woven absorption liner, a waterproof urethane inner layer and a woven reinforcing blue bottom layer protect both the patient's skin and bedding/cushions/furniture from urine spills. Machine Washable. Meets CA #117 flame retardancy standards.



## MONITORING

**AWARNING** Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity and proper circulation.

## INSPECTION

**AWARNING** Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

## INCONTINENCE AND TURN PAD

The Posey Incontinence and Turn Pad will absorb up to 1,000cc, or 4 cups of fluid. The multiple layer pad is machine washable and more cost-effective than disposables. Nylon handles along the length of the pad help the caregiver to quickly and safely turn or reposition the patient. Sold in packages of six pads. The load capacity is 220 lbs. (100 kgs.).

**REF 8145** Incontinence Pad (6 pk.) 35½"L x 35½"W (90 cm x 90 cm)

**AWARNING** DO NOT use this product as a patient transfer device, it is not designed to hold the patient's weight during transfer. Patient may fall or severe injury may occur.

## APPLICATION INSTRUCTIONS

Place the pad on the bed, underneath a patient with the blue side down. Handles should be oriented along the side of the mattress.



## WHEELCHAIR CUSHION INCONTINENCE PAD

The Posey Wheelchair Cushion Incontinence Pad will absorb up to 300cc, or 10 oz. of fluid. Non-latex elastic straps at all four corners secure the pad to the cushion.

**REF 6303** Incontinence Pad (Each) Fits cushions up to 20" W x 18" D x 2" H (51 cm x 46 cm x 5 cm)

## APPLICATION INSTRUCTIONS

Place on a cushion, blue side down, and secure to the cushion using the four corner straps (Fig. 1). Pad can also be used without a cushion by simply placing it underneath the individual.



## LAUNDERING INSTRUCTIONS:

Wash hot, bleach as directed, dry on low.



## DISPOSAL

**AWARNING** Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

## STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

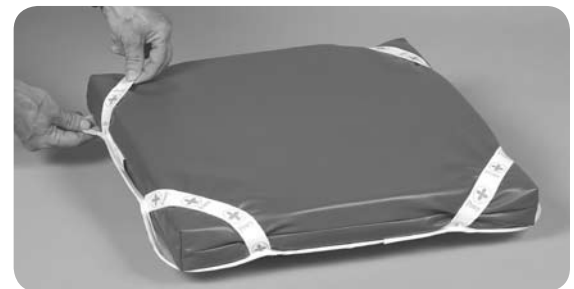


Fig. 1 - Attaching corner straps to cushion.