




DeRoyal Sports Ankle Brace 2

Single Patient Use Only

SIZE	PRODUCT #	CIRCUMFERENCE
X-Small	15600004	10"-11"
Small	15600005	11"-12"
Medium	15600006	12"-13"
Large	15600007	13"-14"
X-Large	15600008	14"-15"
XX-Large	15600009	15"-16"

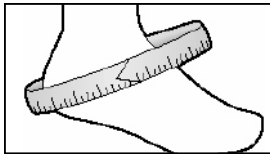

*Measure circumference from the base of the heel around the ankle.

IMPORTANT INFORMATION

Please read all instructions before use. Correct application is essential to proper functioning of product.

INTENDED USE

The universal stabilization strap can be directed to the inside or the outside of the ankle joint. The directional stabilization strap provides external support over the injury site, creates compression and forces the heel into inversion or eversion (depending on the nature of the injury).

INDICATIONS



Chronic medial and lateral ankle instability; Assist in prevention or treatment of acute ankle sprains

PRECAUTIONS

Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or prolonged pain while using this product.

DIRECTIONS FOR USE


1. Place foot with the heel to the back of the brace and position the tongue between the top of the foot and laces (**Note:** brace should be worn with an athletic sock).
2. Lace up ankle brace from the bottom to top and tie laces together.
3. Detach "Y-tab" from the end of nylon strap.
4. Feed the nylon strap under the cuff closure to the side of the ankle that needs support.

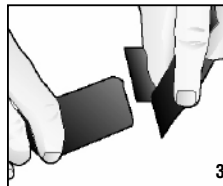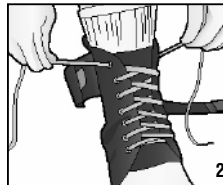

DeRoyal Sports Ankle Brace 2

Single Patient Use Only

SIZE	PRODUCT #	CIRCUMFERENCE
X-Small	15600004	10"-11"
Small	15600005	11"-12"
Medium	15600006	12"-13"
Large	15600007	13"-14"
X-Large	15600008	14"-15"
XX-Large	15600009	15"-16"

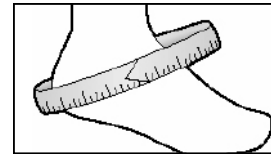

*Measure circumference from the base of the heel around the ankle.

IMPORTANT INFORMATION

Please read all instructions before use. Correct application is essential to proper functioning of product.

INTENDED USE

The universal stabilization strap can be directed to the inside or the outside of the ankle joint. The directional stabilization strap provides external support over the injury site, creates compression and forces the heel into inversion or eversion (depending on the nature of the injury).

INDICATIONS



Chronic medial and lateral ankle instability; Assist in prevention or treatment of acute ankle sprains

PRECAUTIONS

Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or prolonged pain while using this product.


DIRECTIONS FOR USE

1. Place foot with the heel to the back of the brace and position the tongue between the top of the foot and laces (**Note:** brace should be worn with an athletic sock).
2. Lace up ankle brace from the bottom to top and tie laces together.
3. Detach "Y-tab" from the end of nylon strap.
4. Feed the nylon strap under the cuff closure to the side of the ankle that needs support.

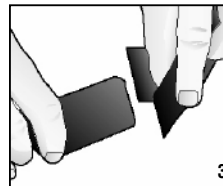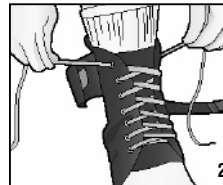




5. Reattach the "Y-tab" to the end of the strap (**Note:** Hook should be facing the body of brace to ensure proper attachment).

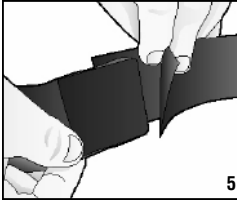



6. Bring stabilizing strap across the top of the foot, through the arch and under the heel.



7. Secure the hook to the body of brace (snug).

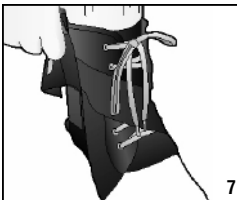

8. Secure the elastic cuff closed over laces and strap. DeRoyal logo should be centered to the front of brace to ensure proper compression.

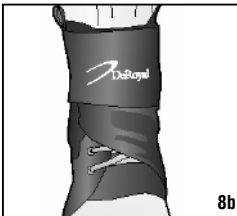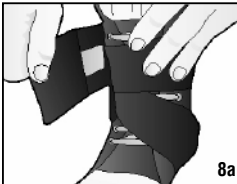

CARE INSTRUCTIONS

Hand wash in lukewarm water and air dry.



Manufacturer
DeRoyal
200 DeBusk Lane, Powell, TN 37849 U.S.A.
888.938.7828
865.938.7828

Authorized Representative
DeRoyal Europe, Ltd.
Virginia Road, Kells, County Meath Ireland
353.46.9280100


©2004 DeRoyal. All Rights Reserved.
Part #0-1801 Revision A
Revised 4/04




5. Reattach the "Y-tab" to the end of the strap (**Note:** Hook should be facing the body of brace to ensure proper attachment).



6. Bring stabilizing strap across the top of the foot, through the arch and under the heel.

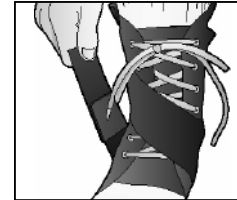

7. Secure the hook to the body of brace (snug).

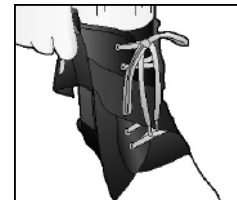

8. Secure the elastic cuff closed over laces and strap. DeRoyal logo should be centered to the front of brace to ensure proper compression.



CARE INSTRUCTIONS

Hand wash in lukewarm water and air dry.



Manufacturer
DeRoyal
200 DeBusk Lane, Powell, TN 37849 U.S.A.
888.938.7828
865.938.7828

Authorized Representative
DeRoyal Europe, Ltd.
Virginia Road, Kells, County Meath Ireland
353.46.9280100

©2004 DeRoyal. All Rights Reserved.
Part #0-1801 Revision A
Revised 4/04