

BIRD & CRONIN[®] INC.

Improving lives with quality medical products

Stack Finger Splint Kit

30-Piece Assortment

Clear Color Reorder No.	Flesh Color Reorder No.	Splint Size	DIP Circumference	Kit Quantity	Reorder Quantity
0814 2100	0814 2226	Assorted	--	--	KIT
0814 2101	0814 2001	1	5.0 cm or less	3 EA*	3/PK
0814 2102	0814 2002	2	5.0 cm to 5.5 cm	3 EA*	3/PK
0814 2103	0814 2003	3	5.5 cm to 5.8 cm	3 EA*	3/PK
0814 2104	0814 2004	4	5.8 cm to 6.3 cm	5 EA*	3/PK
0814 2105	0814 2005	5	6.3 cm to 6.8 cm	4 EA*	3/PK
0814 2155	0814 2055	5.5	6.8 cm to 7.1 cm	4 EA*	3/PK
0814 2106	0814 2006	6	7.1 cm to 7.3 cm	4 EA*	3/PK
0814 2107	0814 2007	7	7.3 cm to 8.4 cm	4 EA*	3/PK


* Kit contains clear or flesh colored splints based on style ordered

**NOT MADE WITH
natural rubber latex**

BIRD & CRONIN[®] INC.

Improving lives with quality medical products

Stack Finger Splint Kit

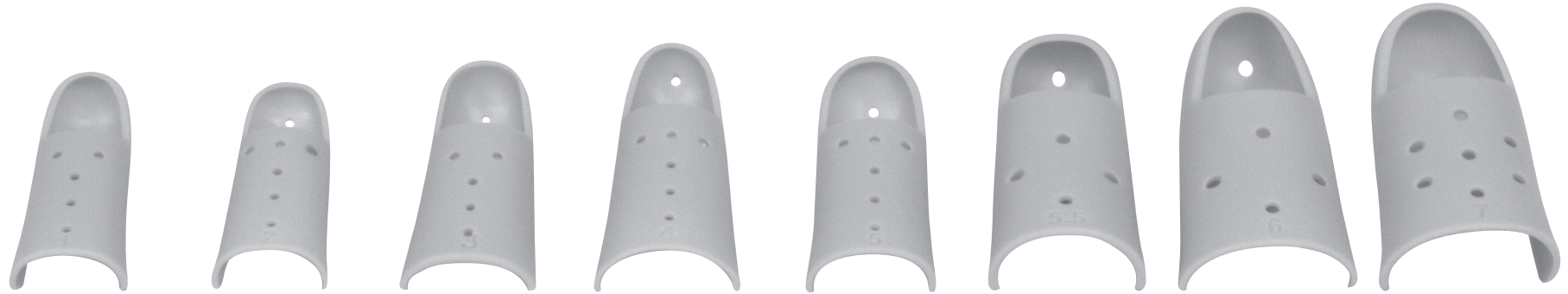

Size 1

Clear: 0814 2101
Flesh: 0814 2001

Size 2

Clear: 0814 2102
Flesh: 0814 2002

Size 3

Clear: 0814 2103
Flesh: 0814 2003

Size 4

Clear: 0814 2104
Flesh: 0814 2004

Size 5

Clear: 0814 2105
Flesh: 0814 2005

Size 5.5

Clear: 0814 2155
Flesh: 0814 2055

Size 6

Clear: 0814 2106
Flesh: 0814 2006

Size 7

Clear: 0814 2107
Flesh: 0814 2007

* Stack Finger Splints are ideally suited for the repair of ruptured tendons. They are designed to support the distal joint of the finger in extension while permitting unrestricted movement of the proximal interphalangeal joint. The Splint is normally used without padding and is fixed to the middle phalanx with a small strip of adhesive tape, thus automatically elevating the distal joint to its proper extended position.

Occasionally, the dorsum of the finger may be too tender to tolerate a normal size Stack Splint. A much larger size, padded on the inside, should be used until the swelling has subsided enough to replace it with a Splint of the proper size.

*H. Graham Stack, F.R.C.S.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.