

# User Manual



**DEALER:** This manual MUST be given to the user of the product.

**USER:** BEFORE using this product, read this manual and save for future reference.

Yes, you can.

Invacare<sup>®</sup> Matrix<sup>®</sup> PS



Model Number PS-V



---

## I General

---

### I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.



#### **WARNING**

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

#### **CAUTION**

Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.



Gives useful tips, recommendations and information for efficient, trouble-free use.

### I.2 Overview

#### **Important Information**

The best way to avoid problems related to pressure sores is to understand their causes and your role in a skin management program.

Your therapist and physician should be consulted if you have questions regarding individual limitations and needs. All cushions should be selected carefully. Working with your therapist and physician is the best way to assure that a cushion choice matches your individual needs. As the needs of the individual become more complex, cushion evaluation becomes more important.

#### **Invacare Matrix PS**

The Invacare Matrix PS cushion features an anatomically contoured shape with a unique waffled ischial relief recess and an innovative reversible outer cover.

#### **Outer Cover and Inner Cover**

The outer cover utilizes a unique reversible Dartex™ outer cover, which provides two unique cover options to satisfy your needs in terms of comfort and aesthetics. This specially designed, reversible stretch fabric outer cover may be installed with either the knitted surface exposed or with the smooth surface exposed. The outer cover is moisture resistant and breathable.

The inner cover is moisture resistant to protect the foam. Regular cleaning and inspection of the covers is recommended. Refer to Care and Maintenance on page 3.

---

## 2 Safety

---



Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

### 2.1 Intended Use

The Invacare Matrix PS is designed to provide superior positioning, stability and comfort.

## 2.2 General Warnings



### WARNING

DO NOT USE THIS PRODUCT OR ANY AVAILABLE OPTIONAL EQUIPMENT WITHOUT FIRST COMPLETELY READING AND UNDERSTANDING THESE INSTRUCTIONS AND ANY ADDITIONAL INSTRUCTIONAL MATERIAL SUCH AS OWNER'S MANUALS, SERVICE MANUALS OR INSTRUCTION SHEETS SUPPLIED WITH THIS PRODUCT OR OPTIONAL EQUIPMENT. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS OR INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNEL BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE, INJURY OR DAMAGE MAY OCCUR.

### ACCESSORIES WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

### INSTALLATION WARNING

After any adjustments, repair or service and before use, make sure that all attaching component parts are secure.

Ensure the seat cushion is properly secured to the wheelchair before using. Otherwise injury or damage may occur.

Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to become unstable, potentially resulting in injury.

### WARNING

Skin condition should be checked very frequently after the installation of any new seating system. Your therapist and physician should be consulted if you have any questions regarding individual limitations and needs.

Working with your therapist, physician, and equipment supplier is the best way to assure that a seating choice matches your individual needs.

As the needs of the individual become more complex, the seating evaluation becomes more important.

The cushion is designed to be used only at room temperature. If it is left at lower temperatures, always allow it to warm to room temperature before using. Sitting on extremely cold or hot surfaces can cause skin damage.

Avoid sharp objects or exposure to excessive heat or open flame.

## 2.3 Weight Limitation and Model Numbers



Refer to the chart for the weight limitation and stock model numbers.

Configured model numbers are: PS-V

Grey = size not available

WIDTH	DEPTH					WEIGHT LIMIT*
	12 IN (30 CM)	14 IN (36 CM)	16 IN (41 CM)	18 IN (46 CM)	20 IN (51 CM)	
12 in (30 cm)	PS1212	PS1214	PS1216			150 lbs (68 kg)
14 in (36 cm)	PS1412	PS1414	PS1416			
16 in (41 cm)			PS1616	PS1618	PS1620	300 lbs (136 kg)
18 in (46 cm)			PS1816	PS1818	PS1820	
20 in (51 cm)			PS2016	PS2018	PS2020	

\*Matrx PS Heavy Duty models are custom configured and have a weight limitation of 600 lbs (272 kg). These cushions are available in sizes ranging from 21 to 30 inches wide and 16 to 30 inches deep.

---

## 3 Usage

---

### 3.1 Installing the Invacare Matrix PS

1. Ensure the desired side of the outer cover is exposed. Refer to [Reversing the Outer Cover](#) on page 3.
2. Position the cushion so the material identification tags (not shown) face the rear.
3. Align the hook and loop strips (not shown) on the bottom of the cushion with those on the wheelchair seat.
4. Position the cushion onto the wheelchair as far back as possible.
5. Press down to ensure the hook and loop strips are firmly engaged.



### 3.2 Reversing the Outer Cover

1. Unzip the outer cushion cover.
2. Remove the outer cushion cover.
3. Turn the outer cushion cover inside out to the desired side.
4. Install the cushion with inner cover into the outer cover with the waffled recess facing down.

---

## 4 Care and Maintenance

---

### 4.1 Cleaning

#### Outer Cover

The cover can be wiped clean with a damp towel.

The cover can be removed for laundering. Unzip the cover and carefully remove the foam with inner cover. The zipper should be closed prior to laundering.

Dry clean or machine wash using the delicate cycle and hang dry or tumble dry on low.

#### Inner Cover

The inner cover can be wiped clean with a damp cloth. The inner cover should not be removed from the foam.

### 4.2 Inspection



#### **WARNING**

DO NOT continue to use this product if problems are discovered. Corrective maintenance can be performed at or arranged through your equipment supplier.

Visually inspect parts including upholstery materials, foams (if accessible), and plastics, for deformation, corrosion, breakage, wear or compression.

---

## 5 After Use

---

### 5.1 Reuse



#### **WARNING: Risk of Injury**

Always have a dealer inspect the product for damage before transferring the product to a different user. If any damage is found, DO NOT use the product. Otherwise injury may occur.

# 6 Warranty

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare/Motion Concepts warrants this product to be free from defects in materials and workmanship for two years of use by original purchaser. This warranty does not apply to punctures, tears or burns, nor to the removable cushion cover. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's/Motion Concepts' option, with refurbished or new parts. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Product repairs shall not extend this warranty - coverage for repaired product shall end when this limited warranty terminates. Invacare's/Motion Concepts' sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare/Motion Concepts product. In the event you do not receive satisfactory warranty service, please write directly to Invacare/Motion Concepts at the address on the back cover. Provide dealer's name, address, model number, the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation/Motion Concepts will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S/MOTION CONCEPTS' EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE/MOTION CONCEPTS; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S/MOTION CONCEPTS' CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE/MOTION CONCEPTS DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE/MOTION CONCEPTS.**

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE/MOTION CONCEPTS SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

**Invacare**

[www.invacare.com](http://www.invacare.com)



**Yes, you can.**

## USA

One Invacare Way  
Elyria, Ohio USA  
44036-2125  
440-329-6000  
800-333-6900

## Canada

570 Matheson Blvd. E Unit 8  
Mississauga Ontario  
L4Z 4G4 Canada  
800-668-5324

**Motion Concepts**

[www.motionconcepts.com](http://www.motionconcepts.com)



## USA

700 Ensminger Rd. Suite 112  
Tonawanda, NY 14150  
888-433-6818

## Canada

84 Citation Dr.  
Concord, Ontario L4K 3C1  
905-695-0134  
866-748-7943



© 2011 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted. Dartex is a trademark of Dartex Coatings Limited Corporation UK.

Part No 1171735

Rev A - 02/11

