



MJM International Corporation

118-3-KD

Assembly Instructions


"We Do Care - We Do More"

IDENTIFY COMPONENTS TO START ASSEMBLY



- 1 SEAT SECTION
- 1 BASE SECTION.
- 2 ARM REST.
- 1 PVC BACKREST WITH MESH SLING.
- 4 VERTICAL HEIGHTS.
- 12 CLAMP LOCKING SYSTEM.
- BOX WITH CASTERS AND SCREWS
- 2 CASTERS WITH BRAKES. 2 CASTERS NON BRAKE.

STEP 1 – CASTERS

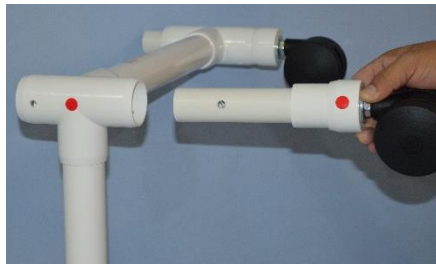

- INSERT CASTER LEG STEMS INTO BASE SECTION
- SECURE USING 1/2" RETAINING SCREWS AND PILOT HOLES AS REFERENCE.

NOTE: A PHILLIPS SCREWDRIVER OR A CORDLESS HAND DRILL IS REQUIRED TO TIGHTEN / SECURE ALL SCREWS.

STEP 2 HEIGHTS



- INSERT VERTICAL HEIGHTS INTO BASE SECTION MATCH COLOR CODED DOTS.
- SECURE HEIGHTS WITH CLAMP LOCKS.



STEP 3 SEAT SECTION



- INSERT SEAT SECTION ONTO VERTICAL HEIGHTS MATCH COLOR CODED DOTS.
- SECURE SEAT SECTION WITH CLAMP LOCKS.





MJM International Corporation

118-3-KD

"We Do Care - We Do More"

Assembly Instructions

STEP 4 ARMRESTS



- INSERT ARMRESTS INTO SEAT SECTION MATCH COLOR CODED DOTS.
- SECURE ARMRESTS WITH CLAMP LOCKS.



STEP 5 BACKREST & MESH SLING



- .

STEP 6 BACKREST & MESH SLING



- INSERT PVC BACKREST INTO T-FITTING LOCATED ON ARMREST SECTIONS.
- SECURE USING 1/2" RETAINING SCREWS AND PILOT HOLES AS REFERENCE



STEP 7



- PRODUCT COMPLETELY ASSEMBLED

NOTE: CASTERS, PVC BACKREST, ARMS REST, PVC HEIGHTS, SEAT SECTION MUST BE CHECKED AND SECURED PRIOR TO USING SHOWER CHAIR.

Rev: 1.3



MJM International Corporation

CARE INSTRUCTIONS

"We Do Care - We Do More"

PVC PIPING AND CASTERS

Wipe down piping with non-abrasive cleanser. **DO NOT** use solvents. Restore luster with Johnson's Pledge or similar spray wax. If casters are used in wet environment, keep lubricated with white lithium paste or similar lubricant; remove hair and mop- thread build up. Provide a weekly maintenance to avoid premature rusting. Rust Proof or Plastic caster need the same care to get the maximum usage.

MESH FABRIC (SLING)

If laundering is required, use only **COLD** water in large machine, with regular detergent and one-cup bleach. Do not use softeners or solvents. Spin-dry only; machine heat will shrink mesh fabric. Hang damp fabric on frame and allow to air dry, returning to its original shape.

SOLID FABRIC (PRIVACY SKIRTS)

Wipe down panels with sponge using solution of mild detergent, one-cup bleach and three gallons water, **DO NOT** launder in machine. To keep vinyl flexible, occasionally spray with furniture polish (e.g. Pledge) or vinyl formula (Armor All). This will help drying out and cracking.

FOOTREST AND DROP ARMS

Remove soap build-up from sliding surface; with a soft bristle brush and soapy solution. Lubricate sliding mechanism with furniture spray wax to restore ease in movement.

When lifting drop-arm, use two hands, one at each end of the section. Lifting only at hand grip will put undue pressure against the sliding fitting.

SOFT SEAT

Wipe with non-abrasive disinfectant wash. Avoid contact with sharp or rough surfaces.

CAUTION: **DO NOT** use chair if casters are rusted and fail to move easily or if cracks occur in fittings.

THE GLOBAL LEADER IN PVC MEDICAL EQUIPMENT!