



Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

# Instruction Sheet

## HTS Platform Swing - 126015 w/Handles 126017



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

### MATERIALS LIST

- 1-Platform Swing
- 1-Instruction Sheet
- 1-Maintenance Checklist

### WORKING LOAD 200 LBS.

The doorway brackets and swing are capable of supporting 200 lbs. This is dependant on the structural integrity of your doorway frame and proper installation of the brackets.

### MINIMUM HEIGHT REQUIREMENT

This product is intended to hang from the Southpaw Home Therapy System's Doorway Support.

### HANGING PROCEDURE

The ropes on each side of the swing are connected to a steel ring. Attach the steel rings to the corresponding safety snap on the doorway support. Adjust the swing to the desired height by adjusting the position of the Doorway Support in the hanging bracket (see instruction sheet for the Doorway Support for details).

### MAINTENANCE CHECKLIST

- All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.
- Check metal rings at top for excessive wear.
- Check loops at top and bottom of ropes for fraying. If fraying is found, discontinue use until repaired.
- Check metal rope clips (the metal ring at each end of ropes).
- Check wood for splinters or cracks. If splinters are found, sand smooth and refinish with lacquer. If cracks are found, discontinue use immediately.
- **Note:** If a rope clip comes loose or if a rope frays, the swing must be returned (for repair or replacement).

### STORAGE PROCEDURE

- Store where not accessible by clients without proper supervision.
- Store out of direct sunlight.

### CLEANING INSTRUCTIONS

- Wipe with a damp cloth
- Do not immerse in water.
- Dry before using.

### OPTIONAL CARPET

- You may purchase a carpet kit for this swing to provide an alternate tactile sensation for your child. Call the number below and order item number 126016 - Carpet Kit for HTS Platform Swing.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



southpaw enterprises, inc.®

## Instruction Sheet

### 126016: Carpet Kit

## Home Therapy System Platform Swing

Store these instructions in a safe place or with the enclosed maintenance checklist



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

#### MATERIALS LIST

- 1 - Carpet for the Home Therapy System Platform Swing
- 4 - Adhesive loop squares
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

#### Directions

Your carpet comes with the four adhesive loop squares attached to the hook squares on the bottom of the carpet.

1. Place the platform swing on the floor. Place the carpet on top of the platform swing, centered between the ropes and square with the edges of the platform.
2. Once the carpet is in place and square, kneel on the carpet to hold it in place. Turn one corner up, revealing the hook and loop squares. Peel the backing off of the adhesive loop square, leaving it attached to the hook square. Carefully lay the corner of the carpet back down, and press firmly on top of the corner.
3. Repeat step 2 with the remaining corners. Your carpet is now ready to use.


**NOTE: To make sure that the adhesive squares never release from the board, you can staple them in place. Be sure to use 3/8" staples, so that they do not go all the way through the wood. Staple them according to the pattern to the right.**

#### STORAGE PROCEDURE

- Store where not accessible by clients.
- Store out of direct sunlight.

#### CLEANING INSTRUCTIONS

- Vacuum carpet.
- Carpet may be cleaned with foam carpet cleaner.
- Carpet is not machine washable.

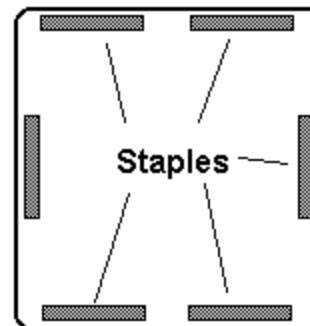

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.