



southpaw enterprises, inc.®

Store these instructions in a safe place or with the enclosed maintenance checklist

Instruction Sheet

Sensory Shaker 1395



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST

- 1 - Sensory Shaker
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

WORKING LOAD: 100 lbs.

The Sensory Shaker may be filled with different kinds of media such as playpen balls or other things deemed appropriate.

CAUTION: Do not suspend the Sensory Shaker with a client in the bag.

MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check for ripped seams or loose stitches. If ripped seams or loose stitches are found, discontinue use until repaired.

CLEANING INSTRUCTIONS

Check with your infectious disease department before using these procedures.

- Check for client allergies first.
- Machine wash using cold water and mild detergent.
- Tumble dry low heat.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.