**DRIVE MEDICAL SIMCAIR®
MEDICAL SUPPORT MATTRESS**

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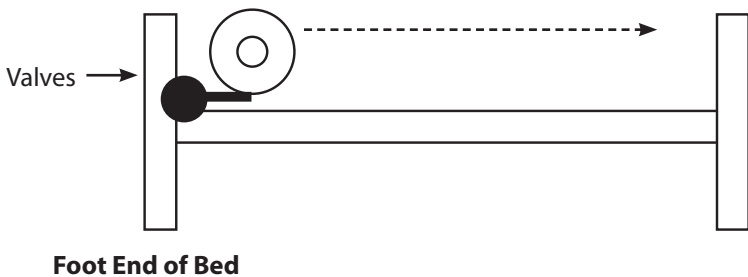
Placing The Drive Simcair® Mattress In Use

- Remove the DRIVE SIMCAIR® Mattress from the outer packaging. Do not use Knives, Scissors or sharp Objects to cut packaging when opening.
- Open the zip of the outer mattress cover at the base of the mattress to expose the two black valves
- Place the mattress on the bed with the Logo facing upwards to the patient side.
- Open the valves (see below) and unroll the mattress manually until it is flat on the bed.

To Inflate The Drive Simcair® Mattress

Turn the valves at the foot end of the mattress Counter-Clockwise as shown in Fig.A. Three complete turns will open the valves fully. The Drive - Simcair® mattress will self-inflate in approximately 5-10 minutes. When the mattress has inflated, close both valves firmly to achieve a full seal (See Fig. B). Close the zipper of the outer cover when inner valves are sealed and the mattress is ready for use.

DO NOT ADJUST THE AMOUNT OF AIR IN THE DRIVE SIMCAIR® MATTRESS AFTER FIRST INFLATION. VARIABLE ADJUSTMENT WILL NOT PROVIDE ADDITIONAL BENEFITS TO THE PATIENT. THE DRIVE - SIMCAIR® MATTRESS IS DESIGNED TO INTAKE THE CORRECT AMOUNT OF AIR ON FIRST INFLATION.

NOTE: The DRIVE - SIMCAIR® Mattress will require more time to self-inflate when first removed from its original packaging. Full Self-inflation on first use requires minimum 10 minutes with valves fully opened.

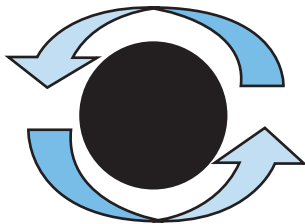
Fig. A

Turn both valves Counter-Clockwise as shown to allow mattress to self-inflate.

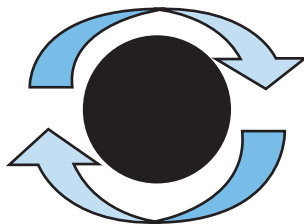
Fig. B

Close the Valve securely by turning Clockwise to seal the mattress before use

Deflating Drive Simcair® After Use Or Prior To Transport

Place the Drive SimCair® Mattress on the floor.

Open the zipper of the outer cover at the base to expose the black valves and open one valve as shown in Fig. A. Close the other valve.

Use a standard vacuum and hold the hose of the vacuum over the opened valve of the mattress until the mattress has deflated. This should take approximately 3 to 4 minutes.

Alternatively, with both valves fully opened, roll the mattress by hand as tightly as possible in the direction of the valve end of the mattress. You will hear the air exhausting through the valve. When the Mattress is fully rolled and compressed, close the valve securely as in Fig. B, it is ready for transport.

Note: You may need to press the mattress with your knees as you are rolling it and when it is fully rolled in order to push additional air from the mattress.

Drive Simcair® - Cleaning And Care

Hygienic Wipes can be used or the mattress can be wiped down with water and a mild detergent. Please ensure the correct dilution is used.

DO NOT use Phenols or organic solvents.

DO NOT Tumble Dry

DO NOT Dry Clean

DO NOT Machine Wash the mattress.

The cover of the mattress can be machine washed at up to 95 degrees C and/or can be wiped down as per instructions above.

Wipe/Wash down with correct detergents/disinfectants should be sufficient to ensure infection control. IF cleaning Drive-SimCair® Products in a Decontamination/Steam Tunnel, the maximum temperature must be 95 degrees C and for no longer than 3 minutes in a single cleaning procedure. The valves must be securely closed prior to cleaning/decontamination. It is recommended that Decontamination procedure are used ONLY when necessary and not as a regular cleaning procedure.

Other Advice

The Drive SimCair® Mattress should not be adjusted or varied for each patient after the first full inflation. The valves should be closed tightly as in Fig. B.

The Drive SimCair® Mattress is suitable for patients weighing up to 225Kgs (500lbs)*.

*Clinical Guide Only - Each patient must be assessed individually by a suitably qualified clinical expert in pressure care/wound care to assess suitability of any pressure care product.

The Drive SimCair® Mattress is suitable for patients at Low, Medium and High Risk of developing Pressure Ulcers, as well as for treatment as part of a Comprehensive Ulcer Management Program

NEVER use mechanical or electrically powered pumps to inflate the Drive SimCair® mattress.

NEVER apply direct flame/heat to the mattress.

NEVER Use phenol based cleaning agents.

NEVER use organic solvents to clean the mattress.

NEVER try to apply heat or clothes Iron directly on the mattress.

Limited 3 Year Warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for three years of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This 3 Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as accessories, covers, carry bag, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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