



southpaw enterprises, inc.®

Store these instructions in a safe place or with the enclosed maintenance checklist

# Instruction Sheet

## Sling Seat 1625 HTS Sling Seat 126040



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

### MATERIALS LIST

- 1 - Seat with ropes attached.
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

**Minimum height and space needed: 8 ft.**

**Working load: 400 lbs.**

**Keep a clear distance of 6 ft. from walls and other obstacale.**

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

### HANGING INSTRUCTIONS

To hang your Southpaw Sling Seat, use a Southpaw 100 Saftey Snap to attach it to your suspension point. The use of a Southpaw 5000 Height Adjuster is suggested for ease of adjustment. A Southpaw 2000 Safety Rotational Device is required if there is rotational movement.

### MAINTENANCE CHECKLIST

All the following wear points should be checked on a monthly basis and recorded on the checklist enclosed:

- Check for fraying of rope loops on Safety Snaps. Check Safety Snaps for wear. ( Snaps not included ).
  - Check for loose or missing splice locks \* on ropes. Discontinue use if ropes are frayed, or splice locks \* are loose or missing.
  - Check for loose or missing stitches. Discontinue use if loose or missing stitches are found.
- \* Splice locks are the rope staples near the end of the rope.

### CLEANING INSTRUCTIONS

- Check for client allergies first. Check with your infectious disease department before using these procedures.
- May be hand washed in mild soap. Do not bleach. Hang to dry.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.