

## Safety in the spotlight.

MOBELI® - always a well-rounded solution.



**MOBELI®**  
The innovative Original.



Focus on safety.



Clarke Health Care Products, Inc  
1003 International Drive  
Oakdale PA 15071  
Toll Free: 1-888-347-4537  
www.clarkehealthcare.com

**ROTH®**

## Mobile assistants.

MOBELI® - small helpers, big impact.

Our mobile helpers support you in a variety of ways in daily living. Just position our helpers at the desired place to mount a hairdryer, shaver, or hair-brush – that's it.

The helpful shower positioner is available with a flexible swivel arm as an option. The mobile telescopic mirrors – with optional LED light – are quickly and easily positioned. They make it possible to look into the mirror while seated or are very helpful during catheterizing or for stoma control.

If your canes lie on the floor far too often, then the mobile Duo stick holder will make life easier for you. It can be mounted on the wall or under the tabletop. It's always quick and safe and you never need tools.



MOBELI®	Order no.
Mobile clip holder with swivel arm	14992 04 S
Mobile telescopic mirror (31.5")	14002 09 S
LED micro light with clip (incl. 50 h battery)	14092 09
Shower head holder (picture p. 10)	14002 02 S
Shower head holder with swivel arm	14992 02 S
Duo canes holder	14002 06 S

9



The name is program: view screen and splash guard; very popular and ideal for patient care. The jointed arm can be tilted upward or swiveled to the side if mounted vertically. The integrated brake holds the Mobeli arm automatically in the desired position. The 720 – 1050 mm long three-part elastic arm has been conceived in such a way that no strong leverage forces are exerted onto the sustaining wall – even when leaning on it.



MOBELI®	Order no.
Jointed arm for view screen/splash guard (without curtain)	14002 05 S
Curtain (width/length): (47.2" x 31.5")	140612 08

10

## Support systems.

Safe grip. Exactly where you need it.

The supporting system QuattroPower Support with its four suction pads has been designed in such a way that the leverage forces acting on the wall are considerably reduced. It can even be attached on tiled cavity walls and carry a nominal load of 80 kg, thus eliminating the need for expensive rebuilding to reinforce the walls so that screw-on grab handles might be attached.

Depending upon requirements and room conditions, this supporting system can be attached e.g. upward or downward on both sides of the WC. The QuattroPower support grip can be attached at the side, above, or in front of WC, shower, or bath tub; it increases your action radius considerably and facilitates getting up or relocating.



MOBELI®	Order no.
QuattroPower Support	14002 65 S
QuattroPower Support with grip	14002 66 S
QuattroPower Tub	14002 61 S
QuattroPower Tub with grip extension	14002 62 S

11



Optimized for the bath tub: The Mobeli supporting system QuattroPower Tub is excellently suited for getting into and out of the bath tub. It can sustain large tensile and supporting loads. Before entering or leaving the bath tub, seize the grab bar with both hands and let go of it only after you are sitting safely or standing firmly. By the way: The QuattroPower Tub can be attached to the outside or to the inside and holds even under water.



12



MOBELI® - living an active life. With safety.



**MOBELI®**

Mobile grab handles – ingeniously simple and safe.

# Mobile grab handle

Stay mobile. Overcome limitations.

The patented Mobeli system by ROTH has proven itself for ten years as a practical helper in everyday life for handicapped and elderly persons.

The mobile grips are quickly and easily attached and just as easily removed. Simply press both suction pads with a diameter of 120 mm each onto a flat, non-porous surface, press down the vacuum levers – that's it.


All grab handles are manufactured with excellent workmanship and certified by the German Technical Inspection Agency TÜV-Süd; they provide an enormously high adhesive strength and are available in different lengths. Useful accessories such as shower head positioners, dual hooks, bath tub shorteners or trays as well as numerous other little "assistants" complete the Mobeli system in a useful way.

MOBELI®	Grip length (inch)	Nominal load	Order no.
1-hand grab handle	4.3	176 lbs	14002 20 S
Grab handle	8.3	166 lbs	14002 25 S
Grab handle	13.8	154 lbs	14002 26 S
Grab handle	25.6	154 lbs	14002 27 S
Grab handle telescope	7.9-13.6	144 lbs	14002 21 S
Grab handle telescope	13.4-18.3	144 lbs	14002 22 S
Grab handle telescope	17.3-22.3	144 lbs	14002 23 S
Grab handle telescope	26.2-31.1	132 lbs	14002 24 S

# MOBELI®

### Suction pad

The internal interlocking of suction pad and vacuum head results in a substantially higher holding strength. The suction pads, made of special rubber, are even suitable for slightly textured bathroom and WC tile surfaces.

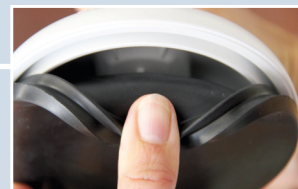

### Telescopic grab bar

We have developed the telescopic grip to enable you to attach your holding grip independently of the tile grid. The grips are made of extremely high-impact resistant and tough plastic.

### Quick-lock coupling

All Mobeli grab handles can be conveniently installed without any tools and just as easily disassembled for travelling. The quick-lock coupling is manufactured of stainless steel, as are all metal parts of the Mobeli system.

### Safety indicator

The latest innovation of the Mobeli grab handle is simply revolutionary. We present you a true novelty: **The first holding grip that permanently monitors the vacuum state – and thus its adhesive strength.** The Mobeli safety jack continuously examines the vacuum. If the warning jack is sunk flat in the lever, the adhesive strength is correct and the grip is fully able to carry its load. If the warning jack rises and the red edge of the jack becomes visible, the Mobeli suction head must be released and fastened anew. The safety indication is also suitable for persons with impaired eyesight because in case of vacuum loss the risen warning jack can easily be felt.


**Important note:** All MOBELI grab handles are subject to the strict production control by the German Technical Inspection Agency TÜV-SÜD and have been designed and are prescribable as medical products (conform to the directive 93/42/EWG).

# Safety indication


Mobile and safe. Because safety comes first.

Mobeli is the original among the mobile grab handle. Our product family is constantly being further developed and enhanced. All grab handles of the Mobeli system are developed and manufactured in Germany by ROTH, a company rich in tradition.

Mode of operation of the Mobeli safety jack: The vacuum is permanently monitored.

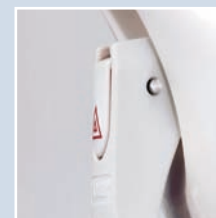

**Full load capacity.** Indicated nominal carrying load with 1.5 times safety allowance (e.g., 70 kg = 105 kg).

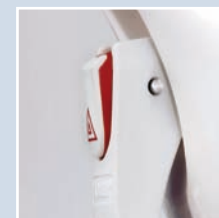

**Approx. 100 % load capacity.** Indicated nominal carrying load without safety buffer. Attention: Reattach now!



**Attention: RED Alert.** The vacuum volume of the holding grip has been used up. The grip will fall off. Reattach immediately!


Well-suited for travelling. Our mobile grab handles can easily be disassembled. They take up little space in your luggage and are just as easily reassembled at your destination – without any tools.

Versatile and extensible. The Mobeli system offers a solution for almost any problem. Please inquire with our team of experts or the competent specialized trade.




# Mobility assistants.

More mobility. At home and when travelling.

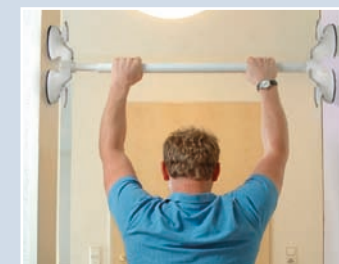


High carrying load variant: The mobile vacuum holding grip QuattroPlus can be fastened at the wall, across a corner, or serve as horizontal training grab bar. Thanks to the integrated universal joints it is possible to fasten the QuattroPlus even at the door frame of a room.

The angularly adjustable hand rail combines all the advantages of a horizontal holding grab bar to pull yourself up with the advantages of a vertical grab bar, e.g. as support for holding while entering and leaving a shower cubicle. Thus it conforms completely to the new DIN 18040 on barrier-free living for horizontal and vertical grab handle in shower cubicles.

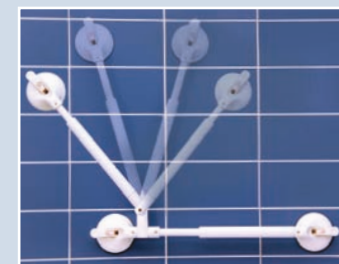


MOBELI®	Grip length (inch)	Nominal load	Order no.
QuattroPlus	7.9-12.8	276 lbs	14002 41 S
QuattroPlus	12.6-17.5	276 lbs	14002 42 S
QuattroPlus	16.5-21.5	142 lbs	14002 43 S
QuattroPlus	25.4-30.3	142 lbs	14002 44 S
Angularly adjustable hand rail	13.4-18.3	142 lbs	1400222H2S

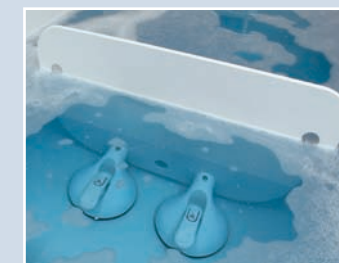

Handy and comfortable. The continuously positionable bath tub shortener prevents "getting lost under water" in the bath tub. It is suitable just as well as a tray directly at the bath tub.

Stabilization grips. You are looking for assistance to stabilize your hand: Mobeli Stabi – in two variants. It can be quickly and conveniently attached on all pore-tight surfaces like painted wood, plastic, or metal.

The Stabi Horizontal stabilizes in the horizontal direction; Stabi Vario can alternatively be swiveled vertically, horizontally, or at an angle of 45° to the left or right.

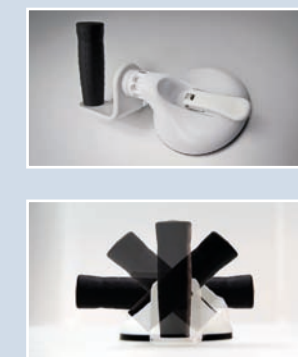

MOBELI®	Bath tub width (inch)	Order no.
Bath tub shortener (2 suction pads)	from 20.1	14002 81 S
Bath tub shortener (3 suction pads)	20.1-27.6	14002 82 S

	Grip (inch)	Nominal load	Order no.
Stabi Horizontal	1.2	34 lbs	14002 95 S
Stabi Vario	1.2	34 lbs	14002 96 S