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INSTRUCCIONES AL DORSO

## LIFETIME LIMITED WARRANTY

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer.

MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners.

Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.

MOBB Healthcare Ltd.  
116A Viceroy Rd. Unit 5-6  
Concord, Ontario, Canada  
L4K 2M1  
[www.mobbhhc.com](http://www.mobbhhc.com)

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2

## CARE AND CLEANING

- REGULAR CLEANING IS RECOMMENDED
- USE A NON-ABRASIVE CLEANER OR DETERGENT. RINSE WITH WARM WATER AND DRY COMPLETELY.
- ABRASIVE CLOTHS MAY DAMAGE THE SURFACE
- PLEASE REGULARLY INSPECT YOUR FOLDING TOILET SAFETY FRAME AND IT'S PARTS FOR ANY CORROSION, BREAKAGE, COMPRESSION OR WEAR.

## SET UP & USAGE

### 1. Unboxing

- The folding toilet safety frame comes fully assembled and folded in the box. Remove from box and rest the horizontal bar against your legs.

### 2. Unfolding

- With the horizontal bar facing you, hold one padded armrest closest to you. Grasp the padded armrest attached to the left or right section of the frame (which ever is furthest from you) and push it away from you so it swings open. Continue to swing until you hear an audible click. Repeat for the other side of the frame.

### 3. Height Adjustment of Legs.

- All four legs operate the same way. Push the lock button in (1) and slide the leg tube up or down to the desired height. Ensure an audible click is heard indicating the leg is locked and the Folding Toilet Safety Frame is then safe to use. ENSURE ALL LEGS ARE ADJUSTED TO THE SAME HEIGHT. Consult your healthcare practioner and/or home health care dealer for determining the appropriate height of the arm rests.

### 4. Folding

- To fold for storage or transport purposes, first depress the locking leaver down (2) to release the locking pin. While depressed, swing that side of the frame towards the horizontal bar in the middle. Repeat this step for the other side.

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