



MAT PLATFORMS

All Sizes of MODELS: 240, 241, 242,
260 & 262



ASSEMBLY INSTRUCTIONS

ASSEMBLY INSTRUCTIONS FOR MAT PLATFORMS

All sizes of **MODELS 240, 241, 242, 260 & 262,**

Thank you for counting on Clinton to meet your needs by purchasing this fine equipment.

All Clinton equipment is designed and manufactured to provide years of dependable service because we want you as a customer now and in the future. Below are easy to follow instructions on how to properly assemble your equipment. Please contact us at 800-441-9131 if you experience any difficulty with the assembly or have questions.

INTENDED USE


Clinton Mat Platforms, models 240, 241, 242, 260 & 262 are specifically designed and engineered for physical therapy use. The combination of force exerted on the table during treatment should not exceed the individual product weight capacity.

Hardware Package

- 14 3¹/₂" Hex Head Bolts
- 28 7/8" Washers
- 14 1/2" Keps Nuts

Tools Required

- 1/2" Wrench

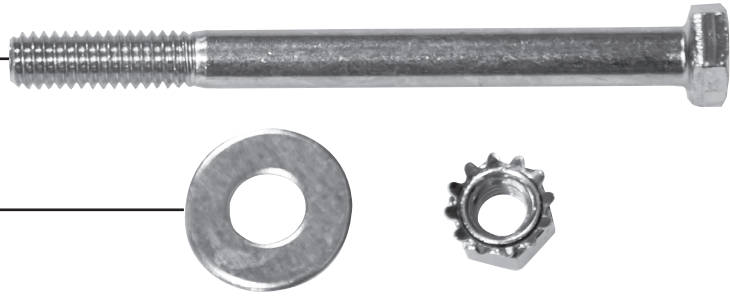

1

Invert top of platform and attach corner legs by inserting the 3¹/₂" hex head bolts with a washer through the frame and the coordinating holes in the corner legs. Secure the legs by placing a washer and keps nut on each bolt and tightening with a wrench.



2

Secure the side legs (or center leg for models 260 & 262) in the same manner as the corner legs.



FOR SAFETY PURPOSES, TAKE TIME TO CHECK ALL NUTS TO INSURE THAT THEY ARE SECURELY TIGHTENED AND THAT THE PLATFORM HAS BEEN PROPERLY ASSEMBLED ACCORDING TO THESE INSTRUCTIONS.



800-441-9131 • 717-848-3519

Toll-Free Fax 866-522-1233

Fax 717-843-5871

www.clinton-ind.com

