

Wired Mouse-Style Video Magnifier

Item # 251065

Features:

- Video magnification for a television
- 8 levels of magnification, from 2X – 32X
- 4 different color viewing options, including full color, digital black on white, digital white on black, and high contrast
- Screen freeze mode
- LED illumination

Package Contents

- Power adapter with USB cord
- Wired mouse-style magnifier, with video out plug and an AC power connector
- Soft pouch

Set Up

- Take the “Y” cord that is attached to the mouse and insert the yellow video input plug into the yellow Video Input socket of a standard television.
- Plug the USB plug (rectangular) from the power cord into the bottom of the AC adapter (power plug).
- Plug the AC adapter into an electrical outlet.
- Plug the other end of the AC adapter into the AC power jack of the mouse “Y” cord.

To Operate

- Turn on the television.
- Turn on the wired mouse, using the button on the left hand side of the mouse. A green light will appear and the mouse will emit a high pitched, loud beep.
- Place the mouse directly on top of the desired reading material and roll the mouse from side to side.

To Increase Magnification

- Slowly press the “+” button on the top of the mouse. Wait for the magnification icon on the TV screen to disappear from the screen before pressing the “+” button again.

- Magnification ranges from 2X to 32X in 8 step increments.

To Decrease Magnification

- Slowly press the “-” button on the top of the mouse. Wait for the magnification icon on the TV screen to disappear from the screen before pressing the “-” button again.

To Change Color Modes

- The wireless mouse has 4 color modes – full color, digital black on white, digital white on black, and high contrast. To change between these modes, press the “M” button slowly on the left side of the magnification (+ / -) toggle. Keep scrolling through the different modes to find the best reading position.

Screen Lock

- The screen lock feature allows the user to freeze an image on the television screen.
- Press the “*” button on the right side of the magnification toggle. This freezes the current image on the TV screen, even if the mouse is moved.
- To unfreeze the image, and release the mouse for general viewing, press the “*” button again.

Shutting Down

- When done using the mouse, turn off the power button on the left hand side of the mouse.
- Turn off the television, if no longer using.