
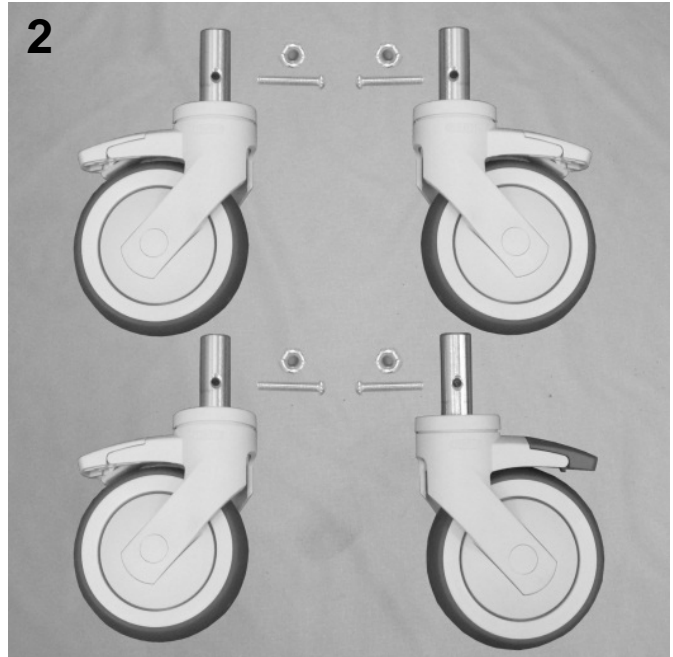


6900 Elite Care Cliner Parts List

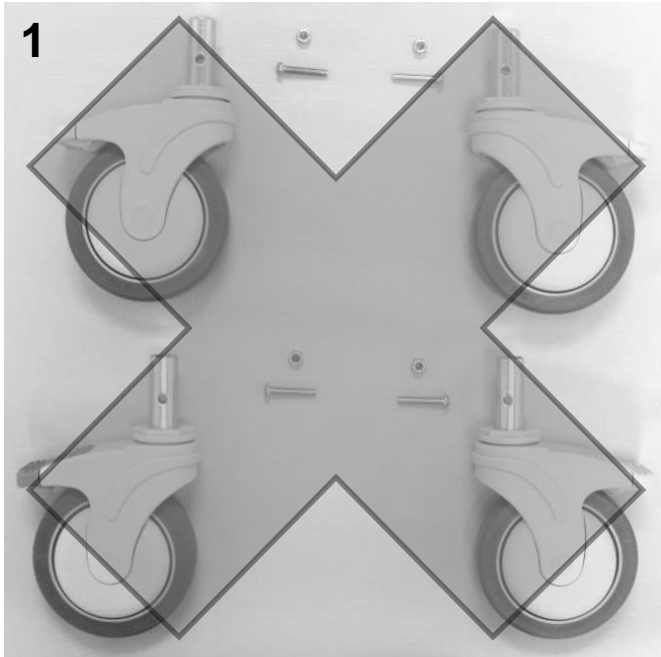



PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
VERSION 1				
1	A,D	400550	CASTER, NYLON 5" TOTAL LOCK (NO LONGER AVAILABLE)	3
1	A,C,D	400560	CASTER, NYLON 5" DIRECTNL LOCK (NO LONGER AVAILABLE)	1
2	E	200560K	OBSOLETE DIRECTIONAL CASTER REPLACEMENT KIT	1
VERSION 2				
2	B	400575	CASTER, NYLON 5" TOTAL LOCK W/PRECISION BEARING WHEEL, STD COLOR	3
2	B, C	400585	CASTER, NYLON 5" DIRECT. LOCK W/PRECISION BEARING WHEEL, STD COLOR	1
1,2		566400	BOLT-CARRIAGE ZINC 1/4-20 X 1 1/4	4
1,2		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4

NOTES

Note	Description
A	VERSION 1 = 690's made BEFORE serial # 690102561
B	VERSION 2 = 690's made from serial # 690102561 to Present
C	400560 or 400585 is installed on the LEFT REAR of chair frame. (GREEN BRAKE TAB)
D	This item is no longer available. MUST order kit 200560K . See NOTE F
E	This kit is ONLY available as a replacement for 400560. Includes the 1st 4 items listed under VERSION 2
	<i>Left & Right is determined - as if sitting in the chair</i>

6900 Elite Care Cliner Parts List

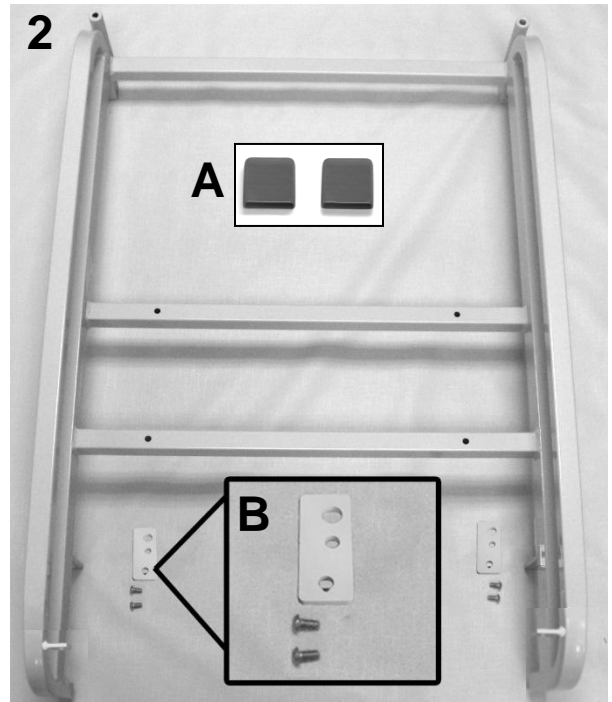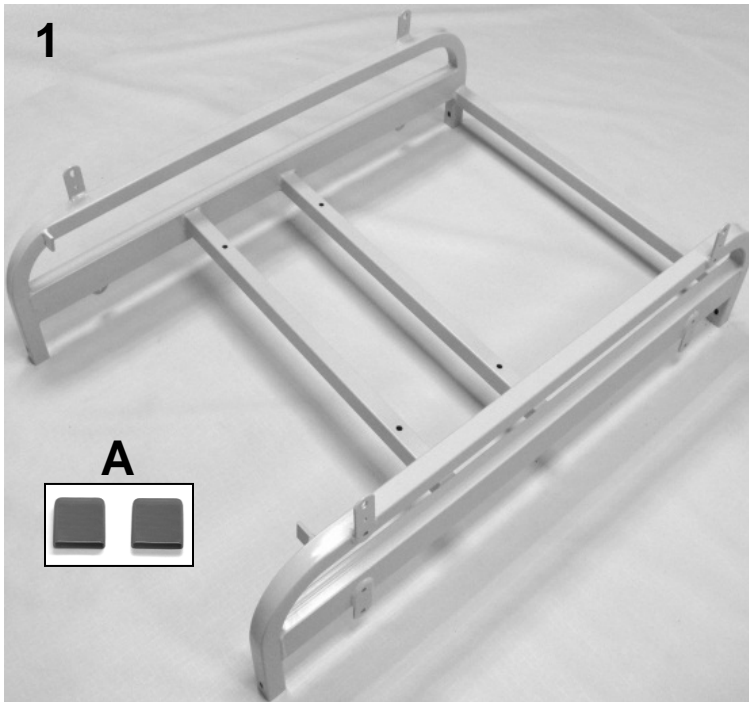





PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
VERSION 1				
1	C	246900	690 MAIN FRAME WELDED	1
1A	C	700890	CAP, FLAT RECTANGULAR , GRAY, .25" X 1"	2
VERSION 2				
2	C	246902	694 SWING-ARM MAIN FRAME, WELDED	1
2A	C	700890	CAP, FLAT RECTANGULAR , GRAY, .25" X 1"	2
2B	C	202830	LATCH PLATE, FIXED ARM (690 STYLE) PAINTED	2
2B	C	566572	SCREW, BUTTON HEAD 1/4-20 X 3/4"	4

NOTES


Note	Description
A	VERSION 1 = 690's with serial #'s that contain <u>no letter</u> or an "A" (EXAMPLE: 690123456 or 690 A 123456)
B	VERSION 2 = 690's with serial #'s that contain a "B" (EXAMPLE: 690 B 123456)
C	Part #'s 246900 & 700890 in VERSION 1 or 246902 & 700890 in VERSION 2 MUST be ordered together <i>Left & Right is determined - as if sitting in the chair</i>

6900 Elite Care Cliner Parts List

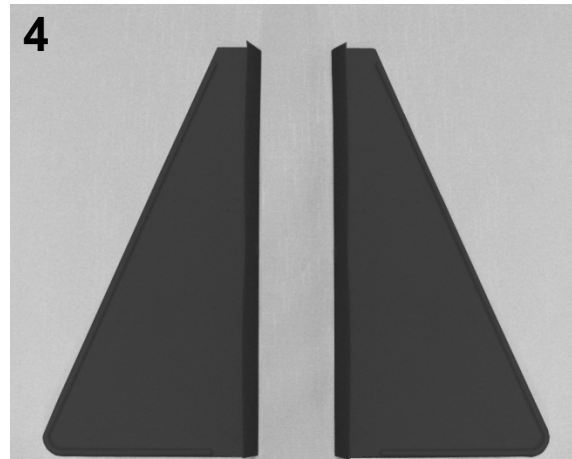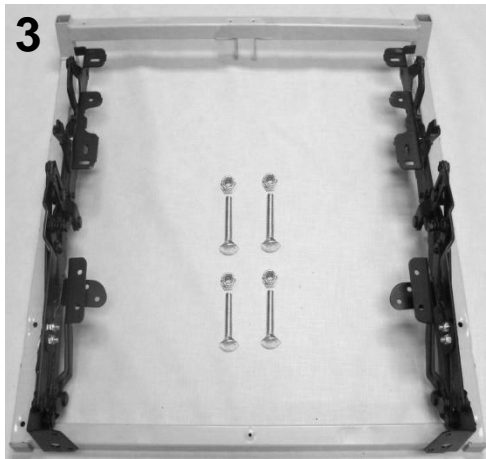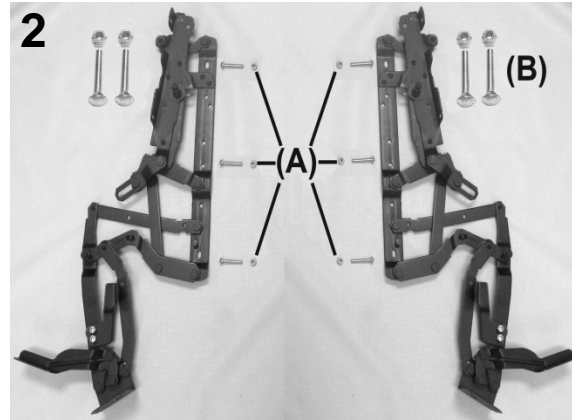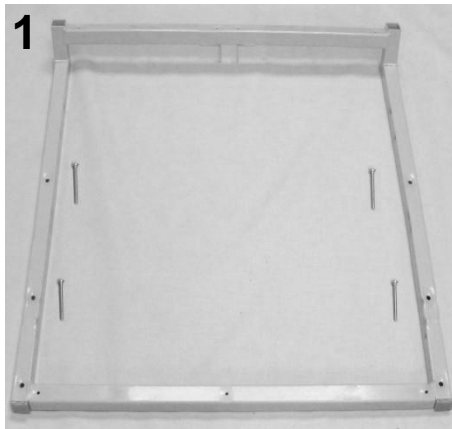



PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1		216730K	557A STYLE SEAT FRAME KIT	1
1	A	216730	WELDED SEAT FRAME 557A	1
1	A	566515	SCREW-MACH 10-24 X 1 3/4 PAN PHILLIP M/S ZINC	4
1	A	700700	SQ 1" CAP, GRAY	2
2		200800K	653 MECHANISM KIT	1
2	A	400811	MECHANISM, RECLINER, LEFT 525/529/653	1
2	A	400821	MECHANISM, RECLINER, RIGHT 525/529/653	1
2	A	402670	MID OTTOMAN BRKT LEFT	1
2	A	402680	MID OTTOMAN BRKT RIGHT	1
2A	A	566506	SCREW-MACH 1/4-20 X 1 1/2 RND PHIL M/S ZINC	6
2B, 3	A	566401	BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC	4
2	A	566504	SCREW-MACH 1/4-20 X 3/4 RND PHILLIP M/S ZINC	4
2A, 2B, 3	A	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	14
3	B	216740K	557A STYLE SEAT FRAME W/MECH KIT	1
4		704300	SAFETY SHIELD, BLACK (L&R PAIR)	1 PAIR
Note	Description			
A	It is NOT recommended that these parts be sold seperately however in special cases they can be.			
B	All parts listed above are included in this kit			
	<i>Left & Right is determined - as if sitting in the chair</i>			

6900 Elite Care Cliner Parts List

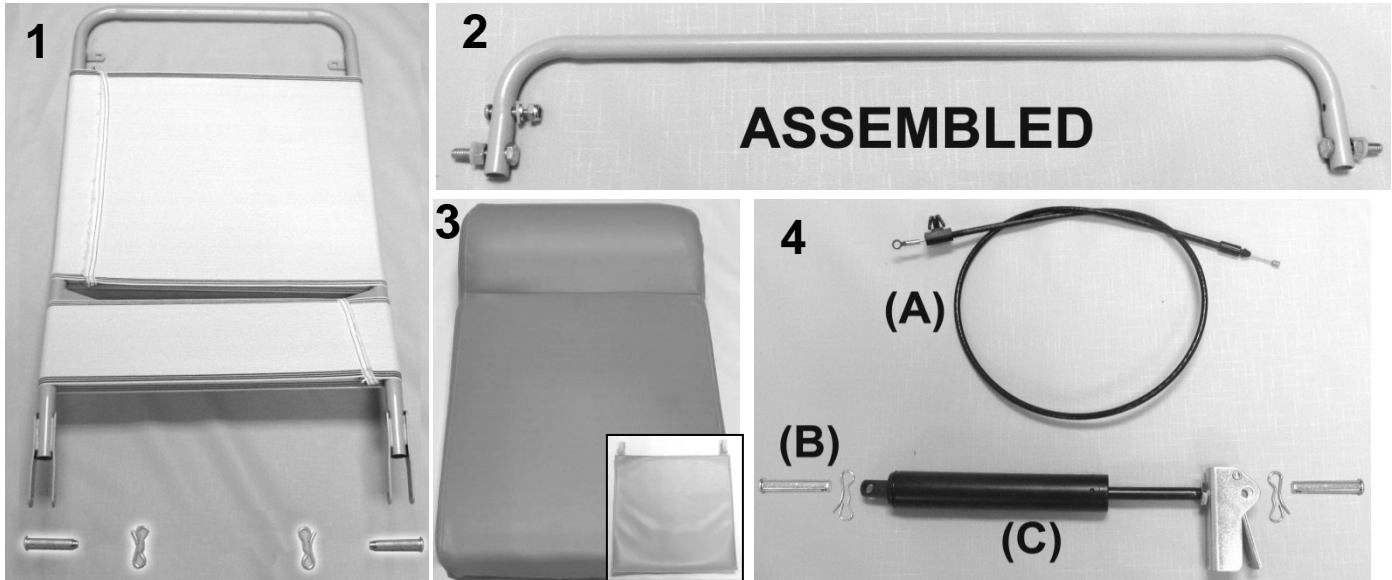


PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1		243204W	BACK FRAME W/WEBBING (653-655-690-694)	1
1	A	200025K	CLEVIS/COTTER PIN ASSM. KIT	2
2		243205K	653 RELEASE HANDLE ASSM KIT	1
2	D	243205	653 RELEASE HANDLE	1
2	D	566512	SCREW-MACH 10-24 X 1 PAN PHILLIP M/S ZINC	1
2	D	566321	WASHER-SBUP 3/16 STEEL BACK WASHER	1
2	D	566219	CAP NUT, 10-24 STAINLESS STEEL	1
2	D	566310	WASHER-1/4 NYLON F/W (.275 X .50 X .0625)	3
2	D	566606	1/4-20 X 1 1/2 HEX CAP SCREW, ZINC, FULLY THREADED	2
2	D	566206	NUT-1/4-20 HEX 2-WAY LOCK NUT ZINC	2
3	B,F	U0208XX	UPHOLSTERED BACK	1
3	B,C	HC00-XX	HEADREST COVER	1
4A		203500K	CONTROL CABLE REPLACEMENT KIT	1
4A		403500	CONTROL CABLE 34"	1
N/A		566512	SCREW-MACH 10-24 X 1 PAN PHILLIP M/S ZINC	1
N/A		566219	CAP NUT, 10-24 STAINLESS STEEL	1
N/A		566321	WASHER-SBUP 3/16 STEEL BACK WASHER	1
N/A		566310	WASHER-1/4 NYLON F/W (.275 X .50 X .0625)	1
4B		200025K	CLEVIS/COTTER PIN ASSM. KIT	2
4C	E	205210	CLINICAL CHAIR GAS CYL. W/OPERATOR ASSM'D & ADJUSTED	1
NOTES				
Note	Description			
A	Parts are included with 243204W			
B	XX- Use color code for specific part number i.e. 02 - Royal Blue			
C	HEADREST COVER is a separate item & is NOT included with the uph. back when ordering			
D	It is NOT recommended that these parts be sold separately however in special cases they can be.			
E	When ordering it is recommended that part # 200025K be ordered as well.			
F	For chairs with Heat and Massage or Heat ONLY option see "Heat & Massage Identifiers" below			
OPTIONS	For TB133 Add TB after Part Number i.e. U020802TB			
	Use 00 if option is not needed i.e. U020802 00 HM for upholstered back without TB & with HM/HT			
Heat & Massage Identifiers				
M5	For chairs w/ heat/massage made 04-01-14 to present add identifier after TB option i.e. U0207XXTB XX			
M4	For chairs w/ heat/massage made 05-18-12 to 03-31-14 add identifier after TB option i.e. U0207XXTB XX			
M3	For chairs w/ heat/massage made 06-22-10 to 05-17-12 add identifier after TB option i.e. U0207XXTB XX			
M1	For chairs w/ heat/massage made 01-01-08 to 06-21-10 add identifier after TB option i.e. U0207XXTB XX			
	Any chairs manufactured before January 2008 may not be available. A photo must be taken of controller (located on bottom of chair) & sent to customer service/engineering for verification.			
Heat ONLY Identifiers				
H5	For chairs w/ heat only made 04-01-14 to present add identifier after TB option i.e. U0207XXTB XX			
H4	For chairs w/ heat only made 05-18-12 to 03-31-14 add identifier after TB option i.e. U0207XXTB XX			
H3	For chairs w/ heat only made 06-22-10 to 05-17-12 add identifier after TB option i.e. U0207XXTB XX			
H1	For chairs w/ heat only made 01-01-08 to 06-21-10 add identifier after TB option i.e. U0207XXTB XX			
	Any chairs manufactured before January 2008 may not be available. A photo must be taken of controller (located on bottom of chair) & sent to customer service/engineering for verification.			

6900 Elite Care Cliner Parts List

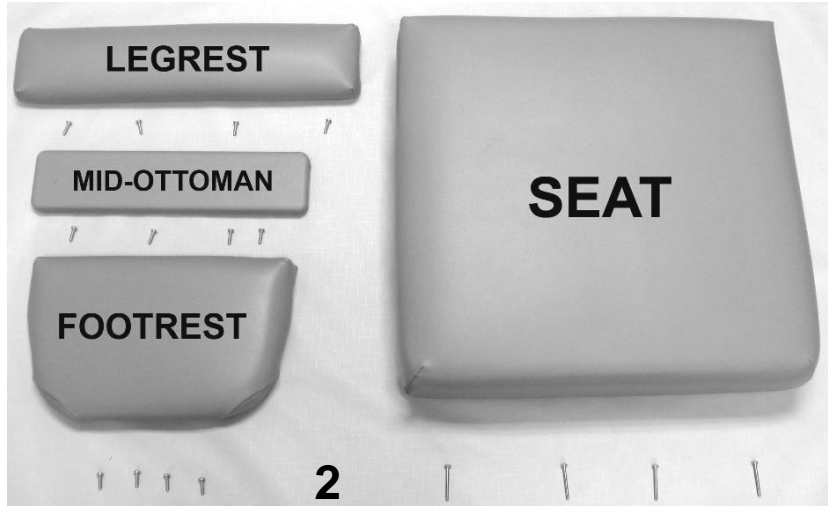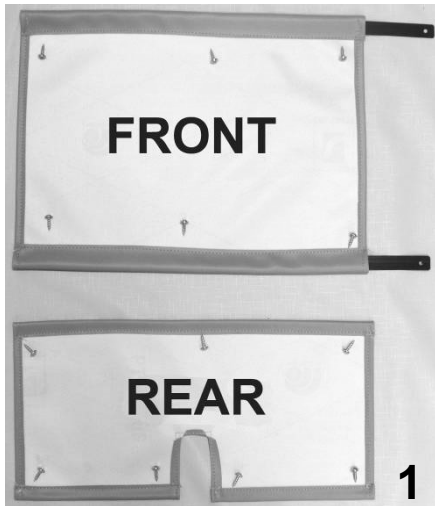


PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1	A	U1403XX	FRONT APRON	1
1	A	U1503XX	REAR APRON	1
1		229910	APRON SLAT, FRONT (653 STYLE)	2
1		566534	SCREW-TAP 10 X 3/4 RND WASHER SQ T/S ZINC	13
2	A	U1003XX	UPHOLSTERED LEGREST	1
2	A	U1201XX	UPHOLSTERED MID-OTTOMAN	1
2	A	U1104XX	UPHOLSTERED FOOTREST	1
2		566511	SCREW-MACH 10-24 X <u>3/4</u> " PAN PHIL M/S ZINC	12
2	A,B,C	U0303XX	UPHOLSTERED SEAT	1
2		566515	SCREW-MACH 10-24 X <u>1 3/4</u> " PAN PHILLIP M/S ZINC	4
N/A	C	490014	JUMPER , SEAT HEATING ELEMENT TO CONTROL MOD.	1
NOTES				
Note	Description			
A	XX- Use color code for specific part number i.e. 02 - Royal Blue Screws needed to attach this product are included			
B	For chairs with Heat and Massage or Heat ONLY option see " Heat & Massage Identifiers " below			
C	Any heat/massage or heat only chairs manufactured before January 2008 may need 490014 (listed above) ordered w/ UPH seat. A photo must be taken of controller (located on bottom of chair) & sent to customer service/engineering for verification.			
OPTIONS	For TB133 Add TB after Part Number i.e. U0303XX TB			
	Use 00 if option is not needed i.e. U0303XX 00 HT For Upholstered Seat without TB w/ HT			
	<i>Left & Right is determined - as if sitting in the chair</i>			
Heat & Massage or Heat ONLY Identifiers				
H5	For chairs made 04-01-14 to present add identifier after TB option i.e. U0207XX TBXX			
H4	For chairs made 05-18-12 to 03-31-14 add identifier after TB option i.e. U0207XX TBXX			
H3	For chairs made 06-22-10 to 05-17-12 add identifier after TB option i.e. U0207XX TBXX			
H1	For chairs made 01-01-08 to 06-21-10 add identifier after TB option i.e. U0207XX TBXX			
	<i>Left & Right is determined - as if sitting in the chair</i>			

6900 Elite Care Cliner Parts List

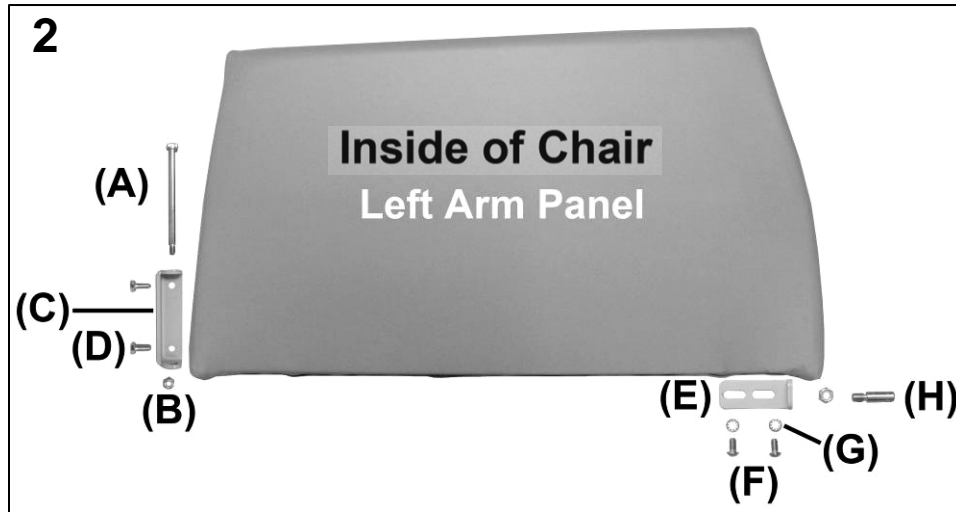

PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
VERSION 1				
1	C,D	U0401XX	690 UPHOLSTERED ARM PANEL, LEFT	1
1	C,D	U0501XX	690 UPHOLSTERED ARM PANEL, RIGHT	1
1		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	8
VERSION 2				
2	C,E	U0403XX	694 UPH. SWING-ARM PANEL - LEFT	1
2	C,E	U0503XX	694 UPH. SWING-ARM PANEL - RIGHT	1
2B	E,F	566205	NUT-5/16-18 HEX NYLON LOCK NUT ZINC	SEE NOTES
2C	F	202735	WELDED HINGE BRKT REINF.	1
2D	F	566410	BOLT - 5/16 X 3/4 HEX HEAD	2
2E	F	202740	LATCH BRACKET ARM, GRAY	1
2F	F	566572	SCREW, BUTTON HEAD 1/4-20 X 3/4"	2
2G	F	566301	WASHER-1/4 INT. TOOTH LOCKWASHER ZINC	2
2H	F	202770	STRIKER PIN, 3/8 X 1" (5/16-18 THREAD)	1
2A	E	566555	3/8 X 4 1/2, ZINC SKT SHOULDER SCREW	1
NOTES				
Note	Description			
A	VERSION 1 = 690's with serial #'s that contain <u>no</u> letter or an "A" (EXAMPLE: 690123456 or 690 A 123456)			
B	VERSION 2 = 690's with serial #'s that contain a "B" (EXAMPLE: 690 B 123456)			
C	XX- Use color code for specific part number i.e. 02 - Royal Blue			
D	Hardware needed to attach this product is included			
E	VERSION 2 - 2B is included when ordering Arm-Panel - 2B must be ordered separate when ordering 566555			
F	It is NOT recommended that these parts be sold separately however in special cases they can be.			
OPTIONS	For TB133 Add TB after Part Number i.e. U0403 XTB			
	For Heat & Massage or Heat Only option see page 8			
	Use 00 if option is not needed i.e. U0403XX 00 HT For Upholstered Seat without TB w/ HT			
	<i>Left & Right is determined - as if sitting in the chair</i>			

6900 Elite Care Cliner Parts List

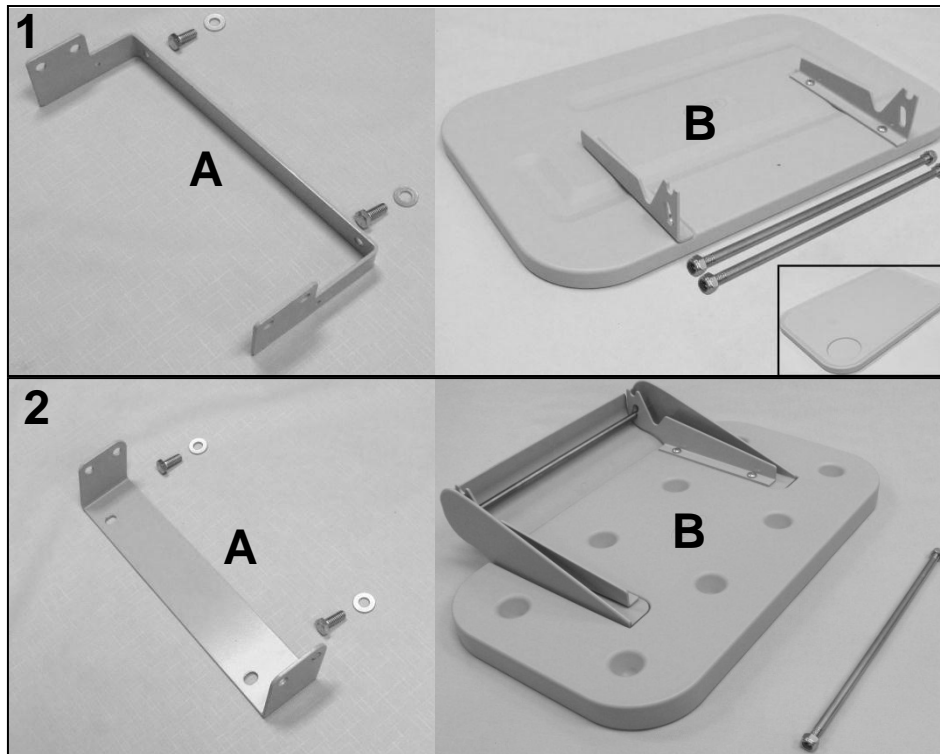


PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
VERSION 1				
1	A	261620K	SIDE TABLE "ADD ON" KIT ASSM. 690/900 LEFT	1
1A	D	246920	690 LEFT TABLE BRKT FORMED	1
1A	C,D	566410	BOLT - 5/16 X 3/4 HEX HEAD	2
1A	C,D	566304	WASHER-5/16 SAE FLAT WASHER ZINC	2
1B	C,D	700420	SIDE TABLE BLOW MOLDED PLASTIC VERSION ONE W/CUP HOLDER	1
1B	C,D	203150	SIDE TABLE HINGE BRKT (PR), GRAY	1
1B	C,D	566300	WASHER-10 INTERNAL TOOTH WASHER ZINC	4
1B	C,D	566509	SCREW-MACH 10-24 X 3/8 PAN PHIL M/S ZINC	4
1B	C,D	243273	THREADED ROD FOR SIDE TABLE KIT	2
1B	C,D	566205	NUT-5/16-18 HEX NYLON LOCK NUT ZINC	4
1	A	261623K	SIDE TABLE "ADD ON" KIT ASSM. 690/900 RIGHT	1
1A	D	246915	690 RIGHT TABLE BRKT FORMED	1
VERSION 2				
2	B	261640K	SIDETABLE/COVER REPLACEMENT KIT BOLT-ON BRKT STYLE	1
2A	C	202685	SIDE TABLE MOUNTING BRKT - GRAY - GRAY - GRAY	1
2A	C	566416	BOLT - 5/16 X 3/4 HEX CAP SCREW, GR.5, ZINC, W/BUE NYLON PA	2
2A	C	566304	WASHER-5/16 SAE FLAT WASHER ZINC	2
2B	C	700425	SIDE TABLE, BLOW MOLDED (USED W/COVER)	1
2B	C	700426	COVER, INJECTION MOLDED (FOR SIDE TABLE)	1
2B	C	203150	SIDE TABLE HINGE BRKT (PR), GRAY	1
2B	C	566561	#12 X 3/4" PANHEAD PHILLIPS TAP SCREW, ZINC	4
2B	C	243270	5/16 SS ROD FOR SIDE TABLE	1
2B	C	243273	THREADED ROD FOR SIDE TABLE KIT	1
2B	C	566205	NUT-5/16-18 HEX NYLON LOCK NUT ZINC	2

NOTES

Note	Description
A	VERSION 1 = 690's made PRIOR to January 13th, 2006
B	VERSION 2 = 690's made AFTER January 13th, 2006
C	Parts included in LEFT and/or RIGHT "ADD ON" kits
D	It is NOT recommended that these parts be sold separately however in special cases they can be.
	<i>Left & Right is determined - as if sitting in the chair</i>

6900 Elite Care Cliner Parts List

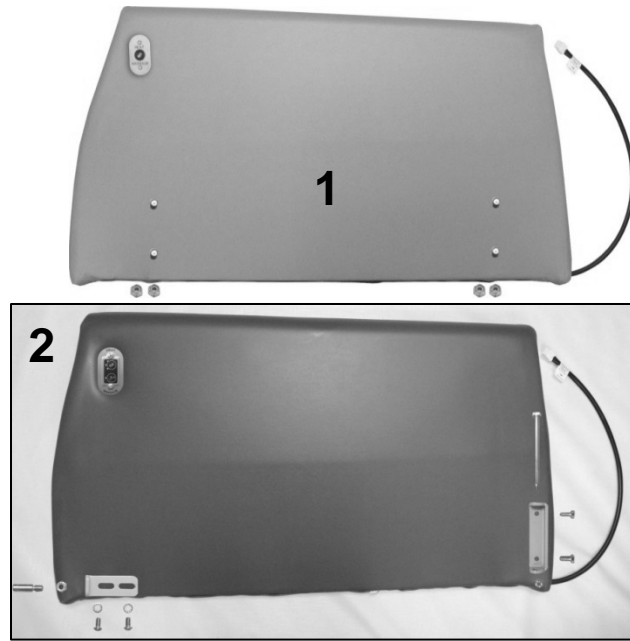

PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
VERSION 1				
1	A,B,D	U0501XX00**	RIGHT UPHOLSTERED ARM PANEL W/ HM-HT ASSEMBLY KIT	1
1		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
VERSION 2				
2	A,C,D	U0503XX00**	RIGHT UPHOLSTERED ARM PANEL W/ HM-HT ASSEMBLY KIT	1
NOTES				
Note	Description			
A	XX- Use color code for specific part number i.e. 02 - Royal Blue See PAGE 6 for a complete list of hardware available for the RIGHT arm panel. Hardware needed to attach this product is included			
B	VERSION 1 = 690's with serial #'s WITHOUT a "B" (EXAMPLE: 690123456 or 690A123456)			
C	VERSION 2 = 690's with serial #'s that contain a "B" (EXAMPLE: 690B123456)			
D	For chairs with Heat and Massage or Heat ONLY option see " Heat & Massage Identifiers " below			
**Heat & Massage Identifiers				
M5	For chairs w/ heat/massage made 04-01-14 to present add identifier after TB option i.e. U0207XXTB XX			
M4	For chairs w/ heat/massage made 05-18-12 to 03-31-14 add identifier after TB option i.e. U0207XXTB XX			
M3	For chairs w/ heat/massage made 06-22-10 to 05-17-12 add identifier after TB option i.e. U0207XXTB XX			
M1	For chairs w/ heat/massage made 01-01-08 to 06-21-10 add identifier after TB option i.e. U0207XXTB XX Any chairs manufactured before January 2008 may not be available. A photo must be taken of controller (located on bottom of chair) & sent to customer service/engineering for verification.			
**Heat ONLY Identifiers				
H5	For chairs w/ heat only made 04-01-14 to present add identifier after TB option i.e. U0207XXTB XX			
H4	For chairs w/ heat only made 05-18-12 to 03-31-14 add identifier after TB option i.e. U0207XXTB XX			
H3	For chairs w/ heat only made 06-22-10 to 05-17-12 add identifier after TB option i.e. U0207XXTB XX			
H1	For chairs w/ heat only made 01-01-08 to 06-21-10 add identifier after TB option i.e. U0207XXTB XX Any chairs manufactured before January 2008 may not be available. A photo must be taken of controller (located on bottom of chair) & sent to customer service/engineering for verification.			

For additional options please see: [HM-HT OPTIONS PAGE, PIVOT TABLE OPTIONS PAGE,](#)

[IV POLE OPTIONS PAGE, OXYGEN TANK HOLDER OPTIONS PAGE](#)