

---

## Saucer Switches

---

### *New textures!*

Our popular saucer switches are now available with a new texture making it especially pleasing to all users. The slightest movement activates the device of your choice. Size: 7"D. Weight: 1¾ lbs.

### Operation:

1. The Saucer Switch does *not* require any batteries to operate.
2. Plug the Saucer Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Saucer Switch and your toy/device is tight. There should be no gaps.
5. Simply press anywhere on the Saucer Switch, to activate your toy/device.
6. The toy/device will only remain activated while the Saucer Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

### Troubleshooting:

**Problem:** The Saucer Switch does not activate your toy/device.

**Action #1:** Make sure that the Saucer Switch is on a flat surface (not tilted or vertical). This provides for optimal functioning.

**Action #2:** Make sure that the connection between the Saucer Switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

**Action #3:** Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

**Action #4:** Try a different adapter (if applicable) to rule this out as the source of the problem.

### Care of Unit:

The Saucer Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.