

Activates any toy or device!

A great new colorful somatosensory experience. Our bumpy switch was designed to provide tactile stimulation while activating your toy or device. *Size: 8"L x 4½"W x 1/8"H. Weight: ¼ lb.*

Operation:

1. Plug the Super Bumpy Switch into the jack on your switch-adapted toy/device.
2. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
3. Make sure that the connection between the Super Bumpy Switch and your toy/device is plugged in all the way. There should be no gaps.
4. Simply press anywhere on the textured yellow plate, to activate your toy/device.
5. The toy/device will only remain activated while the yellow plate is depressed. Once you release the yellow plate, the toy/device will turn off.

Troubleshooting:

Problem: The Super Bumpy Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Super Bumpy Switch and your toy/device is plugged in all the way. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Super Bumpy Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.