



## ULTRA-CUSHION


Description	Cover	Size (inches)	Reorder #	Size (inches)	Reorder#	Size (inches)	Reorder #
Foam Cushion	Vinyl Only	16 x 16 x 3½	754312 □	16 x 18 x 3½	754310 □	16 x 20 x 3½	754314 □
Foam Cushion	Vinyl w/Cloth	16 x 16 x 3½	754322 □	16 x 18 x 3½	754320 □	16 x 20 x 3½	754324 □
Gel-Foam Cushion	Vinyl w/Cloth	16 x 16 x 4	754332 □	16 x 18 x 4	754330 □	16 x 20 x 4	754334 □
Thin-Line Gel Foam	Vinyl w/LSI	16 x 16 x 2½	754339 □	18 x 16 x 2½	754340 □	16 x 16 x 1	914481 □
		18 x 16 x 1	914482 □	20 x 16 x 1.5	750142 □		
		20 x 16 x 1	914483 □	20 x 16 x 1.5	751043 □		
		16 x 16 x 1.5	751038 □				
		18 x 16 x 1.5	751041 □				

### PURPOSE

The Ultra-Cushion is recommended for wheelchair-bound individuals who require a firm-foundation cushion. Available in both foam-only and gel-foam versions.

• **GEL DECREASES PRESSURE AND INCREASES COMFORT** For maximum pressure relief, Ultra-Cushions are available with a self-contained chamber of heat-dissipating gel that evenly distributes resident's weight.

• **PROVIDES COMFORT** An ample layer of pliant high-density foam offers maximum cushioning for resident comfort.

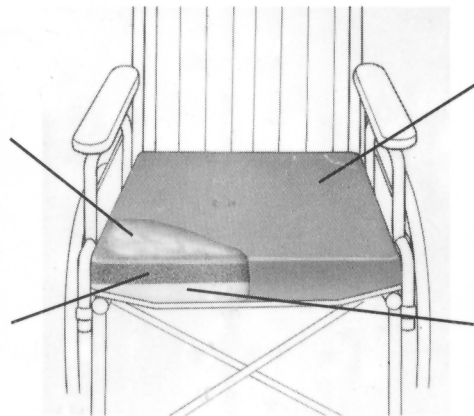

• **URINE CAN'T PENETRATE COVER** The Ultra-Cushion is protected by a factory-sealed, incontinent-proof vinyl cover.

• **ELIMINATES WHEELCHAIR SEAT HAMMOCKING** Noncompressing firm foam foundation eliminates the sling-seat hammocking that causes spinal pressure. The contoured foundation rests below wheelchair seat frame—your resident sits lower in the chair.

### IMPORTANT SAFETY FEATURE

All Ultra-Cushion covers—vinyl or cloth—have tie cords that prevent the cushion from slipping off the wheelchair seat. For user safety, these cords must be secured to the wheelchair frame.

### COVER CARE INSTRUCTIONS

Vinyl covers may be cleaned with a mild spray-type cleaner and damp cloth or sponge. Cloth covers can be removed for laundering. Do not launder or dry cloth covers at temperatures in excess of 180°F (82°C). Bleaching is not recommended.