

8500-8510  
Treatment Table



CUSTOMER INSTRUCTIONS

*PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT. If you have trouble understanding these instructions contact your dealer or Winco customer support, (800) 237-3377 before attempting to use this product; otherwise injury may occur.*

***Winco assumes no responsibility for damage or injury caused by improper assembly, installation, use, or maintenance of these products.***



1. READ AND FOLLOW ALL DIRECTIONS.
2. NEVER stand on table or INJURY MAY OCCUR.
3. NEVER allow user to sleep unattended on table or INJURY MAY OCCUR.
4. NEVER leave an unconscious user unattended on table or INJURY MAY OCCUR.
5. DO NOT use table for transporting user in or with ANY type of vehicle or trailer. Winco tables have not been tested or approved for use by an occupant in any type of vehicle or trailer.
6. NEVER leave an infant or child unattended on table or INJURY MAY OCCUR.
7. DO NOT put hands, feet or clothing into any openings when changing positions on table. Attendant MUST confirm that users hands and feet are safe while changing table positions or INJURY MAY RESULT.
8. Periodically, recheck tightness of all nuts, bolts and screws.
9. USE ONLY WINCO AUTHORIZED REPLACEMENT PARTS.
10. NEVER EXCEED the recommended **weight capacity of 400 lbs.** (181.4 kg).
11. SAVE THESE INSTRUCTIONS for future reference and training.

8500-8510 Weight Capacity = 400 lbs. (181.4 kg.)

Tools needed:

- Phillips-head screwdriver
- 7/16" wrench (multiple size wrench included)
- Scissors or box cutter
- - If using powered tools always start hardware together by hand first & wear safety goggles

*PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE PROCEEDING WITH ASSEMBLY*

**SEE "Parts Reference" (pg.3) for visual reference during assembly.**

1. After checking your product for any shipping damage, table needs to be positioned upside-down on cardboard or protective surface. Remove all packing material and hardware that was secured for shipping. Cut tape on corners of box so that cardboard will lay flat while you assemble table.
2. You will find two **Table Legs**, in-between and attached to the **Table Frame Rails**. Pivot up the (2) **Table Legs**.

**NOTE:** Each **Table Leg** has an **Upper Leg Brace** that need to align with two threaded holes in the wood of the **Upholstered Top**.

3. Install (4) ¼-20 x 1-½" screws through (2) holes in each upper brace of the **Table Leg**, and into (2) threaded holes (each end) in the wood of the **Upholstered Top**.

**NOTE:** **Do not** fully tighten at this time.

4. Place (4) ¼-20 x 1-½” carriage bolts into the (4) square holes through the **Lower Leg Brace** of the **Table Legs**. Heads of screws should face the ends of table, for better appearance.
5. Install the **Cross-brace**. Welds on the **Cross-brace** must face down toward floor when the table is upright. Attach with the (4) screws that you just installed on step 4 and secure with (4) ¼-20 nuts.

*If the **Optional Shelf** and **Reinforced Cross-brace** have been ordered; make sure that the smooth surface (side with tape) of the **Reinforced Cross-brace** will face upward (when the table is in the upright position) prior to fastening to the **Table Legs**.*

6. TIGHTEN ALL nuts & bolts.
7. Install the four (4) **Glide Feet** completely into the **Threaded Inserts** located on the bottom of the **Table Legs**.

### **Additional instructions for 8510 models ----- (8500 model go to step 11)**

8. Attach the **Face Slot Tube** so that one hole is aligned with the center hole of the **Upper Leg Brace** (flat side of Face Slot Tube faces table bottom). Position the opposite end of the **Face Slot Tube** over the 1” nylon spacer and the hole in table bottom. Install two (2) 1-3/4” Panhead Phillips screws through both ends of the **Face Slot Tube**, then tighten into the wood table bottom with a Phillips Head Screwdriver.
9. Install **Face Paper Holder** using (2) 1” “drywall-type” screws in pre-drilled holes on either side of Face Slot.
10. Install **Face Paper Cutter** on **Table Frame Rail** (where desired) opposite **Face Paper Holder**. Peel backing from tape on back of cutter, press cutter firmly into place.

*After peeling paper from adhesive tape:  
Use damp sponge to moisten tape to allow face paper cutter to be adjusted during install.  
Once tape is dry face paper cutter will stay in place.*

11. Using at least two people, turn table upright.
12. Place **Paper Holder Rod** through the (2) **Paper Rod Clamps**, (with paper roll in-between).
13. Adjust the (4) **Glide Feet** to level table.

### **Optional Shelf installation:**

14. Double-sided tape has been installed on the **Cross-brace**.
15. Before removing paper on tape, “test fit” the shelf. Shelf should be approximately ¼” from edges of Lower Leg Brace.
16. Once you are certain of where you will position the shelf, peel off the paper to expose adhesive.

*After peeling paper from adhesive tape:  
Use damp sponge to moisten tape to allow shelf to be adjusted during install.  
Once tape is dry shelf will stay in place.*

17. Carefully position shelf in place on adhesive.

**Vinyl Strap Adjustment** – Should the vinyl strap on your Winco table need to be adjusted, unscrew the screw that currently holds either side of it in place. Pierce another hole in the strap to give desired tension and re-insert screw.

**Wrinkles in Vinyl** – Storage and shipping can cause wrinkles of the vinyl on your Winco table. This is normal. Please allow 1-2 weeks at room temperature (approx. 70 degrees) to allow for vinyl to expand back to original appearance.



***Do not over tighten nuts and bolts. Periodically recheck all fasteners for tightness.***

# Parts Reference

