

8550 Recovery Couch



CUSTOMER INSTRUCTIONS

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT. If you have trouble understanding these instructions contact your dealer or Winco customer support, (800) 237-3377 before attempting to use this product; otherwise injury may occur.

Winco assumes no responsibility for damage or injury caused by improper assembly, installation, use, or maintenance of these products.



1. **READ AND FOLLOW ALL DIRECTIONS.**
2. NEVER stand on table or INJURY MAY OCCUR.
3. NEVER allow user to sleep unattended on table or INJURY MAY OCCUR.
4. NEVER leave an unconscious user unattended on table or INJURY MAY OCCUR.
5. DO NOT use table for transporting user in or with ANY type of vehicle or trailer. Winco tables have not been tested or approved for use by an occupant in any type of vehicle or trailer.
6. NEVER leave an infant or child unattended on table or INJURY MAY OCCUR.
7. Periodically, recheck tightness of all nuts, bolts and screws.
8. USE ONLY WINCO AUTHORIZED REPLACEMENT PARTS.
9. NEVER EXCEED the recommended **weight capacity of 400 lbs.** (181.4 kg).
10. SAVE THESE INSTRUCTIONS for future reference and training.

8550 Weight Capacity = 400 lbs. (181.4 kg.)

Tools needed:

- Phillips-head screwdriver
- 7/16" wrench (multiple size wrench included)
- Scissors or box cutter
- Cardboard shipping container (stays under table during assembly to protect upholstery)
- - If using powered tools, always start hardware together by hand first & wear safety goggles

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE PROCEEDING WITH ASSEMBLY

SEE "Parts Reference" (pg.3) for visual reference during assembly.

1. After checking your product for any shipping damage, table needs to be positioned upside-down on cardboard or protective surface. Remove all packing material and hardware that was secured for shipping. Cut tape on corners of box so that cardboard will lay flat while you assemble table.

2. You will find two **Table Legs**, in-between and attached to the **Table Frame Rails**. Pivot up the (2) **Table Legs**.

NOTE: The table legs have braces that need to align with two threaded holes in the wood of the **Upholstered Top**.

3. Install (4) ¼-20 x 1-½" screws through (2) holes in each **Upper Leg Brace** of the **Table Legs**, and into (2) threaded holes (each end) in the wood of the **Upholstered Top**.

NOTE: Do not fully tighten at this time.

4. Place (4) ¼-20 x 1-½" carriage bolts into the (4) holes through the **Lower Leg Brace** of the **Table Legs**. Heads of screws should face the ends of table, for better appearance.
5. Align the **Cross-brace** (smoothest side up) with the (4) screws that you just installed on step 4, and fasten with (4) ¼-20 nuts.

If the **Optional Shelf** and **Reinforced Cross-brace** have been ordered; make sure that the smooth surface (side with tape) of the **Cross-brace** will face upward (when the table is in the upright position) prior to fastening to **Table Legs**.

6. TIGHTEN ALL nuts and bolts at this time.
7. Install the four (4) **Threaded Inserts** with **Glide Feet** into the ends of the table legs.
8. Using at least two people, turn table upright.
9. Remove (1) of the white plastic caps on the **Paper Rod** and install into the (2) **Paper Rod Clamps**, replace cap. Adjust (4) 5/16" **Glide Feet** to level table.

Optional Shelf installation:

10. Double-sided tape has been installed on the **Reinforced Cross-brace**.
11. Before removing paper on tape, "test fit" the shelf. Shelf should be approximately ¼" from edge of the **Lower Leg Brace** of the **Table Legs**.
12. Once you are certain of where you will position the shelf, peel off the paper to expose adhesive.

*After peeling paper from adhesive tape:
Use damp sponge to moisten tape to allow shelf to be adjusted during install.
Once tape is dry shelf will stay in place.*

13. Carefully position shelf in place on adhesive.

Vinyl Strap Adjustment – Should the vinyl strap on your Winco table need to be adjusted, unscrew the screw that currently holds either side of it in place. Pierce another hole in the strap to give desired tension and re-insert screw.

Wrinkles in Vinyl – Storage and shipping can cause wrinkles of the vinyl on your Winco table. This is normal. Please allow 1-2 weeks at room temperature (approx. 70 degrees) to allow for vinyl to expand back to original appearance.

Do not over tighten nuts and bolts. Periodically check all fasteners for tightness.

Parts Reference

