
Relaxing Neck & Shoulder Heating Pad

Dry Heat

Item: 619-5136-1900, 18" x 15" (45.8cm x 38.1 cm)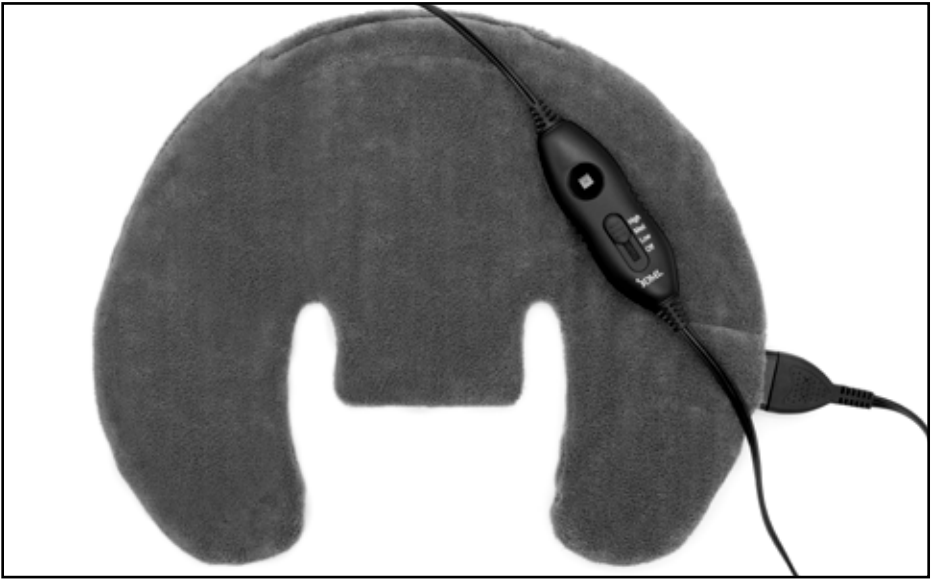

Please read this guidebook
completely before operating the unit.

SAVE THESE INSTRUCTIONS

**DETAILED
GUIDEBOOK**

English • Spanish

IMPORTANT SAFETY INSTRUCTIONS

DO NOT DESTROY

WHEN USING THIS OR ANY OTHER ELECTRICAL APPLIANCE, ALWAYS FOLLOW THESE BASIC SAFETY INSTRUCTIONS. CONSULT THE INCLUDED INSTRUCTION MANUAL FOR DETAILED PRODUCT USE AND CARE INFORMATION.

WET PROOF VARIABLE HEAT ELECTRIC HEATING PAD FOR HOUSEHOLD/PERSONAL USE ONLY


DANGER: BURNS WILL RESULT FROM IMPROPER USE. TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK AND FIRE, THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE FOLLOWING INSTRUCTIONS:

- 1. READ ALL INSTRUCTIONS.**
- DO NOT USE WHILE SLEEPING.
- BURNS MAY OCCUR REGARDLESS OF CONTROL SETTING. CHECK THE SKIN UNDER THE PAD FREQUENTLY TO AVOID BURNING AND BLISTERING.
- DO NOT USE ON AN INFANT OR ON AN ANIMAL.
- THIS PAD IS NOT TO BE USED BY OR ON AN INVALID, SLEEPING OR UNCONSCIOUS PERSON, OR A PERSON WITH POOR BLOOD CIRCULATION UNLESS CAREFULLY ATTENDED.
- DO NOT USE A HEATING PAD ON AREAS OF SENSITIVE SKIN.
- NEVER USE PAD WITHOUT THE REMOVABLE COVER IN PLACE.
- DO NOT USE IN AN OXYGEN ENRICHED ENVIRONMENT OR NEAR EQUIPMENT WHICH STORES/EMITS OXYGEN.
- PLACE PAD ON TOP OF AND NOT UNDER THE PART OF THE BODY NEEDING HEAT.
- DO NOT SIT ON, OR AGAINST PAD; DO NOT CRUSH PAD. AVOID SHARP FOLDS.
- DO NOT PUT THIS PAD IN A MICROWAVE.
- NEVER PULL THIS PAD BY THE SUPPLY CORD. DO NOT USE CORD AS A HANDLE.
- UNPLUG WHEN NOT IN USE.
- CAREFULLY EXAMINE INNER VINYL COVER BEFORE EACH USE. DISCARD THE PAD IF VINYL COVERING SHOWS ANY SIGNS OF DETERIORATION (SUCH AS BLISTERING OR CRACKING).
- LOOP CORD LOOSELY WHEN STORING AS TIGHT WRAPPING MAY DAMAGE CORD AND INTERNAL PARTS.
- THIS UNIT SHOULD NOT BE USED BY CHILDREN WITHOUT ADULT SUPERVISION.
- DO NOT TAMPER WITH THIS PAD IN ANY WAY.
- NEVER USE PINS OR OTHER METALLIC MEANS TO FASTEN THIS PAD IN PLACE.
- THERE ARE NO USER-SERVICEABLE PARTS. IF FOR ANY REASON THIS PAD DOES NOT FUNCTION SATISFACTORILY, SEE WARRANTY FOR CONSUMER SERVICE INFORMATION.
- DO NOT USE THIS PAD WITH LINIMENT, SALVE OR OINTMENT PREPARATIONS, ESPECIALLY ONES THAT CONTAIN HEAT-PRODUCTION INGREDIENTS AS SKIN BURNS MAY RESULT.
- DO NOT BEND OR PINCH CORD; DO NOT USE IF CORD IS DAMAGED.
- NEVER LEAVE THE HEATING PAD UNATTENDED, ESPECIALLY IF CHILDREN ARE PRESENT.
- USE THIS PAD ONLY ON A 120 VOLT AC CIRCUIT.

CAUTION: READ AND FOLLOW ALL INSTRUCTIONS BEFORE USE. SAVE THESE INSTRUCTIONS.

POLARIZED PLUG

This product comes equipped with a polarized electrical plug (one blade is wider than the other) and will only fit into the outlet one way. This is meant to reduce the risk of an electrical shock. If the plug does not fit into the outlet, reverse the plug. If it still does not fit, consult a qualified electrician. **DO NOT ATTEMPT TO MODIFY THIS PLUG OR CIRCUMVENT THIS SAFETY FEATURE.**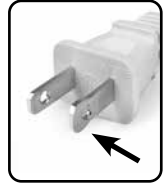

FCC STATEMENT AND EMC COMPATIBILITY

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment Generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

CAUTION: Any changes or modification to this device could void the user's authority to operate the equipment.


INSTRUCTIONS FOR USE

IMPORTANT: DO NOT REMOVE OR ATTEMPT TO REMOVE THE FABRIC COVER FROM THIS DEVICE, SERIOUS BURNS OR INJURY COULD RESULT

Please read all warnings and caution statements within this manual before using.

FOR DRY HEAT APPLICATIONS ONLY

1. Connect the power cord to the receptacle on the heating pad by matching up the arrows on the both the power cord and receptacle. The plug on the power cord will only fit into the receptacle one way.
2. Insert the plug into an appropriate 120 volt AC electrical outlet, noting the plug is polarized and will only fit one way.

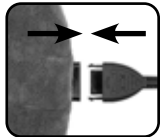3. Place the heating pad on shoulders. The tall collar should sit on the back of the neck while the two front edges connect with plastic snap to help keep the wrap in place. **NOTE: Neck wrap should comfortably fit around the neck and NEVER restrict the airway.**

WARNING: THE PAD SHOULD ALWAYS BE PLACE ON TOP OF THE TREATMENT AREA. DO NOT PLACE PAD UNDERNEATH TREATMENT AREA AS TEMPERATURES CAN RISE TO UNINTENDED/UNCOMFORTABLE LEVELS.

4. Using the controller, slide the button to reach the desired heat setting (Lo, Med, Hi). To turn the heating pad off, slide the button to the 'Off' position.
5. Pay close attention to the heat level during use and adjust the setting as needed.
6. The heating pad will reach the desired temperature in approximately 10~20 minutes.

NOTE: This heating pad will automatically turn OFF after 2 hours of use. Unplug heating pad when not in use.

USING THE CONTROLLER

Heat Level	Timing to Reach Heat Level
Low	10 min
Medium	10 min
High	20 min

CARE AND MAINTENANCE

WARNING: ALWAYS DISCONNECT THE HEATING PAD FROM THE ELECTRICAL OUTLET AFTER USE. ALWAYS DISCONNECT THE POWER CORD FROM THE NECK WRAP BEFORE ATTEMPTING TO CLEAN IT.

WASHING INSTRUCTIONS

- **Hand Washing:** use a mild detergent and gently wash in cold water
- **Machine Washing:** disconnect the power cord and wash in cold water on gentle cycle only.

DRYING INSTRUCTIONS

- Hang dry or lay flat. DO NOT machine dry
- Heating pad must be completely dry before reconnecting the power cord. DO NOT use unless the heating pad is 100% dry.

STORING INSTRUCTIONS

- Heating pad must be completely dry before storing
- Be sure to store in a cool, dry location
- Avoid sharply folding the heating pad, as cracks may start to appear over time in the vinyl

DO NOT ATTEMPT TO CLEAN OR WASH THE HEATING PAD IN ANY OTHER METHOD THAN AS DESCRIBED ABOVE. DOING SO MAY CAUSE DAMAGE AND VOID THE WARRANTY.

LIMITED 1-YEAR WARRANTY

This warranty covers only normal use, and does not apply for use in any clinical or commercial applications. The warranty is voided if the product is misused or abused in any manner.

If the unit fails to operate during the first year from the time originally purchased, please call our Customer Care Help Line at 800-622-4714 and we will try to resolve the issue as soon as possible. If the warrantor determines the unit failed to operate due to a manufacturing defect, the unit will be replaced at the option of the warrantor. The purchaser will be responsible for the prepaid postage along with \$9.99 to cover return shipping and insurance. The purchaser must return the original unit prior to a replacement product being provided. Replacement of the product is the only remedy covered by the Limited 1-Year Warranty.

We make no other express or implied warranties, arising by operation of law or otherwise, or any warranty of merchantability or fitness for a particular use or purpose whether or not the use or purpose has been disclosed to the warrantor in specifications, drawings or otherwise, and whether or not the warrantor's products are specifically designed and/or manufactured by the warrantor for the buyer's use or purposes, except for the limited warranty stated above. The warrantor will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss, including, but not limited to, damage to or loss of other property or equipment and personal injuries, whether to purchaser or others. The warrantor shall in no event be liable to the purchaser for any amount in excess of the cost to replace the product.

SPECIFICATIONS

- Product Name: DMI Neck & Shoulder Heating Pad
- Item Number: 619-5136-1900
- Color: Gray
- Fabric: 100% polyester
- Voltage: 120v AC
- Frequency: 60 HZ
- Wattage: 60w
- Amperage: 0.5A
- Cord Length: 9ft (274cm)

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