




Rolling Walker

Provides added stability while walking



1 Part Check

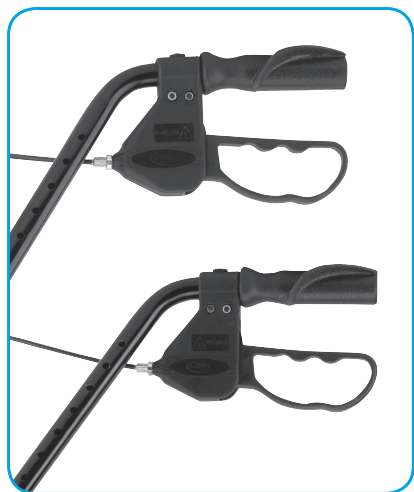

2 Handles



1 Backrest



6 Hand screws



2 Front wheels



2 Rear wheels



6 Bolts with plastic washers

Note: actual items may vary slightly from what is shown.

6 Installing Backrest





1




2



5 Adjusting the handles

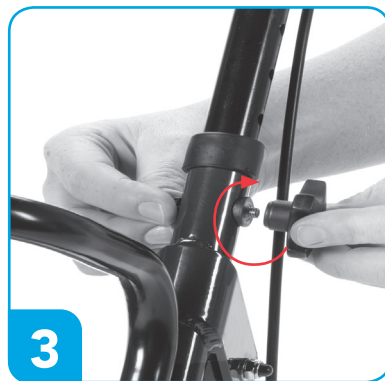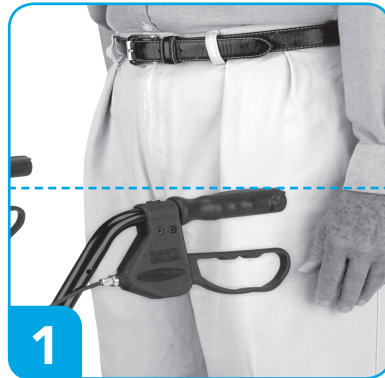

2 Front wheel assembly

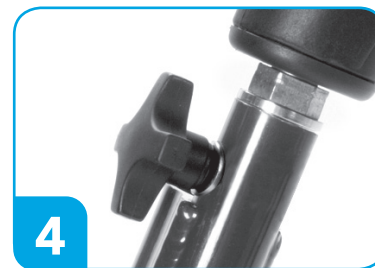




3 Rear wheel & brake assembly

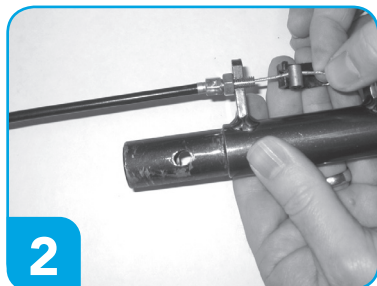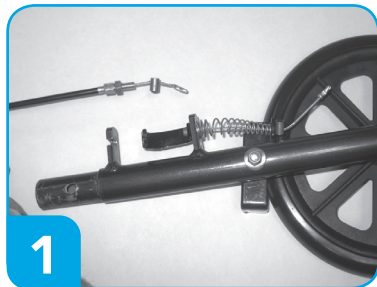

Brake Assembly



4 Unfolding walker & installing handles

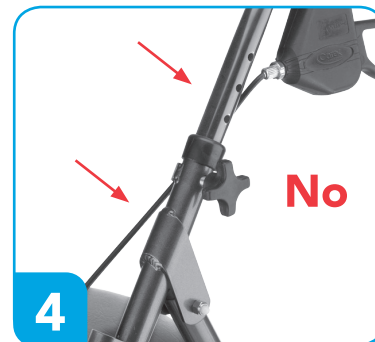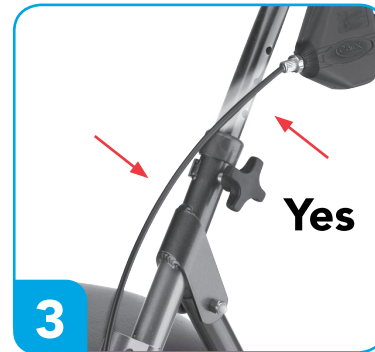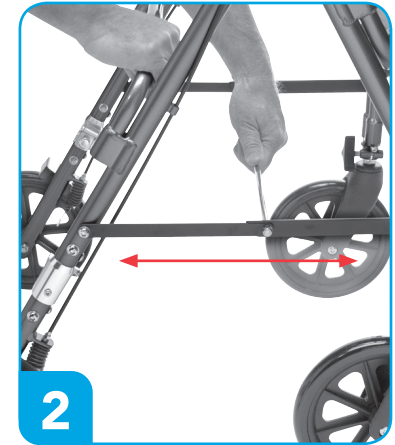

Handle brakes lock for safety.
1. Squeeze handle to brake
2. Press handle down to engage lock

