



Carex Health Brands



Mobility & Comfort for
your active lifestyle



Explorer[®] Walker

Folding walker with wheels and glides



Carex Health Brands

1 - Part Check: **FRAME**

LEGS (x2)



2 - Unfold Walker Frame



3 - Make sure to hear the "click" to ensure walker is locked into place



4 - To attach legs, flip walker frame over and insert legs by pressing in the pop pins until you hear a click



Align wheels with front cross bar on outside of frame

5 - Adjust to proper height using pop pin



6 - Easily fold or unfold walker by pushing down on the release button

