

---

---

**4-in-1 Joystick Switch #7301**

---

---

**USER'S GUIDE**



Need to activate more than one switch? Our joystick is great for the user who needs to access multiple switch adapted devices or a single device with multiple switch inputs such as our adapted TV remote module (#5150). Can also be used to teach directionality – left, right, up, down. Size: 5¾"D x 4½"H. Weight: ¾ lb.

1. Connect up to four cords to external switch enabled toys or devices via the four 1/8-inch jacks on the side of the Joystick. If you need to use 1/4- to 1/8-inch adaptors, they must be mono adaptors, not stereo.
2. To operate a toy or device connected to the first jack, push the Joystick in the corresponding direction for that toy/device. To activate any additional connected toys/device repeat as before.
3. The toy/device will only remain activated while the Joystick is being actuated in one of the four directions. Once you release the Joystick, the toy/device will turn off.

### **Troubleshooting:**

**Problem:** The 4-in-1 Joystick switch does not activate your toy/device.

**Action #1:** Make sure that the 4-in-1 Joystick switch is on a flat surface (not tilted or vertical). This provides for optimal functioning.

**Action #2:** Make sure that the connection between the 4-in-1 Joystick switch and your toy/device is plugged in all the way. There should be **no** gaps. This is a common error and an easy fix.

**Action #3:** Try a different switch with your toy/device to rule out the 4-in-1 Joystick switch as the source of the problem.

**Action #4:** Try a different adapter (if applicable) to rule this out as the source of the problem.

### **Care of Unit:**

The 4-in-1 Joystick switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.