

Adjustable Wing & Bolster

**BRODA's Adjustable Wings are the ideal option for superior upper lateral support!**


Adjustable vertically as well as angular, our Adjustable Wings accommodate a wide variety of users. Good for:

- ✓ Short or tall residents
- ✓ Narrow or wide shoulders
- ✓ Swinging away for easy side transfers

The Adjustable Wings are both functional AND durable. Engaging holes along the wing frame allow the wing to lock into place, without slipping out of position.

- Adjustable in angles of 0°, 22°, 45° and full swing away at varying heights
- 16 gauge tubular steel frame
- Can be used on BRODA models: Elite, Basic, Midline, Pedal Chair and Pedal Rocker
- Available factory or non-factory installed, and may be retro-fitted to existing Broda chairs
- Must be used in conjunction with Adjustable Wing new Shoulder Bolsters


## Easy as 1-2-3!

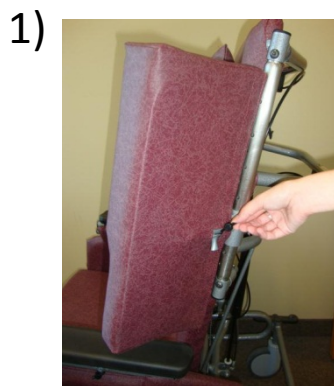

Loosen wing nut



Squeeze pin towards front of chair while adjusting wing to desired angle and height increment. Ensure wing locks into place

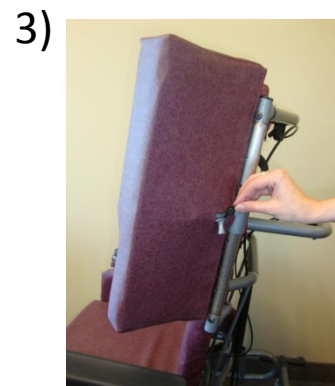


Once wing is locked into place, tighten wing nut.


## Variety of Angles for Positioning:




0 degrees

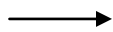

22 degrees

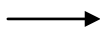

45 degrees

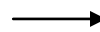

Swing away

- The various angles of the adjustable wings allow for a more customized fit for your resident!
- Plus, the swing away feature eliminates the need for full wing removal- making transfers quicker and easier to save time, and eliminate the chance of losing a removable part.



Detent Pins

### To Attach the Wing:

- Using a 3/16" Allen Key, loosen the bolts on the bottom end of the wing.
- Insert lower and upper ends of the wing into the lower and upper receptacles, respectively together, while depressing detent pins.
- Release detent pins in holes to secure wing into place.
- Once the wing is secured in place, use the 3/16" Allen Key to tighten the bottom bolts.

**Not recommended for users with severe agitation or involuntary movement.**