



southpaw enterprises, inc.®

Instruction Sheet

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

Advantage Line 2-in-1 Bolster 120060



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment. If this Instruction Sheet is misplaced, please call or write for a replacement. (See address below).

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

MATERIALS LIST:

- 1 - 2-in-1 Bolster Swing
- 3 - Safety Snaps
- 1 - Instruction Sheet
- 1 - Safety Snap Instruction Sheet
- 1 - Maintenance Checklist

The Southpaw 2-in-1 Bolster can be used as a bolster swing or, with the bolster detached, can be used as a trapeze bar. The bolster comes with a safety snap to hang it from a single suspension point but by adding another snap it can be hung from two points between 30" to 36" apart.

To use as a trapeze bar, remove the safety snaps from the web loops. Leave the safety snaps on the ends of the bolster ropes so that they won't hit clients using the trapeze bar. Reassemble in reverse order. **Note:** Make sure the safety snaps are properly closed when in use. Failure to close the gate of the safety snaps may cause it to bend or break.

MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED:

- Can be used with an 8 foot ceiling.
- Caution must be taken to keep a clear distance of 6 feet from walls and other obstacles.

HEIGHT ADJUSTER AND ROTATIONAL DEVICE NEEDED:

- For ease of adjustment, a Southpaw Height Adjuster (Part # 5000) is recommended.
- A Southpaw Safety Rotational Device (Part # 2000) should be used to prevent damage to your equipment.

WORKING LOAD: 300 POUNDS

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed

- Check ropes for fraying (be sure to check end of rope under black boot).
- Check Safety Snaps for wear. Refer to the Safety Snap Instruction Sheet for detailed maintenance checklist.
- Check ropes for loose or missing metal rope clips. If loose or missing rope clips are found, discontinue use until repaired.
- Check bar for cracks or splinters. If cracks are found discontinue use. If splinters are found sand until smooth and refinish with clear lacquer.

STORAGE PROCEDURES:

- Store out of Direct Sunlight.
- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS:

- Check for client allergies first.
- Check with your infectious disease department before using these procedures.
- Wash using a damp cloth with mild soap and water. Dry before using.
- Do not use abrasive cleaners.

CAUTION

- ◆ Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.
- ◆ Floor mats of proper thickness should be positioned properly under and around all suspended equipment.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

