


⚠ CAUTION

See the Alliance Patient Lift User Manual for additional instructions.

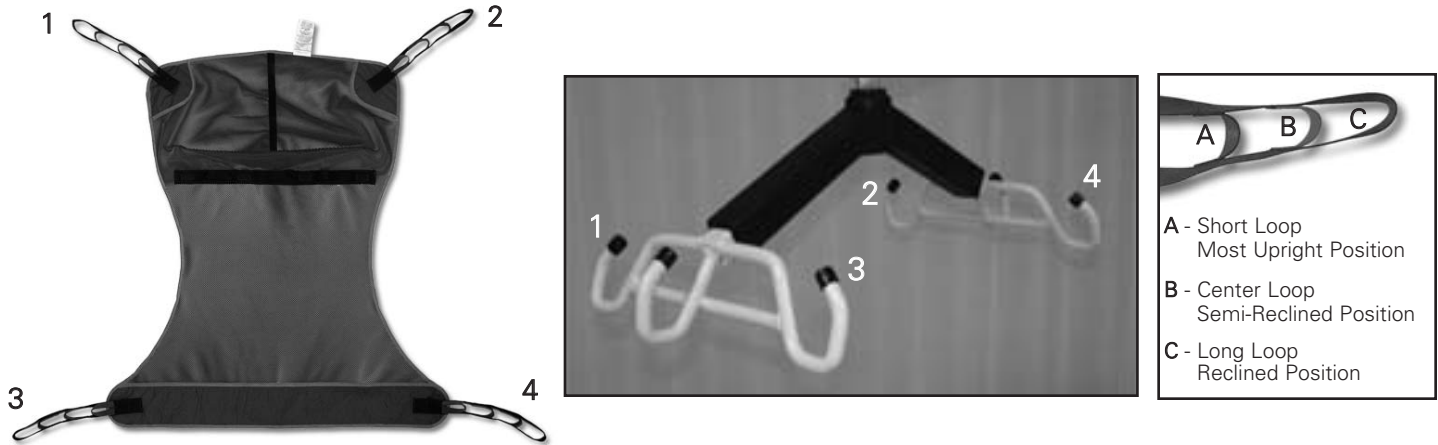

Installation Instructions:

1. Check the sling weight capacity for the patient being lifted. Do not exceed the maximum safe working load of 400 lbs (181 kg).
2. Review the following procedures carefully, prior to attempting lift.
3. Position the sling under and behind the patient's back with the color adjustment strap outside. Straps 1 and 2 should be at the patient's shoulder height (see the above image of the Invacare Compatible Sling for callout of straps).
4. Carefully, with the help from another caregiver, pull the sling straps 3 and 4 forward to the bend of the patient's knees (**as shown in Figure 1**).
5. Push the lift towards the patient. Widen the base of the lift by pressing the right foot pedal down (**as shown in Figure 2**). Apply the brakes on both rear casters.
6. Hook all straps to the corresponding hook on the spreader bar (see the above images of the Invacare Compatible Sling and the spreader bar).

NOTE: The loops on the Invacare Compatible sling straps provide multiple patient comfort options. Each patient will find the optimum comfort setting for their own body. Using loops closer to the patient's body will generally accelerate the lift. Regardless of which loop is used, make sure the same color loop is used on the opposite side.

7. Release the brakes to allow the lift to properly balance the weight of the patient before lifting.
8. Proceed with lift and transfer of the patient.



Figure 1

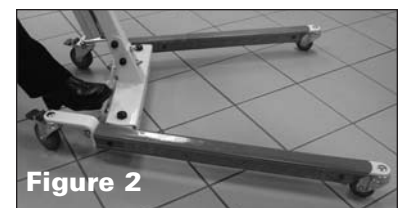


Figure 2

Alliance™ Slings

Universal Sling

This "Split Leg" sling is designed to meet most general patient lifting needs and can easily be applied or removed from a patient while seated or reclining. Available in both mesh and deluxe padded versions, the universal sling features six-point connectivity. The universal mesh sling is designed to allow maximum breathability and quick drying for bathing applications. The deluxe padded sling is designed to provide more comfort at the pressure points.

Universal Mesh Sling

- 19501 Medium, 400 lb (181 kg) max.
- 19502 Large, 400 lb (181 kg) max.

Universal Deluxe Padded Sling

- 19504 Medium, 400 lb (181 kg) max.
- 19505 Large, 400 lb (181 kg) max.
- 19506 Bariatric, 600 lb (272 kg) max.

Hammock Sling

This deluxe padded sling features six-point connectivity and provides additional support to the entire body and is the best choice for patients with limited muscle control.

- 19508 Medium, 400 lb (181 kg) max.
- 19509 Large, 400 lb (181 kg) max.

Toileting Sling

This deluxe padded sling is designed to provide unobstructed access to the perineal area, while allowing easier clothing adjustments than standard commode opening style slings.

- 19512 Medium, 400 lb (181 kg) max.
- 19513 Large, 400 lb (181 kg) max.

Stand Assist Sling

This deluxe padded sling features soft support panels under the arm and lumbar area for added comfort. The unique cross over strap and non-slip inner pad ensure the sling stays in place and does not ride up during lifting. An optional buttocks strap is available for initial lift assistance.

- 19515 Standard, 30"-50" (76 cm - 127 cm) torso
- 19516 Large, 38"-62" (97 cm - 163 cm) torso
- 19517 Buttock Support Strap

Pediatric Sling

Available in both mesh and deluxe padded versions, the pediatric sling is designed with head support and a padded belt to cradle the patient in a secure non-slip position.

- 19500 Standard Mesh, 18"-28" (46 cm - 71 cm) torso, 100 lb (45 kg) max.
- 19503 Standard Padded, 18"-28" (46 cm - 71 cm) torso, 100 lb (45 kg) max.

Invacare® Compatible Sling

Available in a mesh or solid body style with or without commode opening, this four-point connecting sling is compatible to the sling size and weight specifications offered by Invacare®.

Mesh Full Body Sling

- 19110 Medium, 450 lb (204 kg) max.
- 19111 Large, 450 lb (204 kg) max.
- 19140 Bariatric, 600 lb (272 kg) max.

Mesh Full Body Sling with Commode Opening

- 19114 Medium, 450 lb (204 kg) max.
- 19115 Large, 450 lb (204 kg) max.
- 19116 X-Large, 450 lb (204 kg) max.
- 19141 Bariatric, 600 lb (272 kg) max.

Solid Full Body Sling

- 19112 Medium, 450 lb (204 kg) max.
- 19113 Large, 450 lb (204 kg) max.
- 19117 X-Large, 450 lb (204 kg)

Invacare® is a registered trademark of Invacare Corporation.

© 2005 Encore Medical, L.P.

19547A



4717 Adams Road
Hixson, TN 37343 U.S.A.
1-423-870-2281
1-800-592-7329 U.S.A.
1-800-361-6661 CANADA
+ 1-423-870-2046 OUTSIDE U.S.A. FAX
www.chattgroup.com