




Fitting Protocol

Amputee Sling
Model 25040, 25140

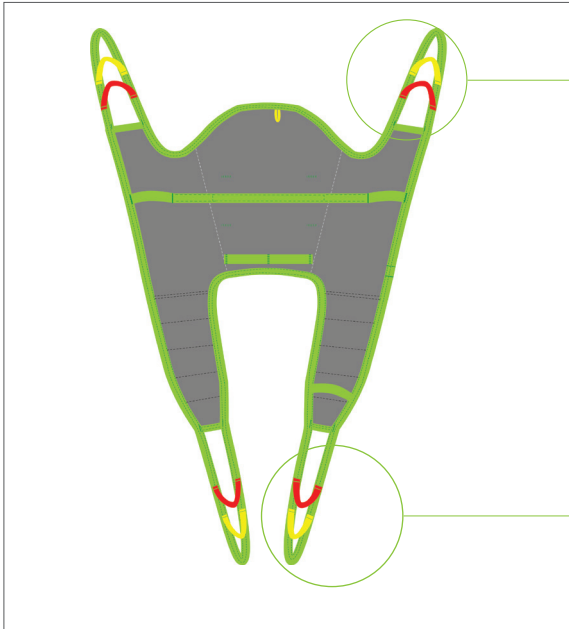

- Green Strap
- Yellow Strap
- Red Strap

Extension Loops


- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

- Green Strap
- Yellow Strap
- Red Strap

Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.

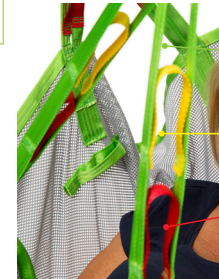

- Green Strap**
Increased Recline
Less hip flex and greatest angle
- Yellow Strap**
Semi-Recumbant
Mid-hip angle
- Red Strap**
Upright Position
Least hip angle

Match colours on both sides of the patient for a 90 degree angle in the hips.

Maximum User Weight: 300kg / 660lb


Model 25040 - Polyester

Model 25140 - Polyester Net


Transfer Situation:



Leg Placement:

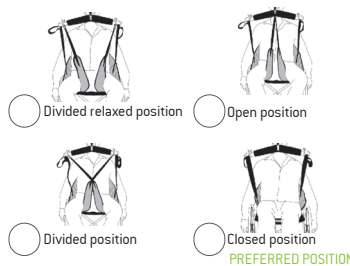

Suspension:

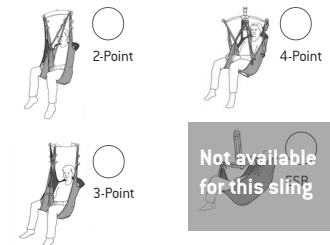

Available sizes:

Indicate size required



Accessories required:



- 21146 Leg Padding - Small
- 21146 Leg Padding - Medium
- 21146 Leg Padding - Large



- 25102 Reinforced Leg Support - One Size



- 50866 Upholstered Cover for Hanger Bar. **Do not use in wet environments.**



- 25145.*
Waist Belt
(Velcro)
- | | |
|----------------------|---------------------|
| *Junior Sizes | *Adult Sizes |
| -40B (40cm/15.7") | -60 (60cm/23.6") |
| -50B (50cm/19.7") | -70 (70cm/27.6") |
| -60B (60cm/23.6") | -80 (80cm/31.5") |
| | -90 (90cm/34.4") |



- 25146.*
Waist Belt
(Safety Buckle)
- Size (eg. -60B)
- | |
|---------------------------------|
| *Adult Sizes |
| -100 [†] (100cm/39.4") |
| -110 (110cm/43.3") |
| -120 (120cm/47.3") |
| -130 (130cm/51.2") |
- [†]only for Model #25146

Patient Information:

Name	
Facility	Room
Lift Used	Hanger Bar Used

The Amputee Sling is a low height model with divided leg parts. It features a reinforced back and is suitable for patients with double-sided or single leg amputations as well as patients with hip problems. Because of the slightly recline position, it is also suitable for patients with an increased risk of sliding out of the sling. The user of this sling must have good head control. The Amputee sling is available in Polyester or Polyester Net fabrics and can be used with an optional belt for added security.

Additional Instructions

To staff/caregiver:

To patient:

Fitting Protocol Completed by:

Name	
Date	Position/Title



Important! This fitting protocol does not replace the user manual. Read all instructions before use!