

## Two Year Limited Warranty

Your Inspired by Drive branded product is warranted to be free of defects in materials and workmanship for two years of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Inspired by Drive device or this warranty, please contact an authorized Inspired by Drive dealer.



2017 Medical Depot, Inc. All rights reserved.

Drive DeVilbiss Healthcare

99 Seaview Boulevard

Port Washington N.Y. 11050 USA

Made in China



## Tusky Tilt & Recline Positioning Chair

Item # APT 3000



EU Authorized Representative  
Drive Medical LTD  
Ainley's Industrial Estate  
Elland, West Yorkshire,  
United Kingdom HX5 9JP

[www.inspiredbydrive.com](http://www.inspiredbydrive.com)

## assembly:


1. Remove all positioning components from box.
2. Remove bolts (A) and insert back mounting bracket (AG) in the following order:
  - Seat pan (AH)
  - Back mounting bracket (AG)Reinsert bolts.
3. Lift up back to desired angle and seat depth and tighten bolts (A).
4. Insert arm rest assembly into mounting brackets (C).




5. Insert laterals in bracket (B) and tighten knob (V)



6. Insert footrest assembly (D) into bracket (F) under the seat of the chair and tighten knob (Y).
7. Insert abductor (G) into round receptacle bracket (Z) and tighten knob (E).

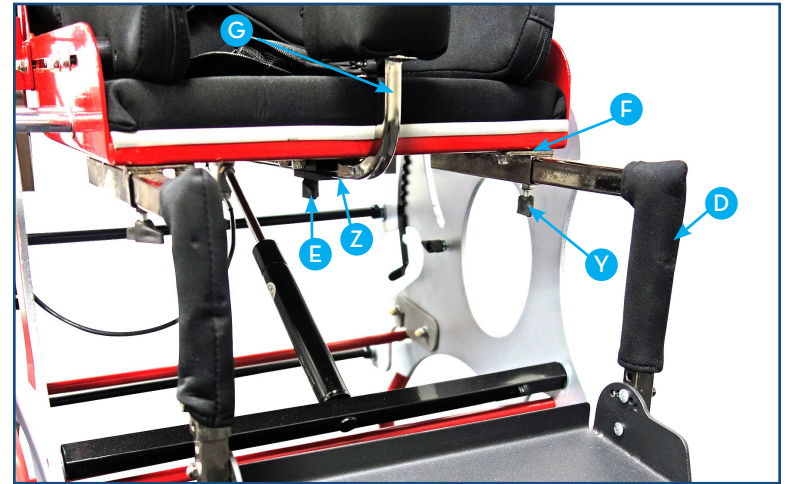



8. Insert vertical pole of headrest (H) into bracket (I) on back of chair and turn knob (J) to tighten. Tighten allen screws (K) on triangular bracket (L) and on the back of the headrest cushion (M)

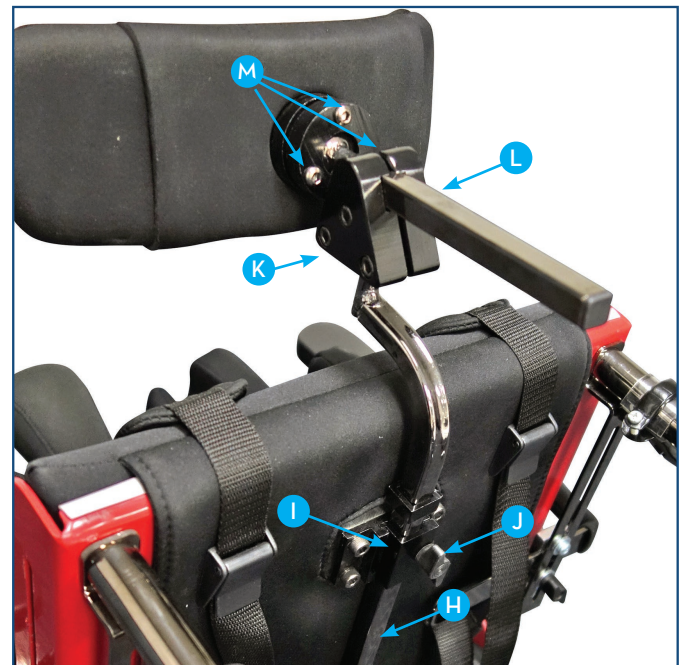

## adjustments:


### TILT POSITION

Open knob (N) and slide the securing bracket (O) away. Press tilt actuator (P) and push down on chair while holding on to both handles (Q) to desired tilt angle. Return securing bracket (O) and tighten knob (N).

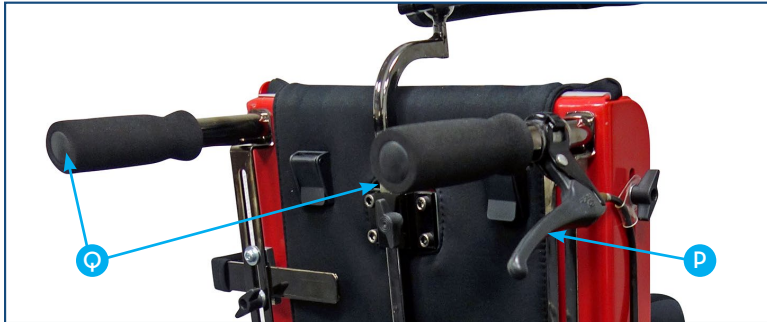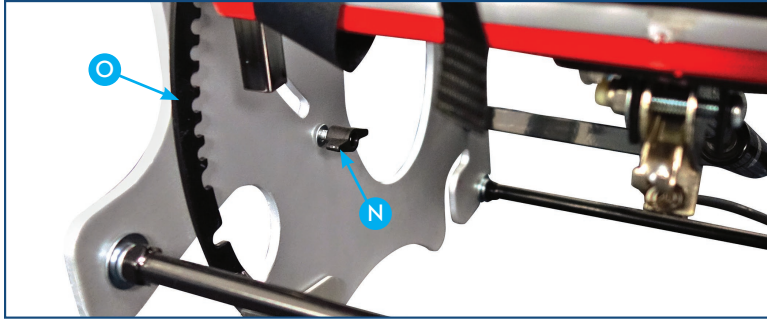

### SEAT TO BACK ANGLE AND SEAT DEPTH

Loosen bolts (A) and position back at desired depth. Place seat to back angle at desired setting.

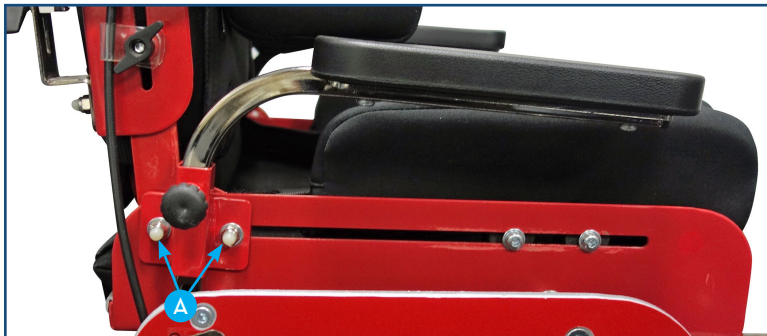

### ARM RESTS


Height - Loosen knob (S) and adjust armrest assembly.

Depth - Loosen bolts (AF) and adjust.



### SEAT WIDTH

There are two sets of hip guides included with this product. The set you use will determine the seat width. They can be switched by removing bolt (T) and replacing with the other set. Each set comes with its own bolts.

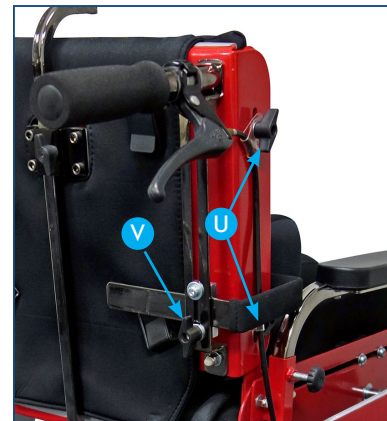

### BACK HEIGHT

Back height can be adjusted by loosening knobs (U) and raising or lowering back.

### LATERALS

Laterals can be adjusted in height and width. Loosen knob (V) and adjust height and width.

## HEADRESTS

### Height


Loosen knob (J) and raise or lower vertical assembly pole.

### Depth

Loosen 3 Allen screws (K) on triangular rotating bracket and push/pull headrest to desired position. For additional depth adjustability, mount the vertical assembly pole with the curve either facing inward or outward.

### Angle

Loosen 3 Allen screws (M) on circular bracket on headrest and tilt headrest to desired angle. Additional angle adjustability can be achieved by loosening the 3 Allen screws (K) on the triangular rotating bracket and adjust accordingly.

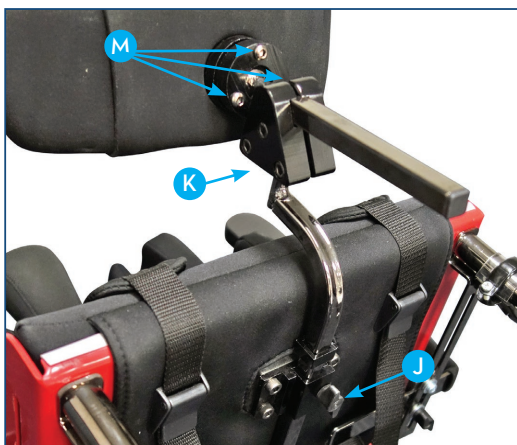

## FOOTRESTS


Height - Loosen bolts (W & X) and raise or lower bracket.

Angle - Loosen bolt (X).



## ABDUCTOR

Abductor can be removed and adjusted by loosen the knob (Z). Slide the abductor in or out for proper depth and tighten the knob.

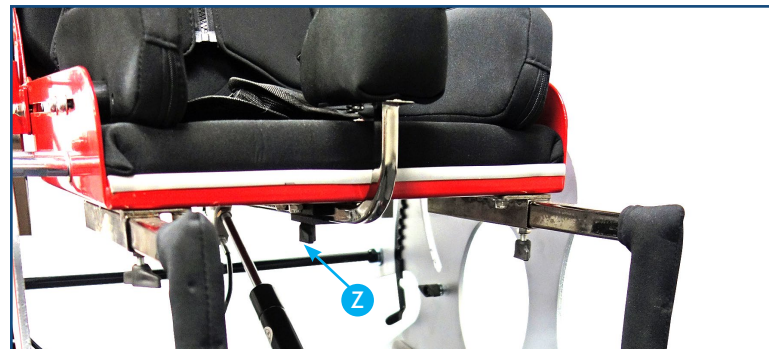


## LAP BELT

Length - Use buckle &/or change the position of the hook and loop straps under the seat.

## HARNESS

The top and side hook and loop straps on the back of the chair adjust the length of the straps.

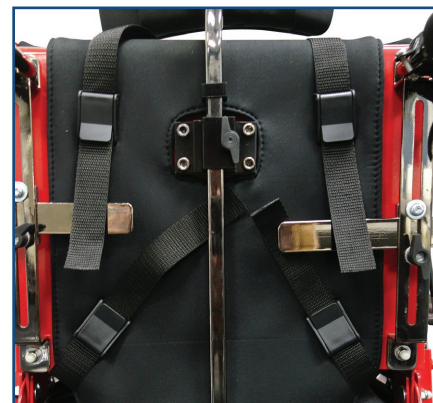

## ACCESSORIES BASE

APT-A0014 – High base

APT-A0015 – Low base

Mount chair onto either base. Lock chair onto base by sliding bracket (AC) over cross bar (AD).

