

AQ-1000 Beach and Trail Wheelchair

1. Insert the push rail with handles into the side rails as shown in photo 1 and photo 2. Be sure the fittings and pipe are inserted all the way. Tap on them if needed. Face the handles towards the back.
2. Use the provided screws at the predrilled locations on the front and back of the push bar fittings. Shown in Photo 2, 3 and photo 4.

3. Clip the back rest frame to the cross bar behind the seat cushion. Be sure to position the clips as shown in photo 5 and photo 6.

4. Place the back rest in the upright position and use the straps with snap and attach to the snap on the back rest.

5. The brake is operated by removing the pins and dropping the rubber pads to the ground and replace pins. The rubber feet can also be placed deeper in soft surfaces like sand. Be sure to raise and re-pin the brake before being mobile.



Photo 1



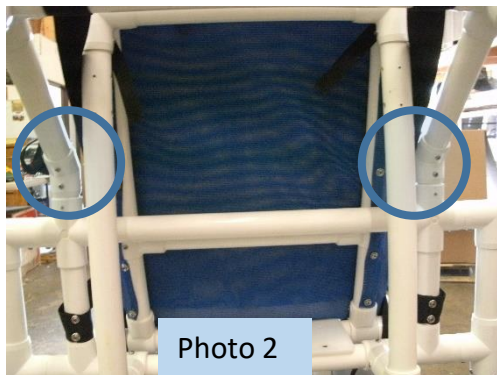


Photo 2

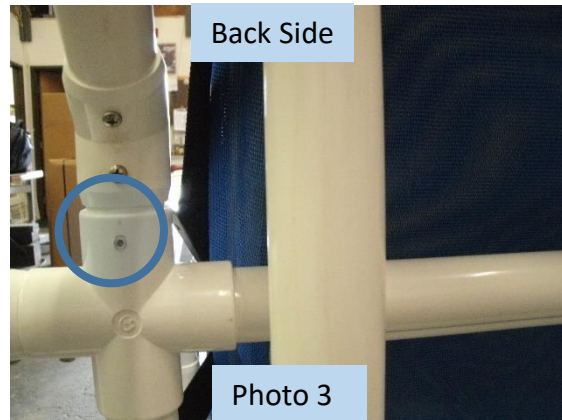

Back Side

Photo 3

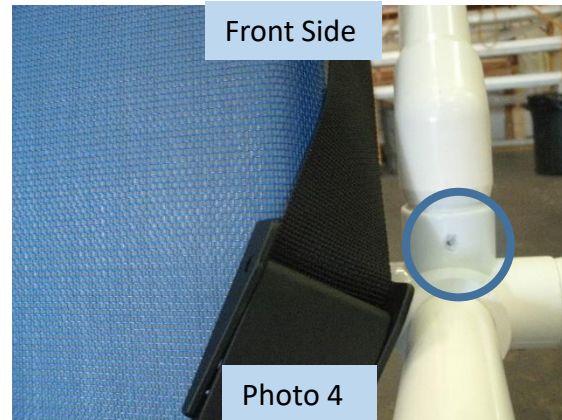

Front Side

Photo 4



Photo 5



Photo 6

AQ-1000

