



Aqua Swivel Seat

FOR RESIDENTIAL USE ONLY

MANUAL for the Aqua Swivel Seat:

Patent # 10,610,433

Hot Tub Chair AQS400HT





Aqua Swivel Seat

READ ALL SAFETY INFORMATION BEFORE USING

IMPORTANT SAFETY INFORMATION:

- **DO NOT STAND ON THE AQUA SWIVEL SEAT**
- **DO NOT EXCEED THE MAXIMUM WEIGHT LIMIT: 400LBS**
- ALWAYS ensure the legs are securely adjusted, level and pins (buttons) are engaged
- INSPECT at regular intervals for wear or damage. Check all plastic parts and rubber feet are in good condition.
- DO NOT use if any parts and/or rubber feet are damaged.
- ALWAYS with individuals who are severely cognitively, physically or visually compromised have someone frequently monitor individual using the Aqua Swivel Seat.

After Use:

Rinse chair thoroughly with clean water and dry with clean towel

NOTE: Failure to follow the above safety instructions may lead to injury



Aqua Swivel Seat

Hot Tub Chair

Check List

Item No: AQS400HT

1 Chair Assembly

2 sets of legs

Wrench

Manual

Assembly

Remove assembled chair, legs, wrench and manual from box .

Install two Short Legs to the front of the chair - Install two Long Legs to the back of the Chair—make sure the legs and screwed in tight and use the wrench to tighten the nut.

Adjust all four legs height for level seating (see page 4 for instructions)

Adjust the clamp to fit on the hot tub side width




Aqua Swivel Seat

HOW TO ADJUST LEG HEIGHT



1.

Release leg clamp (**do not loosen nut on clamp**). Pull lever outward.

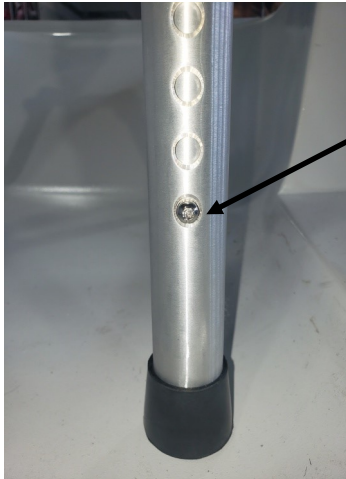


2.

Push Button in adjust height of legs, adjust the legs to proper height slide the legs until leg button engages to proper height, make sure button is engaged properly

Position on all four legs evenly

Buttons on legs MUST be used at all times

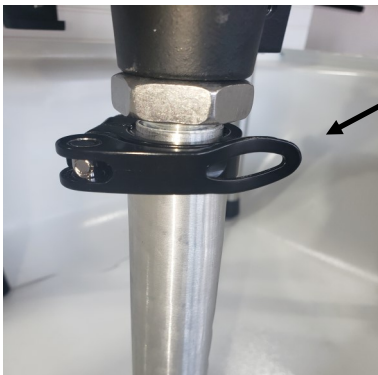

3.

Push lever inward to lock leg clamp

Aqua Swivel Seat

HOW TO ADJUST HOT TUB CLAMP SLIDE



1. Ensure clamp knobs and slide are outside of tub.
2. Turn the two screw knobs on each side counterclockwise to loosen.
3. Use notch in slide as handle for easy adjustment of clamp. Move clamp slide to fit clamp to side of tub securely.
4. Turn knobs clockwise to lock clamp slide (DO NOT OVER Tighten).

HOW TO LOCK AND UNLOCK



Pull on lanyard on the side of the chair - start to turn chair and let go of the lanyard. The chair will self lock when the chair to turn 180°.



Aqua Swivel Seat

FOR RESIDENTIAL USE ONLY

NOT FOR COMMERCIAL USE

Limited Lifetime Warranty

AQUA SWIVEL stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

AQUA SWIVEL extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. **AQUA SWIVEL** warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty (Seat Assembly and Legs)
Non-Metal/Metal parts	One Year Limited Warranty (Rubber tips, all other plastic parts)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. **Items that have been subject to negligent abuse or modification are not covered within this warranty.**

For warranty service, please contact the dealer from whom you purchased your **AQUA SWIVEL** product. In the event that you do not receive satisfactory service, please call us directly at the number below. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. **DO NOT** return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of products.