


ASSEMBLY INSTRUCTIONS



1



2

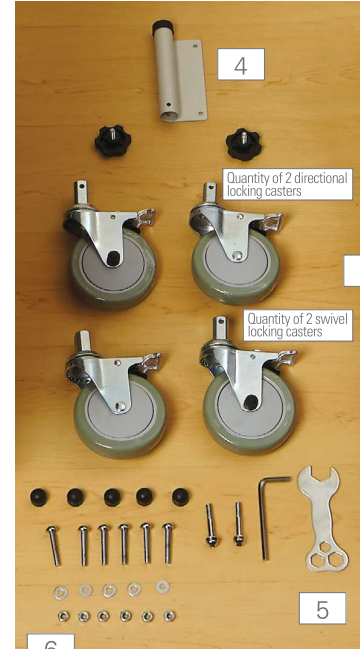

3

4

Quantity of 2 directional locking casters


Quantity of 2 swivel locking casters

5

Components:

1. Chair Base
2. Reclining Back Assembly with attached back post and extension leverage bar.
3. Casters 2 Swivel Locking, 2 Directional Locking
4. IV Pole Holder
5. Wrench
6. Hardware Package (6 bolts, 6 nuts, 5 caps and 2 screws.)

Step A

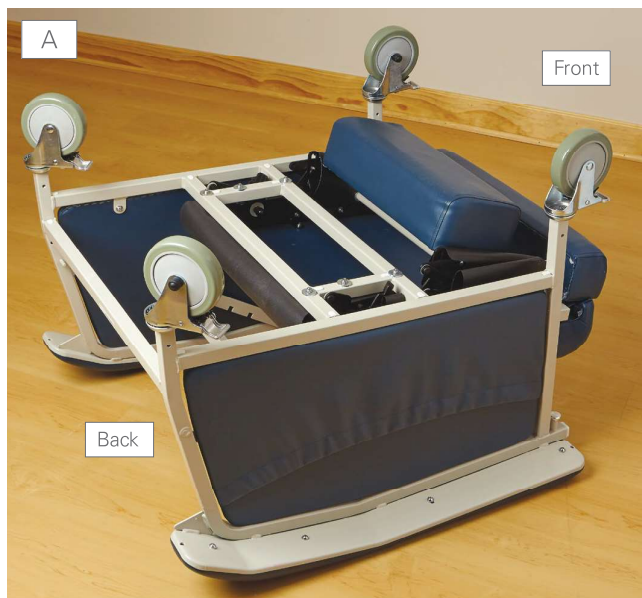


A. Turn the chair base upside down and attach the casters to the legs of the chair base. The swivel locking casters should be placed on back legs of the chair base and secured with the nuts, bolts and caps provided. The front of the chair receives the directional non swivel locking casters.

Step B



B. Turn the chair upright and attach the IV Pole holder to the right rear caster receiver. Unsecure the bottom screw on the right rear locking caster and align the IV pole holder with the two holes, then secure it with the provided screws.

Step C

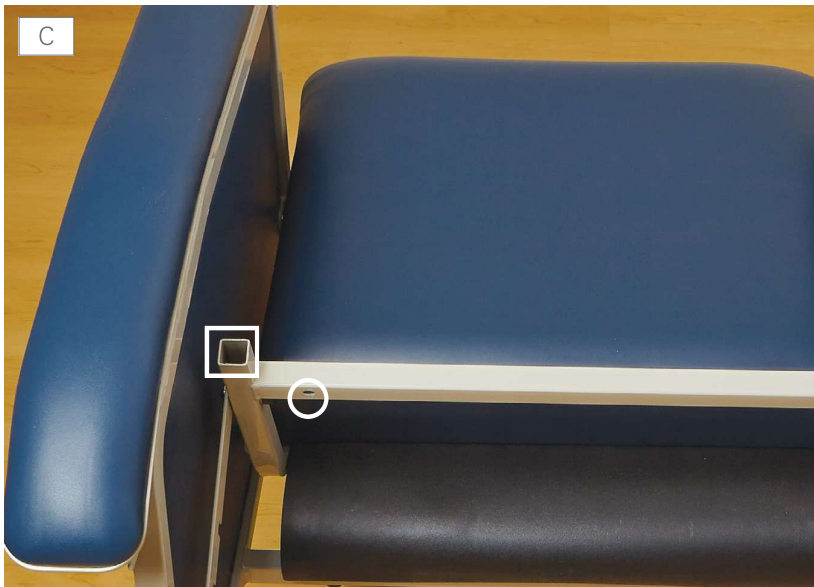

C. Insert the reclining back posts into the base receivers (see **WHITE SQUARES**) and secure the back to the base with the Allen Screws (see **WHITE CIRCLE**).