


# LiteGait<sup>®</sup> AutoStep

Quick Start Guide

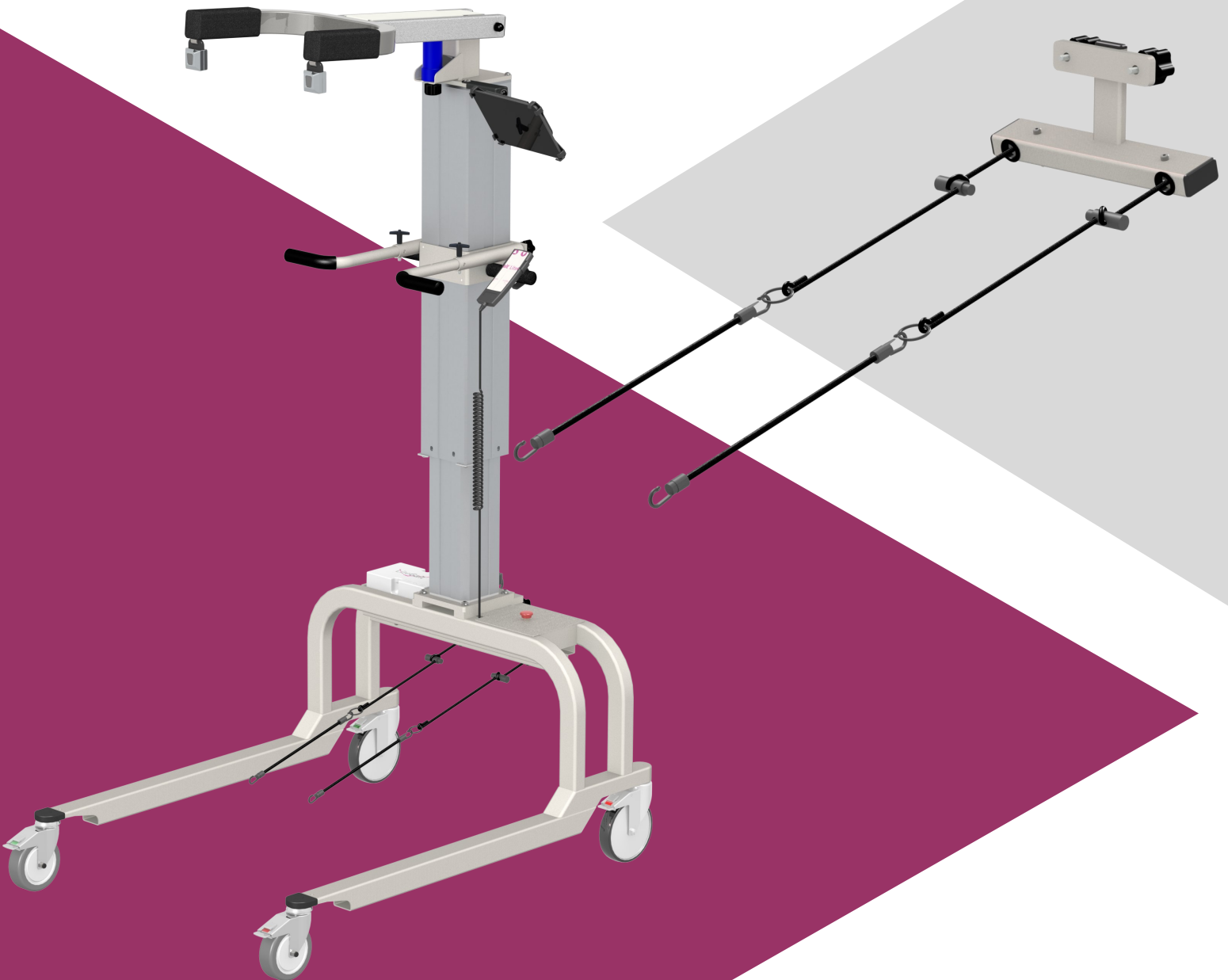



Congratulations on your new AutoStep which provides an easy adjustable method to provide queues within the gait cycle. Please follow these instructions to install, set up and start using AutoStep. Photos may differ from your version of the product

If you purchased an AutoStep to attach to a LiteGait that you already are using, start on **Section 1**.


If you are **replacing** an older AutoStep model or attaching your new AutoStep to your **new** LiteGait please start on **Section 2**

### Section 1


Remove **AND DISCARD** the two rear bolts that support the Actuator and discard them. Loosely insert the two bolts that were shipped with AutoStep.

#### WARNING

It is critical the new **longer bolts** that were provided with your AutoStep are used.

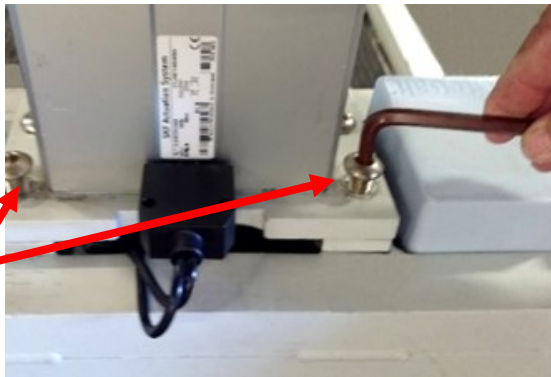


Position the AutoStep attachment bracket behind the bolts. Notice two slots for the bolts, and one slot for the actuator cabling.

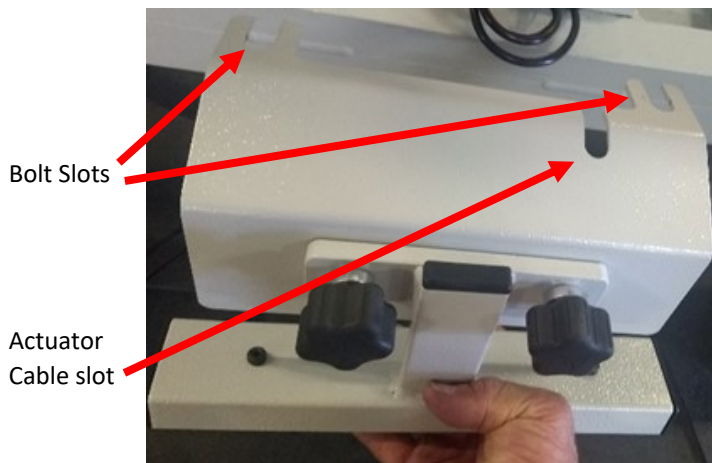

Align the actuator connector cable and the bolts into the specified slots and slip the bracket into place.

Slide AutoStep bracket into place

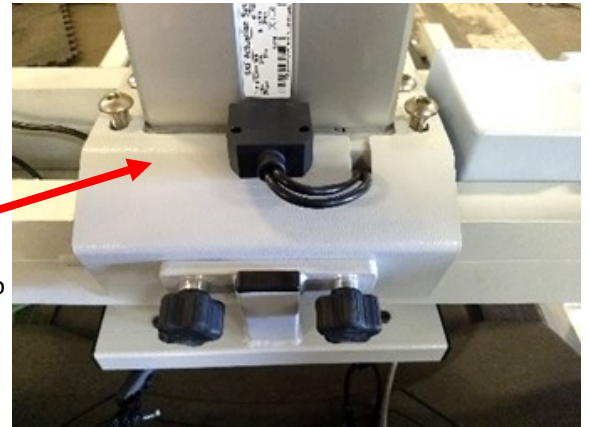


*Your bracket may differ based upon LiteGait model (see images above and below)*

Securely Tighten both of the longer bolts that were included with your LiteGait.

#### WARNING

It is critical the new **longer bolts** that were provided with your AutoStep are used.

Tighten bolts securely

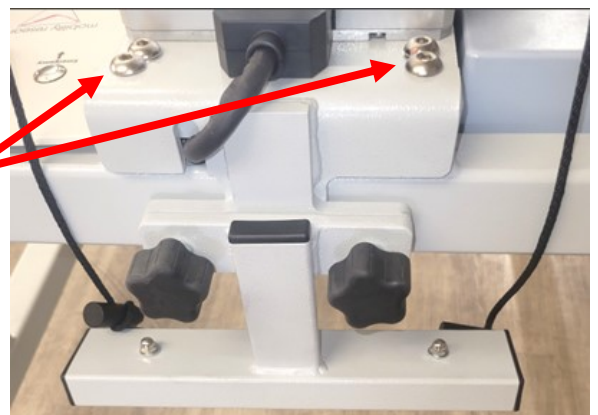

Proceed to **Section 3**




### Section 2

Attach AutoStep to the LiteGait base using the provided knobs or bolts, If not already attached.



Attach one Elastic Band to each of the AutoStep attachment rings.

*Note the black Elastic Bands have a heavier resistance than the gray Elastic Bands.*

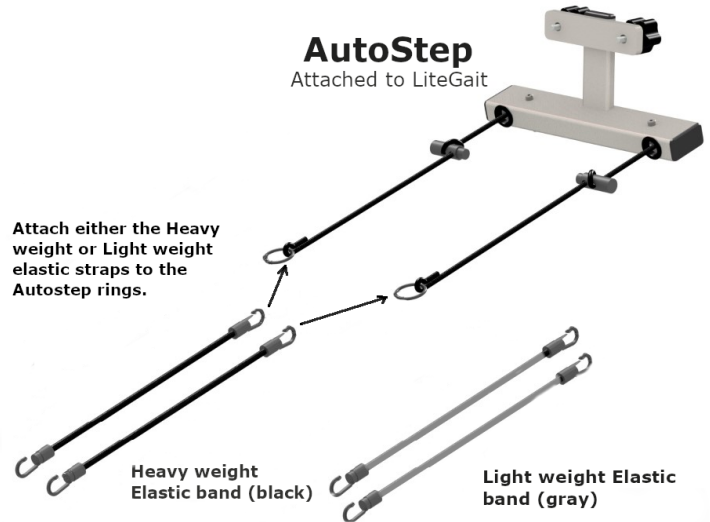

### Section 3

AutoStep is provided with a soft goods package with these components.

ID	Description	QTY
1	Light weight elastic band (gray)	2
2	Heavy weight elastic band (black)	2
3	Toe Strap	2
4	Ankle Strap	2



Feed the straps towards the patient



Pull the straps towards the patient



### FOOT/ANKLE STRAPS


The Toe and Ankle straps can be tightened around mid-section of foot, the ankle, or below the knee.

#### Attach

Toe Straps

or

Ankle Straps

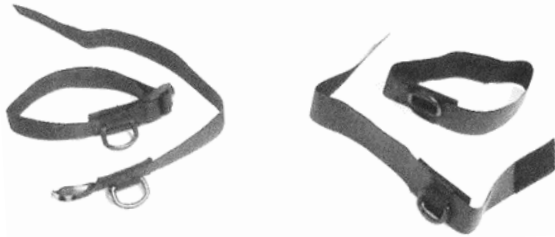

Attach the appropriate AutoStep elastic band to the D ring on the ankle or toe straps. It is also possible to attach directly to shoelaces when present



### Adjust Step Length

To adjust step length, press cord lock together, and slide it.




**NOTE-** distance from cord lock to the hook should be the same on the left as the right


Move cord lock toggle towards the **hook** to lengthen step length

cord lock toggles

Move cord lock toggle towards the **LiteGait** to shorten step length.

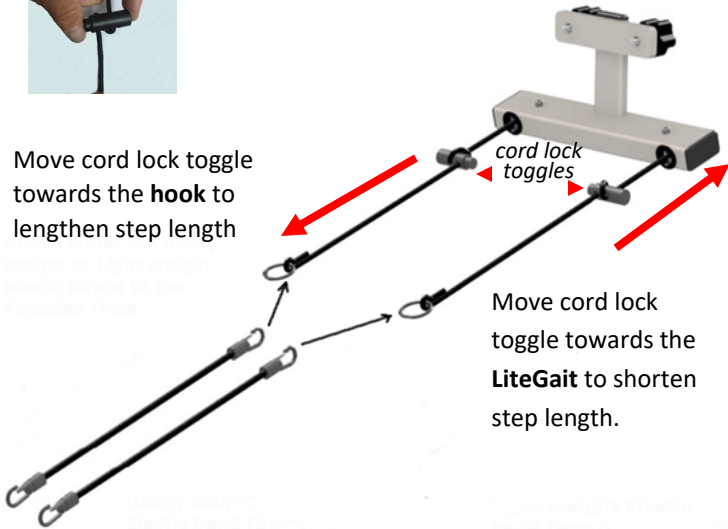

To remove AutoStep loosen the two black knobs on the back, remove the lower bracket, and place the knobs back into the threads.

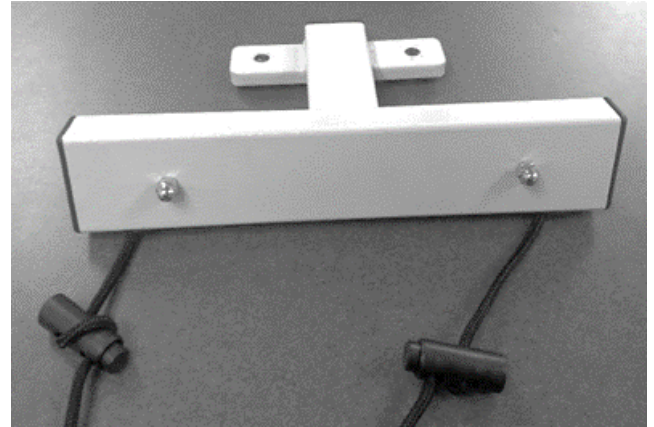

Hook the straps to the handlebars when using LiteGait without AutoStep.

