




## Fitting Protocol

Back Belt

Models 25130, NSB-500/501/501C

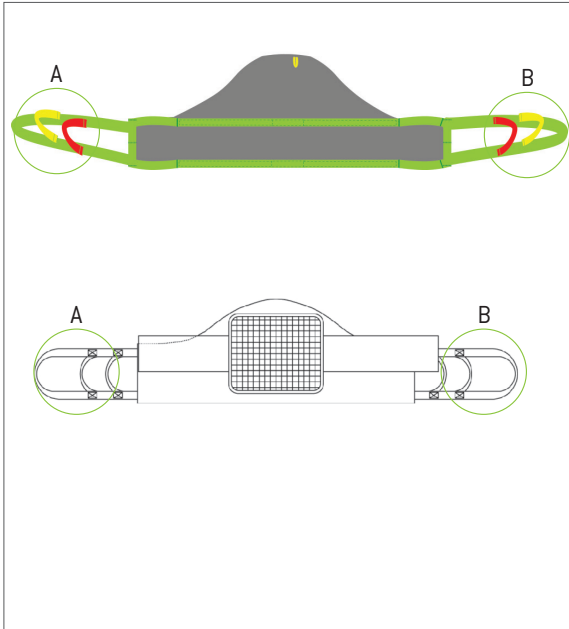

**Strap A**

- Red Strap
- Yellow Strap
- Green Strap

**Strap B**

- Red Strap
- Yellow Strap
- Green Strap

- Model 25130 - Polyester**
- Model NSB-500 - Polyester**
- Model NSB-501 - Cotton, Single Patient Use**
- Model NSB-501C - Cotton, Single Patient Use, Clips**

Choose the loop combination that will give the best distance from the Sit-to-Stand for the intended use. When securing straps to Sit-to-Stand device, match colours on both sides of the patient.

**Maximum User Weight: 150kg/330.7lb**

**Transfer Situation:**

- Raise/Lower To/From a Bed
- Raise/Lower To/From a Chair
- Gait Training/Mobilization
- Raise/Lower To/From a Toilet

**Available sizes:**

Indicate size required

- S
- M
- L
- XL

Note: NSB-501 size XL is not available

**Leg Placement:**

- Not applicable for this sling
- Not applicable for this sling
- Not applicable for this sling
- Not applicable for this sling

**Suspension:**

- 2-Point

**Accessories required:**

**Patient Information:**

Name	
Facility	Room
Lift Used	Hanger Bar Used

Back belts are designed for use with a Sit-to-Stand device and help safely raise or lower patients from a bed, chair or toilet. The wide reinforced back support allows for gait training or mobilization exercises when the float plate is removed from the device. Back belts are available in reusable Polyester models or single patient use cotton models. These belts are recommended for patients with poor trunk stability and difficulty standing. Some leg control is necessary to use this product.

### Additional Instructions

To staff/caregiver:


To patient:


Fitting Protocol Completed by:

Name	
Date	Position/Title



**Important!** This fitting protocol does not replace the user manual. Read all instructions before use!