




## Fitting Protocol

Positioning Sling  
Model NSB-401-ST, NSB-410-ST,  
NSB-451-U

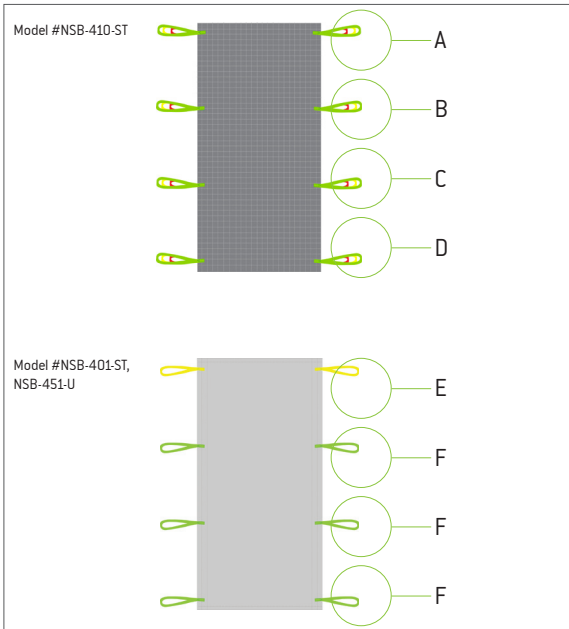

- Strap A**
- Red Strap
  - Yellow Strap
  - Green Strap

- Strap B**
- Red Strap
  - Yellow Strap
  - Green Strap

- Strap C**
- Red Strap
  - Yellow Strap
  - Green Strap

- Strap D**
- Red Strap
  - Yellow Strap
  - Green Strap

- Yellow Strap E
- Green Strap F

Extension Loop Size:  
Strap A: \_\_\_\_\_  
Strap B: \_\_\_\_\_  
Strap C: \_\_\_\_\_  
Strap D: \_\_\_\_\_

Choose the loop combination that will give the most comfortable and safe suspension angle, keeping in mind the patient's ability to support themselves.



- Green Strap**  
Increased Recline  
Less hip flex and greatest angle
- Yellow Strap**  
Semi-Recumbent  
Mid-hip angle
- Red Strap**  
Upright Position  
Least hip angle

Match colours on both sides of the patient for a 90 degree angle in the hips.

For Models NSB-401-ST and NSB-451-U angle adjustment not applicable.




- Yellow Strap**  
Head positioning
- Green Strap**  
Body positioning

Maximum User Weight: 454kg/1000lb


Maximum User Weight: 318kg/700lb

- Model NSB-401-ST, NSB-410-ST, Ventilated Polyester
- Model NSB-451-U - Single Patient Use Polyester/Cotton


### Transfer Situation:

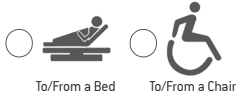

### Leg Placement:

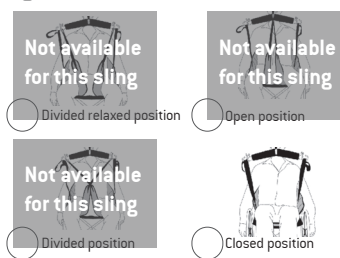

### Suspension:

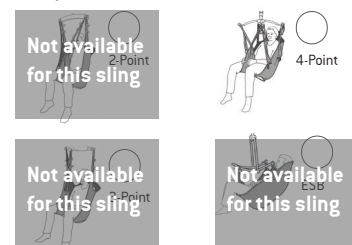

### Available sizes:

- Indicate size required
- ONE SIZE
  - L
  - XL
- Model #NSB-410-ST only

### Accessories required:



#### Extension Loops

- 21180-10 (10cm/3.9")
- 21180-15 (15cm/5.9")
- 21180-20 (20cm/7.8")
- 21180-25 (25cm/9.8")
- 21180-35 (35cm/13.7")

### Patient Information:

Name	
Facility	Room
Lift Used	Hanger Bar Used

The Positioning Sling is designed to assist a caregiver to safely and securely perform safe patient handling during sudden or intended positioning/repositioning, turning and lateral transfer from a supine position. This sling is offered in a reusable or single patient use model. The reusable model can be placed behind the patient much like a sheet and can remain behind the patient for short periods of time. The unique material facilitates air flow and breathability and does not promote skin breakdown. The single patient use model offers the convenient option of disposal after use combined with the added benefit of hygiene control. All slings fit standard hospital beds.

### Additional Instructions

To staff/caregiver:


To patient:


Fitting Protocol Completed by:

Name		
Date	Position/Title	



**Important!** This fitting protocol does not replace the user manual. Read all instructions before use!