

# BARIATRIC TRANSFER SHEET INSTRUCTION GUIDE

900 LBS SLIDING WEIGHT CAPACITY

To perform a transfer with a glide sheet, always get help from another caregiver.

1

Fold the glide sheet in half lengthwise, roll the patient onto one side and place the sheet on the exposed side of the mattress.



2

Roll the patient onto the other side and pull the sheet through to the other side of the mattress.



3

Ensure that the stretcher or bed you are transferring to is at the same height as the patient. Perform the transfer by gripping the handles and pulling on the transfer sheet to slide the patient from one lateral surface to another. The sheet will reduce the friction, allow for a smooth transfer for the patient, and minimize the strain of manual lifting.



4

Repeat steps one and two to remove the glide sheet.



## CARE INSTRUCTIONS:

Do not use bleach. Wash with a mild detergent and warm water not exceeding 150°F (65°C). Dry on low setting. Do not exceed max. weight specified.

## CAUTION:

This sheet must be inspected prior to each use for any rips, tears, frayed, or bleached areas which are unsafe and could result in injury.



Medline Industries, Inc.

FOLLOW US



Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2017 Medline industries, Inc. Medline is registered trademarks of Medline Industries, Inc.