

Your First Line of Defense

Introducing The WAFFLE® Bariatric Mattress Overlay



Prevention and Treatment of Pressure Ulcers

	Product Code	Dimensions	Warranty
Bariatric Overlay	1076BM	76" x 47" x 3"	90 Day
Bariatric Overlay with Pump	1076BMP	76" x 47" x 3"	90 Day

Weight limit up to 800 pounds depending on body type.

LATEX FREE

WAFFLE M.A.D® Hand Pump – 72 Strokes (Bariatric Overlay)

The WAFFLE® M.A.D. Pump provides non-electrical, exact inflation for all WAFFLE® Brand Products. It ensures therapeutic effectiveness and maximizes the medically-proven benefits of static air support.



WAFFLE® Electric Inflator – Inflation Range - 12 to 15 Seconds

To inflate, using the WAFFLE® Electric Inflator, place the overlay with the valve side on top and at the foot end of the bed. Open the clear valve, insert the pump and inflate. The overlay should inflate within 12 to 15 seconds. The product should be closely monitored while inflating, as the exact time may vary.




The unique design of the WAFFLE Bariatric Mattress Overlay allows the patient to sink down into the product providing protection (and comfort) while contouring to the body. The low profile chamber of air redistributes the patient's weight evenly on the surface. The Bariatric Mattress Overlay is clinically effective in keeping the patient stable and comfortable while preventing pressure ulcers.

The WAFFLE Bariatric Overlay has been weight tested for patients up to **800 pounds**.

“Caring for our bariatric patients can be very challenging,” said Charlotte Lisco, RN, BSN, CWOCN. “The healthcare industry has been waiting a long time for a product like the WAFFLE Bariatric Overlay that can prevent and treat pressure ulcers in the morbidly obese population”.

Charlotte Lisco, RN, BSN, CWOCN
Kaiser Permanente, Fontana, CA

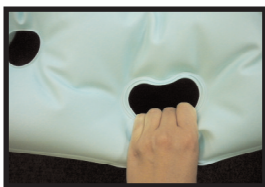

The new WAFFLE Overlay is designed for a bariatric-size bed, measuring 76" x 47" x 3", and it may also be used on a standard full-size bed for the home. The overlay can easily accompany the patient from acute care to long term care and to their homes. The patent-pending dogbone-shaped handwells, safely assist the caregiver with patient and product handling.

The WAFFLE® Bariatric Overlay meets the CFR 1632 (Cigarette Test) and both CAL 129 and ASTM E1590 (Full Product Burn). The overlay has undergone extensive antimicrobial testing on 10 bacteria (documentation available upon request). The overlay meets the FDA requirements for bio-compatibility and is latex free.



Also Available – The WAFFLE® Bariatric Seat Cushion

- Weight tested for patients up to 700 pounds.
- Low profile design contours and cradles the patient while seated.
- Redistributes weight and reduces friction and shear.
- Diamond-shaped hole pattern provides temperature and moisture control.