



GK-Basic +



User manual
06/2007

■ Contents

Contents	...	2
For your safety	...	3
Preparations for bathing	...	4
Components	...	5-7
Functions	...	8-10
Raising & Lowering of the tub	...	8
Setting the temperature	...	8
Shower	...	8
Filling the tub	...	9
Drain the tub	...	9
Disinfection	...	9
Flowmeter	...	9
Hydro massage	...	10
Care & cleaning of the tub	...	11
Service	...	12
Notes	...	13-14

■ **For your safety**

- Make certain that this bathing system is operated only by trained personnel.
- Always check the water temperature before the client enters the bathtub or when the shower spray is applied to the client.
- Make certain that the equipment is thoroughly cleaned after every use.
- Make sure that each client is assessed for it's ambulatory capability, ensuring the client's safety when entering and exiting the bathing system (utilize handgrips, correct entry height, correct client transferring device, etc.).
- When using disinfection, wear rubber gloves and eye protection. In case of eye or skin contact, follow the procedures on the disinfectant container and material safety data sheet.
- Always make sure that the device is supplied with power.
- Make sure that there are no materials right next to the tub to ensure a safe up and down movement. Disregarding this tip can cause bad injuries or damage the device.
- Don't deposit any things on the bathtub. Because of movement and vibrations, things can fall on the floor or in the tub.
- We want to point out that a lifting bathtub is a nursing tub and the bathing of the inhabitant has to be effected by use of a suitable lifter.
- Should no lift be used we recommend to furnish the bathtub with handles.

Note: *The safety rules and instructions in this operating manual must be observed in order to ensure safe and proper use of the equipment. Adjustments and repairs must be carried out by authorized and appropriately qualified persons. All claims under guarantee become void if these requirements are not met.*

■ Preparations for bathing

- If the drain knob isn't in the drain system put it back now.
- Put the pillow to the right position.
- If you are using a stretcher you don't have to apply the pillow.
- To fill the tub, move the inflow valve against the clockwise direction.
- Select a pleasant fill temperature.
- As soon as the water has achieved the desired level, help the client into the bathtub or bring the stretcher in the right position.
- Use the "UP" or "DOWN" buttons to bring the tub in a comfortable working height.
- If you want to use the shower at any time during the bath cycle, then move the shower valve against the clockwise direction and remove the white shower wand from its holder. To start showering press the black trigger.
- Now lower the tub and bring the client out of the water.
- Drain the tub.
- Before you start a new bath cycle you should clean the tub and go through the disinfection procedure.



■ **Components**

1. Disinfection
2. Shower
3. Stop-valve fill-in
4. Stop-valve shower
5. Fill-in
6. Thermostat mixing valve
7. Overflow
8. Hydro jets
9. Up/down-button
10. Drain
11. Adjustable foot
12. Hydro-button



- Fittings

- Disinfection with stop-valve



- Hydro-system

■ Functions

Raising & Lowering the tub

This tub allows caregivers to bathe clients in an ergonomic environment. Adjusting the tub to a comfortable working height enhances the bathing process. The "UP" and "DOWN" buttons are used to raise or lower the tub. The movement continues for as long as the button is pressed. When the button is released, the movement stops. In the upper and lower end positions the tub stops automatically.

Setting the temperature



You can set the temperature for tub fill and shower to any desired value, by turning the knob of the mixing valve.

Note:

For safety reasons the temperature is mechanically limited to 38°C (standard mixer). To override this security appliance, press the red button on the knob. Now it's possible to reach a higher temperature. Use extreme caution when overriding the temperature.

The thermostatic mixer is adjusted at 3 bar flow pressure at both supplies. If you have different supply pressure it may be necessary to do a readjustment.

Shower



To use the shower move the shower valve against the clockwise direction and remove the white shower wand from its holder. Press and hold the black trigger directly on the white shower wand to begin showering. If you want to stop showering, close the valve, release the pressure from the hose by pressing the black trigger and remove the shower wand.

Filling the tub



Move the inflow valve against the clockwise direction to fill the tub. If the right amount of water is in the tub close the valve to stop the water flow.

Drain the tub



To open or close the drain, turn the chrome drain plug knob in the respective direction. We recommend to close the drain system when the inflow is active.

Disinfection



The bathtub should be disinfected only when the tub is empty. Move the disinfection valve against the clockwise direction and remove the red shower wand from its holder. When activated, an accurately measured amount of disinfectant is directed into the wand for optimum disinfection and cleaning. Press and hold the black trigger directly on the red shower wand and spray all internal tub surfaces. Remove the red shower into its holder and close the disinfection valve. After the prescribed contact time use the white shower wand to rinse the interior tub surfaces thoroughly. For cleaning, maximum temperature should be selected.

Flowmeter



The flowmeter is used to adjust the mixture ratio between water and disinfectant. The table next to it should help you to find the right values. Always follow the instructions of the liquid manufacturer. Generally the flowmeter setting should be done when you install the tub.

Hydro massage

This therapy is only available with completely filled tub. As soon as the tub is filled, you can start the therapy by pressing the button on the side of the tub. With the first pressing of the button you start the Hydro-jets. If you want to stop the therapy you have to press the button again.

■ Care & Cleaning of the tub

After use, always rinse the bath tub thoroughly with the shower hose. Any dirt which adheres to the bathtub surfaces must be removed with a soft sponge or cloth - **do not use plastic scourers or scouring pads.**

To prolong the life of your bath tub we recommend that it is polished once a month. This will give the bathtub a smooth, water repellent surface and will make routine cleaning considerably easier - it will also enhance the appearance of your GK bath.


We recommend using our approved range of GK products with your GK Basic+ bath.

Any product used to clean or disinfect your GK Basic+ bath must be free from any abrasive, alcohol, acid, alkali & corrosive ingredients. If you are in any doubt about your usual cleaning products please contact your GK representative **before use**, proving COSHH data sheets where applicable.

NB: Any damage caused by unapproved / unsuitable products will not be covered under the manufacturers warranty.

A disinfection cycle should always be performed after cleaning. Where fitted, the hydromassage jet coverings should also be cleaned & disinfected.

The following diagrams show how to remove and clean the hydromassage jets.

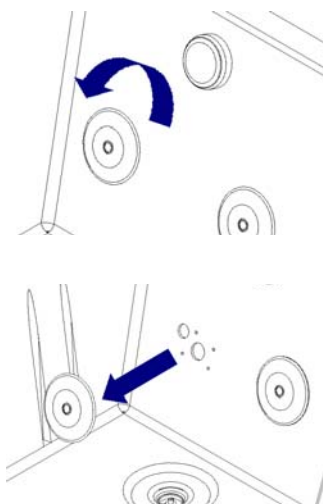

Rotate the jet cover anti-clockwise until it can be removed easily.

Clean the jet cover, the inlet aperture and the outlet aperture thoroughly. Replace the jet cover by rotating clockwise.

NB: Do not use excessive force!

If you have any further questions or require more information please contact your local GK representative, who will be happy to advise.

■ Service

- Check the hoses, tubes and connections from time to time.
- Make certain that the disinfectant container is monitored.
- We recommend that you take advantage of our maintenance service and request a safety and functional check every 12 months.
- Adjustments and repairs must be carried out by authorized and appropriately qualified persons.
- Check if there are any cracks or other damages on the tub body.
- Check the sieves of the shower wands and clean them if necessary.
- When servicing the bathtub the unit must be unplugged and locked out.

■ Notes

■ Notes



Typing, spelling or transmission errors excepted for all contents. All product-related information is not subject to change for technical production or other reasons.