

Erasing Recordings



Single Operation

1. On the back of the device, move the Multi/Single switch to **Single**.
2. Press and release the REC button. A solid red LED on the front indicates your Big Button is ready to erase the message.
3. Press and release the Play button and the red LED will flash. Press again within one second and the message is erased.
4. To exit Record mode, press and release the REC button. The red LED will turn off.

Multi Operation

1. On the back of the device, move the Multi/Single switch to **Multi**.
2. Press and release the REC button. A solid red LED on the front indicates your Big Button is ready to erase message one, two, or three.
3. To erase message one, two, or three, press and release the Play button once, twice, or three times to access the message. The red LED will flash after a one-second delay. Press again within one second and the message is erased.
4. To exit Record mode, press and release the REC button. The red LED will turn off.

Record Lock

To disable Record, turn the device off, press and hold the Volume up button , and turn the device on. Release the Volume up button  when two ascending beeps play, indicating the Record Lock is on. To unlock, repeat the directions above. Two descending beeps indicate Record Lock is off.

Quick Start

Basics

- On/Off switch on the back
- 2 AAA batteries included
- REC button on the back
- Adjustable volume
- Threaded insert and keyhole on back for mounting
- Overlay storage on back

Operations

- Choose between Single or Multi messages with the toggle switch on the back
- Single operation has one message with a 30-second recording capacity
- Multi operation has up to three 30-second messages
- For Multi operation: record and play messages by pressing the Play button once, twice, or three times


Touch the edge or the top to play a message.



Attainment Company's

Big Button

easy-touch
access!

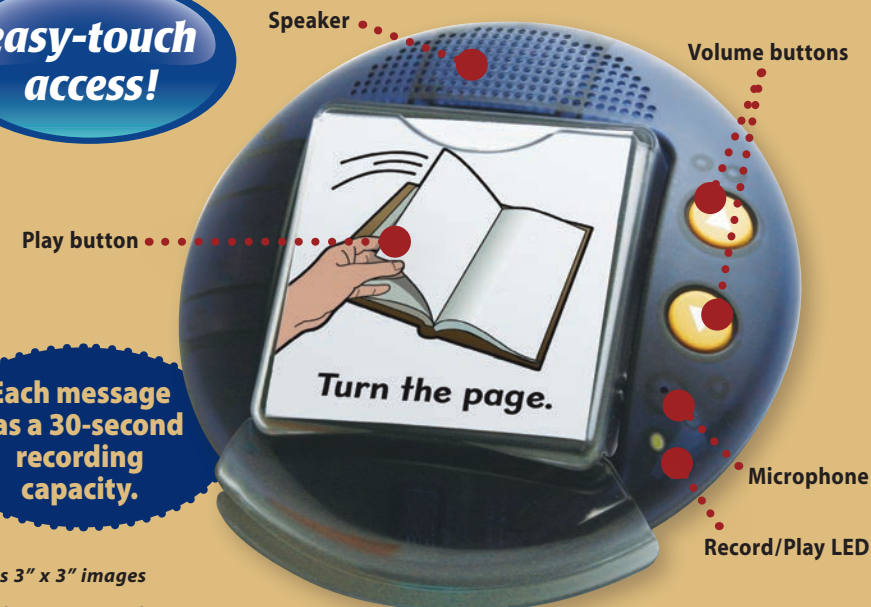

Each message
has a 30-second
recording
capacity.

Holds 3" x 3" images

Weight: 8.1 oz. (.23 kg)

Size: 5 7/8" across (14.6 cm)

User's Guide

The **Big Button** is a powerful communication tool with a single large Play button. A removable transparent cap snaps onto the Play button and has an opening on top for sliding in overlays or icons. Remove the cap by firmly pulling up on the lip until it dislodges. Then pull on the top part of the cap to remove it. The cap extends nearly to the table surface, creating a large activation area with an extremely low profile. In fact, you can play a message without lifting your hand off the table. Only a little pressure is needed to activate speech.

Low-profile,
easy-touch
activation!



Volume

Volume up and down buttons are on the front right side of the Big Button. It beeps when you adjust volume up or down.

Removable,
transparent cap for
easy cleaning

Large,
easy-touch
Play button




Single or Multi Operations

Choose from two operational styles—**Single** or **Multi**—with the toggle switch on the back.

In **Single** operation, the same message plays each time the Play button is pressed.

In **Multi** operation, you can select one of the three messages based on the number of Play button presses: One press for the first message, two presses in quick succession for the second message, and three presses for the third.

Single Operation Recording


1. On the back of the device, move the Multi/Single switch to **Single**.
2. Press and release the REC (Record) button. A solid red LED on the front indicates your Big Button is ready to record.
3. Press and release the Play button and the red LED will flash. You are now recording. Speak clearly, about 8" from the Big Button. *Note: The click you hear when you press the Play button will not record.*
4. Press and release the Play button when you are done recording. Otherwise, recording will automatically stop after 30 seconds. The LED will return to solid red.
5. Press and release the Volume down button  to hear your recording. To re-record, follow recording directions 3–4.
6. To exit Record mode, press and release the REC button. The red LED will turn off.

Playing

Press and release the Play button. The Record/Play LED will flash green during playback. Press and release the Play button during playback to stop the message.

Each message
has a 30-second
recording
capacity.

Multi Operation Recording

1. Move the Multi/Single switch to **Multi**.
2. Press and release the REC button. A solid red LED will turn on, and you are ready to record one, two, or three messages.
3. To record message one, press and release the Play button once. The red LED will flash after a one-second delay, indicating you are recording. Speak clearly, about 8" from the Big Button. *Note: The click you hear when you press the Play button will not record.*
4. Press and release the Play button when you are done recording. Otherwise, recording will automatically stop after 30 seconds. The LED will return to solid red.
5. Press and release the Volume down button  to hear what you have just recorded. To re-record, follow recording directions 3–4.
6. To record message two, press and release the Play button twice in quick succession. The red LED will flash after a one-second delay, indicating you are recording. Press and release the Play button when you are done recording.
7. To record message three, press and release the Play button three times. The red LED will flash after a one-second delay, indicating you are recording. Press and release the Play button when you are done recording.
8. To exit Record mode, press and release the REC button. The red LED will turn off.

Playing

Press and release the Play button once to play message one, twice in quick succession to play message two, and three times to play message three. The Record/Play LED will flash green during playback. To stop the playback of a message, press and release the Play button.

