## Drag Lock

You can easily drag an object with the BIGtrack 2 without having to press and hold the left click button.

To enable drag lock:

1. Hover the cursor over the item you would like to drag.
2. Double click the right click button to lock on to the item.
3. Move item to the desired location.
4. Release the drag lock by clicking the left click or right click button.

## FCC

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

## USB Connection

Plug the cable into your computer's USB port or a USB hub connected to your computer. You do not need to turn off your computer before connecting or disconnecting a USB device.

## Switch Use

The BIGtrack 2 has two switch ports located on the left and right side of the device. For switch use, plug a switch into each switch jack. Each switch will perform the same function as clicking the left or right button on the device.

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## For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that

used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

## For professional users in the European Union:

If you wish to discard electrical and electronic equipment(EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union(EU). If you wish to discard this product, please contact your local authorities or dealer and ask for the correct method of disposal.

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