



**southpaw enterprises, inc.**<sup>®</sup>

# Instruction Sheet

## Bouncy Platform Swing 1830

Store these instructions in a safe place or with the enclosed maintenance checklist



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location. This Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

**Warning:** Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

### MATERIALS LIST

- 1 - Bouncy Platform Swing
- 1 - Carpet
- 1 - Safety Snap
- 1 - Instruction Sheet
- 1 - Safety Snap Instruction Sheet
- 1 - Vertical Stimulation Device Instruction Sheet
- 1 - Maintenance Checklist

### MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED

Minimum height required is 8 ft.  
Caution must be used to keep a clear distance of 6 ft. from walls and other obstacles.

### HEIGHT ADJUSTER AND SAFETY ROTATIONAL DEVICE NEEDED

Use of the Southpaw 5000 Height Adjuster is recommended for ease of adjusting the height of your equipment.

A Southpaw 2000 Safety Rotational Device should be used to prevent damage to your equipment.

**WORKING LOAD:** 240 lbs.

### HANGING PROCEDURE



Hang Platform Swing by attaching the Safety Snap that is on the top of the Platform Swing to your height adjustment system.

With four control lines connected to the main suspension line, the platform remains relatively stable.

For a full range of movement, disconnect one or all of the stability lines from the main suspension line.

**Special Note:** An occasional spot of grease on the U-bolt (top of disc) will keep it noise free.

**Caution:** Edge is padded, but will give you a good bump if you're not careful.

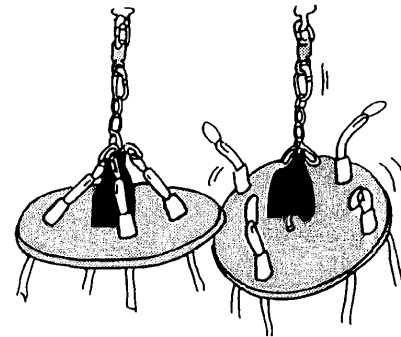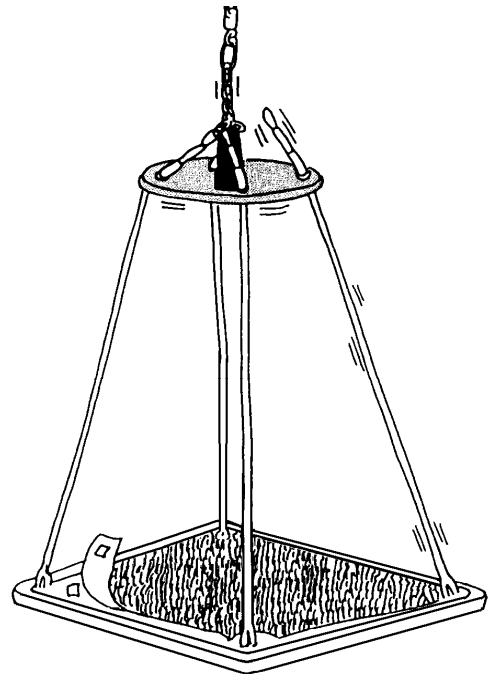

Stability lines will unclip to give a total range of movement

## MAINTENANCE CHECKLIST

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

- Check the chain and Safety Snap for wear. (See Safety Snap Instruction Sheet). Also check the threaded chain links, near the safety snap, to which the control lines are attached. Make sure that the screw gate is securely tightened.
- Stimulation Loops — Check for loose stitching around the “ball” near the top and the bottom, and holding the loops on the top and bottom. If found discontinue use until repaired or replaced. (See VSD Instruction Sheet)
- Check the U-bolt under the black boot for wear. If wear is more than 30% of the diameter of the bolt, discontinue use.
- Check ropes for loose or missing splice locks.\* If loose or missing splice locks are found, discontinue use until repaired.
- Check wood for cracks or splinters. If cracked, discontinue use. For splinters, sand until smooth.
- Check edge padding to see that it is securely attached.

\* A splice lock is the metal clip near the end of the ropes.

## STORAGE PROCEDURES

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

## CLEANING INSTRUCTIONS

- Check for client allergies first.
- Check with your infectious disease department before using these procedures.
- Dust or wipe with a damp cloth.

## CAUTION

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.

