


Aeromesh™ Calf Strap and Calf Panel


INSTALLATION AND USER'S INSTRUCTIONS

ART NOS. SP102S, SP102M, SP102L, SP103S, SP103M & SP103L

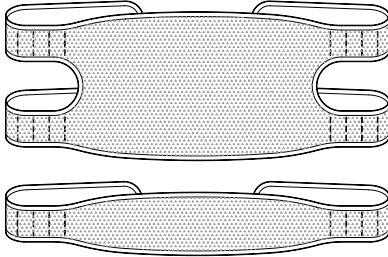

PACKAGE CONTENTS

- Aeromesh™ Calf Panel
- or
- Aeromesh™ Calf Strap
- Adhesive Backed Hook and Loop Patch

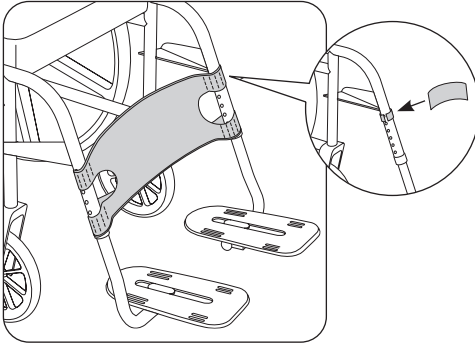


PERIODIC SAFETY AND PERFORMANCE

CHECK: To ensure user safety, this product must be checked periodically for function and signs of wear. If the product does not function correctly or if significant wear is found in the buckles, mounting points, webbing, padding, or stitches, stop using it and contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons — health and safety depend on it!

⚠ WARNING! If a serious incident occurs related to the use of this product, it should be reported to the manufacturer (Bodypoint, Inc.) and the local Competent Authority.

🧺🔥🕒🧺 CLEANING: Machine wash, hot, 60°C (140°F). Do not bleach. Tumble dry, low temperature, or drip dry. Do not iron. (Placing the product inside a cloth bag during washing helps prevent scratches to the product and the machine.)

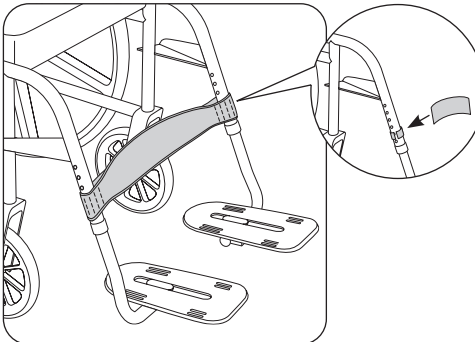

SCRAP/DISPOSAL: Product is made of materials which can be disposed of safely without special precautions at the end of its useful life.

WARRANTY: This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.