




The STABILO® is a seat of stable positioning with the capability to be easily shaped in any desired form and create to maintain it. In maintaining the corrected positions following the rehabilitation, it improves the posture and the body maintenance (spasticity, scoliosis, kyphosis...).

The STABILO can easily be positioned on the seat of the All-Terrain and Ski Wheelchair. It is possible to transfer it from one wheelchair to another, and it is an excellent complement of therapeutic sittings. The rigidity of the seat is adjusted with the vacuum pump that pulls the air contained in the seat to create vacuum and to set in this way the desired shape.



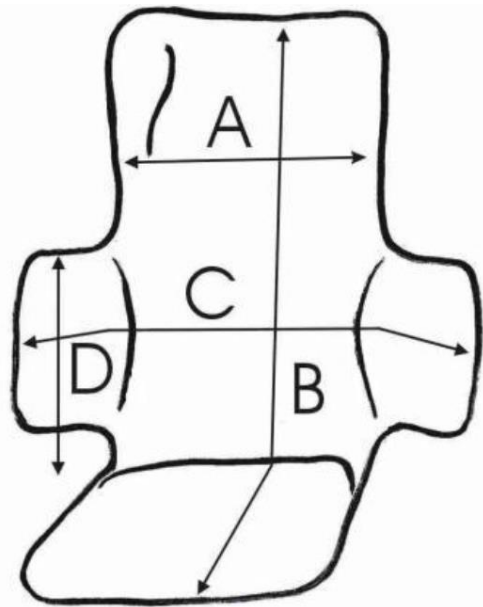

DIMENSIONS (cm)				
SIZES	A	B	C	D
S	30	80	58	28
M	35	95	65	28
L	40	115	75	35
XL	45	120	85	35

ADDITIONAL DETAILS

COMPOSITION	<p>The STABILO® seat is composed of synthetic rubber covered of a blue anti-allergenic jersey nylon cover that obtains the adequate water and airtightness to get the vacuum. Fine granules with a high friction capacity allowing the lightness and the malleability of the seat. The seat cover in neoprene is hyper deformable, ensures the waterproofing and the desired shape memory property.</p>
FUNCTIONING	<p>The pump creates an air vacuum by fixing the granules while letting them mobiles each other due to the weight and pressure effect: it offers an unlimited possibilities of shapes and hardness for a particularly stable system. The granules avoid the pressures on the whole support seating surfaces.</p> <p>To change the shape of the seat, you just have to open the valve and to let enter the air in to get a new shape, then to recreate the vacuum to obtain the hardness or the desired softness.</p>
RIGIDITY AND DURABILITY	<p>The seat keeps its shape and its density several months following the user behavior: in effect, if the user is agitated or spastic, the change of positions and the spasms necessitate a bi-monthly adjustment.</p> <p>A temperature increase will tend to soften the seat, whereas the cold will rigidify it: this variation is always minimal and correctable with the vacuum pump.</p>
WASHING	<p>The coating in jersey nylon is easy care: it is advised to wash it in a shower tray or a bath at 60° max. The use of chlorine or solvent is forbidden. Do not put it in the washing machine.</p>
DRYING	<p>The drying can be achieved with paper towels or a hair dryer.</p> <p>It is possible to use the drier and not to exceed 60°.</p> <p>After drying, the product is immediately ready for use.</p>