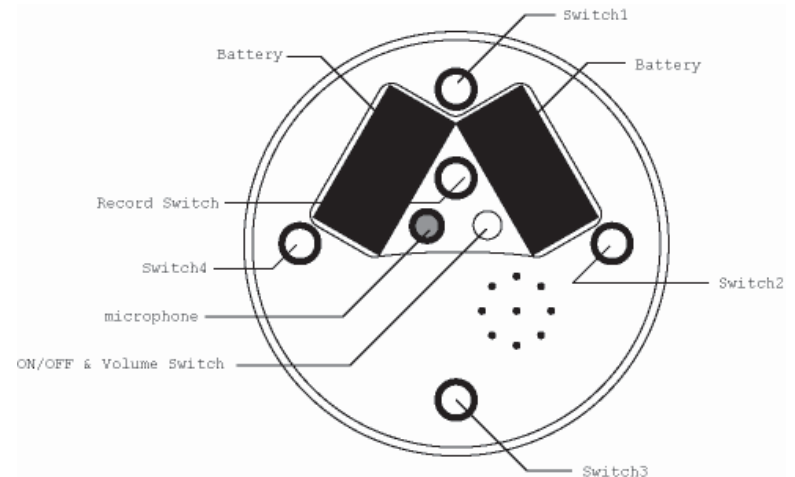

Tail Light Switch Say-It-Play-it

Touch its rim ever so lightly and hear your pre-recorded message while vibrating and lighting stimuli go into action. Plug in your toy or device and it will start when edge is touched. Can be used on a tabletop or mounted on a door or wall. Wall mounting bracket sold separately. See item #466 below. Size: 7 3/4"W x 2 1/2"H. Requires 4 AA Batteries (Not included).

Operation:

1. Carefully turn over unit and remove the battery case covers to reveal battery compartments. Observing polarity, install four AA size batteries (Not included) into the holders. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well.
2. The microphone and the red "RECORD" button are located on the back panel of the unit. To record, first depress and hold the red "RECORD" button. Then while holding the "RECORD" button, depress the No.2 or No.4 padded buttons (see illustration) and speak into the microphone. Release both the padded button and the recording button when the recording is over. Place the Tail Light on a flat surface, or use the optional wall mount. Press anywhere on the Tail Light and your message will play, along with lights and vibration.
3. The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when a new recording is performed.
4. Playback volume can be adjusted using the "ON/OFF/VOLUME" control knob located on the back panel of the unit.
5. To use as a switch, plug one end of the supplied double ended 1/8" to 1/8" cord into the jack located on the side of the Tail Light. Next plug the other end into your switch adapted toy/device. Press anywhere on the Tail light, and your toy/device will activate along with your pre-recorded message, lights and vibration. Once you release the Tail Light your toy/device will shut off.



Important Notes:

1. The memory of this unit is capable of holding up to a total of 20 seconds of speech information. If message length exceeds available memory amount (e.g., 20 seconds), it will wrap around over itself at its beginning.
2. The Tail Light Switch will shut off automatically after playback, to preserve battery life. However, the device still drains a very small current in this "sleep" mode. Therefore, if the unit is not going to be used for a few days, it is recommended that the unit be turned off, to extend overall battery life. This can be done by turning the "ON/OFF/VOLUME" switch counterclockwise, until it "clicks" off.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Make sure that you are using the Tail Light on a flat surface.

Action #2: Disconnect the power, by removing one of the batteries from each compartment for 20 seconds, to reset the unit. Reconnect the battery. If the unit still does not work correctly, replace all batteries.

Action # 3: Make sure all connections between the Tail Light, cord and toy/device. There should be no gaps.

Care of Unit:

The Tail Light Say-It-Play-It can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit as it will damage the electrical components.

Do not use abrasive cleaners as they will scratch the surface of the unit.

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697
customer_support@enablingdevices.com

Tail Light Switch Say-It-Play-It

Catalog #464

User's Guide

Gum Ball Switches #782R, B, G, Y, P

USER'S GUIDE

A high quality and dependable switch. Meets most needs of switch users. It features a 2½ inch activation surface and is recommended for all who can access a smaller target area. Provides extraordinary tactile and audible feedback. Base has three holes for easy mounting. We have also made the mounting plate detachable since many users do not need the plate. Available in 5 colors. Size: 3¼" Base x ¾" High.

Operation:

1. The Gum Ball Switch does **not** require any batteries to operate.
2. The Gum Ball Switches come with a removable base. To remove the base turn the switch over and unscrew the two Phillips Head screws. Next pull the base ring off the switch.
3. Plug the Gum Ball Switch into the jack on your switch-adapted toy/device.
4. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
5. Make sure that the connection between the Gum Ball Switch and your toy/device is tight. There should be no gaps.
6. Simply press anywhere on the Gum Ball Switch's colored top to activate your toy/device.
7. The toy/device will only remain activated while the Gum Ball Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Troubleshooting:

Problem: The Gum Ball Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Gum Ball Switch and your toy/device is tight. There should be **no** gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Gum Ball Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Some Other Popular Switches:

Bright Red Switch #262

3”X5” Plate Switch with Latch/Timer #810-X

5”X8” Plate Switch with Latch/Timer #820-X

Leaf Switch #730

Vertical Plate Switch #1030

String Switch #1080

Petite Pillow Switch w/Gooseneck Mounting #1675

The Twitch Switch #1061

Tilt Switch #990

Roller Switch Music Box #880

Button Click Switch #775

Mini Gum Ball Switches #783R, B, G, Y, P

USER’S GUIDE

The Mini-Gumball switch is versatile and durable. It has a 1¼ inch activation surface. This switch is small enough to be easily mounted anywhere. Base has three holes for easy mounting. We have also made the mounting plate detachable since many users do not need the plate. It provides tactile and audible feedback. Available in 5 colors. Size: 2" base x ½" high.

Operation:

1. The Mini Gum Ball Switch does *not* require any batteries to operate. The base is removable if desired. To remove the base turn the switch over and unscrew the two Phillips Head screws. Next pull the base ring off the switch. After the base has been removed you can place the included piece of non skid rubber, to give the switch a firm grip on smooth surfaces.
2. Plug the Mini Gum Ball Switch into the jack on your switch-adapted toy/device.
4. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
5. Make sure that the connection between the Mini Gum Ball Switch and your toy/device is tight. There should be no gaps.
6. Simply press anywhere on the Mini Gum Ball Switch's colored top to activate your toy/device.
7. The toy/device will only remain activated while the Mini Gum Ball Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Troubleshooting:

Problem: The Mini Gum Ball Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Mini Gum Ball Switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

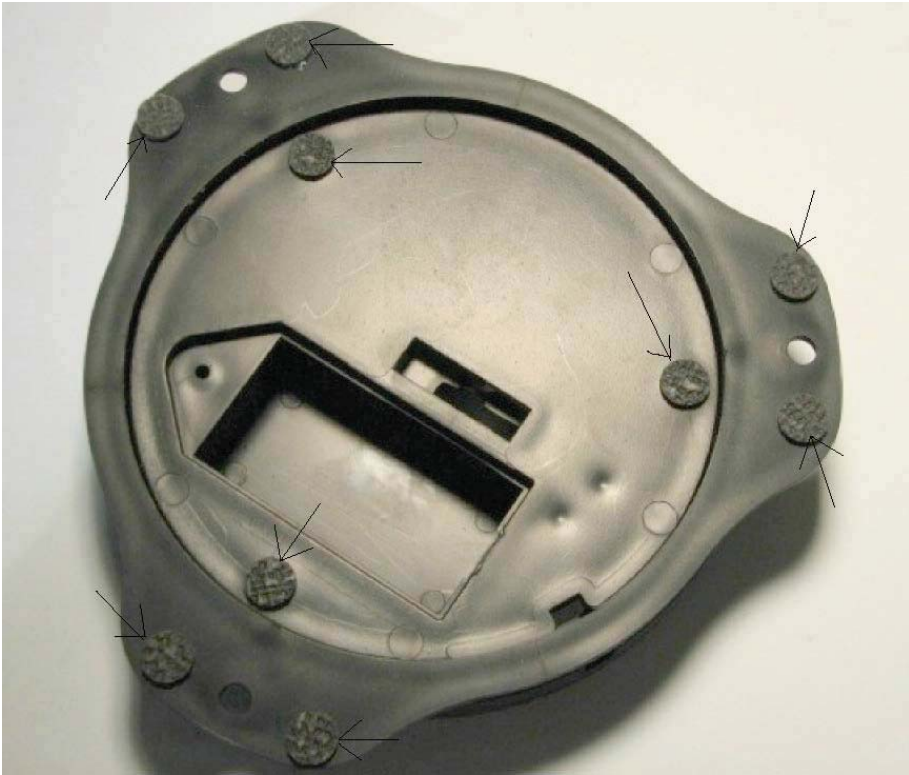
Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Gum Ball Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.



Sticker Placement Guide

**Jumbo Switch- #787-R, Y, B, G, & P
(Set of 5 Switches) #789**

USER'S GUIDE

This jumbo switch is an improved version of similar round switches. We have also made the mounting plate detachable since many users do not need the plate. Size: 5 1/8"Diameter x 1"H.

Operation:

1. The Jumbo Switch does *not* require any batteries to operate.
2. Plug the Jumbo Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Jumbo Switch and your toy/device is tight. There should be no gaps.
5. Simply press anywhere on the Jumbo Switch's colored top to activate your toy/device.
6. The toy/device will only remain activated while the Jumbo Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Mounting Plate Options:

The Jumbo Switch has a removable mounting base option. To use without the base, simply remove the two small Phillips Head screws holding the base from sides of the switch body. Included are a set of 9 self-adhesive Non-Skid feet. When using the Jumbo Switch without the base only 3 feet are needed. (See Photo for proper placement). If using with the base all 9 feet are needed. (See Photo).

Troubleshooting:

Problem: The Jumbo Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Jumbo Switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Jumbo Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Koosh™ Switch

Spongy, springy fun! Children of all ages will enjoy our colorful Koosh™ Switch. Touch the compelling ball of rubber strings and you are instantly rewarded with lights, music and vibrations. Great fun! Lights and music are disabled when used as a switch.

Requires 2 AA Batteries

Note: This product is not suitable for anyone with allergies to latex.

Operation:

1. The Koosh™ Switch requires two AA batteries. The battery compartment is located under the base of the unit. When installing new batteries, be careful to **observe battery polarity**. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well.
2. To operate the Koosh™ Switch as a switch, connect a cord to an external toy or device via the 1/8-inch jack on the unit. The Koosh™ Switch #883 comes with a 1/8- to 1/8-inch cord for activating external toys or devices with 1/8-inch jacks. If you need to use a 1/4- to 1/8-inch adapter, it must be a mono adapter, not stereo.

Troubleshooting:

Note: When this unit is **not** connected to an external toy or device, the device cord must be removed from the jack or the unit will not operate properly. Lights and music are disabled when used as a switch.

Problem: The Koosh™ Switch does not operate properly.

Action #1: Remove the 1/8- to 1/8-inch cord from the jack (see *Note* above.) If unit still does not work, make sure the batteries are in the battery holder properly and making good contact.

Action #2: If the unit still does not work properly, replace the batteries.

Care of Unit:

The Koosh™ Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents of the top and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-TEC-TOYS or 1-800-832-8697

Some Other Popular Toys/Switches

Somatosensory Bead Chain with Music Switch #357

Deluxe Razzle Dazzle Bead Chain Curtain #361

Glitter Roll Music Box/Switch #696

Oval Texture Music Box/Switch #712

Oval Texture Multi-Sensory Center/Switch #725

Lighted Musical Plate Switch #740

Lighted Plate Switch #750

Music Box Plate Switch #770

Roller Switch Music Box #880

1-, 2- and 3-Shape Switch/Music Box #900, 910, 920

1-, 2- and 3-Shape Switch #930, 940, 950

Interchangeable Music Puzzle/Switch #921

Lighted Signal Switch #956

Bitty Bubble #5100

Lighted Vibrating Cone #6100

Koosh™ Switch

Catalog #883

User's Guide

Ultimate Switch - #1671

USER'S GUIDE

Our exclusive Ultimate Switch is THE most universal, versatile, multi-faceted switch. It's designed to be used by even the most profoundly challenged or neurologically impaired individuals. It requires only the slightest touch for activation (10 grams of force), yet can easily withstand the abuse of gross body movements or severe blows. 19" Gooseneck and universal clamp included.

Operation:

1. Using the mounting clamp, attach the Ultimate Switch to a Wheel Chair, Table or Desk. Tighten the clamp enough to hold the Ultimate Switch securely, but do not over-Tighten the clamp. (See Photos).



2. Plug the Ultimate Switch into your switch adapted toy/device. Make sure that the connection is tight. There should be NO gaps. If you use an adapter for devices with ¼ inch jacks, make sure that it is a MONO adapter, not stereo.
3. Actuate the Ultimate switch in order to turn on your toy/device. Your toy/device will only remain activated while you hold the Ultimate switch in the actuated position. Once you release pressure on the Ultimate switch, the toy/device will turn off.

Troubleshooting:

Problem #1: The toy/device is not activating.

Action #1: Make sure that the connection to the toy/device is very tight. There should be NO gaps. Make sure that you are using a MONO adapter, if applicable.

Action #2: Try using your toy/device with a different switch in order to rule out the toy/device as the source of the problem. You may need to change the batteries in the toy/device.

Care of Unit:

The Ultimate Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.