



## **CLEANING & CARE INSTRUCTIONS FOR (COATED POSITIONERS & TABLE PADS) & (CLOSED CELL POSITIONERS)**

Thank you for choosing Z&Z Medical, Inc. for your medical imaging accessory needs. To ensure the proper care of your new coated or closed cell sponges and/or coated table pads, we recommend the following guidelines for cleaning, care, and storage.

### **CLEANING:**

- **TO CLEAN** the surface of coated and closed cell sponges and/or coated table pads that have become dirty or dusty with normal use and storage, wash with cold water and a mild general purpose soap (409 spray, etc.).
- Avoid using detergent cleaners that contain hydrocarbon solvents like acetone or ammonia.
- Avoid rigorous scrubbing, abrasive pads, or stiff brushes as they can damage the surface.
  
- **TO DISINFECT** the surface of your coated and closed cell sponges and/or coated table pads that have become soiled with blood, bodily fluids, barium, betadine, or other fluids, we recommend the use of hospital approved disinfectant wipes or sprays. Allow manufacturer's recommended drying time in between patients.
- A 10% Bleach solution can be used as a disinfectant if manufacture's guidelines are strictly followed. Failure to comply with these guidelines will result in a void of warranty.
- Do NOT autoclave, steam, soak, launder or submerge in water.

### **CARE & WARRANTY:**

- Z&Z Medical foam products are designed to provide aid to technologist when positioning a patient, and comfort for the patient during imaging procedures. The surface provides a barrier that can be cleaned in between patients. If your coated foam product is punctured or torn, this barrier does not provide a barrier of protection for infection control and should be taken out of service immediately and replaced. **This does NOT apply to closed cell foam products as the crosslink polyethylene foam is non-porous and resistant to fluid even if punctured or torn.**
- Because of the nature of foam products, there is a 30 day manufacturing warranty on sponges and/or table pads to be free from defects in material and workmanship under normal use. Warranty does not cover damage from misuse, mis-handling or neglect.



**STORAGE:**

- Store sponges in a clean dry area when not in use.
- Table pads should be kept on a flat surface to prevent creases.