## **AbleNet**

**QUICK START GUIDE** 

Cling

mounting arm

## WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

## To get started:

- Place suction cups on a flat, nonporous surface. Move the lever on each suction cup to the lock position to secure it to the flat, non-porous surface.
- 2. Attach threaded bolt side of mounting plate to end of mounting arm.
- Use the locking knob to adjust the mounting arm. Turning the locking knob counterclockwise to loosen and then clockwise to tighten after positioning the mounting arm.

## 3 Device Overview



 Attach mounting plate here. Compatible with AbleNet Quick Ready Mounting Plate for Assistive Technology, AbleNet Quick Ready Tablet Holder, or other mounting plates with a 1/4"-20 male threaded bolt.

Lock and Release Suction Cups