

JAY[®]

JAY CUSHIONS AND BACKS PARTS MANUAL

JANUARY 2023



improving people's lives

JAY CUSHIONS & BACKS	6
GLOSSARY OF ABBREVIATIONS	6
JAY BACKS	6
JAY BASIC BACK	6
JAY BASIC BACK CLAMP STYLE	7
JAY BASIC BACK STRAP STYLE	9
JAY BASIC BACK Discontinued	10
JAY GO BACK	11
JAY CARE BACK	12
JAY FOCUS POINT BACK	13
JAY J2 BACK	14
JAY J2 PLUS	14
JAY J2 PLUS BACK COVER	15
JAY J2 PLUS BACK PLATE ASSM	16
HEADREST	18
JAY J2 PLUS BACK SIDE PLATE	19
JAY J2 PLUS BACK SWING-AWAY LATERALS	21
JAY J2 PLUS BACK FIXED LATERALS	22
JAY J2 BACK	23
JAY J2 DEEP CONTOUR BACK	25
JAY J2 & J2 DEEP CONTOUR MOUNTING HARDWARE GUIDE	27
JAY J2 & J2 DEEP CONTOUR MOUNTING HARDWARE CHART	29
JAY J2 & J2 DEEP CONTOUR CLAMP STYLE HARDWARE	30
HEADREST	31
UNIVERSAL HEADREST MOUNTING KIT	32
JAY J3 BACK	32
J3 LT BACKS	32
J3 LT PA BACKREST	33
J3 LT PD BACKREST	35
J3 LT PDC BACKREST	36
J3 CARBON LT PA BACK	37
J3 MT BACKS	38
J3 MT PA BACKREST	39
J3 HD MT PA BACKREST	42
J3 MT PL BACKREST	44
J3 MT PD BACK	47
J3 MT PDC BACKREST	49
J3 CARBON MT PA BACK	51
J3 UT BACKS	51
J3 UT PA BACKREST	52
J3 HD UT PA BACKREST	55
J3 UT PL BACKREST	56
J3 UT PDL BACKREST	59
J3 UT PD BACK	61
J3 UT PDC BACKREST	63
J3 SH BACKS	64
J3 SH PA BACKREST	65
J3 SH PL BACKREST	67
J3 SH PDL BACKREST	69
J3 HD SH PA BACKREST	71
J3 SH PD BACK	72
J3 SH PDC BACKREST	74
MOUNTING HARDWARE	75
J3 BACK HARDWARE	76
J3 4 POINT BACK HARDWARE	78
J3 RH & RHF BACK HARDWARE	79
J3 TH BACK HARDWARE	80
LATERALS	80
JAY SWING-AWAY LATERALS	81
CRADLE-U LATERALS	83
J3 SWING-AWAY LATERAL	84
J3 FIXED LATERAL	85
HARNESSES	85
CENTER OPENING HARNESS	86
TRUNK STRAP PAD	87
CLASSIC HARNESS	88
CONTOUR HARNESS	89
J3 HARNESS ATTACHMENT HARDWARE	90
HEADRESTS	90
AXYS HEADREST	91

COBRA HEADREST	92
LINX HEADREST	93
ONYX HEADREST	95
UNIVERSAL HEADREST MOUNT	97
ADJUST-A-PLUSH HEADREST PAD	98
CONTOUR CRADLE HEADREST PAD	99
PLUSH HEADREST PAD	100
HEADS UP	102
VANITY FLAP	105
J3 TOOLS	106
JAY J3 BACK SPINE ALIGN KIT	107
J3 PLUS BACK	107
J3 PLUS BACK MT	108
J3 PLUS BACK MT WITH VANITY EXTENSION	109
J3 PLUS BACK UT	111
J3 PLUS BACK SH	112
J3 CUSTOM BACK	112
J3 MT PA CUSTOM BACKREST (FIP)	113
J3 UT PA CUSTOM BACKREST (FIP)	114
J3 SH PA CUSTOM BACKREST (FIP)	115
JAY ZIP BACK	115
JAY ZIP BACK	116
JAY ZIP BACK HARDWARE (Beginning 3/25/19)	118
JAY ZIP BACK HARDWARE (Prior to 3/25/19)	120
JAY ZIP SWING AWAY LATERALS	122
JAY ZIP FIXED LATERALS	123
JAY ZIP BACK POSITIONING COMPONENTS	124
JAY ZIP BACK HARNESS ASSEMBLIES	125
VANITY FLAP	126
JAY ZIP BACK HEADREST	127
JAY ZIP BACK TOOLS	128
JAY LUMBAR SUPPORT	129
JAY CUSHIONS	129
JAY BASIC CUSHION	130
JAY BASIC PRO CUSHION	132
JAY CARE CUSHION	134
JAY SOFT COMBI P CUSHION	135
JAY DUO CUSHION	136
JAY EASY CUSHION	138
JAY FUSION	141
JAY FUSION EVALUATION KIT	142
JAY FUSION FLUID	143
JAY FUSION AIR	149
JAY FUSION AIR - REDUCED PROFILE	155
JAY FUSION FLUID - REDUCED PROFILE	161
JAY FUSION AIR POSITIONING	167
JAY FUSION FLUID POSITIONING	173
JAY FUSION CORE	179
JAY FUSION CRYO	184
JAY FUSION CRYO - REDUCED PROFILE	188
JAY FUSION CRYO POSITIONING	192
JAY GO CUSHION	196
JAY GS CUSHION	199
JAY ION	201
JAY J2 CUSHION	204
JAY J2 CUSHION	205
JAY J2 DEEP CONTOUR CUSHION	207
JAY J2 PLUS CUSHION	212
JAY J3 CUSHION	215
JAY J3 STANDARD CUSHION	216
JAY J3 POSITIONING CUSHION	219
JAY J3 CUSHION DEMO KIT	224
JAY J3 POSITIONING CUSHION DEMO KIT	225
JAY J3 STAMPED PART NUMBER CROSS REFERENCE	226
JAY LITE CUSHION	228
JAY UNION	232
JAY X2 CUSHION	237
JAY ZIP	239
JAY CUSTOM DROPSEAT OPTION PARTS	242
JAY SOLID SEAT	243

JAY ACCESSORIES	244
JAY BOX	245
JAY J3 BACK SPINE ALIGN KIT	247
JAY J3 CUSHION DEMO KIT	248
JAY J3 POSITIONING CUSHION DEMO KIT	249
JAY CUSTOM DROPSEAT OPTION PARTS	250
JAY ADJUSTABLE SEAT	251
JAY SOLID SEAT	253
JAY NON SKID SOLID SEAT INSERT	255
JAY PROTECTOR	257
JAY SPINAL FLUID PAD	258
JAY FULL FOOT BOX WITH DIVIDER	259
JAY FULL FOOT BOX	261
JAY SINGLE FOOT BOX	262
JAY ARM TROUGH FIXED & ROTATIONAL	263
JAY LUMBAR SUPPORT	264
back cover	264

GLOSSARY OF ABBREVIATIONS


Note: see Informations Panel for file.

1AD One Arm Drive	CZ Cozy	INH Inhibit	PCB Pediatric Center of Gravity (AKA: Zippie Iris)	S/A Swing Away
3DCR 3DX Microclimatic cover for Cryo style inserts	DAB Depth-adjustable Back	INORG Inorganic coating	PCM Power Center Mount	S/F Side Frame
AAA Adult Angle Adjustable	DB Dynamic Back	INSRT Insert	PED Pediatric	SCW Screw
AACM Adjustable Center Mount	DBL Double	I/O Swing In Out	PEL Pelvic	SDLE Saddle
ABD Abductor	DC Deep Contour	IRIS Intelligent Rotation in Space	PH Push Handle	SG Side Guard
AC All Court	DL Desk Length	ISM Intelligent Seating Module	PHIL Phillips	SGTG Super Grip Thumb Guard
AD Arm Drive	DP Dual Post Armrest	JA Jay Active	P/L Pin Lock	SHCS Socket Head Cap Screw
ADJ Adjustable	D-Post Dual Post Armrest	JNR Junior	PL Plated	SHLDR Shoulder
ADPT Adapter	D-PST Dual Post Armrest	KAA Kids Angle Adjustable	PLL Pull	SHT Short
AE Air Exchange	DRV Drive	KDS Kids	PLT Plate	SLD Solid
AHAB ASAP II	DTT Diagnostic Test Tool	KT Kit (usually consists of all mounting hardware for one side)	PLST Plastic	SLNG Sling
ANG Angle	DY Dynamic Back	KVLR Kevlar	PLY Plywood	SLTD Slotted
ALR Articulating Legrest	EA Each	KZ Quickie Kidz/Zippie Kidz	PNEU Pneumatic	SLV Sleeve
ALUM Aluminum	ELR Elevating Legrest	L Left	PNK Pink	SM Small
AMH Adjustable Mounting Hardware	EXT Extension	LAT Lateral	PNT Paint	SNGL Single
ANO Anodized	EXTND Extended	LCK Lock	PNTD Painted	SOC Socket
ASAP Adjustable Seating and Positioning	F Folding Frame	LFH Left Front Horizontal	POLY Polyurethane	SPCR Spacer
ASL Adaptive Switch Laboratories	FHCS Flat Head Cap Screw	LFT Left	POST Posterior	SPEC CLR Specify Color
ASM Assembly (includes other parts)	FL Full Length	LFV Left Front Vertical	PR Pressure Relieving	SPIA Single Post Height Adjustable
ASSM Assembly	FLD Folding	LH Left Hand	PR Power Recline	SPK Spoke
ASSMLY Assembly	FL FT Flip-up Footrest	LH Left Horizontal	PR Pair	SPR Super Grip Handrim
ASSY Assembly	FLP Flip	LK Lock	(PR) Pair	SQZ Squeeze
AT Anti-tip	FLRD Flared	LNG Long	PRCR Premium Leatherette cover for Cryo style inserts	SS Stainless Steel
ATT Attachment	FM Foam	LO Lift Off	PRE-ISC Pre-Ischial	ST Seat
BB Basketball	FM Front Mount	L/O Lift Off	PREM Premium Leatherette cover material	ST/AP Self Tapping
BC Breezy Chrome	FNDR Fender	LP Loop	PROJ Projection	STCR Stretch Cover for Cryo Style inserts
BCK Back	FPH Folding Push Handle	LPS Loops	PRPL Purple	STD Standard
BCKBRD Backboard	FT Foot	LRG Large	PTL Partial	STL Style
BH Bultion Head	F/REST Footrest	LRH Left Rear Horizontal	PSH Push	STR Stretch cover material
BHCS Bultion Head Cap Screw	FRNT Front	LRV Left Rear Vertical	PT Point	STRLR Stroller
BKL Buckle	FT PLT Footplate	LS Low Sheer	PTRN Pattern	STRT Straight
BKRST Backrest	FTRST Footrest	LT Long Tab	PWR Power	SUP Support
B-LOCK Back Lock	F/UP Flip-up	LT Natural Fit Lite Handrims	Q1 Quickie 1	SYS System
BLD Build Up	FWD Front Wheel Drive	LTRL Lateral	Q2 Quickie 2	T Tab
BLK Black	FXT Fixed	LUMB Lumber	Q2HIP Quickie 2 High Performance	TG Thumb Guard (std)
BLU Blue	G8 Grade 8 higher grade = stronger	LV Left Vertical	QC Quickie Carbon	THRD Threaded
BR Breezy	GP Quickie GP	LW Low	QR Quick Release	Ti Titanium
BRD Board	GPS Quickie GPS	LX Quickie LX	QRx Quickie RX	TL Tall
BRNG Bearing	GPV Quickie GPV	LXI Quickie LXi	QTI Quickie Titanium	TLC Tilt Lift Comb
BRKT Bracket	GNRC Generic	MACH Machine	QT/QTs Quickie Tilt & Space	TLT Tilt
BSHNG Bushing	H Hole	MECH Mechanism	QUAD Quadrant Pad	TNNS Tennis
BTIN Bultion	HB High Brake	MED Medium	R Right	TIS Tilt-in-Space
BTTRY Battery	HD Head or Heavy Duty	mm Millimeter	RCLNR Recliner	TTS Tilt-in-Space
BZ Breezy	HVD Heavy Duty	MNT Mount	REC Receiver	UNIV Universal
CAA Composite Angle Adjustable	HDRST Headrest	MNTG Mounting	REG Regular	UNTHRD Unthreaded
CG Center of Gravity	HDWR Hardware	MOD Modify or Modification	REL Release	UPH Upholstery
CGT Center of Gravity Tilt (AKA: IRIS)	HEX Hexagon	M/S Machine Screw	REV Reverse	VERT Vertical
CHRGR Charger	HH Hemi-Height or Hex Head	MWD Mid Wheel Drive	RH Right Hand	W With
CJSM Color Joystick Module	HK Hook	Nat.Fit Natural Fit Handrim	RH Right Horizontal	WD Wide
CLMP Clamp	HL Heel Loops	NAUG Naugahyde	RFH Right Front Horizontal	W/F Water Fall
CLNCHR Cincher	HM High Mount	NF Non-folding	RFV Right Front Vertical	W/O Without
CLR Clear	HNDL Handle	NF Natural Fit Handrim	RNUT(S) Rivnut(s)	WHL Wheel
COMP Complete	HNGR Hanger	NFLT Natural Fit Lite Handrim	RoHS Restriction of Hazardous Substances	WLDMNT Weldment
CMPST Composite	HP High Pressure	NR Non-recessed	RR Rear	XS Cross Spoke
CRM Chrome	HR Handrim	OBLIQ Obliquity	RRH Right Rear Horizontal	YLW Yellow
C/S Cap Screw	HRDWR Hardware	OD Oblique	RRV Right Rear Vertical	Z2 Zippie 2
C-S Countersunk	HRNSS Harness	OD Outer/Outside Dimension	RT Right	ZN Zinc
CSTR Caster	HS High Speed	O/S Old Style	RV Right Vertical	ZP Zippie
CUSH Cushion	HT Height or Heat Treated	OUTBRD Outboard	RV Quickie Revolution	ZP2 Zippie 2
CV Conventional	ID Inner/Inside Dimension Diameter	P1 Quickie P100	RWD Rear Wheel Drive	
CVR Cover	IN In/Out (for swing in/out hangers)	P2 Quickie P200	SA Swing Away	
CVRS Covers	INBRD Inboard	P3 Quickie P300		
CYL Cylinder	INC Incontinent	P5 Quickie/Zippie P500		

Pos.	Item Number	Description	Remarks
		GLOSSARY OF ABBREVIATIONS	see attachment in Informations Panel

Note: Effective August 25 2008

Note: Item 1 Back Shell not available as a replacement part; please order complete back from Jay Order Form if replacement shell is needed.

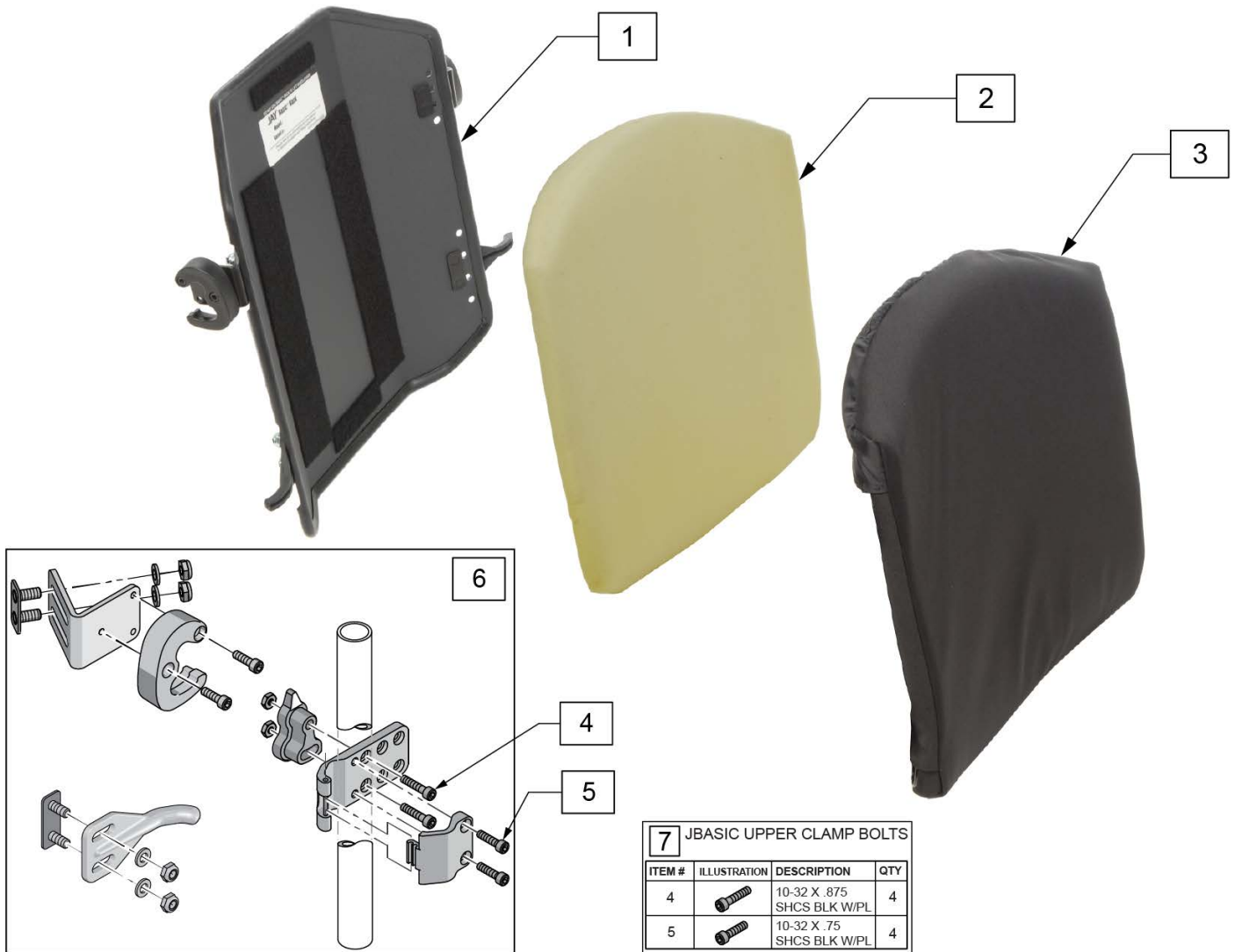


7 JBASIC UPPER CLAMP BOLTS			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
4		10-32 X .875 SHCS BLK W/PL	4
5		10-32 X .75 SHCS BLK W/PL	4

Pos.	Item Number	Description	Remarks
2	B3300	JBASIC BACK FOAM BASE,20x20	20x20
2	B3306	JBASIC BACK FOAM BASE,20x16	20x16
2	B3308	JBASIC BACK FOAM BASE,20x18	20x18
2	B3340	JBASIC BACK FOAM BASE,14x20	14x20
2	B3346	JBASIC BACK FOAM BASE,14x16	14x16
2	B3348	JBASIC BACK FOAM BASE,14x18	14x18
2	B3360	JBASIC BACK FOAM BASE,16x20	16x20
2	B3366	JBASIC BACK FOAM BASE,16x16	16x16
2	B3368	JBASIC BACK FOAM BASE,16x18	16x18
2	B3380	JBASIC BACK FOAM BASE,18x20	18x20

Pos.	Item Number	Description	Remarks
2	B3386	JBASIC BACK FOAM BASE,18x16	18x16
2	B3388	JBASIC BACK FOAM BASE,18x18	18x18
3	C3300FC	JBASIC BACK COVER,20x20	20x20
3	C3306FC	JBASIC BACK COVER,20x16	20x16
3	C3308FC	JBASIC BACK COVER,20x18	20x18
3	C3340FC	JBASIC BACK COVER,14x20	14x20
3	C3346FC	JBASIC BACK COVER,14x16	14x16
3	C3348FC	JBASIC BACK COVER,14x18	14x18
3	C3360FC	JBASIC BACK COVER,16x20	16x20
3	C3366FC	JBASIC BACK COVER,16x16	16x16
3	C3368FC	JBASIC BACK COVER,16x18	16x18
3	C3380FC	JBASIC BACK COVER,18x20	18x20
3	C3386FC	JBASIC BACK COVER,18x16	18x16
3	C3388FC	JBASIC BACK COVER,18x18	18x18
6	JBHDWUL	JBASIC UPPER CLAMP LEFT	COMPLETE UPPER LEFT CLAMP
6	JBHDWUR	JBASIC UPPER CLAMP RIGHT	COMPLETE LOWER RIGHT CLAMP
6	JBHDWH	JBASIC CLAMP HARDWARE (PAIR)	COMPLETE CLAMP HDW, L & R
6	101575S	ASSM MNT HRDW RH BASIC BK	Right
6	101576S	ASSM MNT HRDW LH BASIC BK	Left
7	JBHDWUB	JBASIC UPPER CLAMP BOLTS	ALL BOLTS, LEFT OR RIGHT

Note: effective August 25 2008



Note: Item 1 Back Shell not available as a replacement part; please order complete back from Jay Order Form if replacement shell is needed.

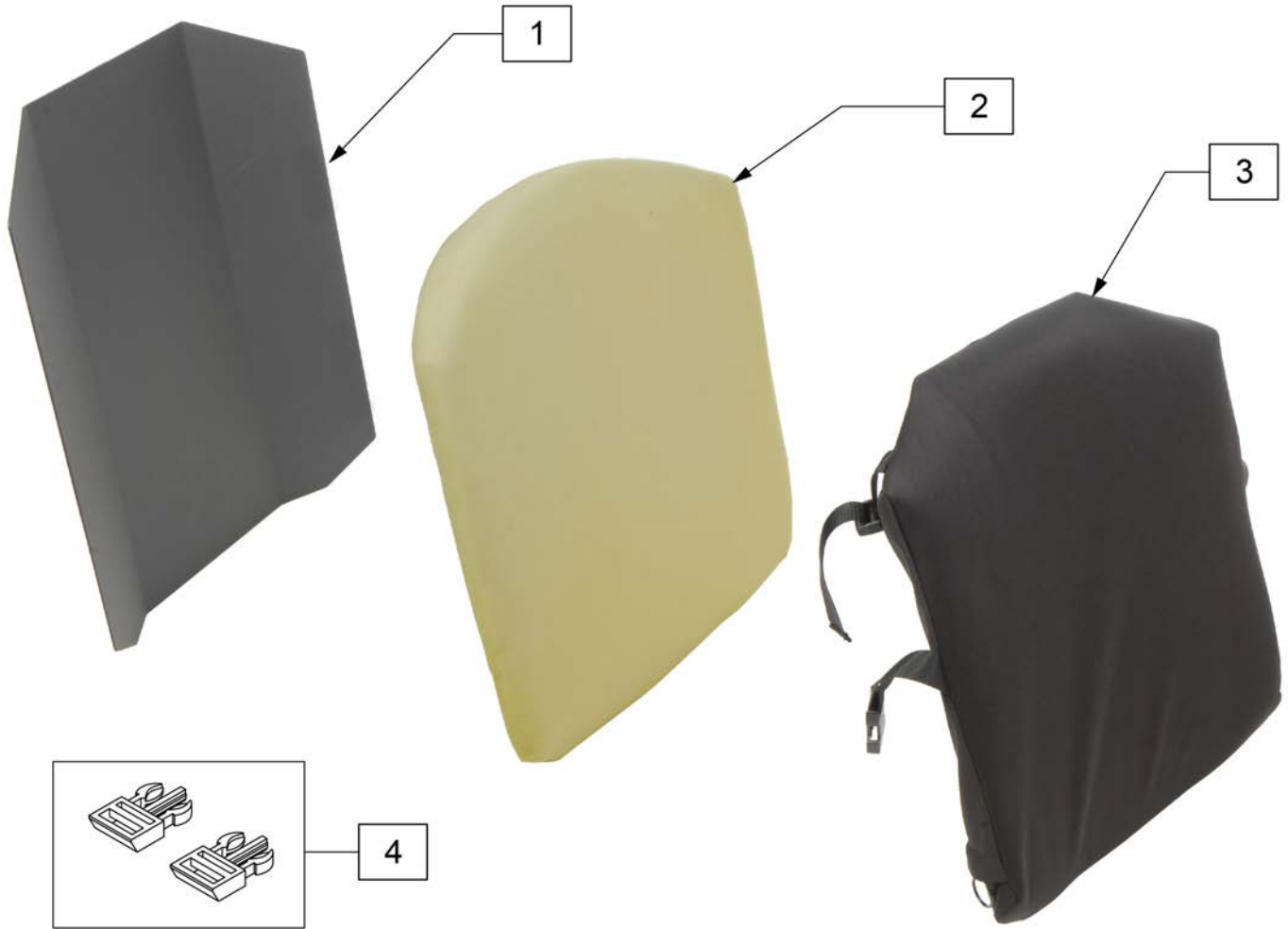

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
2	B3300	JBASIC BACK FOAM BASE,20x20	20x20	3	C3306FS	JBASIC BACK COVER,20x16	20x16
2	B3306	JBASIC BACK FOAM BASE,20x16	20x16	3	C3308FS	JBASIC BACK COVER,20x18	20x18
2	B3308	JBASIC BACK FOAM BASE,20x18	20x18	3	C3340FS	JBASIC BACK COVER,14x20	14x20
2	B3340	JBASIC BACK FOAM BASE,14x20	14x20	3	C3346FS	JBASIC BACK COVER,14x16	14x16
2	B3346	JBASIC BACK FOAM BASE,14x16	14x16	3	C3348FS	JBASIC BACK COVER,14x18	14x18
2	B3348	JBASIC BACK FOAM BASE,14x18	14x18	3	C3360FS	JBASIC BACK COVER,16x20	16x20
2	B3360	JBASIC BACK FOAM BASE,16x20	16x20	3	C3366FS	JBASIC BACK COVER,16x16	16x16
2	B3366	JBASIC BACK FOAM BASE,16x16	16x16	3	C3368FS	JBASIC BACK COVER,16x18	16x18
2	B3368	JBASIC BACK FOAM BASE,16x18	16x18	3	C3380FS	JBASIC BACK COVER,18x20	18x20
2	B3380	JBASIC BACK FOAM BASE,18x20	18x20	3	C3386FS	JBASIC BACK COVER,18x16	18x16
2	B3386	JBASIC BACK FOAM BASE,18x16	18x16	3	C3388FS	JBASIC BACK COVER,18x18	18x18
2	B3388	JBASIC BACK FOAM BASE,18x18	18x18	4	JBHDWSC	JBASIC BACK STRAP BUCKLE	
3	C3300FS	JBASIC BACK COVER,20x20	20x20				

Note: parts only; discontinued August 25, 2008

Note: Item 3 Back Shell not available as a replacement part; please order complete back from Jay Order Form if replacement shell is needed.

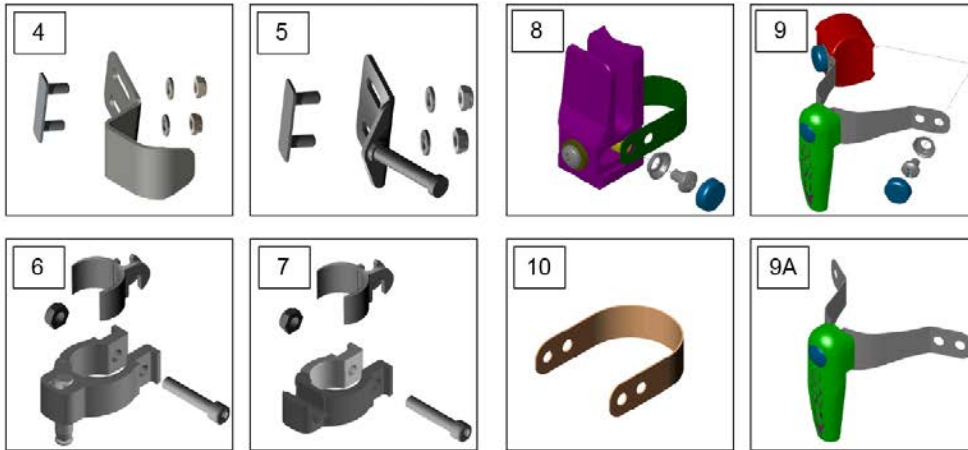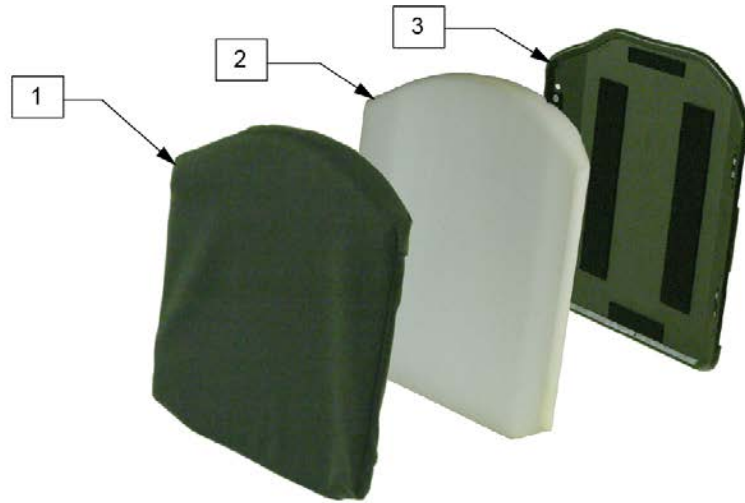


Pos.	Item Number	Description	Pos.	Item Number	Description
1	C3346	COVER BASIC BACK 14 X 16	2	B3340	FOAM BASIC BACK 14 X 20
1	C3348	COVER BASIC BACK 14 X 18	2	B3366	FOAM BASIC BACK 16 X 16
1	C3340	COVER BASIC BACK 14 X 20	2	B3368	FOAM BASIC BACK 16 X 18
1	C3366	COVER BASIC BACK 16 X 16	2	B3360	FOAM BASIC BACK 16 X 20
1	C3368	COVER BASIC BACK 16 X 18	2	B3386	FOAM BASIC BACK 18 X 16
1	C3360	COVER BASIC BACK 16 X 20	2	B3388	FOAM BASIC BACK 18 X 18
1	C3386	COVER BASIC BACK 18 X 16	2	B3380	FOAM BASIC BACK 18 X 20
1	C3388	COVER BASIC BACK 18 X 18	2	B3306	FOAM BASIC BACK 20 X 16
1	C3380	COVER BASIC BACK 18 X 20	2	B3308	FOAM BASIC BACK 20 X 18
1	C3306	COVER BASIC BACK 20 X 16	2	B3300	FOAM BASIC BACK 20 X 20
1	C3308	COVER BASIC BACK 20 X 18	4, 5	100197	BASIC BACK MOUNT HARDWARE
1	C3300	COVER BASIC BACK 20 X 20	6, 7	M-250C	KIT CLAMP UNIVERSAL RETRO-FIT
2	B3346	FOAM BASIC BACK 14 X 16	8, 9, 9A, 10	M-250S	KIT STRAP UNIVERSAL RETRO-FIT
2	B3348	FOAM BASIC BACK 14 X 18			

Note: Individual replacement parts are no longer available for 12, 16, 20 wide backs. New Jay Go back must be ordered for those sizes.




Pos.	Item Number	Description
1	SJGOB14S	JAY GO BCK 14W SHT STRAP ASM
1	SJGOB18S	JAY GO BCK 18W SHT STRAP ASM
1	SJGOB22S	JAY GO BCK 22W SHT STRAP ASM
1	SJGOB14T	JAY GO BCK 14W TAL STRAP ASM
1	SJGOB18T	JAY GO BCK 18W TAL STRAP ASM
1	SJGOB22T	JAY GO BCK 22W TAL STRAP ASM
2,3	CJGOB14S	JAY GO BACK 14W SHORT COVER
2,3	CJGOB18S	JAY GO BACK 18W SHORT COVER
2,3	CJGOB22S	JAY GO BACK 22W SHORT COVER
2,3	CJGOB14T	JAY GO BACK 14W TALL COVER
2,3	CJGOB18T	JAY GO BACK 18W TALL COVER
2,3	CJGOB22T	JAY GO BACK 22W TALL COVER
2,3	CJGOB14SM	JGO BK 14W SHORT MICRO COVER
2,3	CJGOB14TM	JGO BK 14W TALL MICRO COVER
2,3	CJGOB18SM	JGO BK 18W SHORT MICRO COVER
2,3	CJGOB18TM	JGO BK 18W TALL MICRO COVER
2,3	CJGOB22SM	JGO BK 22W SHORT MICRO COVER
2,3	CJGOB22TM	JGO BK 22W TALL MICRO COVER

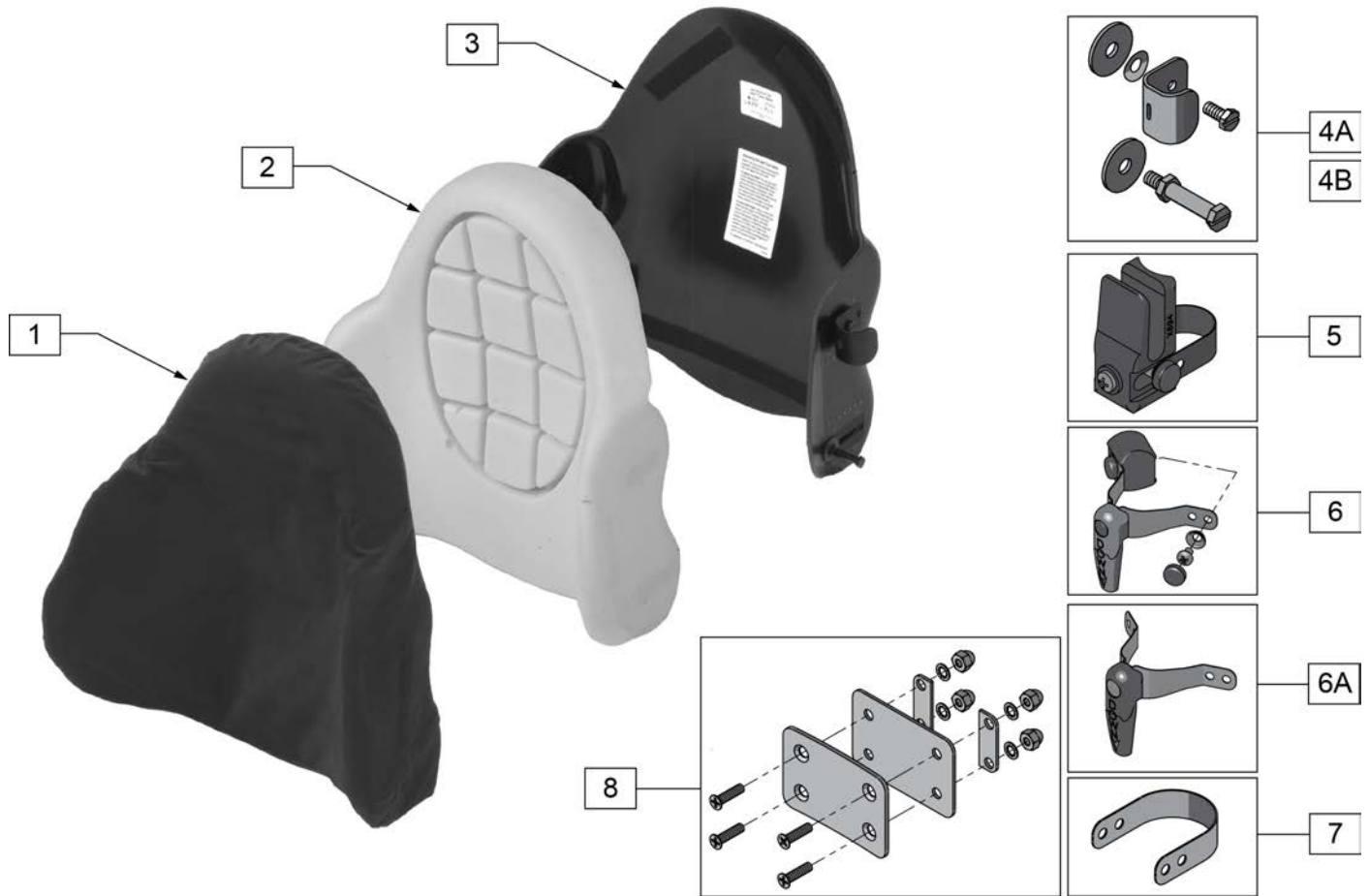

Pos.	Item Number	Description	Remarks
1	C3500B	JAYCARE COVER BACK REGULAR AD BLK	
1	C3502B	JAYCARE COVER BACK 20 BLK	
1	C3505B	JAYCARE COVER BACK 16 BLK	
2	B3500	JAYCARE BASE BACK 18	
2	B3502	JAYCARE BASE BACK 20	
2	B3505	JAYCARE BASE BACK 16	
3	S3500	JAYCARE SHELL BACK 18	
3	S3502	JAYCARE SHELL BACK 20	
3	S3505	JAYCARE SHELL BACK 16	
4A	M-300	STANDARD HARDWARE KIT-JAYCARE SHELL	
4B	M-301	EXTERIOR HARDWARE KIT-JAYCARE SHELL	
5	680C	BOTTOM UNIVERSAL POST BRACKETS	
6	681C	TOP BRACKET ASSM - "QUARTER-TURN"	fits 5/8" to 1.0" diameter tubing
6A	2681C	TOP LONG 1/4 BRACKET ASSM	fits 1.0" to 1.25" diameter tubing
7	880997	LONG STRAP KIT	USED ON 1 1/8" DIAMETER BACK POST
8	M-500	HEADREST ADAPTOR KIT	


Note: Parts only, for complete Backrest see Jay Focus Point Back order form.

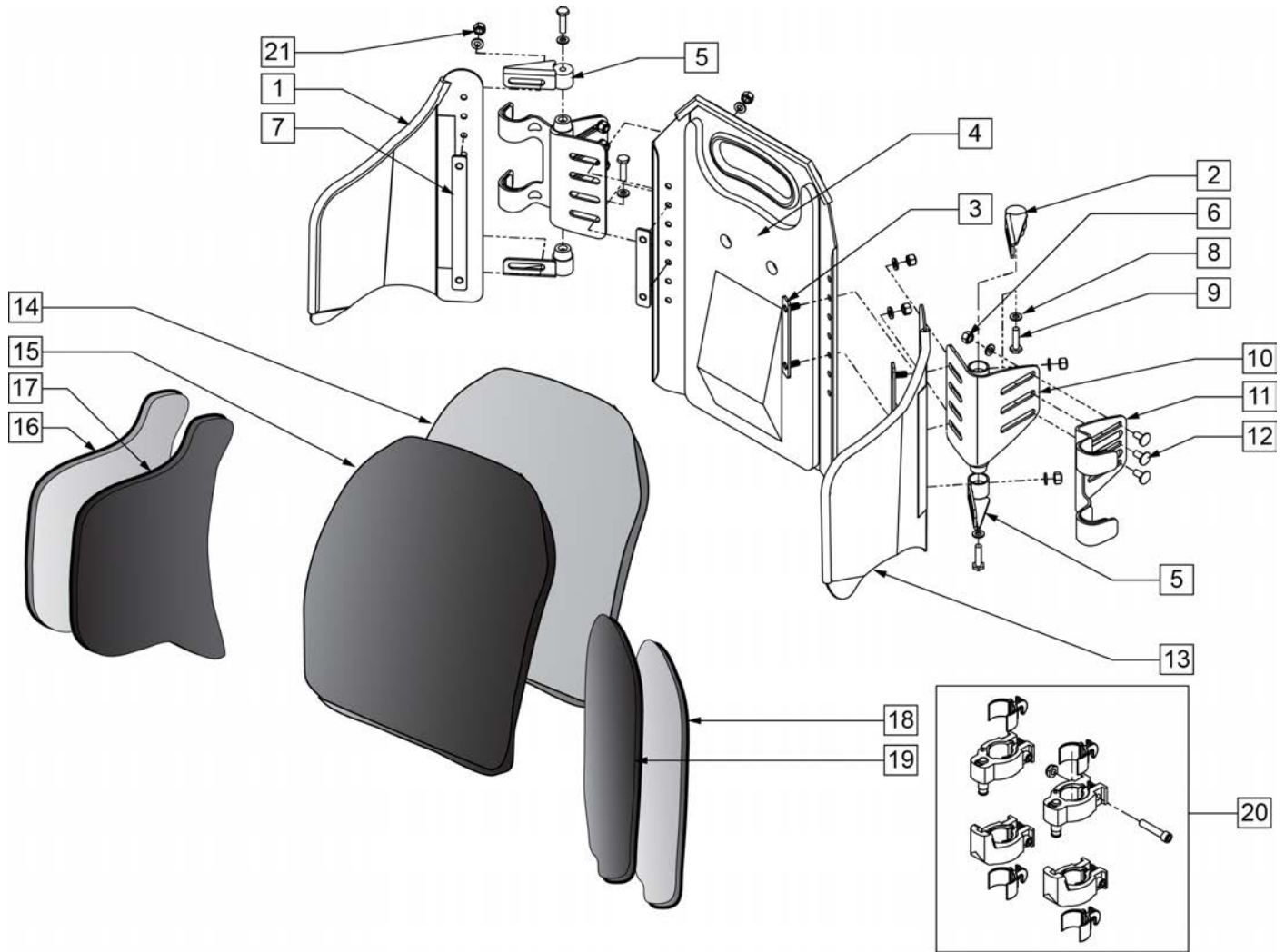

Pos.	Item Number	Description	Pos.	Item Number	Description
1	FP-LS-1415R	FP RT LATERAL SHELL 14-15	7	FP-NPLT-LNG	FP NUTPLATE LONG
1	FP-LS-1617R	FP RT LATERAL SHELL 16-17	8	106943	1/4 AN WASHER FLAT
1	FP-LS-1819R	FP RT LATERAL SHELL 18-19	9	106958	1/4-20 X 1 HEX C/S
1	FP-LS-2021R	FP RT LATERAL SHELL 20-21	10	FP-MBRKT	FP MAIN BRKT
2	FP-HPLT-ML	FP HINGE PLT MALE	11	FP-HANG-V3	FP HANGER V3
3	FP-NPLT-SHT	FP NUTPLATE SHORT	12	M-455036-QU	1/4-28 X 9/16 SPEC ROUND HD
4	FP-CS-1415T	FP CENTER SHELL 14-15 TALL	13	FP-LS-1415L	FP LFT LATERAL SHELL 14-15
4	FP-CS-1415	FP CENTER SHELL 14-15	13	FP-LS-1617L	FP LFT LATERAL SHELL 16-17
4	FP-CS-1617T	FP CENTER SHELL 16-17 TALL	13	FP-LS-1819L	FP LFT LATERAL SHELL 18-19
4	FP-CS-1617	FP CENTER SHELL 16-17	13	FP-LS-2021L	FP LFT LATERAL SHELL 20-21
4	FP-CS-1819T	FP CENTER SHELL 18-19 TALL	14	FP-CF-1415T	FP BCK FOAM CENTER 1415T
4	FP-CS-1819	FP CENTER SHELL 18-19	14	FP-CF-1415	FP BCK FOAM CENTER 1415
4	FP-CS-2021T	FP CENTER SHELL 20-21 TALL	14	FP-CF-1617T	FP BCK FOAM CENTER 1617T
4	FP-CS-2021	FP CENTER SHELL 20-21	14	FP-CF-1617	FP BCK FOAM CENTER 1617
5	FP-HPLT-FL	FP HINGE PLT FEMALE	14	FP-CF-1819T	FP BCK FOAM CENTER 1819T
6	106935	NUT NYLOCK THIN 1/4-28 ROHS	14	FP-CF-1819	FP BCK FOAM CENTER 1819


Pos.	Item Number	Description	Pos.	Item Number	Description
14	FP-CF-2021T	FP BCK FOAM CENTER 2021T	17	FP-LCVR-1415R	FP LATERAL COVER RT 14-15
14	FP-CF-2021	FP BCK FOAM CENTER 2021	17	FP-LCVR-1617R	FP LATERAL COVER RT 16-17
15	FP-CVR-1415T	FP BCK COVER CENTER 1415T	17	FP-LCVR-1819R	FP LATERAL COVER RT 18-19
15	FP-CVR-1415	FP BCK COVER CENTER 1415	17	FP-LCVR-2021R	FP LATERAL COVER RT 20-21
15	FP-CVR-1617T	FP BCK COVER CENTER 1617T	18	FP-LF-1415L	FP LATERAL FOAM LFT 14-15
15	FP-CVR-1617	FP BCK COVER CENTER 1617	18	FP-LF-1617L	FP LATERAL FOAM LFT 16-17
15	FP-CVR-1819T	FP BCK COVER CENTER 1819T	18	FP-LF-1819L	FP LATERAL FOAM LFT 18-19
15	FP-CVR-1819	FP BCK COVER CENTER 1819	18	FP-LF-2021L	FP LATERAL FOAM LFT 20-21
15	FP-CVR-2021T	FP BCK COVER CENTER 2021T	19	FP-LCVR-1415L	FP LATERAL COVER LFT 14-15
15	FP-CVR-2021	FP BCK COVER CENTER 2021	19	FP-LCVR-1617L	FP LATERAL COVER LFT 16-17
16	FP-LF-1415R	FP LATERAL FOAM RT 14-15	19	FP-LCVR-1819L	FP LATERAL COVER LFT 18-19
16	FP-LF-1617R	FP LATERAL FOAM RT 16-17	19	FP-LCVR-2021L	FP LATERAL COVER LFT 20-21
16	FP-LF-1819R	FP LATERAL FOAM RT 18-19	20	M-260	KIT CLAMP FP BACK
16	FP-LF-2021R	FP LATERAL FOAM RT 20-21	21	M-201105	1/4-20 NYLOCK

Note: parts only; for complete Backrest see J2 Plus Back order form.

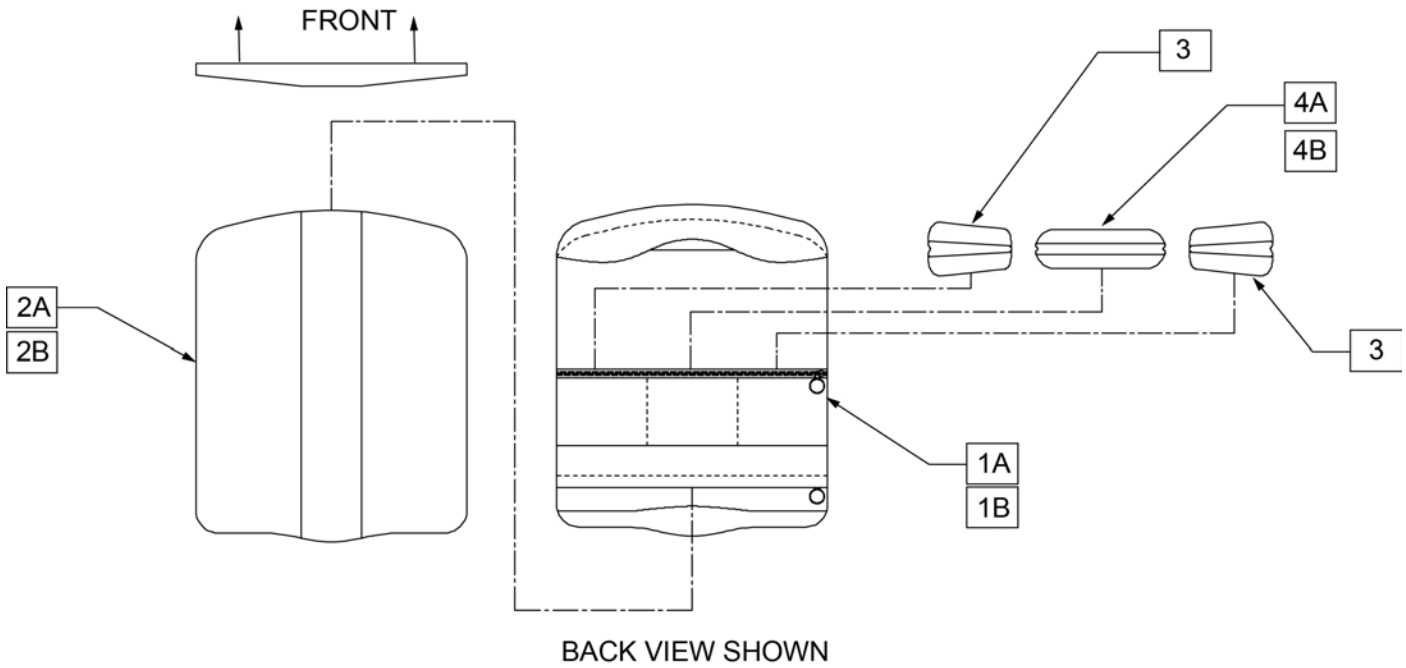

Pos.	Item Number	Description
1A	C202023A	J2 PLUS BACK COVER 20 - 23 AIR
1C	C202326A	J2 PLUS BACK COVER 23 - 26 AIR
2A	B202023	J2 PLUS BACK FOAM 20 - 23
2B	B202426	J2 PLUS BACK FOAM 24 - 26
3	B200007	ASIS FOAM BARIATRIC
4A	B200006	SACURM FOAM SMALL BARIATRIC
4B	B200005	SACRUM FOAM LARGE BARIATRIC
1A,2A,3,4A	C202023AB	J2 PLUS COVER ASSM 20 - 23 AIR
1B,2B,3,4B	C202326AB	J2PLUS BACK CVR ASM 23-26" AIR


Note: Parts only, for complete Backrest see J2 Plus Back order form.

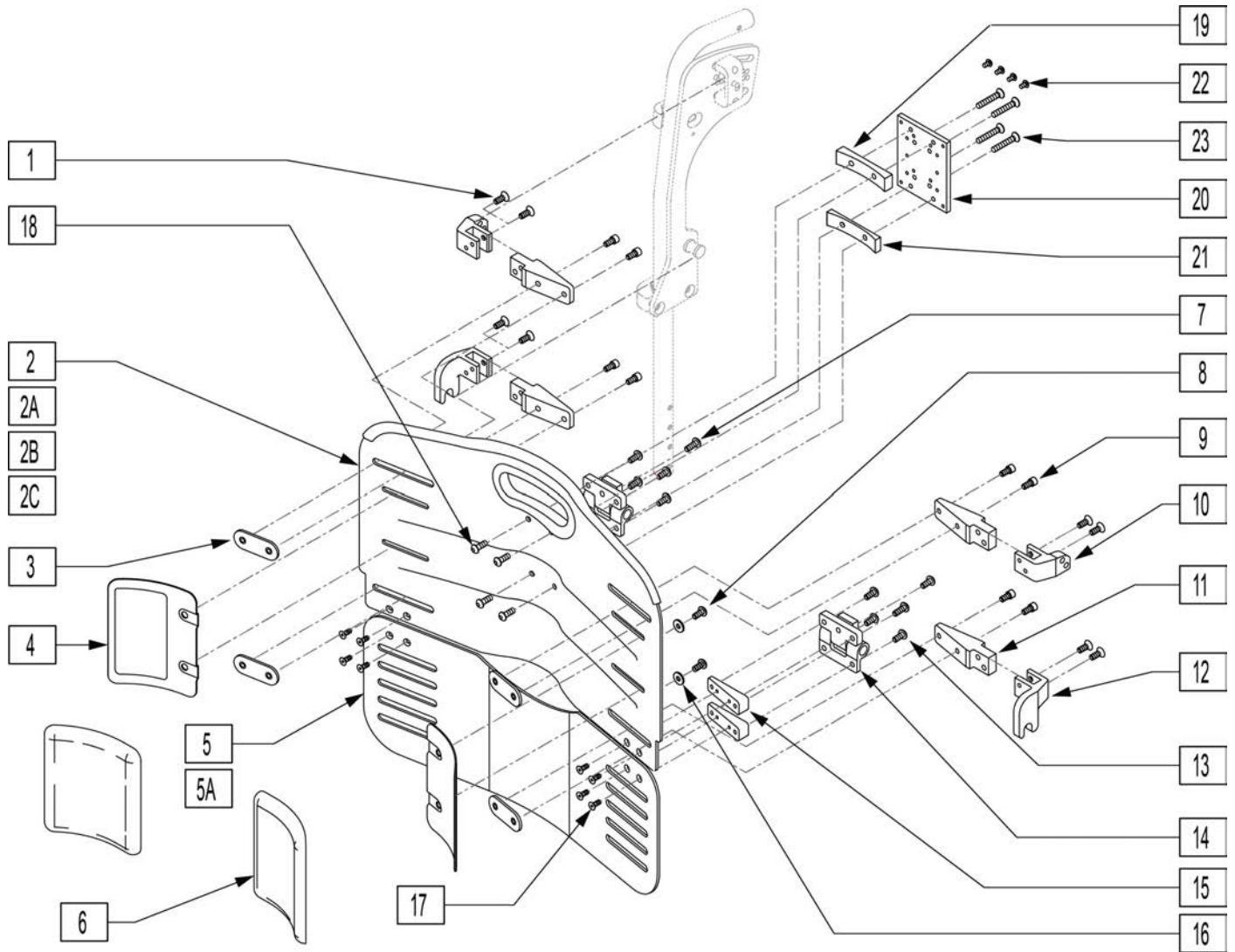


Pos.	Item Number	Description
1	M-200007	FHCS .250 - 20 X 1.00 W/ PATCH BLK
3	200010-MJ	NUT PLATE BARIATRIC
4	200110-MJ	J2 PLUS GROWTH PLATE ASSM
5	202023-MJ	J2 PLUS VANITY FLAP ASSM 20 - 23
5A	202326-MJ	J2 PLUS VANITY FLAP ASSM 23 - 26
6	B200001	GROWTH PLATE FOAM
7	M-200010	BHCS .250 - 20 X .38 W/ PATCH BLK
8	M-200043	BHCS .250 - 20 X .38 W/ PATCH BLK
9	M-200039	BHCS .250 - 20 X .38 W/ PATCH BLK
10	200009-MJ	UPPER SHELL CONNECTOR
12	200006-MJ	LOWER SHELL CONNECTOR
13	M-200029	BHCS .250 - 20 X .38 W/ PATCH BLK
14	M-200006	J2 PLUS HINGE FRICTION ADJUST
15	200014-MJ	J2 PLUS HINGE PLATE SHIM

Pos.	Item Number	Description
16	M-200036	WASHER .25 ANSI SERIES N PLAIN BLK
17	M-200040	FHCS 10 - 32 X .38 W/ PATCH BLK
18	M-200042	BHCS .25 - 28 X 63 W/ PATCH BLK
19	200023-MJ	UNIVERSAL HEADREST UPPER MOUNTING
20	XM2025	MNTNG PLATE/PEDIAT.HEAD REST
21	200022-MJ	UNIVERSAL HEADREST LOWER MOUNTING
22	226018-QU	BHCS 10 - 32 X 3/8 W/ PATCH BLK
23	479034	FHCS, 250 - 28 X 1.50
9,16-18	200121-MJ	J2 PLUS HINGE ASSM
1, 13, 14	200102-MJ	J2 PLUS UPPER CONNECTOR ASM
1, 14, 15	200100-MJ	J2 PLUS LOWER CONNECTOR ASSM
1-2, 3-5, 6-17	200130H-MJ	BCKRST ASM 20-23 J2PLUS HEADREST
1-2A, 3-5, 6-17	200130-MJ	J2 PLUS BACKREST ASSM 20-23
1-2C, 3-5, 6-17	200131H-MJ	J2 PLUS BACKREST ASSM 23-26 HEADREST
1-2B, 3-5A, 6-21	200131-MJ	J2 PLUS BACKREST ASSM 23-26
18-21	200206-MJ	J2 HEADREST HARDWARE KIT
18-21, 23	200207-MJ	J2 HEADREST HARDWARE KIT, WT
18-22	200208-MJ	J2 HEADREST HARDWARE KIT, OT


HEADREST

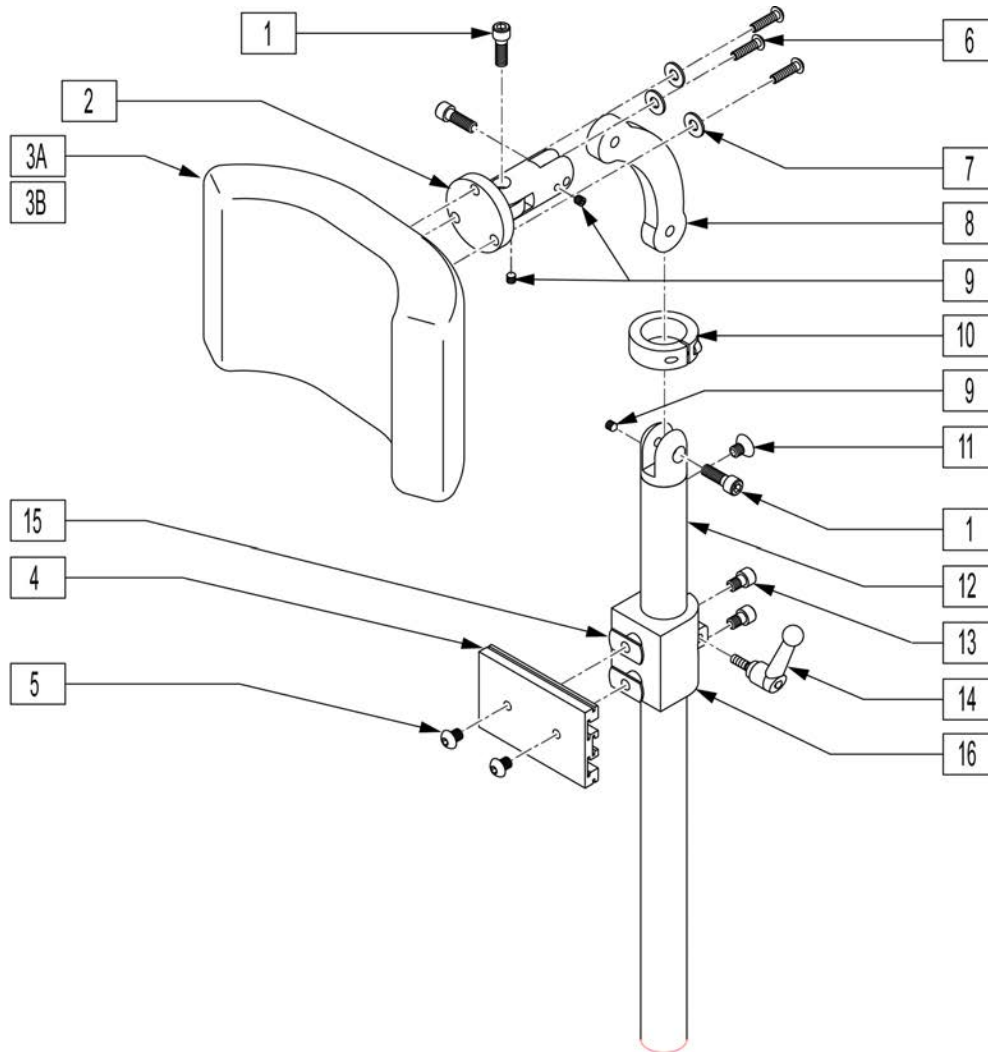


Pos.	Item Number	Description	Remarks
1	221115	1/4-20 x 3/4" SOC HD C/S BLK	
3A	750696	HEADREST W/ZIPPER MEDIUM CG	8x4.5 pad
3B	750697	HEADREST W/ZIPPER LARGE CG	9x4.5 pad
4	505932	HEADREST MOUNTING TRACK	
5	476304	1/4-28 x .31 BHCS W/PATCH	
6	221010	10-32 x .75 SHCS BLK W/PL	
7	261204	#10 AN WASHER .063	
8	331077	HEADREST LINK 6 INCH	
9	341045	#10-24 CONE POINT SET SCREW	
10	505933	HEADREST COLLAR	
11	226072	1/4-20 x 1/2 FH C/S BLK	
13	226095	1/4-20 x 3/8" SOC C/S BLK ZN	
14	341046	HEADREST CLAMPING HANDLE SCREW	
15	340033	1/4-20 WELD NUTS SLAB BASE	
4,5,13-16	888101	CLAMP HEADREST RIGID BACK	
1,2,4,6-16	881003	HEADREST MOUNTING BRACKET ASSM	
1-3A,4-16	881907	HEADREST ASSEMBLY MEDIUM	8x4.5 pad
1-3B,4-16	881908	HEADREST ASSEMBLY LARGE	9x4.5 pad

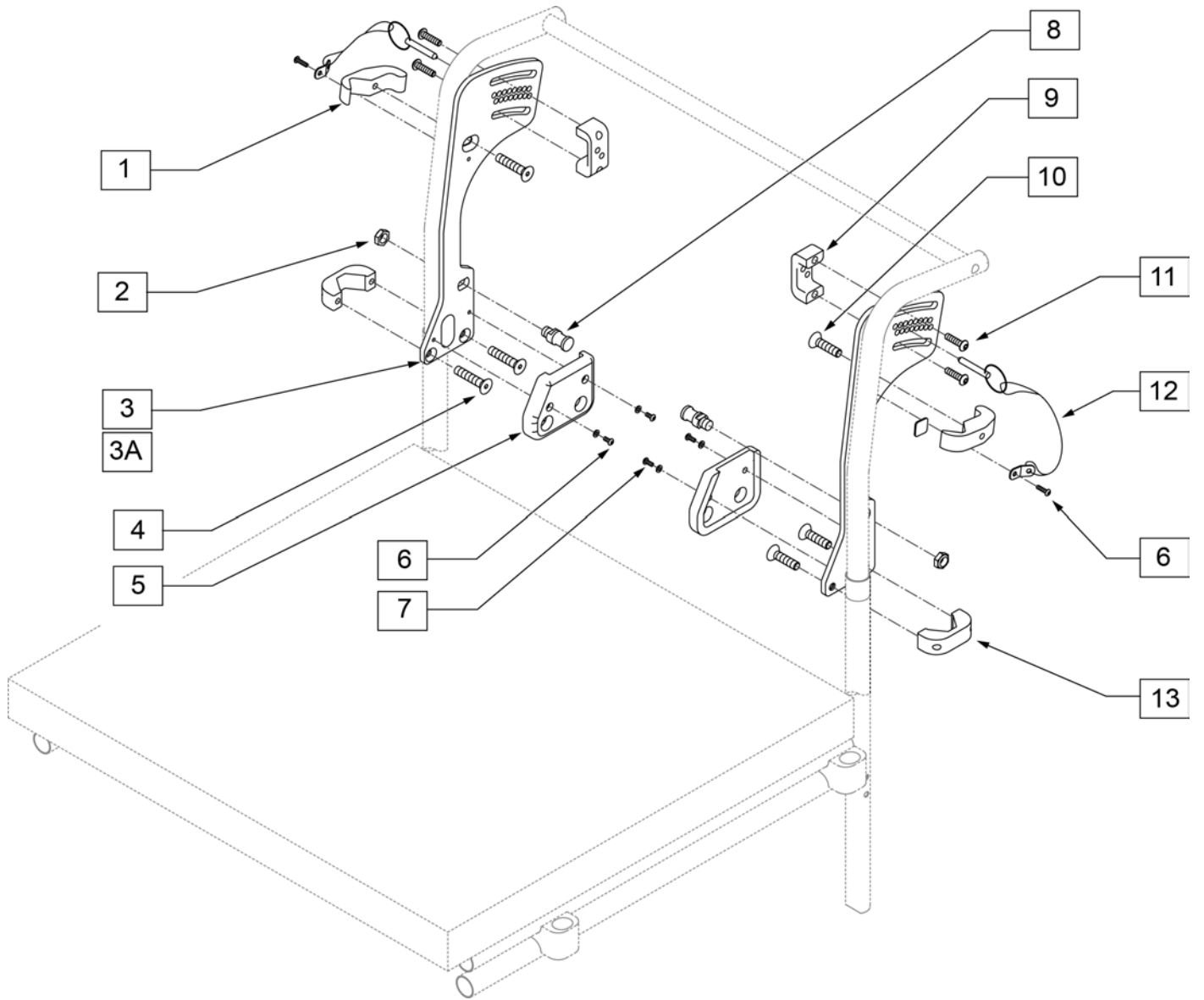

Pos.	Item Number	Description	Remarks
2	M-200030	NUT, JAM NYLOCK .4375-20	
4	M-200034	FHCS, .313 -18 X 1.25 W/PATCH BLK	
5	B200003	SIDE PLATE FOAM J2 PLUS R/H	Right
5	B200004	SIDE PLATE FOAM J2 PLUS L/H	Left
6	468903	10-32 X .50 BHCS W/PATCH	
7	2702	#10 SAE WASHER	
8	200004-MJ	J2 PLUS PSIS PIVOT	
9	200017-MJ	PIN RETAINER L/H J2 PLUS	Left
9	200016-MJ	PIN RETAINER R/H J2 PLUS	Right
10	M-200035	FHCS, .313 -18 X 1.0 W/PATCH BLK	
11	M-200031	BHCS, .313 -18 X 1.0 W/PATCH BLK	
12	M-200003	J2 PLUS LOCATING PIN	
13	200002-MJ	LOWER SIDE CLAMP	
1-13	200202-MJ	SIDE PLATE ASM RH 8 DEG J2PLUS	
1-13	200203-MJ	SIDE PLATE ASM LH 8 DEG J2PLUS	
1,2,3A, 4-13	200200-MJ	SIDE PLATE ASM RH 0 DEG J2PLUS	
1,2,3A, 4-13	200201-MJ	SIDE PLATE ASM LH 0 DEG J2PLUS	

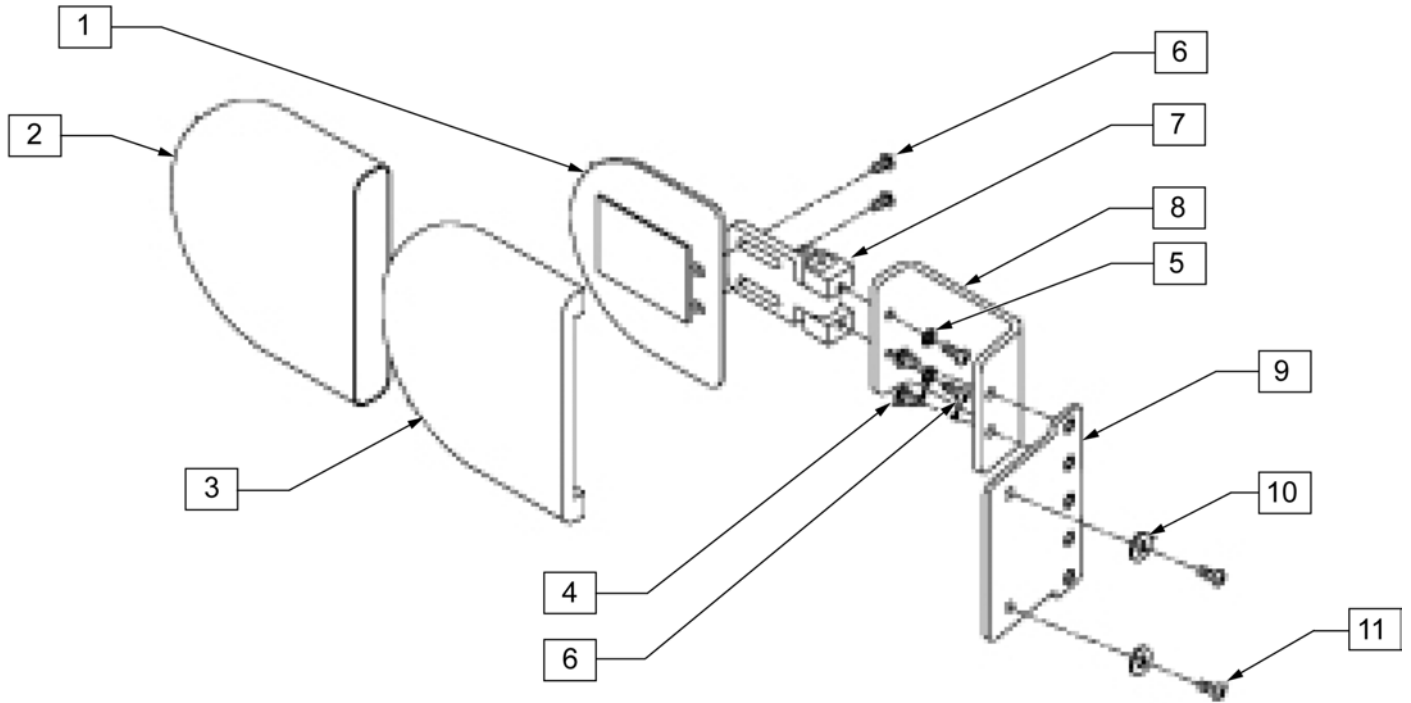


Pos.	Item Number	Description	Remarks
4	226020	1/4-20 X 3/8 BH C/S W/PATCH	
6	XM2141S	1/4-28 x .31 BHSC W/ PATCH	
7	476180	ADJUSTABLE LATERAL SWING AWAY	
8	200026-MJ	J2 PLUS S/A LATERAL BRACKET	
9	200028-MJ	J2 PLUS LATERAL BRACKET PLATE	
10	M-200036	WASHER, .25 ANSI SERIES N PLAIN BLK	
11	M-200037	BHSC .250 - 20 X .63 W/ PATCH BLK	
1-3	476230	LATERAL THORACIC SUPPORT LARGE W/O HARDWARE	
1-11	200124-MJ	J2 PLUS S/A LATERAL SUPPORT ASSM	RIGHT
1-11	200125-MJ	J2 PLUS S/A LATERAL SUPPORT ASSM	LEF
1-11	200127-MJ	J2 PLUS S/A LATERAL SUPPORT ASSM	PAIR



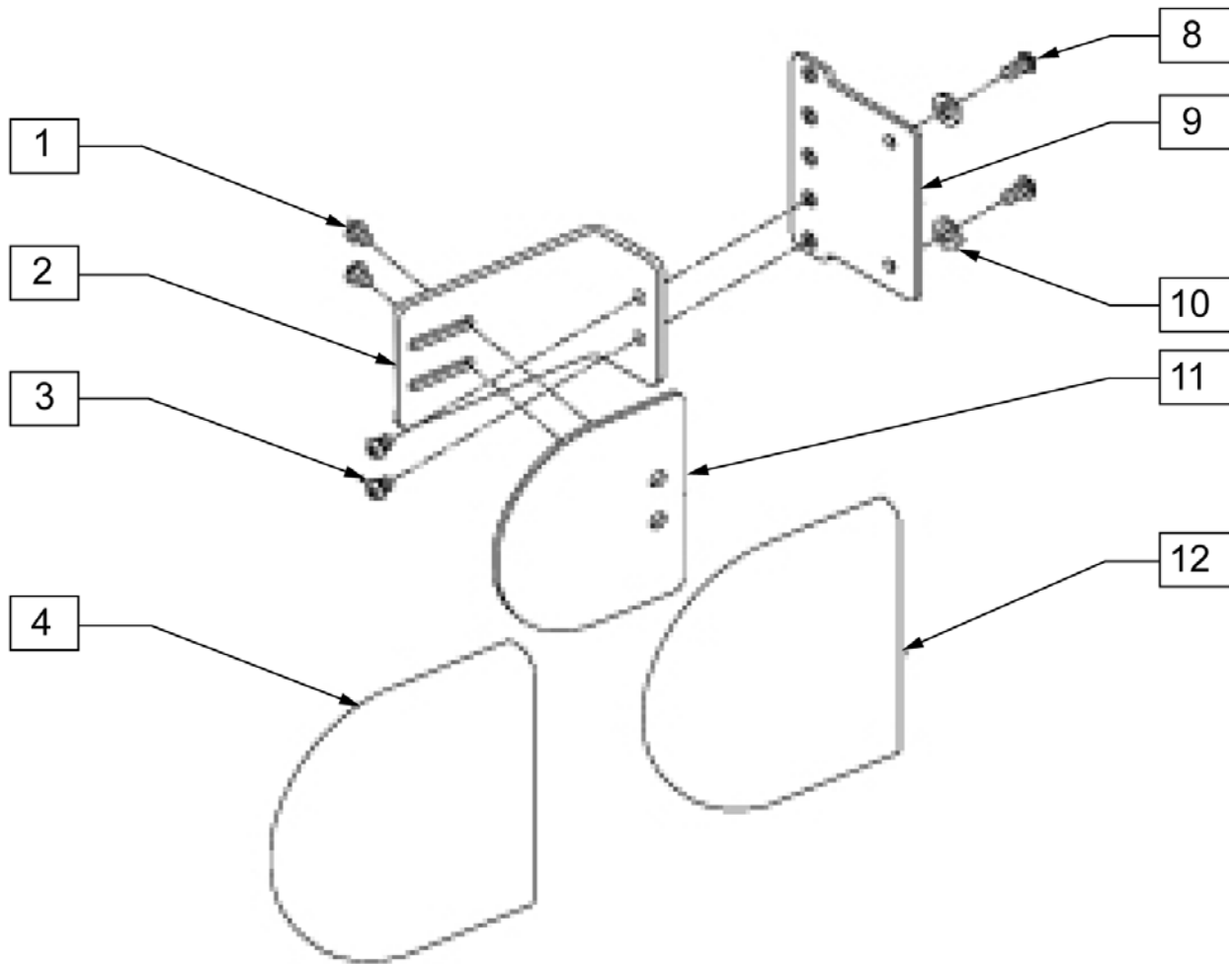


Pos.	Item Number	Description	Remarks
1	226018-QU	BHCS 10 - 32 x 3/8 W/ PATCH BLK	
2	200027-MJ	J2 PLUS FIXED LATERAL BRACKET	
3	M-200029	BHCS .250 - 20 X .38 W/ PATCH BLK	
5	M-200037	BHCS 10 - 32 x .63 W/ PATCH BLK	
6	200028-MJ	J2 PLUS LATERAL BRACKET PLATE	
7	M-200036	WASHER, .25 ANSI SERIES N PLAIN BLK	
4, 8, 9	476230	LATERAL THORACIC SUPPORT LARGE W/O HARDWARE	
1-9	200114-MJ	J2 PLUS LATERAL SUPPORT ASSM	RIGHT
1-9	200115-MJ	J2 PLUS LATERAL SUPPORT ASSM	LEFT
1-8	200116-MJ	J2 PLUS LATERAL SUPPORT ASSM	PAIR

NOTE: SEE JAY J2 & JAY J2 DEEP CONTOUR MOUNTING HARDWARE GUIDE FOR REPLACEMENT PART NUMBERS FOR ALL ATTACHMENT HARDWARE.

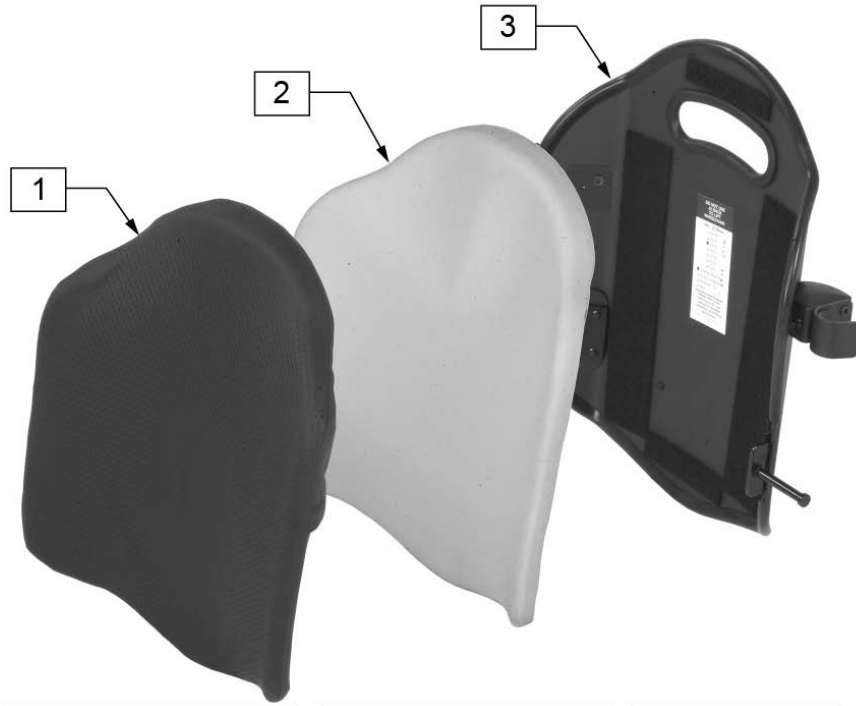

J2 Back Shell Measuring Guide

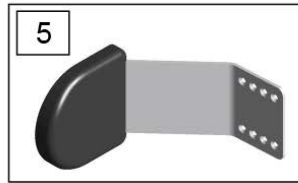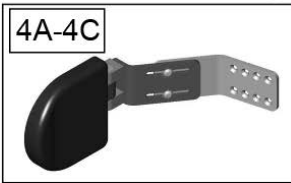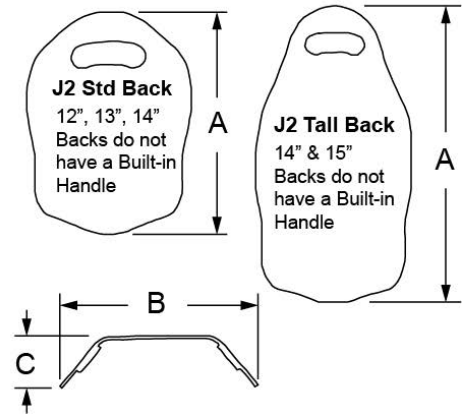

Part #	A	B	C
S2512K	12.38"	8.38"	2.18"
S2513K	12.80"	9.37"	2.24"
S2514K	14.30"	10.58"	2.70"
S2514TK	20.75"	10.62"	2.70"
S2515K	16.30"	11.58"	2.75"
S2515TK	22.75"	11.62"	2.75"
S2516K	16.30"	12.58"	2.85"
S2516TK	22.75"	12.62"	2.85"
S2517K	16.30"	13.58"	2.93"
S2517TK	22.75"	13.62"	2.93"
S2518K	16.30"	14.58"	3.00"
S2518TK	22.75"	14.62"	3.00"
S2519K	16.30"	15.58"	3.12"
S2519TK	22.75"	15.62"	3.12"
S2520K	16.30"	16.58"	3.23"
S2520TK	22.75"	16.62"	3.23"

Pos.	Item Number	Description	Remarks
1	C2512	COVER J2 BACK 12 X 12.5	
1	C2513	COVER J2 BACK 13 X 13.2	
1	C2514	COVER J2 BACK 14 X 14.3	
1	C2515	COVER J2 BACK 15 X 15.3	
1	C2516	COVER J2 BACK 16 X 16.3	
1	C2517	COVER J2 BACK 17 X 16.3	
1	C2518	COVER J2 BACK 18 X 16.3	
1	C2519	COVER J2 BACK 19 X 16.8	
1	C2520	COVER J2 BACK 20 X 17.3	
1	C2514T	COVER J2 BACK 14 X 21	
1	C2515T	COVER J2 BACK 15 X 21	
1	C2516T	COVER J2 BACK 16 X 21.8	
1	C2517T	COVER J2 BACK 17 X 21.8	
1	C2518T	COVER J2 BACK 18 X 21.8	
1	C2519T	COVER J2 BACK 19 X 22.8	
1	C2520T	COVER J2 BACK 20 X 22.8	
2	B2512	BACK FOAM 12 X 12.5	
2	B2513	BACK FOAM 13 X 13.2	
2	B2514	BACK FOAM 14 X 14.3	

Pos.	Item Number	Description	Remarks
2	B2515	BACK FOAM 15 X 15.3	
2	B2516	BACK FOAM 16 X 16.3	
2	B2517	BACK FOAM 17 X 16.3	
2	B2518	BACK FOAM 18 X 16.3	
2	B2519	BACK FOAM 19 X 16.8	
2	B2520	BACK FOAM 20 X 17.3	
2	B2514T	BACK FOAM 14 X 21	
2	B2515T	BACK FOAM 15 X 21	
2	B2516T	BACK FOAM 16 X 21.8	
2	B2517T	BACK FOAM 17 X 21.8	
2	B2518T	BACK FOAM 18 X 21.8	
2	B2519T	BACK FOAM 19 X 22.8	
2	B2520T	BACK FOAM 20 X 22.8	
3	S2512K	J2 BACK SHELL W/ UNI MOUNT 12 X 12.5	
3	S2513K	J2 BACK SHELL W/ UNI MOUNT 13 X 13.2	
3	S2514K	J2 BACK SHELL W/ UNI MOUNT 14 X 14.3	
3	S2515K	J2 BACK SHELL W/ UNI MOUNT 15 X 15.3	
3	S2516K	J2 BACK SHELL W/ UNI MOUNT 16 X 16.3	
3	S2517K	J2 BACK SHELL W/ UNI MOUNT 17 X 16.3	
3	S2518K	J2 BACK SHELL W/ UNI MOUNT 118 X 16.3	
3	S2519K	J2 BACK SHELL W/ UNI MOUNT 19 X 16.8	
3	S2520K	J2 BACK SHELL W/ UNI MOUNT 20 X 17.3	
3	S2514TK	J2 BACK SHELL W/ UNI MOUNT 14 X 21	
3	S2515TK	J2 BACK SHELL W/ UNI MOUNT 15 X 21	
3	S2516TK	J2 BACK SHELL W/ UNI MOUNT 16 X 21.8	
3	S2517TK	J2 BACK SHELL W/ UNI MOUNT 17 X 21.8	
3	S2518TK	J2 BACK SHELL W/ UNI MOUNT 18 X 21.8	
3	S2519TK	J2 BACK SHELL W/ UNI MOUNT 19 X 22.8	
3	S2520TK	J2 BACK SHELL W/ UNI MOUNT 20 X 22.8	
4A	B2560SA	SWING AWAY BRACKET ONLY	
4B	B2560LSA	SWING AWAY BRACKET ONLY -LONG	
5	2560J	J2 BACK LATERAL SUPPORT 12/13 SMALL	
5	2560-MJ	J2 BACK LATERAL SUPPORT 4 X 4 SMALL	
5	2562J	J2 BACK LATERAL SUPPORT 12/13 MEDIUM	
5	2562	J2 BACK LATERAL SUPPORT 5 X 5 MEDIUM	
5	2562L	J2 BACK EXT LATERAL SUPPORT MEDIUM	
5	2564	J2 BACK LARGE LATERAL SUPPORT 6 X 6	
6A	P2560S	SMALL J2 LATERAL-PAD ONLY	PAIR - INCLUDES SMALL PADDLE, FOAM, COVER, AND HARDWARE
6A	B662	BASE FOR LATERAL SUPPORT SM	EACH - INCLUDES SMALL FOAM ONLY
6B	P2562M	MEDIUM J2 LATERAL-PAD ONLY	PAIR - INCLUDES MED PADDLE, FOAM, COVER, AND HARDWARE
6B	B672	BASE EXTENDED LATERAL MED	EACH - INCLUDES MED FOAM ONLY
6C	P2564L	LARGE J2 LATERAL-PAD ONLY	PAIR - INCLUDES LARGE PADDLE, FOAM, COVER, AND HARDWARE
6C	B682	BASE EXTENDED LATERAL LARGE	EACH - INCLUDES LARGE FOAM ONLY
4C,6A	2560JSA	J2 BACK LATERAL SUPPORT SA 12/13 SMALL	
4B,6A	2560LSA	SMALL J2 SA LATERAL LONG	
4A,6A	2560SA	J2 BACK LATERAL SUPPORT SA SMALL	
4C,6B	2562JSA	J2 BACK LATERAL SUPPORT SA 12/13 MEDIUM	
4A,6B	2562SA	MEDIUM SA LATERAL SUPPORT	
4B,6B	2562LSA	J2 BACK LATERAL LONG SA MEDIUM	
4A,6C	2564SA	J2 BACK LATERAL SA LARGE	
4B,6C	2564LSA	LARGE J2 SA LATERAL LONG	
	M-251	MNTING ASSEMB KIT J2 BK LAT.	Lateral hardware kit- 4 screws, washers & nuts

NOTE: SEE JAY J2 & JAY J2 DEEP CONTOUR MOUNTING HARDWARE GUIDE FOR REPLACEMENT PART NUMBERS FOR ALL ATTACHMENT HARDWARE.


NOTE: CARRYING HANDLE IS NOT AVAILABLE FOR BACK SHELL WIDTHS 12" THROUGH 15" WIDE.



Pos.	Item Number	Description	Remarks
1	S2314	J2 DC 14 BACK SHELL ASSM 14 X 17	14 X 17
1	S2315	J2 DC 14 BACK SHELL ASSM 15 X 18	15 X 18
1	S2316	J2 DC 14 BACK SHELL ASSM 16 X 19	16 X 19
1	S2317	J2 DC 14 BACK SHELL ASSM 17 X 19	17 X 19
1	S2318	J2 DC 14 BACK SHELL ASSM 18 X 19	18 X 19
1	S2319	J2 DC 14 BACK SHELL ASSM 19 X 20	19 X 20
1	S2320	J2 DC 14 BACK SHELL ASSM 20 X 20	20 X 20
2	B2314F	J2 DC BACK BASE 14 X 17	14 X 17
2	B2315F	J2 DC BACK BASE 15 X 18	15 X 18
2	B2316F	J2 DC BACK BASE 16 X 19	
2	B2317F	J2 DC BACK BASE 17 X 19	
2	B2318F	J2 DC BACK BASE 18 X 19	
2	B2319F	J2 DC BACK BASE 19 X 20	
2	B2320F	J2 DC BACK BASE 20 X 20	

Pos.	Item Number	Description	Remarks
3	C2314	J2 DC BACK COVER 14 X 17	
3	C2315	J2 DC BACK COVER 15 X 18	
3	C2316	J2 DC BACK COVER 16 X 19	
3	C2317	J2 DC BACK COVER 17 X 19	
3	C2318	J2 DC BACK COVER 18 X 19	
3	C2319	J2 DC BACK COVER 19 X 20	19 X 20
3	C2320	J2 DC BACK COVER 20 X 20	20 X 20
4	B2320W	J2 DC BACK VELCRO LUMBAR SUPPORT	
5	2370	J2 DC POST LATERAL LUMBAR SUPPORTS	PAIR
6	2360	J2 LATERAL SUPPORT ADJUSTABLE WEDGES	PAIR

NOTE: if adding Jay J2 Deep Contour Back to a chair that has an Angle Adjustable Back please order 880987 (item 13).

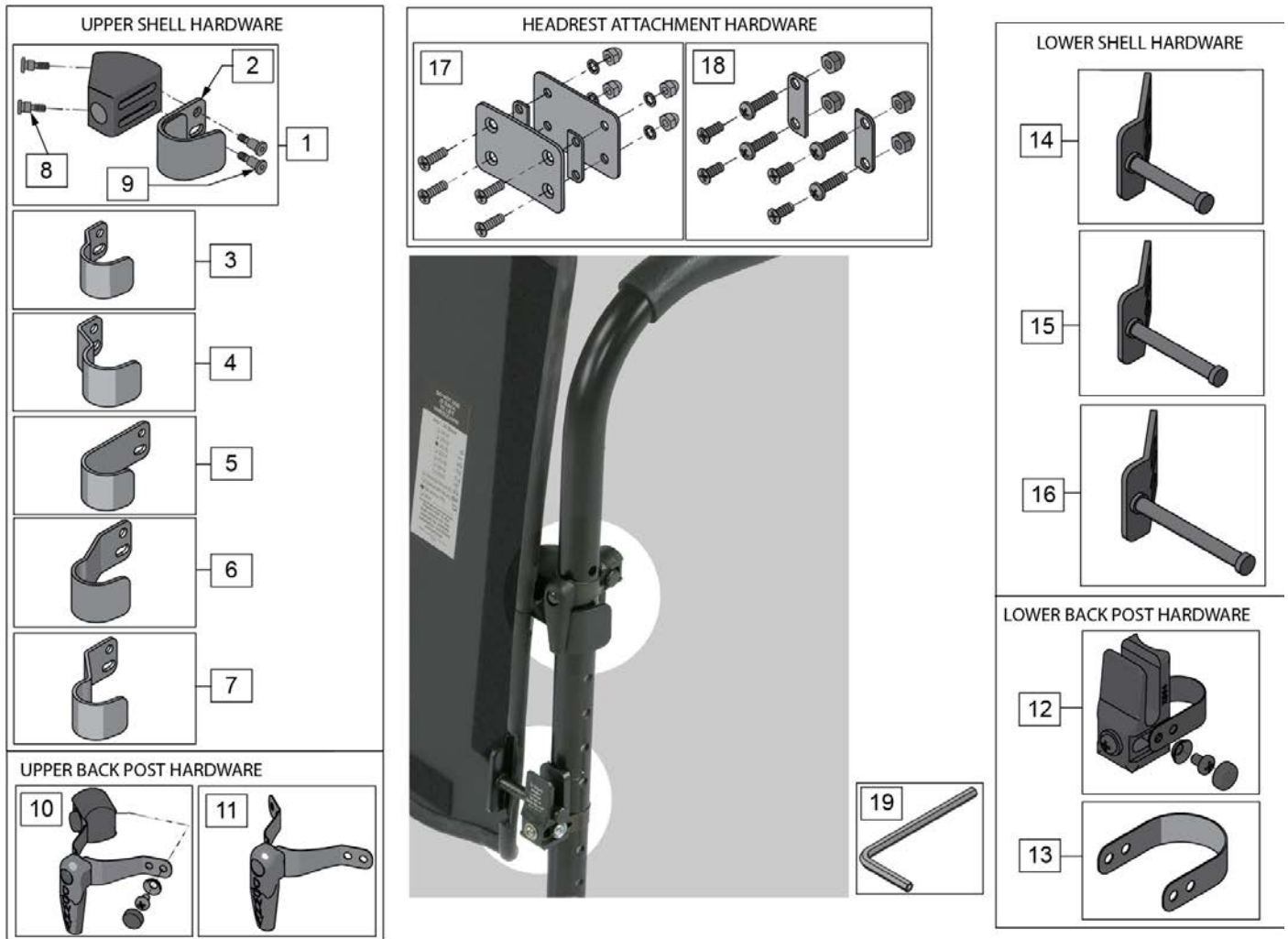

Pos.	Item Number	Description	Remarks
1	M-254	TOP ADJUSTERS	
2	M-253	TOP HOOK KIT (SHORT)	Ships Standard w/ J2 & J2 Deep Backs
3	M-2312	SHORT HOOKS W/ 1/2 OFFSET	
4	M-2301	SHORT HOOKS W/ 1 OFFSET	
5	M-256	TOP HOOK KIT (LONG)	
6	M-2322	LONG HOOKS W/ 1/2 OFFSET	
7	M-2311	LONG HOOKS W/ 1 OFFSET	
8	M-241	SHOULDER BOLT 1/4 X 1/4	
9	M-240	SHOULDER BOLT 1/4 X 10-32	
10	681C	TOP BRACKET ASSM - 'QUARTER-TURN'	fits 5/8" to 1.0" diameter tubing
11	2681C	TOP LONG 1/4 BRACKET ASSM	fits 1.0" to 1.25" diameter tubing
12	680C	BOTTOM UNIVERSAL POST BRACKETS	fits 5/8" to 1.0" diameter tubing
13	880997	LONG STRAP KIT	Used on 1 1/8" diameter Back Post
14	M-258	BOTTOM PINS STANDARD	Pin Length = 2"
15	M-2313	BOTTOM PINS 1/2 EXTENDED	Pin Length = 2.5"
16	M-2323	BOTTOM PINS 1 EXTENDED	Pin Length = 3"

Pos.	Item Number	Description	Remarks
17	M-500	HEADREST ADAPTOR KIT	
18	M-509	HEADREST ADAPTOR KIT-TALL BACK	
19	M-228	ALLEN WRENCH	
2,10,12	M-250	UNIV.RETRO-FIT KIT J2 BK	



SHORT HOOKS OFTEN USED WITH ANGLED BACKPOST,
LONG HOOKS MORE COMMON WITH STRAIGHT BACKPOST.


J2	J2 TALL	J2 DC	WIDTH	NARROWEST SETTING	WIDEST SETTING	1/2" OFFSET HOOK	1" OFFSET HOOK
2512K	2512TK	N/A	12" WIDE	11	12.5	13.5	14.5
2513K	2513TK	N/A	13" WIDE	12	13.5	14.5	15.5
2514K	2514TK	2314	14" WIDE	13	14.5	15.5	16.5
2515K	2515TK	2315	15" WIDE	14	15.5	16.5	17.5
2516K	2516TK	2316	16" WIDE	15	16.5	17.5	18.5
2517K	2517TK	2317	17" WIDE	16	17.5	18.5	19.5
2518K	2518TK	2318	18" WIDE	17	18.5	19.5	20.5
2519K	2519TK	2319	19" WIDE	18	19.5	20.5	N/A
2520K	2520TK	2320	20" WIDE	19	20.5	N/A	N/A

Pos.	Description
-	SEE CHART ABOVE


NOTE: Kit includes pair of upper and lower clamps with spacers to fit 1 1/8", 1" and 7/8" Back Tubes.

NOTE: Please refer to "Jay J2 & Deep Contour Mounting Hardware Guide" page for shell mounting bracket hardware.


Note: Clamp Style hardware is not available with the Xenon because the Back Post/Push Handle tubing is less than 7/8".

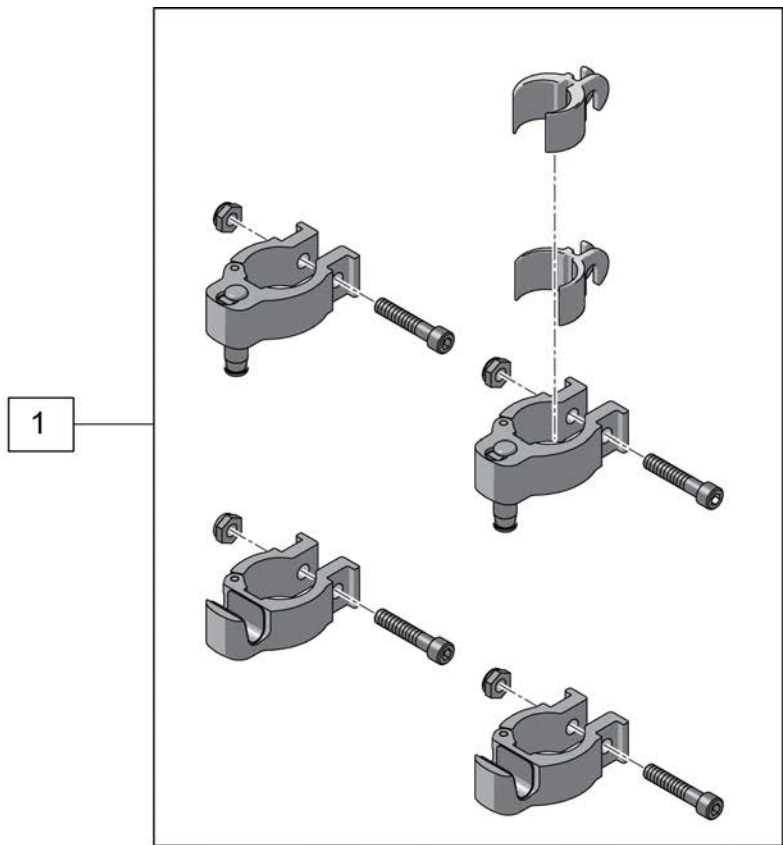

Pos.	Item Number	Description
1	XM250C	KIT CLAMP UNIV RETRO-FIT

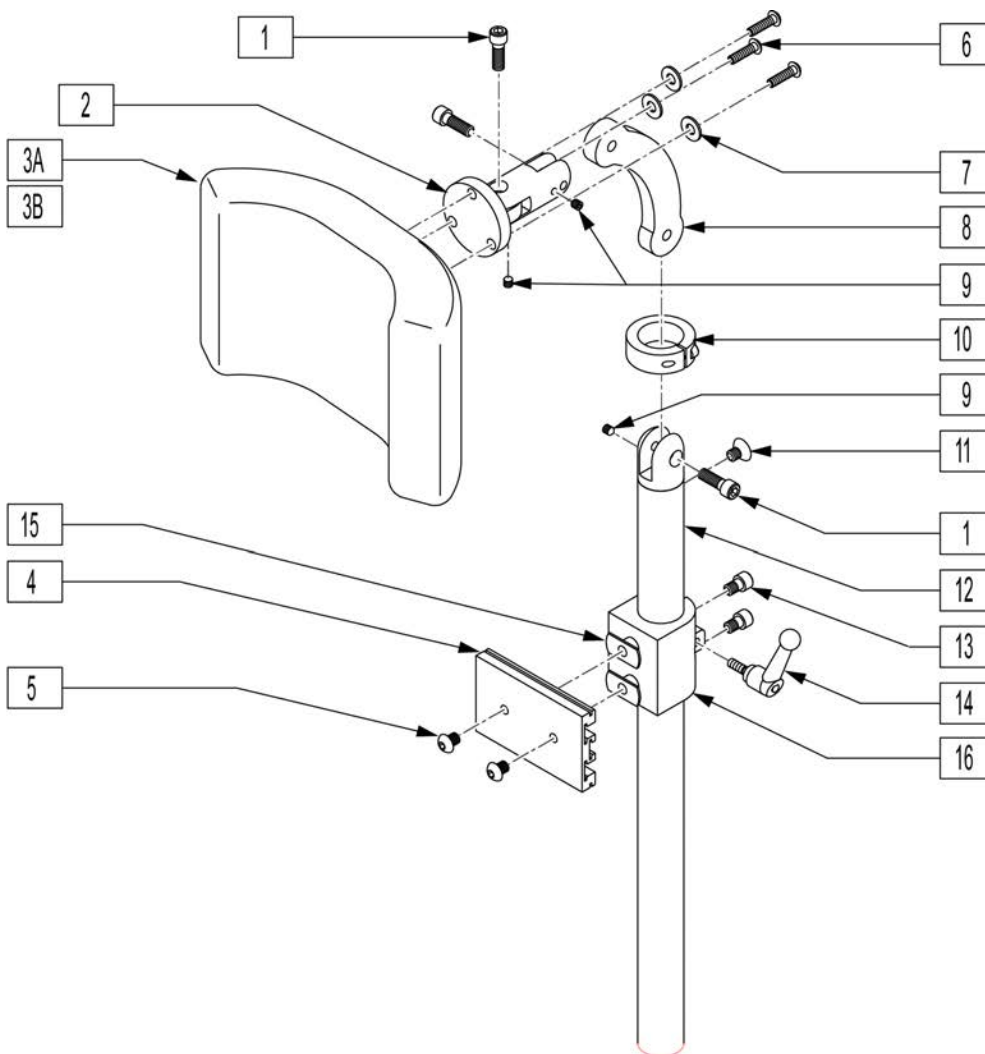


Pos.	Item Number	Description	Remarks
1	221115	1/4-20 x 3/4" SOC HD C/S BLK	
3A	750696	HEADREST W/ZIPPER MEDIUM CG	8x4.5 pad
3B	750697	HEADREST W/ZIPPER LARGE CG	9x4.5 pad
4	505932	HEADREST MOUNTING TRACK	
5	476304	1/4-28 x .31 BHCS W/PATCH	
6	221010	10-32 x .75 SHCS BLK W/PL	
7	261204	#10 AN WASHER .063	
8	331077	HEADREST LINK 6 INCH	
9	341045	#10-24 CONE POINT SET SCREW	
10	505933	HEADREST COLLAR	
11	226072	1/4-20 x 1/2 FH C/S BLK	
13	226095	1/4-20 x 3/8" SOC C/S BLK ZN	
14	341046	HEADREST CLAMPING HANDLE SCREW	
15	340033	1/4-20 WELD NUTS SLAB BASE	
4,5,13-16	888101	CLAMP HEADREST RIGID BACK	
1,2,4,6-16	881003	HEADREST MOUNTING BRACKET ASSM	
1-3A,4-16	881907	HEADREST ASSEMBLY MEDIUM	8x4.5 pad
1-3B,4-16	881908	HEADREST ASSEMBLY LARGE	9x4.5 pad

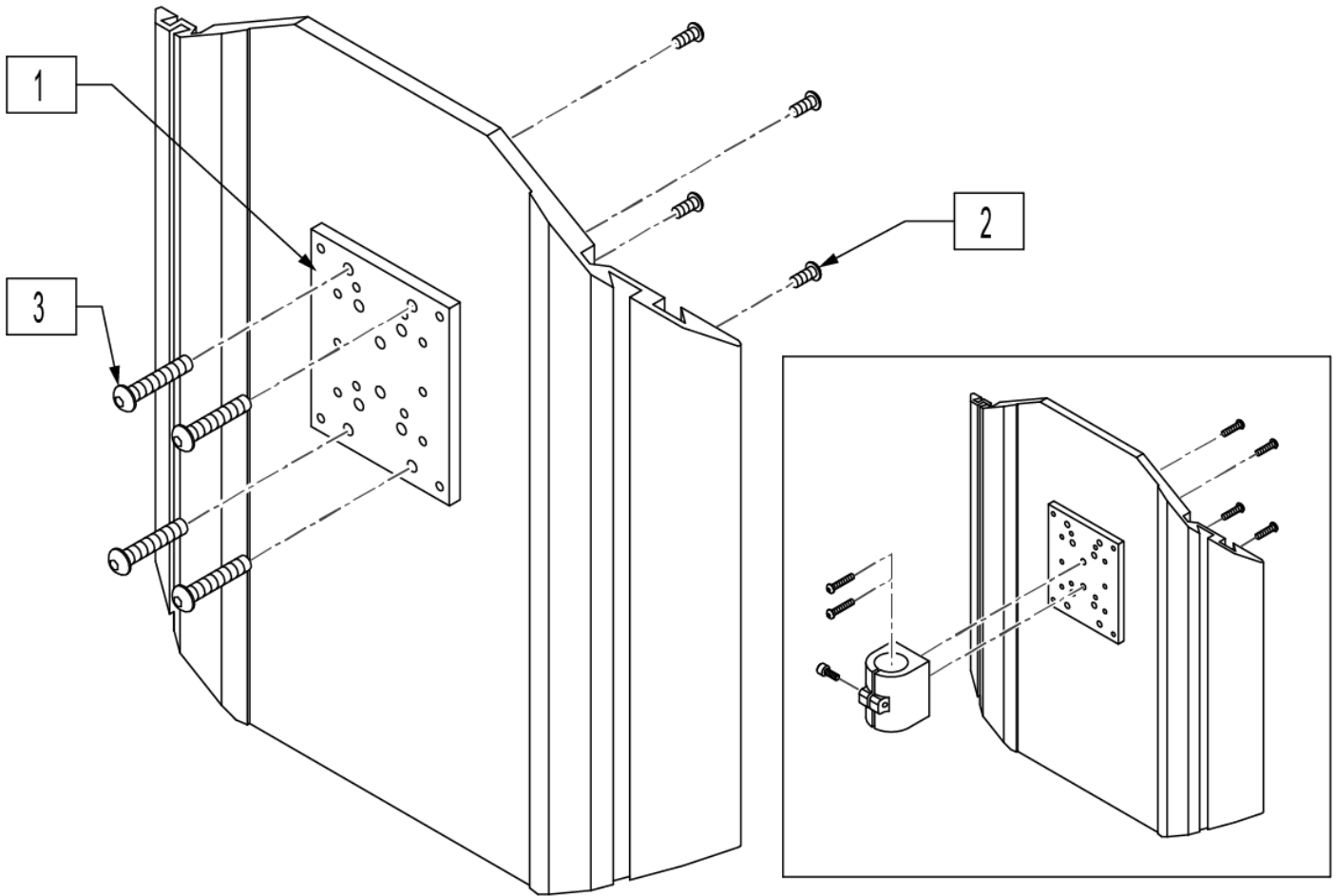

Pos.	Item Number	Description
1	XM2025	HDRST MTG PLT JAY FIT SYSTEM
2	476304	1/4-28 x .31 BHCS W/PATCH
3	226018-QU	10-32 X 3/8 BH C/S W/PATCH BLK
1-3	476314	HEADREST NMT KIT W/HDWR POWER

NOTE: Parts only, for complete Backrest see J3 Back order form.

Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

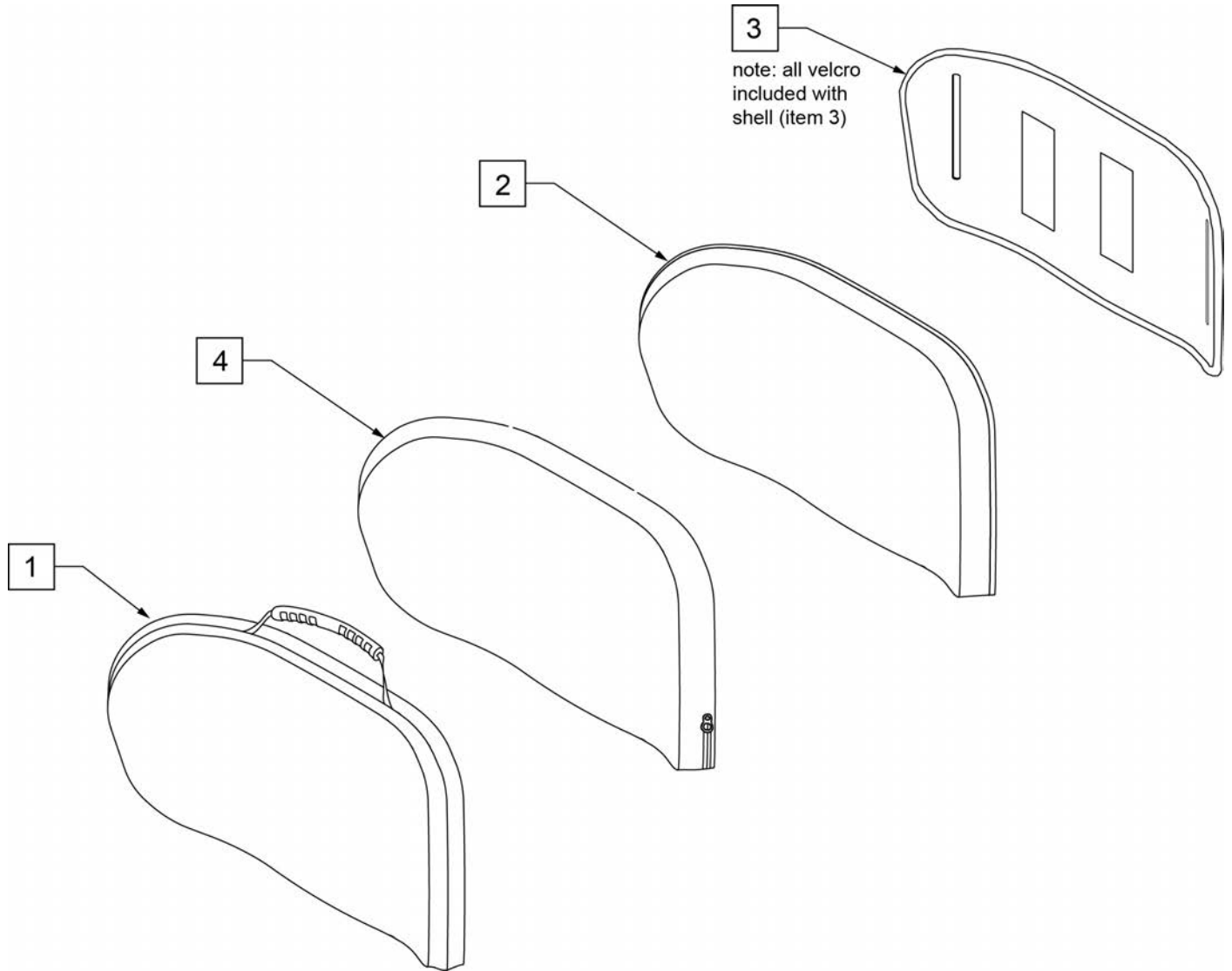


Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	CJ3PALTM12	J3 CVR POS LT 12W MED		1	CJ3PALTT18	J3 CVR, POS LT 18W TALL	
1	CJ3PALTT12	J3 CVR POS LT 12W TAL		1	CJ3PALTT20	J3 CVR, POS LT 20W TALL	
1	CJ3PALTS14	J3 CVR, POS LT 14W STD		2	BJ3PALTM12	J3 FOAM POS LT 12W MED	
1	CJ3PALTM14	J3 CVR, POS LT 14W MED		2	BJ3PALTT12	J3 FOAM POS LT 12W TAL	
1	CJ3PALTT14	J3 CVR, POS LT 14W TALL		2	BJ3PALTS14	J3 FOAM, POS LT 14W STD	
1	CJ3PALTS16	J3 CVR, POS LT 16W STD		2	BJ3PALTM14	J3 FOAM, POS LT 14W MED	
1	CJ3PALTM16	J3 CVR, POS LT 16W MED		2	BJ3PALTT14	J3 FOAM, POS LT 14W TALL	
1	CJ3PALTT16	J3 CVR, POS LT 16W TALL		2	BJ3PALTS16	J3 FOAM, POS LT 16W STD	
1	CJ3PALTS18	J3 CVR, POS LT 18W STD		2	BJ3PALTM16	J3 FOAM, POS LT 16W MED	
1	CJ3PALTM18	J3 CVR, POS LT 18W MED		2	BJ3PALTT16	J3 FOAM, POS LT 16W TALL	

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
2	BJ3PALTS18	J3 FOAM, POS LT 18W STD		3	SJ3PALTT18	J3 SHELL, POS LT 18W TALL	
2	BJ3PALTM18	J3 FOAM, POS LT 18W MED		3	SJ3PALTT20	J3 SHELL, POS LT 20W TALL	
2	BJ3PALTT18	J3 FOAM, POS LT 18W TALL		4	ICJ3PALTM12	J3 BK POS LT 12W MED SH	inner cover
2	BJ3PALTT20	J3 FOAM, POS LT 20W TALL		4	ICJ3PALTM14	J3 BK POS LT 14W MED SH	inner cover
3	SJ3PALTM12	J3 SHELL POS LT 12W MED		4	ICJ3PALTM16	J3 BK POS LT 16W MED SH	inner cover
3	SJ3PALTT12	J3 SHELL POS LT 12W TAL		4	ICJ3PALTM18	J3 BK POS LT 18W MED SH	inner cover
3	SJ3PALTS14	J3 SHELL, POS LT 14W STD		4	ICJ3PALTS14	J3 BK POS LT 14W STD SH	inner cover
3	SJ3PALTM14	J3 SHELL, POS LT 14W MED		4	ICJ3PALTS16	J3 BK POS LT 16W STD SH	inner cover
3	SJ3PALTT14	J3 SHELL, POS LT 14W TALL		4	ICJ3PALTS18	J3 BK POS LT 18W STD SH	inner cover
3	SJ3PALTS16	J3 SHELL, POS LT 16W STD		4	ICJ3PALTT12	J3 BK POS LT 12W TALL SH	inner cover
3	SJ3PALTM16	J3 SHELL, POS LT 16W MED		4	ICJ3PALTT14	J3 BK POS LT 14W TALL SH	inner cover
3	SJ3PALTT16	J3 SHELL, POS LT 16W TALL		4	ICJ3PALTT16	J3 BK POS LT 16W TALL SH	inner cover
3	SJ3PALTS18	J3 SHELL, POS LT 18W STD		4	ICJ3PALTT18	J3 BK POS LT 18W TALL SH	inner cover
3	SJ3PALTM18	J3 SHELL, POS LT 18W MED		4	ICJ3PALTT20	J3 BK POS LT 20W TALL SH	inner cover

NOTE: Parts only, for complete Backrest see J3 Back order form.


Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

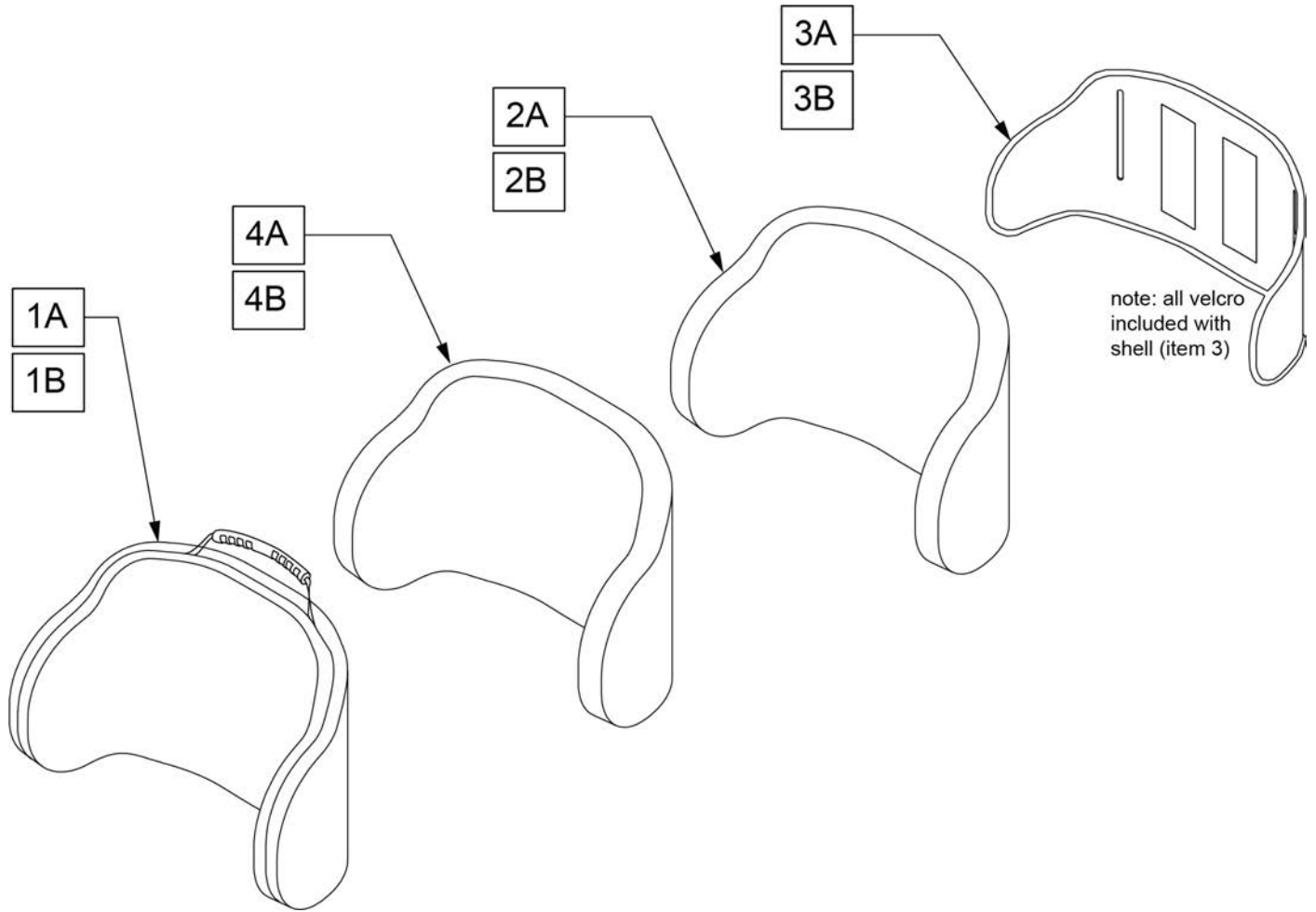

Pos.	Item Number	Description	Pos.	Item Number	Description
1A	CJ3PDLTM14	J3 BK CVR PD LT 14W MED	3A	SJ3PDLTM14	J3 BK SHL PD LT 14W MED
1A	CJ3PDLTM16	J3 BK CVR PD LT 16W MED	3A	SJ3PDLTM16	J3 BK SHL PD LT 16W MED
1A	CJ3PDLTM18	J3 BK CVR PD LT 18W MED	3A	SJ3PDLTM18	J3 BK SHL PD LT 18W MED
1B	CJ3PDLTT14	J3 BK CVR PD LT 14W TALL	3B	SJ3PDLTT14	J3 BK SHL PD LT 14W TALL
1B	CJ3PDLTT16	J3 BK CVR PD LT 16W TALL	3B	SJ3PDLTT16	J3 BK SHL PD LT 16W TALL
1B	CJ3PDLTT18	J3 BK CVR PD LT 18W TALL	3B	SJ3PDLTT18	J3 BK SHL PD LT 18W TALL
1B	CJ3PDLTT20	J3 BK CVR PD LT 20W TALL	3B	SJ3PDLTT20	J3 BK SHL PD LT 20W TALL
2A	BJ3PDLTM14	J3 BK FOAM PD LT 14W MED	4A	ICJ3PDLTM14	J3 BK INNER CVR PD LT 14W MED
2A	BJ3PDLTM16	J3 BK FOAM PD LT 16W MED	4A	ICJ3PDLTM16	J3 BK INNER CVR PD LT 16W MED
2A	BJ3PDLTM18	J3 BK FOAM PD LT 18W MED	4A	ICJ3PDLTM18	J3 BK INNER CVR PD LT 18W MED
2B	BJ3PDLTT14	J3 BK FOAM PD LT 14W TALL	4B	ICJ3PDLTT14	J3 BK INNER CVR PD LT 14W TALL
2B	BJ3PDLTT16	J3 BK FOAM PD LT 16W TALL	4B	ICJ3PDLTT16	J3 BK INNER CVR PD LT 16W TALL
2B	BJ3PDLTT18	J3 BK FOAM PD LT 18W TALL	4B	ICJ3PDLTT18	J3 BK INNER CVR PD LT 18W TALL
2B	BJ3PDLTT20	J3 BK FOAM PD LT 20W TALL	4B	ICJ3PDLTT20	J3 BK INNER CVR PD LT 20W TALL


NOTE: Parts only, for complete Backrest see J3 Back order form.

Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

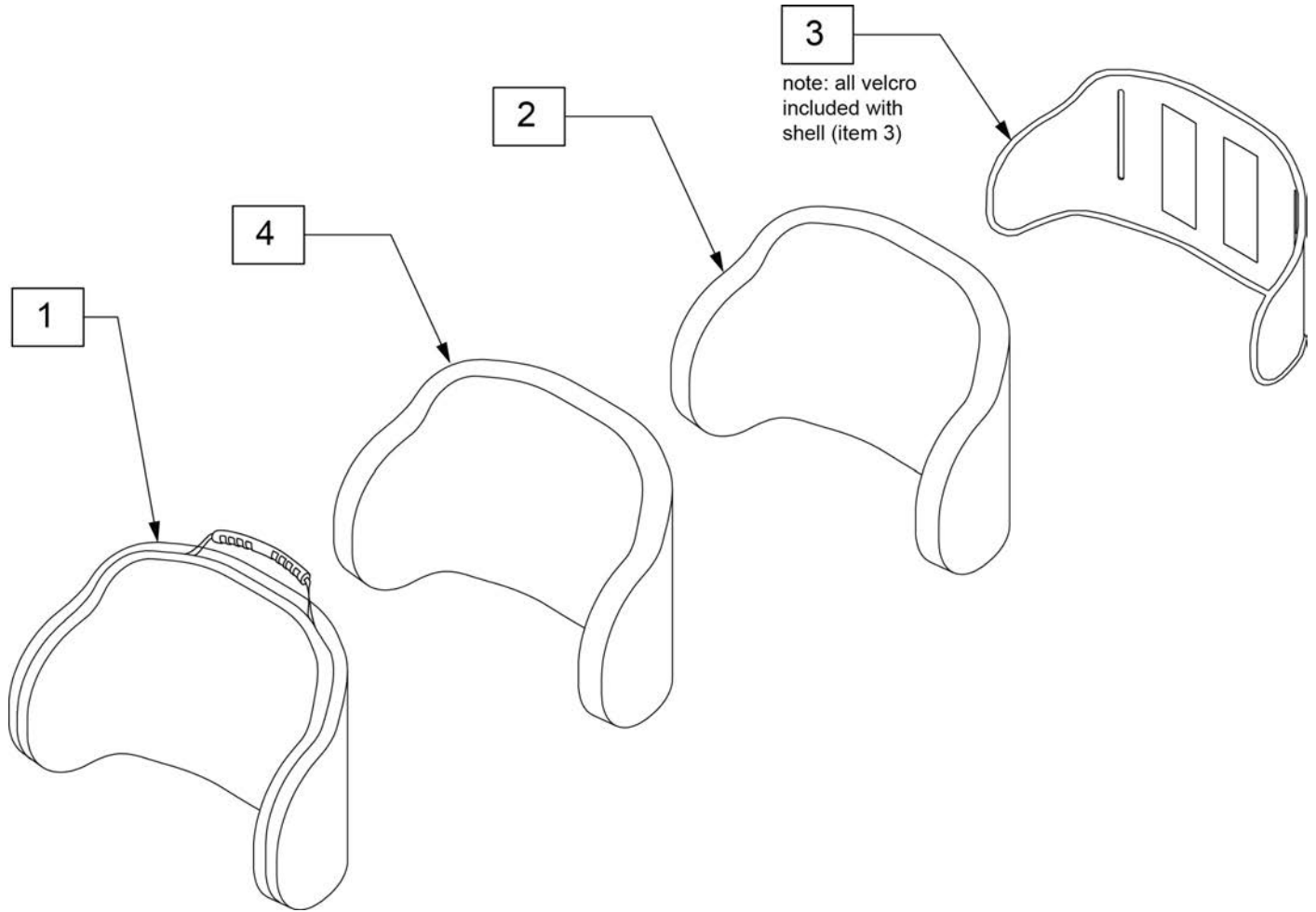

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	CJ3PDCLTM14	J3 CVR PDC LT 14W MED		3	SJ3PDCLTM14	J3 SHELL PDC LT 14W MED	
1	CJ3PDCLTT14	J3 CVR PDC LT 14W TAL		3	SJ3PDCLTT14	J3 SHELL PDC LT 14W TAL	
1	CJ3PDCLTM16	J3 CVR PDC LT 16W MED		3	SJ3PDCLTM16	J3 SHELL PDC LT 16W MED	
1	CJ3PDCLTT16	J3 CVR PDC LT 16W TAL		3	SJ3PDCLTT16	J3 SHELL PDC LT 16W TAL	
1	CJ3PDCLTM18	J3 CVR PDC LT 18W MED		3	SJ3PDCLTM18	J3 SHELL PDC LT 18W MED	
1	CJ3PDCLTT18	J3 CVR PDC LT 18W TAL		3	SJ3PDCLTT18	J3 SHELL PDC LT 18W TAL	
1	CJ3PDCLTT20	J3 CVR PDC LT 20W TAL		3	SJ3PDCLTT20	J3 SHELL PDC LT 20W TAL	
2	BJ3PDCLTM14	J3 FOAM PDC LT 14W MED		4	ICJ3PDCLTM14	J3 BK PDC LT 14W MED SH	inner cover
2	BJ3PDCLTT14	J3 FOAM PDC LT 14W TAL		4	ICJ3PDCLTM16	J3 BK PDC LT 16W MED SH	inner cover
2	BJ3PDCLTM16	J3 FOAM PDC LT 16W MED		4	ICJ3PDCLTM18	J3 BK PDC LT 18W MED SH	inner cover
2	BJ3PDCLTT16	J3 FOAM PDC LT 16W TAL		4	ICJ3PDCLTT14	J3 BK PDC LT 14W TALL SH	inner cover
2	BJ3PDCLTM18	J3 FOAM PDC LT 18W MED		4	ICJ3PDCLTT16	J3 BK PDC LT 16W TALL SH	inner cover
2	BJ3PDCLTT18	J3 FOAM PDC LT 18W TAL		4	ICJ3PDCLTT18	J3 BK PDC LT 18W TALL SH	inner cover
2	BJ3PDCLTT20	J3 FOAM PDC LT 20W TAL		4	ICJ3PDCLTT20	J3 BK PDC LT 20W TALL SH	inner cover

NOTE: Parts only, for complete Backrest see J3 Back order form.

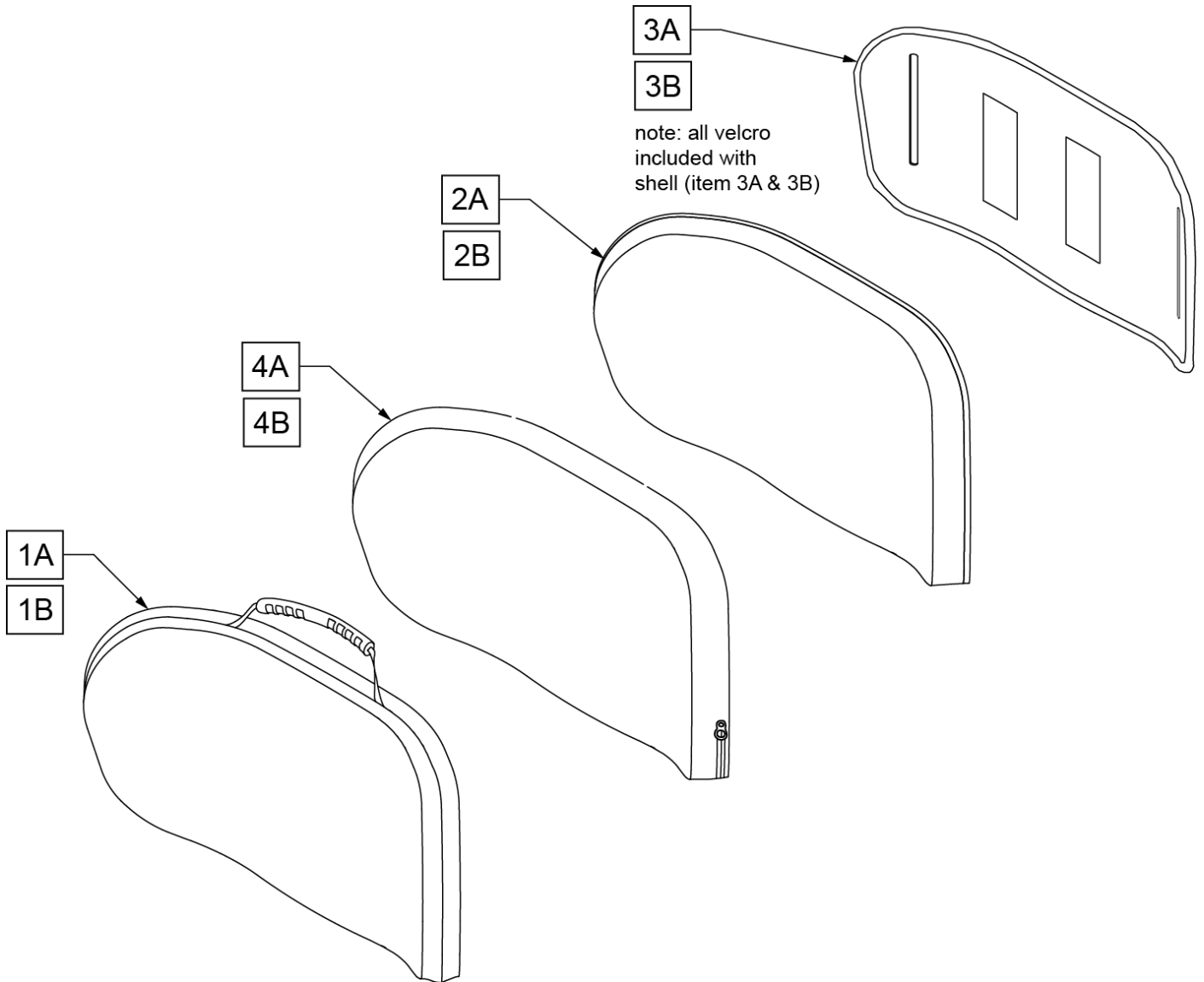


Pos.	Item Number	Description	Pos.	Item Number	Description
1A	CJ3CFLTS14	COVER J3 CF BACK PA LT SHT 14W	2B	BJ3PALTT18	J3 FOAM, POS LT 18W TALL
1A	CJ3CFLTS16	COVER J3 CF BACK PA LT SHT 16W	2B	BJ3PALTT20	J3 FOAM, POS LT 20W TALL
1A	CJ3CFLTS18	COVER J3 CF BACK PA LT SHT 18W	3A	SJ3CFLTS14	SHELL J3 CF BACK PA LT SHT 14W
1B	CJ3CFLTT14	COVER J3 CF BACK PA LT TALL 14W	3A	SJ3CFLTS16	SHELL J3 CF BACK PA LT SHT 16W
1B	CJ3CFLTT16	COVER J3 CF BACK PA LT TALL 16W	3A	SJ3CFLTS18	SHELL J3 CF BACK PA LT SHT 18W
1B	CJ3CFLTT18	COVER J3 CF BACK PA LT TALL 18W	3B	SJ3CFLTT14	SHELL J3 CF BACK PA LT TALL 14W
1B	CJ3CFLTT20	COVER J3 CF BACK PA LT TALL 20W	3B	SJ3CFLTT16	SHELL J3 CF BACK PA LT TALL 16W
2A	BJ3PALTS14	J3 FOAM, POS LT 14W STD	3B	SJ3CFLTT18	SHELL J3 CF BACK PA LT TALL 18W
2A	BJ3PALTS16	J3 FOAM, POS LT 16W STD	3B	SJ3CFLTT20	SHELL J3 CF BACK PA LT TALL 20W
2A	BJ3PALTS18	J3 FOAM, POS LT 18W STD	4A	ICJ3PALTS14	J3 BK POS LT 14W STD SH
2B	BJ3PALTT14	J3 FOAM, POS LT 14W TALL	4A	ICJ3PALTT16	J3 BK POS LT 16W TALL SH
2B	BJ3PALTT16	J3 FOAM, POS LT 16W TALL	4A	ICJ3PALTS18	J3 BK POS LT 18W STD SH

Pos.	Item Number	Description	Pos.	Item Number	Description
4B	ICJ3PALTT14	J3 BK POS LT 14W TALL SH	4B	ICJ3PALTT18	J3 BK POS LT 18W TALL SH
4B	ICJ3PALTS16	J3 BK POS LT 16W STD SH	4B	ICJ3PALTT20	J3 BK POS LT 20W TALL SH

NOTE: Parts only, for complete Backrest see J3 Back order form.

Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

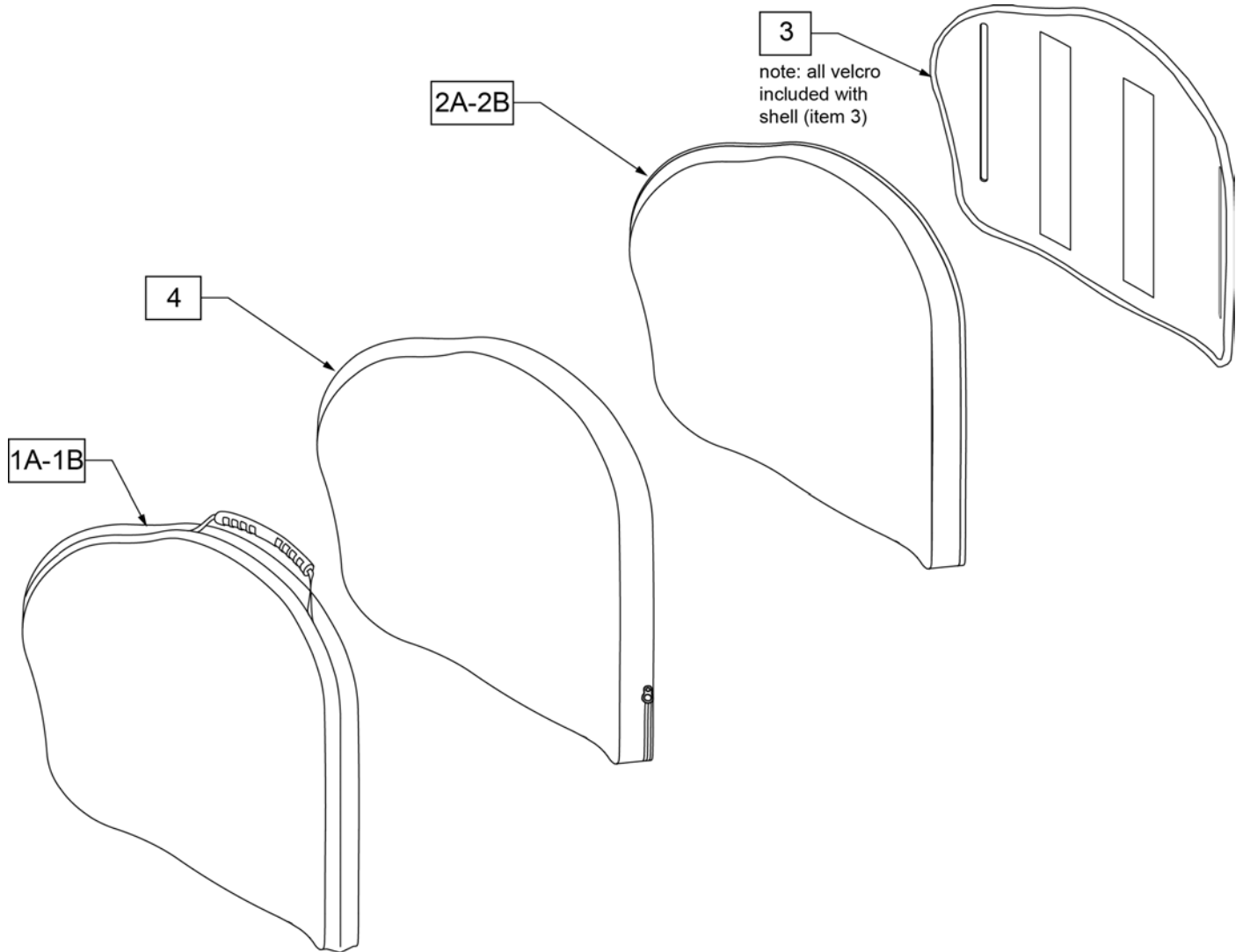


Pos.	Item Number	Description	Remarks
1A	CJ3PAMTS12	J3 CVR POS MT 12W STD	
1A	CJ3PAMTM12	J3 CVR POS MT 12W MED	
1A	CJ3PAMTT12	J3 CVR POS MT 12W TAL	
1A	CJ3PAMTS14	J3 CVR, POS MT 14W STD	
1A	CJ3PAMTM14	J3 CVR, POS MT 14W MED	
1A	CJ3PAMTT14	J3 CVR, POS MT 14W TALL	
1A	CJ3PAMTS16	J3 CVR, POS MT 16W STD	
1A	CJ3PAMTM16	J3 CVR, POS MT 16W MED	
1A	CJ3PAMTT16	J3 CVR, POS MT 16W TALL	
1A	CJ3PAMTS18	J3 CVR, POS MT 18W STD	

Pos.	Item Number	Description	Remarks
1A	CJ3PAMTM18	J3 CVR, POS MT 18W MED	
1A	CJ3PAMTT18	J3 CVR, POS MT 18W TALL	
1A	CJ3PAMTS20	J3 CVR, POS MT 20W STD	
1A	CJ3PAMTM20	J3 CVR, POS MT 20W MED	
1A	CJ3PAMTT20	J3 CVR, POS MT 20W TALL	
1B	CJ3PAMTS16CU	CVR J3 BK POS MT 16W STD CU	use w/ Cradle-U
1B	CJ3PAMTM16CU	CVR J3 BK POS MT 16W MED CU	use w/ Cradle-U
1B	CJ3PAMTT16CU	CVR J3 BK POS MT 16W TALL CU	use w/ Cradle-U
1B	CJ3PAMTS18CU	CVR J3 BK POS MT 18W STD CU	use w/ Cradle-U
1B	CJ3PAMTM18CU	CVR J3 BK POS MT 18W MED CU	use w/ Cradle-U
1B	CJ3PAMTT18CU	CVR J3 BK POS MT 18W TALL CU	use w/ Cradle-U
1B	CJ3PAMTS20CU	CVR J3 BK POS MT 20W STD CU	use w/ Cradle-U
1B	CJ3PAMTM20CU	CVR J3 BK POS MT 20W MED CU	use w/ Cradle-U
1B	CJ3PAMTT20CU	CVR J3 BK POS MT 20W TALL CU	use w/ Cradle-U
2A	BJ3PAMTS12	J3 FOAM POS MT 12W STD	
2A	BJ3PAMTM12	J3 FOAM POS MT 12W MED	
2A	BJ3PAMTT12	J3 FOAM POS MT 12W TAL	
2A	BJ3PAMTS14	J3 FOAM, POS MT 14W STD	
2A	BJ3PAMTM14	J3 FOAM, POS MT 14W MED	
2A	BJ3PAMTT14	J3 FOAM, POS MT 14W TALL	
2A	BJ3PAMTS16	J3 FOAM, POS MT 16W STD	
2A	BJ3PAMTM16	J3 FOAM, POS MT 16W MED	
2A	BJ3PAMTT16	J3 FOAM, POS MT 16W TALL	
2A	BJ3PAMTS18	J3 FOAM, POS MT 18W STD	
2A	BJ3PAMTM18	J3 FOAM, POS MT 18W MED	
2A	BJ3PAMTT18	J3 FOAM, POS MT 18W TALL	
2A	BJ3PAMTT20	J3 FOAM, POS MT 20W TALL	
2A	BJ3PAMTS20	J3 FOAM, POS MT 20W STD	
2B	BJ3PAMTS16CU	BASE J3 BK POS MT 16W STD CU	use w/ Cradle-U
2B	BJ3PAMTM16CU	BASE J3 BK POS MT 16W MED CU	use w/ Cradle-U
2B	BJ3PAMTT16CU	BASE J3 BK POS MT 16W TALL CU	use w/ Cradle-U
2B	BJ3PAMTS18CU	BASE J3 BK POS MT 18W STD CU	use w/ Cradle-U
2B	BJ3PAMTM18CU	BASE J3 BK POS MT 18W MED CU	use w/ Cradle-U
2B	BJ3PAMTT18CU	BASE J3 BK POS MT 18W TALL CU	use w/ Cradle-U
2B	BJ3PAMTS20CU	BASE J3 BK POS MT 20W STD CU	use w/ Cradle-U
2B	BJ3PAMTM20CU	BASE J3 BK POS MT 20W MED CU	use w/ Cradle-U
3	SJ3PAMTS12	J3 SHELL POS MT 12W STD	
3	SJ3PAMTM12	J3 SHELL POS MT 12W MED	
3	SJ3PAMTT12	J3 SHELL POS MT 12W TAL	
3	SJ3PAMTS14	J3 SHELL, POS MT 14W STD	
3	SJ3PAMTM14	J3 SHELL, POS MT 14W MED	
3	SJ3PAMTT14	J3 SHELL, POS MT 14W TALL	
3	SJ3PAMTS16	J3 SHELL, POS MT 16W STD	
3	SJ3PAMTM16	J3 SHELL, POS MT 16W MED	
3	SJ3PAMTT16	J3 SHELL, POS MT 16W TALL	
3	SJ3PAMTS18	J3 SHELL, POS MT 18W STD	
3	SJ3PAMTM18	J3 SHELL, POS MT 18W MED	
3	SJ3PAMTT18	J3 SHELL, POS MT 18W TALL	
3	SJ3PAMTS20	J3 SHELL, POS MT 20W STD	
3	SJ3PAMTM20	J3 SHELL, POS MT 20W MED	
3	SJ3PAMTT20	J3 SHELL, POS MT 20W TALL	
4	ICJ3PAMTS12	J3 BK POS MT 12W STD SH	inner cover
4	ICJ3PAMTM12	J3 BK POS MT 12W MED SH	inner cover
4	ICJ3PAMTT12	J3 BK POS MT 12W TALL SH	inner cover
4	ICJ3PAMTS14	J3 BK POS MT 14W STD SH	inner cover
4	ICJ3PAMTM14	J3 BK POS MT 14W MED SH	inner cover
4	ICJ3PAMTT14	J3 BK POS MT 14W TALL SH	inner cover

Pos.	Item Number	Description	Remarks
4	ICJ3PAMTS16	J3 BK POS MT 16W STD SH	inner cover
4	ICJ3PAMTM16	J3 BK POS MT 16W MED SH	inner cover
4	ICJ3PAMTT16	J3 BK POS MT 16W TALL SH	inner cover
4	ICJ3PAMTS18	J3 BK POS MT 18W STD SH	inner cover
4	ICJ3PAMTM18	J3 BK POS MT 18W MED SH	inner cover
4	ICJ3PAMTT18	J3 BK POS MT 18W TALL SH	inner cover
4	ICJ3PAMTM20	J3 BK POS MT 20W MED SH	inner cover
4	ICJ3PAMTT20	J3 BK POS MT 20W TALL SH	inner cover
4	ICJ3PAMTS20	J3 BK POS MT 20W STD SH	inner cover

NOTE: Parts only, for complete Backrest see J3 Back order form.

Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

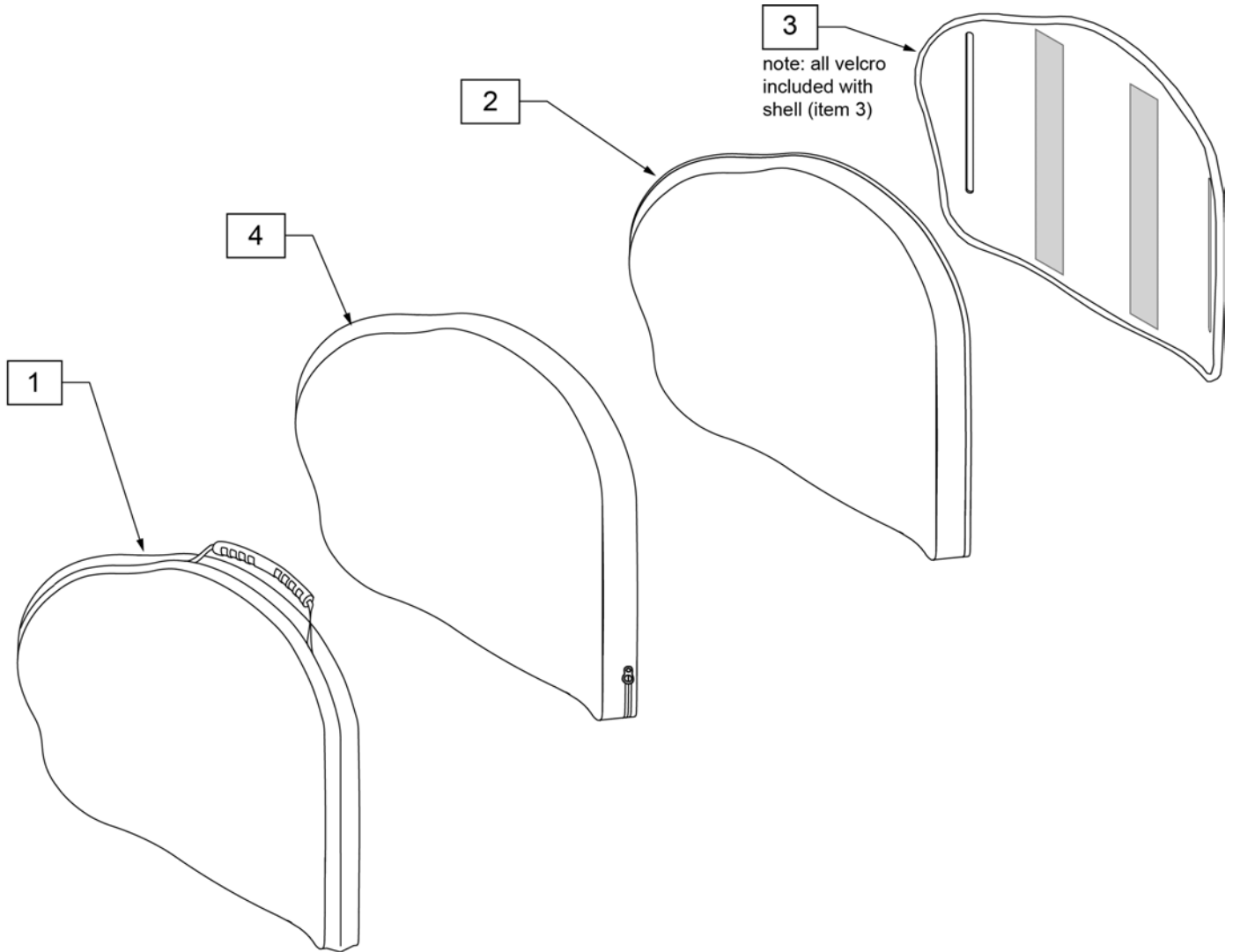


Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJ3PAMTS22	J3HD CVR POS MT 22W STD	2	BJ3PAMTT26	J3HD FOAM POS MT 26W TAL
1	CJ3PAMTT22	J3HD CVR POS MT 22W TAL	3	SJ3PAMTS22	J3HD SHELL POS MT 22W STD
1	CJ3PAMTS24	J3HD CVR POS MT 24W STD	3	SJ3PAMTT22	J3HD SHELL POS MT 22W TAL
1	CJ3PAMTT24	J3HD CVR POS MT 24W TAL	3	SJ3PAMTS24	J3HD SHELL POS MT 24W STD
1	CJ3PAMTS26	J3HD CVR POS MT 26W STD	3	SJ3PAMTT24	J3HD SHELL POS MT 24W TAL
1	CJ3PAMTT26	J3HD CVR POS MT 26W TAL	3	SJ3PAMTS26	J3HD SHELL POS MT 26W STD
2	BJ3PAMTS22	J3HD FOAM POS MT 22W STD	3	SJ3PAMTT26	J3HD SHELL POS MT 26W TAL
2	BJ3PAMTT22	J3HD FOAM POS MT 22W TAL	4	ICJ3PAMTS22	J3 HD BK,POS MT 22W STD, FS
2	BJ3PAMTS24	J3HD FOAM POS MT 24W STD	4	ICJ3PAMTT22	J3 HD BK,POS MT 22W TALL, FS
2	BJ3PAMTT24	J3HD FOAM POS MT 24W TAL	4	ICJ3PAMTS24	J3 HD BK,POS MT 24W STD, FS
2	BJ3PAMTS26	J3HD FOAM POS MT 26W STD	4	ICJ3PAMTT24	J3 HD BK,POS MT 24W TALL, FS

Pos.	Item Number	Description	Pos.	Item Number	Description
4	ICJ3PAMTS26	J3 HD BK,POS MT 26W STD, FS	4	ICJ3PAMTT26	J3 HD BK,POS MT 26W TALL, FS

Note: parts only; for complete Backrest see J3 Back order form.

Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

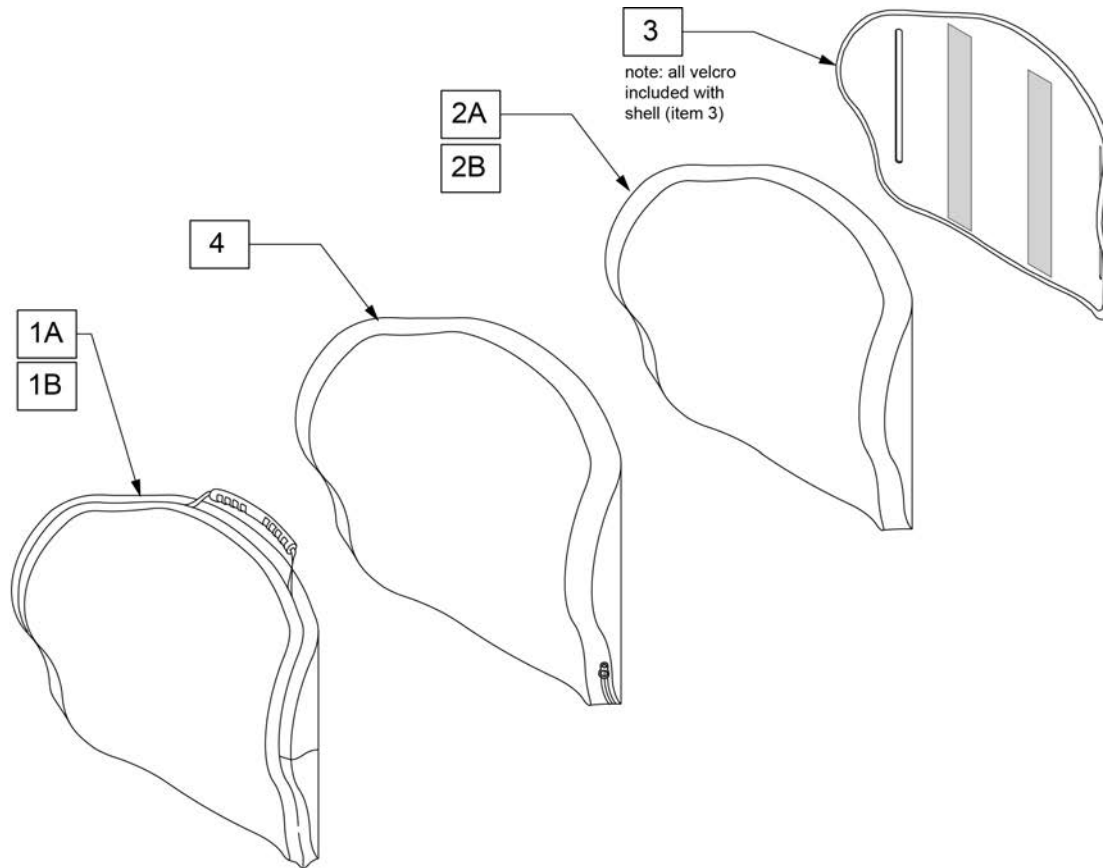


Pos.	Item Number	Description	Remarks
1A	CJ3PLMTS12	J3 CVR SHAL MT 12W STD	
1A	CJ3PLMTM12	J3 CVR SHAL MT 12W MED	
1A	CJ3PLMTT12	J3 CVR SHAL MT 12W TAL	
1A	CJ3PLMTS14	J3 CVR, SHAL MT 14W STD	
1A	CJ3PLMTM14	J3 CVR, SHAL MT 14W MED	
1A	CJ3PLMTT14	J3 CVR, SHAL MT 14W TALL	
1A	CJ3PLMTS16	J3 CVR, SHAL MT 16W STD	
1A	CJ3PLMTM16	J3 CVR, SHAL MT 16W MED	
1A	CJ3PLMTT16	J3 CVR, SHAL MT 16W TALL	
1A	CJ3PLMTS18	J3 CVR, SHAL MT 18W STD	
1A	CJ3PLMTM18	J3 CVR, SHAL MT 18W MED	
1A	CJ3PLMTT18	J3 CVR, SHAL MT 18W TALL	
1A	CJ3PLMTS20	J3 CVR, SHAL MT 20W STD	
1A	CJ3PLMTM20	J3 CVR, SHAL MT 20W MED	
1A	CJ3PLMTT20	J3 CVR, SHAL MT 20W TALL	
1B	CJ3PLMTS16CU	CVR J3 BK SHAL MT 16W STD CU	use w/ Cradle-U
1B	CJ3PLMTM16CU	CVR J3 BK SHAL MT 16W MED CU	use w/ Cradle-U

Pos.	Item Number	Description	Remarks
1B	CJ3PLMTT16CU	CVR J3 BK SHAL MT 16W TALL CU	use w/ Cradle-U
1B	CJ3PLMTS18CU	CVR J3 BK SHAL MT 18W STD CU	use w/ Cradle-U
1B	CJ3PLMTM18CU	CVR J3 BK SHAL MT 18W MED CU	use w/ Cradle-U
1B	CJ3PLMTT18CU	CVR J3 BK SHAL MT 18W TALL CU	use w/ Cradle-U
1B	CJ3PLMTS20CU	CVR J3 BK SHAL MT 20W STD CU	use w/ Cradle-U
1B	CJ3PLMTM20CU	CVR J3 BK SHAL MT 20W MED CU	use w/ Cradle-U
1B	CJ3PLMTT20CU	CVR J3 BK SHAL MT 20W TALL CU	use w/ Cradle-U
2A	BJ3PLMTS12	J3 FOAM SHAL MT 12W STD	
2A	BJ3PLMTM12	J3 FOAM SHAL MT 12W MED	
2A	BJ3PLMTT12	J3 FOAM SHAL MT 12W TAL	
2A	BJ3PLMTS14	J3 FOAM, SHAL MT 14W STD	
2A	BJ3PLMTM14	J3 FOAM, SHAL MT 14W MED	
2A	BJ3PLMTT14	J3 FOAM, SHAL MT 14W TALL	
2A	BJ3PLMTS16	J3 FOAM, SHAL MT 16W STD	
2A	BJ3PLMTM16	J3 FOAM, SHAL MT 16W MED	
2A	BJ3PLMTT16	J3 FOAM, SHAL MT 16W TALL	
2A	BJ3PLMTS18	J3 FOAM, SHAL MT 18W STD	
2A	BJ3PLMTM18	J3 FOAM, SHAL MT 18W MED	
2A	BJ3PLMTT18	J3 FOAM, SHAL MT 18W TALL	
2A	BJ3PLMTS20	J3 FOAM, SHAL MT 20W STD	
2A	BJ3PLMTM20	J3 FOAM, SHAL MT 20W MED	
2A	BJ3PLMTT20	J3 FOAM, SHAL MT 20W TALL	
2B	BJ3PLMTS16CU	BASE J3 BK SHAL MT 16W STD CU	use w/ Cradle-U
2B	BJ3PLMTM16CU	BASE J3 BK SHAL MT 16W MED CU	use w/ Cradle-U
2B	BJ3PLMTT16CU	BASE J3 BK SHAL MT 16W TALL CU	use w/ Cradle-U
2B	BJ3PLMTS18CU	BASE J3 BK SHAL MT 18W STD CU	use w/ Cradle-U
2B	BJ3PLMTM18CU	BASE J3 BK SHAL MT 18W MED CU	use w/ Cradle-U
2B	BJ3PLMTT18CU	BASE J3 BK SHAL MT 18W TALL CU	use w/ Cradle-U
2B	BJ3PLMTS20CU	BASE J3 BK SHAL MT 20W STD CU	use w/ Cradle-U
2B	BJ3PLMTM20CU	BASE J3 BK SHAL MT 20W MED CU	use w/ Cradle-U
2B	BJ3PLMTT20CU	BASE J3 BK SHAL MT 20W TALL CU	use w/ Cradle-U
3	SJ3PLMTS12	J3 SHELL SHAL MT 12W STD	
3	SJ3PLMTM12	J3 SHELL SHAL MT 12W MED	
3	SJ3PLMTT12	J3 SHELL SHAL MT 12W TAL	
3	SJ3PLMTS14	J3 SHELL, SHAL MT 14W STD	
3	SJ3PLMTM14	J3 SHELL, SHAL MT 14W MED	
3	SJ3PLMTT14	J3 SHELL, SHAL MT 14W TALL	
3	SJ3PLMTS16	J3 SHELL, SHAL MT 16W STD	
3	SJ3PLMTM16	J3 SHELL, SHAL MT 16W MED	
3	SJ3PLMTT16	J3 SHELL, SHAL MT 16W TALL	
3	SJ3PLMTS18	J3 SHELL, SHAL MT 18W STD	
3	SJ3PLMTM18	J3 SHELL, SHAL MT 18W MED	
3	SJ3PLMTT18	J3 SHELL, SHAL MT 18W TALL	
3	SJ3PLMTS20	J3 SHELL, SHAL MT 20W STD	
3	SJ3PLMTM20	J3 SHELL, SHAL MT 20W MED	
3	SJ3PLMTT20	J3 SHELL, SHAL MT 20W TALL	
4	ICJ3PLMTS12	J3 BK SHAL MT 12W STD SH	
4	ICJ3PLMTM12	J3 BK SHAL MT 12W MED SH	
4	ICJ3PLMTT12	J3 BK SHAL MT 12W TALL SH	
4	ICJ3PLMTS14	J3 BK SHAL MT 14W STD SH	
4	ICJ3PLMTM14	J3 BK SHAL MT 14W MED SH	
4	ICJ3PLMTT14	J3 BK SHAL MT 14W TALL SH	
4	ICJ3PLMTS16	J3 BK SHAL MT 16W STD SH	
4	ICJ3PLMTM16	J3 BK SHAL MT 16W MED SH	
4	ICJ3PLMTT16	J3 BK SHAL MT 16W TALL SH	
4	ICJ3PLMTS18	J3 BK SHAL MT 18W STD SH	
4	ICJ3PLMTM18	J3 BK SHAL MT 18W MED SH	

Pos.	Item Number	Description	Remarks
4	ICJ3PLMTT18	J3 BK SHAL MT 18W TALL SH	
4	ICJ3PLMTS20	J3 BK SHAL MT 20W STD SH	
4	ICJ3PLMTM20	J3 BK SHAL MT 20W MED SH	
4	ICJ3PLMTT20	J3 BK SHAL MT 20W TALL SH	

Note: parts only; for complete Backrest see J3 Back order form.

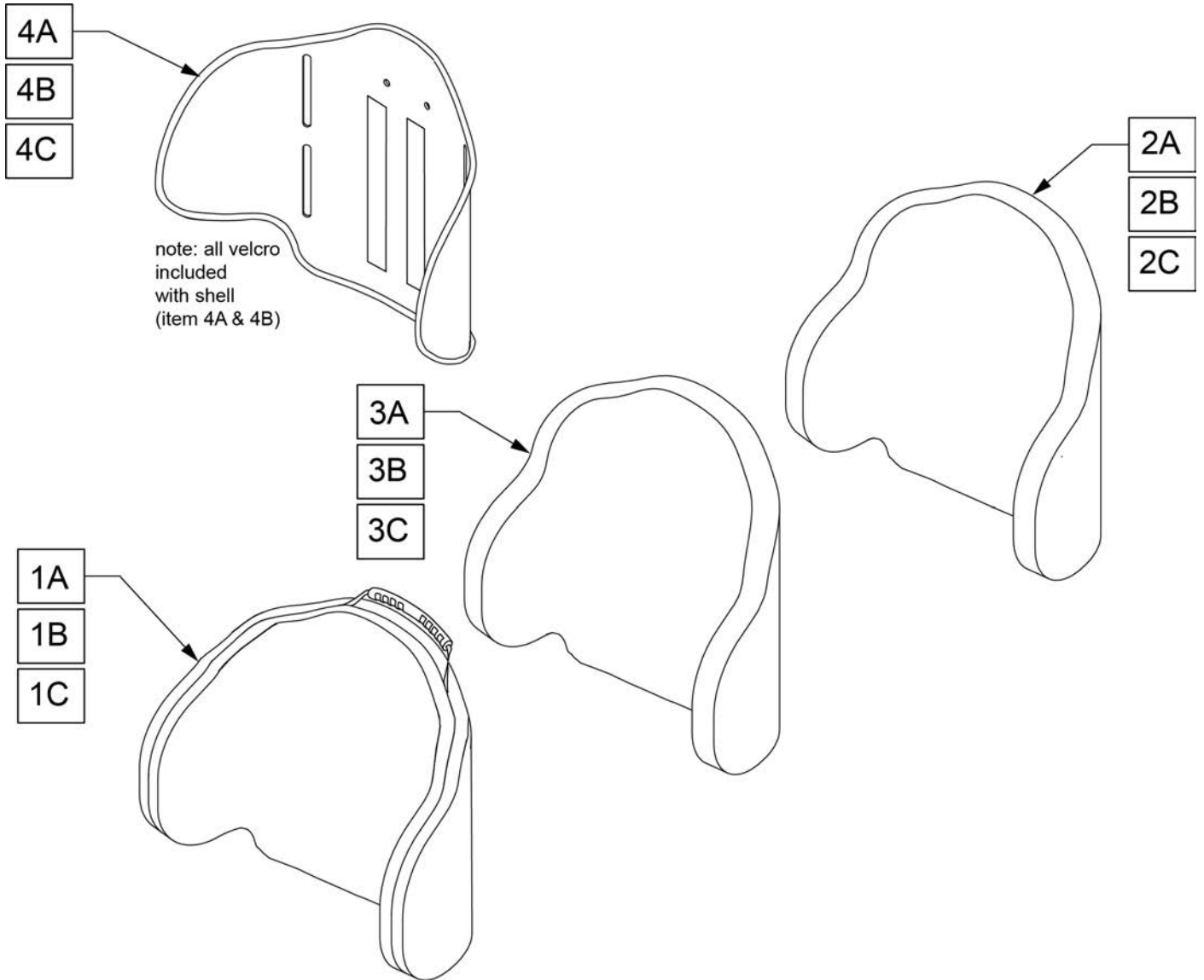


Pos.	Item Number	Description	Pos.	Item Number	Description
1A	CJ3PDMTS12	J3 BK CVR PD MT 12W SHT	1C	CJ3PDMTT16	J3 BK CVR PD MT 16W TALL
1A	CJ3PDMTS14	J3 BK CVR PD MT 14W SHT	1C	CJ3PDMTT18	J3 BK CVR PD MT 18W TALL
1A	CJ3PDMTS16	J3 BK CVR PD MT 16W SHT	1C	CJ3PDMTT20	J3 BK CVR PD MT 20W TALL
1A	CJ3PDMTS18	J3 BK CVR PD MT 18W SHT	2A	BJ3PDMTS12	J3 BK FOAM PD MT 12W SHT
1A	CJ3PDMTS20	J3 BK CVR PD MT 20W SHT	2A	BJ3PDMTS14	J3 BK FOAM PD MT 14W SHT
1B	CJ3PDMTM12	J3 BK CVR PD MT 12W MED	2A	BJ3PDMTS16	J3 BK FOAM PD MT 16W SHT
1B	CJ3PDMTM14	J3 BK CVR PD MT 14W MED	2A	BJ3PDMTS18	J3 BK FOAM PD MT 18W SHT
1B	CJ3PDMTM16	J3 BK CVR PD MT 16W MED	2A	BJ3PDMTS20	J3 BK FOAM PD MT 20W SHT
1B	CJ3PDMTM18	J3 BK CVR PD MT 18W MED	2B	BJ3PDMTM12	J3 BK FOAM PD MT 12W MED
1B	CJ3PDMTM20	J3 BK CVR PD MT 20W MED	2B	BJ3PDMTM14	J3 BK FOAM PD MT 14W MED
1C	CJ3PDMTT12	J3 BK CVR PD MT 12W TALL	2B	BJ3PDMTM16	J3 BK FOAM PD MT 16W MED
1C	CJ3PDMTT14	J3 BK CVR PD MT 14W TALL	2B	BJ3PDMTM18	J3 BK FOAM PD MT 18W MED

Pos.	Item Number	Description	Pos.	Item Number	Description
2B	BJ3PDMTM20	J3 BK FOAM PD MT 20W MED	3C	ICJ3PDMTT16	J3 BK INNER CVR PD MT 16W TALL
2C	BJ3PDMTT12	J3 BK FOAM PD MT 12W TALL	3C	ICJ3PDMTT18	J3 BK INNER CVR PD MT 18W TALL
2C	BJ3PDMTT14	J3 BK FOAM PD MT 14W TALL	3C	ICJ3PDMTT20	J3 BK INNER CVR PD MT 20W TALL
2C	BJ3PDMTT16	J3 BK FOAM PD MT 16W TALL	4A	SJ3PDMTS12	J3 BK SHL PD MT 12W SHT
2C	BJ3PDMTT18	J3 BK FOAM PD MT 18W TALL	4A	SJ3PDMTS14	J3 BK SHL PD MT 14W SHT
2C	BJ3PDMTT20	J3 BK FOAM PD MT 20W TALL	4A	SJ3PDMTS16	J3 BK SHL PD MT 16W SHT
3A	ICJ3PDMTS12	J3 BK INNER CVR PD MT 12W SHT	4A	SJ3PDMTS18	J3 BK SHL PD MT 18W SHT
3A	ICJ3PDMTS14	J3 BK INNER CVR PD MT 14W SHT	4A	SJ3PDMTS20	J3 BK SHL PD MT 20W SHT
3A	ICJ3PDMTS16	J3 BK INNER CVR PD MT 16W SHT	4B	SJ3PDMTM12	J3 BK SHL PD MT 12W MED
3A	ICJ3PDMTS18	J3 BK INNER CVR PD MT 18W SHT	4B	SJ3PDMTM14	J3 BK SHL PD MT 14W MED
3A	ICJ3PDMTS20	J3 BK INNER CVR PD MT 20W SHT	4B	SJ3PDMTM16	J3 BK SHL PD MT 16W MED
3B	ICJ3PDMTM12	J3 BK INNER CVR PD MT 12W MED	4B	SJ3PDMTM18	J3 BK SHL PD MT 18W MED
3B	ICJ3PDMTM14	J3 BK INNER CVR PD MT 14W MED	4B	SJ3PDMTM20	J3 BK SHL PD MT 20W MED
3B	ICJ3PDMTM16	J3 BK INNER CVR PD MT 16W MED	4C	SJ3PDMTT12	J3 BK SHL PD MT 12W TALL
3B	ICJ3PDMTM18	J3 BK INNER CVR PD MT 18W MED	4C	SJ3PDMTT14	J3 BK SHL PD MT 14W TALL
3B	ICJ3PDMTM20	J3 BK INNER CVR PD MT 20W MED	4C	SJ3PDMTT16	J3 BK SHL PD MT 16W TALL
3C	ICJ3PDMTT12	J3 BK INNER CVR PD MT 12W TALL	4C	SJ3PDMTT18	J3 BK SHL PD MT 18W TALL
3C	ICJ3PDMTT14	J3 BK INNER CVR PD MT 14W TALL	4C	SJ3PDMTT20	J3 BK SHL PD MT 20W TALL

NOTE: Parts only, for complete Backrest see J3 Back order form.


Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

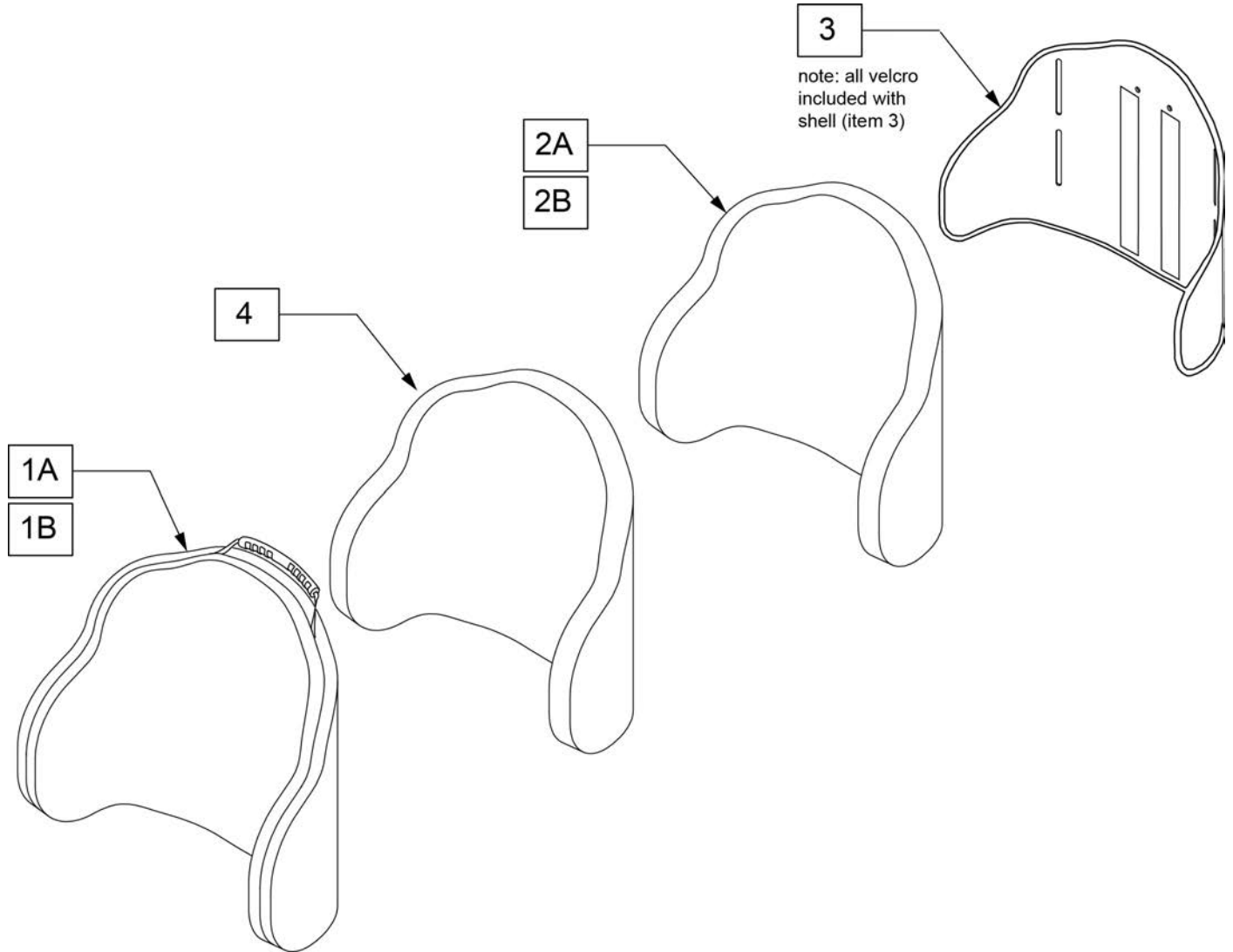

Pos.	Item Number	Description	Remarks
1A	CJ3PDCMTS14	J3 CVR PDC MT 14W STD	
1A	CJ3PDCMTT14	J3 CVR PDC MT 14W TAL	
1A	CJ3PDCMTS16	J3 CVR PDC MT 16W STD	
1A	CJ3PDCMTT16	J3 CVR PDC MT 16W TAL	
1A	CJ3PDCMTS18	J3 CVR PDC MT 18W STD	
1A	CJ3PDCMTT18	J3 CVR PDC MT 18W TAL	
1A	CJ3PDCMTS20	J3 CVR PDC MT 20W STD	
1A	CJ3PDCMTT20	J3 CVR PDC MT 20W TAL	
1B	CJ3PDCMTS16CU	CVR J3 BK PDC MT 16W STD CU	use w/ Cradle-U
1B	CJ3PDCMTT16CU	CVR J3 BK PDC MT 16W TALL CU	use w/ Cradle-U

Pos.	Item Number	Description	Remarks
1B	CJ3PDCMTS18CU	CVR J3 BK PDC MT 18W STD CU	use w/ Cradle-U
1B	CJ3PDCMTT18CU	CVR J3 BK PDC MT 18W TALL CU	use w/ Cradle-U
1B	CJ3PDCMTS20CU	CVR J3 BK PDC MT 20W STD CU	use w/ Cradle-U
1B	CJ3PDCMTT20CU	CVR J3 BK PDC MT 20W TALL CU	use w/ Cradle-U
2A	BJ3PDCMTS14	J3 FOAM PDC MT 14W STD	
2A	BJ3PDCMTT14	J3 FOAM PDC MT 14W TAL	
2A	BJ3PDCMTS16	J3 FOAM PDC MT 16W STD	
2A	BJ3PDCMTT16	J3 FOAM PDC MT 16W TAL	
2A	BJ3PDCMTS18	J3 FOAM PDC MT 18W STD	
2A	BJ3PDCMTT18	J3 FOAM PDC MT 18W TAL	
2A	BJ3PDCMTS20	J3 FOAM PDC MT 20W STD	
2A	BJ3PDCMTT20	J3 FOAM PDC MT 20W TAL	
2B	BJ3PDCMTS16CU	BASE J3 BK PDC MT 16W STD CU	use w/ Cradle-U
2B	BJ3PDCMTT16CU	BASE J3 BK PDC MT 16W TALL CU	use w/ Cradle-U
2B	BJ3PDCMTS18CU	BASE J3 BK PDC MT 18W STD CU	use w/ Cradle-U
2B	BJ3PDCMTT18CU	BASE J3 BK PDC MT 18W TALL CU	use w/ Cradle-U
2B	BJ3PDCMTS20CU	BASE J3 BK PDC MT 20W STD CU	use w/ Cradle-U
2B	BJ3PDCMTT20CU	BASE J3 BK PDC MT 20W TALL CU	use w/ Cradle-U
3	SJ3PDCMTS14	J3 SHELL PDC MT 14W STD	
3	SJ3PDCMTT14	J3 SHELL PDC MT 14W TAL	
3	SJ3PDCMTS16	J3 SHELL PDC MT 16W STD	
3	SJ3PDCMTT16	J3 SHELL PDC MT 16W TAL	
3	SJ3PDCMTS18	J3 SHELL PDC MT 18W STD	
3	SJ3PDCMTT18	J3 SHELL PDC MT 18W TAL	
3	SJ3PDCMTS20	J3 SHELL PDC MT 20W STD	
3	SJ3PDCMTT20	J3 SHELL PDC MT 20W TAL	
4	ICJ3PDCMTS14	J3 BK PDC MT 14W STD SH	
4	ICJ3PDCMTT14	J3 BK PDC MT 14W TALL SH	
4	ICJ3PDCMTS16	J3 BK PDC MT 16W STD SH	
4	ICJ3PDCMTT16	J3 BK PDC MT 16W TALL SH	
4	ICJ3PDCMTS18	J3 BK PDC MT 18W STD SH	
4	ICJ3PDCMTT18	J3 BK PDC MT 18W TALL SH	
4	ICJ3PDCMTS20	J3 BK PDC MT 20W STD SH	
4	ICJ3PDCMTT20	J3 BK PDC MT 20W TALL SH	


Note: parts only; for complete Backrest see J3 Back order form.

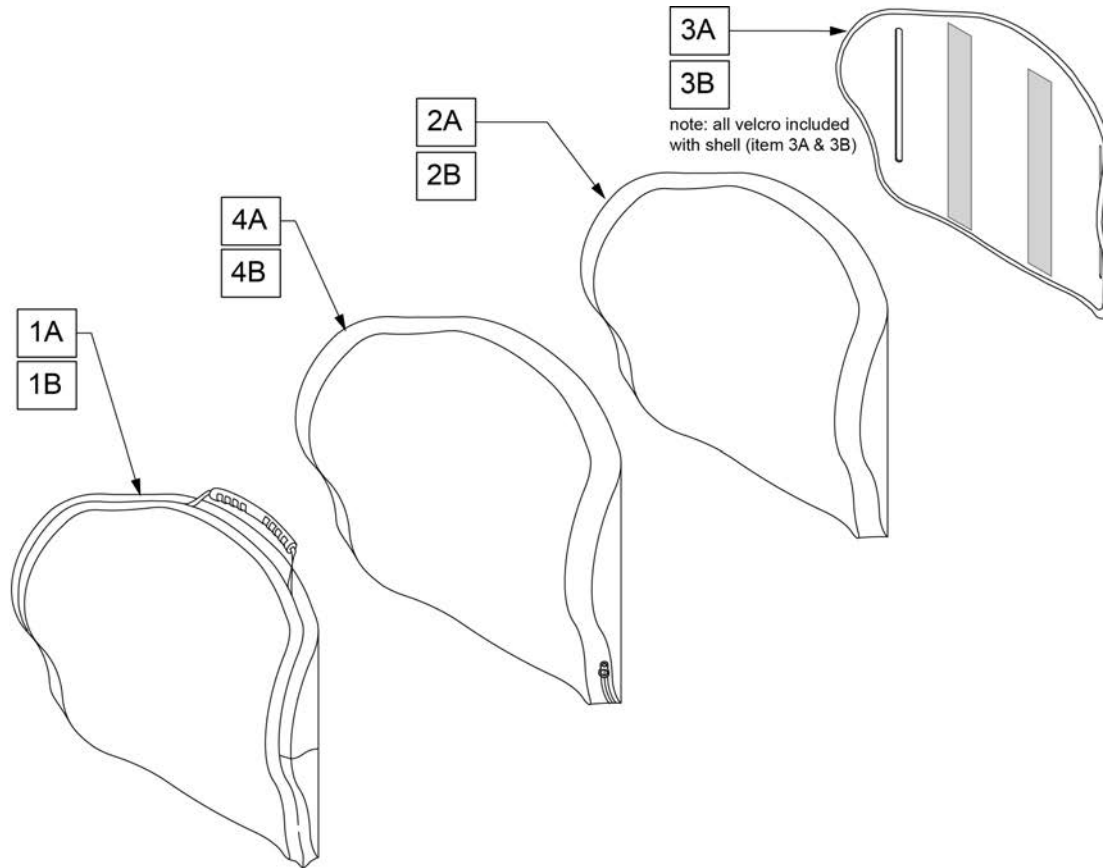

Pos.	Item Number	Description	Pos.	Item Number	Description
1A	CJ3CFMTS14	COVER J3 CF BACK PA MT SHT 14W	3A	SJ3CFMTS14	SHELL J3 CF BACK PA MT SHT 14W
1A	CJ3CFMTS16	COVER J3 CF BACK PA MT SHT 16W	3A	SJ3CFMTS16	SHELL J3 CF BACK PA MT SHT 16W
1A	CJ3CFMTS18	COVER J3 CF BACK PA MT SHT 18W	3A	SJ3CFMTS18	SHELL J3 CF BACK PA MT SHT 18W
1A	CJ3CFMTS20	COVER J3 CF BACK PA MT SHT 20W	3A	SJ3CFMTS20	SHELL J3 CF BACK PA MT SHT 20W
1B	CJ3CFMTT14	COVER J3 CF BACK PA MT TALL 14W	3B	SJ3CFMTT14	SHELL J3 CF BACK PA MT TALL 14W
1B	CJ3CFMTT16	COVER J3 CF BACK PA MT TALL 16W	3B	SJ3CFMTT16	SHELL J3 CF BACK PA MT TALL 16W
1B	CJ3CFMTT18	COVER J3 CF BACK PA MT TALL 18W	3B	SJ3CFMTT18	SHELL J3 CF BACK PA MT TALL 18W
1B	CJ3CFMTT20	COVER J3 CF BACK PA MT TALL 20W	3B	SJ3CFMTT20	SHELL J3 CF BACK PA MT TALL 20W
2A	BJ3PAMTS14	J3 FOAM, POS MT 14W STD	4A	ICJ3PAMTS14	J3BK INNER CVR PA MT 14W SHT
2A	BJ3PAMTS16	J3 FOAM, POS MT 16W STD	4A	ICJ3PAMTS16	J3BK INNER CVR PA MT 16W SHT
2A	BJ3PAMTS18	J3 FOAM, POS MT 18W STD	4A	ICJ3PAMTS18	J3BK INNER CVR PA MT 18W SHT
2A	BJ3PAMTS20	J3 FOAM, POS MT 20W STD	4A	ICJ3PAMTS20	J3BK INNER CVR PA MT 20W SHT
2B	BJ3PAMTT14	J3 FOAM, POS MT 14W TALL	4B	ICJ3PAMTT14	J3BK INNER CVR PA MT 14W TALL
2B	BJ3PAMTT16	J3 FOAM, POS MT 16W TALL	4B	ICJ3PAMTT16	J3BK INNER CVR PA MT 16W TALL
2B	BJ3PAMTT18	J3 FOAM, POS MT 18W TALL	4B	ICJ3PAMTT18	J3BK INNER CVR PA MT 18W TALL
2B	BJ3PAMTT20	J3 FOAM, POS MT 20W TALL	4B	ICJ3PAMTT20	J3BK INNER CVR PA MT 20W TALL

Note: parts only; for complete Backrest see J3 Back order form.

Note: if replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

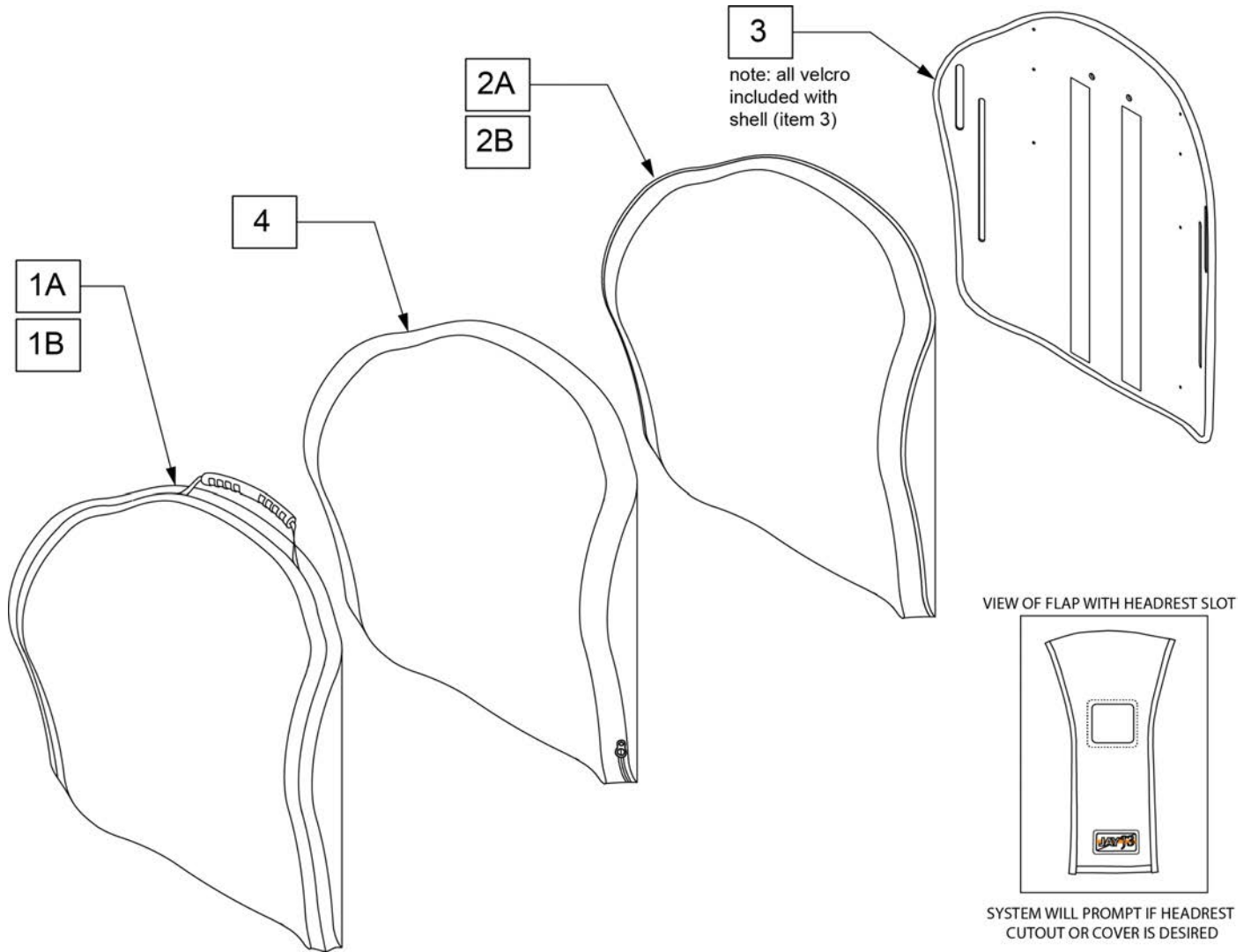


Pos.	Item Number	Description	Remarks
1A	CJ3PAUTS12	J3 CVR POS UT 12W STD	
1A	CJ3PAUTS14	J3 CVR, POS UT 14W STD	
1A	CJ3PAUTM14	J3 CVR, POS UT 14W MED	
1A	CJ3PAUTT14	J3 CVR, POS UT 14W TALL	
1A	CJ3PAUTS16	J3 CVR, POS UT 16W STD	
1A	CJ3PAUTM16	J3 CVR, POS UT 16W MED	
1A	CJ3PAUTT16	J3 CVR, POS UT 16W TALL	
1A	CJ3PAUTS18	J3 CVR, POS UT 18W STD	
1A	CJ3PAUTM18	J3 CVR, POS UT 18W MED	
1A	CJ3PAUTT18	J3 CVR, POS UT 18W TALL	
1A	CJ3PAUTS20	J3 CVR, POS UT 20W STD	

Pos.	Item Number	Description	Remarks
1A	CJ3PAUTM20	J3 CVR, POS UT 20W MED	
1A	CJ3PAUTT20	J3 CVR, POS UT 20W TALL	
1B	CJ3PAUTS16CU	CVR J3 BK POS UT 16W STD CU	use w/ Cradle-U
1B	CJ3PAUTM16CU	CVR J3 BK POS UT 16W MED CU	use w/ Cradle-U
1B	CJ3PAUTT16CU	CVR J3 BK POS UT 16W TALL CU	use w/ Cradle-U
1B	CJ3PAUTS18CU	CVR J3 BK POS UT 18W STD CU	use w/ Cradle-U
1B	CJ3PAUTM18CU	CVR J3 BK POS UT 18W MED CU	use w/ Cradle-U
1B	CJ3PAUTT18CU	CVR J3 BK POS UT 18W TALL CU	use w/ Cradle-U
1B	CJ3PAUTS20CU	CVR J3 BK POS UT 20W STD CU	use w/ Cradle-U
1B	CJ3PAUTM20CU	CVR J3 BK POS UT 20W MED CU	use w/ Cradle-U
1B	CJ3PAUTT20CU	CVR J3 BK POS UT 20W TALL CU	use w/ Cradle-U
2A	BJ3PAUTS12	J3 FOAM POS UT 12W STD	
2A	BJ3PAUTS14	J3 FOAM, POS UT 14W STD	
2A	BJ3PAUTM14	J3 FOAM, POS UT 14W MED	
2A	BJ3PAUTT14	J3 FOAM, POS UT 14W TALL	
2A	BJ3PAUTS16	J3 FOAM, POS UT 16W STD	
2A	BJ3PAUTM16	J3 FOAM, POS UT 16W MED	
2A	BJ3PAUTT16	J3 FOAM, POS UT 16W TALL	
2A	BJ3PAUTS18	J3 FOAM, POS UT 18W STD	
2A	BJ3PAUTM18	J3 FOAM, POS UT 18W MED	
2A	BJ3PAUTT18	J3 FOAM, POS UT 18W TALL	
2A	BJ3PAUTS20	J3 FOAM, POS UT 20W STD	
2A	BJ3PAUTM20	J3 FOAM, POS UT 20W MED	
2A	BJ3PAUTT20	J3 FOAM, POS UT 20W TALL	
2B	BJ3PAUTS16CU	BASE J3 BK POS UT 16W STD CU	use w/ Cradle-U
2B	BJ3PAUTM16CU	BASE J3 BK POS UT 16W MED CU	use w/ Cradle-U
2B	BJ3PAUTT16CU	BASE J3 BK POS UT 16W TALL CU	use w/ Cradle-U
2B	BJ3PAUTS18CU	BASE J3 BK POS UT 18W STD CU	use w/ Cradle-U
2B	BJ3PAUTM18CU	BASE J3 BK POS UT 18W MED CU	use w/ Cradle-U
2B	BJ3PAUTT18CU	BASE J3 BK POS UT 18W TALL CU	use w/ Cradle-U
2B	BJ3PAUTS20CU	BASE J3 BK POS UT 20W STD CU	use w/ Cradle-U
2B	BJ3PAUTM20CU	BASE J3 BK POS UT 20W MED CU	use w/ Cradle-U
2B	BJ3PAUTT20CU	BASE J3 BK POS UT 20W TALL CU	use w/ Cradle-U
3	SJ3PAUTS12	J3 SHELL POS UT 12W STD	
3	SJ3PAUTS14	J3 SHELL, POS UT 14W STD	
3	SJ3PAUTM14	J3 SHELL, POS UT 14W MED	
3	SJ3PAUTT14	J3 SHELL, POS UT 14W TALL	
3	SJ3PAUTS16	J3 SHELL, POS UT 16W STD	
3	SJ3PAUTM16	J3 SHELL, POS UT 16W MED	
3	SJ3PAUTT16	J3 SHELL, POS UT 16W TALL	
3	SJ3PAUTS18	J3 SHELL, POS UT 18W STD	
3	SJ3PAUTM18	J3 SHELL, POS UT 18W MED	
3	SJ3PAUTT18	J3 SHELL, POS UT 18W TALL	
3	SJ3PAUTS20	J3 SHELL, POS UT 20W STD	
3	SJ3PAUTM20	J3 SHELL, POS UT 20W MED	
3	SJ3PAUTT20	J3 SHELL, POS UT 20W TALL	
4	ICJ3PAUTS12	J3 BK POS UT 12W STD SH	
4	ICJ3PAUTS14	J3 BK POS UT 14W STD SH	
4	ICJ3PAUTM14	J3 BK POS UT 14W MED SH	
4	ICJ3PAUTT14	J3 BK POS UT 14W TALL SH	
4	ICJ3PAUTS16	J3 BK POS UT 16W STD SH	
4	ICJ3PAUTM16	J3 BK POS UT 16W MED SH	
4	ICJ3PAUTT16	J3 BK POS UT 16W TALL SH	
4	ICJ3PAUTS18	J3 BK POS UT 18W STD SH	
4	ICJ3PAUTM18	J3 BK POS UT 18W MED SH	
4	ICJ3PAUTT18	J3 BK POS UT 18W TALL SH	
4	ICJ3PAUTS20	J3 BK POS UT 20W STD SH	

Pos.	Item Number	Description	Remarks
4	ICJ3PAUTM20	J3 BK POS UT 20W MED SH	
4	ICJ3PAUTT20	J3 BK POS UT 20W TALL SH	

NOTE: Parts only, for complete Backrest see J3 Back order form.


Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

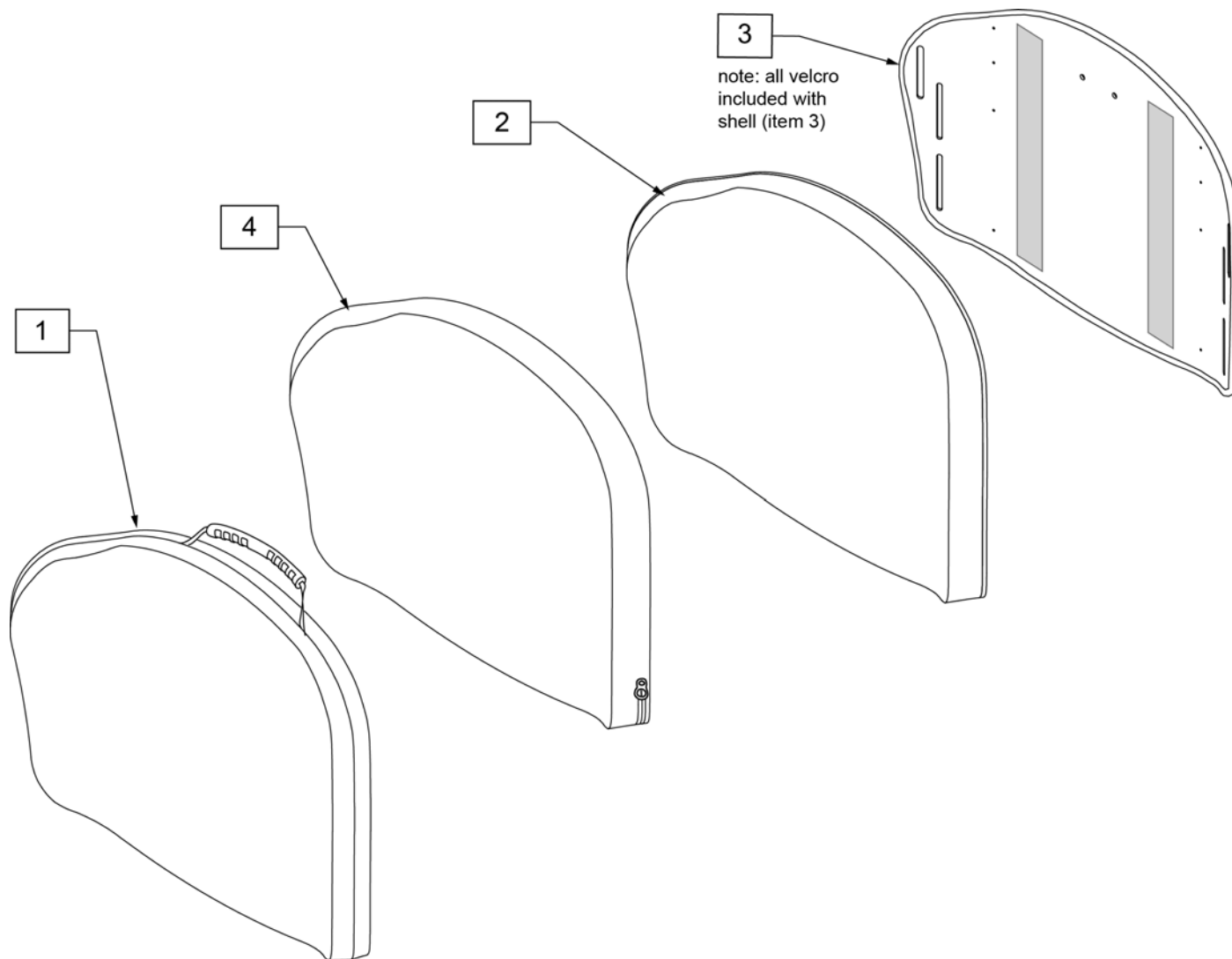

Pos.	Item Number	Description
1	CJ3PAUTM22	J3HD CVR POS UT 22W MED
1	CJ3PAUTM24	J3HD CVR POS UT 24W MED
2	BJ3PAUTM22	J3HD FOAM POS UT 22W MED
2	BJ3PAUTM24	J3HD FOAM POS UT 24W MED
3	SJ3PAUTM22	J3HD SHELL POS UT 22W MED
3	SJ3PAUTM24	J3HD SHELL POS UT 24W MED
4	ICJ3PAUTM22	J3 HD BK,POS UT 22W MED, FS
4	ICJ3PAUTM24	J3 HD BK,POS UT 24W MED, FS

Note: Parts only; for complete Backrest see J3 Back order form.

Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

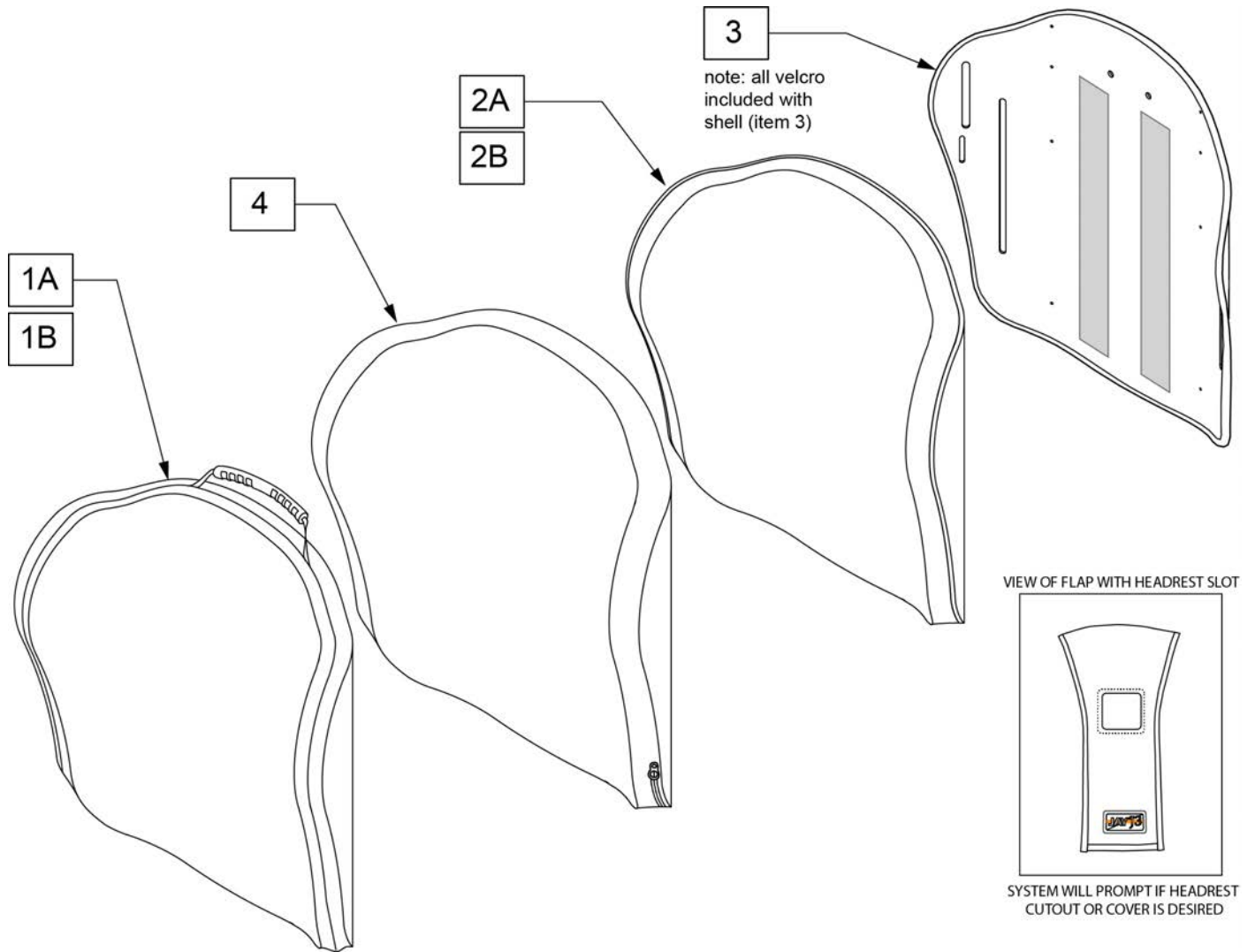


Pos.	Item Number	Description	Remarks
1A	CJ3PLUTS12	J3 CVR SHAL UT 12W STD	
1A	CJ3PLUTS14	J3 CVR, SHAL UT 14W STD	
1A	CJ3PLUTM14	J3 CVR, SHAL UT 14W MED	
1A	CJ3PLUTT14	J3 CVR, SHAL UT 14W TALL	
1A	CJ3PLUTS16	J3 CVR, SHAL UT 16W STD	
1A	CJ3PLUTM16	J3 CVR, SHAL UT 16W MED	
1A	CJ3PLUTT16	J3 CVR, SHAL UT 16W TALL	
1A	CJ3PLUTS18	J3 CVR, SHAL UT 18W STD	
1A	CJ3PLUTM18	J3 CVR, SHAL UT 18W MED	
1A	CJ3PLUTT18	J3 CVR, SHAL UT 18W TALL	
1A	CJ3PLUTS20	J3 CVR, SHAL UT 20W STD	

Pos.	Item Number	Description	Remarks
1A	CJ3PLUTM20	J3 CVR, SHAL UT 20W MED	
1A	CJ3PLUTT20	J3 CVR, SHAL UT 20W TALL	
1B	CJ3PLUTS16CU	CVR J3 BK SHAL UT 16W STD CU	use w/ Cradle-U
1B	CJ3PLUTM16CU	CVR J3 BK SHAL UT 16W MED CU	use w/ Cradle-U
1B	CJ3PLUTT16CU	CVR J3 BK SHAL UT 16W TALL CU	use w/ Cradle-U
1B	CJ3PLUTS18CU	CVR J3 BK SHAL UT 18W STD CU	use w/ Cradle-U
1B	CJ3PLUTM18CU	CVR J3 BK SHAL UT 18W MED CU	use w/ Cradle-U
1B	CJ3PLUTT18CU	CVR J3 BK SHAL UT 18W TALL CU	use w/ Cradle-U
1B	CJ3PLUTS20CU	CVR J3 BK SHAL UT 20W STD CU	use w/ Cradle-U
1B	CJ3PLUTM20CU	CVR J3 BK SHAL UT 20W MED CU	use w/ Cradle-U
1B	CJ3PLUTT20CU	CVR J3 BK SHAL UT 20W TALL CU	use w/ Cradle-U
2A	BJ3PLUTS12	J3 FOAM SHAL UT 12W STD	
2A	BJ3PLUTS14	J3 FOAM, SHAL UT 14W STD	
2A	BJ3PLUTM14	J3 FOAM, SHAL UT 14W MED	
2A	BJ3PLUTT14	J3 FOAM, SHAL UT 14W TALL	
2A	BJ3PLUTS16	J3 FOAM, SHAL UT 16W STD	
2A	BJ3PLUTM16	J3 FOAM, SHAL UT 16W MED	
2A	BJ3PLUTT16	J3 FOAM, SHAL UT 16W TALL	
2A	BJ3PLUTS18	J3 FOAM, SHAL UT 18W STD	
2A	BJ3PLUTM18	J3 FOAM, SHAL UT 18W MED	
2A	BJ3PLUTT18	J3 FOAM, SHAL UT 18W TALL	
2A	BJ3PLUTS20	J3 FOAM, SHAL UT 20W STD	
2A	BJ3PLUTM20	J3 FOAM, SHAL UT 20W MED	
2A	BJ3PLUTT20	J3 FOAM, SHAL UT 20W TALL	
2B	BJ3PLUTS16CU	BASE J3 BK SHAL UT 16W STD CU	use w/ Cradle-U
2B	BJ3PLUTM16CU	BASE J3 BK SHAL UT 16W MED CU	use w/ Cradle-U
2B	BJ3PLUTT16CU	BASE J3 BK SHAL UT 16W TALL CU	use w/ Cradle-U
2B	BJ3PLUTS18CU	BASE J3 BK SHAL UT 18W STD CU	use w/ Cradle-U
2B	BJ3PLUTM18CU	BASE J3 BK SHAL UT 18W MED CU	use w/ Cradle-U
2B	BJ3PLUTT18CU	BASE J3 BK SHAL UT 18W TALL CU	use w/ Cradle-U
2B	BJ3PLUTS20CU	BASE J3 BK SHAL UT 20W STD CU	use w/ Cradle-U
2B	BJ3PLUTM20CU	BASE J3 BK SHAL UT 20W MED CU	use w/ Cradle-U
2B	BJ3PLUTT20CU	BASE J3 BK SHAL UT 20W TALL CU	use w/ Cradle-U
3	SJ3PLUTS12	J3 SHELL SHAL UT 12W STD	
3	SJ3PLUTS14	J3 SHELL, SHAL UT 14W STD	
3	SJ3PLUTM14	J3 SHELL, SHAL UT 14W MED	
3	SJ3PLUTT14	J3 SHELL, SHAL UT 14W TALL	
3	SJ3PLUTS16	J3 SHELL, SHAL UT 16W STD	
3	SJ3PLUTM16	J3 SHELL, SHAL UT 16W MED	
3	SJ3PLUTT16	J3 SHELL, SHAL UT 16W TALL	
3	SJ3PLUTS18	J3 SHELL, SHAL UT 18W STD	
3	SJ3PLUTM18	J3 SHELL, SHAL UT 18W MED	
3	SJ3PLUTT18	J3 SHELL, SHAL UT 18W TALL	
3	SJ3PLUTS20	J3 SHELL, SHAL UT 20W STD	
3	SJ3PLUTM20	J3 SHELL, SHAL UT 20W MED	
3	SJ3PLUTT20	J3 SHELL, SHAL UT 20W TALL	
4	ICJ3PLUTS12	J3 BK SHAL UT 12W STD SH	
4	ICJ3PLUTS14	J3 BK SHAL UT 14W STD SH	
4	ICJ3PLUTM14	J3 BK SHAL UT 14W MED SH	
4	ICJ3PLUTT14	J3 BK SHAL UT 14W TALL SH	
4	ICJ3PLUTS16	J3 BK SHAL UT 16W STD SH	
4	ICJ3PLUTM16	J3 BK SHAL UT 16W MED SH	
4	ICJ3PLUTT16	J3 BK SHAL UT 16W TALL SH	
4	ICJ3PLUTS18	J3 BK SHAL UT 18W STD SH	
4	ICJ3PLUTM18	J3 BK SHAL UT 18W MED SH	
4	ICJ3PLUTT18	J3 BK SHAL UT 18W TALL SH	
4	ICJ3PLUTS20	J3 BK SHAL UT 20W STD SH	

Pos.	Item Number	Description	Remarks
4	ICJ3PLUTM20	J3 BK SHAL UT 20W MED SH	
4	ICJ3PLUTT20	J3 BK SHAL UT 20W TALL SH	

Note: Parts only; for complete Backrest see J3 Back order form.


Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

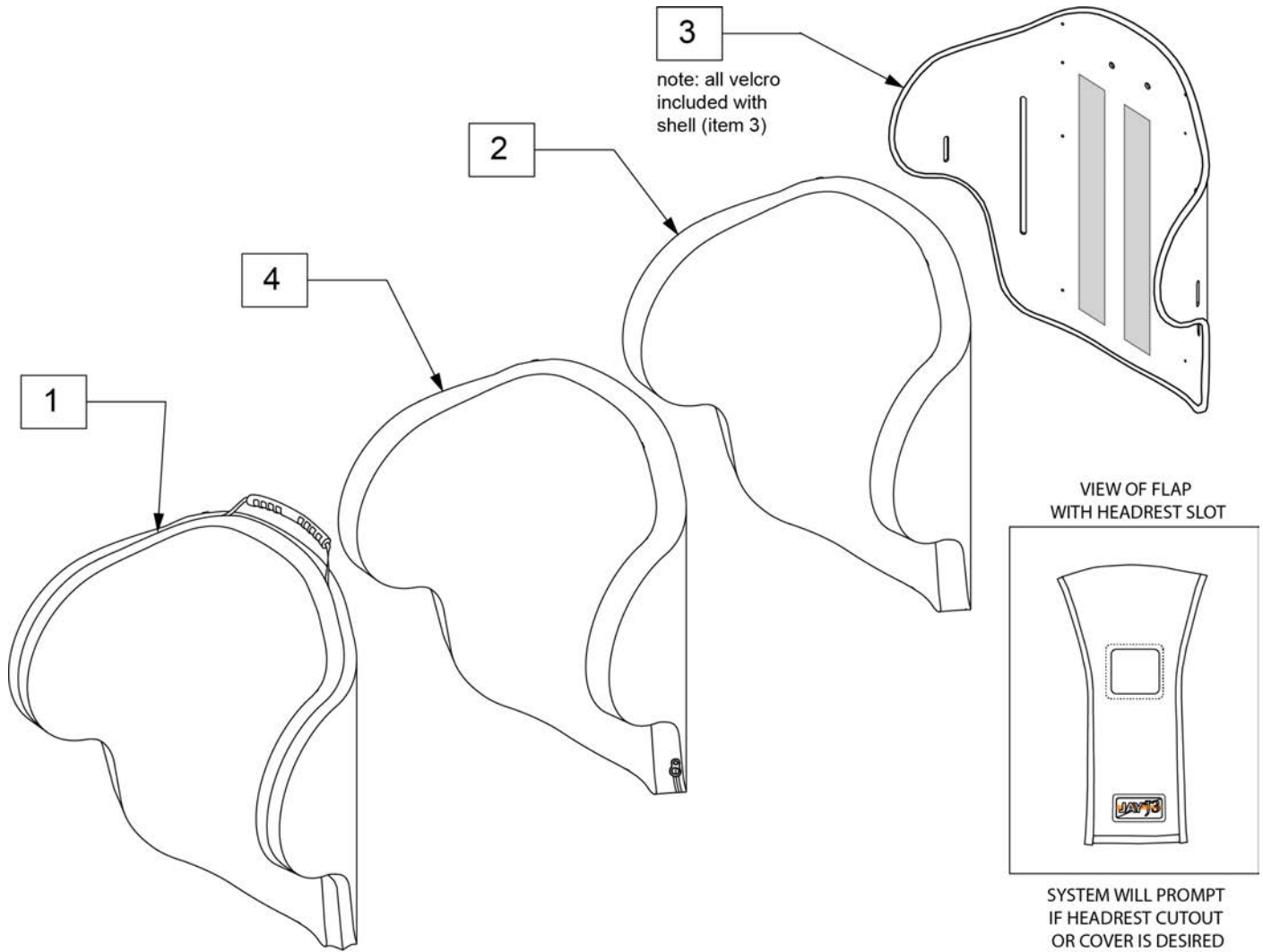

Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJ3PDLUTS14	J3 CVR, DEEP UT 14W STD	1	CJ3PDLUTT20	J3 CVR, DEEP UT 20W TALL
1	CJ3PDLUTM14	J3 CVR, DEEP UT 14W MED	2	BJ3PDLUTS14	J3 FOAM, DEEP UT 14W STD
1	CJ3PDLUTT14	J3 CVR, DEEP UT 14W TALL	2	BJ3PDLUTM14	J3 FOAM, DEEP UT 14W MED
1	CJ3PDLUTS16	J3 CVR, DEEP UT 16W STD	2	BJ3PDLUTT14	J3 FOAM, DEEP UT 14W TALL
1	CJ3PDLUTM16	J3 CVR, DEEP UT 16W MED	2	BJ3PDLUTS16	J3 FOAM, DEEP UT 16W STD
1	CJ3PDLUTT16	J3 CVR, DEEP UT 16W TALL	2	BJ3PDLUTM16	J3 FOAM, DEEP UT 16W MED
1	CJ3PDLUTS18	J3 CVR, DEEP UT 18W STD	2	BJ3PDLUTT16	J3 FOAM, DEEP UT 16W TALL
1	CJ3PDLUTM18	J3 CVR, DEEP UT 18W MED	2	BJ3PDLUTS18	J3 FOAM, DEEP UT 18W STD
1	CJ3PDLUTT18	J3 CVR, DEEP UT 18W TALL	2	BJ3PDLUTM18	J3 FOAM, DEEP UT 18W MED
1	CJ3PDLUTS20	J3 CVR, DEEP UT 20W STD	2	BJ3PDLUTT18	J3 FOAM, DEEP UT 18W TALL
1	CJ3PDLUTM20	J3 CVR, DEEP UT 20W MED	2	BJ3PDLUTS20	J3 FOAM, DEEP UT 20W STD

Pos.	Item Number	Description	Pos.	Item Number	Description
2	BJ3PDLUTM20	J3 FOAM, DEEP UT 20W MED	3	SJ3PDLUTT20	J3 SHELL, DEEP UT 20W TALL
2	BJ3PDLUTT20	J3 FOAM, DEEP UT 20W TALL	4	ICJ3PDLUTS14	J3 BK DEEP UT 14W STD SH
3	SJ3PDLUTS14	J3 SHELL, DEEP UT 14W STD	4	ICJ3PDLUTM14	J3 BK DEEP UT 14W MED SH
3	SJ3PDLUTM14	J3 SHELL, DEEP UT 14W MED	4	ICJ3PDLUTT14	J3 BK DEEP UT 14W TALL SH
3	SJ3PDLUTT14	J3 SHELL, DEEP UT 14W TALL	4	ICJ3PDLUTS16	J3 BK DEEP UT 16W STD SH
3	SJ3PDLUTS16	J3 SHELL, DEEP UT 16W STD	4	ICJ3PDLUTM16	J3 BK DEEP UT 16W MED SH
3	SJ3PDLUTM16	J3 SHELL, DEEP UT 16W MED	4	ICJ3PDLUTT16	J3 BK DEEP UT 16W TALL SH
3	SJ3PDLUTT16	J3 SHELL, DEEP UT 16W TALL	4	ICJ3PDLUTS18	J3 BK DEEP UT 18W STD SH
3	SJ3PDLUTS18	J3 SHELL, DEEP UT 18W STD	4	ICJ3PDLUTM18	J3 BK DEEP UT 18W MED SH
3	SJ3PDLUTM18	J3 SHELL, DEEP UT 18W MED	4	ICJ3PDLUTT18	J3 BK DEEP UT 18W TALL SH
3	SJ3PDLUTT18	J3 SHELL, DEEP UT 18W TALL	4	ICJ3PDLUTS20	J3 BK DEEP UT 20W STD SH
3	SJ3PDLUTS20	J3 SHELL, DEEP UT 20W STD	4	ICJ3PDLUTM20	J3 BK DEEP UT 20W MED SH
3	SJ3PDLUTM20	J3 SHELL, DEEP UT 20W MED	4	ICJ3PDLUTT20	J3 BK DEEP UT 20W TALL SH

Note: parts only; for complete Backrest see J3 Back order form.

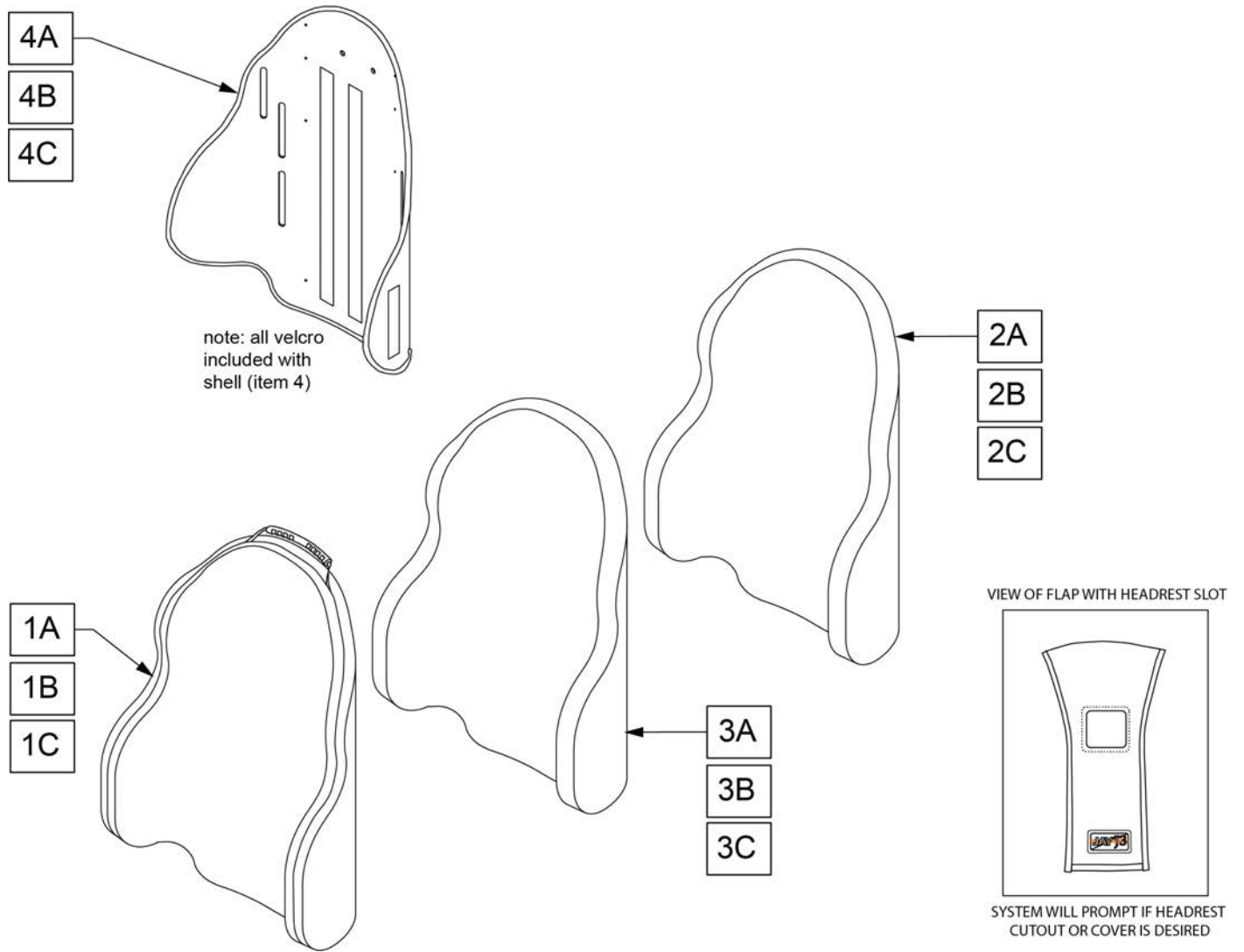


Pos.	Item Number	Description	Pos.	Item Number	Description
1A	CJ3PDUTS12	J3 BK CVR PD UT 12W SHT	2A	BJ3PDUTS14	J3 BK FOAM PD UT 14W SHT
1A	CJ3PDUTS14	J3 BK CVR PD UT 14W SHT	2A	BJ3PDUTS16	J3 BK FOAM PD UT 16W SHT
1A	CJ3PDUTS16	J3 BK CVR PD UT 16W SHT	2A	BJ3PDUTS18	J3 BK FOAM PD UT 18W SHT
1A	CJ3PDUTS18	J3 BK CVR PD UT 18W SHT	2B	BJ3PDUTS20	J3 BK FOAM PD UT 20W SHT
1B	CJ3PDUTS20	J3 BK CVR PD UT 20W SHT	2B	BJ3PDUTM14	J3 BK FOAM PD UT 14W MED
1B	CJ3PDUTM14	J3 BK CVR PD UT 14W MED	2B	BJ3PDUTM16	J3 BK FOAM PD UT 16W MED
1B	CJ3PDUTM16	J3 BK CVR PD UT 16W MED	2B	BJ3PDUTM18	J3 BK FOAM PD UT 18W MED
1B	CJ3PDUTM18	J3 BK CVR PD UT 18W MED	2B	BJ3PDUTM20	J3 BK FOAM PD UT 20W MED
1B	CJ3PDUTM20	J3 BK CVR PD UT 20W MED	2C	BJ3PDUTT14	J3 BK FOAM PD UT 14W TALL
1C	CJ3PDUTT14	J3 BK CVR PD UT 14W TALL	2C	BJ3PDUTT16	J3 BK FOAM PD UT 16W TALL
1C	CJ3PDUTT16	J3 BK CVR PD UT 16W TALL	2C	BJ3PDUTT18	J3 BK FOAM PD UT 18W TALL
1C	CJ3PDUTT18	J3 BK CVR PD UT 18W TALL	2C	BJ3PDUTT20	J3 BK FOAM PD UT 20W TALL
1C	CJ3PDUTT20	J3 BK CVR PD UT 20W TALL	3A	ICJ3PDUTS12	J3 BK INNER CVR PD UT 12W SHT
2A	BJ3PDUTS12	J3 BK FOAM PD UT 12W SHT	3A	ICJ3PDUTS14	J3 BK INNER CVR PD UT 14W SHT

Pos.	Item Number	Description	Pos.	Item Number	Description
3A	ICJ3PDUTS16	J3 BK INNER CVR PD UT 16W SHT	4A	SJ3PDUTS14	J3 BK SHL PD UT 14W SHT
3A	ICJ3PDUTS18	J3 BK INNER CVR PD UT 18W SHT	4A	SJ3PDUTS16	J3 BK SHL PD UT 16W SHT
3A	ICJ3PDUTS20	J3 BK INNER CVR PD UT 20W SHT	4A	SJ3PDUTS18	J3 BK SHL PD UT 18W SHT
3B	ICJ3PDUTM14	J3 BK INNER CVR PD UT 14W MED	4A	SJ3PDUTS20	J3 BK SHL PD UT 20W SHT
3B	ICJ3PDUTM16	J3 BK INNER CVR PD UT 16W MED	4B	SJ3PDUTM14	J3 BK SHL PD UT 14W MED
3B	ICJ3PDUTM18	J3 BK INNER CVR PD UT 18W MED	4B	SJ3PDUTM16	J3 BK SHL PD UT 16W MED
3B	ICJ3PDUTM20	J3 BK INNER CVR PD UT 20W MED	4B	SJ3PDUTM18	J3 BK SHL PD UT 18W MED
3C	ICJ3PDUTT14	J3 BK INNER CVR PD UT 14W TALL	4B	SJ3PDUTM20	J3 BK SHL PD UT 20W MED
3C	ICJ3PDUTT16	J3 BK INNER CVR PD UT 16W TALL	4C	SJ3PDUTT14	J3 BK SHL PD UT 14W TALL
3C	ICJ3PDUTT18	J3 BK INNER CVR PD UT 18W TALL	4C	SJ3PDUTT16	J3 BK SHL PD UT 16W TALL
3C	ICJ3PDUTT20	J3 BK INNER CVR PD UT 20W TALL	4C	SJ3PDUTT18	J3 BK SHL PD UT 18W TALL
4A	SJ3PDUTS12	J3 BK SHL PD UT 12W SHT	4C	SJ3PDUTT20	J3 BK SHL PD UT 20W TALL

NOTE: Parts only, for complete Backrest see J3 Back order form.

Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

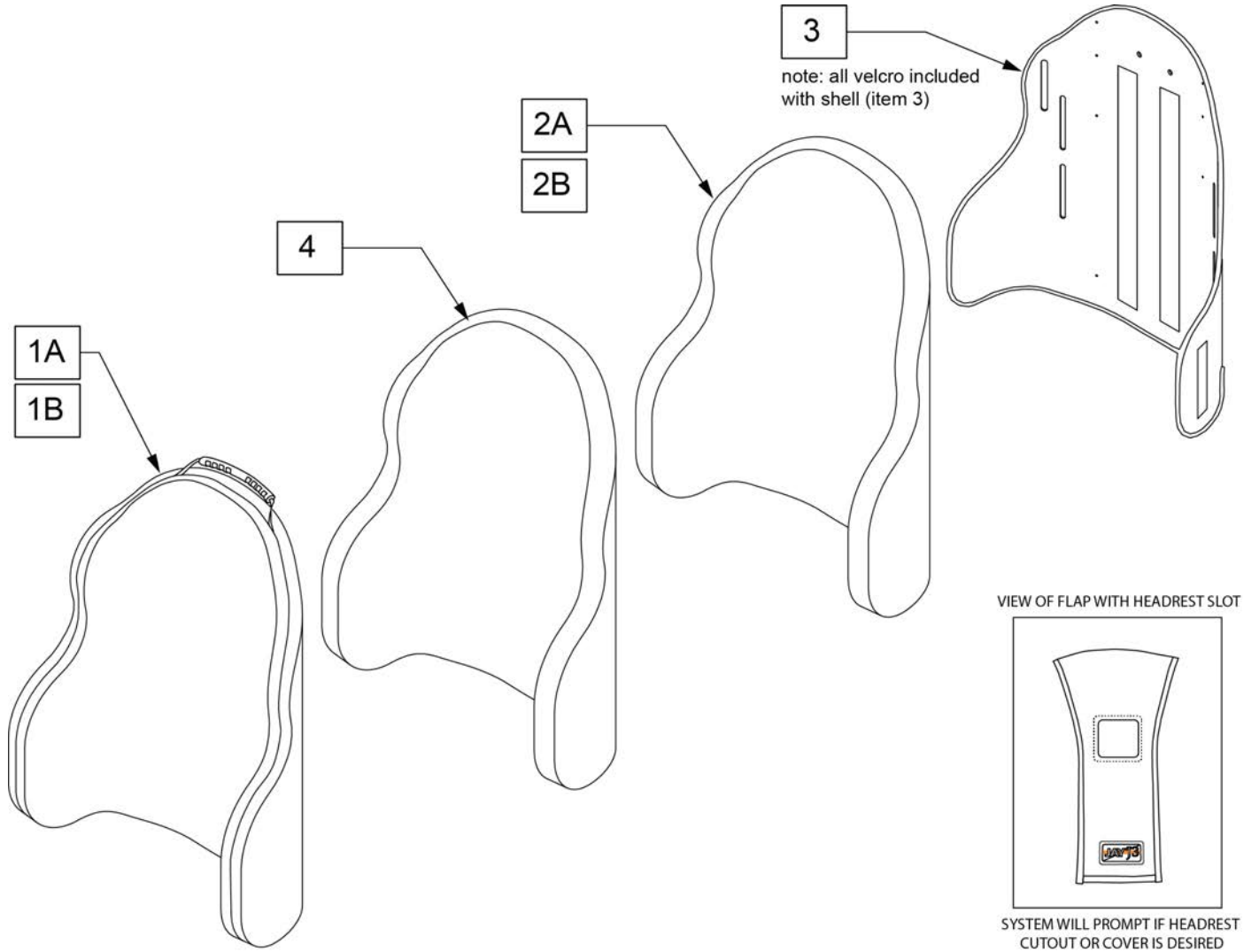


Pos.	Item Number	Description	Remarks
1A	CJ3PDCUTM14	J3 CVR PDC UT 14W MED	
1A	CJ3PDCUTM16	J3 CVR PDC UT 16W MED	
1A	CJ3PDCUTM18	J3 CVR PDC UT 18W MED	
1A	CJ3PDCUTM20	J3 CVR PDC UT 20W MED	
1B	CJ3PDCUTM16CU	CVR J3 BK PDC UT 16W MED CU	use w/ Cradle-U
1B	CJ3PDCUTM18CU	CVR J3 BK PDC UT 18W MED CU	use w/ Cradle-U
1B	CJ3PDCUTM20CU	CVR J3 BK PDC UT 20W MED CU	use w/ Cradle-U
2A	BJ3PDCUTM14	J3 FOAM PDC UT 14W MED	
2A	BJ3PDCUTM16	J3 FOAM PDC UT 16W MED	
2A	BJ3PDCUTM18	J3 FOAM PDC UT 18W MED	
2A	BJ3PDCUTM20	J3 FOAM PDC UT 20W MED	

Pos.	Item Number	Description	Remarks
2B	BJ3PDCUTM16CU	BASE J3 BK PDC UT 16W MED CU	use w/ Cradle-U
2B	BJ3PDCUTM18CU	BASE J3 BK PDC UT 18W MED CU	use w/ Cradle-U
2B	BJ3PDCUTM20CU	BASE J3 BK PDC UT 20W MED CU	use w/ Cradle-U
3	SJ3PDCUTM14	J3 SHELL PDC UT 14W MED	
3	SJ3PDCUTM16	J3 SHELL PDC UT 16W MED	
3	SJ3PDCUTM18	J3 SHELL PDC UT 18W MED	
3	SJ3PDCUTM20	J3 SHELL PDC UT 20W MED	
4	ICJ3PDCUTM14	J3 BK PDC UT 14W MED SH	
4	ICJ3PDCUTM16	J3 BK PDC UT 16W MED SH	
4	ICJ3PDCUTM18	J3 BK PDC UT 18W MED SH	
4	ICJ3PDCUTM20	J3 BK PDC UT 20W MED SH	

Note: Parts only; for complete Backrest see J3 Back order form.

Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

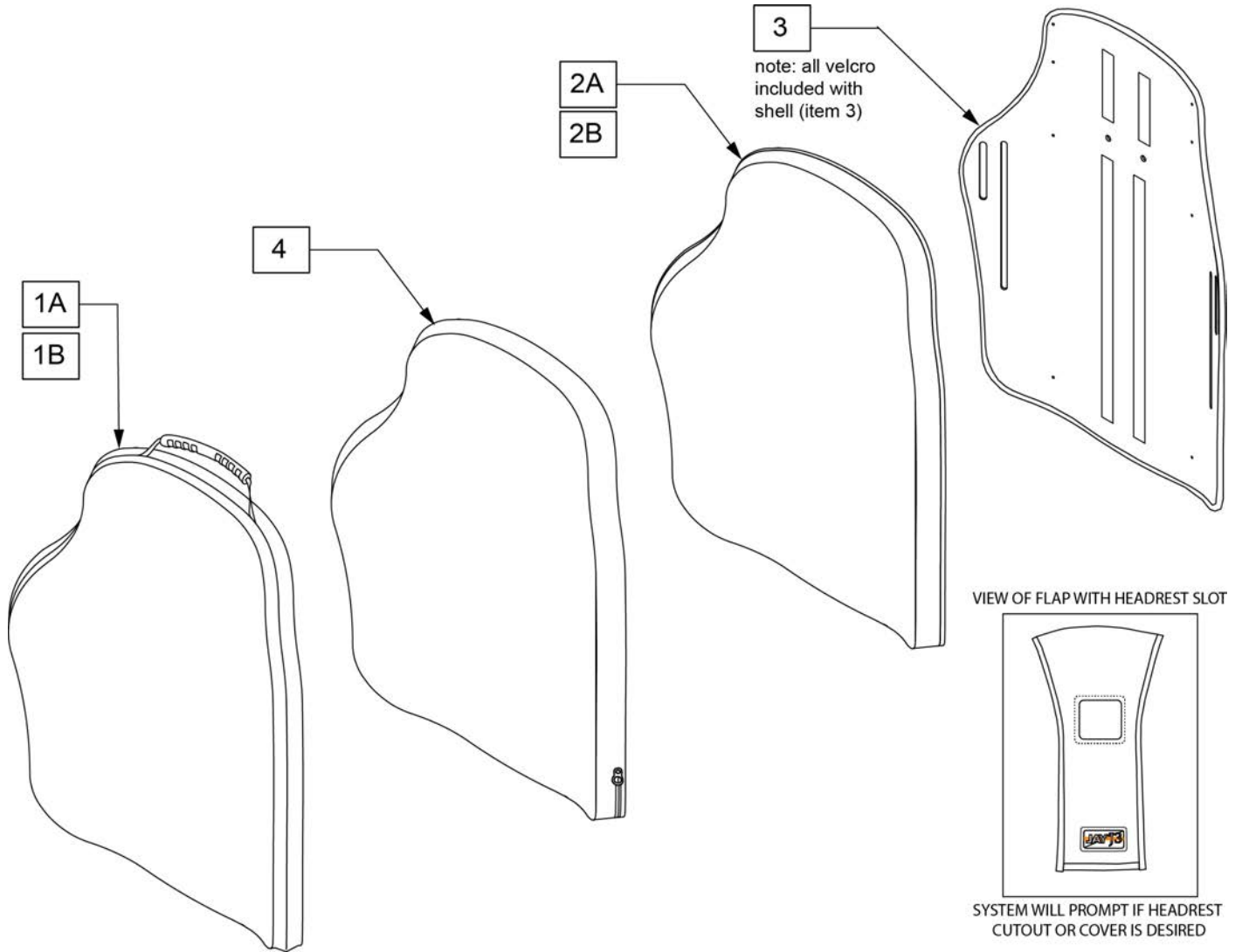


Pos.	Item Number	Description	Remarks
1A	CJ3PASHS14	J3 CVR, POS SH 14W STD	
1A	CJ3PASHS16	J3 CVR, POS SH 16W STD	
1A	CJ3PASHM16	J3 CVR, POS SH 16W MED	
1A	CJ3PASHS18	J3 CVR, POS SH 18W STD	
1A	CJ3PASHM18	J3 CVR, POS SH 18W MED	
1A	CJ3PASHT18	J3 CVR, POS SH 18W TALL	
1A	CJ3PASHS20	J3 CVR, POS SH 20W STD	
1A	CJ3PASHM20	J3 CVR, POS SH 20W MED	
1A	CJ3PASHT20	J3 CVR, POS SH 20W TALL	
1B	CJ3PASHS16CU	CVR J3 BK POS SH 16W STD CU	use w/ Cradle-U

Pos.	Item Number	Description	Remarks
1B	CJ3PASHM16CU	CVR J3 BK POS SH 16W MED CU	use w/ Cradle-U
1B	CJ3PASHS18CU	CVR J3 BK POS SH 18W STD CU	use w/ Cradle-U
1B	CJ3PASHM18CU	CVR J3 BK POS SH 18W MED CU	use w/ Cradle-U
1B	CJ3PASHT18CU	CVR J3 BK POS SH 18W TALL CU	use w/ Cradle-U
1B	CJ3PASHS20CU	CVR J3 BK POS SH 20W STD CU	use w/ Cradle-U
1B	CJ3PASHM20CU	CVR J3 BK POS SH 20W MED CU	use w/ Cradle-U
1B	CJ3PASHT20CU	CVR J3 BK POS SH 20W TALL CU	use w/ Cradle-U
2A	BJ3PASHS14	J3 FOAM, POS SH 14W STD	
2A	BJ3PASHS16	J3 FOAM, POS SH 16W STD	
2A	BJ3PASHM16	J3 FOAM, POS SH 16W MED	
2A	BJ3PASHS18	J3 FOAM, POS SH 18W STD	
2A	BJ3PASHM18	J3 FOAM, POS SH 18W MED	
2A	BJ3PASHT18	J3 FOAM, POS SH 18W TALL	
2A	BJ3PASHS20	J3 FOAM, POS SH 20W STD	
2A	BJ3PASHM20	J3 FOAM, POS SH 20W MED	
2A	BJ3PASHT20	J3 FOAM, POS SH 20W TALL	
2B	BJ3PASHS16CU	BASE J3 BK POS SH 16W STD CU	use w/ Cradle-U
2B	BJ3PASHM16CU	BASE J3 BK POS SH 16W MED CU	use w/ Cradle-U
2B	BJ3PASHS18CU	BASE J3 BK POS SH 18W STD CU	use w/ Cradle-U
2B	BJ3PASHM18CU	BASE J3 BK POS SH 18W MED CU	use w/ Cradle-U
2B	BJ3PASHT18CU	BASE J3 BK POS SH 18W TALL CU	use w/ Cradle-U
2B	BJ3PASHS20CU	BASE J3 BK POS SH 20W STD CU	use w/ Cradle-U
2B	BJ3PASHM20CU	BASE J3 BK POS SH 20W MED CU	use w/ Cradle-U
2B	BJ3PASHT20CU	BASE J3 BK POS SH 20W TALL CU	use w/ Cradle-U
3	SJ3PASHS14	J3 SHELL, POS SH 14W STD	
3	SJ3PASHS16	J3 SHELL, POS SH 16W STD	
3	SJ3PASHM16	J3 SHELL, POS SH 16W MED	
3	SJ3PASHS18	J3 SHELL, POS SH 18W STD	
3	SJ3PASHM18	J3 SHELL, POS SH 18W MED	
3	SJ3PASHT18	J3 SHELL, POS SH 18W TALL	
3	SJ3PASHS20	J3 SHELL, POS SH 20W STD	
3	SJ3PASHM20	J3 SHELL, POS SH 20W MED	
3	SJ3PASHT20	J3 SHELL, POS SH 20W TALL	
4	ICJ3PASHS14	J3 BK POS SH 14W STD SH	
4	ICJ3PASHS16	J3 BK POS SH 16W STD SH	
4	ICJ3PASHM16	J3 BK POS SH 16W MED SH	
4	ICJ3PASHS18	J3 BK POS SH 18W STD SH	
4	ICJ3PASHM18	J3 BK POS SH 18W MED SH	
4	ICJ3PASHT18	J3 BK POS SH 18W TALL SH	
4	ICJ3PASHS20	J3 BK POS SH 20W STD SH	
4	ICJ3PASHM20	J3 BK POS SH 20W MED SH	
4	ICJ3PASHT20	J3 BK POS SH 20W TALL SH	

Note: Parts only; for complete Backrest see J3 Back order form.

Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

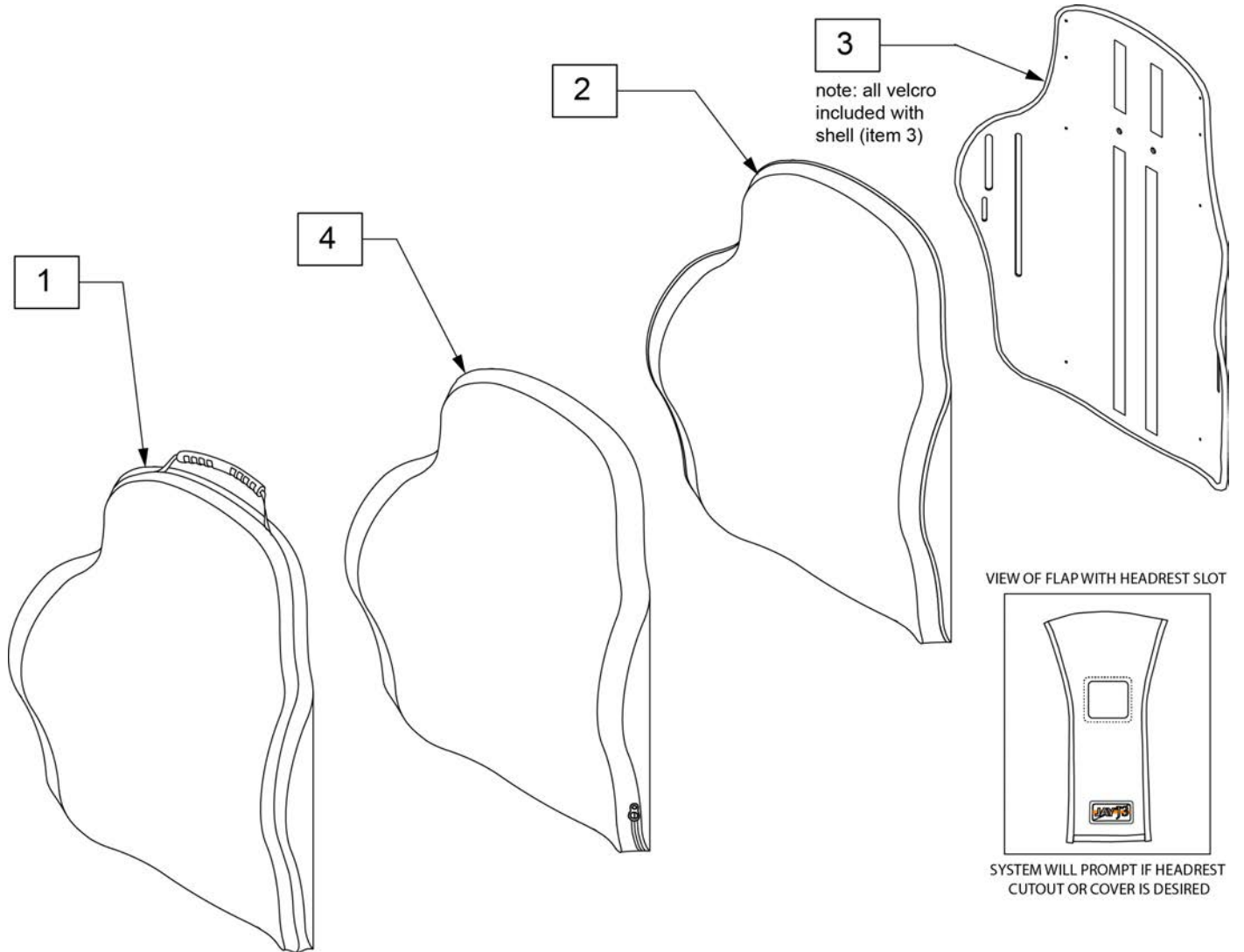


Pos.	Item Number	Description	Remarks
1A	CJ3PLSHS14	J3 CVR, SHAL SH 14W STD	
1A	CJ3PLSHS16	J3 CVR, SHAL SH 16W STD	
1A	CJ3PLSHM16	J3 CVR, SHAL SH 16W MED	
1A	CJ3PLSHS18	J3 CVR, SHAL SH 18W STD	
1A	CJ3PLSHM18	J3 CVR, SHAL SH 18W MED	
1A	CJ3PLSHT18	J3 CVR, SHAL SH 18W TALL	
1A	CJ3PLSHS20	J3 CVR, SHAL SH 20W TALL	
1A	CJ3PLSHM20	J3 CVR, SHAL SH 20W MED	
1A	CJ3PLSHT20	J3 CVR, SHAL SH 20W TALL	
1B	CJ3PLSHS16CU	CVR J3 BK SHAL SH 16W STD CU	use w/ Cradle-U

Pos.	Item Number	Description	Remarks
1B	CJ3PLSHM16CU	CVR J3 BK SHAL SH 16W MED CU	use w/ Cradle-U
1B	CJ3PLSHS18CU	CVR J3 BK SHAL SH 18W STD CU	use w/ Cradle-U
1B	CJ3PLSHM18CU	CVR J3 BK SHAL SH 18W MED CU	use w/ Cradle-U
1B	CJ3PLSHT18CU	CVR J3 BK SHAL SH 18W TALL CU	use w/ Cradle-U
1B	CJ3PLSHS20CU	CVR J3 BK SHAL SH 20W STD CU	use w/ Cradle-U
1B	CJ3PLSHM20CU	CVR J3 BK SHAL SH 20W MED CU	use w/ Cradle-U
1B	CJ3PLSHT20CU	CVR J3 BK SHAL SH 20W TALL CU	use w/ Cradle-U
2A	BJ3PLSHS14	J3 FOAM, SHAL SH 14W STD	
2A	BJ3PLSHS16	J3 FOAM, SHAL SH 16W STD	
2A	BJ3PLSHM16	J3 FOAM, SHAL SH 16W MED	
2A	BJ3PLSHS18	J3 FOAM, SHAL SH 18W STD	
2A	BJ3PLSHM18	J3 FOAM, SHAL SH 18W MED	
2A	BJ3PLSHT18	J3 FOAM, SHAL SH 18W TALL	
2A	BJ3PLSHS20	J3 FOAM, SHAL SH 20W STD	
2A	BJ3PLSHM20	J3 FOAM, SHAL SH 20W MED	
2A	BJ3PLSHT20	J3 FOAM, SHAL SH 20W TALL	
2B	BJ3PLSHS16CU	BASE J3 BK SHAL SH 16W STD CU	use w/ Cradle-U
2B	BJ3PLSHM16CU	BASE J3 BK SHAL SH 16W MED CU	use w/ Cradle-U
2B	BJ3PLSHS18CU	BASE J3 BK SHAL SH 18W STD CU	use w/ Cradle-U
2B	BJ3PLSHM18CU	BASE J3 BK SHAL SH 18W MED CU	use w/ Cradle-U
2B	BJ3PLSHT18CU	BASE J3 BK SHAL SH 18W TALL CU	use w/ Cradle-U
2B	BJ3PLSHS20CU	BASE J3 BK SHAL SH 20W STD CU	use w/ Cradle-U
2B	BJ3PLSHM20CU	BASE J3 BK SHAL SH 20W MED CU	use w/ Cradle-U
2B	BJ3PLSHT20CU	BASE J3 BK SHAL SH 20W TALL CU	use w/ Cradle-U
3	SJ3PLSHS14	J3 SHELL, SHAL SH 14W STD	
3	SJ3PLSHS16	J3 SHELL, SHAL SH 16W STD	
3	SJ3PLSHM16	J3 SHELL, SHAL SH 16W MED	
3	SJ3PLSHS18	J3 SHELL, SHAL SH 18W STD	
3	SJ3PLSHM18	J3 SHELL, SHAL SH 18W MED	
3	SJ3PLSHT18	J3 SHELL, SHAL SH 18W TALL	
3	SJ3PLSHS20	J3 SHELL, SHAL SH 20W STD	
3	SJ3PLSHM20	J3 SHELL, SHAL SH 20W MED	
3	SJ3PLSHT20	J3 SHELL, SHAL SH 20W TALL	
4	ICJ3PLSHS14	J3 BK SHAL SH 14W STD SH	
4	ICJ3PLSHS16	J3 BK SHAL SH 16W STD SH	
4	ICJ3PLSHM16	J3 BK SHAL SH 16W MED SH	
4	ICJ3PLSHS18	J3 BK SHAL SH 18W STD SH	
4	ICJ3PLSHM18	J3 BK SHAL SH 18W MED SH	
4	ICJ3PLSHT18	J3 BK SHAL SH 18W TALL SH	
4	ICJ3PLSHS20	J3 BK SHAL SH 20W STD SH	
4	ICJ3PLSHM20	J3 BK SHAL SH 20W MED SH	
4	ICJ3PLSHT20	J3 BK SHAL SH 20W TALL SH	

Note: Parts only; for complete Backrest see J3 Back order form.

Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

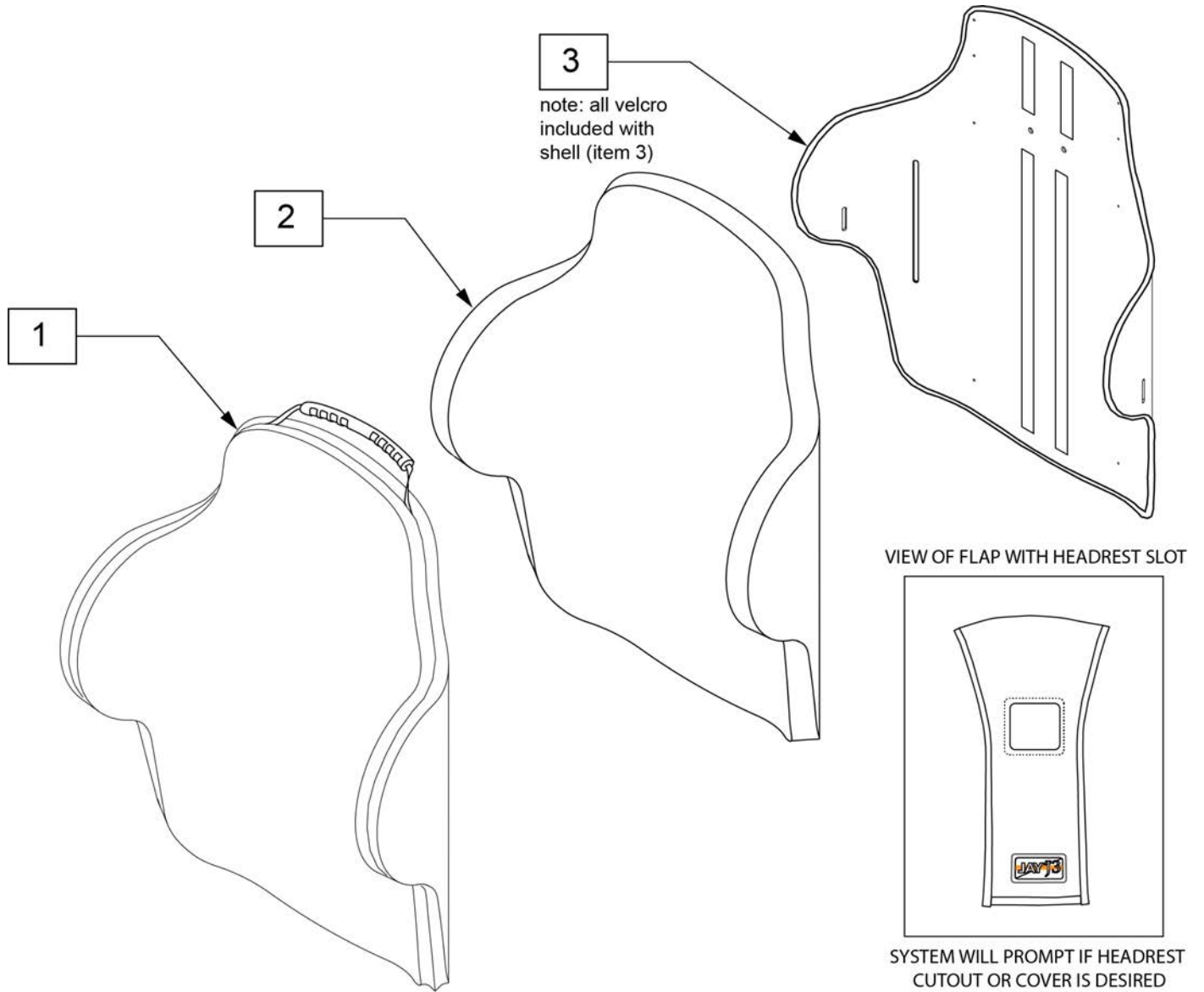


Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJ3PDLSHS14	J3 CVR, DEEP SH 14W STD	1	CJ3PDLSHT20	J3 CVR, DEEP SH 20W TALL
1	CJ3PDLSHS16	J3 CVR, DEEP SH 16W STD	2	BJ3PDLSHS14	J3 FOAM, DEEP SH 14W STD
1	CJ3PDLSHM16	J3 CVR, DEEP SH 16W MED	2	BJ3PDLSHS16	J3 FOAM, DEEP SH 16W STD
1	CJ3PDLSHS18	J3 CVR, DEEP SH 18W STD	2	BJ3PDLSHM16	J3 FOAM, DEEP SH 16W MED
1	CJ3PDLSHM18	J3 CVR, DEEP SH 18W MED	2	BJ3PDLSHS18	J3 FOAM, DEEP SH 18W STD
1	CJ3PDLSHT18	J3 CVR, DEEP SH 18W TALL	2	BJ3PDLSHM18	J3 FOAM, DEEP SH 18W MED
1	CJ3PDLSHS20	J3 CVR, DEEP SH 20W STD	2	BJ3PDLSHT18	J3 FOAM, DEEP SH 18W TALL
1	CJ3PDLSHM20	J3 CVR, DEEP SH 20W MED	2	BJ3PDLSHS20	J3 FOAM, DEEP SH 20W STD

Pos.	Item Number	Description	Pos.	Item Number	Description
2	BJ3PDLSHM20	J3 FOAM, DEEP SH 20W MED	3	SJ3PDLSHT20	J3 SHELL, DEEP SH 20W TALL
2	BJ3PDLSHT20	J3 FOAM, DEEP SH 20W TALL	4	ICJ3PDLSHS14	J3 BK DEEP SH 14W STD SH
3	SJ3PDLSHS14	J3 SHELL, DEEP SH 14W STD	4	ICJ3PDLSHS16	J3 BK DEEP SH 16W STD SH
3	SJ3PDLSHS16	J3 SHELL, DEEP SH 16W STD	4	ICJ3PDLSHM16	J3 BK DEEP SH 16W MED SH
3	SJ3PDLSHM16	J3 SHELL, DEEP SH 16W MED	4	ICJ3PDLSHS18	J3 BK DEEP SH 18W STD SH
3	SJ3PDLSHS18	J3 SHELL, DEEP SH 18W STD	4	ICJ3PDLSHM18	J3 BK DEEP SH 18W MED SH
3	SJ3PDLSHM18	J3 SHELL, DEEP SH 18W MED	4	ICJ3PDLSHT18	J3 BK DEEP SH 18W TALL SH
3	SJ3PDLSHT18	J3 SHELL, DEEP SH 18W TALL	4	ICJ3PDLSHS20	J3 BK DEEP SH 20W STD SH
3	SJ3PDLSHS20	J3 SHELL, DEEP SH 20W STD	4	ICJ3PDLSHM20	J3 BK DEEP SH 20W MED SH
3	SJ3PDLSHM20	J3 SHELL, DEEP SH 20W MED	4	ICJ3PDLSHT20	J3 BK DEEP SH 20W TALL SH


NOTE: Parts only; for complete Backrest see J3 Back order form.

Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

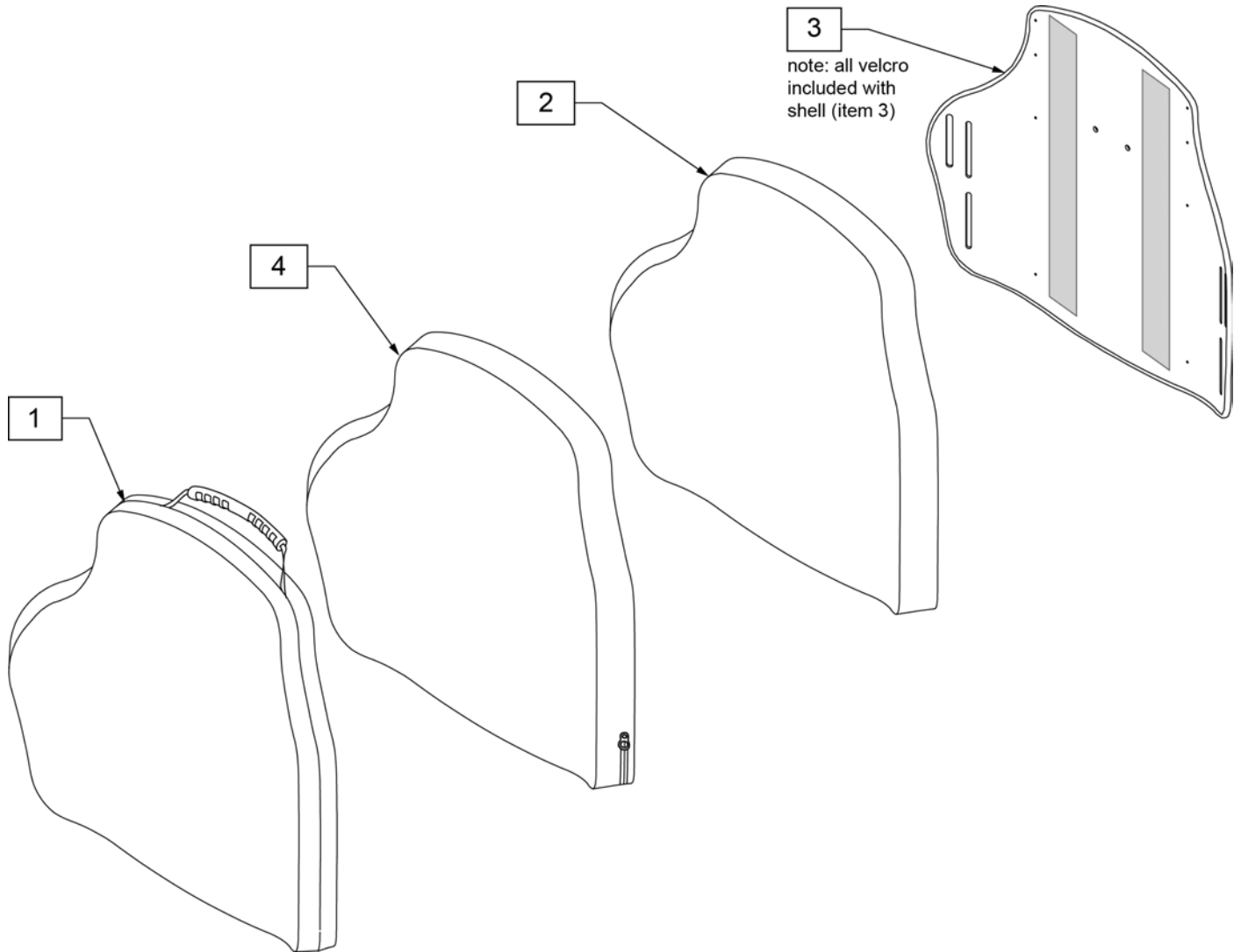

Pos.	Item Number	Description
1	CJ3PASHS22	J3HD CVR POS SH 22W STD
2	BJ3PASHS22	J3HD FOAM POS SH 22W STD
3	SJ3PASHS22	J3HD SHELL POS SH 22W STD
4	ICJ3PASHS22	J3 HD BK,POS SH 22W STD, FS

Note: parts only, for complete Backrest see J3 Back order form.

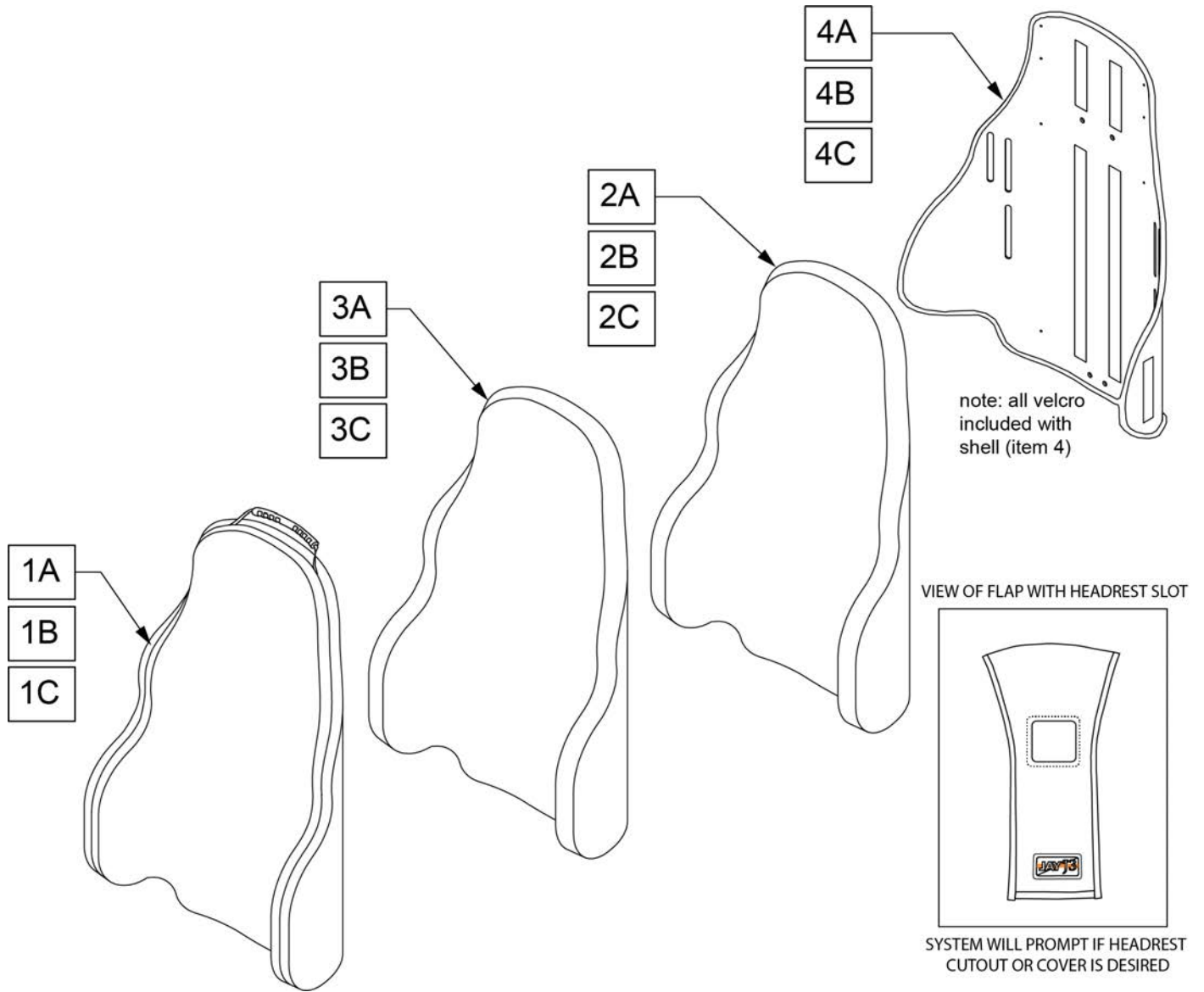


Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1A	CJ3PDSHS14	J3 BK CVR PD SH 14W SHT		2B	BJ3PDSHS20	J3 BK FOAM PD SH 20W SHT	
1A	CJ3PDSHS16	J3 BK CVR PD SH 16W SHT		2B	BJ3PDSHM16	J3 BK FOAM PD SH 16W MED	new
1A	CJ3PDSHS18	J3 BK CVR PD SH 18W SHT		2B	BJ3PDSHM18	J3 BK FOAM PD SH 18W MED	new
1A	CJ3PDSHS20	J3 BK CVR PD SH 20W SHT		2B	BJ3PDSHM20	J3 BK FOAM PD SH 20W MED	new
1B	CJ3PDSHM16	J3 BK CVR PD SH 16W MED	new	2C	BJ3PDSHT18	J3 BK FOAM PD SH 18W TALL	new
1B	CJ3PDSHM18	J3 BK CVR PD SH 18W MED	new	2C	BJ3PDSHT20	J3 BK FOAM PD SH 20W TALL	new
1B	CJ3PDSHM20	J3 BK CVR PD SH 20W MED	new	3A	ICJ3PDSHS14	J3 BK INNER CVR PD SH 14W SHT	
1C	CJ3PDSHT18	J3 BK CVR PD SH 18W TALL	new	3A	ICJ3PDSHS16	J3 BK INNER CVR PD SH 16W SHT	
1C	CJ3PDSHT20	J3 BK CVR PD SH 20W TALL	new	3A	ICJ3PDSHS18	J3 BK INNER CVR PD SH 18W SHT	
2A	BJ3PDSHS14	J3 BK FOAM PD SH 14W SHT		3A	ICJ3PDSHS20	J3 BK INNER CVR PD SH 20W SHT	
2A	BJ3PDSHS16	J3 BK FOAM PD SH 16W SHT		3B	ICJ3PDSHM16	J3 BK INNER CVR PD SH 16W MED	new
2A	BJ3PDSHS18	J3 BK FOAM PD SH 18W SHT		3B	ICJ3PDSHM18	J3 BK INNER CVR PD SH 18W MED	new

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
3B	ICJ3PDSHM20	J3 BK INNER CVR PD SH 20W MED	new	4A	SJ3PDSHS20	J3 BK SHL PD SH 20W SHT	
3C	ICJ3PDSHT18	J3 BK INNER CVR PD SH 18W TALL	new	4B	SJ3PDSHM16	J3 BK SHL PD SH 16W MED	new
3C	ICJ3PDSHT20	J3 BK INNER CVR PD SH 20W TALL	new	4B	SJ3PDSHM18	J3 BK SHL PD SH 18W MED	new
4A	SJ3PDSHS14	J3 BK SHL PD SH 14W SHT		4B	SJ3PDSHM20	J3 BK SHL PD SH 20W MED	new
4A	SJ3PDSHS16	J3 BK SHL PD SH 16W SHT		4C	SJ3PDSHT18	J3 BK SHL PD SH 18W TALL	new
4A	SJ3PDSHS18	J3 BK SHL PD SH 18W SHT		4C	SJ3PDSHT20	J3 BK SHL PD SH 20W TALL	new

NOTE: Parts only; for complete Backrest see J3 Back order form.


Note: If replacing Foam in Jay 3 Backs purchased before August 6, 2013, please order new Cover.

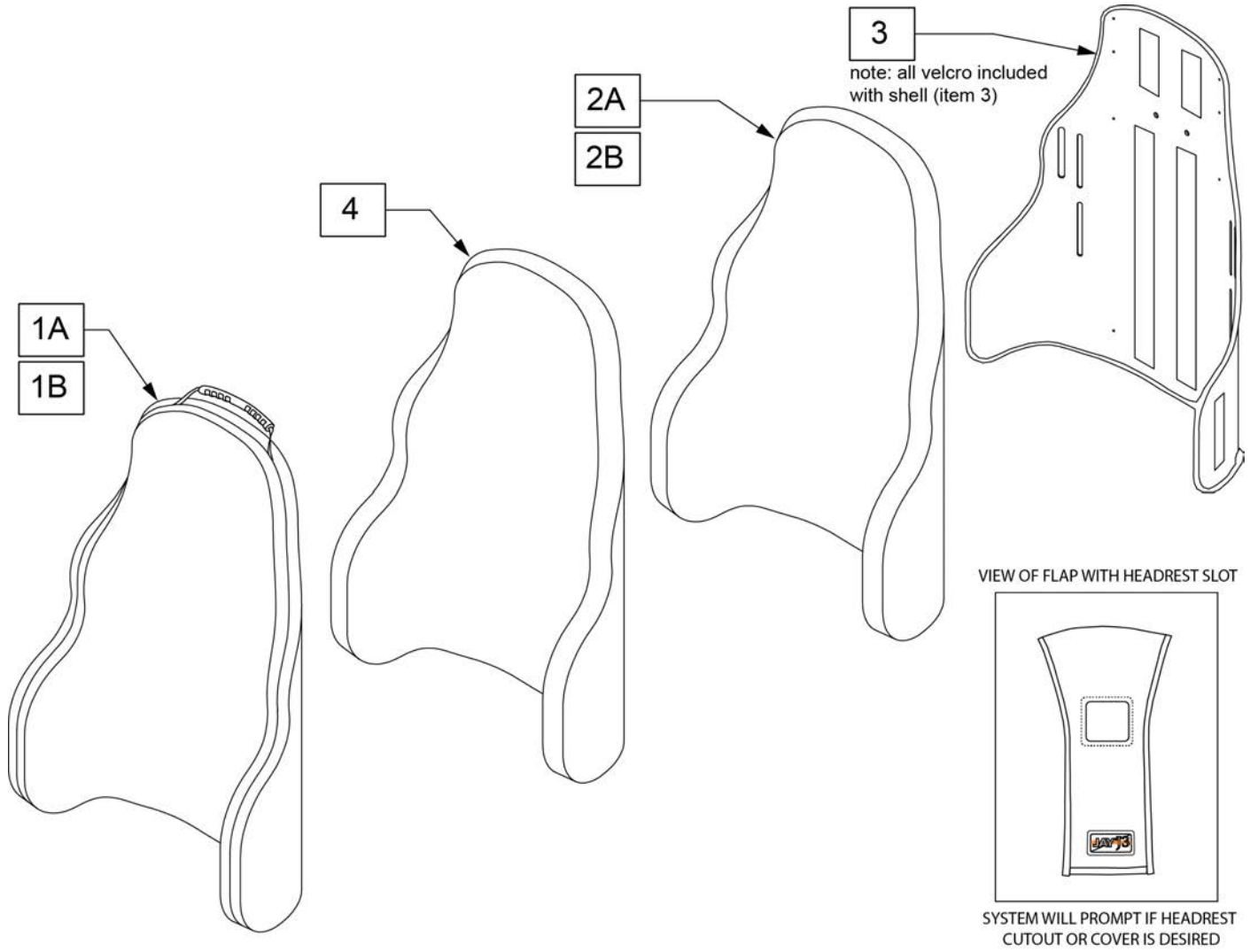

Pos.	Item Number	Description	Remarks
1A	CJ3PDCSHS16	J3 CVR PDC SH 16W STD	
1A	CJ3PDCSHS18	J3 CVR PDC SH 18W STD	
1A	CJ3PDCSHS20	J3 CVR PDC SH 20W STD	
1B	CJ3PDCSHS16CU	CVR J3 BK PDC SH 16W STD CU	use w/ Cradle-U
1B	CJ3PDCSHS18CU	CVR J3 BK PDC SH 18W STD CU	use w/ Cradle-U
1B	CJ3PDCSHS20CU	CVR J3 BK PDC SH 20W STD CU	use w/ Cradle-U
2A	BJ3PDCSHS16	J3 FOAM PDC SH 16W STD	
2A	BJ3PDCSHS18	J3 FOAM PDC SH 18W STD	
2A	BJ3PDCSHS20	J3 FOAM PDC SH 20W STD	
2B	BJ3PDCSHS16CU	BASE J3 BK PDC SH 16W STD CU	use w/ Cradle-U
2B	BJ3PDCSHS18CU	BASE J3 BK PDC SH 18W STD CU	use w/ Cradle-U
2B	BJ3PDCSHS20CU	BASE J3 BK PDC SH 20W STD CU	use w/ Cradle-U
3	SJ3PDCSHS16	J3 SHELL PDC SH 16W STD	
3	SJ3PDCSHS18	J3 SHELL PDC SH 18W STD	
3	SJ3PDCSHS20	J3 SHELL PDC SH 20W STD	
4	ICJ3PDCSHS16	J3 BK PDC SH 16W STD SH	
4	ICJ3PDCSHS18	J3 BK PDC SH 18W STD SH	
4	ICJ3PDCSHS20	J3 BK PDC SH 20W STD SH	

J3 FASTENER KIT GUIDE

Item #	Illustration	Description	Qty
1		WASHER FLAT M6	4
2		SWIVEL PIN THREADED	2
3		WASHER FLAT M5 SS	4
4		SHCS M5 x 0.8 x 25 FULL NP	4
		SHCS M5 x 0.8 x 20 FULL NP	4
5		SHCS M6 x 1 x 22 BLACK	4
6		NUT HEX NYLOCK M6 BLACK	8
7		TWO HOLE WASHER	2
15		HHCS M6 x 1M16 PL ROHS	4

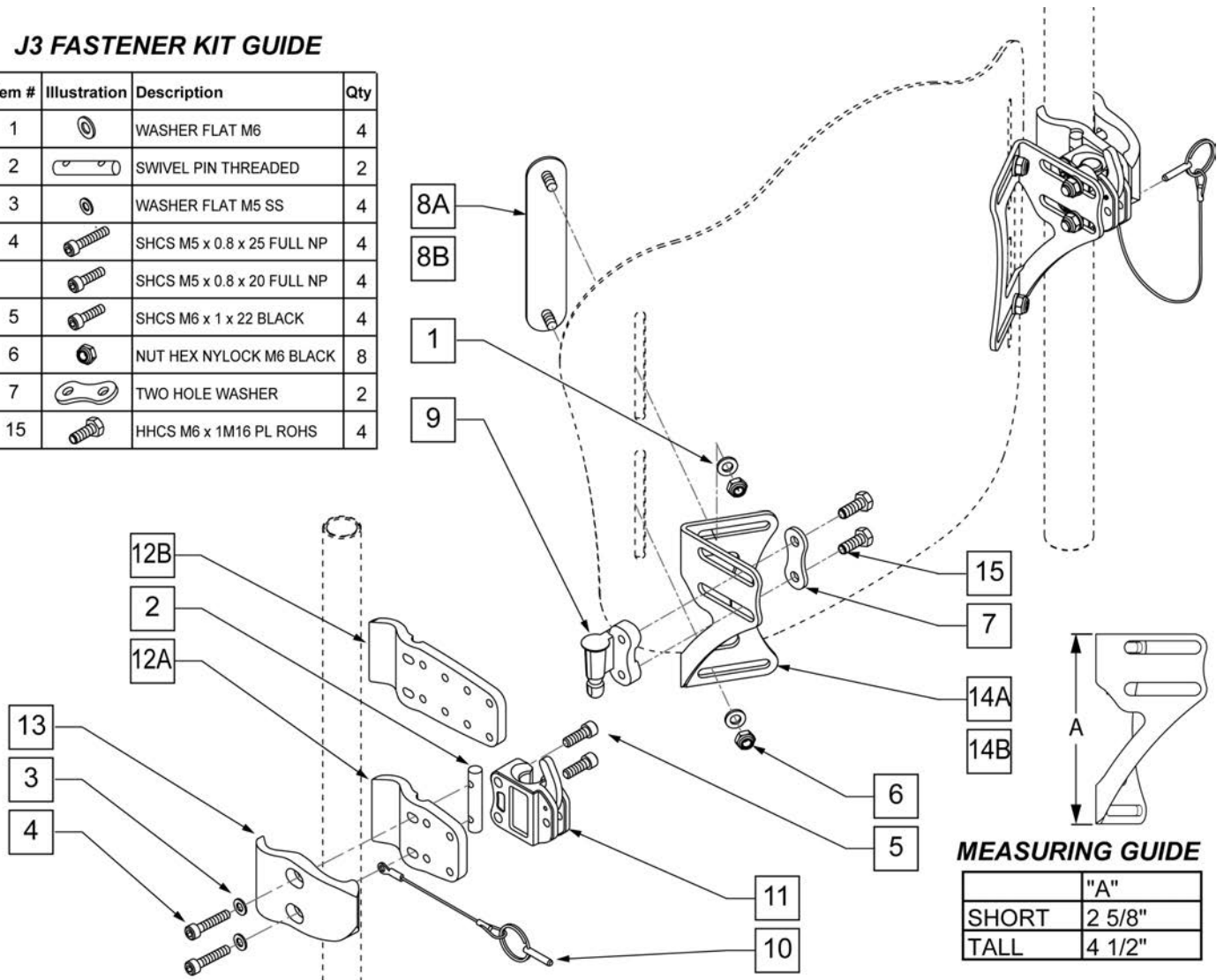


MEASURING GUIDE

	"A"
SHORT	2 5/8"
TALL	4 1/2"

Pos.	Item Number	Description	Remarks
5	J3HDWRM	J3 BK RECEIVER MNTG HDW	Pack of 4
1-7,15	J3HDWF	J3 FASTENER KIT	
8B	J3HDWNT	J3B NUT PLATE TALL	Tall
8A	J3HDWN	J3B NUT PLATE	Short-Lower Thoracic Back only
10	J3LOCKPIN	TRIGGER REL PIN TS/QT/P5	
11	J3HDWLRR	LATCH RECEIVER ASSM, RIGHT	
11	J3HDWLRL	LATCH RECEIVER ASSM, LEFT	
12A,13	J3HDWCS	VARIABLE CLAMP, SHORT	
12B,13	J3HDWCL	VARIABLE CLAMP, LONG	
7,9,15	J3HDWPR	J3B-MOUNTING PIN ASSEMBLY R	
7,9,15	J3HDWPL	J3B-MOUNTING PIN AASSEMBLY L	
2-5,10,11,12A,13	J3SH	J3 STANDARD HARDWARE	
2-5,10,11,12B,13	J3AH	J3 ADJUSTABLE HARDWARE	
1,8B,14B,6	J3HDWTR	J3B-MOUNTING BRACKET, TALL, R	
1,8B,14B,6	J3HDWTL	J3B-MOUNTING BRACKET, TALL, L	
1,8A,14A,6	J3HDWSR	J3B-MOUNTING BRACKET, SHORT, R	Lower Thoracic Back only
1,8A,14A,6	J3HDWSL	J3B-MOUNTING BRACKET, SHORT, L	Lower Thoracic Back only
1-7 ,8A-11,12A,13,14A	117873	J3 MNT STD SHT RETRO KIT	


Pos.	Item Number	Description	Remarks
1-7 ,8A-11,12B,13,14A	117874	J3 MNT ADJ SHT RETRO KIT	
1-7 ,8B-11,12A,13,14B	117871	J3 MNT STD TAL RETRO KIT	
1-7 ,8B-11,12B,13,14B	117872	J3 MNT ADJ TAL RETRO KIT	

20 4 POINT FASTENER KIT			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
1		WASHER FLAT M6	4
2		SWIVEL PIN THREADED	2
3		WASHER FLAT M5 SS	4
4		SHCS, M5X0.8X30, PART, NP	2
5		LOW SHCS M6X1X22 PL BLK	4
6		NUT HEX NYLOCK	8
7		TWO HOLE WASHER	2
15		HHCS M6X1X16 PL	4
16		NUT, JAM 438-20 NYLON	2
17		WASHER, PSIS PIVOT PIN	2
18		PSIS, PIVOT BARIATRIC	2

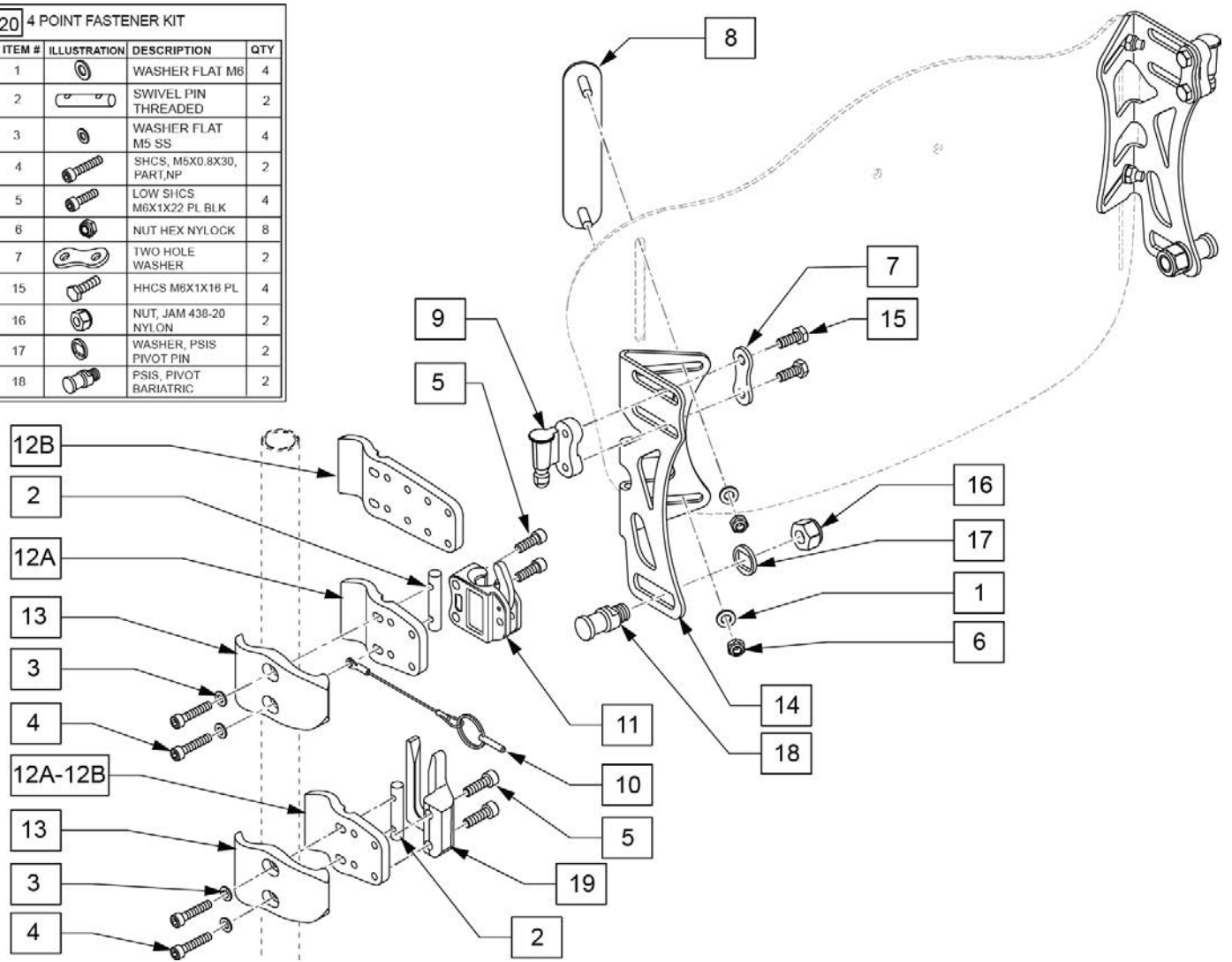


Pos.	Item Number	Description	Remarks
5	J3HDWRM	J3 BK RECIEVER MNTG HDW	PACK OF 4
8	J3HDWNT	J3B NUT PLATE TALL	
10	J3LOCKPIN	TRIGGER REL PIN TS/QT/P5	
11	J3HDWLRR	LATCH RECEIVER ASSM, RIGHT	RIGHT
11	J3HDWLRL	LATCH RECEIVER ASSM, LEFT	LEFT
19	115116	J3 HD BK LOWER RECEIVER LFT	LEFT
19	115117	J3 HD BK LOWER RECEIVER RT	RIGHT
20	117864	4 POINT FASTNER KIT	
7,9,15	J3HDWPR	J3B-Mounting Pin AssblyR	
7,9,15	J3HDWPL	J3B-Mounting Pin AssblyL	
2-4,12A,13,19	117865	J3 LWR CLMP SHT KIT (PR)	PAIR
2-4,12B,13,19	117866	J3 LWR CLMP ADJ KIT (PR)	PAIR
1,6,8,14	117867	4 POINT MNT BRKT TAL R	RIGHT
1,6,8,14	117868	4 POINT MNT BRKT TAL L	LEFT
1-12A,13-19	117869	4 PT MNT STD RETRO KIT	
1-12B,13-19	117870	4 PT MNT ADJ RETRO KIT	

18 J3 RH FASTENER KIT			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
1		WASHER FLAT M6	4
2		BARREL NUT RIG J3 BACK	2
3		WASHER FLAT M5 SS	4
4A		SHCS, M5X0.8X30, PART,NP	4
4B		SHCS, M5X0.8X20, FULL,NP	4
5		NUT HEX NYLOCK	8
6		TWO HOLE WASHER	2
14		LOW SHCS M6X1X22 PL BLK	4
17		HHCS M8X1X16 PL	4


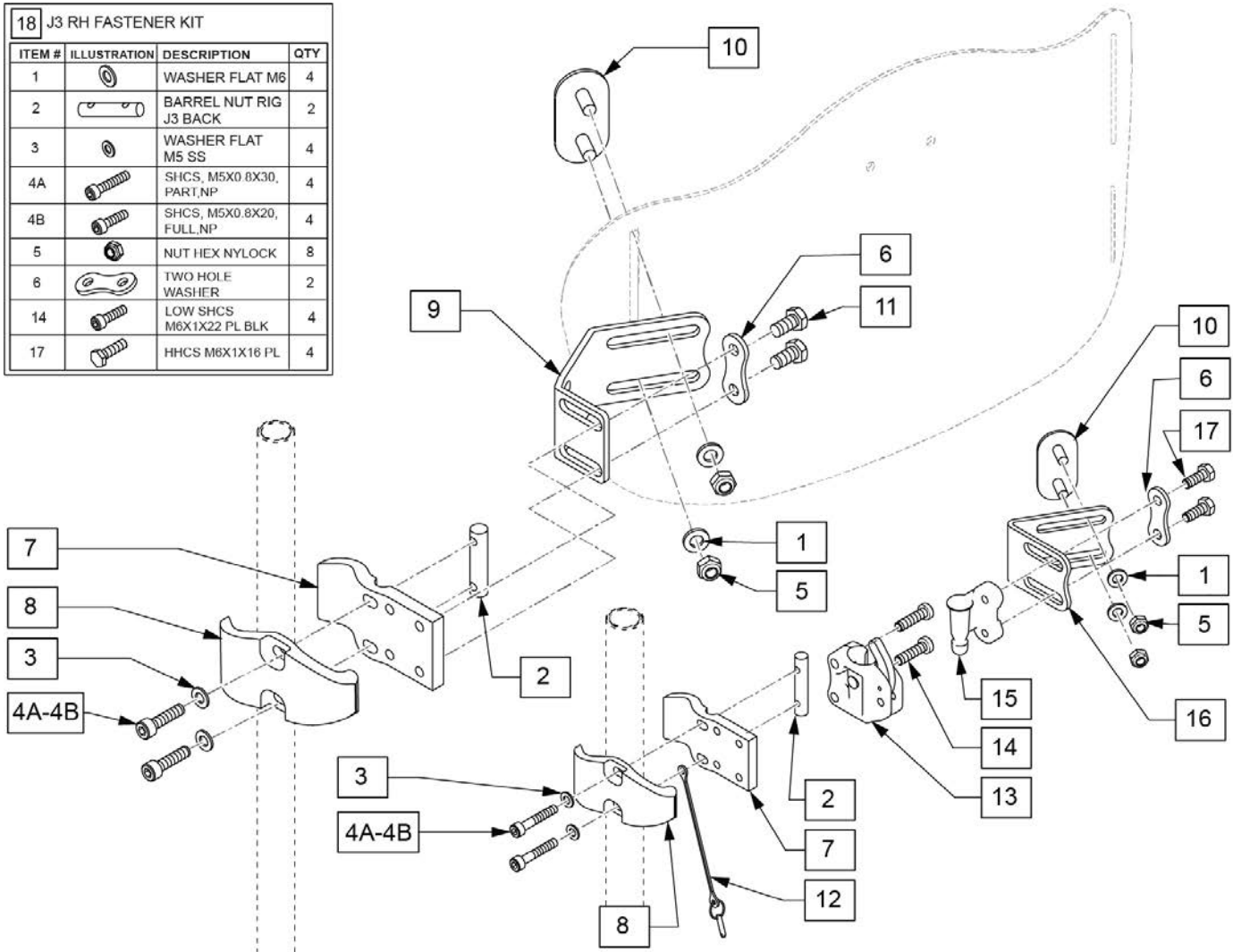


Pos.	Item Number	Description	Remarks
12	J3LOCKPIN	TRIGGER REL PIN TS/QT/P5	
13,14	159398	J3 RH LATCH RECEIVER ASSM RIGHT	RIGHT
13,14	159399	J3 RH LATCH RECEIVER ASSM LEFT	LEFT
1,5,10	159397	J3 RH AND RHF NUT PLATE	1 side
1,5,9,10	159404	J3 RHF MOUNTING BRACKET	1 side
1,5,10,16	159402	J3 RH MOUNTING BRACKET	1 side
6,15,17	159400	J3 MOUNTING PIN ASSM RIGHT	RIGHT
6,15,17	159401	J3 MOUNTING PIN ASSM LEFT	LEFT
1-6,7,8,10,12-17	159395	J3 RIGID HARDWARE (RH)	PAIR
1-11	159403	RIGID HARDWARE FIXED (RHF)	Pair
18	159396	J3 RH FASTENER KIT	for 2 sides

16 J3 TH FASTENER KIT			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
1		WASHER FLAT SAE 1/4 ROHS	4
2		BARREL NUT TILT J3 BACK	2
3		WASHER FLAT M5 SS	4
4A		SHCS, M5X0.8X30, PART, NP	4
4B		SHCS, M5X0.8X20, FULL, NP	4
5		LOW SHCS M6X1X22 PL BLK	4
6		NUT HEX NYLOCK	8
7		WASHER PLATE TILT J3 BACK	2
15		HHCS M6X1X16 PL	4

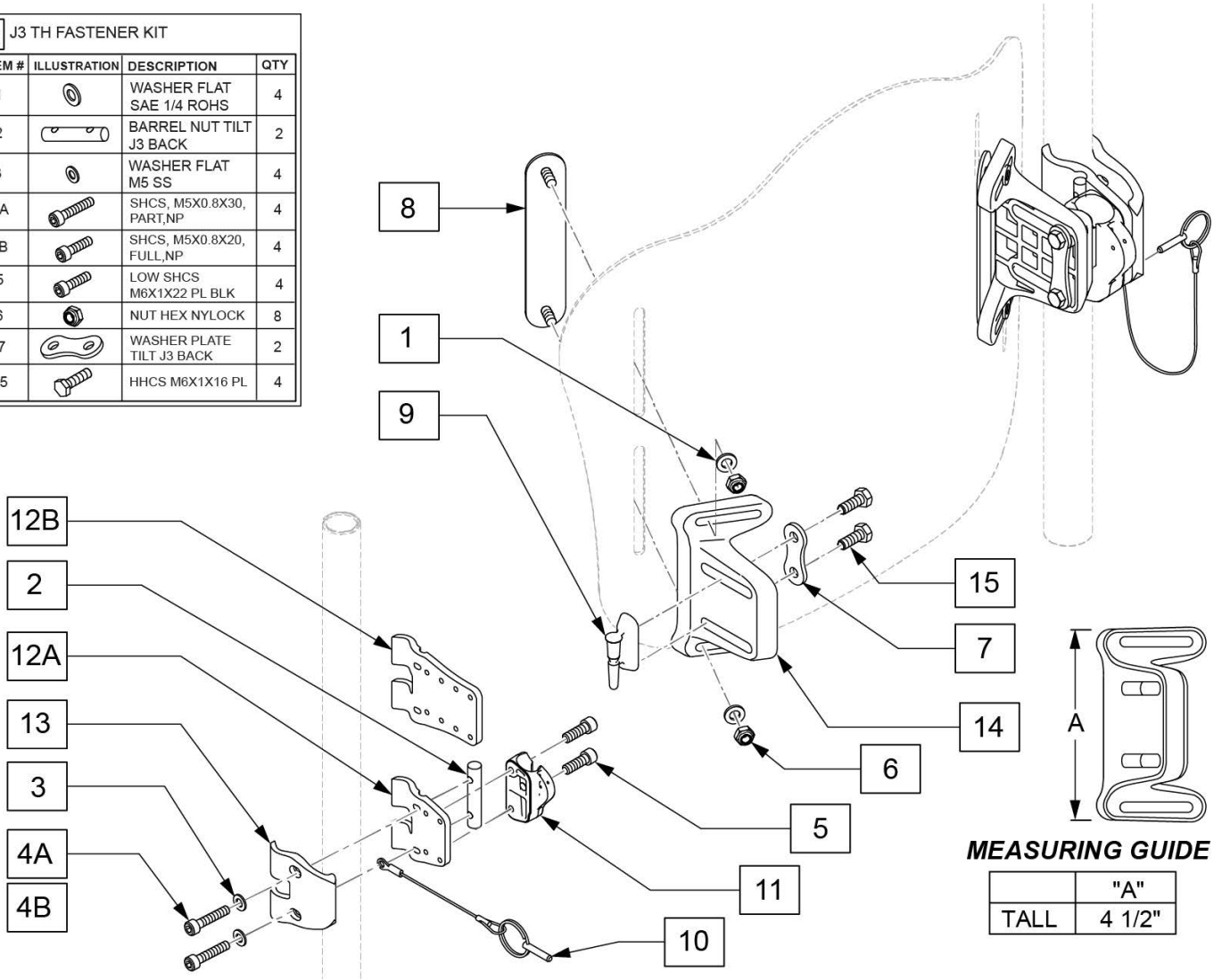

Pos.	Item Number	Description	Remarks
10	J3LOCKPIN	TRIGGER REL PIN TS/QT/P5	
16	159386	J3 TH FASTENER KIT	for 2 sides
1,6,8	159387	J3 TH NUT PLATE	for 1 side
5,11	159389	J3 TH LATCH RECEIVER ASSM RIGHT	RIGHT
5,11	159390	J3 TH LATCH RECEIVER ASSM LEFT	LEFT
7,9,15	159391	J3 TH MOUNTING PIN ASSM	for 1 side
1,6,8,14	159392	J3 TH MOUNTING BRACKET	for 1 side
3-5,12B	159393	J3 TH EXTENDED MOUNTING BRACKET	for 2 sides
3-5,12B,13	159394	J3 TH EXTENDED MOUNTING BRACKET ASSM	for 2 sides
1-15	159385	J3 TILT HARDWARE (TH)	Complete assm left & right

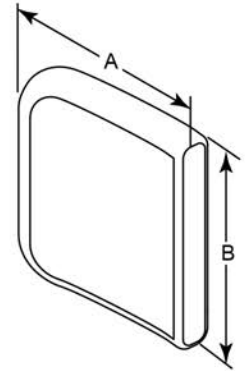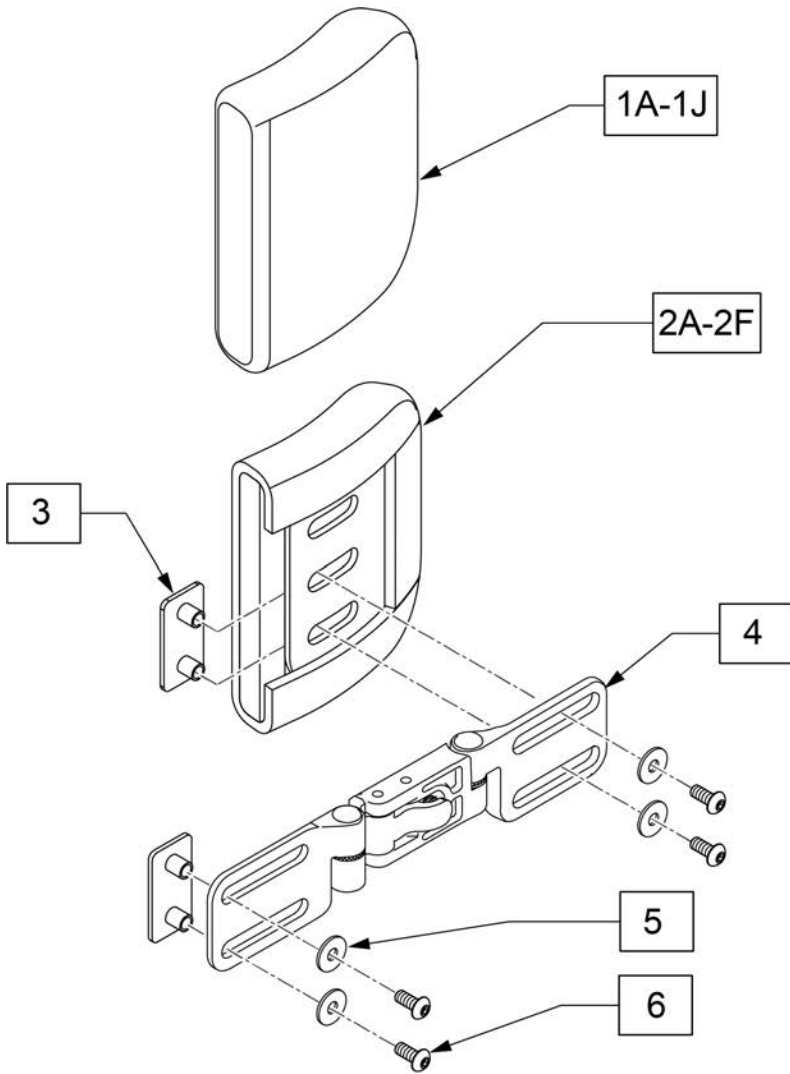

MEASURING GUIDE



DESCRIPTION	DIM A	DIM B
Lateral Cover 4x4	4"	4"
Lateral Cover 4x5	4"	5"
Lateral Cover 4x6	4"	6"
Lateral Cover 6x4	6"	4"
Lateral Cover 6x5	6"	5"
Lateral Cover 6x6	6"	6"
Lateral Base 4x4	4"	4"
Lateral Base 4x5	4"	5"
Lateral Base 4x6	4"	6"
Lateral Base 6x4	6"	4"
Lateral Base 6x5	6"	5"
Lateral Base 6x6	6"	6"

7 J3 S/A-LAT, BAG-O-BOLTS			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
3		NUT PLATE STEEL	2
5		M5 FLAT WASHER	4
6		BHCS, M5X0.8 X12, FULL, PL	4

Pos.	Item Number	Description	Remarks
1A	CJ3LAT4x4P	LATERAL COVER 4x4	
1B	CJ3LAT4x5P	LATERAL COVER 4x5	
1C	CJ3LAT4x6P	LATERAL COVER 4x6	
1D	CJ3LAT6x4P	LATERAL COVER 6x4	
1E	CJ3LAT6x5P	LATERAL COVER 6x5	
1F	CJ3LAT6x6P	LATERAL COVER 6x6	
1A,2A	BJ3LAT4x4P	LATERAL BASE 4x4	Includes cover
1B,2B	BJ3LAT4x5P	LATERAL BASE 4x5	Includes cover
1C,2C	BJ3LAT4x6P	LATERAL BASE 4x6	Includes cover
1D,2D	BJ3LAT6x4P	LATERAL BASE 6x4	Includes cover
1E,2E	BJ3LAT6x5P	LATERAL BASE 6x5	Includes cover
1F,2F	BJ3LAT6x6P	LATERAL BASE 6x6	Includes cover
3-6	256184	JAY SA LAT AND HDW RIGHT	
3-6	256185	JAY SA LAT AND HDW LEFT	
1A,2A,3-6	JLATSA44R	JAY SA LAT 4X4 RH	
1A,2A,3-6	JLATSA44L	JAY SA LAT 4X4 LH	
1B,2B,3-6	JLATSA45L	JAY SA LAT 4X5 LH	
1B,2B,3-6	JLATSA45R	JAY SA LAT 4X5 RH	


Pos.	Item Number	Description	Remarks
1C,2C,3-6	JLATSA46R	JAY SA LAT 4X6 RH	
1C,2C,3-6	JLATSA46L	JAY SA LAT 4X6 LH	
1D,2D,3-6	JLATSA64R	JAY SA LAT 6X4 RH	
1D,2D,3-6	JLATSA64L	JAY SA LAT 6X4 LH	
1E,2E,3-6	JLATSA65R	JAY SA LAT 6X5 RH	
1E,2E,3-6	JLATSA65L	JAY SA LAT 6X5 LH	
1F,2F,3-6	JLATSA66R	JAY SA LAT 6X6 RH	
1F,2F,3-6	JLATSA66L	JAY SA LAT 6X6 LH	
7	J3LATSAP	J3 S/A-LAT. BAG-O-BOLTS	

Note: When retro-fitting the Jay Cradle U on an existing J3 Back please replace the back cover and back foam as the upholstery is wider with this option to ensure lateral plates are covered.

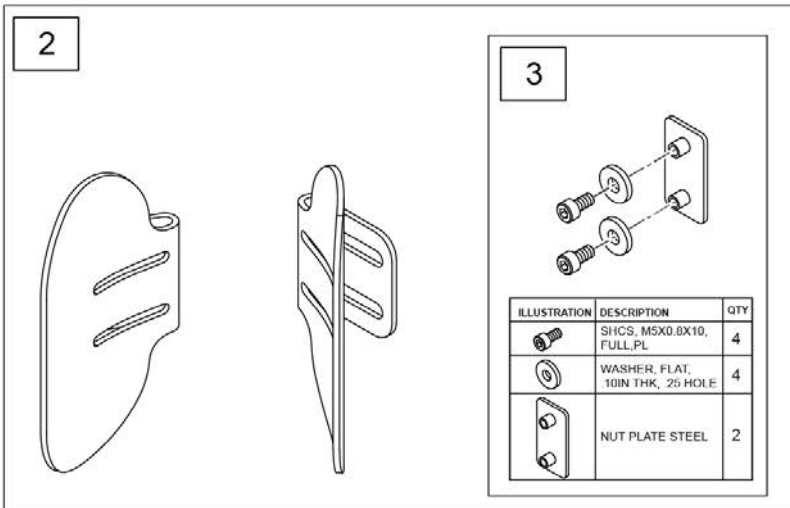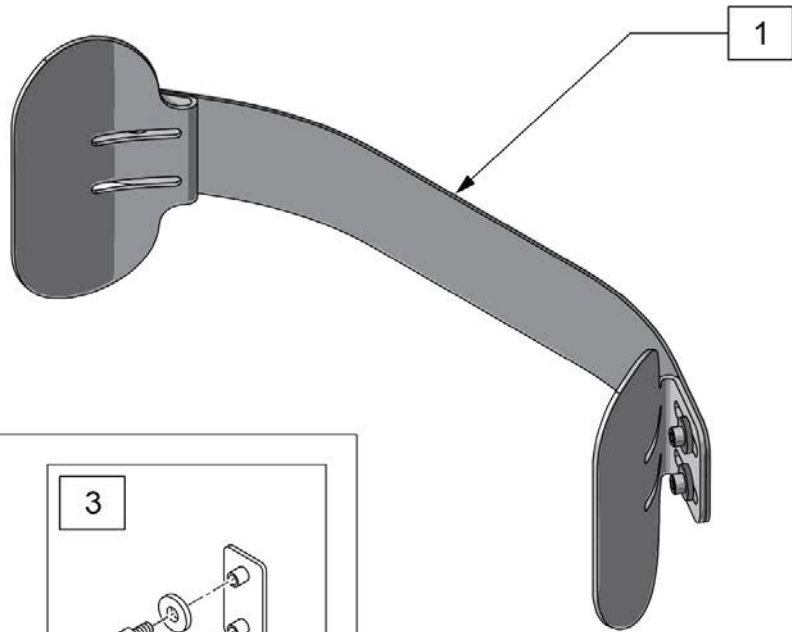


Pos.	Item Number	Description	Remarks
1	CRADLEU16	CRADLE U ASSEMBLY 16 WIDE	complete kit
1	CRADLEU18	CRADLE U ASSEMBLY 18 WIDE	complete kit
1	CRADLEU20	CRADLE U ASSEMBLY 20 WIDE	complete kit
2	126254	CRADLE U PADDLES W/HDWR PAIR	set of pads w/hardware 2 sides
3	126255	CRADLE U HARDWARE PAIR	set of hardware 2 sides

Note: Available prior to September 2010.

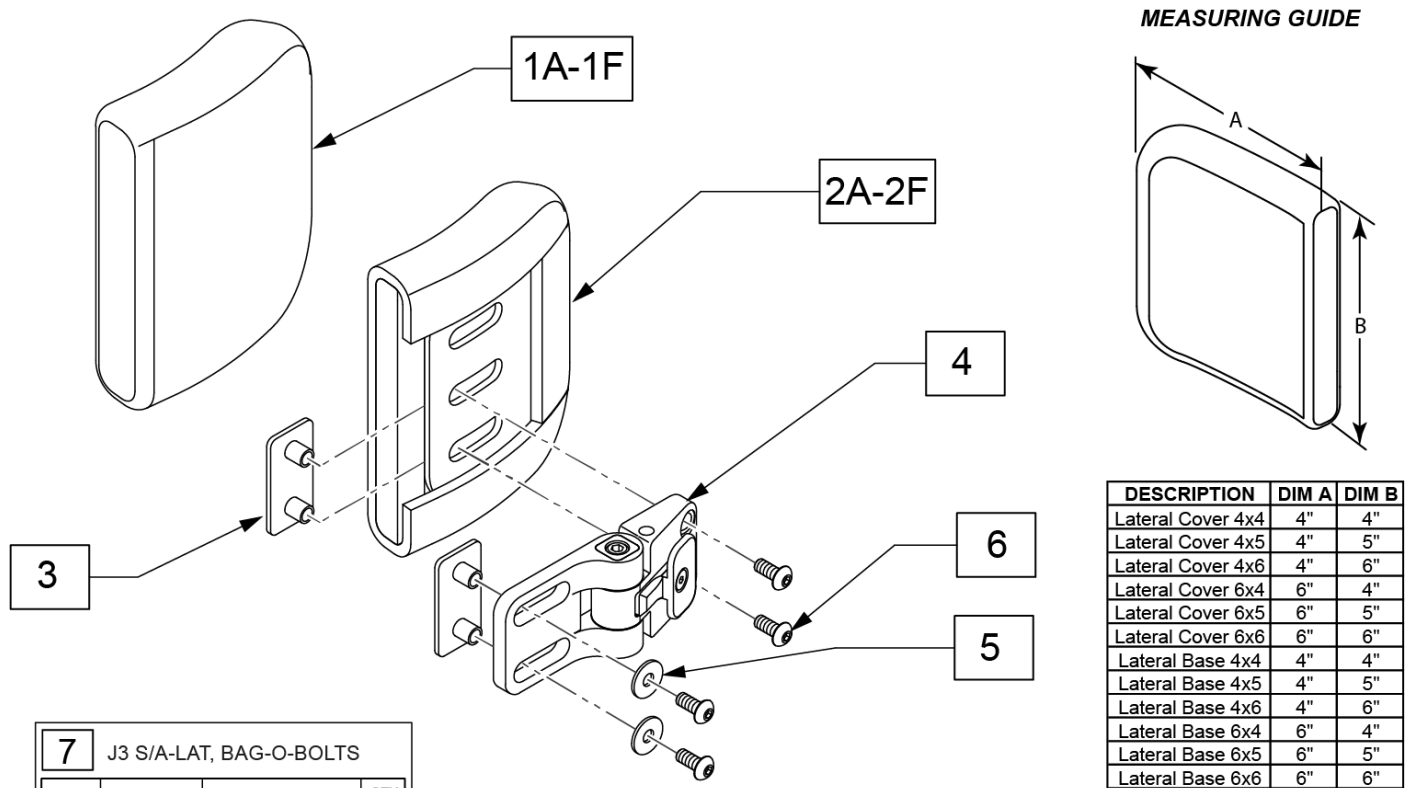


7 J3 S/A-LAT, BAG-O-BOLTS			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
3		NUT PLATE STEEL	2
5		M5 FLAT WASHER	4
6		BHCS, M5X0.8X12, FULL, PL	4

Pos.	Item Number	Description	Remarks
1A	CJ3LAT4x4P	LATERAL COVER 4x4	
1B	CJ3LAT4x5P	LATERAL COVER 4x5	
1C	CJ3LAT4x6P	LATERAL COVER 4x6	
1D	CJ3LAT6x4P	LATERAL COVER 6x4	
1E	CJ3LAT6x5P	LATERAL COVER 6x5	
1F	CJ3LAT6x6P	LATERAL COVER 6x6	
2A	BJ3LAT4x4P	LATERAL BASE 4x4	Includes cover
2B	BJ3LAT4x5P	LATERAL BASE 4x5	Includes cover
2C	BJ3LAT4x6P	LATERAL BASE 4x6	Includes cover
2D	BJ3LAT6x4P	LATERAL BASE 6x4	Includes cover
2E	BJ3LAT6x5P	LATERAL BASE 6x5	Includes cover
2F	BJ3LAT6x6P	LATERAL BASE 6x6	Includes cover
3-6	J3LATSABL	J3LATERAL-S/A BRACKET LH	Left side
3-6	J3LATSABR	J3LATERAL-S/A BRACKET RH	Right side
7	J3LATSAP	J3 S/A-LAT. BAG-O-BOLTS	

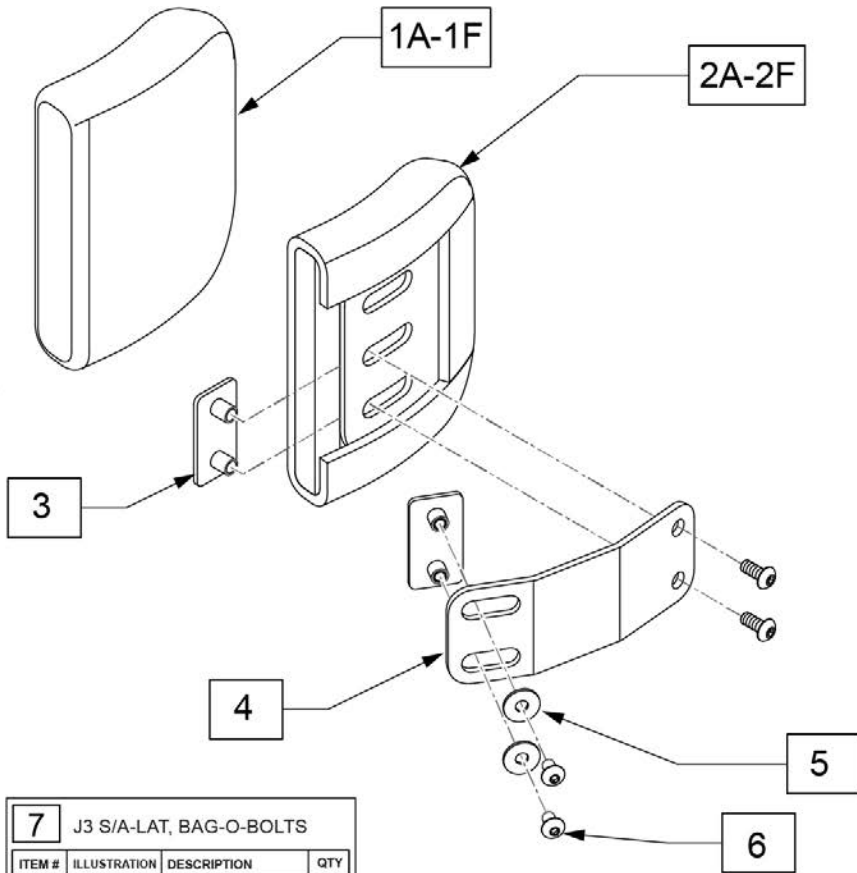


MEASURING GUIDE

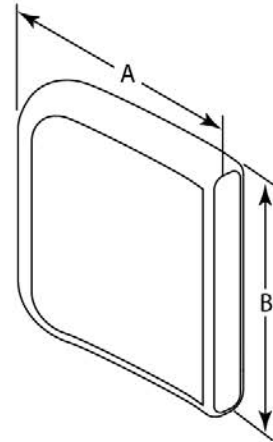

DESCRIPTION	DIM A	DIM B
Lateral Cover 4x4	4"	4"
Lateral Cover 4x5	4"	5"
Lateral Cover 4x6	4"	6"
Lateral Cover 6x4	6"	4"
Lateral Cover 6x5	6"	5"
Lateral Cover 6x6	6"	6"
Lateral Base 4x4	4"	4"
Lateral Base 4x5	4"	5"
Lateral Base 4x6	4"	6"
Lateral Base 6x4	6"	4"
Lateral Base 6x5	6"	5"
Lateral Base 6x6	6"	6"


ITEM #	ILLUSTRATION	DESCRIPTION	QTY
3		NUT PLATE STEEL	2
5		M5 FLAT WASHER	4
6		BHCS, M5X0.8X12, FULL, PL	4

Pos.	Item Number	Description	Remarks
1A	CJ3LAT4x4P	LATERAL COVER 4x4	
1B	CJ3LAT4x5P	LATERAL COVER 4x5	
1C	CJ3LAT4x6P	LATERAL COVER 4x6	
1D	CJ3LAT6x4P	LATERAL COVER 6x4	
1E	CJ3LAT6x5P	LATERAL COVER 6x5	
1F	CJ3LAT6x6P	LATERAL COVER 6x6	
1A,2A	BJ3LAT4x4P	LATERAL BASE 4x4	Includes cover
1B,2B	BJ3LAT4x5P	LATERAL BASE 4x5	Includes cover
1C,2C	BJ3LAT4x6P	LATERAL BASE 4x6	Includes cover
1D,2D	BJ3LAT6x4P	LATERAL BASE 6x4	Includes cover
1E,2E	BJ3LAT6x5P	LATERAL BASE 6x5	Includes cover
1F,2F	BJ3LAT6x6P	LATERAL BASE 6x6	Includes cover
3-6	J3LATFB	J3LATERAL-FXD BRACKET	
7	J3LATFP	J3 FIXED-LAT. BAG-O-BOLTS	

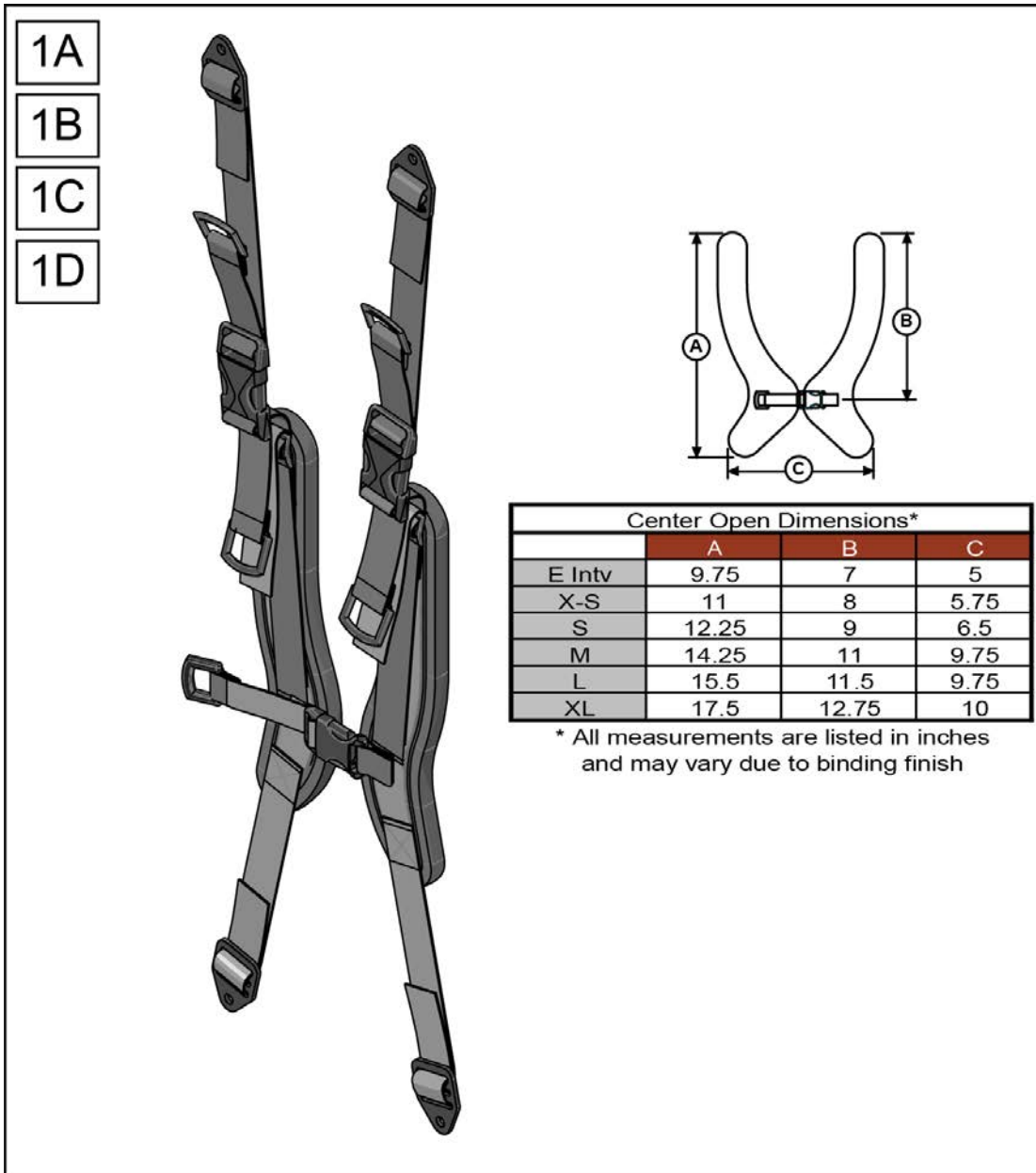


Pos.	Item Number	Description
1A	ATS9401	COMFORFIT, CENTER OPENING, SM
1B	ATS9402	COMFORFIT, CENTER OPENING, MED
1C	ATS9403	COMFORFIT, CENTER OPENING, LG
1D	ATS9404	COMFORFIT, CENTER OPENING, XL

- 1A
- 1B
- 1C
- 1D

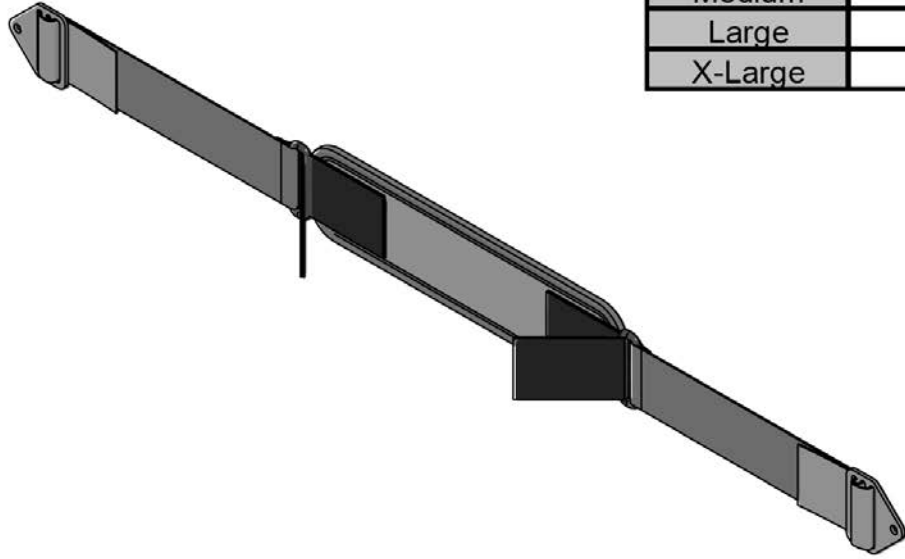


Trunk Strap Pad Dimensions		
	Height	Width
X-Small	3"	8"
Small	3"	10"
Medium	3"	12"
Large	4"	14"
X-Large	4"	16"

Pos.	Item Number	Description
1A	ATS4000	PADDED ANTERIOR TRUNK STRAP SM
1B	ATS4100	PADDED ANTERIOR TRUNK STRAP MD
1C	ATS4200	PADDED ANTERIOR TRUNK STRAP LG
1D	ATS4300	PADDED ANTERIOR TRUNK SPRAP XL

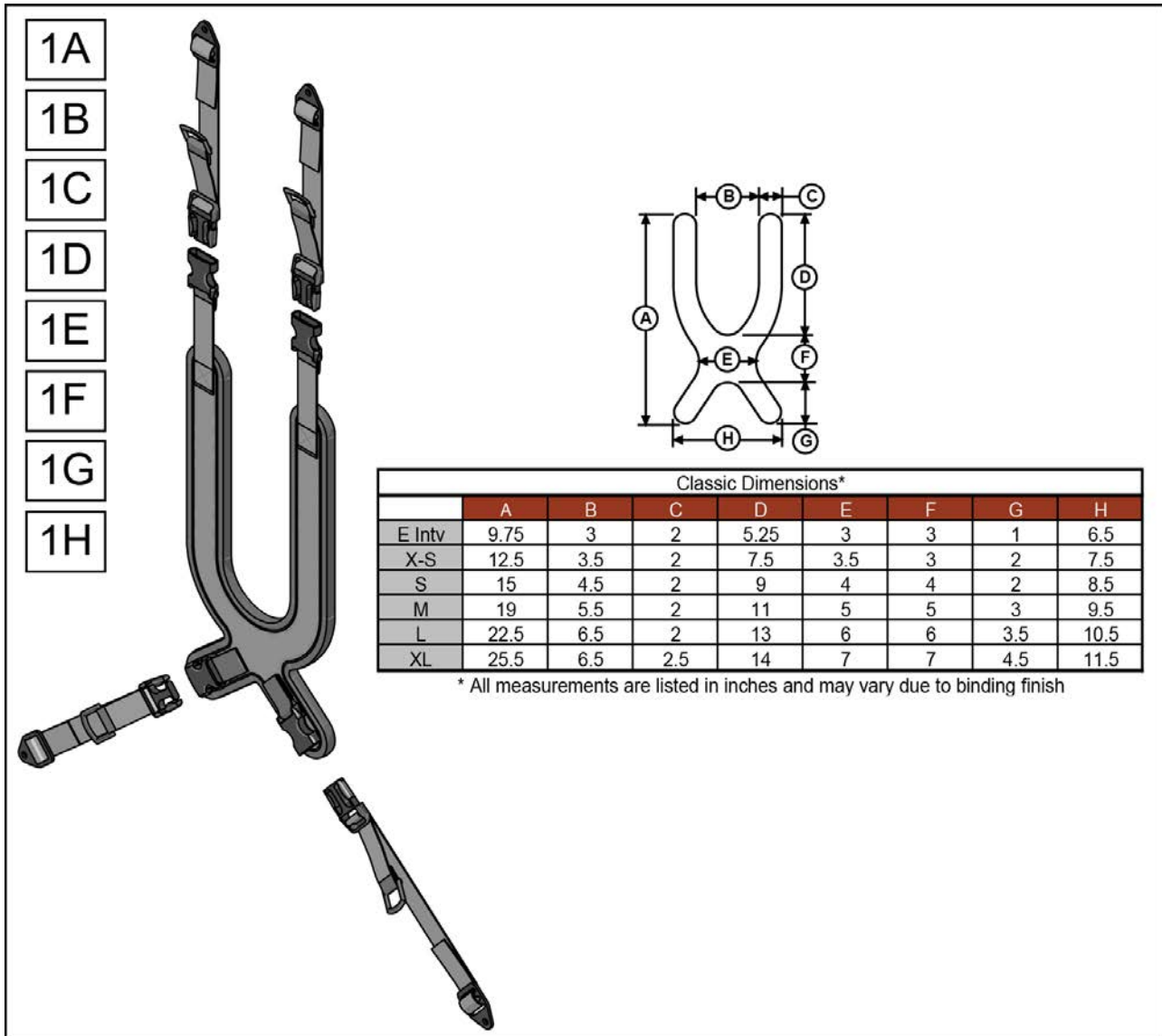

Pos.	Item Number	Description
1A	ATS8101	COMFORFIT PULL-DOWN CLASSIC SM
1B	ATS8102	COMFORFIT PULL-DOWN CLASSIC MD
1C	ATS8103	COMFORFIT PULL-DOWN CLASSIC LG
1D	ATS8104	COMFORFIT PULL-DOWN CLASSIC XL
1E	ATS8301	COMFORFIT STRUCT CLS PD SM
1F	ATS8302	COMFORFIT STRUCT CLS PD MED
1G	ATS8303	COMFORFIT STRUCT CLS PD LG
1H	ATS8304	COMFORFIT STRUCT CLS PD XL


- 1A
- 1B
- 1C
- 1D
- 1E
- 1F
- 1G
- 1H

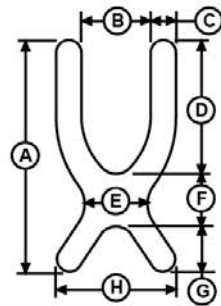

Contour Dimensions*								
	A	B	C	D	E	F	G	H
E Intv	9.75	3	1.75	5.25	1	3.5	1.5	6.5
X-S	12.5	3.5	2	7.5	1	3.5	1.5	7.5
S	15	4.5	2	9	1	4	2	8.5
M	18.5	5.5	2	11	1	5	2.5	9.5
L	22.5	6.5	2	13	1.5	6	3.5	10.5
XL	25.5	6.5	2.5	14	1.5	7	4.5	11.5

* All measurements are listed in inches and may vary due to binding finish


Pos.	Item Number	Description
1A	ATS9101	COMFORFIT PULL-DOWN CONTOUR SM
1B	ATS9102	COMFORFIT PULL-DOWN CONTOUR MD
1C	ATS9103	COMFORFIT PULL-DOWN CONTOUR LG
1D	ATS9104	COMFORFIT PULL-DOWN CONTOUR XL
1E	ATS9301	COMFORFIT, STRUCT, CTR PD SM
1F	ATS9302	COMFORFIT, STRUCT, CTR PD MED
1G	ATS9303	COMFORFIT, STRUCT, CTR PD LG
1H	ATS9304	COMFORFIT, STRUCT, CTR PD XL

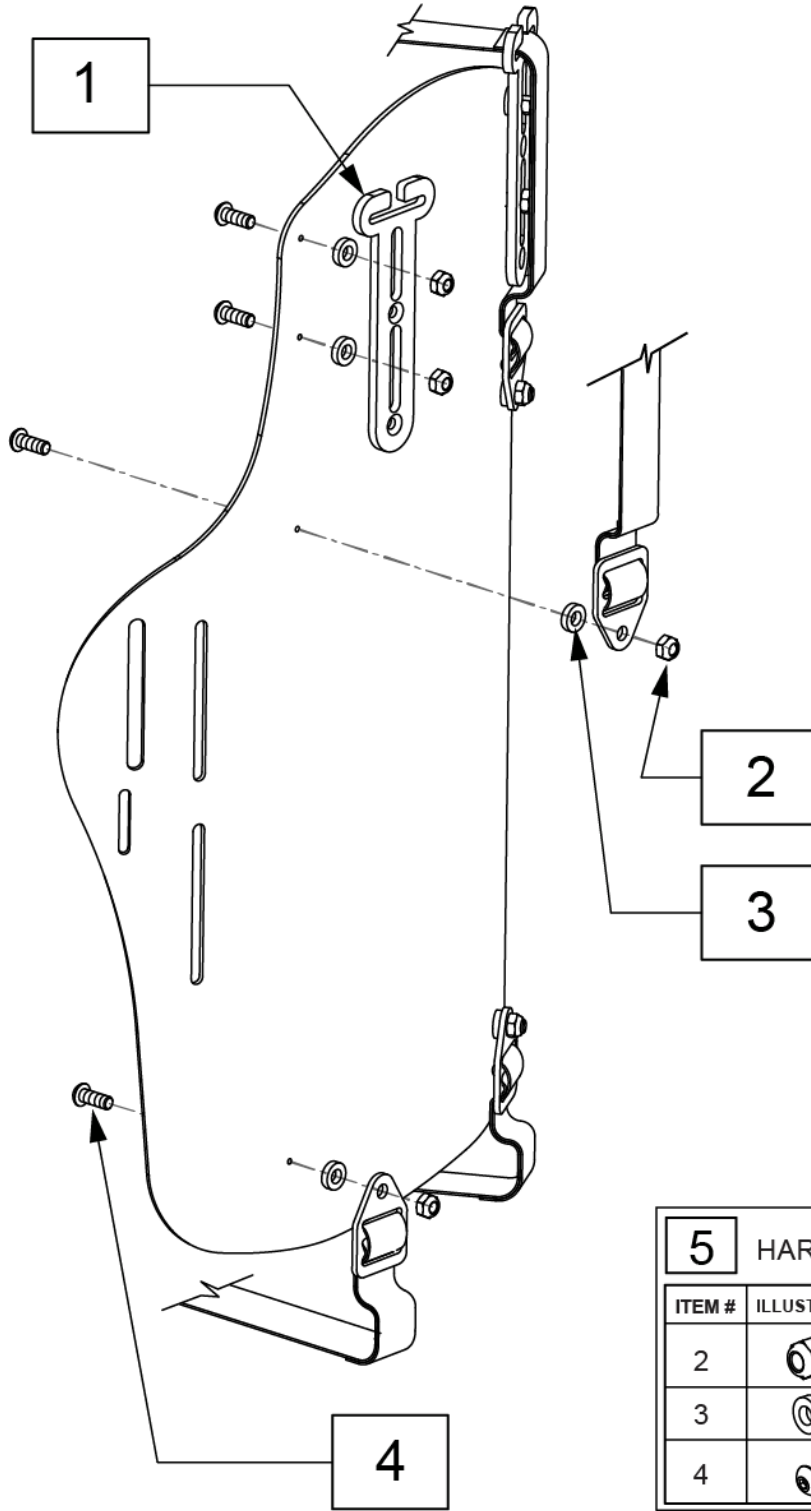

5 HARNESS BAG-O-BOLTS			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
2		NUT HEX NYLOCK	8
3		SPACER NYLON	8
4		BHCS, M6X1X16, FULL,NP	8



Pos.	Item Number	Description
1	J3GPGS	SHOULDER STRAP GUIDES
5	J3HHP	HARNESS BAG-O-BOLTS

Note: if using the Headrest Post on a Quicke Power Chair, please refer to the HEADREST CLAMPS page located in the appropriate power chair parts manual.

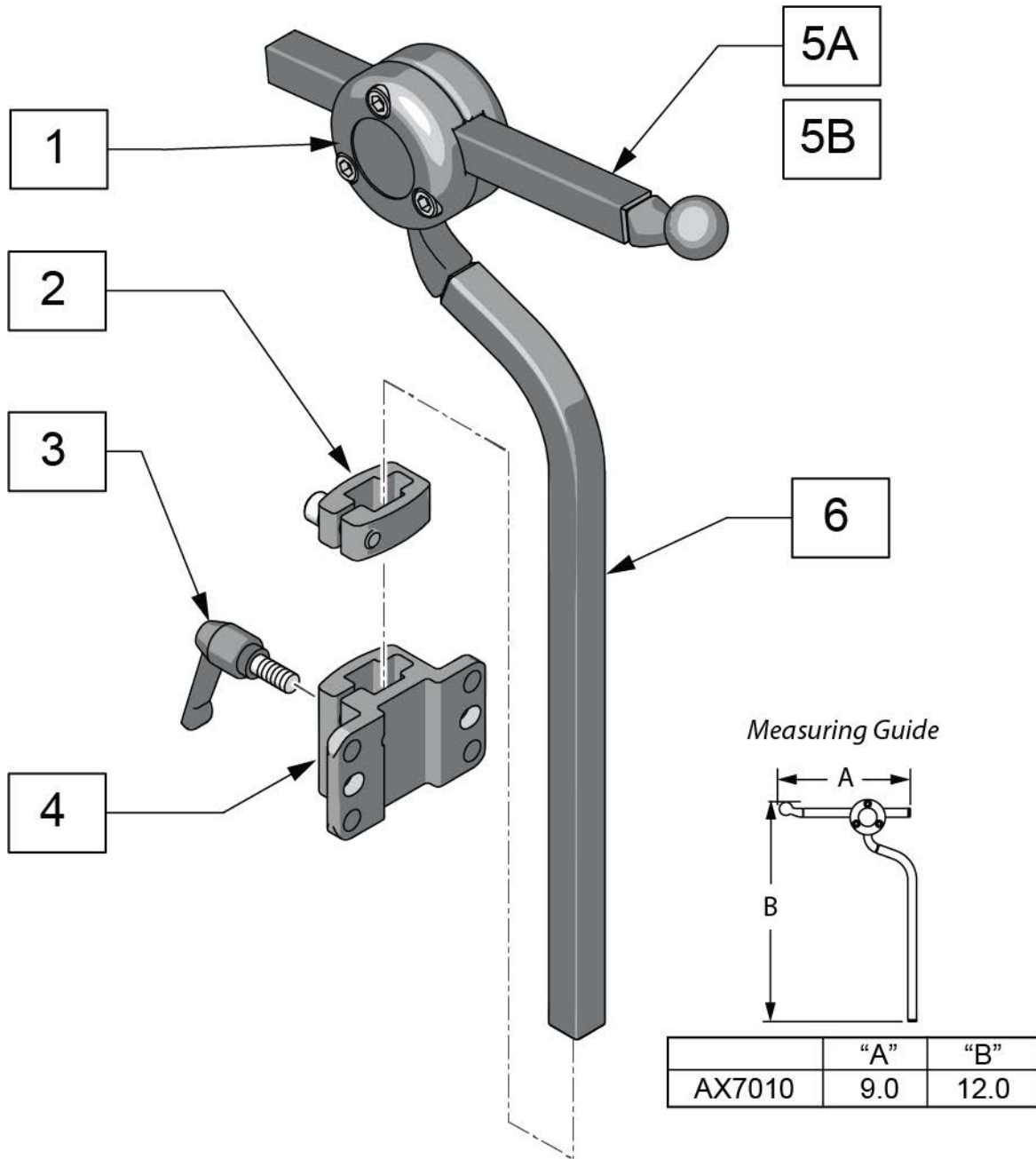


Pos.	Item Number	Description
5A	AX5080	BALLROD AXYS 6" SQUARE BLACK
5B	AX6055	AXYS 12" SQ HORIZ BAR BLK
6	MW070300	AXYS VERT ROD BLK
3,4	115215	J3 BK STD HD RST BRKT KIT
2-4	M2500	AXYS DETACH BRKT
1,5A,6	AX7010	AXYS CORE MOUNT HDWR

Note: if using the Headrest Post on a Quicke Power Chair, please refer to the HEADREST CLAMPS page located in the appropriate power chair parts manual.

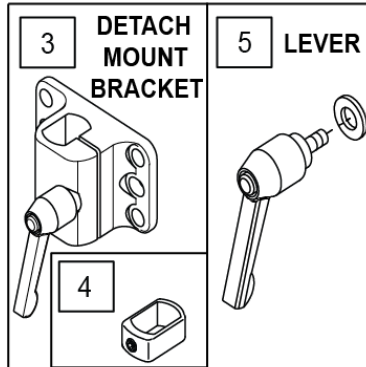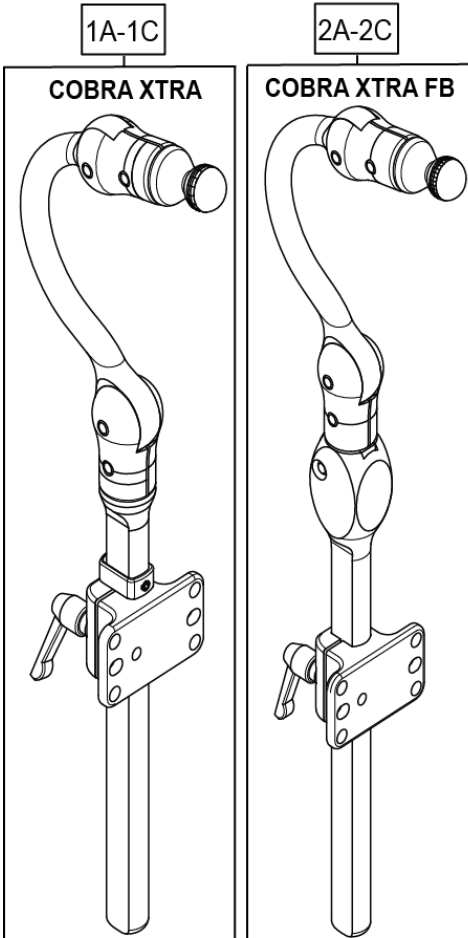

Measuring Guide

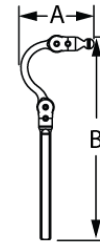

SIZE	P/N	"A"	"B"
ADULT	CBRA7010	5.0	15.0
	CBRA5010	5.0	15.0
PEDIATRIC	CBRA7030	4.5	11.0
	CBRA5030	4.5	11.0
EARLY INTERVENTION	CBRA7080	4.0	10.0
	CBRA5080	4.0	10.0

6 HARDWARE MOUNT KIT

ILLUSTRATION	DESCRIPTION	QTY
	WASHER 1/4" SAE BLK	2
	BOLT 1/4-20 X 3/4" SHCS BLK NI	2
	1/4-20 NYLOK	2
	1/4-20 X 3/8" T-NUT THF	2

7 HARDWARE MOUNT KIT (SLIDE & SWINGAWAY)

ILLUSTRATION	DESCRIPTION	QTY
	WASHER 1/4" SAE BLK	4
	BOLT 1/4-20 X 1/2" SHCS BLK	2
	BOLT 1/4-20X1-1/4" SHCS BLK NI	4
	1/4-20 NYLOK	4
	1/4-20 X 3/8" T-NUT THF	4

Pos.	Item Number	Description	Remarks
1A	CBRA7010	COBRA XTRA MOUNT HDW, ADUL	
1B	CBRA7030	COBRA XTRA MOUNT HDW, PED	
1C	CBRA7080	COBRA XTRA MOUNT HDW, EI	
2A	CBRA5030	COBRA XTRA FB MOUNT HDW, PED	
2B	CBRA5010	COBRA XTRA FB MOUNT HDW, ADUL	
2C	CBRA5080	COBRA XTRA FB MOUNT HDW, EI	
3	M2400-WB	COBRA XTRA DETACH BRKT	INCLUDES QTY 2 1/4-20 X 1/2" SHCS & WASHERS
4	MW070152	COBRA VERT STOP BLK	
5	M0030	KNOB PLAS ADJ LVR 1/4-20 X20mm	
6	M0055	FASTENER PACK DETACH	
7	M0075	FASTENER PACK SWINGAWAY MOUNT	

Note: if using the Headrest Post on a Quicke Power Chair, please refer to the HEADREST CLAMPS page located in the appropriate power chair parts manual.


Measuring Guide

	"A"	"B"	"C"
LX7000	3.75	6.5	10.0
LX8000	3.75	6.5	8.0
LX7804	3.75	6.5	4.0
LX7815	3.75	6.5	15.0

Pos.	Item Number	Description	Remarks
1A	LX7804	LINX CORE STDHRZ/4" VRT MOUNT	4" vertical post
1B	LX8000	LINX CORE MNTG HDW 8"	8" vertical post
1C	LX7000	LINX CORE MOUNT HDW	10" vertical post
1D	LX7815	LINX CORE STDHRZ/15" VRT MOUNT	15" vertical post
1E	LX7000Q	LINX CORE MOUNT HDW QUICK ADJ	
2	MW000680	CAP RATCHET FASTENER BLACK	
3	MW070400	LYNX SYS ASSM 2"	
4	MW006030	.875 SHAFT COLLAR W/ PIN	
5	MC100-WB	COVER 3/4" COLLAR/VINYL BLACK	
6	M0030	KNOB PLAS ADJ LVR 1/4-20 X20mm	
7	M2300	LINX DETACH MOUNT BRKT	
8	M2375	MOUNT FIXED LINX 7/8" SURFACE	
9	M0055	FASTENER PACK DETACH	
10	M0075	FASTENER PACK SWINGAWAY MOUNT	
11A	LX27804	LINX2 CORE MOUNT HDW 4IN	
11B	LX28000	LINX2 CORE MOUNT HDW 8IN	


Pos.	Item Number	Description	Remarks
11C	LX27000	LINX2 CORE MOUNT HDW 10IN	
11D	LX27815	LINX2 CORE MOUNT HDW 15IN	
11E	LX27000Q	LINX2 CORE MNT HDW 10 QCK ADJ	

Note: Please order Mounting Bracket separately.

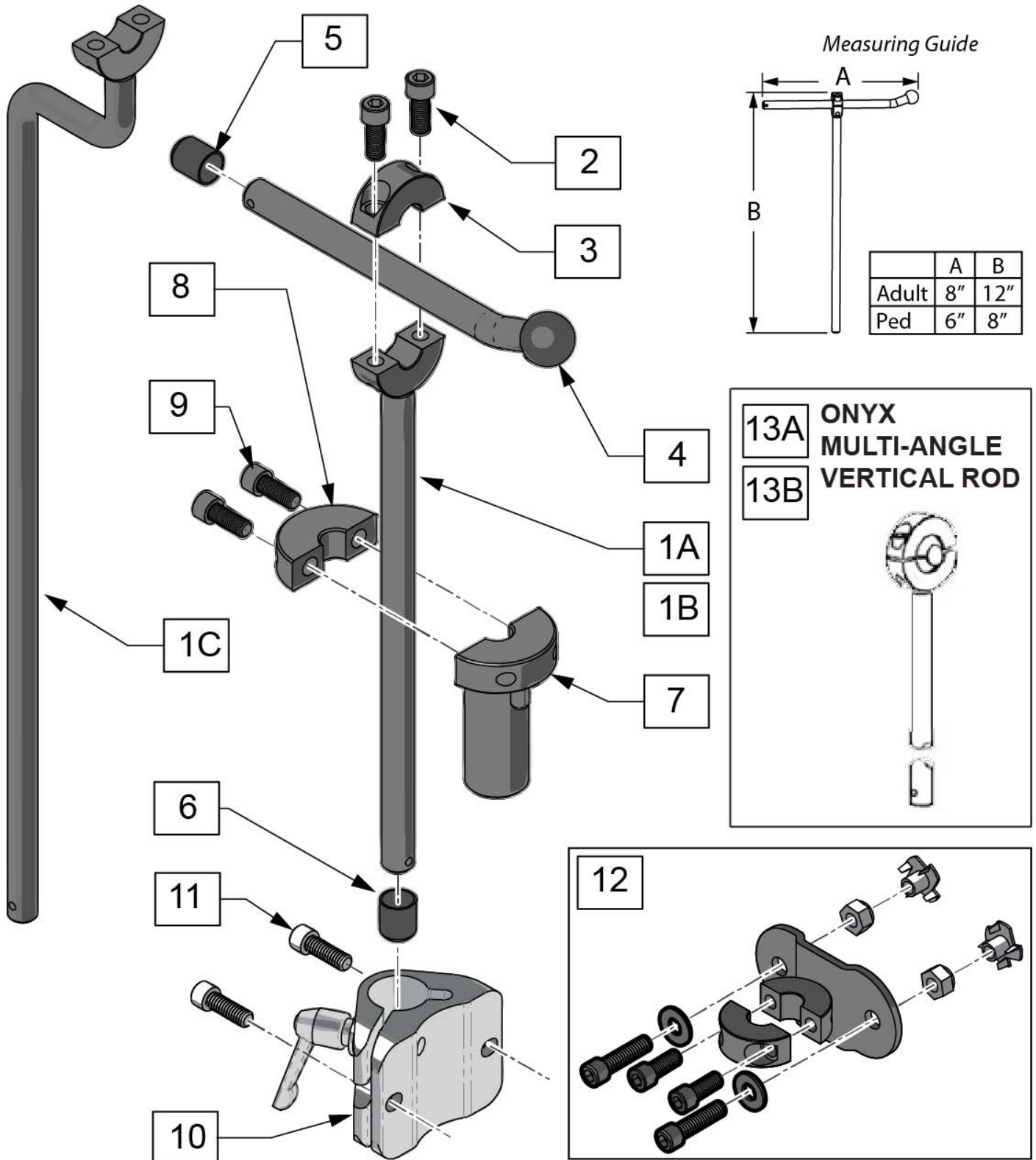

Pos.	Item Number	Description
1A	OX6012	VERTICAL ROD 12" ONYX
1B	OX6008	BALLROD ONYX 8" VERTICAL
1C	OX6030	VERT ROD, 17.5" ONYX W/OFF-SET
4	MW004605	BALLROD 8" FIXD 3/4 W/20DG, BLK
5	MW000800	ENDCAP 7/16 ID VINYL BLK
12	M2175	MOUNT FIXED ONYX 7/16
13A	OX6035	MULTI-ANG VERT ROD, 12" ONYX
13B	OX6050	VERTICAL ROD 12"SQUARE ONYX
1A,2-6	OX7010	ONYX CORE MOUNT HDW, ADULT
1B,2-6	OX7030	ONYX CORE MOUNT HDW, PED
1C,2-6	OX6010	ONYX CORE OFFSET BAR MNTG HDW
7-11	M2100-WB	ONYX DETACH MOUNT BRKT

Note: item 5 used with Linx and Axy's headrest posts only; plate is used to space headrest mount away from back upholstery to allow for easier headrest removal.

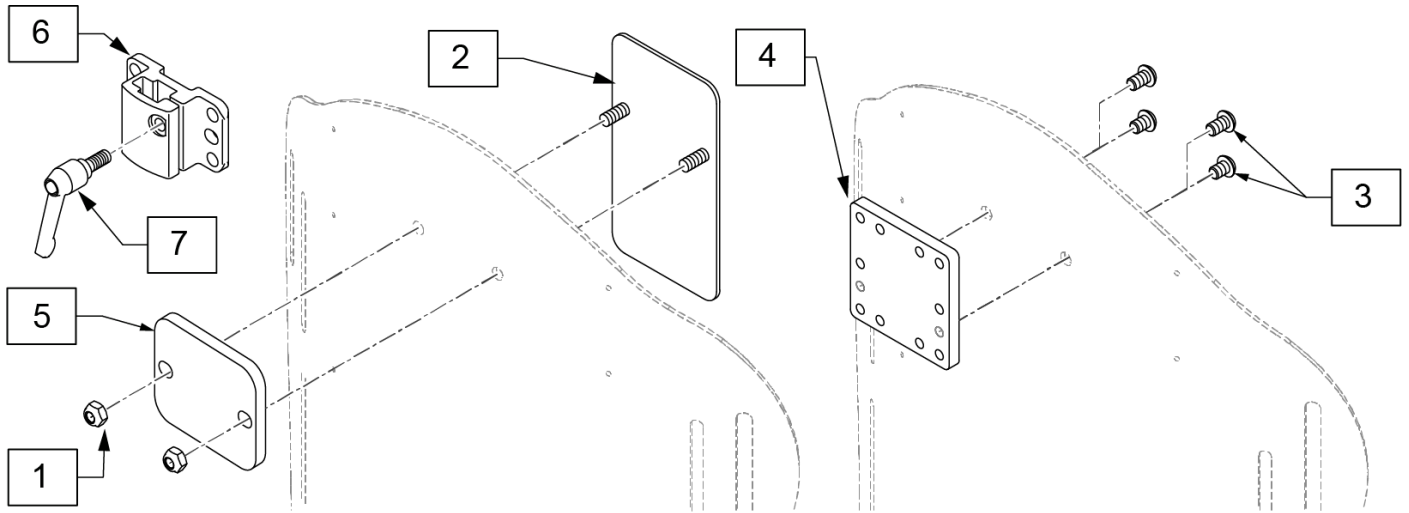


P/N	DESCRIPTION	APPLICATION
J3UHM	HEADREST PLATE ASSY	Universal Headrest Mount designed to mount most headrests in the market today, including Whitmyer
J3HDP	HEADREST BAG-O-BOLTS	Optional Mounting for Whitmyer Headrest Assemblies



ITEM #	ILLUSTRATION	DESCRIPTION	QTY
3		BHCS, M6X1X8, FULL, PL	2
		BHCS, M6X1X10, FULL, PL	2
4		PLATE ADAPTER HDRST UNIVERSAL	1

ITEM #	ILLUSTRATION	DESCRIPTION	QTY	ITEM #	ILLUSTRATION	DESCRIPTION	QTY
1		NUT HEX NYLOCK	2	5		SPACER LINX HEADREST MNT	1
2		NUT PLATE TRANSIT HEADREST	1	6		MT 5 SQR TUBING RECEIVER BLK	1
3		BHCS, M6X1X8, FULL, PL	2	7		KNOB PLAS ADJ LVR 1/4-20X20MM	1
		BHCS, M6X1X10, FULL, PL	2				


Pos.	Item Number	Description
8	J3UHM	HEADREST PLATE ASSY
9	J3HDP	HEADREST BAG-O-BOLTS

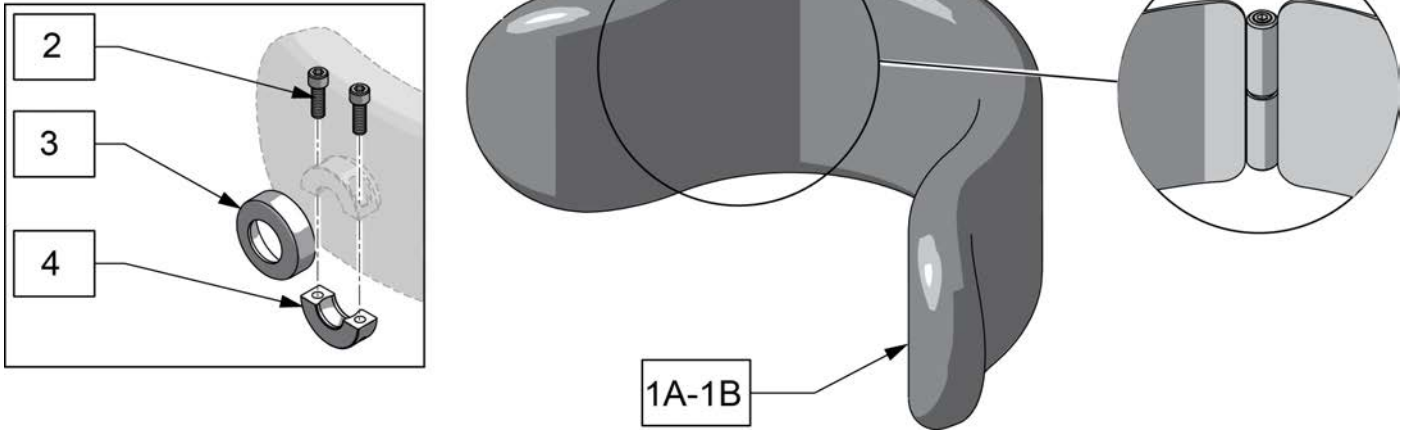

Pos.	Item Number	Description	Remarks
1A	PF8044	FOAM NARROW AJUSTAPLUSH	
1A	PC8044L	COVER, NAR ADJUST-A-PL, LY	Lycra
1A	PC8044V	COVER, NAR ADJUST-A-PL, DR	Dartex
1A	PC8044F	COVER NARROW ADJ-PLUSH/FLEECE	Fleece
1B	PF8088	FOAM STD AJUSTAPLUSH	
1B	PC8088L	COVER, STD ADJUST-A-PL, LY	Lycra
1B	PC8088V	COVER STD ADJ-PLUSH/RD	Dartex
1B	PC8088F	COVER STD AJUSTAPLUSH/FLEECE	Fleece
2	MF014600	1/4-28X5/8" SHCS- COLLAR BOLT	
3	MC100-WB	COVER 3/4"COLLAR/VINYL BLACK	
4	MW027100	COLLAR HALF RAD 3/4" THD. BLCK	
1A,2-4	P8044L	NARROW ADJ-A-PLUSH LYCRA	Lycra
1A,2-4	P8044V	NARROW ADJ-A-PLUSH DARTEX	Dartex
1B,2-4	P8088L	STANDARD ADJ-A-PLUSH LYCRA	Lycra
1B,2-4	P8088V	STANDARD ADJ-A-PLUSH DARTEX	Dartex

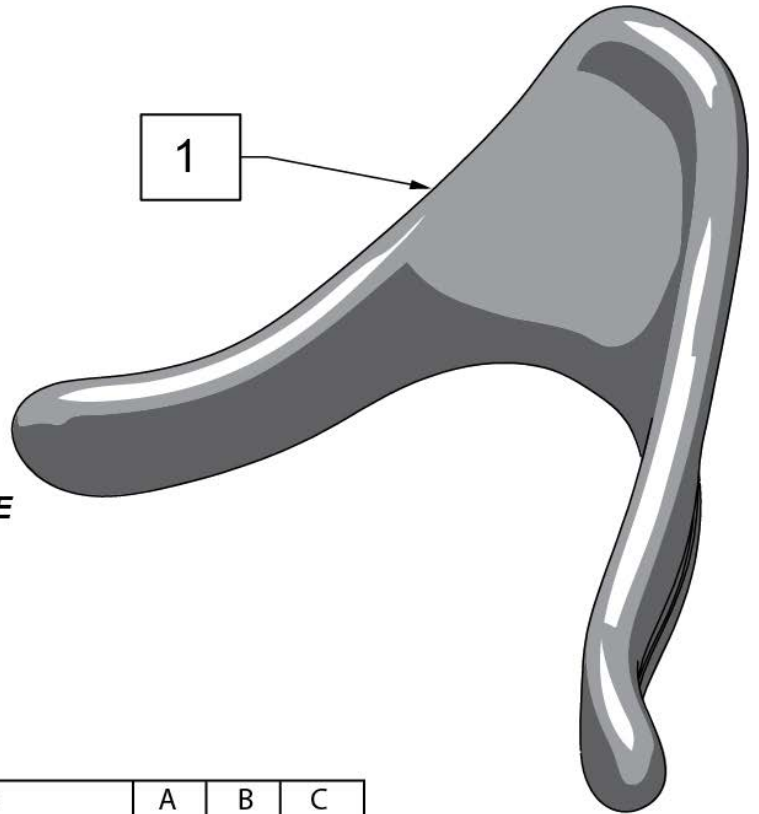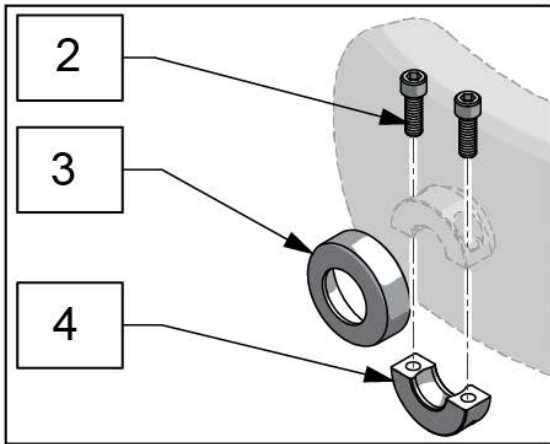


HEADREST MEASURING GUIDE

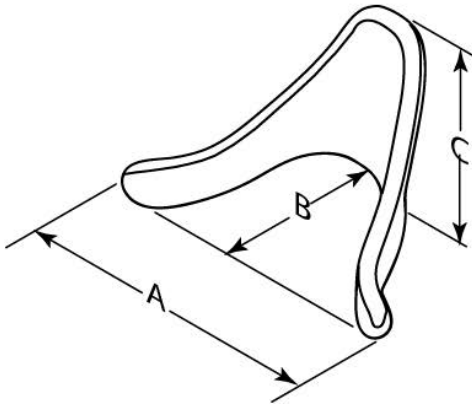


Description	A	B	C
Small Contour Cradle	8.5"	5.5"	4"
Medium Contour Cradle	10"	6.25"	4.75"
Large Contour Cradle	13"	7.5"	6-3/8"


Pos.	Item Number	Description	Remarks
1	PF200	FOAM SMALL CONTOURED CRADLE	Foam only
1	PC200L	COVER SML CONTED CRADLE/LYCRA	Cover only
1	PC200V	COVER SML CONTRD CRADLE/RD	Cover only
1	PC200F	COVER SML CONTED CRADLE/FLEECE	Cover only
1	PF300	FOAM MED CONTOURED CRADLE	Foam only
1	PC300L	COVER MED CONTED CRADLE/LYCRA	Cover only
1	PC300V	COVER MED CONTRD CRADLE/RD	Cover only
1	PC300F	COVER MED CONTED CRADLE/FLEECE	Cover only
1	PF400	FOAM LG CONTOURED CRADLE	Foam only
1	PC400L	COVER LG CONTED CRADLE/LYCRA	Cover only
1	PC400V	COVER LG CONTRD CRADLE/RD	Cover only
1	PC400F	COVER LG CONTED CRADLE/FLEECE	Cover only
2	MF014600	1/4-28X5/8" SHCS- COLLAR BOLT	
3	MC100-WB	COVER 3/4"COLLAR/VINYL BLACK	
4	MW027100	COLLAR HALF RAD 3/4" THD. BLCK	
1-4	P200L	SMALL C-PLUSH LYCRA	
1-4	P200V	SMALL C-PLUSH DARTEX	
1-4	P300L	MEDIUM C-PLUSH LYCRA	
1-4	P300V	MEDIUM C-PLUSH DARTEX	
1-4	P400L	LARGE C-PLUSH LYCRA	
1-4	P400V	LARGE C-PLUSH DARTEX	

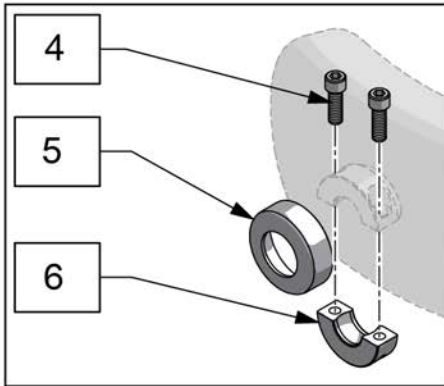

- 1A
- 1B
- 1C
- 1D

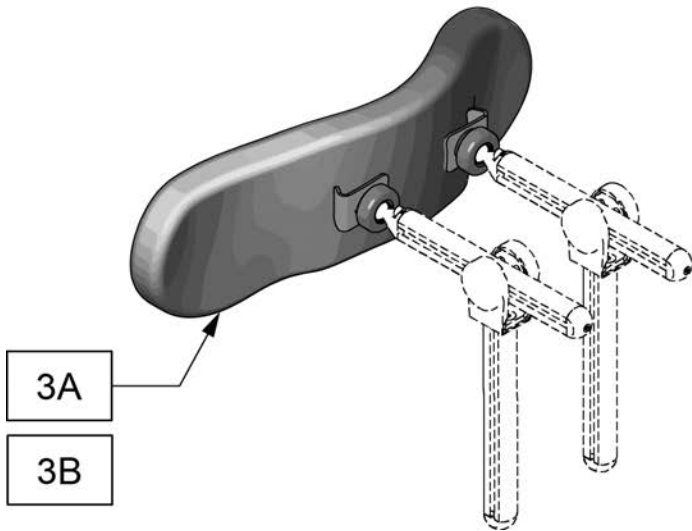


- 2A
- 2B

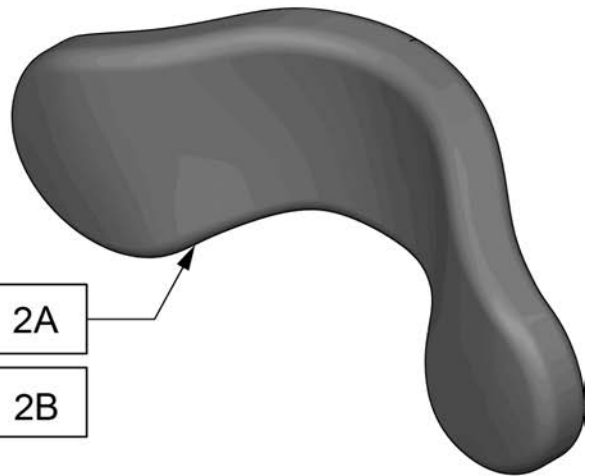

Pos.	Item Number	Description	Remarks
1A	PF605	REFOAM KIT 6" PLUSH	
1A	PC605L	COVER 6" PLUSH/LYCRA	Lycra
1A	PC605V	COVER 6" PLUSH/R'VSD DARTEXX	Dartex
1B	PF800	REFOAM KIT 8" PLUSH	
1B	PC800L	COVER, 8" PLUSH, LYCRA	Lycra
1B	PC800V	COVER, 8" PLUSH, DR	Dartex
1B	PC800F	COVER 8" PLUSH/FLEECE	Fleece
1B	PC800P	COVER 8" PLUSH/ALLURE NH	
1C	PF1000	REFOAM KIT 10" PLUSH	
1C	PC1000L	COVER, 10" PLUSH, LYCRA	Lycra
1C	PC1000V	COVER, 10" PLUSH, DR	Dartex
1C	PC1000F	COVER 10" PLUSH/FLEECE	Fleece
1D	PF1205	REFOAM KIT 12" NARROW PLUSH	
1D	PC1205L	COVER NARROW PLUSH/LYCRA	Lycra
1D	PC1205V	COVER NARROW PLUSH/R'VSD DARTEXX	Dartex
1D	PC1205F	COVER NARROW PLUSH/FLEECE	Fleece
2A	PF1400	REFOAM KIT 14" PLUSH	

Pos.	Item Number	Description	Remarks
2A	PC1400L	COVER, 14" PLUSH, LYCRA	Lycra
2A	PC1400V	COVER, 14" PLUSH, DR	Dartex
2A	PC1400F	COVER 14"PLUSH/FLEECE	Fleece
2B	PF1900	REFOAM KIT 19"PLUSH	
2B	PC1900L	COVER 19"PLUSH/LYCRA	Lycra
2B	PC1900V	COVER 19" PLUSH/R'VSD DARTEXX	Dartex
2B	PC1900F	COVER 19"PLUSH/FLEECE	Fleece
4	MF014600	1/4-28X5/8" SHCS- COLLAR BOLT	
5	MC100-WB	COVER 3/4"COLLAR/VINYL BLACK	
6	MW027100	COLLAR HALF RAD 3/4" THD. BLCK	
1A,4-6	P605L	PLUSH PAD, 6", LYCRA	Lycra
1A,4-6	P605V	PLUSH PAD, 6", DARTEX	Dartex
1B,4-6	P800L	PLUSH PAD, 8 IN LYCRA	Lycra
1B,4-6	P800V	PLUSH PAD, 8 IN DARTEX	Dartex
1C,4-6	P1000L	PLUSH PAD, 10 IN LYCRA	Lycra
1C,4-6	P1000V	PLUSH PAD, 10 IN DARTEX	Dartex
1D,4-6	P1205L	PLUSH PAD, 12 IN NARROW LYCRA	Lycra
1D,4-6	P1205V	PLUSH PAD, 12 IN NARROW DARTEX	Dartex
2A,4-6	P1400L	PLUSH PAD, 14 IN LYCRA	Lycra
2A,4-6	P1400V	PLUSH PAD, 14 IN DARTEX	Dartex
2B,4-6	P1900L	PLUSH PAD, 19 IN LYCRA	Lycra
2B,4-6	P1900V	PLUSH PAD, 19 IN DARTEX	Dartex
3A,4-6	P1420L	PLUSH PAD, 14 IN DUAL MOUNT LYCRA	Lycra
3A,4-6	P1420V	PLUSH PAD, 14 IN DUAL MOUNT DARTEX	Dartex
3B,4-6	P1920L	PLUSH PAD, 19 IN DUAL MOUNT LYCRA	Lycra
3B,4-6	P1920V	PLUSH PAD, 19 IN DUAL MOUNT DARTEX	Dartex


Note: Parts not compatible with Headrest ordered prior to 5/27/2013, please replace complete headrest if ordered prior to 5/27/2013.

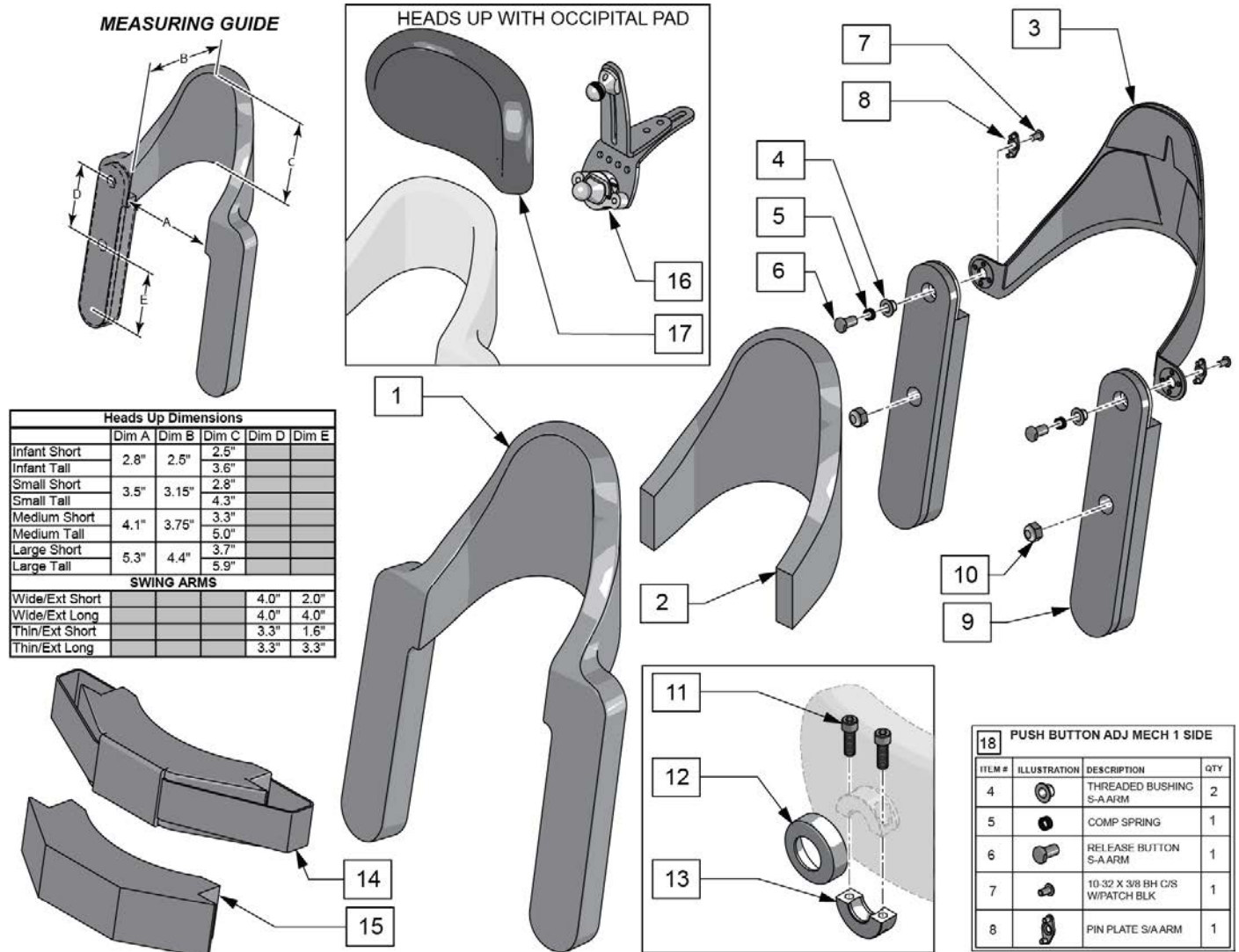


Pos.	Item Number	Description	Remarks
		DARTEX	
1	125375	COV INFANT SHT WIDE SHT DARTEX	Short Cradle, Short Arms
1	125376	COV INFANT SHT WIDE LNG DARTEX	Short Cradle, Long Arms
1	125379	COV INFANT TALL THIN SHT DARTEX	Tall Cradle, Short Arms
1	125380	COV INFANT TALL THIN LNG DARTEX	Tall Cradle, Long Arms
1	125383	COV SMALL SHT WIDE SHT DARTEX	Short Cradle, Short Arms
1	125384	COV SMALL SHT WIDE LNG DARTEX	Short Cradle, Long Arms
1	125387	COV SMALL TALL THIN SHT DARTEX	Tall Cradle, Short Arms
1	125388	COV SMALL TALL THIN LNG DARTEX	Tall Cradle, Long Arms
1	125391	COV MEDIUM SHT WIDE SHT DARTEX	Short Cradle, Short Arms
1	125392	COV MEDIUM SHT WIDE LNG DARTEX	Short Cradle, Long Arms
1	125395	COV MEDIUM TALL THIN SHT DARTEX	Tall Cradle, Short Arms
1	125396	COV MEDIUM TALL THIN LNG DARTEX	Tall Cradle, Long Arms
1	125399	COV LARGE SHT WIDE SHT DARTEX	Short Cradle, Short Arms

Pos.	Item Number	Description	Remarks
1	125400	COV LARGE SHT WIDE LNG DARTEX	Short Cradle, Long Arms
1	125403	COV LARGE TALL THIN SHT DARTEX	Tall Cradle, Short Arms
1	125404	COV LARGE TALL THIN LNG DARTEX	Tall Cradle, Long Arms
		LYCRA	
1	125377	COV INFANT SHT WIDE SHT LYCRA	Short Cradle, Short Arms
1	125378	COV INFANT SHT WIDE LNG LYCRA	Short Cradle, Long Arms
1	125381	COV INFANT TALL THIN SHT LYCRA	Tall Cradle, Short Arms
1	125382	COV INFANT TALL THIN LNG LYCRA	Tall Cradle, Long Arms
1	125385	COV SMALL SHT WIDE SHT LYCRA	Short Cradle, Short Arms
1	125386	COV SMALL SHT WIDE LNG LYCRA	Short Cradle, Long Arms
1	125389	COV SMALL TALL THIN SHT LYCRA	Tall Cradle, Short Arms
1	125390	COV SMALL TALL THIN LNG LYCRA	Tall Cradle, Long Arms
1	125393	COV MEDIUM SHT WIDE SHT LYCRA	Short Cradle, Short Arms
1	125394	COV MEDIUM SHT WIDE LNG LYCRA	Short Cradle, Long Arms
1	125397	COV MEDIUM TALL THIN SHT LYCRA	Tall Cradle, Short Arms
1	125398	COV MEDIUM TALL THIN LNG LYCRA	Tall Cradle, Long Arms
1	125401	COV LARGE SHT WIDE SHT LYCRA	Short Cradle, Short Arms
1	125402	COV LARGE SHT WIDE LNG LYCRA	Short Cradle, Long Arms
1	125405	COV LARGE TALL THIN SHT LYCRA	Tall Cradle, Short Arms
1	125406	COV LARGE TALL THIN LNG LYCRA	Tall Cradle, Long Arms
2	125351	STANDARD CRADLE PAD INF SHT	
2	125352	STANDARD CRADLE PAD INF TL	
2	125353	STANDARD CRADLE PAD SM SHT	
2	125354	STANDARD CRADLE PAD SM TL	
2	125355	STANDARD CRADLE PAD MED SHT	
2	125356	STANDARD CRADLE PAD MED TL	
2	125357	STANDARD CRADLE PAD LRG SHT	
2	125358	STANDARD CRADLE PAD LRG TL	
2	125359	COMFORT CRADLE PAD INF SHT	
2	125360	COMFORT CRADLE PAD INF TL	
2	125361	COMFORT CRADLE PAD SM SHT	
2	125362	COMFORT CRADLE PAD SM TL	
2	125363	COMFORT CRADLE PAD MED SHT	
2	125364	COMFORT CRADLE PAD MED TL	
2	125365	COMFORT CRADLE PAD LRG SHT	
2	125366	COMFORT CRADLE PAD LRG TL	
3	125343	INFANT SHORT CRADLE SHELL	
3	125345	SMALL SHORT CRADLE SHELL	
3	125347	MEDIUM SHORT CRADLE SHELL	
3	125349	LARGE SHORT CRADLE SHELL	
3	125344	INFANT TALL CRADLE SHELL	
3	125346	SMALL TALL CRADLE SHELL	
3	125348	MEDIUM TALL CRADLE SHELL	
3	125350	LARGE TALL CRADLE SHELL	
9	125367	SWING ARM WIDE EXT SHORT RT	Used w/ Med & Large Cradle
9	125368	SWING ARM WIDE EXT SHORT LT	Used w/ Med & Large Cradle
9	125369	SWING ARM WIDE EXT LONG RT	Used w/ Med & Large Cradle
9	125370	SWING ARM WIDE EXT LONG LT	Used w/ Med & Large Cradle
9	125371	SWING ARM THIN EXT SHORT RT	Used w/ Infant & Small Cradle
9	125372	SWING ARM THIN EXT SHORT LT	Used w/ Infant & Small Cradle
9	125373	SWING ARM THIN EXT LONG RT	Used w/ Infant & Small Cradle
9	125374	SWING ARM THIN EXT LONG LT	Used w/ Infant & Small Cradle
10	106933	NUT NYLOCK THIN 1/4-20 ROHS	
11	MF014600	1/4-28x5/8" SHCS-COLLAR BOLT	
12	MC100-WB	COVER 3/4" COLLAR /VINYL BLACK	
13	MW027100	COLLAR HALF RAD 3/4" THD. BLCK	
15	125407	FOAM PAD CHIN PROMPT INFANT	

Pos.	Item Number	Description	Remarks
15	125408	FOAM PAD CHIN PROMPT SMALL	
15	125409	FOAM PAD CHIN PROMPT MEDIUM	
15	125410	FOAM PAD CHIN PROMPT LARGE	
14	125411	COVER PAD CHIN PROMPT INFANT	
14	125412	COVER PAD CHIN PROMPT SMALL	
14	125413	COVER PAD CHIN PROMPT MEDIUM	
14	125414	COVER PAD CHIN PROMPT LARGE	
16	M9275	S.O.F.T. BPA, SINGLE SUB-OCC	
17	P500L-WB	OCCIP PAD, PED, LYCRA	
17	P600L	OCCIP PAD, ADULT, LYCRA	
17	P800L	PLUSH PAD, 8 IN LYCRA	
17	P1000L	PAD 10" PLUSH/LYCRA	
17	P500V	OCCIP PAD, PED, DR	
17	P600V	OCCIP PAD, ADULT, DR	
17	P800V	PLUSH PAD, 8", DR	
17	P1000V	PLUSH PAD, 10", DR	
18	125415	PUSH BUTTON ADJ MECH 1 SIDE	

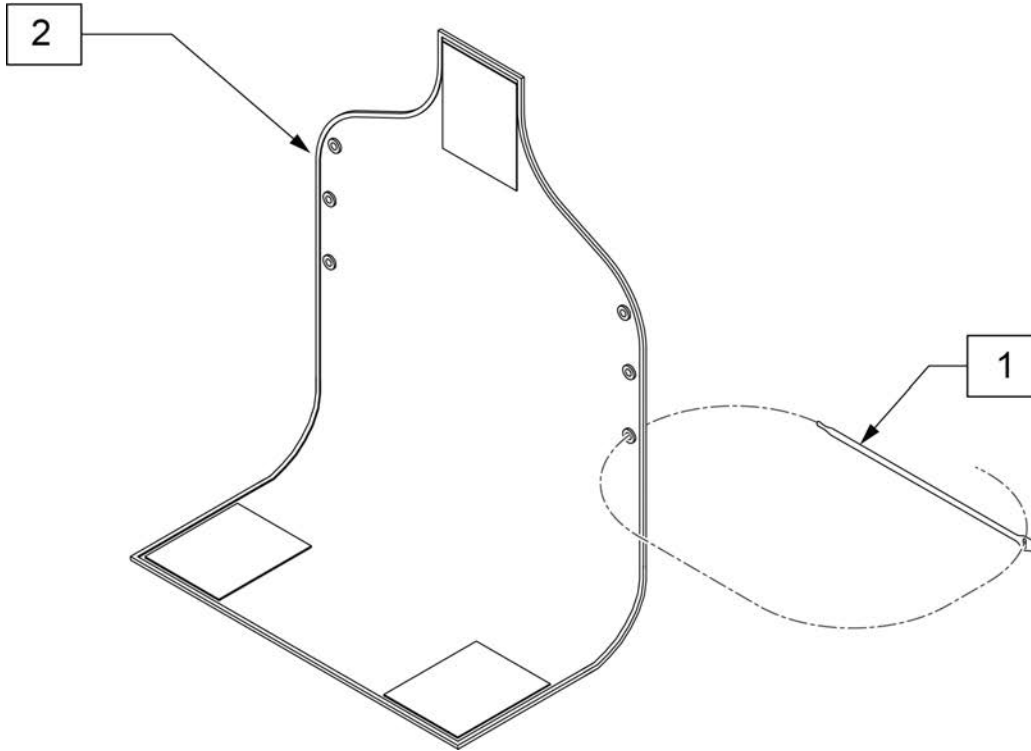


MEASURING GUIDE

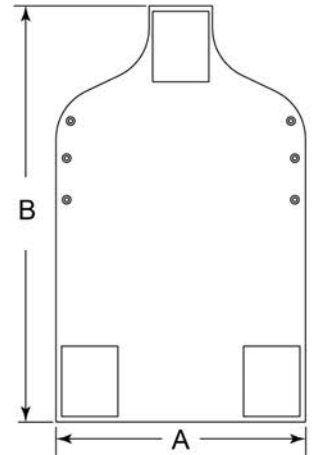


P/N	DIM A	DIM B
116522-001	11 3/4"	25 1/2"
116522-002	13 3/4"	25 1/2"
116522-003	15 3/4"	27 1/2"
116522-004	17 3/4"	29 1/2"
116522-005	19 3/4"	31 1/2"
116522-006	21 3/4"	31 1/2"
116522-007	23 3/4"	33 1/2"
116522-008	25 3/4"	33 1/2"

Pos.	Item Number	Description	Remarks
1	116527S	VELCRO TIE,1 WRAP.75"X12",BLK	
1,2	117096	VANITY FLAP, J3 BACK 12W	Includes 4 velcro ties
1,2	117097	VANITY FLAP, J3 BACK 14W	Includes 4 velcro ties
1,2	117098	VANITY FLAP, J3 BACK 16W	Includes 4 velcro ties
1,2	117099	VANITY FLAP, J3 BACK 18W	Includes 4 velcro ties
1,2	117100	VANITY FLAP, J3 BACK 20W	Includes 4 velcro ties
1,2	117101	VANITY FLAP, J3 BACK 22W	Includes 4 velcro ties
1,2	117102	VANITY FLAP, J3 BACK 24W	Includes 4 velcro ties
1,2	117103	VANITY FLAP, J3 BACK 26W	Includes 4 velcro ties
1,2	VF08	VANITY FLAP 8-9 IN	Includes 4 velcro ties
1,2	VF10	VANITY FLAP 10-11 IN	Includes 4 velcro ties
1,2	VF12	VANITY FLAP 12-13 IN	Includes 4 velcro ties
1,2	VF14	VANITY FLAP 14-15 IN	Includes 4 velcro ties

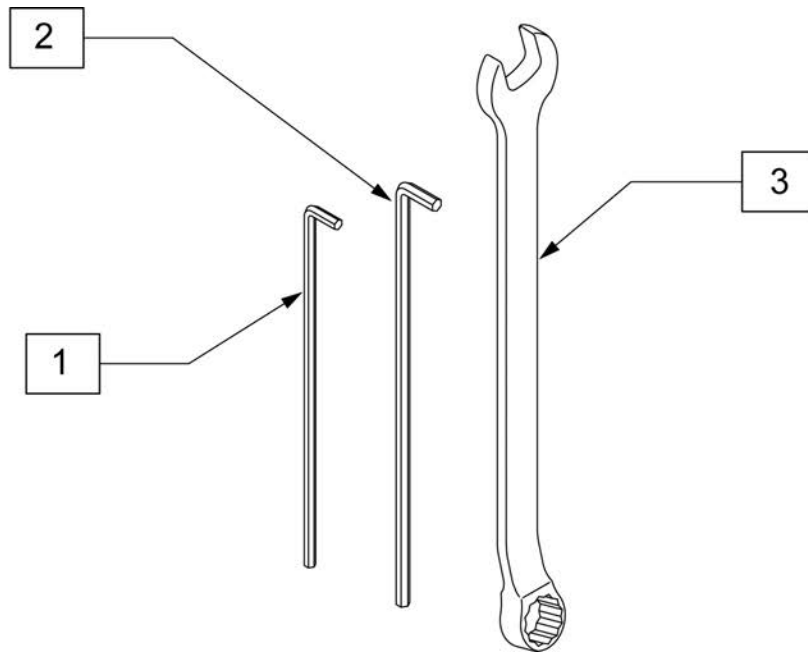


Pos.	Item Number	Description
1	J3HEX3MM	HEX KEY 3MM LONG
2	J3HEX4MM	HEX KEY 4MM LONG
3	J3CW10MM	10 MM COMBO WRENCH

1

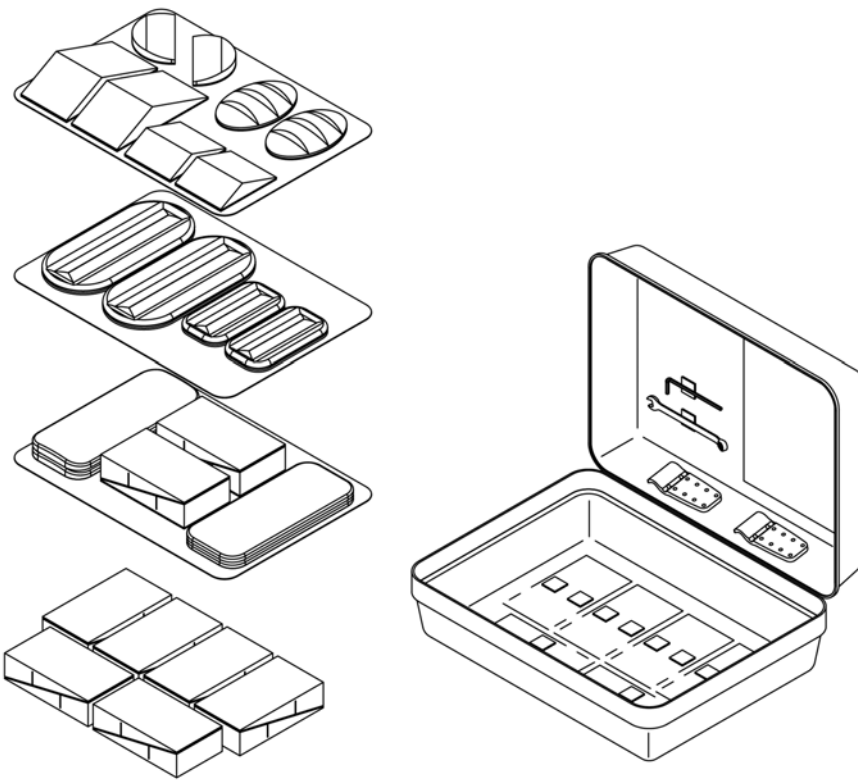


DESCRIPTION	QTY
BACK WEDGE BLK 3.5W W/VEL	16
LATERAL LUMBAR SUPP BLK W/VEL	2
LATERAL SUPP WEDGE BLK W/VEL	2
LUMBAR SUPP LG BLK 3.5W W/VEL	2
FLAT SHIM	2
CURVED SHIM	2
LUMBAR SUPP SM BLK 3.5W W/VEL	2
LARGE BEAN BAG	2
SMALL BEAN BAG	2
VARI DIA CLAMP INNER LONG	2
COMBO WRENCH 10MM	1
HEX KEY 4MM LONG	1
CASE INSERT 1 W/VEL J3 ASSES	1
CASE INSERT 2 W/VEL J3 ASSES	1
CASE INSERT 3 W/VEL J3 ASSES	1
CASE J3 SPINE ALIGN ASSES KIT	1

Pos.	Item Number	Description
1	DJ3SPNKA	ASSESSMENT KIT, SPINE ALIGN

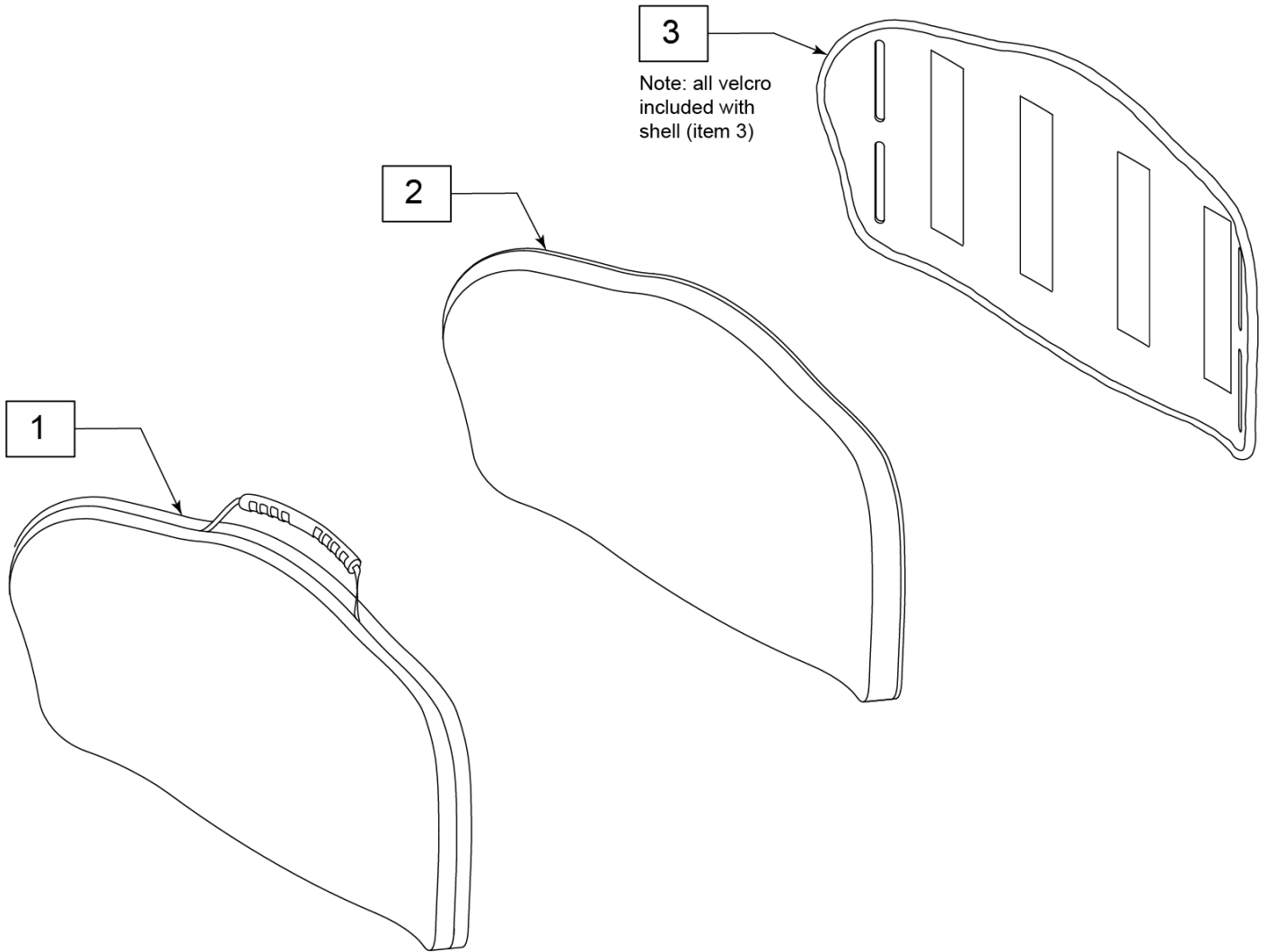


Pos.	Item Number	Description
1	CJ3PAMTS22	J3HD CVR POS MT 22W STD
1	CJ3PAMTT22	J3HD CVR POS MT 22W TAL
1	CJ3PAMTS24	J3HD CVR POS MT 24W STD
1	CJ3PAMTT24	J3HD CVR POS MT 24W TAL
1	CJ3PAMTS26	J3HD CVR POS MT 26W STD
1	CJ3PAMTT26	J3HD CVR POS MT 26W TAL
2	BJ3PAMTS22	J3HD FOAM POS MT 22W STD
2	BJ3PAMTT22	J3HD FOAM POS MT 22W TAL
2	BJ3PAMTS24	J3HD FOAM POS MT 24W STD
2	BJ3PAMTT24	J3HD FOAM POS MT 24W TAL
2	BJ3PAMTS26	J3HD FOAM POS MT 26W STD
2	BJ3PAMTT26	J3HD FOAM POS MT 26W TAL
3	SJ3PAMTS22	J3HD SHELL POS MT 22W STD
3	SJ3PAMTT22	J3HD SHELL POS MT 22W TAL
3	SJ3PAMTS24	J3HD SHELL POS MT 24W STD
3	SJ3PAMTT24	J3HD SHELL POS MT 24W TAL
3	SJ3PAMTS26	J3HD SHELL POS MT 26W STD
3	SJ3PAMTT26	J3HD SHELL POS MT 26W TAL

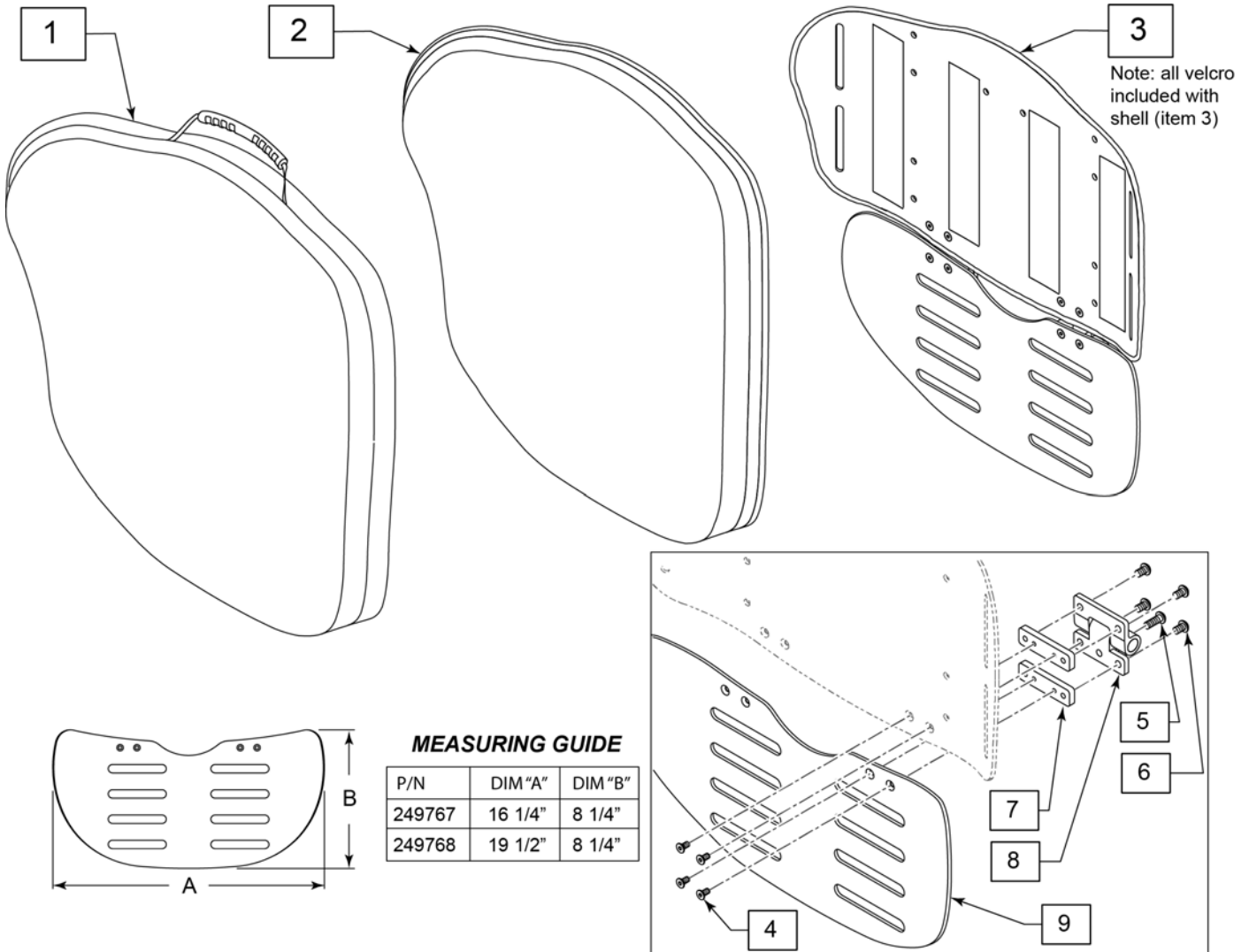


Pos.	Item Number	Description	Remarks
1	CJ3PLUSPAMTS22	J3 PLUS BK CVR PA 22W SHT	
1	CJ3PLUSPAMTT22	J3 PLUS BK CVR PA 22W TLL	
1	CJ3PLUSPAMTS24	J3 PLUS BK CVR PA 24W SHT	
1	CJ3PLUSPAMTT24	J3 PLUS BK CVR PA 24W TLL	
1	CJ3PLUSPAMTS26	J3 PLUS BK CVR PA 26W SHT	
1	CJ3PLUSPAMTT26	J3 PLUS BK CVR PA 26W TLL	
2	BJ3PLUSPAMTS22	J3 PLUS BK 1.5FOAM 22W SHT	
2	BJ3PLUSPAMTT22	J3 PLUS BK 1.5FOAM 22W TLL	
2	BJ3PLUSPAMTS24	J3 PLUS BK 1.5FOAM 24W SHT	
2	BJ3PLUSPAMTT24	J3 PLUS BK 1.5FOAM 24W TLL	
2	BJ3PLUSPAMTS26	J3 PLUS BK 1.5FOAM 26W SHT	
2	BJ3PLUSPAMTT26	J3 PLUS BK 1.5FOAM 26W TLL	
3	SJ3PLUSPAMTS22	J3 PLUS BK SHELL 22W SHT	
3	SJ3PLUSPAMTT22	J3 PLUS BK SHELL 22W TLL	
3	SJ3PLUSPAMTS24	J3 PLUS BK SHELL 24W SHT	
3	SJ3PLUSPAMTT24	J3 PLUS BK SHELL 24W TLL	
3	SJ3PLUSPAMTS26	J3 PLUS BK SHELL 26W SHT	
3	SJ3PLUSPAMTT26	J3 PLUS BK SHELL 26W TLL	
4	XM200040	FHCS 10-32X.38 W PATCH BLK	

Pos.	Item Number	Description	Remarks
5	XM200010	JAY BACK HINGE ADJUST SCREW	
6	XM200029	JAY BACK HINGE MOUNT SCREW	
7	251146	J3 PLUS VANITY FLAP SPACER	
5-8	251149	J3 PLUS HINGE AND SPACER ASSY	
9	249767	VANITY FLAP J3 PLUS 22-24	22" & 24" wide
9	249768	VANITY FLAP J3 PLUS 26	26" wide

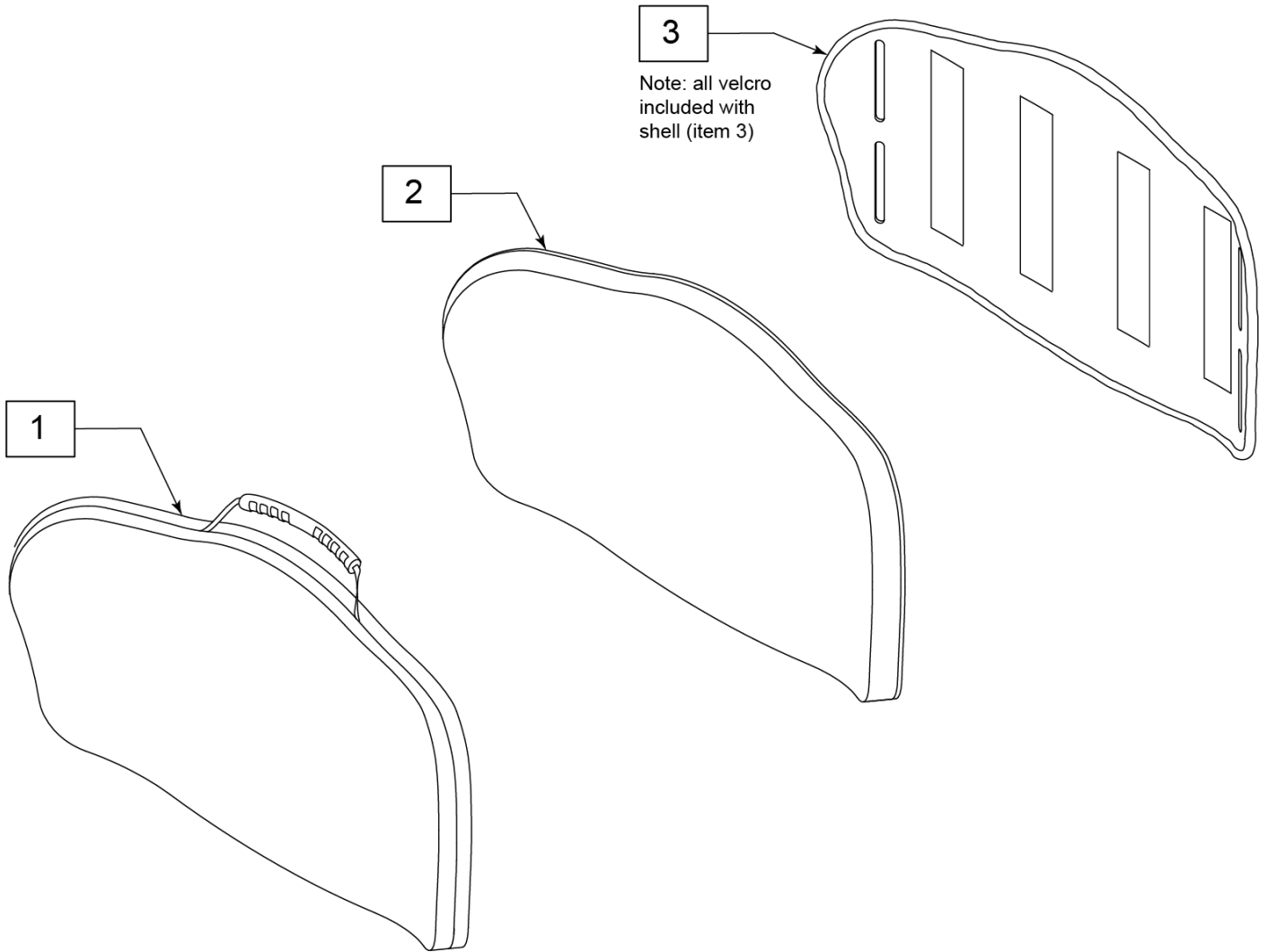


Pos.	Item Number	Description
1	CJ3PAUTM22	J3HD CVR POS UT 22W MED
1	CJ3PAUTM24	J3HD CVR POS UT 24W MED
2	BJ3PAUTM22	J3HD FOAM POS UT 22W MED
2	BJ3PAUTM24	J3HD FOAM POS UT 24W MED
3	SJ3PAUTM22	J3HD SHELL POS UT 22W MED
3	SJ3PAUTM24	J3HD SHELL POS UT 24W MED
4	ICJ3PAUTM22	J3 HD BK,POS UT 22W MED, FS
4	ICJ3PAUTM24	J3 HD BK,POS UT 24W MED, FS

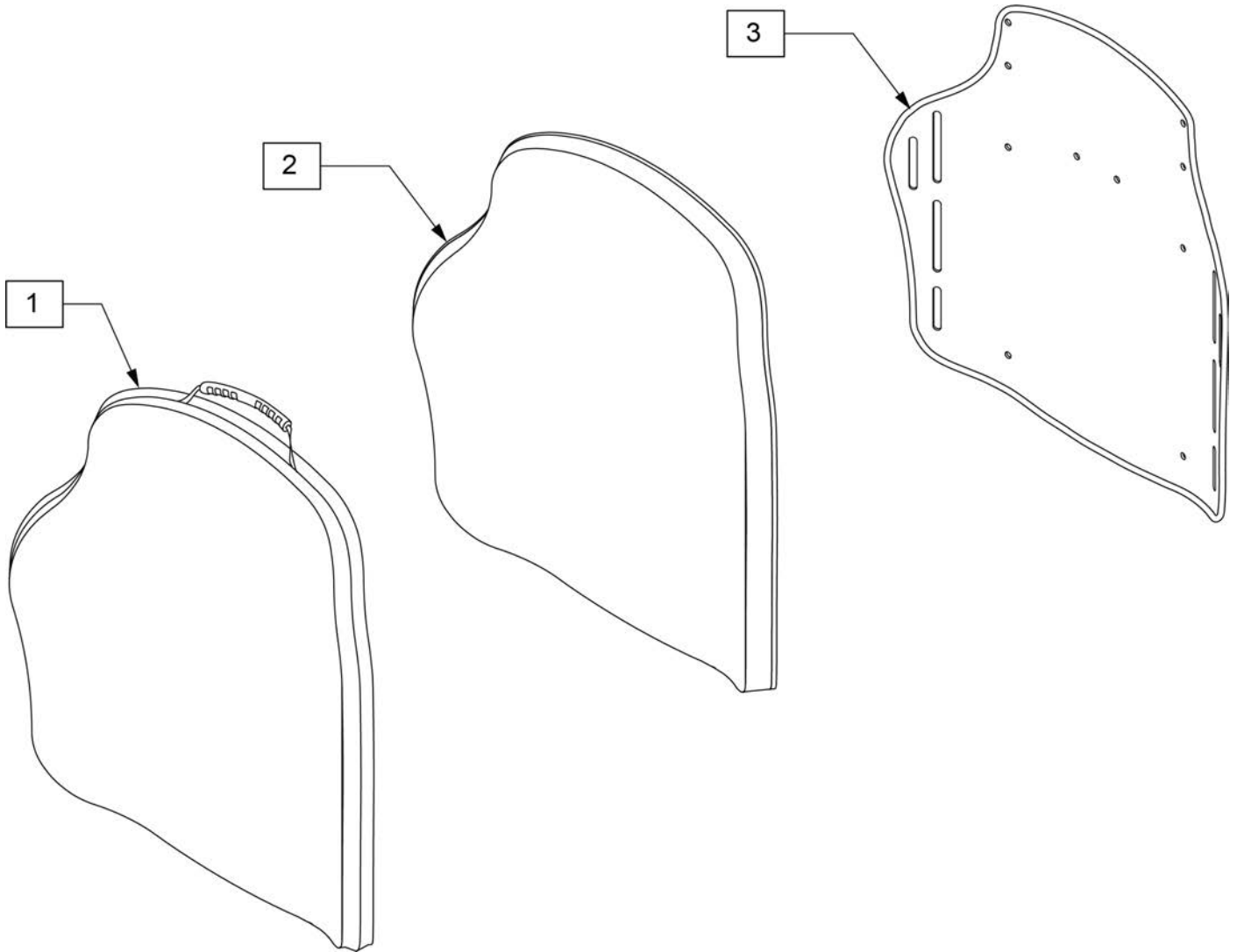


Pos.	Item Number	Description
1	CJ3PASHS22	J3HD CVR POS SH 22W STD
2	BJ3PASHS22	J3HD FOAM POS SH 22W STD
3	SJ3PASHS22	J3HD SHELL POS SH 22W STD
4	ICJ3PASHS22	J3 HD BK,POS SH 22W STD, FS

NOTE: Parts only, for complete Backrest see J3 Custom Back order form.

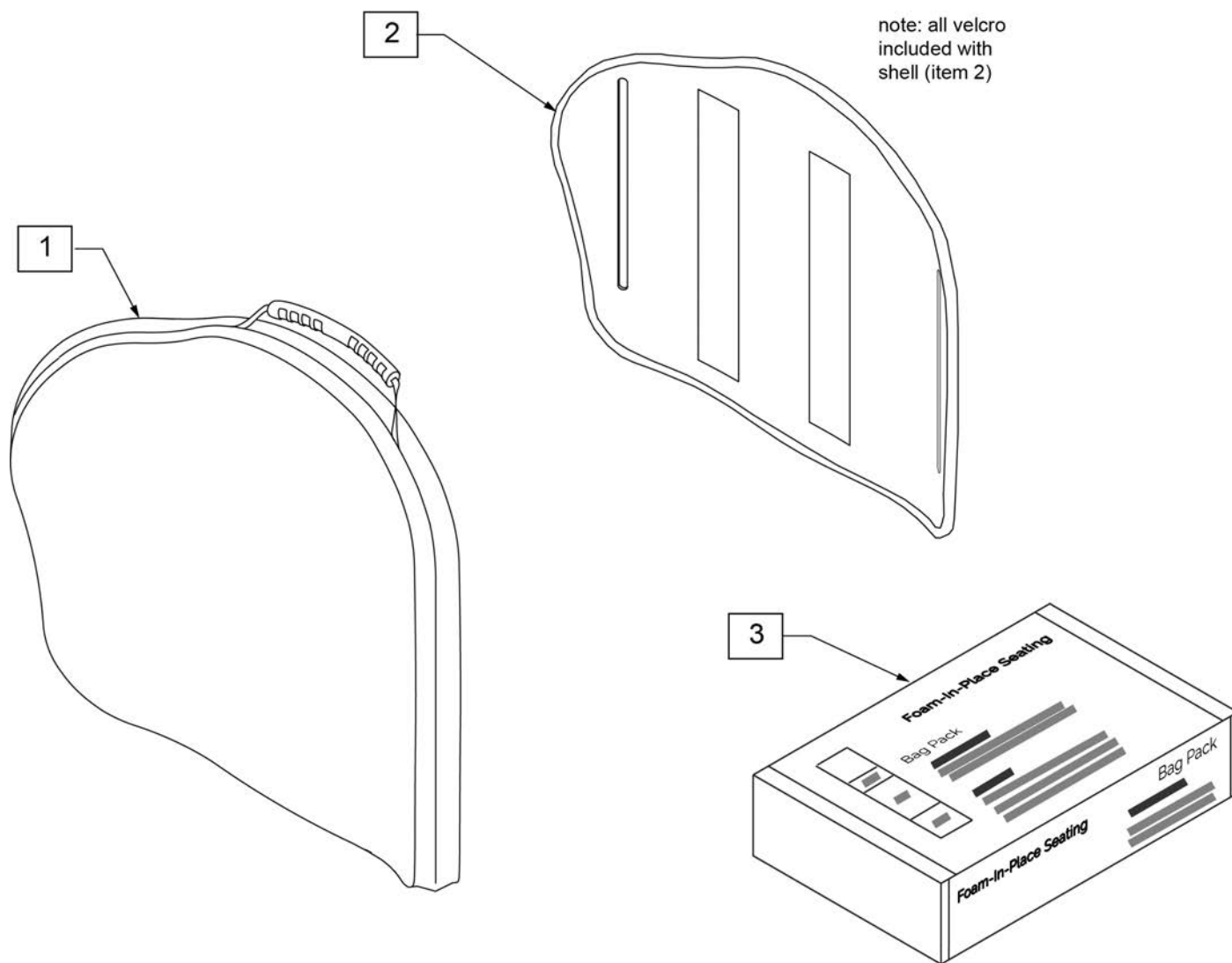


Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJ3FIPPAMTS14	J3 FIP BK CVR MT 14S 2" THICK	2	SJ3PAMTT14	J3 SHELL, POS MT 14W TALL
1	CJ3FIPPAMTM14	J3 FIP BK CVR MT 14M 2" THICK	2	SJ3PAMTS16	J3 SHELL, POS MT 16W STD
1	CJ3FIPPAMTT14	J3 FIP BK CVR MT 14T 2" THICK	2	SJ3PAMTM16	J3 SHELL, POS MT 16W MED
1	CJ3FIPPAMTS16	J3 FIP BK CVR MT 16S 2" THICK	2	SJ3PAMTT16	J3 SHELL, POS MT 16W TALL
1	CJ3FIPPAMTM16	J3 FIP BK CVR MT 16M 2" THICK	2	SJ3PAMTS18	J3 SHELL, POS MT 18W STD
1	CJ3FIPPAMTT16	J3 FIP BK CVR MT 16T 2" THICK	2	SJ3PAMTM18	J3 SHELL, POS MT 18W MED
1	CJ3FIPPAMTS18	J3 FIP BK CVR MT 18S 2" THICK	2	SJ3PAMTT18	J3 SHELL, POS MT 18W TALL
1	CJ3FIPPAMTM18	J3 FIP BK CVR MT 18M 2" THICK	2	SJ3PAMTS20	J3 SHELL, POS MT 20W STD
1	CJ3FIPPAMTT18	J3 FIP BK CVR MT 18T 2" THICK	2	SJ3PAMTM20	J3 SHELL, POS MT 20W MED
1	CJ3FIPPAMTS20	J3 FIP BK CVR MT 20S 2" THICK	2	SJ3PAMTT20	J3 SHELL, POS MT 20W TALL
1	CJ3FIPPAMTM20	J3 FIP BK CVR MT 20M 2" THICK	3	UPH1780B	FIP, X-SOFT (BAG) 1 UNIT
1	CJ3FIPPAMTT20	J3 FIP BK CVR MT 20T 2" THICK	3	UPH1781B	FIP, SOFT (BAG) 1 UNIT
2	SJ3PAMTS14	J3 SHELL, POS MT 14W STD	3	UPH1784B	FIP, MEDIUM (BAG) 1 UNIT
2	SJ3PAMTM14	J3 SHELL, POS MT 14W MED			

Note: parts only; for complete Backrest see J3 Custom Back order form.

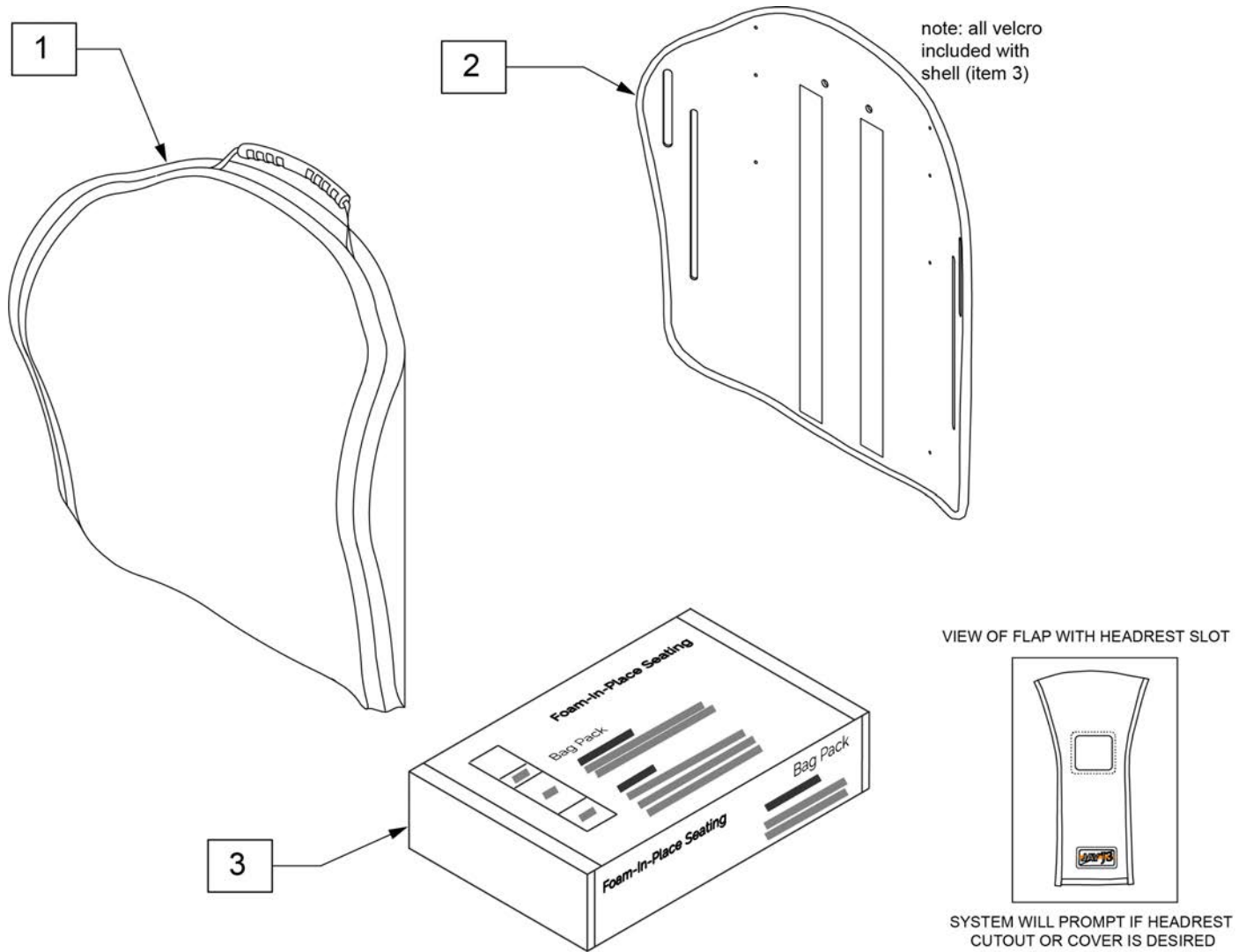


Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJ3FIPPAUTS14	J3 FIP BK CVR UT 14S 2" THICK	2	SJ3PAUTT14	J3 SHELL, POS UT 14W TALL
1	CJ3FIPPAUTM14	J3 FIP BK CVR UT 14M 2" THICK	2	SJ3PAUTS16	J3 SHELL, POS UT 16W STD
1	CJ3FIPPAUTT14	J3 FIP BK CVR UT 14T 2" THICK	2	SJ3PAUTM16	J3 SHELL, POS UT 16W MED
1	CJ3FIPPAUTS16	J3 FIP BK CVR UT 16S 2" THICK	2	SJ3PAUTT16	J3 SHELL, POS UT 16W TALL
1	CJ3FIPPAUTM16	J3 FIP BK CVR UT 16M 2" THICK	2	SJ3PAUTS18	J3 SHELL, POS UT 18W STD
1	CJ3FIPPAUTT16	J3 FIP BK CVR UT 16T 2" THICK	2	SJ3PAUTM18	J3 SHELL, POS UT 18W MED
1	CJ3FIPPAUTS18	J3 FIP BK CVR UT 18S 2" THICK	2	SJ3PAUTT18	J3 SHELL, POS UT 18W TALL
1	CJ3FIPPAUTM18	J3 FIP BK CVR UT 18M 2" THICK	2	SJ3PAUTS20	J3 SHELL, POS UT 20W STD
1	CJ3FIPPAUTT18	J3 FIP BK CVR UT 18T 2" THICK	2	SJ3PAUTM20	J3 SHELL, POS UT 20W MED
1	CJ3FIPPAUTS20	J3 FIP BK CVR UT 20S 2" THICK	2	SJ3PAUTT20	J3 SHELL, POS UT 20W TALL
1	CJ3FIPPAUTM20	J3 FIP BK CVR UT 20M 2" THICK	3	UPH1780B	FIP, X-SOFT (BAG) 1 UNIT
1	CJ3FIPPAUTT20	J3 FIP BK CVR UT 20T 2" THICK	3	UPH1781B	FIP, SOFT (BAG) 1 UNIT
2	SJ3PAUTS14	J3 SHELL, POS UT 14W STD	3	UPH1784B	FIP, MEDIUM (BAG) 1 UNIT
2	SJ3PAUTM14	J3 SHELL, POS UT 14W MED			

Note: Parts only; for complete Backrest see J3 Custom Back order form.

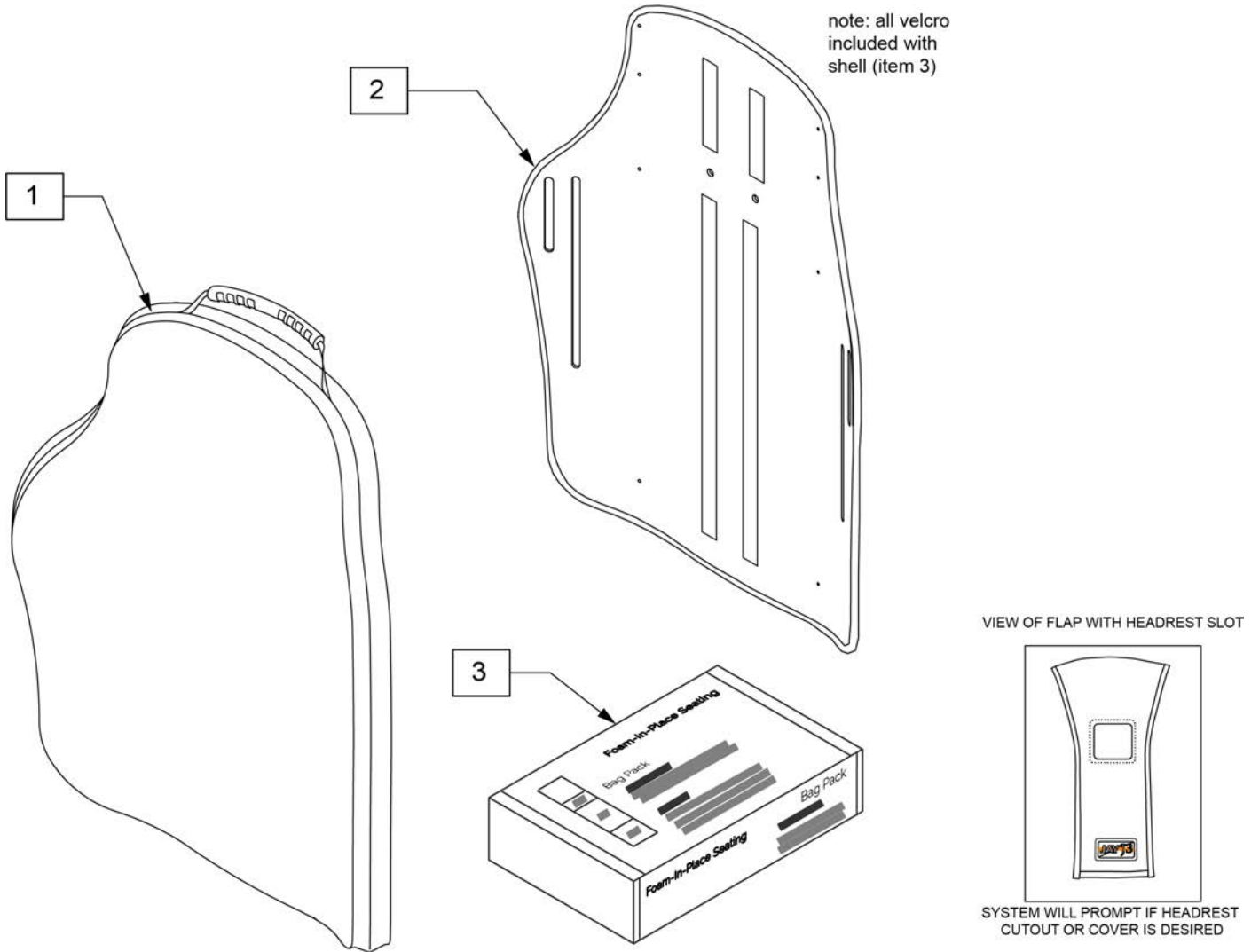


Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJ3FIPPASHS14	J3 FIP BK CVR SH 14S 2" THICK	2	SJ3PASHM16	J3 SHELL, POS SH 16W MED
1	CJ3FIPPASHS16	J3 FIP BK CVR SH 16S 2" THICK	2	SJ3PASHS18	J3 SHELL, POS SH 18W STD
1	CJ3FIPPASHM16	J3 FIP BK CVR SH 16M 2" THICK	2	SJ3PASHM18	J3 SHELL, POS SH 18W MED
1	CJ3FIPPASHS18	J3 FIP BK CVR SH 18S 2" THICK	2	SJ3PASHT18	J3 SHELL, POS SH 18W TALL
1	CJ3FIPPASHM18	J3 FIP BK CVR SH 18M 2" THICK	2	SJ3PASHS20	J3 SHELL, POS SH 20W STD
1	CJ3FIPPASHT18	J3 FIP BK CVR SH 18T 2" THICK	2	SJ3PASHM20	J3 SHELL, POS SH 20W MED
1	CJ3FIPPASHS20	J3 FIP BK CVR SH 20S 2" THICK	2	SJ3PASHT20	J3 SHELL, POS SH 20W TALL
1	CJ3FIPPASHM20	J3 FIP BK CVR SH 20M 2" THICK	3	UPH1780B	FIP, X-SOFT (BAG) 1 UNIT
1	CJ3FIPPASHT20	J3 FIP BK CVR SH 20T 2" THICK	3	UPH1781B	FIP, SOFT (BAG) 1 UNIT
2	SJ3PASHS14	J3 SHELL, POS SH 14W STD	3	UPH1784B	FIP, MEDIUM (BAG) 1 UNIT
2	SJ3PASHS16	J3 SHELL, POS SH 16W STD			

NOTE: Parts only, for complete Backrest see Jay Zip Back order form.

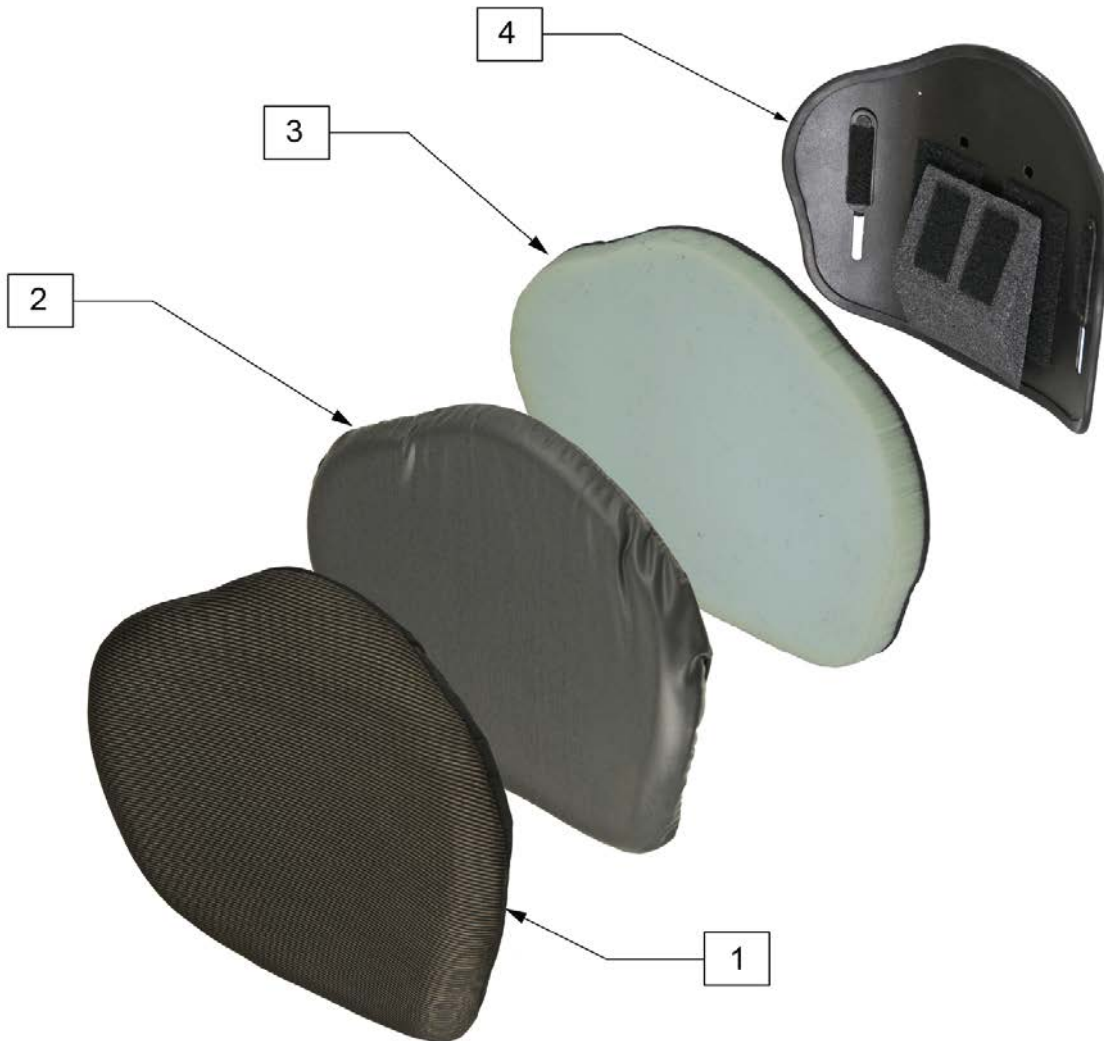

Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJZIPB08M	COVER JAY ZIP BACK 8W MED	1	CJZIPB14X	COVER JAY ZIP BACK 14W XTALL
1	CJZIPB08S	COVER JAY ZIP BACK 8W SHORT	2	ICJZIPB08M	IN COV JAY ZIP BACK 8W MED
1	CJZIPB08T	COVER JAY ZIP BACK 8W TALL	2	ICJZIPB08S	IN COV JAY ZIP BACK 8W SHORT
1	CJZIPB08X	COVER JAY ZIP BACK 8W XTALL	2	ICJZIPB08T	IN COV JAY ZIP BACK 8W TALL
1	CJZIPB10M	COVER JAY ZIP BACK 10W MED	2	ICJZIPB08X	IN COV JAY ZIP BACK 8W XTALL
1	CJZIPB10S	COVER JAY ZIP BACK 10W SHORT	2	ICJZIPB10M	IN COV JAY ZIP BACK 10W MED
1	CJZIPB10T	COVER JAY ZIP BACK 10W TALL	2	ICJZIPB10S	IN COV JAY ZIP BACK 10W SHORT
1	CJZIPB10X	COVER JAY ZIP BACK 10W XTALL	2	ICJZIPB10T	IN COV JAY ZIP BACK 10W TALL
1	CJZIPB12M	COVER JAY ZIP BACK 12W MED	2	ICJZIPB10X	IN COV JAY ZIP BACK 10W XTALL
1	CJZIPB12S	COVER JAY ZIP BACK 12W SHORT	2	ICJZIPB12M	IN COV JAY ZIP BACK 12W MED
1	CJZIPB12T	COVER JAY ZIP BACK 12W TALL	2	ICJZIPB12S	IN COV JAY ZIP BACK 12W SHORT
1	CJZIPB12X	COVER JAY ZIP BACK 12W XTALL	2	ICJZIPB12T	IN COV JAY ZIP BACK 12W TALL
1	CJZIPB14M	COVER JAY ZIP BACK 14W MED	2	ICJZIPB12X	IN COV JAY ZIP BACK 12W XTALL
1	CJZIPB14S	COVER JAY ZIP BACK 14W SHORT	2	ICJZIPB14M	IN COV JAY ZIP BACK 14W MED
1	CJZIPB14T	COVER JAY ZIP BACK 14W TALL	2	ICJZIPB14S	IN COV JAY ZIP BACK 14W SHORT

Pos.	Item Number	Description	Pos.	Item Number	Description
2	ICJZIPB14T	IN COV JAY ZIP BACK 14W TALL	3	BJZIPB14X	FOAM JAY ZIP BACK 14W XTALL
2	ICJZIPB14X	IN COV JAY ZIP BACK 14W XTALL	4	SJZIPB08M	SHELL JAY ZIP BACK 8W MED
3	BJZIPB08M	FOAM JAY ZIP BACK 8W MED	4	SJZIPB08S	SHELL JAY ZIP BACK 8W SHORT
3	BJZIPB08S	FOAM JAY ZIP BACK 8W SHORT	4	SJZIPB08T	SHELL JAY ZIP BACK 8W TALL
3	BJZIPB08T	FOAM JAY ZIP BACK 8W TALL	4	SJZIPB08X	SHELL JAY ZIP BACK 8W XTALL
3	BJZIPB08X	FOAM JAY ZIP BACK 8W XTALL	4	SJZIPB10M	SHELL JAY ZIP BACK 10W MED
3	BJZIPB10M	FOAM JAY ZIP BACK 10W MED	4	SJZIPB10S	SHELL JAY ZIP BACK 10W SHORT
3	BJZIPB10S	FOAM JAY ZIP BACK 10W SHORT	4	SJZIPB10T	SHELL JAY ZIP BACK 10W TALL
3	BJZIPB10T	FOAM JAY ZIP BACK 10W TALL	4	SJZIPB10X	SHELL JAY ZIP BACK 10W XTALL
3	BJZIPB10X	FOAM JAY ZIP BACK 10W XTALL	4	SJZIPB12M	SHELL JAY ZIP BACK 12W MED
3	BJZIPB12M	FOAM JAY ZIP BACK 12W MED	4	SJZIPB12S	SHELL JAY ZIP BACK 12W SHORT
3	BJZIPB12S	FOAM JAY ZIP BACK 12W SHORT	4	SJZIPB12T	SHELL JAY ZIP BACK 12W TALL
3	BJZIPB12T	FOAM JAY ZIP BACK 12W TALL	4	SJZIPB12X	SHELL JAY ZIP BACK 12W XTALL
3	BJZIPB12X	FOAM JAY ZIP BACK 12W XTALL	4	SJZIPB14M	SHELL JAY ZIP BACK 14W MED
3	BJZIPB14M	FOAM JAY ZIP BACK 14W MED	4	SJZIPB14S	SHELL JAY ZIP BACK 14W SHORT
3	BJZIPB14S	FOAM JAY ZIP BACK 14W SHORT	4	SJZIPB14T	SHELL JAY ZIP BACK 14W TALL
3	BJZIPB14T	FOAM JAY ZIP BACK 14W TALL	4	SJZIPB14X	SHELL JAY ZIP BACK 14W XTALL

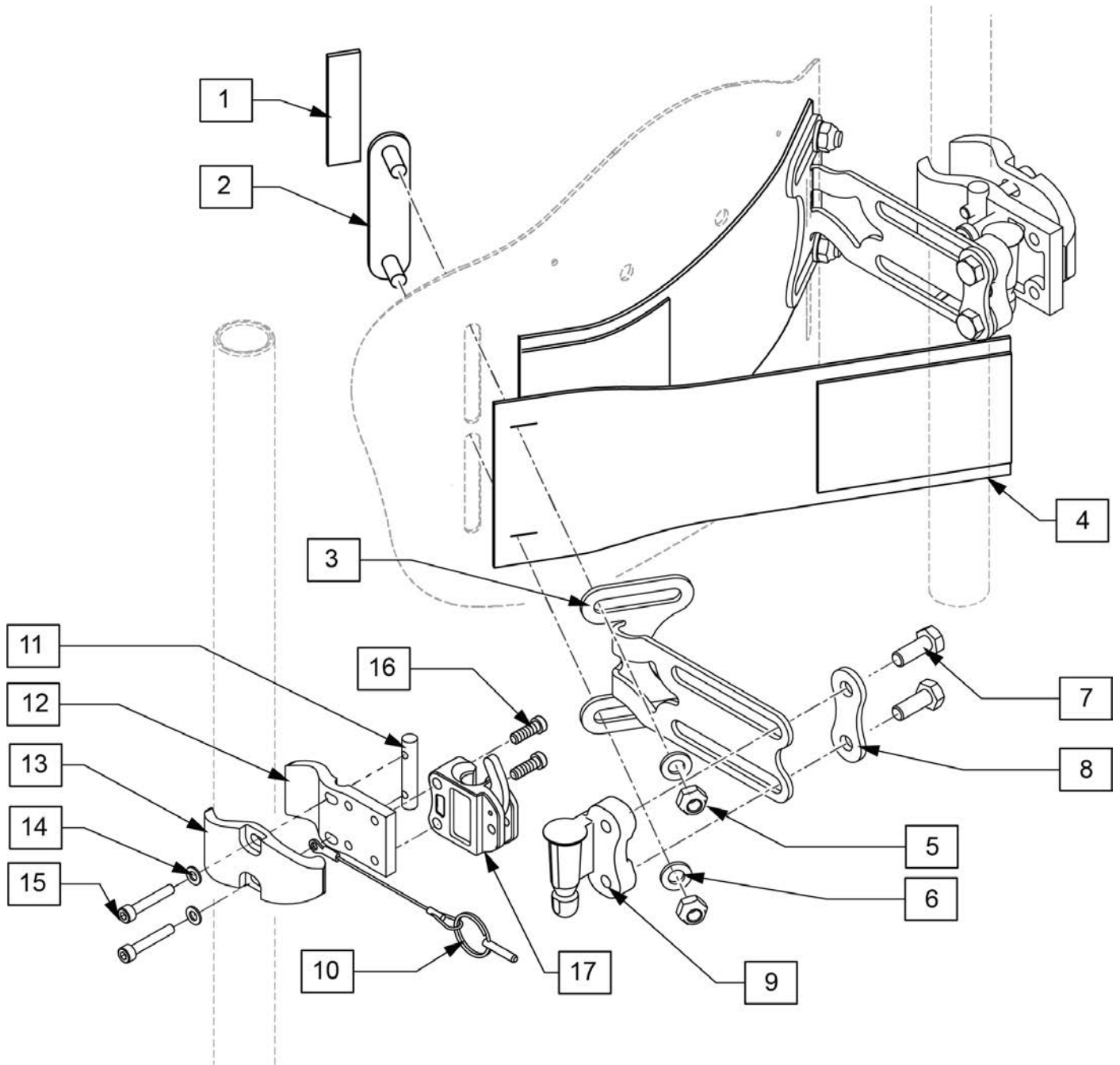


Pos.	Item Number	Description	Remarks
4	250093	ZIP COVER HARDWARE NEOPRENE	For right or left side
7-9	159400	J3 MOUNTING PIN ASSM RIGHT	
7-9	159401	J3 MOUNTING PIN ASSM LEFT	
16,17	159398	J3 RH LATCH RECEIVER ASSM RIGHT	
16,17	159399	3 RH LATCH RECEIVER ASSM LEFT	
1-3,5-17	250663	JAY ZIP RH HDWR AND BRACKET KIT	


NOTE: Parts only, for complete Backrest see JAY ZIP Back order form.

NOTE: Parts only, for complete Backrest see JAY ZIP Back order form.


1		VELCRO	2
2		NUT PLATE M6 X 15 STUD X 51	2
5		NUT HEX NYLOCK M6 BLACK	8
6		HHCS M6 X 1 X 16 NP ROHS	4
8		WASHER FLAT M6	10
9		STUD PIVOT M6	2
11		HHCS M6 X 1 X 18 PL ROHS	2
12		STUD LATCH M6	2
13		NUT BARREL M6 X 9.53 X 12	6

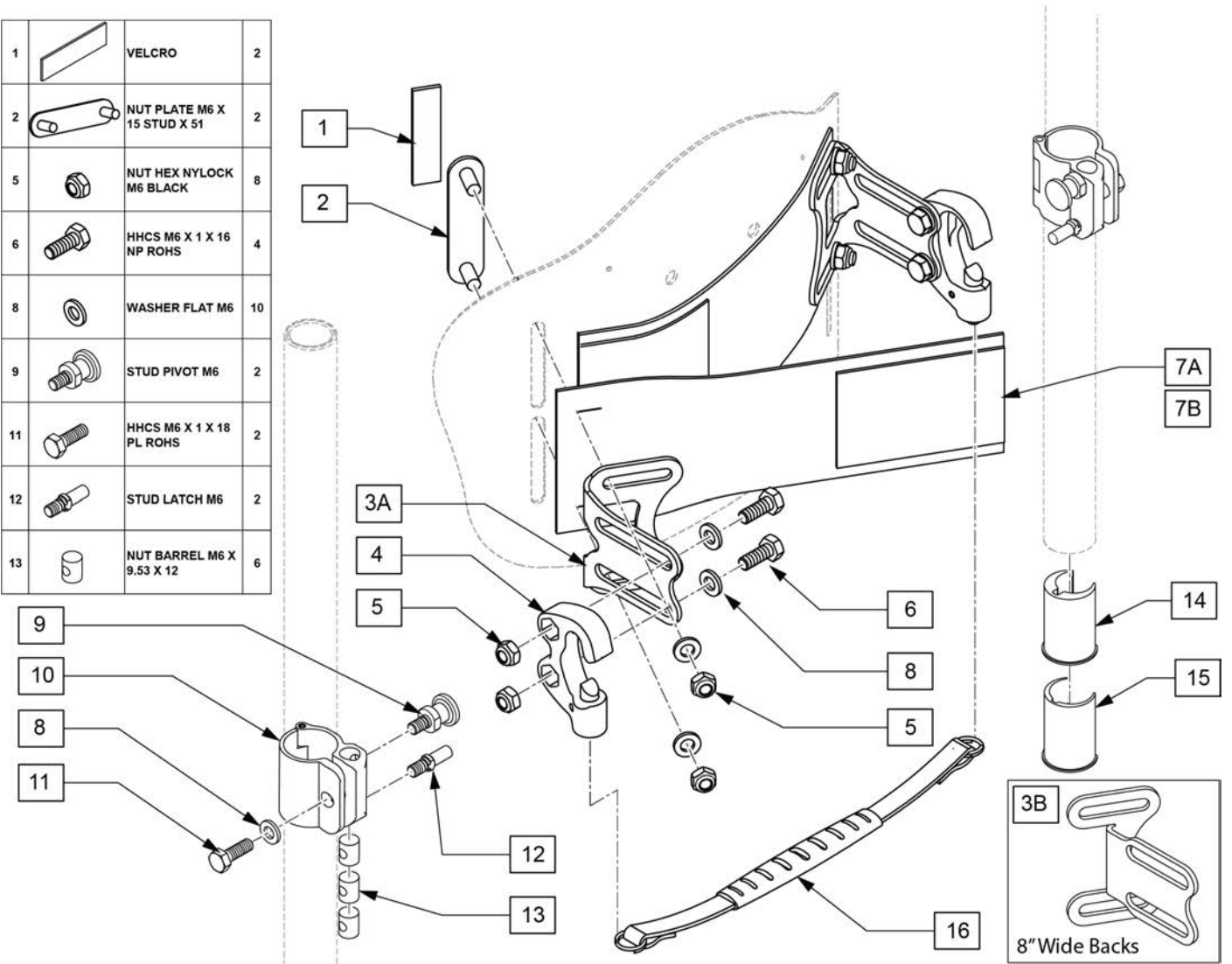


Pos.	Item Number	Description	Remarks
5	210084	NUT HEX NYLOCK M6 BLACK	
7A	120982	COVER HRDWR KIT SM PR	Includes 2 - for 8 & 10 wide backs
7B	120983	COVER HRDWR KIT LG PR	Includes 2 - for 12 & 14 wide backs
8	250104	WASHER FLAT M6	
16	120985	ZIP BACK LANYARD 8-9W	
16	120986	ZIP BACK LANYARD 10-11W	
16	120987	ZIP BACK LANYARD 12-13W	
16	120988	ZIP BACK LANYARD 14-15W	
8-13	120979	TUBE CLAMP ASSM 1IN PR	Pair
14-15	120984	CLAMP INSERTS KIT 2 PR	
1-3A,5,8	120981	SHELL MOUNT BRKT KIT PR	
1-3B,5,8	121111	8W SHELL MOUNT BRKT KIT PR	
4-6,8	120980	MOUNT RECEIVER KIT PR	
1,2,5,6,8,9,11-13	120978	JAY ZIP BACK FASTNER KIT	
1-3B,4-7A,8-16	250577	ZIP C-CLAMP SHELL ASSY 8W	complete pair assembly kit
1-3A,4-7B,8-16	250582	ZIP C-CLAMP SHELL ASSY 10W	complete pair assembly kit
1-3A,4-7B,8-16	250583	ZIP C-CLAMP SHELL ASSY 12W	complete pair assembly kit
1-3A,4-7B,8-16	250584	ZIP C-CLAMP SHELL ASSY 14W	complete pair assembly kit

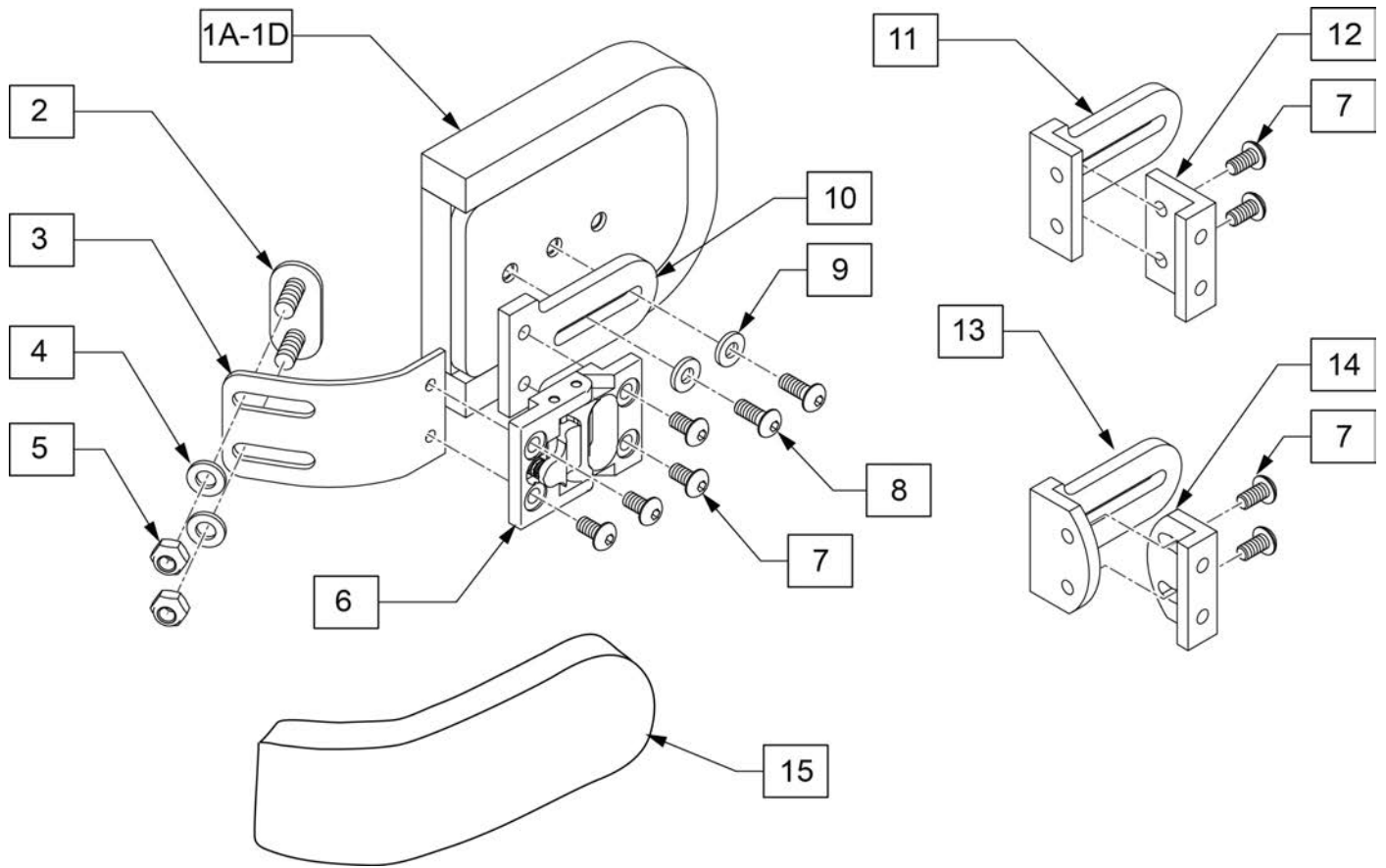


Pos.	Item Number	Description	Remarks
1A	120992	THOR SUPP PAD FLAT 3X4	single pad
1B	120993	THOR SUPP PAD FLAT 4X5	single pad
1C	120994	THOR SUPP PAD CURVED 3X4	single pad
1D	120995	THOR SUPP PAD CURVED 4X5	single pad
4	250104	WASHER FLAT M6	
5	210084	NUT HEX NYLOCK M6 BLACK	
6	PRT7500	LK, HINGE ASSEMBLY	
7	226018-QU	10-32 X 3/8 BH C/S W/PATCH BLK	
8	XM2141S	10-32X.50 BHCS W/PATCH	
9	113282S	WASHER FLAT AN #10 ROHS	
10	PRT7625	LK, PAD MOUNT	
11	PRT7627	LK, PAD MOUNT PLATE, OFFSET	
12	PRT7626	LK, 1" OFF ADAPT FOR E-Z	
13	PRT7653	OFFSET PAD MOUNT 20 DEG, LK	
14	PRT7651	1" OFFSET MOUNT 20 DEG, LK	
15	COV2650	LAT BRKT FABRIC COVER,LK,SHORT	
2-5	120989	ZIP BACK SA LAT BRKT KIT PR	

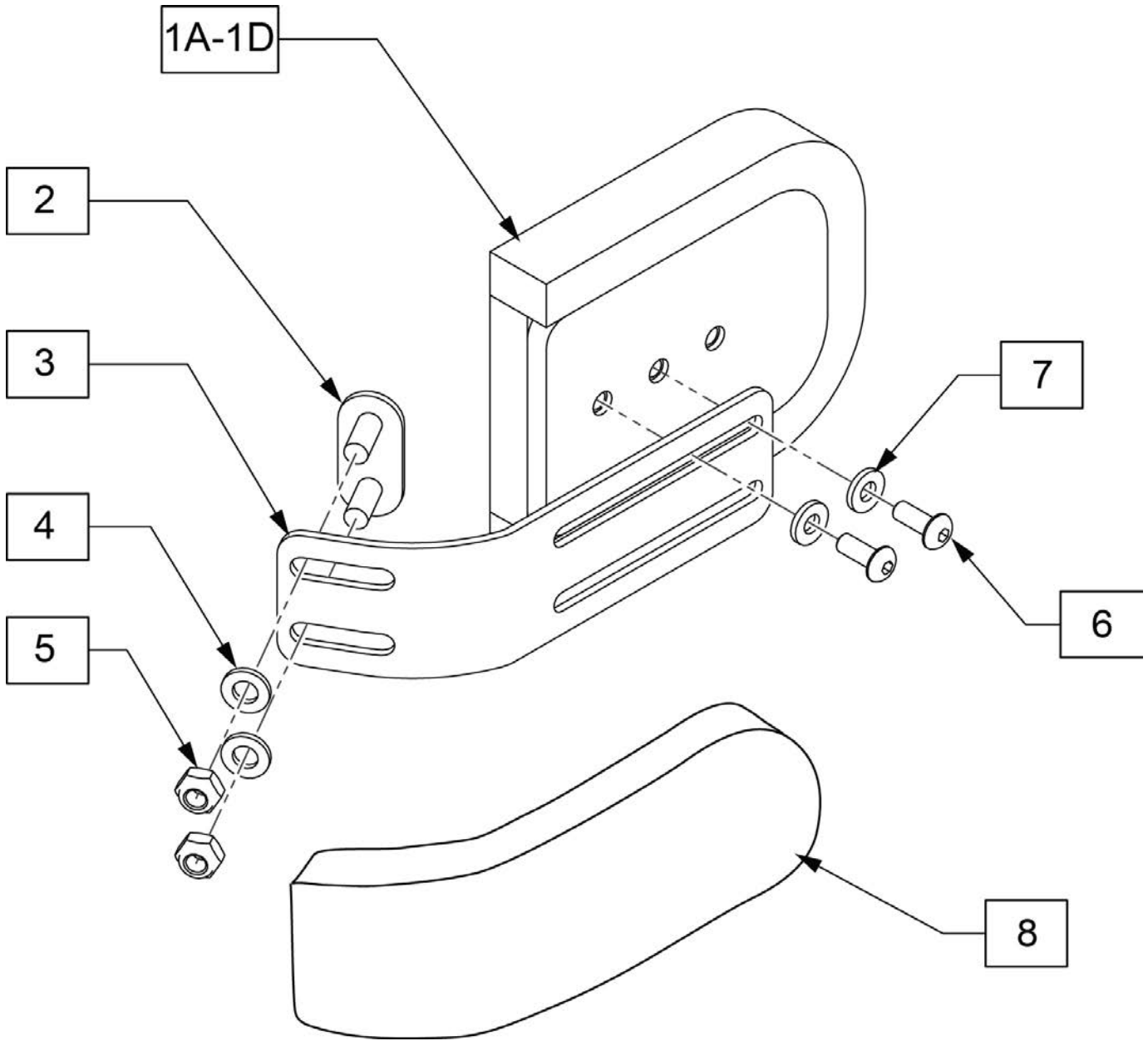

Pos.	Item Number	Description	Remarks
1A	120992	THOR SUPP PAD FLAT 3X4	single pad
1B	120993	THOR SUPP PAD FLAT 4X5	single pad
1C	120994	THOR SUPP PAD CURVED 3X4	single pad
1D	120995	THOR SUPP PAD CURVED 4X5	single pad
4	250104	WASHER FLAT M6	
5	210084	NUT HEX NYLOCK M6 BLACK	
6	XM2141S	10-32X.50 BHCS W/PATCH	
7	113282S	WASHER FLAT AN #10 ROHS	
8	COV2650	LAT BRKT FABRIC COVER,LK,SHORT	
2-5	120990	ZIP BACK FX LAT BRKT KIT PR	

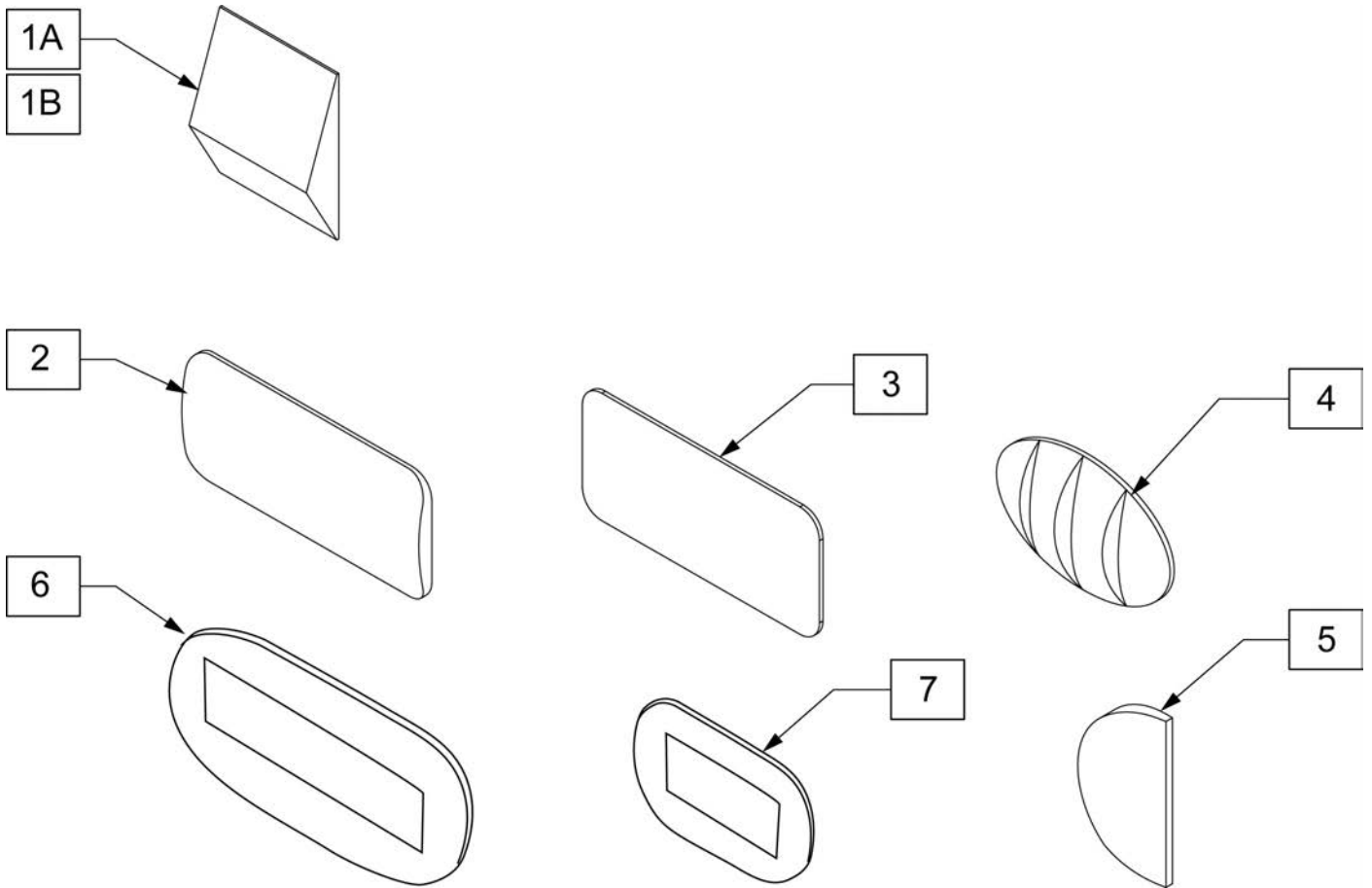


Pos.	Item Number	Description	Remarks
1A	103714-001S	LUMBAR SUPP SM BLK 3.5W W/VEL	
1B	103714-005S	LUMBAR SUPP SM BLK 1.8W W/VEL	
2	J3SPNCR	CURVED SHIM	
3	J3SPNFL	FLAT SHIM	
4	J3SPNLP	POS/LATERAL PELVIC	
5	J3SPNLS	SHIM, LATERAL	
6	J3SPNBL	BEAN BAG, LARGE	Approx 5.25" X 10"
7	J3SPNBS	BEAN BAG, SMALL	Approx 3" X 6"

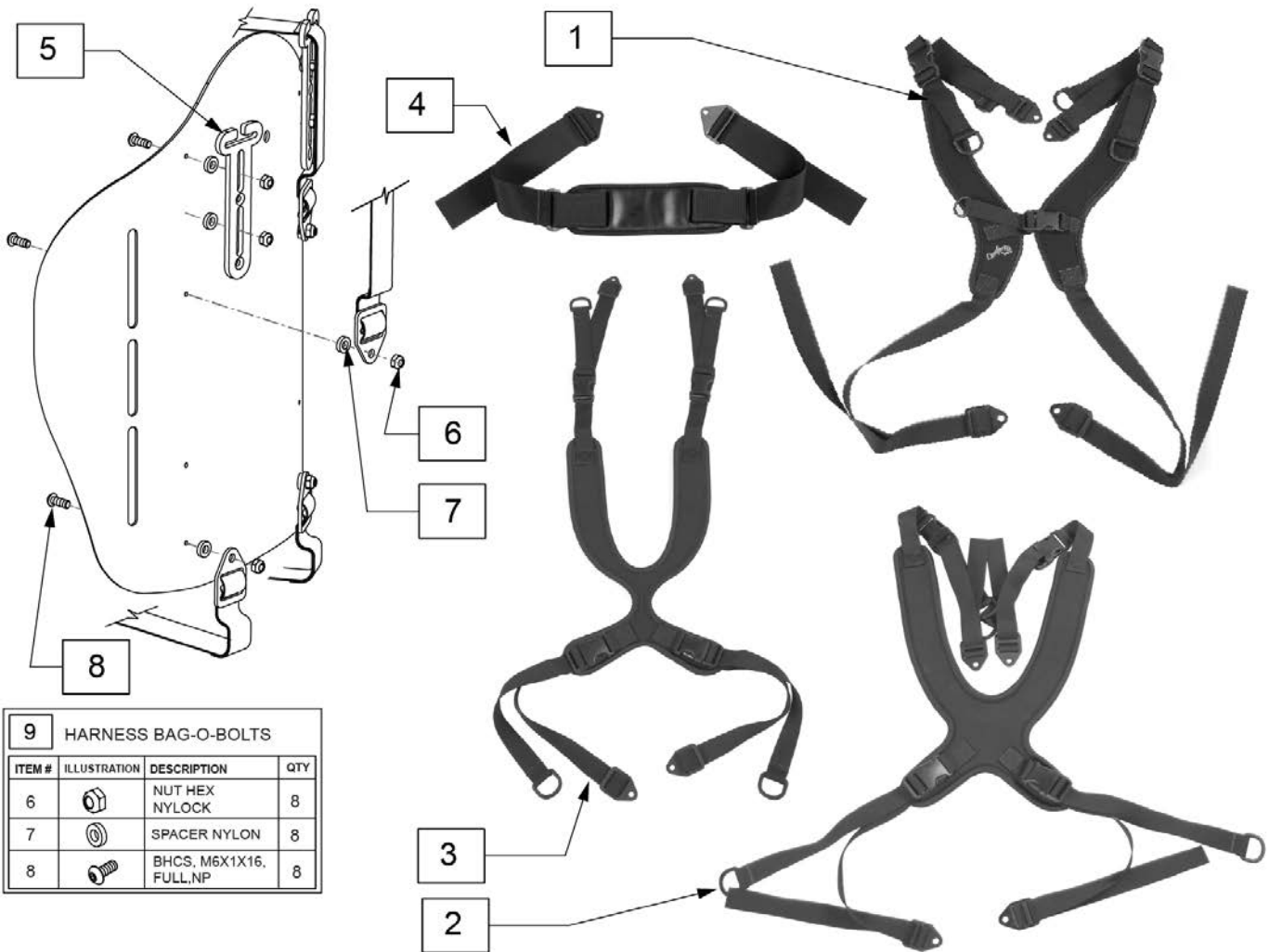


9 HARNESS BAG-O-BOLTS			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
6		NUT HEX NYLOCK	8
7		SPACER NYLON	8
8		BHCS, M6X1X16, FULL.NP	8

Pos.	Item Number	Description	Pos.	Item Number	Description
1	ATS9407	CENTER OPENING W/ BUCKLE (EI)	3	ATS9100	CONTOURED - DYNAMIC PULL DOWN (XS)
1	ATS9400	CENTER OPENING W/ BUCKLE (XS)	3	ATS9101	CONTOURED - DYNAMIC PULL DOWN (S)
1	ATS9401	CENTER OPENING W/ BUCKLE (S)	3	ATS9102	CONTOURED - DYNAMIC PULL DOWN (M)
1	ATS9402	CENTER OPENING W/ BUCKLE (M)	3	ATS9300	CONTOURED - STRUCTURED PULL DOWN (XS)
2	ATS8107	CLASSIC - DYNAMIC PULL DOWN (EI)	3	ATS9301	CONTOURED - STRUCTURED PULL DOWN (S)
2	ATS8100	CLASSIC - DYNAMIC PULL DOWN (XS)	3	ATS9302	CONTOURED - STRUCTURED PULL DOWN (M)
2	ATS8101	CLASSIC - DYNAMIC PULL DOWN (S)	4	ATS4050	PADDED ANTERIOR TRUNK STRAPS (XS)
2	ATS8102	CLASSIC - DYNAMIC PULL DOWN (M)	4	ATS4000	PADDED ANTERIOR TRUNK STRAPS (S)
2	ATS8300	CLASSIC - STRUCTURED PULL DOWN (XS)	4	ATS4100	PADDED ANTERIOR TRUNK STRAPS (M)
2	ATS8301	CLASSIC - STRUCTURED PULL DOWN (S)	5	J3GPGS	GOAL POST SHOULDER STRAP GUIDE
2	ATS8302	CLASSIC - STRUCTURED PULL DOWN (M)	9	J3HHP	HARNESS BAG-O-BOLTS
3	ATS9107	CONTOURED - DYNAMIC PULL DOWN (EI)			

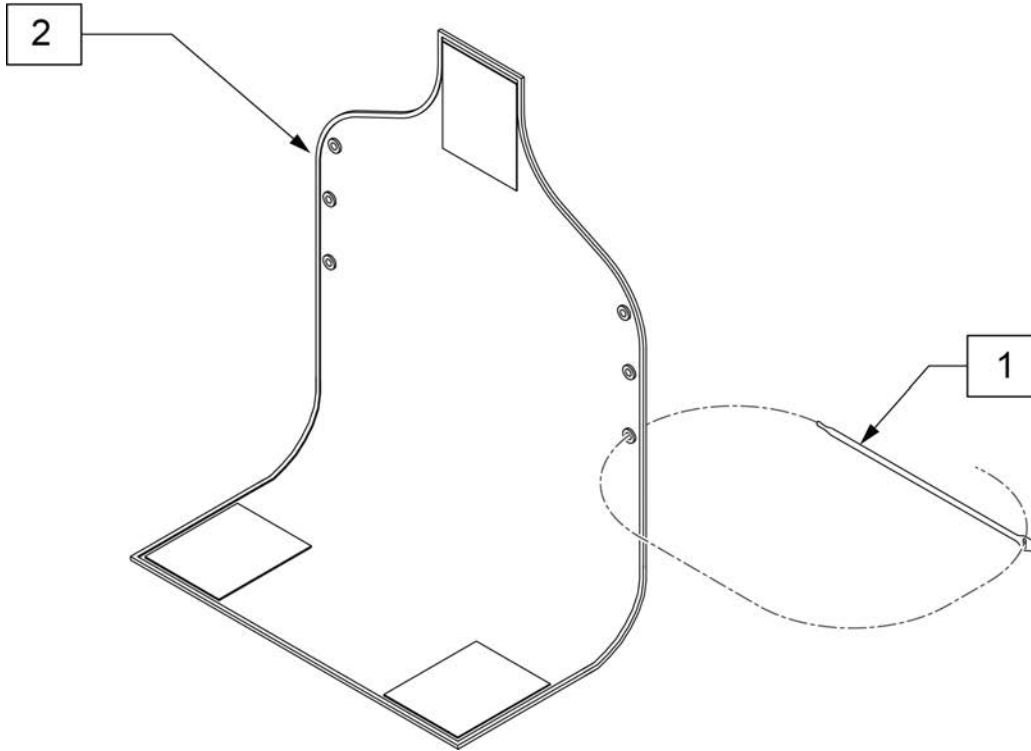


MEASURING GUIDE

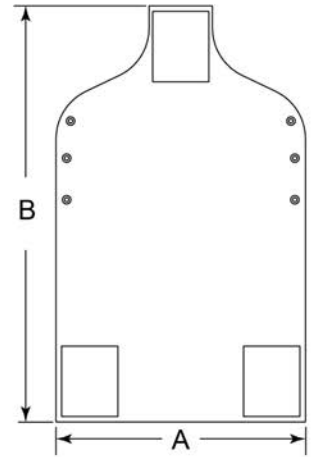


P/N	DIM A	DIM B
116522-001	11 3/4"	25 1/2"
116522-002	13 3/4"	25 1/2"
116522-003	15 3/4"	27 1/2"
116522-004	17 3/4"	29 1/2"
116522-005	19 3/4"	31 1/2"
116522-006	21 3/4"	31 1/2"
116522-007	23 3/4"	33 1/2"
116522-008	25 3/4"	33 1/2"

Pos.	Item Number	Description	Remarks
1	116527S	VELCRO TIE,1 WRAP.75"X12",BLK	
1,2	117096	VANITY FLAP, J3 BACK 12W	Includes 4 velcro ties
1,2	117097	VANITY FLAP, J3 BACK 14W	Includes 4 velcro ties
1,2	117098	VANITY FLAP, J3 BACK 16W	Includes 4 velcro ties
1,2	117099	VANITY FLAP, J3 BACK 18W	Includes 4 velcro ties
1,2	117100	VANITY FLAP, J3 BACK 20W	Includes 4 velcro ties
1,2	117101	VANITY FLAP, J3 BACK 22W	Includes 4 velcro ties
1,2	117102	VANITY FLAP, J3 BACK 24W	Includes 4 velcro ties
1,2	117103	VANITY FLAP, J3 BACK 26W	Includes 4 velcro ties
1,2	VF08	VANITY FLAP 8-9 IN	Includes 4 velcro ties
1,2	VF10	VANITY FLAP 10-11 IN	Includes 4 velcro ties
1,2	VF12	VANITY FLAP 12-13 IN	Includes 4 velcro ties
1,2	VF14	VANITY FLAP 14-15 IN	Includes 4 velcro ties

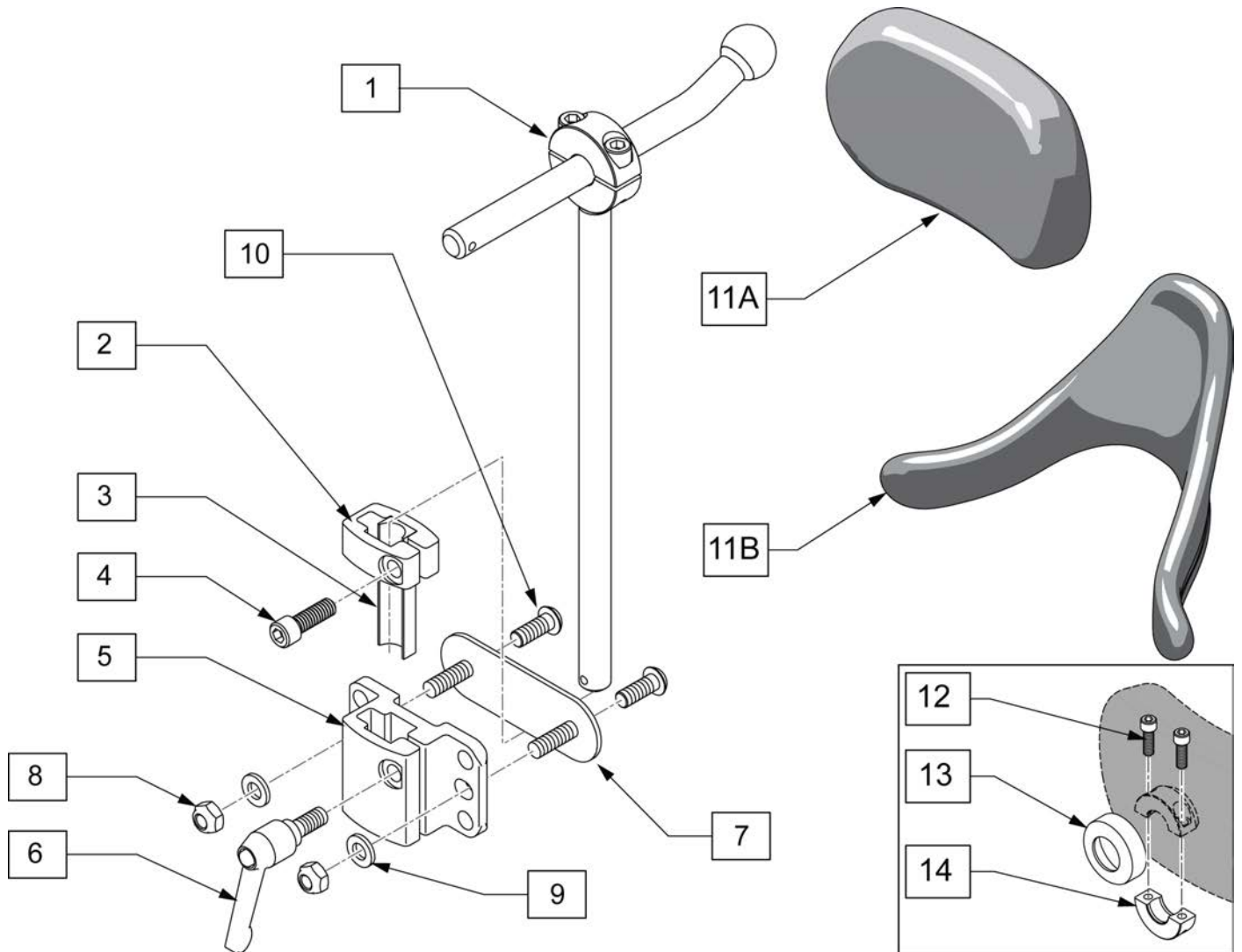


Pos.	Item Number	Description
1	OX7030	ONYX CORE MOUNT HDW, PED
5	MW070230S	MT .5 SQR TUBING RECIEVER BLK
6	M0030	KNOB PLAS ADJ LVR 1/4-20X20MM
7	102971-001S	NUT PLATE M6 STUD SHORT X 51
8	210084	NUT HEX NYLOCK M6 BLACK
9	250104	WASHER FLAT M6
10	103185-006	BHCS M6X1X16 NP BLK
1-10	120452S	ZIP HEADREST ASSM
2-4	120991	HDRST STOP BLK KIT
12	MF014600	1/4-28X5/8" SHCS- COLLAR BOLT
13	MC100-WB	COVER 3/4" COLLAR/VINYL BLACK
14	MW027100	COLLAR HALF RAD 3/4" THD. BLCK
11A-14	P600L	OCCIP PAD, ADULT, LYCRA
11A-14	P800L	PLUSH PAD, 8 IN LYCRA
11B-14	INFT200LS	PAD INF CONTRD CRADLE LYCRA
11B-14	P200L	CONT CRADLE PAD, SM, LYCRA

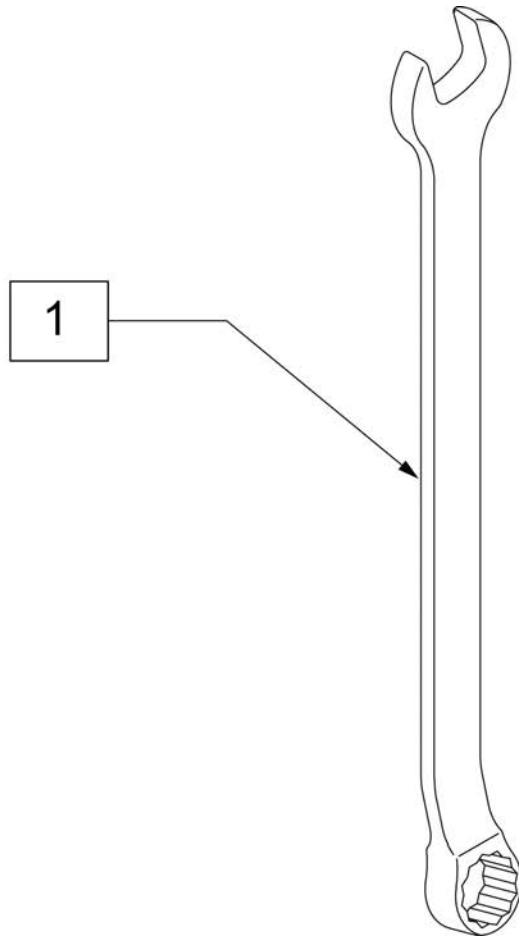

Pos.	Item Number	Description
1	104549S	COMBO WRENCH 10MM



MEASURING GUIDE

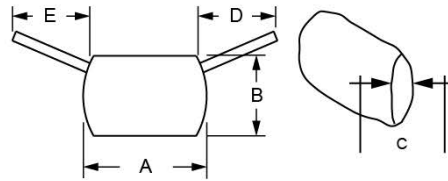

Part Number	Dim A (Width)	Dim B (Height)	Dim C (Thickness)	Dim D (Right Side Strap*)	Dim E (Left Side Strap*)
2250	15"	8.75"	2"	7"	7"
2251-MJ	11"	7"	2"	6.5"	15"

* Strap side is located as if you are resting against the back with the lumbar support on it.

Pos.	Item Number	Description	Remarks
1A	2250	LUMBAR SUPPORT SOFT COMBI	Adult
1B	2251-MJ	LUMBAR SUPPORT SFT COMBI JUNR	Junior



Pos.	Item Number	Description	Pos.	Item Number	Description
1	C331	COVER BASIC INC 14X14	1	C349	COVER BASIC INC 24X24
1	C308	COVER BASIC INCONTINENT 14X16	2	B331-MJ	BASE BASIC CUSHION (14X14)
1	C332	COVER BASIC INC 14X18	2	B308-MJ	FOAM BASE BASIC 14X16
1	C356	COVER BASIC INCONTINENT 15X16	2	B332-MJ	BASE BASIC CUSHION 14X18
1	C333	COVER BASIC INC 15X18	2	B356	FOAM BASE BASIC 15X16
1	C305	COVER BASIC INCONTINENT 16X16	2	B333-MJ	BASE BASIC CUSHION (15X18)
1	C334-MJ	COVER BASIC INC 16X17	2	B305	FOAM BASE BASIC 16X16
1	C306-MJ	COVER BASIC INCONTINENT 16X18	2	B334-MJ	BASE BASIC CUSHION (16X17)
1	C376	COVER BASIC INCONTINENT 17X16	2	B306	FOAM BASE BASIC 16X18
1	C335	COVER BASIC INC 17X17	2	B376	FOAM BASE BASIC 17X16
1	C336	COVER BASIC INC 17X18	2	B335-MJ	BASE BASIC CUSHION (17X17)
1	C300-MJ	COVER BASIC INCONTINENT 18X16	2	B336	BASE BASIC CUSHION (17X18)
1	C337	COVER BASIC INC 18X17	2	B300	FOAM BASE BASIC 18X16
1	C301	COVER BASIC INCONTINENT 18X18	2	B337	BASE BASIC CUSHION (18X17)
1	C338	COVER BASIC INC 18X20	2	B301	FOAM BASE BASIC 18X18
1	C339	COVER BASIC INC 19X16	2	B338	BASE BASIC CUSHION (18X20)
1	C341	COVER BASIC INC 19X18	2	B339	BASE BASIC CUSHION (19X16)
1	C343	COVER BASIC INC 19X20	2	B341-MJ	BASE BASIC CUSHION (19X18)
1	C344	COVER BASIC INC 20X16	2	B343-MJ	BASE BASIC CUSHION (19X20)
1	C309	COVER BASIC 20X18 INC	2	B344-MJ	BASE BASIC CUSHION (20X16)
1	C345	COVER BASIC INC 20X20	2	B309	FOAM BASE BASIC 20X18
1	C328	COVER BASIC 22X18 INC	2	B345	BASE BASIC CUSHION 20X20
1	C320	COVER BASIC 22X20 INC	2	B328	FOAM BASE 22X18 BASIC
1	C346	COVER BASIC INC 22X22	2	B320	FOAM BASE 22X20 BASIC
1	C348	COVER BASIC 24X18 INC	2	B346	BASE BASIC CUSHION 22X22
1	C340	COVER BASIC 24X20 INC	2	B348	FOAM BASE 24X18 BASIC
1	C347	COVER BASIC INC 24X22	2	B340	FOAM BASE 24X20 BASIC



Pos.	Item Number	Description	Pos.	Item Number	Description
2	B347	BASE BASIC CUSHION 24X22	2	B349-MJ	BASE BASIC CUSHION (24X24)



Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJBASICP1414	OUTER CVR BASIC PRO 14X14	1	CJBASICP1918	OUTER CVR BASIC PRO 19X18
1	CJBASICP1416	OUTER CVR BASIC PRO 14X16	1	CJBASICP1920	OUTER CVR BASIC PRO 19X20
1	CJBASICP1418	OUTER CVR BASIC PRO 14X18	1	CJBASICP2016	OUTER CVR BASIC PRO 20X16
1	CJBASICP1518	OUTER CVR BASIC PRO 15X18	1	CJBASICP2018	OUTER CVR BASIC PRO 20X18
1	CJBASICP15516	OUTER CVR BASIC PRO 15.5X16	1	CJBASICP2020	OUTER CVR BASIC PRO 20X20
1	CJBASICP15518	OUTER CVR BASIC PRO 15.5X18	1	CJBASICP2218	OUTER CVR BASIC PRO 22X18
1	CJBASICP1617	OUTER CVR BASIC PRO 16X17	1	CJBASICP2220	OUTER CVR BASIC PRO 22X20
1	CJBASICP1618	OUTER CVR BASIC PRO 16X18	1	CJBASICP2222	OUTER CVR BASIC PRO 22X22
1	CJBASICP1620	OUTER CVR BASIC PRO 16X20	1	CJBASICP2418	OUTER CVR BASIC PRO 24X18
1	CJBASICP1716	OUTER CVR BASIC PRO 17X16	1	CJBASICP2420	OUTER CVR BASIC PRO 24X20
1	CJBASICP1717	OUTER CVR BASIC PRO 17X17	1	CJBASICP2422	OUTER CVR BASIC PRO 24X22
1	CJBASICP1718	OUTER CVR BASIC PRO 17X18	1	CJBASICP2424	OUTER CVR BASIC PRO 24X24
1	CJBASICP1816	OUTER CVR BASIC PRO 18X16	2	ICJBASICP1414	INNER CVR BASIC PRO 14X14
1	CJBASICP1817	OUTER CVR BASIC PRO 18X17	2	ICJBASICP1416	INNER CVR BASIC PRO 14X16
1	CJBASICP1818	OUTER CVR BASIC PRO 18X18	2	ICJBASICP1418	INNER CVR BASIC PRO 14X18
1	CJBASICP1820	OUTER CVR BASIC PRO 18X20	2	ICJBASICP1518	INNER CVR BASIC PRO 15X18
1	CJBASICP1916	OUTER CVR BASIC PRO 19X16	2	ICJBASICP15516	INNER CVR BASIC PRO 15.5X16
1	CJBASICP1917	OUTER CVR BASIC PRO 19X17	2	ICJBASICP15518	INNER CVR BASIC PRO 15.5X18


Pos.	Item Number	Description	Pos.	Item Number	Description
2	ICJBASICP1617	INNER CVR BASIC PRO 16X17	3	BJBASICP1518	BASE BASIC PRO 15X18
2	ICJBASICP1618	INNER CVR BASIC PRO 16X18	3	BJBASICP15516	BASE BASIC PRO 15.5X16
2	ICJBASICP1620	INNER CVR BASIC PRO 16X20	3	BJBASICP15518	BASE BASIC PRO 15.5X18
2	ICJBASICP1716	INNER CVR BASIC PRO 17X16	3	BJBASICP1617	BASE BASIC PRO 16X17
2	ICJBASICP1717	INNER CVR BASIC PRO 17X17	3	BJBASICP1618	BASE BASIC PRO 16X18
2	ICJBASICP1718	INNER CVR BASIC PRO 17X18	3	BJBASICP1620	BASE BASIC PRO 16X20
2	ICJBASICP1816	INNER CVR BASIC PRO 18X16	3	BJBASICP1716	BASE BASIC PRO 17X16
2	ICJBASICP1817	INNER CVR BASIC PRO 18X17	3	BJBASICP1717	BASE BASIC PRO 17X17
2	ICJBASICP1818	INNER CVR BASIC PRO 18X18	3	BJBASICP1718	BASE BASIC PRO 17X18
2	ICJBASICP1820	INNER CVR BASIC PRO 18X20	3	BJBASICP1816	BASE BASIC PRO 18X16
2	ICJBASICP1916	INNER CVR BASIC PRO 19X16	3	BJBASICP1817	BASE BASIC PRO 18X17
2	ICJBASICP1917	INNER CVR BASIC PRO 19X17	3	BJBASICP1818	BASE BASIC PRO 18X18
2	ICJBASICP1918	INNER CVR BASIC PRO 19X18	3	BJBASICP1820	BASE BASIC PRO 18X20
2	ICJBASICP1920	INNER CVR BASIC PRO 19X20	3	BJBASICP1916	BASE BASIC PRO 19X16
2	ICJBASICP2016	INNER CVR BASIC PRO 20X16	3	BJBASICP1917	BASE BASIC PRO 19X17
2	ICJBASICP2018	INNER CVR BASIC PRO 20X18	3	BJBASICP1918	BASE BASIC PRO 19X18
2	ICJBASICP2020	INNER CVR BASIC PRO 20X20	3	BJBASICP1920	BASE BASIC PRO 19X20
2	ICJBASICP2218	INNER CVR BASIC PRO 22X18	3	BJBASICP2016	BASE BASIC PRO 20X16
2	ICJBASICP2220	INNER CVR BASIC PRO 22X20	3	BJBASICP2018	BASE BASIC PRO 20X18
2	ICJBASICP2222	INNER CVR BASIC PRO 22X22	3	BJBASICP2020	BASE BASIC PRO 20X20
2	ICJBASICP2418	INNER CVR BASIC PRO 24X18	3	BJBASICP2218	BASE BASIC PRO 22X18
2	ICJBASICP2420	INNER CVR BASIC PRO 24X20	3	BJBASICP2220	BASE BASIC PRO 22X20
2	ICJBASICP2422	INNER CVR BASIC PRO 24X22	3	BJBASICP2222	BASE BASIC PRO 22X22
2	ICJBASICP2424	INNER CVR BASIC PRO 24X24	3	BJBASICP2418	BASE BASIC PRO 24X18
3	BJBASICP1414	BASE BASIC PRO 14X14	3	BJBASICP2420	BASE BASIC PRO 24X20
3	BJBASICP1416	BASE BASIC PRO 14X16	3	BJBASICP2422	BASE BASIC PRO 24X22
3	BJBASICP1418	BASE BASIC PRO 14X18	3	BJBASICP2424	BASE BASIC PRO 24X24

Note: Cushion length is longer than normal to accommodate posterior pelvic tilt, often associated with a fixed kyphotic posture.

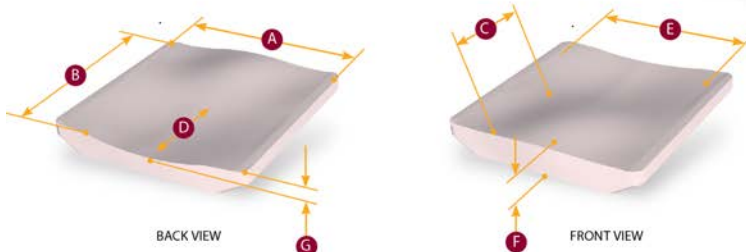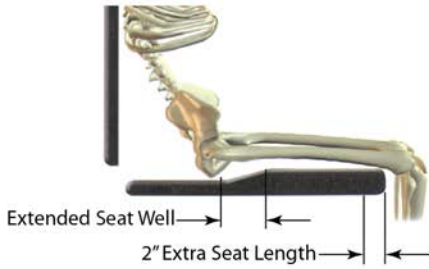

PART #	CHAIR SIZE	ACTUAL WIDTH	ACTUAL LENGTH	FEMORAL SUPPORT LENGTH	PELVIC CONTOUR DEPTH	PELVIC CONTOUR WIDTH	OVERALL HEIGHT	UNLOADED WELL DEPTH
		A	B	C	D	E	F	G
3100B	18x17.5	18.5	17.5	6	8	17	3.75	3.1
3101B	18x19.5	19.25	19.75	7.5	9	17	3.75	2.9
3105B	16x17.5	17	17.75	6.25	8	15	4	2.9
3106B	16x19.5	17.5	19.75	7	9	15.5	4	2.9
3109B	20x19.5	20.75	17.5	6.5	7.75	19.5	3.75	3.1


Pos.	Item Number	Description	Remarks
1	C3100B	COVER JCARE 18X16	
1	C3101B	COVER JCARE 18X18	
1	C3105B	COVER JCARE 16X16	
1	C3106B	COVER JCARE 16X18	
1	C3109B	COVER JCARE 20X18	
2	F3100	JAYCARE FLUID PAD W/BASE	18 X 16
2	F3101	JAYCARE FLUID PAD W/BASE	18 X 18
2	F3105	JAYCARE FLUID PAD W/BASE	16 X 16
2	F3106	JAYCARE FLUID PAD W/BASE	16 X 18
2	F3109	JAYCARE FLUID PAD W/BASE	20 X 18

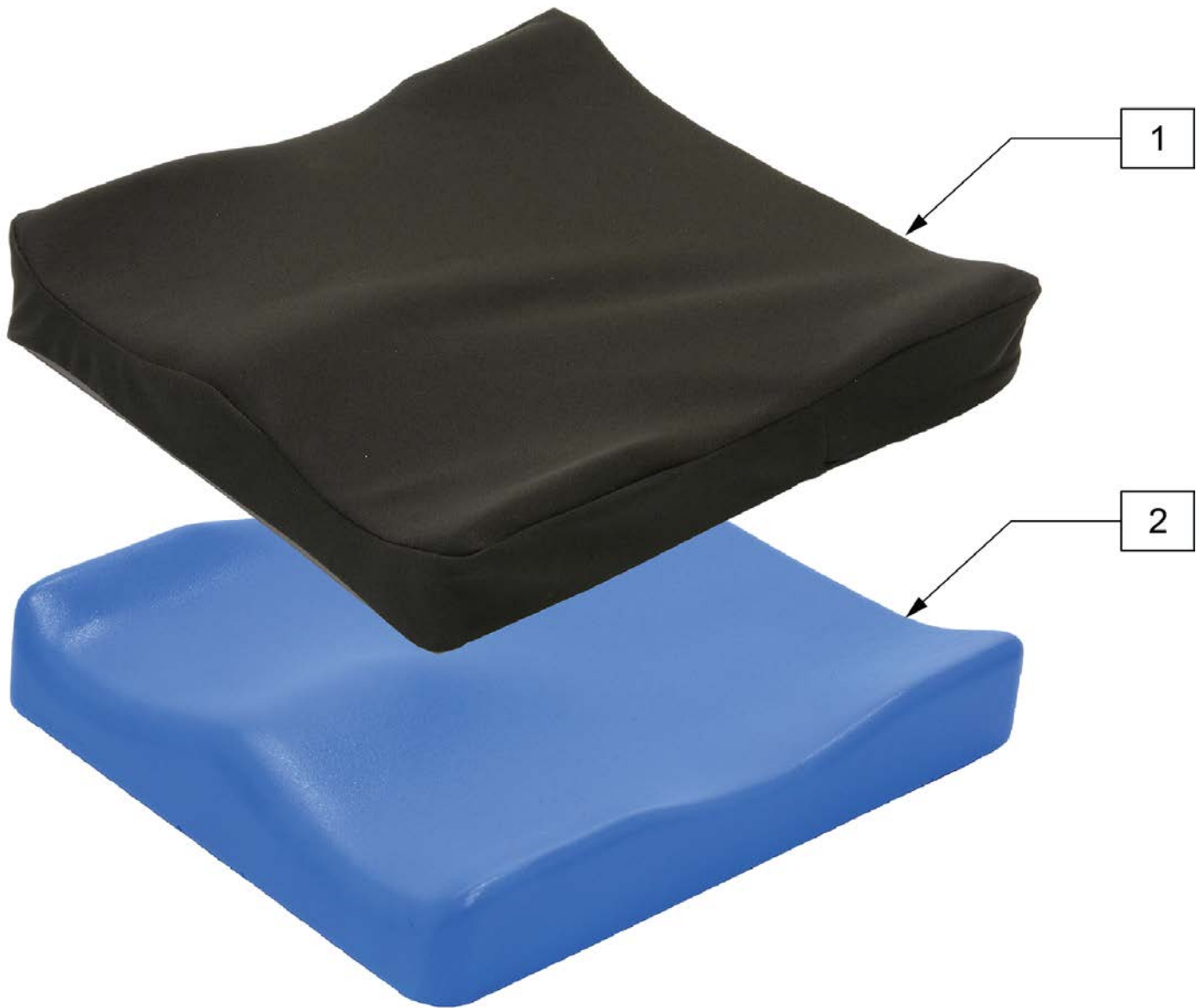

Pos.	Item Number	Description	Remarks
1	C2200P	SOFT COMBI POSITIONING COVER 18x16	
1	C2201P	SOFT COMBI POSITIONING COVER 18x18	
1	C2202P	SOFT COMBI POSITIONING COVER 20x18	
1	C2203P	SOFT COMBI POSITIONING COVER 18x20	
1	C2205P	SOFT COMBI POSITIONING COVER 16x16	
1	C2206P	SOFT COMBI POSITIONING COVER 16x18	actual size 15.5" x18"
1	C2207P	SOFT COMBI POSITIONING COVER 16x20	
1	C2210P	SOFT COMBI POSITIONING COVER 14x14	
1	C2228P	COVER 22X18 SOFT COMBI POS	
1	C2220P	COVER 22X20 SOFT COMBI POS	
1	C2248P	COVER 24X18 SOFT COMBI POS	
1	C2240P	COVER 24X20 SOFT COMBI POS	

NOTE: * FOR WHEELCHAIRS THAT ARE SLIGHTLY NARROWER THAN LISTED DIMENSION, SUCH AS WITH QUICKIE WHEELCHAIRS. THESE PANS ALSO PROVIDE ADDITIONAL CLEARANCE FOR POP-UP FOOTRESTS.


NOTE: ** WHEN ORDERING A DUO FOR A QUICKIE 2 WHEELCHAIR, A SEAT RAIL INSERT STRIP (ITEM 7) MUST BE ORDERED (QUANTITY 2) TO PREVENT THE CLIP FROM CATCHING IN THE UPHOLSTERY SLOT.

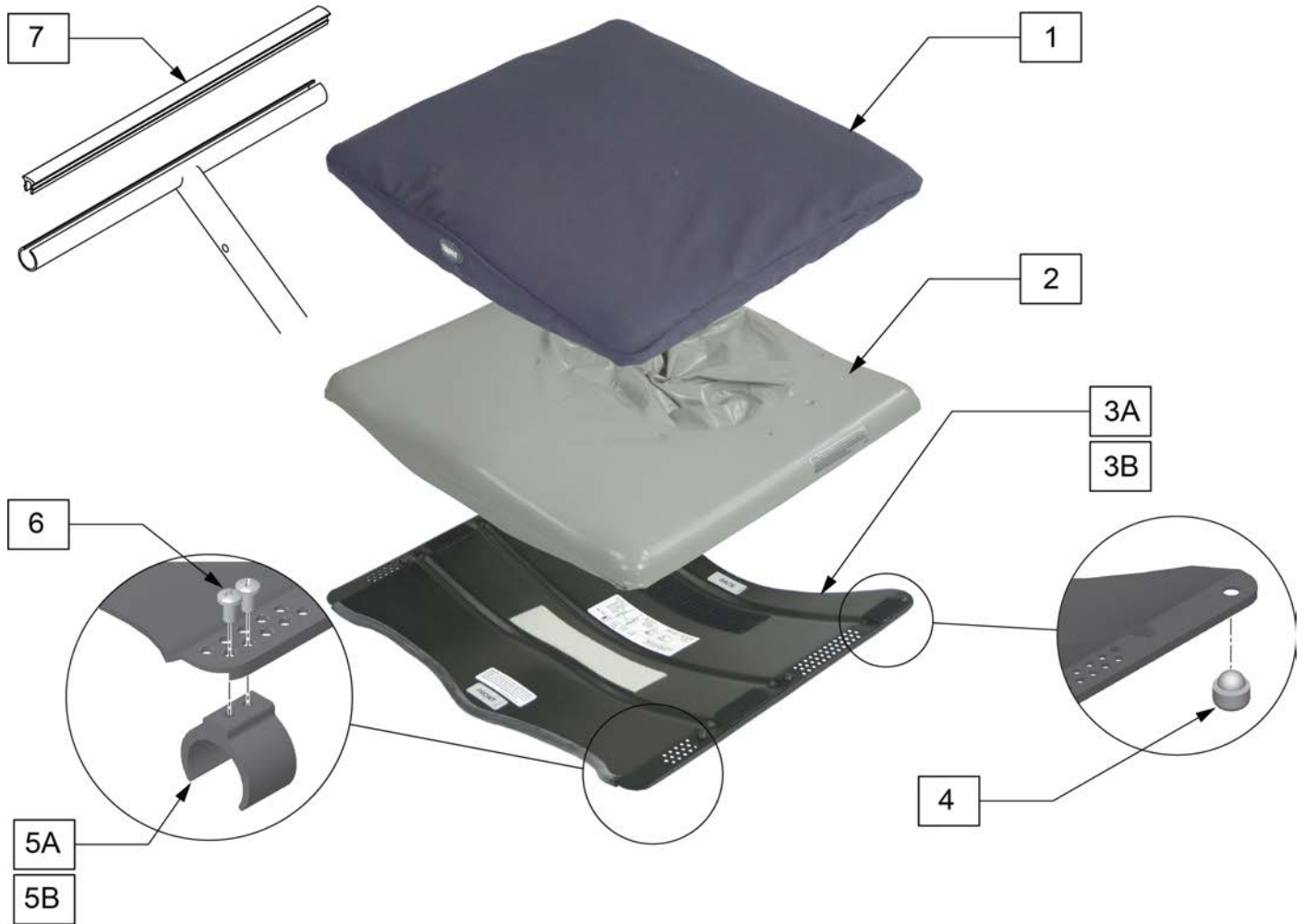

Pos.	Item Number	Description	Remarks
1	C700	JAY DUO COVER 18 x 16	
1	C701	JAY DUO COVER 18 x 18	
1	C705	JAY DUO COVER 16 x 16	
1	C706	JAY DUO COVER 16 x 18	
1	C709	JAY DUO COVER 20 x 18	
1	C726	JAY DUO COVER 20 x 16	
1	C776	JAY DUO COVER 17 x 16	
1	C778	JAY DUO COVER 17 x 18	
2	F700	JAY DUO PAD 18 x 16	
2	F701	JAY DUO PAD 18 x 18	
2	F705	JAY DUO PAD 16 x 16	

Pos.	Item Number	Description	Remarks
2	F706	JAY DUO PAD 16 x 18	
2	F709	JAY DUO PAD 20 x 18	
2	F726	JAY DUO PAD 20 x 16	
2	F776	JAY DUO PAD 17 x 16	
2	F778	JAY DUO PAD 17 x 18	
3A,5A, 5B,6	S700	JAY DUO SEAT PAN 18 x 16	
3A,5A, 5B,6	S701	JAY DUO SEAT PAN 18 x 18	
3A,5A, 5B,6	S705	JAY DUO SEAT PAN 16 x 16	
3A,5A, 5B,6	S706	JAY DUO SEAT PAN 16 x 18	
3A,5A, 5B,6	S709	JAY DUO SEAT PAN 20 x 18	
3A,5A, 5B,6	S726	JAY DUO SEAT PAN 20 x 16	
3A,5A, 5B,6	S776	JAY DUO SEAT PAN 17 x 16	
3A,5A, 5B,6	S778	JAY DUO SEAT PAN 17 x 18	
3B,5A, 5B,6	S700Q	JAY DUO SEAT PAN 18 x 16 (FOR 18 x 16)	* SEE NOTE
3B,5A, 5B,6	S701Q	JAY DUO SEAT PAN 17.5 x 18 (FOR 18 x 18)	* SEE NOTE
3B,5A, 5B,6	S705Q	JAY DUO SEAT PAN 15.5 x 16 (FOR 16 x 16)	* SEE NOTE
3B,5A, 5B,6	S706Q	JAY DUO SEAT PAN 15.5 x 18 (FOR 16 x 18)	* SEE NOTE
3B,5A, 5B,6	S709Q	JAY DUO SEAT PAN 19.5 x 18 (FOR 20 x 18)	* SEE NOTE
3B,5A, 5B,6	S726Q	JAY DUO SEAT PAN 20 x 16 (FOR 20 x 16)	* SEE NOTE
3B,5A, 5B,6	S776Q	JAY DUO SEAT PAN 17 x 16 (FOR 17 x 16)	* SEE NOTE
3B,5A, 5B,6	S778Q	JAY DUO SEAT PAN 17 x 18 (FOR 17 x 18)	* SEE NOTE
4	M-726	5/8" RUBBER BUMPER DUO KIT	SHIPS AS A SET OF 4.
5A,6	M-723	1" NYLON RETAINER CLIP DUO	SHIPS AS A SET OF 4.
5B,6	M-724	7/8" NYLON RETAINER CLIP DUO	
7	580026	SEAT RAIL INSERT STRIP 16"	** SEE NOTE
7	580027	SEAT RAIL INSERT STRIP 17"	** SEE NOTE
7	580028	SEAT RAIL INSERT STRIP 18"	** SEE NOTE
7	580029	SEAT RAIL INSERT STRIP 19"	** SEE NOTE
7	580030	SEAT RAIL INSERT STRIP 20"	** SEE NOTE

NOTE: IN GENERAL, THE (F) FLAT BASE IS USED FOR HARD SEATING SURFACES, SUCH AS A SEAT PAN, AND THE CURVED BASE IS USED FOR SOFT SURFACES, SUCH AS A SEAT SLING.


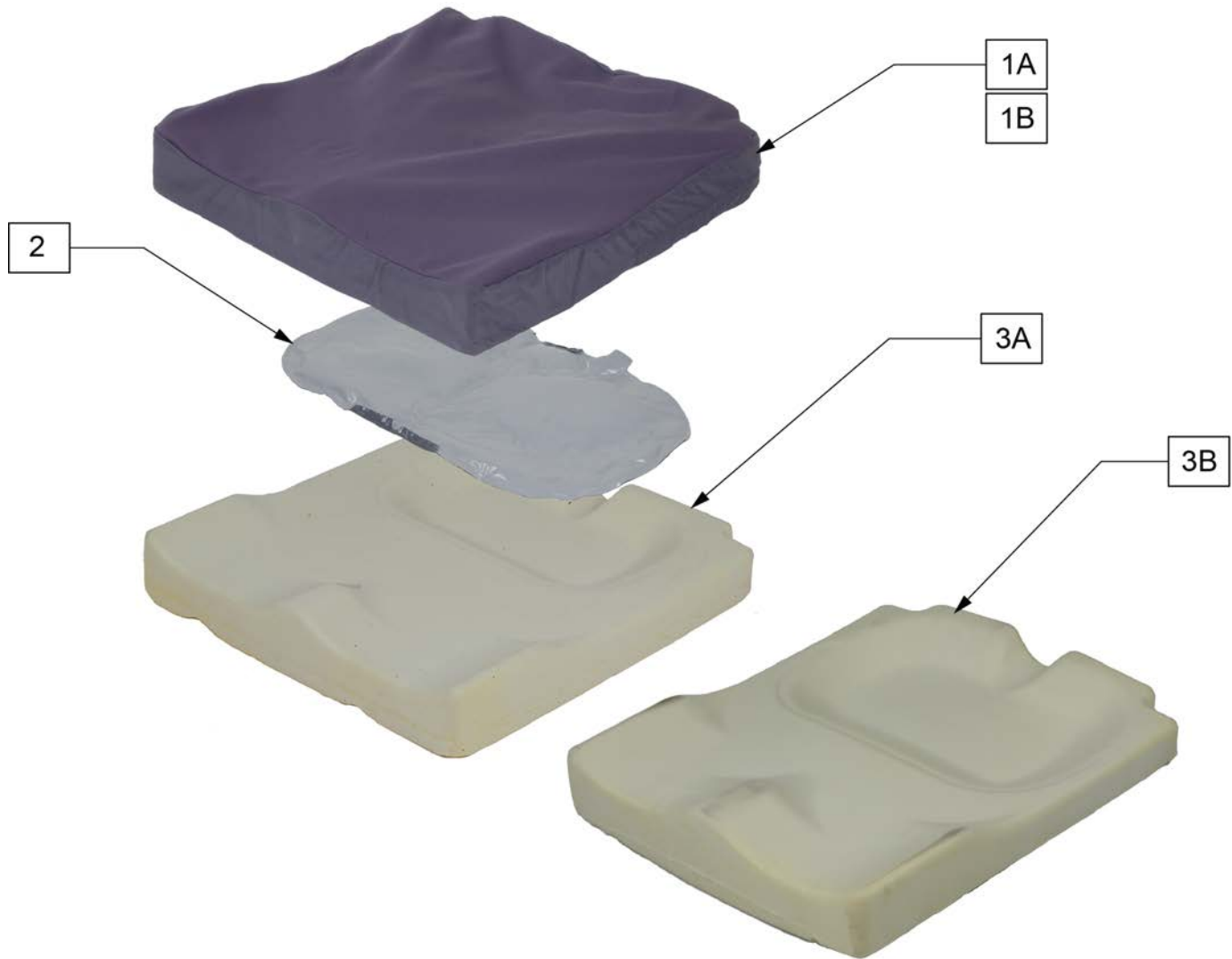


Pos.	Item Number	Description	Remarks
>>>>		JAY EASY COVER FLAT	
1A	CJE1414F	COVER JAY EASY FLAT 14 x 14	
1A	CJE1416F	COVER JAY EASY FLAT 14 x 16	
1A	CJE1418F	COVER JAY EASY FLAT 14 x 18	
1A	CJE1515F	COVER JAY EASY FLAT 15 x 15	
1A	CJE1516F	COVER JAY EASY FLAT 15 x 16	
1A	CJE1517F	COVER JAY EASY FLAT 15 x 17	
1A	CJE1615F	COVER JAY EASY FLAT 16 x 15	
1A	CJE1616F	COVER JAY EASY FLAT 16 x 16	
1A	CJE1617F	COVER JAY EASY FLAT 16 x 17	
1A	CJE1618F	COVER JAY EASY FLAT 16 x 18	
1A	CJE1715F	COVER JAY EASY FLAT 17 x 15	
1A	CJE1716F	COVER JAY EASY FLAT 17 x 16	
1A	CJE1717F	COVER JAY EASY FLAT 17 x 17	

Pos.	Item Number	Description	Remarks
1A	CJE1718F	COVER JAY EASY FLAT 17 x 18	
1A	CJE1816F	COVER JAY EASY FLAT 18 x 16	
1A	CJE1817F	COVER JAY EASY FLAT 18 x 17	
1A	CJE1818F	COVER JAY EASY FLAT 18 x 18	
1A	CJE1820F	COVER JAY EASY FLAT 18 x 20	
1A	CJE1916F	COVER JAY EASY FLAT 19 x 16	
1A	CJE1917F	COVER JAY EASY FLAT 19 x 17	
1A	CJE1918F	COVER JAY EASY FLAT 19 x 18	
1A	CJE1920F	COVER JAY EASY FLAT 19 x 20	
1A	CJE2016F	COVER JAY EASY FLAT 20 x 16	
1A	CJE2018F	COVER JAY EASY FLAT 20 x 18	
1A	CJE2020F	COVER JAY EASY FLAT 20 x 20	
1A	CJE2218F	COVER JAY EASY FLAT 22 x 18	
1A	CJE2220F	COVER JAY EASY FLAT 22 x 20	
1A	CJE2222F	COVER JAY EASY FLAT 22 x 22	
1A	CJE2418F	COVER JAY EASY FLAT 24 x 18	
1A	CJE2420F	COVER JAY EASY FLAT 24 x 20	
1A	CJE2422F	COVER JAY EASY FLAT 24 x 22	
1A	CJE2424F	COVER JAY EASY FLAT 24 x 24	
>>>>		JAY EASY COVER CURVED	
1B	CJE1414C	COVER JAY EASY CURVED 14 x 14	
1B	CJE1416C	COVER JAY EASY CURVED 14 x 16	
1B	CJE1418C	COVER JAY EASY CURVED 14 x 18	
1B	CJE1515C	COVER JAY EASY CURVED 15 x 15	
1B	CJE1516C	COVER JAY EASY CURVED 15 x 16	
1B	CJE1517C	COVER JAY EASY CURVED 15 x 17	
1B	CJE1615C	COVER JAY EASY CURVED 16 x 15	
1B	CJE1616C	COVER JAY EASY CURVED 16 x 16	
1B	CJE1617C	COVER JAY EASY CURVED 16 x 17	
1B	CJE1618C	COVER JAY EASY CURVED 16 x 18	
1B	CJE1715C	COVER JAY EASY CURVED 17 x 15	
1B	CJE1716C	COVER JAY EASY CURVED 17 x 16	
1B	CJE1717C	COVER JAY EASY CURVED 17 x 17	
1B	CJE1718C	COVER JAY EASY CURVED 17 x 18	
1B	CJE1816C	COVER JAY EASY CURVED 18 x 16	
1B	CJE1817C	COVER JAY EASY CURVED 18 x 17	
1B	CJE1818C	COVER JAY EASY CURVED 18 x 18	
1B	CJE1820C	COVER JAY EASY CURVED 18 x 20	
1B	CJE1916C	COVER JAY EASY CURVED 19 x 16	
1B	CJE1917C	COVER JAY EASY CURVED 19 x 17	
1B	CJE1918C	COVER JAY EASY CURVED 19 x 18	
1B	CJE1920C	COVER JAY EASY CURVED 19 x 20	
1B	CJE2016C	COVER JAY EASY CURVED 20 x 16	
1B	CJE2018C	COVER JAY EASY CURVED 20 x 18	
1B	CJE2020C	COVER JAY EASY CURVED 20 x 20	
1B	CJE2218C	COVER JAY EASY CURVED 22 x 18	
1B	CJE2220C	COVER JAY EASY CURVED 22 x 20	
1B	CJE2222C	COVER JAY EASY CURVED 22 x 22	
1B	CJE2418C	COVER JAY EASY CURVED 24 x 18	
1B	CJE2420C	COVER JAY EASY CURVED 24 x 20	
1B	CJE2422C	COVER JAY EASY CURVED 24 x 22	
1B	CJE2424C	COVER JAY EASY CURVED 24 x 24	
>>>>		JAY EASY FLUID PAD	
2	FJE14-15W	FLUID PAD JAY EASY 14-15	Fits 14-15" wide cushions
2	FJE16-18W	FLUID PAD JAY EASY 16-18	Fits 16-18" wide cushions
2	FJE20W	FLUID PAD JAY EASY 20	Fits 19-20" wide cushions
>>>>		JAY EASY BASE FLAT	


Pos.	Item Number	Description	Remarks
3A	BJE1414F	BASE JAY EASY FLAT 14 x 14	
3A	BJE1416F	BASE JAY EASY FLAT 14 x 16	
3A	BJE1418F	BASE JAY EASY FLAT 14 x 18	
3A	BJE1515F	BASE JAY EASY FLAT 15 x 15	
3A	BJE1516F	BASE JAY EASY FLAT 15 x 16	
3A	BJE1517F	BASE JAY EASY FLAT 15 x 17	
3A	BJE1615F	BASE JAY EASY FLAT 16 x 15	
3A	BJE1616F	BASE JAY EASY FLAT 16 x 16	
3A	BJE1617F	BASE JAY EASY FLAT 16 x 17	
3A	BJE1618F	BASE JAY EASY FLAT 16 x 18	
3A	BJE1715F	BASE JAY EASY FLAT 17 x 15	
3A	BJE1716F	BASE JAY EASY FLAT 17 x 16	
3A	BJE1717F	BASE JAY EASY FLAT 17 x 17	
3A	BJE1718F	BASE JAY EASY FLAT 17 x 18	
3A	BJE1816F	BASE JAY EASY FLAT 18 x 16	
3A	BJE1817F	BASE JAY EASY FLAT 18 x 17	
3A	BJE1818F	BASE JAY EASY FLAT 18 x 18	
3A	BJE1820F	BASE JAY EASY FLAT 18 x 20	
3A	BJE1916F	BASE JAY EASY FLAT 19 x 16	
3A	BJE1917F	BASE JAY EASY FLAT 19 x 17	
3A	BJE1918F	BASE JAY EASY FLAT 19 x 18	
3A	BJE1920F	BASE JAY EASY FLAT 19 x 20	
3A	BJE2016F	BASE JAY EASY FLAT 20 x 16	
3A	BJE2018F	BASE JAY EASY FLAT 20 x 18	
3A	BJE2020F	BASE JAY EASY FLAT 20 x 20	
3A	BJE2218F	BASE JAY EASY FLAT 22 x 18	
3A	BJE2220F	BASE JAY EASY FLAT 22 x 20	
3A	BJE2222F	BASE JAY EASY FLAT 22 x 22	
3A	BJE2418F	BASE JAY EASY FLAT 24 x 18	
3A	BJE2420F	BASE JAY EASY FLAT 24 x 20	
3A	BJE2422F	BASE JAY EASY FLAT 24 x 22	
3A	BJE2424F	BASE JAY EASY FLAT 24 x 24	
>>>>		JAY EASY BASE CURVED	
3B	BJE1414C	BASE JAY EASY CURVED 14 x 14	
3B	BJE1416C	BASE JAY EASY CURVED 14 x 16	
3B	BJE1418C	BASE JAY EASY CURVED 14 x 18	
3B	BJE1515C	BASE JAY EASY CURVED 15 x 15	
3B	BJE1516C	BASE JAY EASY CURVED 15 x 16	
3B	BJE1517C	BASE JAY EASY CURVED 15 x 17	
3B	BJE1615C	BASE JAY EASY CURVED 16 x 15	
3B	BJE1616C	BASE JAY EASY CURVED 16 x 16	
3B	BJE1617C	BASE JAY EASY CURVED 16 x 17	
3B	BJE1618C	BASE JAY EASY CURVED 16 x 18	
3B	BJE1715C	BASE JAY EASY CURVED 17 x 15	
3B	BJE1716C	BASE JAY EASY CURVED 17 x 16	
3B	BJE1717C	BASE JAY EASY CURVED 17 x 17	
3B	BJE1718C	BASE JAY EASY CURVED 17 x 18	
3B	BJE1816C	BASE JAY EASY CURVED 18 x 16	
3B	BJE1817C	BASE JAY EASY CURVED 18 x 17	
3B	BJE1818C	BASE JAY EASY CURVED 18 x 18	
3B	BJE1820C	BASE JAY EASY CURVED 18 x 20	
3B	BJE1916C	BASE JAY EASY CURVED 19 x 16	
3B	BJE1917C	BASE JAY EASY CURVED 19 x 17	
3B	BJE1918C	BASE JAY EASY CURVED 19 x 18	
3B	BJE1920C	BASE JAY EASY CURVED 19 x 20	
3B	BJE2016C	BASE JAY EASY CURVED 20 x 16	
3B	BJE2018C	BASE JAY EASY CURVED 20 x 18	

Pos.	Item Number	Description	Remarks
3B	BJE2020C	BASE JAY EASY CURVED 20 x 20	
3B	BJE2218C	BASE JAY EASY CURVED 22 x 18	
3B	BJE2220C	BASE JAY EASY CURVED 22 x 20	
3B	BJE2222C	BASE JAY EASY CURVED 22 x 22	
3B	BJE2418C	BASE JAY EASY CURVED 24 x 18	
3B	BJE2420C	BASE JAY EASY CURVED 24 x 20	
3B	BJE2422C	BASE JAY EASY CURVED 24 x 22	
3B	BJE2424C	BASE JAY EASY CURVED 24 x 24	

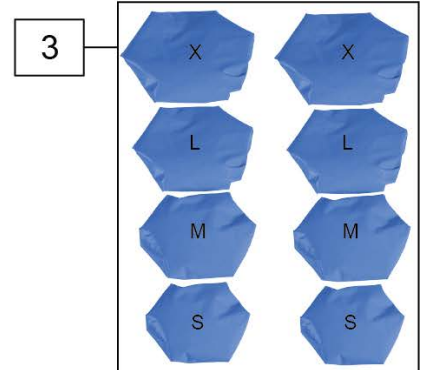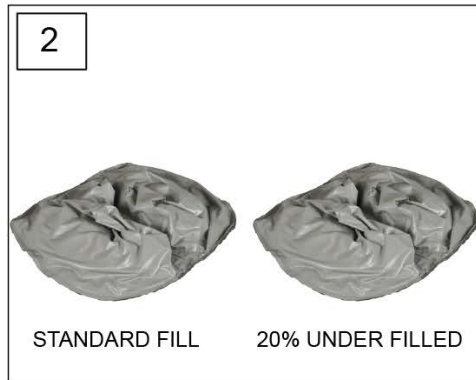

Pos.	Item Number	Description	Remarks
1-3	JFUSIONEK	JAY FUSION EVAL KIT	Order part# JFUSIONEK

Note: Cushion widths 14" & 15" use A size Fluid Bladders, 16" - 20" use B size bladders and 21"-24" use C size bladders.

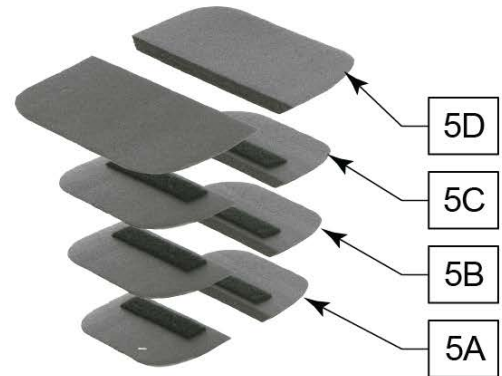

Obliquity Pad Ordering Guide

Cushion Width	1st Layer	2nd Layer
14"-15"	JBALCOP-ALH	JBALCOP-BLH
	JBALCOP-ARH	JBALCOP-BRH
16"-20"	JBALCOP-BLH	JBALCOP-CLH
	JBALCOP-BRH	JBALCOP-CRH
21"-24"	JBALCOP-CLH	JBALCOP-DLH
	JBALCOP-CRH	JBALCOP-DRH

Note: Obliquity Pad is 1/2":
 One layer of pads = 1/2" height
 Two layers of pads = 1" height

Pos.	Item Number	Description	Remarks
1	CJFUSION1414	OUTER CVR JFUSION1414 FLUID	
1	CJFUSION1415	OUTER CVR JFUSION1415 FLUID	
1	CJFUSION1416	OUTER CVR JFUSION1416 FLUID	
1	CJFUSION1417	OUTER CVR JFUSION1417 FLUID	
1	CJFUSION1418	OUTER CVR JFUSION1418 FLUID	
1	CJFUSION1514	OUTER CVR JFUSION1514 FLUID	
1	CJFUSION1515	OUTER CVR JFUSION1515 FLUID	
1	CJFUSION1516	OUTER CVR JFUSION1516 FLUID	
1	CJFUSION1517	OUTER CVR JFUSION1517 FLUID	
1	CJFUSION1518	OUTER CVR JFUSION1518 FLUID	
1	CJFUSION1519	OUTER CVR JFUSION1519 FLUID	
1	CJFUSION1520	OUTER CVR JFUSION1520 FLUID	
1	CJFUSION1521	OUTER CVR JFUSION1521 FLUID	

Pos.	Item Number	Description	Remarks
1	CJFUSION1614	OUTER CVR JFUSION1614 FLUID	
1	CJFUSION1615	OUTER CVR JFUSION1615 FLUID	
1	CJFUSION1616	OUTER CVR JFUSION1616 FLUID	
1	CJFUSION1617	OUTER CVR JFUSION1617 FLUID	
1	CJFUSION1618	OUTER CVR JFUSION1618 FLUID	
1	CJFUSION1619	OUTER CVR JFUSION1619 FLUID	
1	CJFUSION1620	OUTER CVR JFUSION1620 FLUID	
1	CJFUSION1621	OUTER CVR JFUSION1621 FLUID	
1	CJFUSION1622	OUTER CVR JFUSION1622 FLUID	
1	CJFUSION1714	OUTER CVR JFUSION1714 FLUID	
1	CJFUSION1715	OUTER CVR JFUSION1715 FLUID	
1	CJFUSION1716	OUTER CVR JFUSION1716 FLUID	
1	CJFUSION1717	OUTER CVR JFUSION1717 FLUID	
1	CJFUSION1718	OUTER CVR JFUSION1718 FLUID	
1	CJFUSION1719	OUTER CVR JFUSION1719 FLUID	
1	CJFUSION1720	OUTER CVR JFUSION1720 FLUID	
1	CJFUSION1721	OUTER CVR JFUSION1721 FLUID	
1	CJFUSION1722	OUTER CVR JFUSION1722 FLUID	
1	CJFUSION1723	OUTER CVR JFUSION1723 FLUID	
1	CJFUSION1814	OUTER CVR JFUSION1814 FLUID	
1	CJFUSION1815	OUTER CVR JFUSION1815 FLUID	
1	CJFUSION1816	OUTER CVR JFUSION1816 FLUID	
1	CJFUSION1817	OUTER CVR JFUSION1817 FLUID	
1	CJFUSION1818	OUTER CVR JFUSION1818 FLUID	
1	CJFUSION1819	OUTER CVR JFUSION1819 FLUID	
1	CJFUSION1820	OUTER CVR JFUSION1820 FLUID	
1	CJFUSION1821	OUTER CVR JFUSION1821 FLUID	
1	CJFUSION1822	OUTER CVR JFUSION1822 FLUID	
1	CJFUSION1823	OUTER CVR JFUSION1823 FLUID	
1	CJFUSION1824	OUTER CVR JFUSION1824 FLUID	
1	CJFUSION1915	OUTER CVR JFUSION1915 FLUID	
1	CJFUSION1916	OUTER CVR JFUSION1916 FLUID	
1	CJFUSION1917	OUTER CVR JFUSION1917 FLUID	
1	CJFUSION1918	OUTER CVR JFUSION1918 FLUID	
1	CJFUSION1919	OUTER CVR JFUSION1919 FLUID	
1	CJFUSION1920	OUTER CVR JFUSION1920 FLUID	
1	CJFUSION1921	OUTER CVR JFUSION1921 FLUID	
1	CJFUSION1922	OUTER CVR JFUSION1922 FLUID	
1	CJFUSION1923	OUTER CVR JFUSION1923 FLUID	
1	CJFUSION1924	OUTER CVR JFUSION1924 FLUID	
1	CJFUSION2015	OUTER CVR JFUSION2015 FLUID	
1	CJFUSION2016	OUTER CVR JFUSION2016 FLUID	
1	CJFUSION2017	OUTER CVR JFUSION2017 FLUID	
1	CJFUSION2018	OUTER CVR JFUSION2018 FLUID	
1	CJFUSION2019	OUTER CVR JFUSION2019 FLUID	
1	CJFUSION2020	OUTER CVR JFUSION2020 FLUID	
1	CJFUSION2021	OUTER CVR JFUSION2021 FLUID	
1	CJFUSION2022	OUTER CVR JFUSION2022 FLUID	
1	CJFUSION2023	OUTER CVR JFUSION2023 FLUID	
1	CJFUSION2024	OUTER CVR JFUSION2024 FLUID	
1	CJFUSION2117	OUTER CVR JFUSION2117 FLUID	
1	CJFUSION2118	OUTER CVR JFUSION2118 FLUID	
1	CJFUSION2119	OUTER CVR JFUSION2119 FLUID	
1	CJFUSION2120	OUTER CVR JFUSION2120 FLUID	
1	CJFUSION2121	OUTER CVR JFUSION2121 FLUID	
1	CJFUSION2122	OUTER CVR JFUSION2122 FLUID	
1	CJFUSION2123	OUTER CVR JFUSION2123 FLUID	


Pos.	Item Number	Description	Remarks
1	CJFUSION2124	OUTER CVR JFUSION2124 FLUID	
1	CJFUSION2217	OUTER CVR JFUSION2217 FLUID	
1	CJFUSION2218	OUTER CVR JFUSION2218 FLUID	
1	CJFUSION2219	OUTER CVR JFUSION2219 FLUID	
1	CJFUSION2220	OUTER CVR JFUSION2220 FLUID	
1	CJFUSION2221	OUTER CVR JFUSION2221 FLUID	
1	CJFUSION2222	OUTER CVR JFUSION2222 FLUID	
1	CJFUSION2223	OUTER CVR JFUSION2223 FLUID	
1	CJFUSION2224	OUTER CVR JFUSION2224 FLUID	
1	CJFUSION2319	OUTER CVR JFUSION2319 FLUID	
1	CJFUSION2320	OUTER CVR JFUSION2320 FLUID	
1	CJFUSION2321	OUTER CVR JFUSION2321 FLUID	
1	CJFUSION2322	OUTER CVR JFUSION2322 FLUID	
1	CJFUSION2323	OUTER CVR JFUSION2323 FLUID	
1	CJFUSION2324	OUTER CVR JFUSION2324 FLUID	
1	CJFUSION2419	OUTER CVR JFUSION2419 FLUID	
1	CJFUSION2420	OUTER CVR JFUSION2420 FLUID	
1	CJFUSION2421	OUTER CVR JFUSION2421 FLUID	
1	CJFUSION2422	OUTER CVR JFUSION2422 FLUID	
1	CJFUSION2423	OUTER CVR JFUSION2423 FLUID	
1	CJFUSION2424	OUTER CVR JFUSION2424 FLUID	
2	FAJFUSION	FLUID INSERT SIZE A JFUSION	Used in 14"&15" wide
2	FBJFUSION	FLUID INSERT SIZE B JFUSION	Used in 16"-20" wide
2	FCJFUSION	FLUID INSERT SIZE C JFUSION	Used in 21"-24" wide
3	ICJFUSION1414	INNER CVR JFUSION1414 FLUID	
3	ICJFUSION1415	INNER CVR JFUSION1415 FLUID	
3	ICJFUSION1416	INNER CVR JFUSION1416 FLUID	
3	ICJFUSION1417	INNER CVR JFUSION1417 FLUID	
3	ICJFUSION1418	INNER CVR JFUSION1418 FLUID	
3	ICJFUSION1514	INNER CVR JFUSION1514 FLUID	
3	ICJFUSION1515	INNER CVR JFUSION1515 FLUID	
3	ICJFUSION1516	INNER CVR JFUSION1516 FLUID	
3	ICJFUSION1517	INNER CVR JFUSION1517 FLUID	
3	ICJFUSION1518	INNER CVR JFUSION1518 FLUID	
3	ICJFUSION1519	INNER CVR JFUSION1519 FLUID	
3	ICJFUSION1520	INNER CVR JFUSION1520 FLUID	
3	ICJFUSION1521	INNER CVR JFUSION1521 FLUID	
3	ICJFUSION1614	INNER CVR JFUSION1614 FLUID	
3	ICJFUSION1615	INNER CVR JFUSION1615 FLUID	
3	ICJFUSION1616	INNER CVR JFUSION1616 FLUID	
3	ICJFUSION1617	INNER CVR JFUSION1617 FLUID	
3	ICJFUSION1618	INNER CVR JFUSION1618 FLUID	
3	ICJFUSION1619	INNER CVR JFUSION1619 FLUID	
3	ICJFUSION1620	INNER CVR JFUSION1620 FLUID	
3	ICJFUSION1621	INNER CVR JFUSION1621 FLUID	
3	ICJFUSION1622	INNER CVR JFUSION1622 FLUID	
3	ICJFUSION1714	INNER CVR JFUSION1714 FLUID	
3	ICJFUSION1715	INNER CVR JFUSION1715 FLUID	
3	ICJFUSION1716	INNER CVR JFUSION1716 FLUID	
3	ICJFUSION1717	INNER CVR JFUSION1717 FLUID	
3	ICJFUSION1718	INNER CVR JFUSION1718 FLUID	
3	ICJFUSION1719	INNER CVR JFUSION1719 FLUID	
3	ICJFUSION1720	INNER CVR JFUSION1720 FLUID	
3	ICJFUSION1721	INNER CVR JFUSION1721 FLUID	
3	ICJFUSION1722	INNER CVR JFUSION1722 FLUID	
3	ICJFUSION1723	INNER CVR JFUSION1723 FLUID	
3	ICJFUSION1814	INNER CVR JFUSION1814 FLUID	

Pos.	Item Number	Description	Remarks
3	ICJFUSION1815	INNER CVR JFUSION1815 FLUID	
3	ICJFUSION1816	INNER CVR JFUSION1816 FLUID	
3	ICJFUSION1817	INNER CVR JFUSION1817 FLUID	
3	ICJFUSION1818	INNER CVR JFUSION1818 FLUID	
3	ICJFUSION1819	INNER CVR JFUSION1819 FLUID	
3	ICJFUSION1820	INNER CVR JFUSION1820 FLUID	
3	ICJFUSION1821	INNER CVR JFUSION1821 FLUID	
3	ICJFUSION1822	INNER CVR JFUSION1822 FLUID	
3	ICJFUSION1823	INNER CVR JFUSION1823 FLUID	
3	ICJFUSION1824	INNER CVR JFUSION1824 FLUID	
3	ICJFUSION1915	INNER CVR JFUSION1915 FLUID	
3	ICJFUSION1916	INNER CVR JFUSION1916 FLUID	
3	ICJFUSION1917	INNER CVR JFUSION1917 FLUID	
3	ICJFUSION1918	INNER CVR JFUSION1918 FLUID	
3	ICJFUSION1919	INNER CVR JFUSION1919 FLUID	
3	ICJFUSION1920	INNER CVR JFUSION1920 FLUID	
3	ICJFUSION1921	INNER CVR JFUSION1921 FLUID	
3	ICJFUSION1922	INNER CVR JFUSION1922 FLUID	
3	ICJFUSION1923	INNER CVR JFUSION1923 FLUID	
3	ICJFUSION1924	INNER CVR JFUSION1924 FLUID	
3	ICJFUSION2015	INNER CVR JFUSION2015 FLUID	
3	ICJFUSION2016	INNER CVR JFUSION2016 FLUID	
3	ICJFUSION2017	INNER CVR JFUSION2017 FLUID	
3	ICJFUSION2018	INNER CVR JFUSION2018 FLUID	
3	ICJFUSION2019	INNER CVR JFUSION2019 FLUID	
3	ICJFUSION2020	INNER CVR JFUSION2020 FLUID	
3	ICJFUSION2021	INNER CVR JFUSION2021 FLUID	
3	ICJFUSION2022	INNER CVR JFUSION2022 FLUID	
3	ICJFUSION2023	INNER CVR JFUSION2023 FLUID	
3	ICJFUSION2024	INNER CVR JFUSION2024 FLUID	
3	ICJFUSION2117	INNER CVR JFUSION2117 FLUID	
3	ICJFUSION2118	INNER CVR JFUSION2118 FLUID	
3	ICJFUSION2119	INNER CVR JFUSION2119 FLUID	
3	ICJFUSION2120	INNER CVR JFUSION2120 FLUID	
3	ICJFUSION2121	INNER CVR JFUSION2121 FLUID	
3	ICJFUSION2122	INNER CVR JFUSION2122 FLUID	
3	ICJFUSION2123	INNER CVR JFUSION2123 FLUID	
3	ICJFUSION2124	INNER CVR JFUSION2124 FLUID	
3	ICJFUSION2217	INNER CVR JFUSION2217 FLUID	
3	ICJFUSION2218	INNER CVR JFUSION2218 FLUID	
3	ICJFUSION2219	INNER CVR JFUSION2219 FLUID	
3	ICJFUSION2220	INNER CVR JFUSION2220 FLUID	
3	ICJFUSION2221	INNER CVR JFUSION2221 FLUID	
3	ICJFUSION2222	INNER CVR JFUSION2222 FLUID	
3	ICJFUSION2223	INNER CVR JFUSION2223 FLUID	
3	ICJFUSION2224	INNER CVR JFUSION2224 FLUID	
3	ICJFUSION2319	INNER CVR JFUSION2319 FLUID	
3	ICJFUSION2320	INNER CVR JFUSION2320 FLUID	
3	ICJFUSION2321	INNER CVR JFUSION2321 FLUID	
3	ICJFUSION2322	INNER CVR JFUSION2322 FLUID	
3	ICJFUSION2323	INNER CVR JFUSION2323 FLUID	
3	ICJFUSION2324	INNER CVR JFUSION2324 FLUID	
3	ICJFUSION2419	INNER CVR JFUSION2419 FLUID	
3	ICJFUSION2420	INNER CVR JFUSION2420 FLUID	
3	ICJFUSION2421	INNER CVR JFUSION2421 FLUID	
3	ICJFUSION2422	INNER CVR JFUSION2422 FLUID	
3	ICJFUSION2423	INNER CVR JFUSION2423 FLUID	

Pos.	Item Number	Description	Remarks
3	ICJFUSION2424	INNER CVR JFUSION2424 FLUID	
4	BJFUSION1414	BASE JFUSION1414	
4	BJFUSION1415	BASE JFUSION1415	
4	BJFUSION1416	BASE JFUSION1416	
4	BJFUSION1417	BASE JFUSION1417	
4	BJFUSION1418	BASE JFUSION1418	
4	BJFUSION1514	BASE JFUSION1514	
4	BJFUSION1515	BASE JFUSION1515	
4	BJFUSION1516	BASE JFUSION1516	
4	BJFUSION1517	BASE JFUSION1517	
4	BJFUSION1518	BASE JFUSION1518	
4	BJFUSION1519	BASE JFUSION1519	
4	BJFUSION1520	BASE JFUSION1520	
4	BJFUSION1521	BASE JFUSION1521	
4	BJFUSION1614	BASE JFUSION1614	
4	BJFUSION1615	BASE JFUSION1615	
4	BJFUSION1616	BASE JFUSION1616	
4	BJFUSION1617	BASE JFUSION1617	
4	BJFUSION1618	BASE JFUSION1618	
4	BJFUSION1619	BASE JFUSION1619	
4	BJFUSION1620	BASE JFUSION1620	
4	BJFUSION1621	BASE JFUSION1621	
4	BJFUSION1622	BASE JFUSION1622	
4	BJFUSION1714	BASE JFUSION1714	
4	BJFUSION1715	BASE JFUSION1715	
4	BJFUSION1716	BASE JFUSION1716	
4	BJFUSION1717	BASE JFUSION1717	
4	BJFUSION1718	BASE JFUSION1718	
4	BJFUSION1719	BASE JFUSION1719	
4	BJFUSION1720	BASE JFUSION1720	
4	BJFUSION1721	BASE JFUSION1721	
4	BJFUSION1722	BASE JFUSION1722	
4	BJFUSION1723	BASE JFUSION1723	
4	BJFUSION1814	BASE JFUSION1814	
4	BJFUSION1815	BASE JFUSION1815	
4	BJFUSION1816	BASE JFUSION1816	
4	BJFUSION1817	BASE JFUSION1817	
4	BJFUSION1818	BASE JFUSION1818	
4	BJFUSION1819	BASE JFUSION1819	
4	BJFUSION1820	BASE JFUSION1820	
4	BJFUSION1821	BASE JFUSION1821	
4	BJFUSION1822	BASE JFUSION1822	
4	BJFUSION1823	BASE JFUSION1823	
4	BJFUSION1824	BASE JFUSION1824	
4	BJFUSION1915	BASE JFUSION1915	
4	BJFUSION1916	BASE JFUSION1916	
4	BJFUSION1917	BASE JFUSION1917	
4	BJFUSION1918	BASE JFUSION1918	
4	BJFUSION1919	BASE JFUSION1919	
4	BJFUSION1920	BASE JFUSION1920	
4	BJFUSION1921	BASE JFUSION1921	
4	BJFUSION1922	BASE JFUSION1922	
4	BJFUSION1923	BASE JFUSION1923	
4	BJFUSION1924	BASE JFUSION1924	
4	BJFUSION2015	BASE JFUSION2015	
4	BJFUSION2016	BASE JFUSION2016	
4	BJFUSION2017	BASE JFUSION2017	

Pos.	Item Number	Description	Remarks
4	BJFUSION2018	BASE JFUSION2018	
4	BJFUSION2019	BASE JFUSION2019	
4	BJFUSION2020	BASE JFUSION2020	
4	BJFUSION2021	BASE JFUSION2021	
4	BJFUSION2022	BASE JFUSION2022	
4	BJFUSION2023	BASE JFUSION2023	
4	BJFUSION2024	BASE JFUSION2024	
4	BJFUSION2117	BASE JFUSION2117	
4	BJFUSION2118	BASE JFUSION2118	
4	BJFUSION2119	BASE JFUSION2119	
4	BJFUSION2120	BASE JFUSION2120	
4	BJFUSION2121	BASE JFUSION2121	
4	BJFUSION2122	BASE JFUSION2122	
4	BJFUSION2123	BASE JFUSION2123	
4	BJFUSION2124	BASE JFUSION2124	
4	BJFUSION2217	BASE JFUSION2217	
4	BJFUSION2218	BASE JFUSION2218	
4	BJFUSION2219	BASE JFUSION2219	
4	BJFUSION2220	BASE JFUSION2220	
4	BJFUSION2221	BASE JFUSION2221	
4	BJFUSION2222	BASE JFUSION2222	
4	BJFUSION2223	BASE JFUSION2223	
4	BJFUSION2224	BASE JFUSION2224	
4	BJFUSION2319	BASE JFUSION2319	
4	BJFUSION2320	BASE JFUSION2320	
4	BJFUSION2321	BASE JFUSION2321	
4	BJFUSION2322	BASE JFUSION2322	
4	BJFUSION2323	BASE JFUSION2323	
4	BJFUSION2324	BASE JFUSION2324	
4	BJFUSION2419	BASE JFUSION2419	
4	BJFUSION2420	BASE JFUSION2420	
4	BJFUSION2421	BASE JFUSION2421	
4	BJFUSION2422	BASE JFUSION2422	
4	BJFUSION2423	BASE JFUSION2423	
4	BJFUSION2424	BASE JFUSION2424	
5A	JBALCOP-ALH	OBLIQUITY PAD LH PLA A	
5A	JBALCOP-ARH	OBLIQUITY PAD RH PLA A	
5B	JBALCOP-BLH	OBLIQUITY PAD LH PLA B	
5B	JBALCOP-BRH	OBLIQUITY PAD RH PLA B	
5C	JBALCOP-CLH	OBLIQUITY PAD LH PLA C	
5C	JBALCOP-CRH	OLBIQUITY PAD RH PLA C	
5D	JBALCOP-DLH	OBLIQUITY PAD LH PLA D	
5D	JBALCOP-DRH	OBLIQUITY PAD RH PLA D	

Note: Cushion widths 14" & 15" use A size Air Bladders, 16" - 20" use B size bladders and 21"-24" use C size bladders

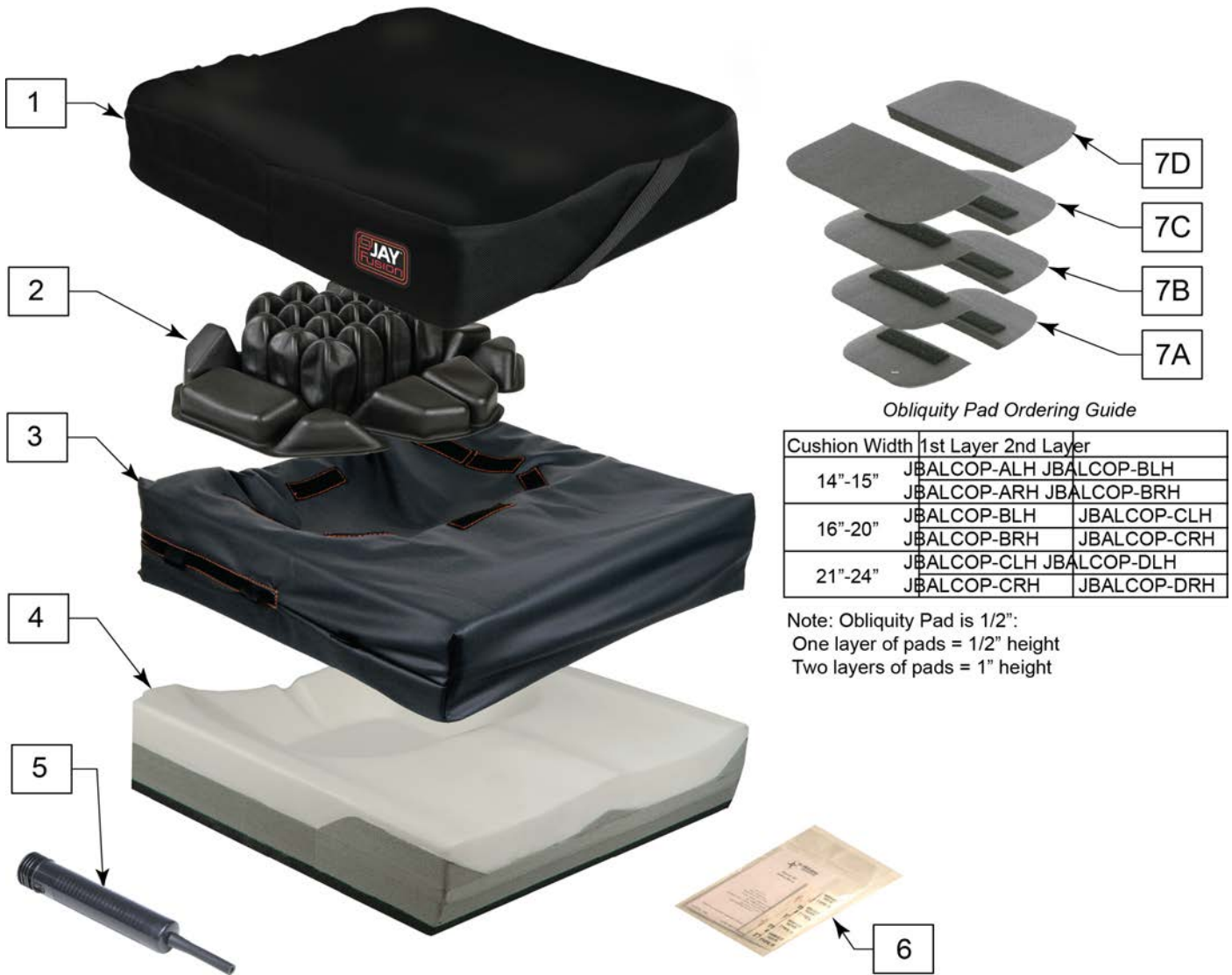

Obliquity Pad Ordering Guide

Cushion Width	1st Layer	2nd Layer
14"-15"	JBALCOP-ALH	JBALCOP-BLH
	JBALCOP-ARH	JBALCOP-BRH
16"-20"	JBALCOP-BLH	JBALCOP-CLH
	JBALCOP-BRH	JBALCOP-CRH
21"-24"	JBALCOP-CLH	JBALCOP-DLH
	JBALCOP-CRH	JBALCOP-DRH

Note: Obliquity Pad is 1/2";
 One layer of pads = 1/2" height
 Two layers of pads = 1" height

Pos.	Item Number	Description	Remarks
1	CJFUSION1414A	OUTER CVR JFUSION1414 AIR	
1	CJFUSION1415A	OUTER CVR JFUSION1415 AIR	
1	CJFUSION1416A	OUTER CVR JFUSION1416 AIR	
1	CJFUSION1417A	OUTER CVR JFUSION1417 AIR	
1	CJFUSION1418A	OUTER CVR JFUSION1418 AIR	
1	CJFUSION1514A	OUTER CVR JFUSION1514 AIR	
1	CJFUSION1515A	OUTER CVR JFUSION1515 AIR	
1	CJFUSION1516A	OUTER CVR JFUSION1516 AIR	
1	CJFUSION1517A	OUTER CVR JFUSION1517 AIR	
1	CJFUSION1518A	OUTER CVR JFUSION1518 AIR	
1	CJFUSION1519A	OUTER CVR JFUSION1519 AIR	
1	CJFUSION1520A	OUTER CVR JFUSION1520 AIR	
1	CJFUSION1521A	OUTER CVR JFUSION1521 AIR	

Pos.	Item Number	Description	Remarks
1	CJFUSION1614A	OUTER CVR JFUSION1614 AIR	
1	CJFUSION1615A	OUTER CVR JFUSION1615 AIR	
1	CJFUSION1616A	OUTER CVR JFUSION1616 AIR	
1	CJFUSION1617A	OUTER CVR JFUSION1617 AIR	
1	CJFUSION1618A	OUTER CVR JFUSION1618 AIR	
1	CJFUSION1619A	OUTER CVR JFUSION1619 AIR	
1	CJFUSION1620A	OUTER CVR JFUSION1620 AIR	
1	CJFUSION1621A	OUTER CVR JFUSION1621 AIR	
1	CJFUSION1622A	OUTER CVR JFUSION1622 AIR	
1	CJFUSION1714A	OUTER CVR JFUSION1714 AIR	
1	CJFUSION1715A	OUTER CVR JFUSION1715 AIR	
1	CJFUSION1716A	OUTER CVR JFUSION1716 AIR	
1	CJFUSION1717A	OUTER CVR JFUSION1717 AIR	
1	CJFUSION1718A	OUTER CVR JFUSION1718 AIR	
1	CJFUSION1719A	OUTER CVR JFUSION1719 AIR	
1	CJFUSION1720A	OUTER CVR JFUSION1720 AIR	
1	CJFUSION1721A	OUTER CVR JFUSION1721 AIR	
1	CJFUSION1722A	OUTER CVR JFUSION1722 AIR	
1	CJFUSION1723A	OUTER CVR JFUSION1723 AIR	
1	CJFUSION1814A	OUTER CVR JFUSION1814 AIR	
1	CJFUSION1815A	OUTER CVR JFUSION1815 AIR	
1	CJFUSION1816A	OUTER CVR JFUSION1816 AIR	
1	CJFUSION1817A	OUTER CVR JFUSION1817 AIR	
1	CJFUSION1818A	OUTER CVR JFUSION1818 AIR	
1	CJFUSION1819A	OUTER CVR JFUSION1819 AIR	
1	CJFUSION1820A	OUTER CVR JFUSION1820 AIR	
1	CJFUSION1821A	OUTER CVR JFUSION1821 AIR	
1	CJFUSION1822A	OUTER CVR JFUSION1822 AIR	
1	CJFUSION1823A	OUTER CVR JFUSION1823 AIR	
1	CJFUSION1824A	OUTER CVR JFUSION1824 AIR	
1	CJFUSION1915A	OUTER CVR JFUSION1915 AIR	
1	CJFUSION1916A	OUTER CVR JFUSION1916 AIR	
1	CJFUSION1917A	OUTER CVR JFUSION1917 AIR	
1	CJFUSION1918A	OUTER CVR JFUSION1918 AIR	
1	CJFUSION1919A	OUTER CVR JFUSION1919 AIR	
1	CJFUSION1920A	OUTER CVR JFUSION1920 AIR	
1	CJFUSION1921A	OUTER CVR JFUSION1921 AIR	
1	CJFUSION1922A	OUTER CVR JFUSION1922 AIR	
1	CJFUSION1923A	OUTER CVR JFUSION1923 AIR	
1	CJFUSION1924A	OUTER CVR JFUSION1924 AIR	
1	CJFUSION2015A	OUTER CVR JFUSION2015 AIR	
1	CJFUSION2016A	OUTER CVR JFUSION2016 AIR	
1	CJFUSION2017A	OUTER CVR JFUSION2017 AIR	
1	CJFUSION2018A	OUTER CVR JFUSION2018 AIR	
1	CJFUSION2019A	OUTER CVR JFUSION2019 AIR	
1	CJFUSION2020A	OUTER CVR JFUSION2020 AIR	
1	CJFUSION2021A	OUTER CVR JFUSION2021 AIR	
1	CJFUSION2022A	OUTER CVR JFUSION2022 AIR	
1	CJFUSION2023A	OUTER CVR JFUSION2023 AIR	
1	CJFUSION2024A	OUTER CVR JFUSION2024 AIR	
1	CJFUSION2117A	OUTER CVR JFUSION2117 AIR	
1	CJFUSION2118A	OUTER CVR JFUSION2118 AIR	
1	CJFUSION2119A	OUTER CVR JFUSION2119 AIR	
1	CJFUSION2120A	OUTER CVR JFUSION2120 AIR	
1	CJFUSION2121A	OUTER CVR JFUSION2121 AIR	
1	CJFUSION2122A	OUTER CVR JFUSION2122 AIR	
1	CJFUSION2123A	OUTER CVR JFUSION2123 AIR	

Pos.	Item Number	Description	Remarks
1	CJFUSION2124A	OUTER CVR JFUSION2124 AIR	
1	CJFUSION2217A	OUTER CVR JFUSION2217 AIR	
1	CJFUSION2218A	OUTER CVR JFUSION2218 AIR	
1	CJFUSION2219A	OUTER CVR JFUSION2219 AIR	
1	CJFUSION2220A	OUTER CVR JFUSION2220 AIR	
1	CJFUSION2221A	OUTER CVR JFUSION2221 AIR	
1	CJFUSION2222A	OUTER CVR JFUSION2222 AIR	
1	CJFUSION2223A	OUTER CVR JFUSION2223 AIR	
1	CJFUSION2224A	OUTER CVR JFUSION2224 AIR	
1	CJFUSION2319A	OUTER CVR JFUSION2319 AIR	
1	CJFUSION2320A	OUTER CVR JFUSION2320 AIR	
1	CJFUSION2321A	OUTER CVR JFUSION2321 AIR	
1	CJFUSION2322A	OUTER CVR JFUSION2322 AIR	
1	CJFUSION2323A	OUTER CVR JFUSION2323 AIR	
1	CJFUSION2324A	OUTER CVR JFUSION2324 AIR	
1	CJFUSION2419A	OUTER CVR JFUSION2419 AIR	
1	CJFUSION2420A	OUTER CVR JFUSION2420 AIR	
1	CJFUSION2421A	OUTER CVR JFUSION2421 AIR	
1	CJFUSION2422A	OUTER CVR JFUSION2422 AIR	
1	CJFUSION2423A	OUTER CVR JFUSION2423 AIR	
1	CJFUSION2424A	OUTER CVR JFUSION2424 AIR	
2	ASAJFUSION	AIR INSERT SINGLE VALVE SIZE A	Used in 14"&15" wide
2	ASBJFUSION	AIR INSERT SINGLE VALVE SIZE B	Used in 16"-20" wide
2	ASCJFUSION	AIR INSERT SINGLE VALVE SIZE C	Used in 21"-24" wide
2	ADAJFUSION	AIR INSERT DUAL VALVE SIZE A	Used in 14"&15" wide
2	ADBJFUSION	AIR INSERT DUAL VALVE SIZE B	Used in 16"-20" wide
2	ADCJFUSION	AIR INSERT DUAL VALVE SIZE C	Used in 21"-24" wide
3	ICJFUSION1414A	INNER CVR JFUSION1414 AIR	
3	ICJFUSION1415A	INNER CVR JFUSION1415 AIR	
3	ICJFUSION1416A	INNER CVR JFUSION1416 AIR	
3	ICJFUSION1417A	INNER CVR JFUSION1417 AIR	
3	ICJFUSION1418A	INNER CVR JFUSION1418 AIR	
3	ICJFUSION1514A	INNER CVR JFUSION1514 AIR	
3	ICJFUSION1515A	INNER CVR JFUSION1515 AIR	
3	ICJFUSION1516A	INNER CVR JFUSION1516 AIR	
3	ICJFUSION1517A	INNER CVR JFUSION1517 AIR	
3	ICJFUSION1518A	INNER CVR JFUSION1518 AIR	
3	ICJFUSION1519A	INNER CVR JFUSION1519 AIR	
3	ICJFUSION1520A	INNER CVR JFUSION1520 AIR	
3	ICJFUSION1521A	INNER CVR JFUSION1521 AIR	
3	ICJFUSION1614A	INNER CVR JFUSION1614 AIR	
3	ICJFUSION1615A	INNER CVR JFUSION1615 AIR	
3	ICJFUSION1616A	INNER CVR JFUSION1616 AIR	
3	ICJFUSION1617A	INNER CVR JFUSION1617 AIR	
3	ICJFUSION1618A	INNER CVR JFUSION1618 AIR	
3	ICJFUSION1619A	INNER CVR JFUSION1619 AIR	
3	ICJFUSION1620A	INNER CVR JFUSION1620 AIR	
3	ICJFUSION1621A	INNER CVR JFUSION1621 AIR	
3	ICJFUSION1622A	INNER CVR JFUSION1622 AIR	
3	ICJFUSION1714A	INNER CVR JFUSION1714 AIR	
3	ICJFUSION1715A	INNER CVR JFUSION1715 AIR	
3	ICJFUSION1716A	INNER CVR JFUSION1716 AIR	
3	ICJFUSION1717A	INNER CVR JFUSION1717 AIR	
3	ICJFUSION1718A	INNER CVR JFUSION1718 AIR	
3	ICJFUSION1719A	INNER CVR JFUSION1719 AIR	
3	ICJFUSION1720A	INNER CVR JFUSION1720 AIR	
3	ICJFUSION1721A	INNER CVR JFUSION1721 AIR	

Pos.	Item Number	Description	Remarks
3	ICJFUSION1722A	INNER CVR JFUSION1722 AIR	
3	ICJFUSION1723A	INNER CVR JFUSION1723 AIR	
3	ICJFUSION1814A	INNER CVR JFUSION1814 AIR	
3	ICJFUSION1815A	INNER CVR JFUSION1815 AIR	
3	ICJFUSION1816A	INNER CVR JFUSION1816 AIR	
3	ICJFUSION1817A	INNER CVR JFUSION1817 AIR	
3	ICJFUSION1818A	INNER CVR JFUSION1818 AIR	
3	ICJFUSION1819A	INNER CVR JFUSION1819 AIR	
3	ICJFUSION1820A	INNER CVR JFUSION1820 AIR	
3	ICJFUSION1821A	INNER CVR JFUSION1821 AIR	
3	ICJFUSION1822A	INNER CVR JFUSION1822 AIR	
3	ICJFUSION1823A	INNER CVR JFUSION1823 AIR	
3	ICJFUSION1824A	INNER CVR JFUSION1824 AIR	
3	ICJFUSION1915A	INNER CVR JFUSION1915 AIR	
3	ICJFUSION1916A	INNER CVR JFUSION1916 AIR	
3	ICJFUSION1917A	INNER CVR JFUSION1917 AIR	
3	ICJFUSION1918A	INNER CVR JFUSION1918 AIR	
3	ICJFUSION1919A	INNER CVR JFUSION1919 AIR	
3	ICJFUSION1920A	INNER CVR JFUSION1920 AIR	
3	ICJFUSION1921A	INNER CVR JFUSION1921 AIR	
3	ICJFUSION1922A	INNER CVR JFUSION1922 AIR	
3	ICJFUSION1923A	INNER CVR JFUSION1923 AIR	
3	ICJFUSION1924A	INNER CVR JFUSION1924 AIR	
3	ICJFUSION2015A	INNER CVR JFUSION2015 AIR	
3	ICJFUSION2016A	INNER CVR JFUSION2016 AIR	
3	ICJFUSION2017A	INNER CVR JFUSION2017 AIR	
3	ICJFUSION2018A	INNER CVR JFUSION2018 AIR	
3	ICJFUSION2019A	INNER CVR JFUSION2019 AIR	
3	ICJFUSION2020A	INNER CVR JFUSION2020 AIR	
3	ICJFUSION2021A	INNER CVR JFUSION2021 AIR	
3	ICJFUSION2022A	INNER CVR JFUSION2022 AIR	
3	ICJFUSION2023A	INNER CVR JFUSION2023 AIR	
3	ICJFUSION2024A	INNER CVR JFUSION2024 AIR	
3	ICJFUSION2117A	INNER CVR JFUSION2117 AIR	
3	ICJFUSION2118A	INNER CVR JFUSION2118 AIR	
3	ICJFUSION2119A	INNER CVR JFUSION2119 AIR	
3	ICJFUSION2120A	INNER CVR JFUSION2120 AIR	
3	ICJFUSION2121A	INNER CVR JFUSION2121 AIR	
3	ICJFUSION2122A	INNER CVR JFUSION2122 AIR	
3	ICJFUSION2123A	INNER CVR JFUSION2123 AIR	
3	ICJFUSION2124A	INNER CVR JFUSION2124 AIR	
3	ICJFUSION2217A	INNER CVR JFUSION2217 AIR	
3	ICJFUSION2218A	INNER CVR JFUSION2218 AIR	
3	ICJFUSION2219A	INNER CVR JFUSION2219 AIR	
3	ICJFUSION2220A	INNER CVR JFUSION2220 AIR	
3	ICJFUSION2221A	INNER CVR JFUSION2221 AIR	
3	ICJFUSION2222A	INNER CVR JFUSION2222 AIR	
3	ICJFUSION2223A	INNER CVR JFUSION2223 AIR	
3	ICJFUSION2224A	INNER CVR JFUSION2224 AIR	
3	ICJFUSION2319A	INNER CVR JFUSION2319 AIR	
3	ICJFUSION2320A	INNER CVR JFUSION2320 AIR	
3	ICJFUSION2321A	INNER CVR JFUSION2321 AIR	
3	ICJFUSION2322A	INNER CVR JFUSION2322 AIR	
3	ICJFUSION2323A	INNER CVR JFUSION2323 AIR	
3	ICJFUSION2324A	INNER CVR JFUSION2324 AIR	
3	ICJFUSION2419A	INNER CVR JFUSION2419 AIR	
3	ICJFUSION2420A	INNER CVR JFUSION2420 AIR	


Pos.	Item Number	Description	Remarks
3	ICJFUSION2421A	INNER CVR JFUSION2421 AIR	
3	ICJFUSION2422A	INNER CVR JFUSION2422 AIR	
3	ICJFUSION2423A	INNER CVR JFUSION2423 AIR	
3	ICJFUSION2424A	INNER CVR JFUSION2424 AIR	
4	BJFUSION1414	BASE JFUSION1414	
4	BJFUSION1415	BASE JFUSION1415	
4	BJFUSION1416	BASE JFUSION1416	
4	BJFUSION1417	BASE JFUSION1417	
4	BJFUSION1418	BASE JFUSION1418	
4	BJFUSION1514	BASE JFUSION1514	
4	BJFUSION1515	BASE JFUSION1515	
4	BJFUSION1516	BASE JFUSION1516	
4	BJFUSION1517	BASE JFUSION1517	
4	BJFUSION1518	BASE JFUSION1518	
4	BJFUSION1519	BASE JFUSION1519	
4	BJFUSION1520	BASE JFUSION1520	
4	BJFUSION1521	BASE JFUSION1521	
4	BJFUSION1614	BASE JFUSION1614	
4	BJFUSION1615	BASE JFUSION1615	
4	BJFUSION1616	BASE JFUSION1616	
4	BJFUSION1617	BASE JFUSION1617	
4	BJFUSION1618	BASE JFUSION1618	
4	BJFUSION1619	BASE JFUSION1619	
4	BJFUSION1620	BASE JFUSION1620	
4	BJFUSION1621	BASE JFUSION1621	
4	BJFUSION1622	BASE JFUSION1622	
4	BJFUSION1714	BASE JFUSION1714	
4	BJFUSION1715	BASE JFUSION1715	
4	BJFUSION1716	BASE JFUSION1716	
4	BJFUSION1717	BASE JFUSION1717	
4	BJFUSION1718	BASE JFUSION1718	
4	BJFUSION1719	BASE JFUSION1719	
4	BJFUSION1720	BASE JFUSION1720	
4	BJFUSION1721	BASE JFUSION1721	
4	BJFUSION1722	BASE JFUSION1722	
4	BJFUSION1723	BASE JFUSION1723	
4	BJFUSION1814	BASE JFUSION1814	
4	BJFUSION1815	BASE JFUSION1815	
4	BJFUSION1816	BASE JFUSION1816	
4	BJFUSION1817	BASE JFUSION1817	
4	BJFUSION1818	BASE JFUSION1818	
4	BJFUSION1819	BASE JFUSION1819	
4	BJFUSION1820	BASE JFUSION1820	
4	BJFUSION1821	BASE JFUSION1821	
4	BJFUSION1822	BASE JFUSION1822	
4	BJFUSION1823	BASE JFUSION1823	
4	BJFUSION1824	BASE JFUSION1824	
4	BJFUSION1915	BASE JFUSION1915	
4	BJFUSION1916	BASE JFUSION1916	
4	BJFUSION1917	BASE JFUSION1917	
4	BJFUSION1918	BASE JFUSION1918	
4	BJFUSION1919	BASE JFUSION1919	
4	BJFUSION1920	BASE JFUSION1920	
4	BJFUSION1921	BASE JFUSION1921	
4	BJFUSION1922	BASE JFUSION1922	
4	BJFUSION1923	BASE JFUSION1923	
4	BJFUSION1924	BASE JFUSION1924	

Pos.	Item Number	Description	Remarks
4	BJFUSION2015	BASE JFUSION2015	
4	BJFUSION2016	BASE JFUSION2016	
4	BJFUSION2017	BASE JFUSION2017	
4	BJFUSION2018	BASE JFUSION2018	
4	BJFUSION2019	BASE JFUSION2019	
4	BJFUSION2020	BASE JFUSION2020	
4	BJFUSION2021	BASE JFUSION2021	
4	BJFUSION2022	BASE JFUSION2022	
4	BJFUSION2023	BASE JFUSION2023	
4	BJFUSION2024	BASE JFUSION2024	
4	BJFUSION2117	BASE JFUSION2117	
4	BJFUSION2118	BASE JFUSION2118	
4	BJFUSION2119	BASE JFUSION2119	
4	BJFUSION2120	BASE JFUSION2120	
4	BJFUSION2121	BASE JFUSION2121	
4	BJFUSION2122	BASE JFUSION2122	
4	BJFUSION2123	BASE JFUSION2123	
4	BJFUSION2124	BASE JFUSION2124	
4	BJFUSION2217	BASE JFUSION2217	
4	BJFUSION2218	BASE JFUSION2218	
4	BJFUSION2219	BASE JFUSION2219	
4	BJFUSION2220	BASE JFUSION2220	
4	BJFUSION2221	BASE JFUSION2221	
4	BJFUSION2222	BASE JFUSION2222	
4	BJFUSION2223	BASE JFUSION2223	
4	BJFUSION2224	BASE JFUSION2224	
4	BJFUSION2319	BASE JFUSION2319	
4	BJFUSION2320	BASE JFUSION2320	
4	BJFUSION2321	BASE JFUSION2321	
4	BJFUSION2322	BASE JFUSION2322	
4	BJFUSION2323	BASE JFUSION2323	
4	BJFUSION2324	BASE JFUSION2324	
4	BJFUSION2419	BASE JFUSION2419	
4	BJFUSION2420	BASE JFUSION2420	
4	BJFUSION2421	BASE JFUSION2421	
4	BJFUSION2422	BASE JFUSION2422	
4	BJFUSION2423	BASE JFUSION2423	
4	BJFUSION2424	BASE JFUSION2424	
5	AIRPUMP	AIR PUMP	
6	PATCHKIT	AIR PATCH KIT	
7A	JBALCOP-ALH	OBLIQUITY PAD LH PLA A	
7A	JBALCOP-ARH	OBLIQUITY PAD RH PLA A	
7B	JBALCOP-BLH	OBLIQUITY PAD LH PLA B	
7B	JBALCOP-BRH	OBLIQUITY PAD RH PLA B	
7C	JBALCOP-CLH	OBLIQUITY PAD LH PLA C	
7C	JBALCOP-CRH	OBLIQUITY PAD RH PLA C	
7D	JBALCOP-DLH	OBLIQUITY PAD LH PLA D	
7D	JBALCOP-DRH	OBLIQUITY PAD RH PLA D	

Note: for part number information, please see attachment in informations panel.

Note: Cushion widths 14" & 15" use A size Air Bladders, 16" - 20" use B size bladders and 21"-24" use C size bladders

Note: for Obliquity Pads (items 7A-7D) second layer is one size smaller than the Obliquity Pad (i.e. for bottom layer size B, then top layer size A).

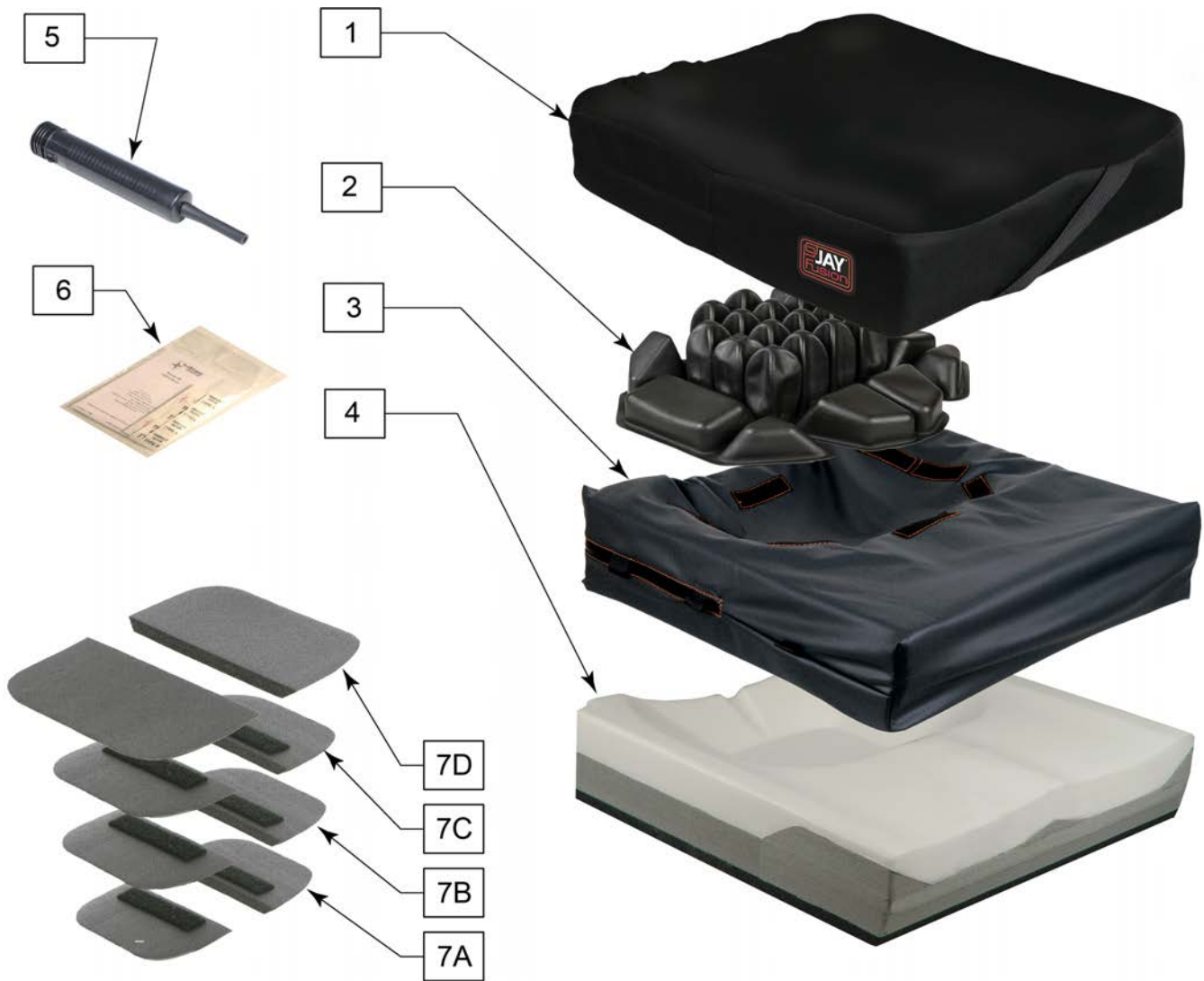

Pos.	Item Number	Description	Remarks
1	CJFUSION1414RPA	COV OUT FUSION RP AIR 14X14	
1	CJFUSION1415RPA	COV OUT FUSION RP AIR 14X15	
1	CJFUSION1416RPA	COV OUT FUSION RP AIR 14X16	
1	CJFUSION1417RPA	COV OUT FUSION RP AIR 14X17	
1	CJFUSION1418RPA	COV OUT FUSION RP AIR 14X18	
1	CJFUSION1514RPA	COV OUT FUSION RP AIR 15X14	

Pos.	Item Number	Description	Remarks
1	CJFUSION1515RPA	COV OUT FUSION RP AIR 15X15	
1	CJFUSION1516RPA	COV OUT FUSION RP AIR 15X16	
1	CJFUSION1517RPA	COV OUT FUSION RP AIR 15X17	
1	CJFUSION1518RPA	COV OUT FUSION RP AIR 15X18	
1	CJFUSION1519RPA	COV OUT FUSION RP AIR 15X19	
1	CJFUSION1520RPA	COV OUT FUSION RP AIR 15X20	
1	CJFUSION1521RPA	COV OUT FUSION RP AIR 15X21	
1	CJFUSION1614RPA	COV OUT FUSION RP AIR 16X14	
1	CJFUSION1615RPA	COV OUT FUSION RP AIR 16X15	
1	CJFUSION1616RPA	COV OUT FUSION RP AIR 16X16	
1	CJFUSION1617RPA	COV OUT FUSION RP AIR 16X17	
1	CJFUSION1618RPA	COV OUT FUSION RP AIR 16X18	
1	CJFUSION1619RPA	COV OUT FUSION RP AIR 16X19	
1	CJFUSION1620RPA	COV OUT FUSION RP AIR 16X20	
1	CJFUSION1621RPA	COV OUT FUSION RP AIR 16X21	
1	CJFUSION1622RPA	COV OUT FUSION RP AIR 16X22	
1	CJFUSION1714RPA	COV OUT FUSION RP AIR 17X14	
1	CJFUSION1715RPA	COV OUT FUSION RP AIR 17X15	
1	CJFUSION1716RPA	COV OUT FUSION RP AIR 17X16	
1	CJFUSION1717RPA	COV OUT FUSION RP AIR 17X17	
1	CJFUSION1718RPA	COV OUT FUSION RP AIR 17X18	
1	CJFUSION1719RPA	COV OUT FUSION RP AIR 17X19	
1	CJFUSION1720RPA	COV OUT FUSION RP AIR 17X20	
1	CJFUSION1721RPA	COV OUT FUSION RP AIR 17X21	
1	CJFUSION1722RPA	COV OUT FUSION RP AIR 17X22	
1	CJFUSION1723RPA	COV OUT FUSION RP AIR 17X23	
1	CJFUSION1814RPA	COV OUT FUSION RP AIR 18X14	
1	CJFUSION1815RPA	COV OUT FUSION RP AIR 18X15	
1	CJFUSION1816RPA	COV OUT FUSION RP AIR 18X16	
1	CJFUSION1817RPA	COV OUT FUSION RP AIR 18X17	
1	CJFUSION1818RPA	COV OUT FUSION RP AIR 18X18	
1	CJFUSION1819RPA	COV OUT FUSION RP AIR 18X19	
1	CJFUSION1820RPA	COV OUT FUSION RP AIR 18X20	
1	CJFUSION1821RPA	COV OUT FUSION RP AIR 18X21	
1	CJFUSION1822RPA	COV OUT FUSION RP AIR 18X22	
1	CJFUSION1823RPA	COV OUT FUSION RP AIR 18X23	
1	CJFUSION1824RPA	COV OUT FUSION RP AIR 18X24	
1	CJFUSION1915RPA	COV OUT FUSION RP AIR 19X15	
1	CJFUSION1916RPA	COV OUT FUSION RP AIR 19X16	
1	CJFUSION1917RPA	COV OUT FUSION RP AIR 19X17	
1	CJFUSION1918RPA	COV OUT FUSION RP AIR 19X18	
1	CJFUSION1919RPA	COV OUT FUSION RP AIR 19X19	
1	CJFUSION1920RPA	COV OUT FUSION RP AIR 19X20	
1	CJFUSION1921RPA	COV OUT FUSION RP AIR 19X21	
1	CJFUSION1922RPA	COV OUT FUSION RP AIR 19X22	
1	CJFUSION1923RPA	COV OUT FUSION RP AIR 19X23	
1	CJFUSION1924RPA	COV OUT FUSION RP AIR 19X24	
1	CJFUSION2015RPA	COV OUT FUSION RP AIR 20X15	
1	CJFUSION2016RPA	COV OUT FUSION RP AIR 20X16	
1	CJFUSION2017RPA	COV OUT FUSION RP AIR 20X17	
1	CJFUSION2018RPA	COV OUT FUSION RP AIR 20X18	
1	CJFUSION2019RPA	COV OUT FUSION RP AIR 20X19	
1	CJFUSION2020RPA	COV OUT FUSION RP AIR 20X20	
1	CJFUSION2021RPA	COV OUT FUSION RP AIR 20X21	
1	CJFUSION2022RPA	COV OUT FUSION RP AIR 20X22	
1	CJFUSION2023RPA	COV OUT FUSION RP AIR 20X23	
1	CJFUSION2024RPA	COV OUT FUSION RP AIR 20X24	

Pos.	Item Number	Description	Remarks
1	CJFUSION2117RPA	COV OUT FUSION RP AIR 21X17	
1	CJFUSION2118RPA	COV OUT FUSION RP AIR 21X18	
1	CJFUSION2119RPA	COV OUT FUSION RP AIR 21X19	
1	CJFUSION2120RPA	COV OUT FUSION RP AIR 21X20	
1	CJFUSION2121RPA	COV OUT FUSION RP AIR 21X21	
1	CJFUSION2122RPA	COV OUT FUSION RP AIR 21X22	
1	CJFUSION2123RPA	COV OUT FUSION RP AIR 21X23	
1	CJFUSION2124RPA	COV OUT FUSION RP AIR 21X24	
1	CJFUSION2217RPA	COV OUT FUSION RP AIR 22X17	
1	CJFUSION2218RPA	COV OUT FUSION RP AIR 22X18	
1	CJFUSION2219RPA	COV OUT FUSION RP AIR 22X19	
1	CJFUSION2220RPA	COV OUT FUSION RP AIR 22X20	
1	CJFUSION2221RPA	COV OUT FUSION RP AIR 22X21	
1	CJFUSION2222RPA	COV OUT FUSION RP AIR 22X22	
1	CJFUSION2223RPA	COV OUT FUSION RP AIR 22X23	
1	CJFUSION2224RPA	COV OUT FUSION RP AIR 22X24	
1	CJFUSION2319RPA	COV OUT FUSION RP AIR 23X19	
1	CJFUSION2320RPA	COV OUT FUSION RP AIR 23X20	
1	CJFUSION2321RPA	COV OUT FUSION RP AIR 23X21	
1	CJFUSION2322RPA	COV OUT FUSION RP AIR 23X22	
1	CJFUSION2323RPA	COV OUT FUSION RP AIR 23X23	
1	CJFUSION2324RPA	COV OUT FUSION RP AIR 23X24	
1	CJFUSION2419RPA	COV OUT FUSION RP AIR 24X19	
1	CJFUSION2420RPA	COV OUT FUSION RP AIR 24X20	
1	CJFUSION2421RPA	COV OUT FUSION RP AIR 24X21	
1	CJFUSION2422RPA	COV OUT FUSION RP AIR 24X22	
1	CJFUSION2423RPA	COV OUT FUSION RP AIR 24X23	
1	CJFUSION2424RPA	COV OUT FUSION RP AIR 24X24	
2	ASAJFUSION	AIR INSERT SINGLE VALVE SIZE A	Used in 14"&15" wide
2	ASBJFUSION	AIR INSERT SINGLE VALVE SIZE B	Used in 16"-20" wide
2	ASCJFUSION	AIR INSERT SINGLE VALVE SIZE C	Used in 21"-24" wide
2	ADAJFUSION	AIR INSERT DUAL VALVE SIZE A	Used in 14"&15" wide
2	ADBJFUSION	AIR INSERT DUAL VALVE SIZE B	Used in 16"-20" wide
2	ADCJFUSION	AIR INSERT DUAL VALVE SIZE C	Used in 21"-24" wide
3	ICJFUSION1414RPA	INN COV FUSION RP AIR 14X14	
3	ICJFUSION1415RPA	INN COV FUSION RP AIR 14X15	
3	ICJFUSION1416RPA	INN COV FUSION RP AIR 14X16	
3	ICJFUSION1417RPA	INN COV FUSION RP AIR 14X17	
3	ICJFUSION1418RPA	INN COV FUSION RP AIR 14X18	
3	ICJFUSION1514RPA	INN COV FUSION RP AIR 15X14	
3	ICJFUSION1515RPA	INN COV FUSION RP AIR 15X15	
3	ICJFUSION1516RPA	INN COV FUSION RP AIR 15X16	
3	ICJFUSION1517RPA	INN COV FUSION RP AIR 15X17	
3	ICJFUSION1518RPA	INN COV FUSION RP AIR 15X18	
3	ICJFUSION1519RPA	INN COV FUSION RP AIR 15X19	
3	ICJFUSION1520RPA	INN COV FUSION RP AIR 15X19	
3	ICJFUSION1521RPA	INN COV FUSION RP AIR 15X21	
3	ICJFUSION1614RPA	INN COV FUSION RP AIR 16X14	
3	ICJFUSION1615RPA	INN COV FUSION RP AIR 16X15	
3	ICJFUSION1616RPA	INN COV FUSION RP AIR 16X16	
3	ICJFUSION1617RPA	INN COV FUSION RP AIR 16X17	
3	ICJFUSION1618RPA	INN COV FUSION RP AIR 16X18	
3	ICJFUSION1619RPA	INN COV FUSION RP AIR 16X19	
3	ICJFUSION1620RPA	INN COV FUSION RP AIR 16X20	
3	ICJFUSION1621RPA	INN COV FUSION RP AIR 16X21	
3	ICJFUSION1622RPA	INN COV FUSION RP AIR 16X22	
3	ICJFUSION1714RPA	INN COV FUSION RP AIR 17X14	

Pos.	Item Number	Description	Remarks
3	ICJFUSION1715RPA	INN COV FUSION RP AIR 17X15	
3	ICJFUSION1716RPA	INN COV FUSION RP AIR 17X16	
3	ICJFUSION1717RPA	INN COV FUSION RP AIR 17X17	
3	ICJFUSION1718RPA	INN COV FUSION RP AIR 17X18	
3	ICJFUSION1719RPA	INN COV FUSION RP AIR 17X19	
3	ICJFUSION1720RPA	INN COV FUSION RP AIR 17X20	
3	ICJFUSION1721RPA	INN COV FUSION RP AIR 17X21	
3	ICJFUSION1722RPA	INN COV FUSION RP AIR 17X22	
3	ICJFUSION1723RPA	INN COV FUSION RP AIR 17X23	
3	ICJFUSION1814RPA	INN COV FUSION RP AIR 18X14	
3	ICJFUSION1815RPA	INN COV FUSION RP AIR 18X15	
3	ICJFUSION1816RPA	INN COV FUSION RP AIR 18X16	
3	ICJFUSION1817RPA	INN COV FUSION RP AIR 18X17	
3	ICJFUSION1818RPA	INN COV FUSION RP AIR 18X18	
3	ICJFUSION1819RPA	INN COV FUSION RP AIR 18X19	
3	ICJFUSION1820RPA	INN COV FUSION RP AIR 18X20	
3	ICJFUSION1821RPA	INN COV FUSION RP AIR 18X21	
3	ICJFUSION1822RPA	INN COV FUSION RP AIR 18X22	
3	ICJFUSION1823RPA	INN COV FUSION RP AIR 18X23	
3	ICJFUSION1824RPA	INN COV FUSION RP AIR 18X24	
3	ICJFUSION1915RPA	INN COV FUSION RP AIR 19X15	
3	ICJFUSION1916RPA	INN COV FUSION RP AIR 19X16	
3	ICJFUSION1917RPA	INN COV FUSION RP AIR 19X17	
3	ICJFUSION1918RPA	INN COV FUSION RP AIR 19X18	
3	ICJFUSION1919RPA	INN COV FUSION RP AIR 19X19	
3	ICJFUSION1920RPA	INN COV FUSION RP AIR 19X20	
3	ICJFUSION1921RPA	INN COV FUSION RP AIR 19X21	
3	ICJFUSION1922RPA	INN COV FUSION RP AIR 19X22	
3	ICJFUSION1923RPA	INN COV FUSION RP AIR 19X23	
3	ICJFUSION1924RPA	INN COV FUSION RP AIR 19X24	
3	ICJFUSION2015RPA	INN COV FUSION RP AIR 20X15	
3	ICJFUSION2016RPA	INN COV FUSION RP AIR 20X16	
3	ICJFUSION2017RPA	INN COV FUSION RP AIR 20X17	
3	ICJFUSION2018RPA	INN COV FUSION RP AIR 20X18	
3	ICJFUSION2019RPA	INN COV FUSION RP AIR 20X19	
3	ICJFUSION2020RPA	INN COV FUSION RP AIR 20X20	
3	ICJFUSION2021RPA	INN COV FUSION RP AIR 20X21	
3	ICJFUSION2022RPA	INN COV FUSION RP AIR 20X22	
3	ICJFUSION2023RPA	INN COV FUSION RP AIR 20X23	
3	ICJFUSION2024RPA	INN COV FUSION RP AIR 20X24	
3	ICJFUSION2117RPA	INN COV FUSION RP AIR 21X17	
3	ICJFUSION2118RPA	INN COV FUSION RP AIR 21X18	
3	ICJFUSION2119RPA	INN COV FUSION RP AIR 21X19	
3	ICJFUSION2120RPA	INN COV FUSION RP AIR 21X20	
3	ICJFUSION2121RPA	INN COV FUSION RP AIR 21X21	
3	ICJFUSION2122RPA	INN COV FUSION RP AIR 21X22	
3	ICJFUSION2123RPA	INN COV FUSION RP AIR 21X23	
3	ICJFUSION2124RPA	INN COV FUSION RP AIR 21X24	
3	ICJFUSION2217RPA	INN COV FUSION RP AIR 22X17	
3	ICJFUSION2218RPA	INN COV FUSION RP AIR 22X18	
3	ICJFUSION2219RPA	INN COV FUSION RP AIR 22X19	
3	ICJFUSION2220RPA	INN COV FUSION RP AIR 22X20	
3	ICJFUSION2221RPA	INN COV FUSION RP AIR 22X21	
3	ICJFUSION2222RPA	INN COV FUSION RP AIR 22X22	
3	ICJFUSION2223RPA	INN COV FUSION RP AIR 22X23	
3	ICJFUSION2224RPA	INN COV FUSION RP AIR 22X24	
3	ICJFUSION2319RPA	INN COV FUSION RP AIR 23X19	


Pos.	Item Number	Description	Remarks
3	ICJFUSION2320RPA	INN COV FUSION RP AIR 23X20	
3	ICJFUSION2321RPA	INN COV FUSION RP AIR 23X21	
3	ICJFUSION2322RPA	INN COV FUSION RP AIR 23X22	
3	ICJFUSION2323RPA	INN COV FUSION RP AIR 23X23	
3	ICJFUSION2324RPA	INN COV FUSION RP AIR 23X24	
3	ICJFUSION2419RPA	INN COV FUSION RP AIR 24X19	
3	ICJFUSION2420RPA	INN COV FUSION RP AIR 24X20	
3	ICJFUSION2421RPA	INN COV FUSION RP AIR 24X21	
3	ICJFUSION2422RPA	INN COV FUSION RP AIR 24X22	
3	ICJFUSION2423RPA	INN COV FUSION RP AIR 24X23	
3	ICJFUSION2424RPA	INN COV FUSION RP AIR 24X24	
4	BJFUSION1414RP	BASE JAY FUSION RPROF 14X14	
4	BJFUSION1415RP	BASE JAY FUSION RPROF 14X15	
4	BJFUSION1416RP	BASE JAY FUSION RPROF 14X16	
4	BJFUSION1417RP	BASE JAY FUSION RPROF 14X17	
4	BJFUSION1418RP	BASE JAY FUSION RPROF 14X18	
4	BJFUSION1514RP	BASE JAY FUSION RPROF 15X14	
4	BJFUSION1515RP	BASE JAY FUSION RPROF 15X15	
4	BJFUSION1516RP	BASE JAY FUSION RPROF 15X16	
4	BJFUSION1517RP	BASE JAY FUSION RPROF 15X17	
4	BJFUSION1518RP	BASE JAY FUSION RPROF 15X18	
4	BJFUSION1519RP	BASE JAY FUSION RPROF 15X19	
4	BJFUSION1520RP	BASE JAY FUSION RPROF 15X19	
4	BJFUSION1521RP	BASE JAY FUSION RPROF 15X21	
4	BJFUSION1614RP	BASE JAY FUSION RPROF 16X14	
4	BJFUSION1615RP	BASE JAY FUSION RPROF 16X15	
4	BJFUSION1616RP	BASE JAY FUSION RPROF 16X16	
4	BJFUSION1617RP	BASE JAY FUSION RPROF 16X17	
4	BJFUSION1618RP	BASE JAY FUSION RPROF 16X18	
4	BJFUSION1619RP	BASE JAY FUSION RPROF 16X19	
4	BJFUSION1620RP	BASE JAY FUSION RPROF 16X20	
4	BJFUSION1621RP	BASE JAY FUSION RPROF 16X21	
4	BJFUSION1622RP	BASE JAY FUSION RPROF 16X22	
4	BJFUSION1714RP	BASE JAY FUSION RPROF 17X14	
4	BJFUSION1715RP	BASE JAY FUSION RPROF 17X15	
4	BJFUSION1716RP	BASE JAY FUSION RPROF 17X16	
4	BJFUSION1717RP	BASE JAY FUSION RPROF 17X17	
4	BJFUSION1718RP	BASE JAY FUSION RPROF 17X18	
4	BJFUSION1719RP	BASE JAY FUSION RPROF 17X19	
4	BJFUSION1720RP	BASE JAY FUSION RPROF 17X20	
4	BJFUSION1721RP	BASE JAY FUSION RPROF 17X21	
4	BJFUSION1722RP	BASE JAY FUSION RPROF 17X22	
4	BJFUSION1723RP	BASE JAY FUSION RPROF 17X23	
4	BJFUSION1814RP	BASE JAY FUSION RPROF 18X14	
4	BJFUSION1815RP	BASE JAY FUSION RPROF 18X15	
4	BJFUSION1816RP	BASE JAY FUSION RPROF 18X16	
4	BJFUSION1817RP	BASE JAY FUSION RPROF 18X17	
4	BJFUSION1818RP	BASE JAY FUSION RPROF 18X18	
4	BJFUSION1819RP	BASE JAY FUSION RPROF 18X19	
4	BJFUSION1820RP	BASE JAY FUSION RPROF 18X20	
4	BJFUSION1821RP	BASE JAY FUSION RPROF 18X21	
4	BJFUSION1822RP	BASE JAY FUSION RPROF 18X22	
4	BJFUSION1823RP	BASE JAY FUSION RPROF 18X23	
4	BJFUSION1824RP	BASE JAY FUSION RPROF 18X24	
4	BJFUSION1915RP	BASE JAY FUSION RPROF 19X15	
4	BJFUSION1916RP	BASE JAY FUSION RPROF 19X16	
4	BJFUSION1917RP	BASE JAY FUSION RPROF 19X17	

Pos.	Item Number	Description	Remarks
4	BJFUSION1918RP	BASE JAY FUSION RPROF 19X18	
4	BJFUSION1919RP	BASE JAY FUSION RPROF 19X19	
4	BJFUSION1920RP	BASE JAY FUSION RPROF 19X20	
4	BJFUSION1921RP	BASE JAY FUSION RPROF 19X21	
4	BJFUSION1922RP	BASE JAY FUSION RPROF 19X22	
4	BJFUSION1923RP	BASE JAY FUSION RPROF 19X23	
4	BJFUSION1924RP	BASE JAY FUSION RPROF 19X24	
4	BJFUSION2015RP	BASE JAY FUSION RPROF 20X15	
4	BJFUSION2016RP	BASE JAY FUSION RPROF 20X16	
4	BJFUSION2017RP	BASE JAY FUSION RPROF 20X17	
4	BJFUSION2018RP	BASE JAY FUSION RPROF 20X18	
4	BJFUSION2019RP	BASE JAY FUSION RPROF 20X19	
4	BJFUSION2020RP	BASE JAY FUSION RPROF 20X20	
4	BJFUSION2021RP	BASE JAY FUSION RPROF 20X21	
4	BJFUSION2022RP	BASE JAY FUSION RPROF 20X22	
4	BJFUSION2023RP	BASE JAY FUSION RPROF 20X23	
4	BJFUSION2024RP	BASE JAY FUSION RPROF 20X24	
4	BJFUSION2117RP	BASE JAY FUSION RPROF 21X17	
4	BJFUSION2118RP	BASE JAY FUSION RPROF 21X18	
4	BJFUSION2119RP	BASE JAY FUSION RPROF 21X19	
4	BJFUSION2120RP	BASE JAY FUSION RPROF 21X20	
4	BJFUSION2121RP	BASE JAY FUSION RPROF 21X21	
4	BJFUSION2122RP	BASE JAY FUSION RPROF 21X22	
4	BJFUSION2123RP	BASE JAY FUSION RPROF 21X23	
4	BJFUSION2124RP	BASE JAY FUSION RPROF 21X24	
4	BJFUSION2217RP	BASE JAY FUSION RPROF 22X17	
4	BJFUSION2218RP	BASE JAY FUSION RPROF 22X18	
4	BJFUSION2219RP	BASE JAY FUSION RPROF 22X19	
4	BJFUSION2220RP	BASE JAY FUSION RPROF 22X20	
4	BJFUSION2221RP	BASE JAY FUSION RPROF 22X21	
4	BJFUSION2222RP	BASE JAY FUSION RPROF 22X22	
4	BJFUSION2223RP	BASE JAY FUSION RPROF 22X23	
4	BJFUSION2224RP	BASE JAY FUSION RPROF 22X24	
4	BJFUSION2319RP	BASE JAY FUSION RPROF 23X19	
4	BJFUSION2320RP	BASE JAY FUSION RPROF 23X20	
4	BJFUSION2321RP	BASE JAY FUSION RPROF 23X21	
4	BJFUSION2322RP	BASE JAY FUSION RPROF 23X22	
4	BJFUSION2323RP	BASE JAY FUSION RPROF 23X23	
4	BJFUSION2324RP	BASE JAY FUSION RPROF 23X24	
4	BJFUSION2419RP	BASE JAY FUSION RPROF 24X19	
4	BJFUSION2420RP	BASE JAY FUSION RPROF 24X20	
4	BJFUSION2421RP	BASE JAY FUSION RPROF 24X21	
4	BJFUSION2422RP	BASE JAY FUSION RPROF 24X22	
4	BJFUSION2423RP	BASE JAY FUSION RPROF 24X23	
4	BJFUSION2424RP	BASE JAY FUSION RPROF 24X24	
5	AIRPUMP	AIR PUMP	
6	PATCHKIT	AIR PATCH KIT	
7A	JBALCOP-ALH	OBLIQUITY PAD LH PLA A	Left- 14 & 15 wide
7A	JBALCOP-ARH	OBLIQUITY PAD RH PLA A	Right- 14 & 15 wide
7B	JBALCOP-BLH	OBLIQUITY PAD LH PLA B	Left- 16 thru 21 wide
7B	JBALCOP-BRH	OBLIQUITY PAD RH PLA B	Right- 16 thru 21 wide
7C	JBALCOP-CLH	OBLIQUITY PAD LH PLA C	Left- 22 thru 24 wide
7C	JBALCOP-CRH	OBLIQUITY PAD RH PLA C	Right- 22 thru 24 wide
7D	JBALCOP-DLH	OBLIQUITY PAD LH PLA D	Left
7D	JBALCOP-DRH	OBLIQUITY PAD RH PLA D	Right

Note: for part number information, please see attachment in informations panel.

Note: Cushion widths 14" & 15" use A size Fluid Bladders, 16" - 20" use B size bladders and 21"-24" use C size bladders.

Note: for Obliquity Pads (items 5A-5D) second layer is one size smaller than the Obliquity Pad (i.e. for bottom layer size B, then top layer size A).

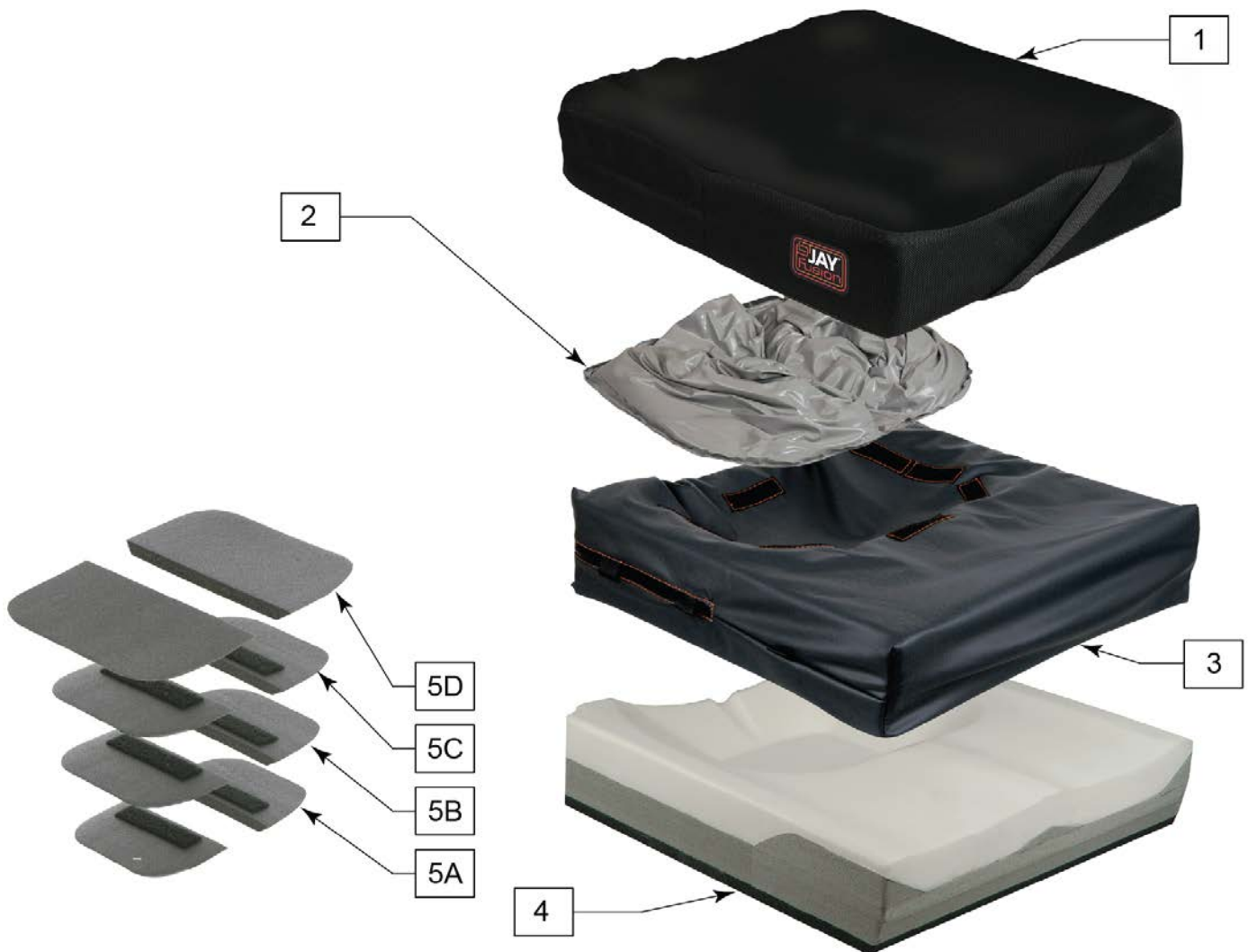

Pos.	Item Number	Description	Remarks
1	CJFUSION1414RP	COV OUT FUSION RP FLD 14X14	
1	CJFUSION1415RP	COV OUT FUSION RP FLD 14X15	
1	CJFUSION1416RP	COV OUT FUSION RP FLD 14X16	
1	CJFUSION1417RP	COV OUT FUSION RP FLD 14X17	
1	CJFUSION1418RP	COV OUT FUSION RP FLD 14X18	
1	CJFUSION1514RP	COV OUT FUSION RP FLD 15X14	
1	CJFUSION1515RP	COV OUT FUSION RP FLD 15X15	

Pos.	Item Number	Description	Remarks
1	CJFUSION1516RP	COV OUT FUSION RP FLD 15X16	
1	CJFUSION1517RP	COV OUT FUSION RP FLD 15X17	
1	CJFUSION1518RP	COV OUT FUSION RP FLD 15X18	
1	CJFUSION1519RP	COV OUT FUSION RP FLD 15X19	
1	CJFUSION1520RP	COV OUT FUSION RP FLD 15X19	
1	CJFUSION1521RP	COV OUT FUSION RP FLD 15X21	
1	CJFUSION1614RP	COV OUT FUSION RP FLD 16X14	
1	CJFUSION1615RP	COV OUT FUSION RP FLD 16X15	
1	CJFUSION1616RP	COV OUT FUSION RP FLD 16X16	
1	CJFUSION1617RP	COV OUT FUSION RP FLD 16X17	
1	CJFUSION1618RP	COV OUT FUSION RP FLD 16X18	
1	CJFUSION1619RP	COV OUT FUSION RP FLD 16X19	
1	CJFUSION1620RP	COV OUT FUSION RP FLD 16X20	
1	CJFUSION1621RP	COV OUT FUSION RP FLD 16X21	
1	CJFUSION1622RP	COV OUT FUSION RP FLD 16X22	
1	CJFUSION1714RP	COV OUT FUSION RP FLD 17X14	
1	CJFUSION1715RP	COV OUT FUSION RP FLD 17X15	
1	CJFUSION1716RP	COV OUT FUSION RP FLD 17X16	
1	CJFUSION1717RP	COV OUT FUSION RP FLD 17X17	
1	CJFUSION1718RP	COV OUT FUSION RP FLD 17X18	
1	CJFUSION1719RP	COV OUT FUSION RP FLD 17X19	
1	CJFUSION1720RP	COV OUT FUSION RP FLD 17X20	
1	CJFUSION1721RP	COV OUT FUSION RP FLD 17X21	
1	CJFUSION1722RP	COV OUT FUSION RP FLD 17X22	
1	CJFUSION1723RP	COV OUT FUSION RP FLD 17X23	
1	CJFUSION1814RP	COV OUT FUSION RP FLD 18X14	
1	CJFUSION1815RP	COV OUT FUSION RP FLD 18X15	
1	CJFUSION1816RP	COV OUT FUSION RP FLD 18X16	
1	CJFUSION1817RP	COV OUT FUSION RP FLD 18X17	
1	CJFUSION1818RP	COV OUT FUSION RP FLD 18X18	
1	CJFUSION1819RP	COV OUT FUSION RP FLD 18X19	
1	CJFUSION1820RP	COV OUT FUSION RP FLD 18X20	
1	CJFUSION1821RP	COV OUT FUSION RP FLD 18X21	
1	CJFUSION1822RP	COV OUT FUSION RP FLD 18X22	
1	CJFUSION1823RP	COV OUT FUSION RP FLD 18X23	
1	CJFUSION1824RP	COV OUT FUSION RP FLD 18X24	
1	CJFUSION1915RP	COV OUT FUSION RP FLD 19X15	
1	CJFUSION1916RP	COV OUT FUSION RP FLD 19X16	
1	CJFUSION1917RP	COV OUT FUSION RP FLD 19X17	
1	CJFUSION1918RP	COV OUT FUSION RP FLD 19X18	
1	CJFUSION1919RP	COV OUT FUSION RP FLD 19X19	
1	CJFUSION1920RP	COV OUT FUSION RP FLD 19X20	
1	CJFUSION1921RP	COV OUT FUSION RP FLD 19X21	
1	CJFUSION1922RP	COV OUT FUSION RP FLD 19X22	
1	CJFUSION1923RP	COV OUT FUSION RP FLD 19X23	
1	CJFUSION1924RP	COV OUT FUSION RP FLD 19X24	
1	CJFUSION2015RP	COV OUT FUSION RP FLD 20X15	
1	CJFUSION2016RP	COV OUT FUSION RP FLD 20X16	
1	CJFUSION2017RP	COV OUT FUSION RP FLD 20X17	
1	CJFUSION2018RP	COV OUT FUSION RP FLD 20X18	
1	CJFUSION2019RP	COV OUT FUSION RP FLD 20X19	
1	CJFUSION2020RP	COV OUT FUSION RP FLD 20X20	
1	CJFUSION2021RP	COV OUT FUSION RP FLD 20X21	
1	CJFUSION2022RP	COV OUT FUSION RP FLD 20X22	
1	CJFUSION2023RP	COV OUT FUSION RP FLD 20X23	
1	CJFUSION2024RP	COV OUT FUSION RP FLD 20X24	
1	CJFUSION2117RP	COV OUT FUSION RP FLD 21X17	

Pos.	Item Number	Description	Remarks
1	CJFUSION2118RP	COV OUT FUSION RP FLD 21X18	
1	CJFUSION2119RP	COV OUT FUSION RP FLD 21X19	
1	CJFUSION2120RP	COV OUT FUSION RP FLD 21X20	
1	CJFUSION2121RP	COV OUT FUSION RP FLD 21X21	
1	CJFUSION2122RP	COV OUT FUSION RP FLD 21X22	
1	CJFUSION2123RP	COV OUT FUSION RP FLD 21X23	
1	CJFUSION2124RP	COV OUT FUSION RP FLD 21X24	
1	CJFUSION2217RP	COV OUT FUSION RP FLD 22X17	
1	CJFUSION2218RP	COV OUT FUSION RP FLD 22X18	
1	CJFUSION2219RP	COV OUT FUSION RP FLD 22X19	
1	CJFUSION2220RP	COV OUT FUSION RP FLD 22X20	
1	CJFUSION2221RP	COV OUT FUSION RP FLD 22X21	
1	CJFUSION2222RP	COV OUT FUSION RP FLD 22X22	
1	CJFUSION2223RP	COV OUT FUSION RP FLD 22X23	
1	CJFUSION2224RP	COV OUT FUSION RP FLD 22X24	
1	CJFUSION2319RP	COV OUT FUSION RP FLD 23X19	
1	CJFUSION2320RP	COV OUT FUSION RP FLD 23X20	
1	CJFUSION2321RP	COV OUT FUSION RP FLD 23X21	
1	CJFUSION2322RP	COV OUT FUSION RP FLD 23X22	
1	CJFUSION2323RP	COV OUT FUSION RP FLD 23X23	
1	CJFUSION2324RP	COV OUT FUSION RP FLD 23X24	
1	CJFUSION2419RP	COV OUT FUSION RP FLD 24X19	
1	CJFUSION2420RP	COV OUT FUSION RP FLD 24X20	
1	CJFUSION2421RP	COV OUT FUSION RP FLD 24X21	
1	CJFUSION2422RP	COV OUT FUSION RP FLD 24X22	
1	CJFUSION2423RP	COV OUT FUSION RP FLD 24X23	
1	CJFUSION2424RP	COV OUT FUSION RP FLD 24X24	
2	FAJFUSION	FLUID INSERT SIZE A JFUSION	Used in 14"&15" wide
2	FBJFUSION	FLUID INSERT SIZE B JFUSION	Used in 16"-20" wide
2	FCJFUSION	FLUID INSERT SIZE C JFUSION	Used in 21"-24" wide
3	ICJFUSION1414RP	INN COV FUSION RP FLD 14X14	
3	ICJFUSION1415RP	INN COV FUSION RP FLD 14X15	
3	ICJFUSION1416RP	INN COV FUSION RP FLD 14X16	
3	ICJFUSION1417RP	INN COV FUSION RP FLD 14X17	
3	ICJFUSION1418RP	INN COV FUSION RP FLD 14X18	
3	ICJFUSION1514RP	INN COV FUSION RP FLD 15X14	
3	ICJFUSION1515RP	INN COV FUSION RP FLD 15X15	
3	ICJFUSION1516RP	INN COV FUSION RP FLD 15X16	
3	ICJFUSION1517RP	INN COV FUSION RP FLD 15X17	
3	ICJFUSION1518RP	INN COV FUSION RP FLD 15X18	
3	ICJFUSION1519RP	INN COV FUSION RP FLD 15X19	
3	ICJFUSION1520RP	INN COV FUSION RP FLD 15X19	
3	ICJFUSION1521RP	INN COV FUSION RP FLD 15X21	
3	ICJFUSION1614RP	INN COV FUSION RP FLD 16X14	
3	ICJFUSION1615RP	INN COV FUSION RP FLD 16X15	
3	ICJFUSION1616RP	INN COV FUSION RP FLD 16X16	
3	ICJFUSION1617RP	INN COV FUSION RP FLD 16X17	
3	ICJFUSION1618RP	INN COV FUSION RP FLD 16X18	
3	ICJFUSION1619RP	INN COV FUSION RP FLD 16X19	
3	ICJFUSION1620RP	INN COV FUSION RP FLD 16X20	
3	ICJFUSION1621RP	INN COV FUSION RP FLD 16X21	
3	ICJFUSION1622RP	INN COV FUSION RP FLD 16X22	
3	ICJFUSION1714RP	INN COV FUSION RP FLD 17X14	
3	ICJFUSION1715RP	INN COV FUSION RP FLD 17X15	
3	ICJFUSION1716RP	INN COV FUSION RP FLD 17X16	
3	ICJFUSION1717RP	INN COV FUSION RP FLD 17X17	
3	ICJFUSION1718RP	INN COV FUSION RP FLD 17X18	

Pos.	Item Number	Description	Remarks
3	ICJFUSION1719RP	INN COV FUSION RP FLD 17X19	
3	ICJFUSION1720RP	INN COV FUSION RP FLD 17X20	
3	ICJFUSION1721RP	INN COV FUSION RP FLD 17X21	
3	ICJFUSION1722RP	INN COV FUSION RP FLD 17X22	
3	ICJFUSION1723RP	INN COV FUSION RP FLD 17X23	
3	ICJFUSION1814RP	INN COV FUSION RP FLD 18X14	
3	ICJFUSION1815RP	INN COV FUSION RP FLD 18X15	
3	ICJFUSION1816RP	INN COV FUSION RP FLD 18X16	
3	ICJFUSION1817RP	INN COV FUSION RP FLD 18X17	
3	ICJFUSION1818RP	INN COV FUSION RP FLD 18X18	
3	ICJFUSION1819RP	INN COV FUSION RP FLD 18X19	
3	ICJFUSION1820RP	INN COV FUSION RP FLD 18X20	
3	ICJFUSION1821RP	INN COV FUSION RP FLD 18X21	
3	ICJFUSION1822RP	INN COV FUSION RP FLD 18X22	
3	ICJFUSION1823RP	INN COV FUSION RP FLD 18X23	
3	ICJFUSION1824RP	INN COV FUSION RP FLD 18X24	
3	ICJFUSION1915RP	INN COV FUSION RP FLD 19X15	
3	ICJFUSION1916RP	INN COV FUSION RP FLD 19X16	
3	ICJFUSION1917RP	INN COV FUSION RP FLD 19X17	
3	ICJFUSION1918RP	INN COV FUSION RP FLD 19X18	
3	ICJFUSION1919RP	INN COV FUSION RP FLD 19X19	
3	ICJFUSION1920RP	INN COV FUSION RP FLD 19X20	
3	ICJFUSION1921RP	INN COV FUSION RP FLD 19X21	
3	ICJFUSION1922RP	INN COV FUSION RP FLD 19X22	
3	ICJFUSION1923RP	INN COV FUSION RP FLD 19X23	
3	ICJFUSION1924RP	INN COV FUSION RP FLD 19X24	
3	ICJFUSION2015RP	INN COV FUSION RP FLD 20X15	
3	ICJFUSION2016RP	INN COV FUSION RP FLD 20X16	
3	ICJFUSION2017RP	INN COV FUSION RP FLD 20X17	
3	ICJFUSION2018RP	INN COV FUSION RP FLD 20X18	
3	ICJFUSION2019RP	INN COV FUSION RP FLD 20X19	
3	ICJFUSION2020RP	INN COV FUSION RP FLD 20X20	
3	ICJFUSION2021RP	INN COV FUSION RP FLD 20X21	
3	ICJFUSION2022RP	INN COV FUSION RP FLD 20X22	
3	ICJFUSION2023RP	INN COV FUSION RP FLD 20X23	
3	ICJFUSION2024RP	INN COV FUSION RP FLD 20X24	
3	ICJFUSION2117RP	INN COV FUSION RP FLD 21X17	
3	ICJFUSION2118RP	INN COV FUSION RP FLD 21X18	
3	ICJFUSION2119RP	INN COV FUSION RP FLD 21X19	
3	ICJFUSION2120RP	INN COV FUSION RP FLD 21X20	
3	ICJFUSION2121RP	INN COV FUSION RP FLD 21X21	
3	ICJFUSION2122RP	INN COV FUSION RP FLD 21X22	
3	ICJFUSION2123RP	INN COV FUSION RP FLD 21X23	
3	ICJFUSION2124RP	INN COV FUSION RP FLD 21X24	
3	ICJFUSION2217RP	INN COV FUSION RP FLD 22X17	
3	ICJFUSION2218RP	INN COV FUSION RP FLD 22X18	
3	ICJFUSION2219RP	INN COV FUSION RP FLD 22X19	
3	ICJFUSION2220RP	INN COV FUSION RP FLD 22X20	
3	ICJFUSION2221RP	INN COV FUSION RP FLD 22X21	
3	ICJFUSION2222RP	INN COV FUSION RP FLD 22X22	
3	ICJFUSION2223RP	INN COV FUSION RP FLD 22X23	
3	ICJFUSION2224RP	INN COV FUSION RP FLD 22X24	
3	ICJFUSION2319RP	INN COV FUSION RP FLD 23X19	
3	ICJFUSION2320RP	INN COV FUSION RP FLD 23X20	
3	ICJFUSION2321RP	INN COV FUSION RP FLD 23X21	
3	ICJFUSION2322RP	INN COV FUSION RP FLD 23X22	
3	ICJFUSION2323RP	INN COV FUSION RP FLD 23X23	

Pos.	Item Number	Description	Remarks
3	ICJFUSION2324RP	INN COV FUSION RP FLD 23X24	
3	ICJFUSION2419RP	INN COV FUSION RP FLD 24X19	
3	ICJFUSION2420RP	INN COV FUSION RP FLD 24X20	
3	ICJFUSION2421RP	INN COV FUSION RP FLD 24X21	
3	ICJFUSION2422RP	INN COV FUSION RP FLD 24X22	
3	ICJFUSION2423RP	INN COV FUSION RP FLD 24X23	
3	ICJFUSION2424RP	INN COV FUSION RP FLD 24X24	
4	BJFUSION1414RP	BASE JAY FUSION RPROF 14X14	
4	BJFUSION1415RP	BASE JAY FUSION RPROF 14X15	
4	BJFUSION1416RP	BASE JAY FUSION RPROF 14X16	
4	BJFUSION1417RP	BASE JAY FUSION RPROF 14X17	
4	BJFUSION1418RP	BASE JAY FUSION RPROF 14X18	
4	BJFUSION1514RP	BASE JAY FUSION RPROF 15X14	
4	BJFUSION1515RP	BASE JAY FUSION RPROF 15X15	
4	BJFUSION1516RP	BASE JAY FUSION RPROF 15X16	
4	BJFUSION1517RP	BASE JAY FUSION RPROF 15X17	
4	BJFUSION1518RP	BASE JAY FUSION RPROF 15X18	
4	BJFUSION1519RP	BASE JAY FUSION RPROF 15X19	
4	BJFUSION1520RP	BASE JAY FUSION RPROF 15X19	
4	BJFUSION1521RP	BASE JAY FUSION RPROF 15X21	
4	BJFUSION1614RP	BASE JAY FUSION RPROF 16X14	
4	BJFUSION1615RP	BASE JAY FUSION RPROF 16X15	
4	BJFUSION1616RP	BASE JAY FUSION RPROF 16X16	
4	BJFUSION1617RP	BASE JAY FUSION RPROF 16X17	
4	BJFUSION1618RP	BASE JAY FUSION RPROF 16X18	
4	BJFUSION1619RP	BASE JAY FUSION RPROF 16X19	
4	BJFUSION1620RP	BASE JAY FUSION RPROF 16X20	
4	BJFUSION1621RP	BASE JAY FUSION RPROF 16X21	
4	BJFUSION1622RP	BASE JAY FUSION RPROF 16X22	
4	BJFUSION1714RP	BASE JAY FUSION RPROF 17X14	
4	BJFUSION1715RP	BASE JAY FUSION RPROF 17X15	
4	BJFUSION1716RP	BASE JAY FUSION RPROF 17X16	
4	BJFUSION1717RP	BASE JAY FUSION RPROF 17X17	
4	BJFUSION1718RP	BASE JAY FUSION RPROF 17X18	
4	BJFUSION1719RP	BASE JAY FUSION RPROF 17X19	
4	BJFUSION1720RP	BASE JAY FUSION RPROF 17X20	
4	BJFUSION1721RP	BASE JAY FUSION RPROF 17X21	
4	BJFUSION1722RP	BASE JAY FUSION RPROF 17X22	
4	BJFUSION1723RP	BASE JAY FUSION RPROF 17X23	
4	BJFUSION1814RP	BASE JAY FUSION RPROF 18X14	
4	BJFUSION1815RP	BASE JAY FUSION RPROF 18X15	
4	BJFUSION1816RP	BASE JAY FUSION RPROF 18X16	
4	BJFUSION1817RP	BASE JAY FUSION RPROF 18X17	
4	BJFUSION1818RP	BASE JAY FUSION RPROF 18X18	
4	BJFUSION1819RP	BASE JAY FUSION RPROF 18X19	
4	BJFUSION1820RP	BASE JAY FUSION RPROF 18X20	
4	BJFUSION1821RP	BASE JAY FUSION RPROF 18X21	
4	BJFUSION1822RP	BASE JAY FUSION RPROF 18X22	
4	BJFUSION1823RP	BASE JAY FUSION RPROF 18X23	
4	BJFUSION1824RP	BASE JAY FUSION RPROF 18X24	
4	BJFUSION1915RP	BASE JAY FUSION RPROF 19X15	
4	BJFUSION1916RP	BASE JAY FUSION RPROF 19X16	
4	BJFUSION1917RP	BASE JAY FUSION RPROF 19X17	
4	BJFUSION1918RP	BASE JAY FUSION RPROF 19X18	
4	BJFUSION1919RP	BASE JAY FUSION RPROF 19X19	
4	BJFUSION1920RP	BASE JAY FUSION RPROF 19X20	
4	BJFUSION1921RP	BASE JAY FUSION RPROF 19X21	

Pos.	Item Number	Description	Remarks
4	BJFUSION1922RP	BASE JAY FUSION RPROF 19X22	
4	BJFUSION1923RP	BASE JAY FUSION RPROF 19X23	
4	BJFUSION1924RP	BASE JAY FUSION RPROF 19X24	
4	BJFUSION2015RP	BASE JAY FUSION RPROF 20X15	
4	BJFUSION2016RP	BASE JAY FUSION RPROF 20X16	
4	BJFUSION2017RP	BASE JAY FUSION RPROF 20X17	
4	BJFUSION2018RP	BASE JAY FUSION RPROF 20X18	
4	BJFUSION2019RP	BASE JAY FUSION RPROF 20X19	
4	BJFUSION2020RP	BASE JAY FUSION RPROF 20X20	
4	BJFUSION2021RP	BASE JAY FUSION RPROF 20X21	
4	BJFUSION2022RP	BASE JAY FUSION RPROF 20X22	
4	BJFUSION2023RP	BASE JAY FUSION RPROF 20X23	
4	BJFUSION2024RP	BASE JAY FUSION RPROF 20X24	
4	BJFUSION2117RP	BASE JAY FUSION RPROF 21X17	
4	BJFUSION2118RP	BASE JAY FUSION RPROF 21X18	
4	BJFUSION2119RP	BASE JAY FUSION RPROF 21X19	
4	BJFUSION2120RP	BASE JAY FUSION RPROF 21X20	
4	BJFUSION2121RP	BASE JAY FUSION RPROF 21X21	
4	BJFUSION2122RP	BASE JAY FUSION RPROF 21X22	
4	BJFUSION2123RP	BASE JAY FUSION RPROF 21X23	
4	BJFUSION2124RP	BASE JAY FUSION RPROF 21X24	
4	BJFUSION2217RP	BASE JAY FUSION RPROF 22X17	
4	BJFUSION2218RP	BASE JAY FUSION RPROF 22X18	
4	BJFUSION2219RP	BASE JAY FUSION RPROF 22X19	
4	BJFUSION2220RP	BASE JAY FUSION RPROF 22X20	
4	BJFUSION2221RP	BASE JAY FUSION RPROF 22X21	
4	BJFUSION2222RP	BASE JAY FUSION RPROF 22X22	
4	BJFUSION2223RP	BASE JAY FUSION RPROF 22X23	
4	BJFUSION2224RP	BASE JAY FUSION RPROF 22X24	
4	BJFUSION2319RP	BASE JAY FUSION RPROF 23X19	
4	BJFUSION2320RP	BASE JAY FUSION RPROF 23X20	
4	BJFUSION2321RP	BASE JAY FUSION RPROF 23X21	
4	BJFUSION2322RP	BASE JAY FUSION RPROF 23X22	
4	BJFUSION2323RP	BASE JAY FUSION RPROF 23X23	
4	BJFUSION2324RP	BASE JAY FUSION RPROF 23X24	
4	BJFUSION2419RP	BASE JAY FUSION RPROF 24X19	
4	BJFUSION2420RP	BASE JAY FUSION RPROF 24X20	
4	BJFUSION2421RP	BASE JAY FUSION RPROF 24X21	
4	BJFUSION2422RP	BASE JAY FUSION RPROF 24X22	
4	BJFUSION2423RP	BASE JAY FUSION RPROF 24X23	
4	BJFUSION2424	BASE JFUSION2424	
5A	JBALCOP-ALH	OBLIQUITY PAD LH PLA A	Left- 14 & 15 wide
5A	JBALCOP-ARH	OBLIQUITY PAD RH PLA A	Right- 14 & 15 wide
5B	JBALCOP-BLH	OBLIQUITY PAD LH PLA B	Left- 16 thru 21 wide
5B	JBALCOP-BRH	OBLIQUITY PAD RH PLA B	Right- 16 thru 21 wide
5C	JBALCOP-CLH	OBLIQUITY PAD LH PLA C	Left- 22 thru 24 wide
5C	JBALCOP-CRH	OBLIQUITY PAD RH PLA C	Right- 22 thru 24 wide
5D	JBALCOP-DLH	OBLIQUITY PAD LH PLA D	Left
5D	JBALCOP-DRH	OBLIQUITY PAD RH PLA D	Right

Note: for part number information, please see attachment in informations panel.

Note: Cushion widths 14" & 15" use A size Air Bladders, 16" - 20" use B size bladders and 21"-24" use C size bladders



Pos.	Item Number	Description	Remarks
1	CJFUSION1414PA	COV OUT FUSION POS AIR 14X14	
1	CJFUSION1415PA	COV OUT FUSION POS AIR 14X15	
1	CJFUSION1416PA	COV OUT FUSION POS AIR 14X16	
1	CJFUSION1417PA	COV OUT FUSION POS AIR 14X17	
1	CJFUSION1418PA	COV OUT FUSION POS AIR 14X18	
1	CJFUSION1514PA	COV OUT FUSION POS AIR 15X14	
1	CJFUSION1515PA	COV OUT FUSION POS AIR 15X15	
1	CJFUSION1516PA	COV OUT FUSION POS AIR 15X16	
1	CJFUSION1517PA	COV OUT FUSION POS AIR 15X17	
1	CJFUSION1518PA	COV OUT FUSION POS AIR 15X18	
1	CJFUSION1519PA	COV OUT FUSION POS AIR 15X19	

Pos.	Item Number	Description	Remarks
1	CJFUSION1520PA	COV OUT FUSION POS AIR 15X19	
1	CJFUSION1521PA	COV OUT FUSION POS AIR 15X21	
1	CJFUSION1614PA	COV OUT FUSION POS AIR 16X14	
1	CJFUSION1615PA	COV OUT FUSION POS AIR 16X15	
1	CJFUSION1616PA	COV OUT FUSION POS AIR 16X16	
1	CJFUSION1617PA	COV OUT FUSION POS AIR 16X17	
1	CJFUSION1618PA	COV OUT FUSION POS AIR 16X18	
1	CJFUSION1619PA	COV OUT FUSION POS AIR 16X19	
1	CJFUSION1620PA	COV OUT FUSION POS AIR 16X20	
1	CJFUSION1621PA	COV OUT FUSION POS AIR 16X21	
1	CJFUSION1622PA	COV OUT FUSION POS AIR 16X22	
1	CJFUSION1714PA	COV OUT FUSION POS AIR 17X14	
1	CJFUSION1715PA	COV OUT FUSION POS AIR 17X15	
1	CJFUSION1716PA	COV OUT FUSION POS AIR 17X16	
1	CJFUSION1717PA	COV OUT FUSION POS AIR 17X17	
1	CJFUSION1718PA	COV OUT FUSION POS AIR 17X18	
1	CJFUSION1719PA	COV OUT FUSION POS AIR 17X19	
1	CJFUSION1720PA	COV OUT FUSION POS AIR 17X20	
1	CJFUSION1721PA	COV OUT FUSION POS AIR 17X21	
1	CJFUSION1722PA	COV OUT FUSION POS AIR 17X22	
1	CJFUSION1723PA	COV OUT FUSION POS AIR 17X23	
1	CJFUSION1814PA	COV OUT FUSION POS AIR 18X14	
1	CJFUSION1815PA	COV OUT FUSION POS AIR 18X15	
1	CJFUSION1816PA	COV OUT FUSION POS AIR 18X16	
1	CJFUSION1817PA	COV OUT FUSION POS AIR 18X17	
1	CJFUSION1818PA	COV OUT FUSION POS AIR 18X18	
1	CJFUSION1819PA	COV OUT FUSION POS AIR 18X19	
1	CJFUSION1820PA	COV OUT FUSION POS AIR 18X20	
1	CJFUSION1821PA	COV OUT FUSION POS AIR 18X21	
1	CJFUSION1822PA	COV OUT FUSION POS AIR 18X22	
1	CJFUSION1823PA	COV OUT FUSION POS AIR 18X23	
1	CJFUSION1824PA	COV OUT FUSION POS AIR 18X24	
1	CJFUSION1915PA	COV OUT FUSION POS AIR 19X15	
1	CJFUSION1916PA	COV OUT FUSION POS AIR 19X16	
1	CJFUSION1917PA	COV OUT FUSION POS AIR 19X17	
1	CJFUSION1918PA	COV OUT FUSION POS AIR 19X18	
1	CJFUSION1919PA	COV OUT FUSION POS AIR 19X19	
1	CJFUSION1920PA	COV OUT FUSION POS AIR 19X20	
1	CJFUSION1921PA	COV OUT FUSION POS AIR 19X21	
1	CJFUSION1922PA	COV OUT FUSION POS AIR 19X22	
1	CJFUSION1923PA	COV OUT FUSION POS AIR 19X23	
1	CJFUSION1924PA	COV OUT FUSION POS AIR 19X24	
1	CJFUSION2015PA	COV OUT FUSION POS AIR 20X15	
1	CJFUSION2016PA	COV OUT FUSION POS AIR 20X16	
1	CJFUSION2017PA	COV OUT FUSION POS AIR 20X17	
1	CJFUSION2018PA	COV OUT FUSION POS AIR 20X18	
1	CJFUSION2019PA	COV OUT FUSION POS AIR 20X19	
1	CJFUSION2020PA	COV OUT FUSION POS AIR 20X20	
1	CJFUSION2021PA	COV OUT FUSION POS AIR 20X21	
1	CJFUSION2022PA	COV OUT FUSION POS AIR 20X22	
1	CJFUSION2023PA	COV OUT FUSION POS AIR 20X23	
1	CJFUSION2024PA	COV OUT FUSION POS AIR 20X24	
1	CJFUSION2117PA	COV OUT FUSION POS AIR 21X17	
1	CJFUSION2118PA	COV OUT FUSION POS AIR 21X18	
1	CJFUSION2119PA	COV OUT FUSION POS AIR 21X19	
1	CJFUSION2120PA	COV OUT FUSION POS AIR 21X20	
1	CJFUSION2121PA	COV OUT FUSION POS AIR 21X21	

Pos.	Item Number	Description	Remarks
1	CJFUSION2122PA	COV OUT FUSION POS AIR 21X22	
1	CJFUSION2123PA	COV OUT FUSION POS AIR 21X23	
1	CJFUSION2124PA	COV OUT FUSION POS AIR 21X24	
1	CJFUSION2217PA	COV OUT FUSION POS AIR 22X17	
1	CJFUSION2218PA	COV OUT FUSION POS AIR 22X18	
1	CJFUSION2219PA	COV OUT FUSION POS AIR 22X19	
1	CJFUSION2220PA	COV OUT FUSION POS AIR 22X20	
1	CJFUSION2221PA	COV OUT FUSION POS AIR 22X21	
1	CJFUSION2222PA	COV OUT FUSION POS AIR 22X22	
1	CJFUSION2223PA	COV OUT FUSION POS AIR 22X23	
1	CJFUSION2224PA	COV OUT FUSION POS AIR 22X24	
1	CJFUSION2319PA	COV OUT FUSION POS AIR 23X19	
1	CJFUSION2320PA	COV OUT FUSION POS AIR 23X20	
1	CJFUSION2321PA	COV OUT FUSION POS AIR 23X21	
1	CJFUSION2322PA	COV OUT FUSION POS AIR 23X22	
1	CJFUSION2323PA	COV OUT FUSION POS AIR 23X23	
1	CJFUSION2324PA	COV OUT FUSION POS AIR 23X24	
1	CJFUSION2419PA	COV OUT FUSION POS AIR 24X19	
1	CJFUSION2420PA	COV OUT FUSION POS AIR 24X20	
1	CJFUSION2421PA	COV OUT FUSION POS AIR 24X21	
1	CJFUSION2422PA	COV OUT FUSION POS AIR 24X22	
1	CJFUSION2423PA	COV OUT FUSION POS AIR 24X23	
1	CJFUSION2424PA	COV OUT FUSION POS AIR 24X24	
2	ASAJFUSION	AIR INSERT SINGLE VALVE SIZE A	Used in 14"&15" wide
2	ASBJFUSION	AIR INSERT SINGLE VALVE SIZE B	Used in 16"-20" wide
2	ASCJFUSION	AIR INSERT SINGLE VALVE SIZE C	Used in 21"-24" wide
2	ADAJFUSION	AIR INSERT DUAL VALVE SIZE A	Used in 14"&15" wide
2	ADBJFUSION	AIR INSERT DUAL VALVE SIZE B	Used in 16"-20" wide
2	ADCJFUSION	AIR INSERT DUAL VALVE SIZE C	Used in 21"-24" wide
3	ICJFUSION1414PA	INN COV FUSION POS AIR 14X14	
3	ICJFUSION1415PA	INN COV FUSION POS AIR 14X15	
3	ICJFUSION1416PA	INN COV FUSION POS AIR 14X16	
3	ICJFUSION1417PA	INN COV FUSION POS AIR 14X17	
3	ICJFUSION1418PA	INN COV FUSION POS AIR 14X18	
3	ICJFUSION1514PA	INN COV FUSION POS AIR 15X14	
3	ICJFUSION1515PA	INN COV FUSION POS AIR 15X15	
3	ICJFUSION1516PA	INN COV FUSION POS AIR 15X16	
3	ICJFUSION1517PA	INN COV FUSION POS AIR 15X17	
3	ICJFUSION1518PA	INN COV FUSION POS AIR 15X18	
3	ICJFUSION1519PA	INN COV FUSION POS AIR 15X19	
3	ICJFUSION1520PA	INN COV FUSION POS AIR 15X19	
3	ICJFUSION1521PA	INN COV FUSION POS AIR 15X21	
3	ICJFUSION1614PA	INN COV FUSION POS AIR 16X14	
3	ICJFUSION1615PA	INN COV FUSION POS AIR 16X15	
3	ICJFUSION1616PA	INN COV FUSION POS AIR 16X16	
3	ICJFUSION1617PA	INN COV FUSION POS AIR 16X17	
3	ICJFUSION1618PA	INN COV FUSION POS AIR 16X18	
3	ICJFUSION1619PA	INN COV FUSION POS AIR 16X19	
3	ICJFUSION1620PA	INN COV FUSION POS AIR 16X20	
3	ICJFUSION1621PA	INN COV FUSION POS AIR 16X21	
3	ICJFUSION1622PA	INN COV FUSION POS AIR 16X22	
3	ICJFUSION1714PA	INN COV FUSION POS AIR 17X14	
3	ICJFUSION1715PA	INN COV FUSION POS AIR 17X15	
3	ICJFUSION1716PA	INN COV FUSION POS AIR 17X16	
3	ICJFUSION1717PA	INN COV FUSION POS AIR 17X17	
3	ICJFUSION1718PA	INN COV FUSION POS AIR 17X18	
3	ICJFUSION1719PA	INN COV FUSION POS AIR 17X19	

Pos.	Item Number	Description	Remarks
3	ICJFUSION1720PA	INN COV FUSION POS AIR 17X20	
3	ICJFUSION1721PA	INN COV FUSION POS AIR 17X21	
3	ICJFUSION1722PA	INN COV FUSION POS AIR 17X22	
3	ICJFUSION1723PA	INN COV FUSION POS AIR 17X23	
3	ICJFUSION1814PA	INN COV FUSION POS AIR 18X14	
3	ICJFUSION1815PA	INN COV FUSION POS AIR 18X15	
3	ICJFUSION1816PA	INN COV FUSION POS AIR 18X16	
3	ICJFUSION1817PA	INN COV FUSION POS AIR 18X17	
3	ICJFUSION1818PA	INN COV FUSION POS AIR 18X18	
3	ICJFUSION1819PA	INN COV FUSION POS AIR 18X19	
3	ICJFUSION1820PA	INN COV FUSION POS AIR 18X20	
3	ICJFUSION1821PA	INN COV FUSION POS AIR 18X21	
3	ICJFUSION1822PA	INN COV FUSION POS AIR 18X22	
3	ICJFUSION1823PA	INN COV FUSION POS AIR 18X23	
3	ICJFUSION1824PA	INN COV FUSION POS AIR 18X24	
3	ICJFUSION1915PA	INN COV FUSION POS AIR 19X15	
3	ICJFUSION1916PA	INN COV FUSION POS AIR 19X16	
3	ICJFUSION1917PA	INN COV FUSION POS AIR 19X17	
3	ICJFUSION1918PA	INN COV FUSION POS AIR 19X18	
3	ICJFUSION1919PA	INN COV FUSION POS AIR 19X19	
3	ICJFUSION1920PA	INN COV FUSION POS AIR 19X20	
3	ICJFUSION1921PA	INN COV FUSION POS AIR 19X21	
3	ICJFUSION1922PA	INN COV FUSION POS AIR 19X22	
3	ICJFUSION1923PA	INN COV FUSION POS AIR 19X23	
3	ICJFUSION1924PA	INN COV FUSION POS AIR 19X24	
3	ICJFUSION2015PA	INN COV FUSION POS AIR 20X15	
3	ICJFUSION2016PA	INN COV FUSION POS AIR 20X16	
3	ICJFUSION2017PA	INN COV FUSION POS AIR 20X17	
3	ICJFUSION2018PA	INN COV FUSION POS AIR 20X18	
3	ICJFUSION2019PA	INN COV FUSION POS AIR 20X19	
3	ICJFUSION2020PA	INN COV FUSION POS AIR 20X20	
3	ICJFUSION2021PA	INN COV FUSION POS AIR 20X21	
3	ICJFUSION2022PA	INN COV FUSION POS AIR 20X22	
3	ICJFUSION2023PA	INN COV FUSION POS AIR 20X23	
3	ICJFUSION2024PA	INN COV FUSION POS AIR 20X24	
3	ICJFUSION2117PA	INN COV FUSION POS AIR 21X17	
3	ICJFUSION2118PA	INN COV FUSION POS AIR 21X18	
3	ICJFUSION2119PA	INN COV FUSION POS AIR 21X19	
3	ICJFUSION2120PA	INN COV FUSION POS AIR 21X20	
3	ICJFUSION2121PA	INN COV FUSION POS AIR 21X21	
3	ICJFUSION2122PA	INN COV FUSION POS AIR 21X22	
3	ICJFUSION2123PA	INN COV FUSION POS AIR 21X23	
3	ICJFUSION2124PA	INN COV FUSION POS AIR 21X24	
3	ICJFUSION2217PA	INN COV FUSION POS AIR 22X17	
3	ICJFUSION2218PA	INN COV FUSION POS AIR 22X18	
3	ICJFUSION2219PA	INN COV FUSION POS AIR 22X19	
3	ICJFUSION2220PA	INN COV FUSION POS AIR 22X20	
3	ICJFUSION2221PA	INN COV FUSION POS AIR 22X21	
3	ICJFUSION2222PA	INN COV FUSION POS AIR 22X22	
3	ICJFUSION2223PA	INN COV FUSION POS AIR 22X23	
3	ICJFUSION2224PA	INN COV FUSION POS AIR 22X24	
3	ICJFUSION2319PA	INN COV FUSION POS AIR 23X19	
3	ICJFUSION2320PA	INN COV FUSION POS AIR 23X20	
3	ICJFUSION2321PA	INN COV FUSION POS AIR 23X21	
3	ICJFUSION2322PA	INN COV FUSION POS AIR 23X22	
3	ICJFUSION2323PA	INN COV FUSION POS AIR 23X23	
3	ICJFUSION2324PA	INN COV FUSION POS AIR 23X24	

Pos.	Item Number	Description	Remarks
3	ICJFUSION2419PA	INN COV FUSION POS AIR 24X19	
3	ICJFUSION2420PA	INN COV FUSION POS AIR 24X20	
3	ICJFUSION2421PA	INN COV FUSION POS AIR 24X21	
3	ICJFUSION2422PA	INN COV FUSION POS AIR 24X22	
3	ICJFUSION2423PA	INN COV FUSION POS AIR 24X23	
3	ICJFUSION2424PA	INN COV FUSION POS AIR 24X24	
4	JBALCPKIT	JAY BALANCE CUSH POS KIT	
5	BJFUSION1414	BASE JFUSION1414	
5	BJFUSION1415	BASE JFUSION1415	
5	BJFUSION1416	BASE JFUSION1416	
5	BJFUSION1417	BASE JFUSION1417	
5	BJFUSION1418	BASE JFUSION1418	
5	BJFUSION1514	BASE JFUSION1514	
5	BJFUSION1515	BASE JFUSION1515	
5	BJFUSION1516	BASE JFUSION1516	
5	BJFUSION1517	BASE JFUSION1517	
5	BJFUSION1518	BASE JFUSION1518	
5	BJFUSION1519	BASE JFUSION1519	
5	BJFUSION1520	BASE JFUSION1520	
5	BJFUSION1521	BASE JFUSION1521	
5	BJFUSION1614	BASE JFUSION1614	
5	BJFUSION1615	BASE JFUSION1615	
5	BJFUSION1616	BASE JFUSION1616	
5	BJFUSION1617	BASE JFUSION1617	
5	BJFUSION1618	BASE JFUSION1618	
5	BJFUSION1619	BASE JFUSION1619	
5	BJFUSION1620	BASE JFUSION1620	
5	BJFUSION1621	BASE JFUSION1621	
5	BJFUSION1622	BASE JFUSION1622	
5	BJFUSION1714	BASE JFUSION1714	
5	BJFUSION1715	BASE JFUSION1715	
5	BJFUSION1716	BASE JFUSION1716	
5	BJFUSION1717	BASE JFUSION1717	
5	BJFUSION1718	BASE JFUSION1718	
5	BJFUSION1719	BASE JFUSION1719	
5	BJFUSION1720	BASE JFUSION1720	
5	BJFUSION1721	BASE JFUSION1721	
5	BJFUSION1722	BASE JFUSION1722	
5	BJFUSION1723	BASE JFUSION1723	
5	BJFUSION1814	BASE JFUSION1814	
5	BJFUSION1815	BASE JFUSION1815	
5	BJFUSION1816	BASE JFUSION1816	
5	BJFUSION1817	BASE JFUSION1817	
5	BJFUSION1818	BASE JFUSION1818	
5	BJFUSION1819	BASE JFUSION1819	
5	BJFUSION1820	BASE JFUSION1820	
5	BJFUSION1821	BASE JFUSION1821	
5	BJFUSION1822	BASE JFUSION1822	
5	BJFUSION1823	BASE JFUSION1823	
5	BJFUSION1824	BASE JFUSION1824	
5	BJFUSION1915	BASE JFUSION1915	
5	BJFUSION1916	BASE JFUSION1916	
5	BJFUSION1917	BASE JFUSION1917	
5	BJFUSION1918	BASE JFUSION1918	
5	BJFUSION1919	BASE JFUSION1919	
5	BJFUSION1920	BASE JFUSION1920	
5	BJFUSION1921	BASE JFUSION1921	

Pos.	Item Number	Description	Remarks
5	BJFUSION1922	BASE JFUSION1922	
5	BJFUSION1923	BASE JFUSION1923	
5	BJFUSION1924	BASE JFUSION1924	
5	BJFUSION2015	BASE JFUSION2015	
5	BJFUSION2016	BASE JFUSION2016	
5	BJFUSION2017	BASE JFUSION2017	
5	BJFUSION2018	BASE JFUSION2018	
5	BJFUSION2019	BASE JFUSION2019	
5	BJFUSION2020	BASE JFUSION2020	
5	BJFUSION2021	BASE JFUSION2021	
5	BJFUSION2022	BASE JFUSION2022	
5	BJFUSION2023	BASE JFUSION2023	
5	BJFUSION2024	BASE JFUSION2024	
5	BJFUSION2117	BASE JFUSION2117	
5	BJFUSION2118	BASE JFUSION2118	
5	BJFUSION2119	BASE JFUSION2119	
5	BJFUSION2120	BASE JFUSION2120	
5	BJFUSION2121	BASE JFUSION2121	
5	BJFUSION2122	BASE JFUSION2122	
5	BJFUSION2123	BASE JFUSION2123	
5	BJFUSION2124	BASE JFUSION2124	
5	BJFUSION2217	BASE JFUSION2217	
5	BJFUSION2218	BASE JFUSION2218	
5	BJFUSION2219	BASE JFUSION2219	
5	BJFUSION2220	BASE JFUSION2220	
5	BJFUSION2221	BASE JFUSION2221	
5	BJFUSION2222	BASE JFUSION2222	
5	BJFUSION2223	BASE JFUSION2223	
5	BJFUSION2224	BASE JFUSION2224	
5	BJFUSION2319	BASE JFUSION2319	
5	BJFUSION2320	BASE JFUSION2320	
5	BJFUSION2321	BASE JFUSION2321	
5	BJFUSION2322	BASE JFUSION2322	
5	BJFUSION2323	BASE JFUSION2323	
5	BJFUSION2324	BASE JFUSION2324	
5	BJFUSION2419	BASE JFUSION2419	
5	BJFUSION2420	BASE JFUSION2420	
5	BJFUSION2421	BASE JFUSION2421	
5	BJFUSION2422	BASE JFUSION2422	
5	BJFUSION2423	BASE JFUSION2423	
5	BJFUSION2424	BASE JFUSION2424	
6	AIRPUMP	AIR PUMP	
7	PATCHKIT	AIR PATCH KIT	

Note: for part number information, please see attachment in informations panel.

Note: Cushion widths 14" & 15" use A size Fluid Bladders, 16" - 20" use B size bladders and 21"-24" use C size bladders.



Pos.	Item Number	Description	Remarks
1	CJFUSION1414P	COV OUT FUSION POS FLD 14X14	
1	CJFUSION1415P	COV OUT FUSION POS FLD 14X15	
1	CJFUSION1416P	COV OUT FUSION POS FLD 14X16	
1	CJFUSION1417P	COV OUT FUSION POS FLD 14X17	
1	CJFUSION1418P	COV OUT FUSION POS FLD 14X18	
1	CJFUSION1514P	COV OUT FUSION POS FLD 15X14	
1	CJFUSION1515P	COV OUT FUSION POS FLD 15X15	
1	CJFUSION1516P	COV OUT FUSION POS FLD 15X16	
1	CJFUSION1517P	COV OUT FUSION POS FLD 15X17	
1	CJFUSION1518P	COV OUT FUSION POS FLD 15X18	
1	CJFUSION1519P	COV OUT FUSION POS FLD 15X19	
1	CJFUSION1520P	COV OUT FUSION POS FLD 15X19	
1	CJFUSION1521P	COV OUT FUSION POS FLD 15X21	
1	CJFUSION1614P	COV OUT FUSION POS FLD 16X14	
1	CJFUSION1615P	COV OUT FUSION POS FLD 16X15	
1	CJFUSION1616P	COV OUT FUSION POS FLD 16X16	
1	CJFUSION1617P	COV OUT FUSION POS FLD 16X17	

Pos.	Item Number	Description	Remarks
1	CJFUSION1618P	COV OUT FUSION POS FLD 16X18	
1	CJFUSION1619P	COV OUT FUSION POS FLD 16X19	
1	CJFUSION1620P	COV OUT FUSION POS FLD 16X20	
1	CJFUSION1621P	COV OUT FUSION POS FLD 16X21	
1	CJFUSION1622P	COV OUT FUSION POS FLD 16X22	
1	CJFUSION1714P	COV OUT FUSION POS FLD 17X14	
1	CJFUSION1715P	COV OUT FUSION POS FLD 17X15	
1	CJFUSION1716P	COV OUT FUSION POS FLD 17X16	
1	CJFUSION1717P	COV OUT FUSION POS FLD 17X17	
1	CJFUSION1718P	COV OUT FUSION POS FLD 17X18	
1	CJFUSION1719P	COV OUT FUSION POS FLD 17X19	
1	CJFUSION1720P	COV OUT FUSION POS FLD 17X20	
1	CJFUSION1721P	COV OUT FUSION POS FLD 17X21	
1	CJFUSION1722P	COV OUT FUSION POS FLD 17X22	
1	CJFUSION1723P	COV OUT FUSION POS FLD 17X23	
1	CJFUSION1814P	COV OUT FUSION POS FLD 18X14	
1	CJFUSION1815P	COV OUT FUSION POS FLD 18X15	
1	CJFUSION1816P	COV OUT FUSION POS FLD 18X16	
1	CJFUSION1817P	COV OUT FUSION POS FLD 18X17	
1	CJFUSION1818P	COV OUT FUSION POS FLD 18X18	
1	CJFUSION1819P	COV OUT FUSION POS FLD 18X19	
1	CJFUSION1820P	COV OUT FUSION POS FLD 18X20	
1	CJFUSION1821P	COV OUT FUSION POS FLD 18X21	
1	CJFUSION1822P	COV OUT FUSION POS FLD 18X22	
1	CJFUSION1823P	COV OUT FUSION POS FLD 18X23	
1	CJFUSION1824P	COV OUT FUSION POS FLD 18X24	
1	CJFUSION1915P	COV OUT FUSION POS FLD 19X15	
1	CJFUSION1916P	COV OUT FUSION POS FLD 19X16	
1	CJFUSION1917P	COV OUT FUSION POS FLD 19X17	
1	CJFUSION1918P	COV OUT FUSION POS FLD 19X18	
1	CJFUSION1919P	COV OUT FUSION POS FLD 19X19	
1	CJFUSION1920P	COV OUT FUSION POS FLD 19X20	
1	CJFUSION1921P	COV OUT FUSION POS FLD 19X21	
1	CJFUSION1922P	COV OUT FUSION POS FLD 19X22	
1	CJFUSION1923P	COV OUT FUSION POS FLD 19X23	
1	CJFUSION1924P	COV OUT FUSION POS FLD 19X24	
1	CJFUSION2015P	COV OUT FUSION POS FLD 20X15	
1	CJFUSION2016P	COV OUT FUSION POS FLD 20X16	
1	CJFUSION2017P	COV OUT FUSION POS FLD 20X17	
1	CJFUSION2018P	COV OUT FUSION POS FLD 20X18	
1	CJFUSION2019P	COV OUT FUSION POS FLD 20X19	
1	CJFUSION2020P	COV OUT FUSION POS FLD 20X20	
1	CJFUSION2021P	COV OUT FUSION POS FLD 20X21	
1	CJFUSION2022P	COV OUT FUSION POS FLD 20X22	
1	CJFUSION2023P	COV OUT FUSION POS FLD 20X23	
1	CJFUSION2024P	COV OUT FUSION POS FLD 20X24	
1	CJFUSION2117P	COV OUT FUSION POS FLD 21X17	
1	CJFUSION2118P	COV OUT FUSION POS FLD 21X18	
1	CJFUSION2119P	COV OUT FUSION POS FLD 21X19	
1	CJFUSION2120P	COV OUT FUSION POS FLD 21X20	
1	CJFUSION2121P	COV OUT FUSION POS FLD 21X21	
1	CJFUSION2122P	COV OUT FUSION POS FLD 21X22	
1	CJFUSION2123P	COV OUT FUSION POS FLD 21X23	
1	CJFUSION2124P	COV OUT FUSION POS FLD 21X24	
1	CJFUSION2217P	COV OUT FUSION POS FLD 22X17	
1	CJFUSION2218P	COV OUT FUSION POS FLD 22X18	
1	CJFUSION2219P	COV OUT FUSION POS FLD 22X19	

Pos.	Item Number	Description	Remarks
1	CJFUSION2220P	COV OUT FUSION POS FLD 22X20	
1	CJFUSION2221P	COV OUT FUSION POS FLD 22X21	
1	CJFUSION2222P	COV OUT FUSION POS FLD 22X22	
1	CJFUSION2223P	COV OUT FUSION POS FLD 22X23	
1	CJFUSION2224P	COV OUT FUSION POS FLD 22X24	
1	CJFUSION2319P	COV OUT FUSION POS FLD 23X19	
1	CJFUSION2320P	COV OUT FUSION POS FLD 23X20	
1	CJFUSION2321P	COV OUT FUSION POS FLD 23X21	
1	CJFUSION2322P	COV OUT FUSION POS FLD 23X22	
1	CJFUSION2323P	COV OUT FUSION POS FLD 23X23	
1	CJFUSION2324P	COV OUT FUSION POS FLD 23X24	
1	CJFUSION2419P	COV OUT FUSION POS FLD 24X19	
1	CJFUSION2420P	COV OUT FUSION POS FLD 24X20	
1	CJFUSION2421P	COV OUT FUSION POS FLD 24X21	
1	CJFUSION2422P	COV OUT FUSION POS FLD 24X22	
1	CJFUSION2423P	COV OUT FUSION POS FLD 24X23	
1	CJFUSION2424P	COV OUT FUSION POS FLD 24X24	
2	FAJFUSION	FLUID INSERT SIZE A JFUSION	Used in 14"&15" wide
2	FBJFUSION	FLUID INSERT SIZE B JFUSION	Used in 16"-20" wide
2	FCJFUSION	FLUID INSERT SIZE C JFUSION	Used in 21"-24" wide
3	JBALCPKIT	JAY BALANCE CUSH POS KIT	5 pieces
4	ICJFUSION1414P	INN COV FUSION POS FLD 14X14	
4	ICJFUSION1415P	INN COV FUSION POS FLD 14X15	
4	ICJFUSION1416P	INN COV FUSION POS FLD 14X16	
4	ICJFUSION1417P	INN COV FUSION POS FLD 14X17	
4	ICJFUSION1418P	INN COV FUSION POS FLD 14X18	
4	ICJFUSION1514P	INN COV FUSION POS FLD 15X14	
4	ICJFUSION1515P	INN COV FUSION POS FLD 15X15	
4	ICJFUSION1516P	INN COV FUSION POS FLD 15X16	
4	ICJFUSION1517P	INN COV FUSION POS FLD 15X17	
4	ICJFUSION1518P	INN COV FUSION POS FLD 15X18	
4	ICJFUSION1519P	INN COV FUSION POS FLD 15X19	
4	ICJFUSION1520P	INN COV FUSION POS FLD 15X19	
4	ICJFUSION1521P	INN COV FUSION POS FLD 15X21	
4	ICJFUSION1614P	INN COV FUSION POS FLD 16X14	
4	ICJFUSION1615P	INN COV FUSION POS FLD 16X15	
4	ICJFUSION1616P	INN COV FUSION POS FLD 16X16	
4	ICJFUSION1617P	INN COV FUSION POS FLD 16X17	
4	ICJFUSION1618P	INN COV FUSION POS FLD 16X18	
4	ICJFUSION1619P	INN COV FUSION POS FLD 16X19	
4	ICJFUSION1620P	INN COV FUSION POS FLD 16X20	
4	ICJFUSION1621P	INN COV FUSION POS FLD 16X21	
4	ICJFUSION1622P	INN COV FUSION POS FLD 16X22	
4	ICJFUSION1714P	INN COV FUSION POS FLD 17X14	
4	ICJFUSION1715P	INN COV FUSION POS FLD 17X15	
4	ICJFUSION1716P	INN COV FUSION POS FLD 17X16	
4	ICJFUSION1717P	INN COV FUSION POS FLD 17X17	
4	ICJFUSION1718P	INN COV FUSION POS FLD 17X18	
4	ICJFUSION1719P	INN COV FUSION POS FLD 17X19	
4	ICJFUSION1720P	INN COV FUSION POS FLD 17X20	
4	ICJFUSION1721P	INN COV FUSION POS FLD 17X21	
4	ICJFUSION1722P	INN COV FUSION POS FLD 17X22	
4	ICJFUSION1723P	INN COV FUSION POS FLD 17X23	
4	ICJFUSION1814P	INN COV FUSION POS FLD 18X14	
4	ICJFUSION1815P	INN COV FUSION POS FLD 18X15	
4	ICJFUSION1816P	INN COV FUSION POS FLD 18X16	
4	ICJFUSION1817P	INN COV FUSION POS FLD 18X17	

Pos.	Item Number	Description	Remarks
4	ICJFUSION1818P	INN COV FUSION POS FLD 18X18	
4	ICJFUSION1819P	INN COV FUSION POS FLD 18X19	
4	ICJFUSION1820P	INN COV FUSION POS FLD 18X20	
4	ICJFUSION1821P	INN COV FUSION POS FLD 18X21	
4	ICJFUSION1822P	INN COV FUSION POS FLD 18X22	
4	ICJFUSION1823P	INN COV FUSION POS FLD 18X23	
4	ICJFUSION1824P	INN COV FUSION POS FLD 18X24	
4	ICJFUSION1915P	INN COV FUSION POS FLD 19X15	
4	ICJFUSION1916P	INN COV FUSION POS FLD 19X16	
4	ICJFUSION1917P	INN COV FUSION POS FLD 19X17	
4	ICJFUSION1918P	INN COV FUSION POS FLD 19X18	
4	ICJFUSION1919P	INN COV FUSION POS FLD 19X19	
4	ICJFUSION1920P	INN COV FUSION POS FLD 19X20	
4	ICJFUSION1921P	INN COV FUSION POS FLD 19X21	
4	ICJFUSION1922P	INN COV FUSION POS FLD 19X22	
4	ICJFUSION1923P	INN COV FUSION POS FLD 19X23	
4	ICJFUSION1924P	INN COV FUSION POS FLD 19X24	
4	ICJFUSION2015P	INN COV FUSION POS FLD 20X15	
4	ICJFUSION2016P	INN COV FUSION POS FLD 20X16	
4	ICJFUSION2017P	INN COV FUSION POS FLD 20X17	
4	ICJFUSION2018P	INN COV FUSION POS FLD 20X18	
4	ICJFUSION2019P	INN COV FUSION POS FLD 20X19	
4	ICJFUSION2020P	INN COV FUSION POS FLD 20X20	
4	ICJFUSION2021P	INN COV FUSION POS FLD 20X21	
4	ICJFUSION2022P	INN COV FUSION POS FLD 20X22	
4	ICJFUSION2023P	INN COV FUSION POS FLD 20X23	
4	ICJFUSION2024P	INN COV FUSION POS FLD 20X24	
4	ICJFUSION2117P	INN COV FUSION POS FLD 21X17	
4	ICJFUSION2118P	INN COV FUSION POS FLD 21X18	
4	ICJFUSION2119P	INN COV FUSION POS FLD 21X19	
4	ICJFUSION2120P	INN COV FUSION POS FLD 21X20	
4	ICJFUSION2121P	INN COV FUSION POS FLD 21X21	
4	ICJFUSION2122P	INN COV FUSION POS FLD 21X22	
4	ICJFUSION2123P	INN COV FUSION POS FLD 21X23	
4	ICJFUSION2124P	INN COV FUSION POS FLD 21X24	
4	ICJFUSION2217P	INN COV FUSION POS FLD 22X17	
4	ICJFUSION2218P	INN COV FUSION POS FLD 22X18	
4	ICJFUSION2219P	INN COV FUSION POS FLD 22X19	
4	ICJFUSION2220P	INN COV FUSION POS FLD 22X20	
4	ICJFUSION2221P	INN COV FUSION POS FLD 22X21	
4	ICJFUSION2222P	INN COV FUSION POS FLD 22X22	
4	ICJFUSION2223P	INN COV FUSION POS FLD 22X23	
4	ICJFUSION2224P	INN COV FUSION POS FLD 22X24	
4	ICJFUSION2319P	INN COV FUSION POS FLD 23X19	
4	ICJFUSION2320P	INN COV FUSION POS FLD 23X20	
4	ICJFUSION2321P	INN COV FUSION POS FLD 23X21	
4	ICJFUSION2322P	INN COV FUSION POS FLD 23X22	
4	ICJFUSION2323P	INN COV FUSION POS FLD 23X23	
4	ICJFUSION2324P	INN COV FUSION POS FLD 23X24	
4	ICJFUSION2419P	INN COV FUSION POS FLD 24X19	
4	ICJFUSION2420P	INN COV FUSION POS FLD 24X20	
4	ICJFUSION2421P	INN COV FUSION POS FLD 24X21	
4	ICJFUSION2422P	INN COV FUSION POS FLD 24X22	
4	ICJFUSION2423P	INN COV FUSION POS FLD 24X23	
4	ICJFUSION2424P	INN COV FUSION POS FLD 24X24	
5	BJFUSION1414	BASE JFUSION1414	
5	BJFUSION1415	BASE JFUSION1415	

Pos.	Item Number	Description	Remarks
5	BJFUSION1416	BASE JFUSION1416	
5	BJFUSION1417	BASE JFUSION1417	
5	BJFUSION1418	BASE JFUSION1418	
5	BJFUSION1514	BASE JFUSION1514	
5	BJFUSION1515	BASE JFUSION1515	
5	BJFUSION1516	BASE JFUSION1516	
5	BJFUSION1517	BASE JFUSION1517	
5	BJFUSION1518	BASE JFUSION1518	
5	BJFUSION1519	BASE JFUSION1519	
5	BJFUSION1520	BASE JFUSION1520	
5	BJFUSION1521	BASE JFUSION1521	
5	BJFUSION1614	BASE JFUSION1614	
5	BJFUSION1615	BASE JFUSION1615	
5	BJFUSION1616	BASE JFUSION1616	
5	BJFUSION1617	BASE JFUSION1617	
5	BJFUSION1618	BASE JFUSION1618	
5	BJFUSION1619	BASE JFUSION1619	
5	BJFUSION1620	BASE JFUSION1620	
5	BJFUSION1621	BASE JFUSION1621	
5	BJFUSION1622	BASE JFUSION1622	
5	BJFUSION1714	BASE JFUSION1714	
5	BJFUSION1715	BASE JFUSION1715	
5	BJFUSION1716	BASE JFUSION1716	
5	BJFUSION1717	BASE JFUSION1717	
5	BJFUSION1718	BASE JFUSION1718	
5	BJFUSION1719	BASE JFUSION1719	
5	BJFUSION1720	BASE JFUSION1720	
5	BJFUSION1721	BASE JFUSION1721	
5	BJFUSION1722	BASE JFUSION1722	
5	BJFUSION1723	BASE JFUSION1723	
5	BJFUSION1814	BASE JFUSION1814	
5	BJFUSION1815	BASE JFUSION1815	
5	BJFUSION1816	BASE JFUSION1816	
5	BJFUSION1817	BASE JFUSION1817	
5	BJFUSION1818	BASE JFUSION1818	
5	BJFUSION1819	BASE JFUSION1819	
5	BJFUSION1820	BASE JFUSION1820	
5	BJFUSION1821	BASE JFUSION1821	
5	BJFUSION1822	BASE JFUSION1822	
5	BJFUSION1823	BASE JFUSION1823	
5	BJFUSION1824	BASE JFUSION1824	
5	BJFUSION1915	BASE JFUSION1915	
5	BJFUSION1916	BASE JFUSION1916	
5	BJFUSION1917	BASE JFUSION1917	
5	BJFUSION1918	BASE JFUSION1918	
5	BJFUSION1919	BASE JFUSION1919	
5	BJFUSION1920	BASE JFUSION1920	
5	BJFUSION1921	BASE JFUSION1921	
5	BJFUSION1922	BASE JFUSION1922	
5	BJFUSION1923	BASE JFUSION1923	
5	BJFUSION1924	BASE JFUSION1924	
5	BJFUSION2015	BASE JFUSION2015	
5	BJFUSION2016	BASE JFUSION2016	
5	BJFUSION2017	BASE JFUSION2017	
5	BJFUSION2018	BASE JFUSION2018	
5	BJFUSION2019	BASE JFUSION2019	
5	BJFUSION2020	BASE JFUSION2020	

Pos.	Item Number	Description	Remarks
5	BJFUSION2021	BASE JFUSION2021	
5	BJFUSION2022	BASE JFUSION2022	
5	BJFUSION2023	BASE JFUSION2023	
5	BJFUSION2024	BASE JFUSION2024	
5	BJFUSION2117	BASE JFUSION2117	
5	BJFUSION2118	BASE JFUSION2118	
5	BJFUSION2119	BASE JFUSION2119	
5	BJFUSION2120	BASE JFUSION2120	
5	BJFUSION2121	BASE JFUSION2121	
5	BJFUSION2122	BASE JFUSION2122	
5	BJFUSION2123	BASE JFUSION2123	
5	BJFUSION2124	BASE JFUSION2124	
5	BJFUSION2217	BASE JFUSION2217	
5	BJFUSION2218	BASE JFUSION2218	
5	BJFUSION2219	BASE JFUSION2219	
5	BJFUSION2220	BASE JFUSION2220	
5	BJFUSION2221	BASE JFUSION2221	
5	BJFUSION2222	BASE JFUSION2222	
5	BJFUSION2223	BASE JFUSION2223	
5	BJFUSION2224	BASE JFUSION2224	
5	BJFUSION2319	BASE JFUSION2319	
5	BJFUSION2320	BASE JFUSION2320	
5	BJFUSION2321	BASE JFUSION2321	
5	BJFUSION2322	BASE JFUSION2322	
5	BJFUSION2323	BASE JFUSION2323	
5	BJFUSION2324	BASE JFUSION2324	
5	BJFUSION2419	BASE JFUSION2419	
5	BJFUSION2420	BASE JFUSION2420	
5	BJFUSION2421	BASE JFUSION2421	
5	BJFUSION2422	BASE JFUSION2422	
5	BJFUSION2423	BASE JFUSION2423	
5	BJFUSION2424	BASE JFUSION2424	

Note: for part number information, please see attachment in informations panel.

Note: to order a replacement JYW ROHO Air Insert for your Core Cushion, see JAY FUSION AIR parts page; for a complete Fusion Cushion with a ROHO Air Insert, see the JAY Fusion Adjustable Cushion Order Form on our website.



Pos.	Item Number	Description	Remarks
1	CJFUSION1414C	COVER OUT FUSION CORE 14X14	
1	CJFUSION1415C	COVER OUT FUSION CORE 14X15	

Pos.	Item Number	Description	Remarks
1	CJFUSION1416C	COVER OUT FUSION CORE 14X16	
1	CJFUSION1417C	COVER OUT FUSION CORE 14X17	
1	CJFUSION1418C	COVER OUT FUSION CORE 14X18	
1	CJFUSION1514C	COVER OUT FUSION CORE 15X14	
1	CJFUSION1515C	COVER OUT FUSION CORE 15X15	
1	CJFUSION1516C	COVER OUT FUSION CORE 15X16	
1	CJFUSION1517C	COVER OUT FUSION CORE 15X17	
1	CJFUSION1518C	COVER OUT FUSION CORE 15X18	
1	CJFUSION1519C	COVER OUT FUSION CORE 15X19	
1	CJFUSION1520C	COVER OUT FUSION CORE 15X20	
1	CJFUSION1521C	COVER OUT FUSION CORE 15X21	
1	CJFUSION1614C	COVER OUT FUSION CORE 16X14	
1	CJFUSION1615C	COVER OUT FUSION CORE 16X15	
1	CJFUSION1616C	COVER OUT FUSION CORE 16X16	
1	CJFUSION1617C	COVER OUT FUSION CORE 16X17	
1	CJFUSION1618C	COVER OUT FUSION CORE 16X18	
1	CJFUSION1619C	COVER OUT FUSION CORE 16X19	
1	CJFUSION1620C	COVER OUT FUSION CORE 16X20	
1	CJFUSION1621C	COVER OUT FUSION CORE 16X21	
1	CJFUSION1622C	COVER OUT FUSION CORE 16X22	
1	CJFUSION1714C	COVER OUT FUSION CORE 17X14	
1	CJFUSION1715C	COVER OUT FUSION CORE 17X15	
1	CJFUSION1716C	COVER OUT FUSION CORE 17X16	
1	CJFUSION1717C	COVER OUT FUSION CORE 17X17	
1	CJFUSION1718C	COVER OUT FUSION CORE 17X18	
1	CJFUSION1719C	COVER OUT FUSION CORE 17X19	
1	CJFUSION1720C	COVER OUT FUSION CORE 17X20	
1	CJFUSION1721C	COVER OUT FUSION CORE 17X21	
1	CJFUSION1722C	COVER OUT FUSION CORE 17X22	
1	CJFUSION1723C	COVER OUT FUSION CORE 17X23	
1	CJFUSION1814C	COVER OUT FUSION CORE 18X14	
1	CJFUSION1815C	COVER OUT FUSION CORE 18X15	
1	CJFUSION1816C	COVER OUT FUSION CORE 18X16	
1	CJFUSION1817C	COVER OUT FUSION CORE 18X17	
1	CJFUSION1818C	COVER OUT FUSION CORE 18X18	
1	CJFUSION1819C	COVER OUT FUSION CORE 18X19	
1	CJFUSION1820C	COVER OUT FUSION CORE 18X20	
1	CJFUSION1821C	COVER OUT FUSION CORE 18X21	
1	CJFUSION1822C	COVER OUT FUSION CORE 18X22	
1	CJFUSION1823C	COVER OUT FUSION CORE 18X23	
1	CJFUSION1824C	COVER OUT FUSION CORE 18X24	
1	CJFUSION1915C	COVER OUT FUSION CORE 19X15	
1	CJFUSION1916C	COVER OUT FUSION CORE 19X16	
1	CJFUSION1917C	COVER OUT FUSION CORE 19X17	
1	CJFUSION1918C	COVER OUT FUSION CORE 19X18	
1	CJFUSION1919C	COVER OUT FUSION CORE 19X19	
1	CJFUSION1920C	COVER OUT FUSION CORE 19X20	
1	CJFUSION1921C	COVER OUT FUSION CORE 19X21	
1	CJFUSION1922C	COVER OUT FUSION CORE 19X22	
1	CJFUSION1923C	COVER OUT FUSION CORE 19X23	
1	CJFUSION1924C	COVER OUT FUSION CORE 19X24	
1	CJFUSION2015C	COVER OUT FUSION CORE 20X15	
1	CJFUSION2016C	COVER OUT FUSION CORE 20X16	
1	CJFUSION2017C	COVER OUT FUSION CORE 20X17	
1	CJFUSION2018C	COVER OUT FUSION CORE 20X18	
1	CJFUSION2019C	COVER OUT FUSION CORE 20X19	
1	CJFUSION2020C	COVER OUT FUSION CORE 20X20	


Pos.	Item Number	Description	Remarks
1	CJFUSION2021C	COVER OUT FUSION CORE 20X21	
1	CJFUSION2022C	COVER OUT FUSION CORE 20X22	
1	CJFUSION2023C	COVER OUT FUSION CORE 20X23	
1	CJFUSION2024C	COVER OUT FUSION CORE 20X24	
1	CJFUSION2117C	COVER OUT FUSION CORE 21X17	
1	CJFUSION2118C	COVER OUT FUSION CORE 21X18	
1	CJFUSION2119C	COVER OUT FUSION CORE 21X19	
1	CJFUSION2120C	COVER OUT FUSION CORE 21X20	
1	CJFUSION2121C	COVER OUT FUSION CORE 21X21	
1	CJFUSION2122C	COVER OUT FUSION CORE 21X22	
1	CJFUSION2123C	COVER OUT FUSION CORE 21X23	
1	CJFUSION2124C	COVER OUT FUSION CORE 21X24	
1	CJFUSION2217C	COVER OUT FUSION CORE 22X17	
1	CJFUSION2218C	COVER OUT FUSION CORE 22X18	
1	CJFUSION2219C	COVER OUT FUSION CORE 22X19	
1	CJFUSION2220C	COVER OUT FUSION CORE 22X20	
1	CJFUSION2221C	COVER OUT FUSION CORE 22X21	
1	CJFUSION2222C	COVER OUT FUSION CORE 22X22	
1	CJFUSION2223C	COVER OUT FUSION CORE 22X23	
1	CJFUSION2224C	COVER OUT FUSION CORE 22X24	
1	CJFUSION2319C	COVER OUT FUSION CORE 23X19	
1	CJFUSION2320C	COVER OUT FUSION CORE 23X20	
1	CJFUSION2321C	COVER OUT FUSION CORE 23X21	
1	CJFUSION2322C	COVER OUT FUSION CORE 23X22	
1	CJFUSION2323C	COVER OUT FUSION CORE 23X23	
1	CJFUSION2324C	COVER OUT FUSION CORE 23X24	
1	CJFUSION2419C	COVER OUT FUSION CORE 24X19	
1	CJFUSION2420C	COVER OUT FUSION CORE 24X20	
1	CJFUSION2421C	COVER OUT FUSION CORE 24X21	
1	CJFUSION2422C	COVER OUT FUSION CORE 24X22	
1	CJFUSION2423C	COVER OUT FUSION CORE 24X23	
1	CJFUSION2424C	COVER OUT FUSION CORE 24X24	
2	FJX2-1	FL PD JAY X2 14X14 14X16	all 14 & 15 wide
2	FJX2-5	FL PD JAY X2 17X17 18X18 18X20	all 16 thru 21 wide
2	FJX2-6	FL PD JAY X2 20X20	all 22 thru 24 wide
3	BJFUSION1414	BASE JFUSION1414	
3	BJFUSION1415	BASE JFUSION1415	
3	BJFUSION1416	BASE JFUSION1416	
3	BJFUSION1417	BASE JFUSION1417	
3	BJFUSION1418	BASE JFUSION1418	
3	BJFUSION1514	BASE JFUSION1514	
3	BJFUSION1515	BASE JFUSION1515	
3	BJFUSION1516	BASE JFUSION1516	
3	BJFUSION1517	BASE JFUSION1517	
3	BJFUSION1518	BASE JFUSION1518	
3	BJFUSION1519	BASE JFUSION1519	
3	BJFUSION1520	BASE JFUSION1520	
3	BJFUSION1521	BASE JFUSION1521	
3	BJFUSION1614	BASE JFUSION1614	
3	BJFUSION1615	BASE JFUSION1615	
3	BJFUSION1616	BASE JFUSION1616	
3	BJFUSION1617	BASE JFUSION1617	
3	BJFUSION1618	BASE JFUSION1618	
3	BJFUSION1619	BASE JFUSION1619	
3	BJFUSION1620	BASE JFUSION1620	
3	BJFUSION1621	BASE JFUSION1621	
3	BJFUSION1622	BASE JFUSION1622	

Pos.	Item Number	Description	Remarks
3	BJFUSION1714	BASE JFUSION1714	
3	BJFUSION1715	BASE JFUSION1715	
3	BJFUSION1716	BASE JFUSION1716	
3	BJFUSION1717	BASE JFUSION1717	
3	BJFUSION1718	BASE JFUSION1718	
3	BJFUSION1719	BASE JFUSION1719	
3	BJFUSION1720	BASE JFUSION1720	
3	BJFUSION1721	BASE JFUSION1721	
3	BJFUSION1722	BASE JFUSION1722	
3	BJFUSION1723	BASE JFUSION1723	
3	BJFUSION1814	BASE JFUSION1814	
3	BJFUSION1815	BASE JFUSION1815	
3	BJFUSION1816	BASE JFUSION1816	
3	BJFUSION1817	BASE JFUSION1817	
3	BJFUSION1818	BASE JFUSION1818	
3	BJFUSION1819	BASE JFUSION1819	
3	BJFUSION1820	BASE JFUSION1820	
3	BJFUSION1821	BASE JFUSION1821	
3	BJFUSION1822	BASE JFUSION1822	
3	BJFUSION1823	BASE JFUSION1823	
3	BJFUSION1824	BASE JFUSION1824	
3	BJFUSION1915	BASE JFUSION1915	
3	BJFUSION1916	BASE JFUSION1916	
3	BJFUSION1917	BASE JFUSION1917	
3	BJFUSION1918	BASE JFUSION1918	
3	BJFUSION1919	BASE JFUSION1919	
3	BJFUSION1920	BASE JFUSION1920	
3	BJFUSION1921	BASE JFUSION1921	
3	BJFUSION1922	BASE JFUSION1922	
3	BJFUSION1923	BASE JFUSION1923	
3	BJFUSION1924	BASE JFUSION1924	
3	BJFUSION2015	BASE JFUSION2015	
3	BJFUSION2016	BASE JFUSION2016	
3	BJFUSION2017	BASE JFUSION2017	
3	BJFUSION2018	BASE JFUSION2018	
3	BJFUSION2019	BASE JFUSION2019	
3	BJFUSION2020	BASE JFUSION2020	
3	BJFUSION2021	BASE JFUSION2021	
3	BJFUSION2022	BASE JFUSION2022	
3	BJFUSION2023	BASE JFUSION2023	
3	BJFUSION2024	BASE JFUSION2024	
3	BJFUSION2117	BASE JFUSION2117	
3	BJFUSION2118	BASE JFUSION2118	
3	BJFUSION2119	BASE JFUSION2119	
3	BJFUSION2120	BASE JFUSION2120	
3	BJFUSION2121	BASE JFUSION2121	
3	BJFUSION2122	BASE JFUSION2122	
3	BJFUSION2123	BASE JFUSION2123	
3	BJFUSION2124	BASE JFUSION2124	
3	BJFUSION2217	BASE JFUSION2217	
3	BJFUSION2218	BASE JFUSION2218	
3	BJFUSION2219	BASE JFUSION2219	
3	BJFUSION2220	BASE JFUSION2220	
3	BJFUSION2221	BASE JFUSION2221	
3	BJFUSION2222	BASE JFUSION2222	
3	BJFUSION2223	BASE JFUSION2223	
3	BJFUSION2224	BASE JFUSION2224	

Pos.	Item Number	Description	Remarks
3	BJFUSION2319	BASE JFUSION2319	
3	BJFUSION2320	BASE JFUSION2320	
3	BJFUSION2321	BASE JFUSION2321	
3	BJFUSION2322	BASE JFUSION2322	
3	BJFUSION2323	BASE JFUSION2323	
3	BJFUSION2324	BASE JFUSION2324	
3	BJFUSION2419	BASE JFUSION2419	
3	BJFUSION2420	BASE JFUSION2420	
3	BJFUSION2421	BASE JFUSION2421	
3	BJFUSION2422	BASE JFUSION2422	
3	BJFUSION2423	BASE JFUSION2423	
3	BJFUSION2424	BASE JFUSION2424	

Note: for part number information, please see attachment in informations panel.

Note: Cushion widths 16" - 20" use B size bladders and 21"-22" use C size bladders.

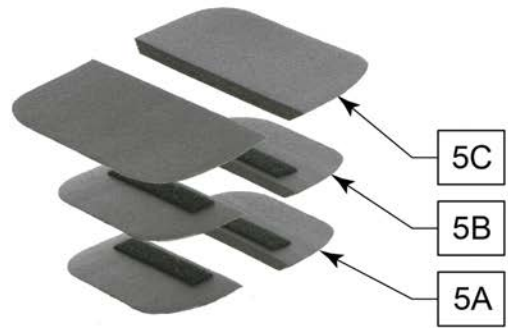

Obliquity Pad Ordering Guide

Cushion Width	1st Layer	2nd Layer
16"-20"	JBALCOP-BLH	JBALCOP-CLH
	JBALCOP-BRH	JBALCOP-CRH
21"-22"	JBALCOP-CLH	JBALCOP-DLH
	JBALCOP-CRH	JBALCOP-DRH

Note: Obliquity Pad is 1/2":
 One layer of pads = 1/2" height
 Two layers of pads = 1" height

Pos.	Item Number	Description	Remarks
1	CJFUSION1616CFH	COV FUSION CUSH CRYO HYB 16X16	
1	CJFUSION1617CFH	COV FUSION CUSH CRYO HYB 16X17	
1	CJFUSION1618CFH	COV FUSION CUSH CRYO HYB 16X18	
1	CJFUSION1619CFH	COV FUSION CUSH CRYO HYB 16X19	
1	CJFUSION1620CFH	COV FUSION CUSH CRYO HYB 16X20	
1	CJFUSION1716CFH	COV FUSION CUSH CRYO HYB 17X16	
1	CJFUSION1717CFH	COV FUSION CUSH CRYO HYB 17X17	
1	CJFUSION1718CFH	COV FUSION CUSH CRYO HYB 17X18	
1	CJFUSION1719CFH	COV FUSION CUSH CRYO HYB 17X19	
1	CJFUSION1720CFH	COV FUSION CUSH CRYO HYB 17X20	
1	CJFUSION1721CFH	COV FUSION CUSH CRYO HYB 17X21	
1	CJFUSION1816CFH	COV FUSION CUSH CRYO HYB 18X16	
1	CJFUSION1817CFH	COV FUSION CUSH CRYO HYB 18X17	
1	CJFUSION1818CFH	COV FUSION CUSH CRYO HYB 18X18	
1	CJFUSION1819CFH	COV FUSION CUSH CRYO HYB 18X19	
1	CJFUSION1820CFH	COV FUSION CUSH CRYO HYB 18X20	
1	CJFUSION1821CFH	COV FUSION CUSH CRYO HYB 18X21	
1	CJFUSION1822CFH	COV FUSION CUSH CRYO HYB 18X22	
1	CJFUSION1916CFH	COV FUSION CUSH CRYO HYB 19X16	


Pos.	Item Number	Description	Remarks
1	CJFUSION1917CFH	COV FUSION CUSH CRYO HYB 19X17	
1	CJFUSION1918CFH	COV FUSION CUSH CRYO HYB 19X18	
1	CJFUSION1919CFH	COV FUSION CUSH CRYO HYB 19X19	
1	CJFUSION1920CFH	COV FUSION CUSH CRYO HYB 19X20	
1	CJFUSION1921CFH	COV FUSION CUSH CRYO HYB 19X21	
1	CJFUSION1922CFH	COV FUSION CUSH CRYO HYB 19X22	
1	CJFUSION2016CFH	COV FUSION CUSH CRYO HYB 20X16	
1	CJFUSION2017CFH	COV FUSION CUSH CRYO HYB 20X17	
1	CJFUSION2018CFH	COV FUSION CUSH CRYO HYB 20X18	
1	CJFUSION2019CFH	COV FUSION CUSH CRYO HYB 20X19	
1	CJFUSION2020CFH	COV FUSION CUSH CRYO HYB 20X20	
1	CJFUSION2020CFH	COV FUSION CUSH CRYO HYB 20X21	
1	CJFUSION2022CFH	COV FUSION CUSH CRYO HYB 20X22	
1	CJFUSION2118CFH	COV FUSION CUSH CRYO HYB 21X18	
1	CJFUSION2119CFH	COV FUSION CUSH CRYO HYB 21X19	
1	CJFUSION2120CFH	COV FUSION CUSH CRYO HYB 21X20	
1	CJFUSION2121CFH	COV FUSION CUSH CRYO HYB 21X21	
1	CJFUSION2122CFH	COV FUSION CUSH CRYO HYB 21X22	
1	CJFUSION2218CFH	COV FUSION CUSH CRYO HYB 22X18	
1	CJFUSION2219CFH	COV FUSION CUSH CRYO HYB 22X19	
1	CJFUSION2220CFH	COV FUSION CUSH CRYO HYB 22X20	
1	CJFUSION2221CFH	COV FUSION CUSH CRYO HYB 22X21	
1	CJFUSION2222CFH	COV FUSION CUSH CRYO HYB 22X22	
2	CFHB	CRYO FLUID HYBRID BLADDER B	Used in 16"-20" wide
2	CFHC	CRYO FLUID HYBRID BLADDER C	Used in 21"-22" wide
3	ICJFUSION1616	INNER CVR JFUSION1616 FLUID	
3	ICJFUSION1617	INNER CVR JFUSION1617 FLUID	
3	ICJFUSION1618	INNER CVR JFUSION1618 FLUID	
3	ICJFUSION1619	INNER CVR JFUSION1619 FLUID	
3	ICJFUSION1620	INNER CVR JFUSION1620 FLUID	
3	ICJFUSION1716	INNER CVR JFUSION1716 FLUID	
3	ICJFUSION1717	INNER CVR JFUSION1717 FLUID	
3	ICJFUSION1718	INNER CVR JFUSION1718 FLUID	
3	ICJFUSION1719	INNER CVR JFUSION1719 FLUID	
3	ICJFUSION1720	INNER CVR JFUSION1720 FLUID	
3	ICJFUSION1721	INNER CVR JFUSION1721 FLUID	
3	ICJFUSION1816	INNER CVR JFUSION1816 FLUID	
3	ICJFUSION1817	INNER CVR JFUSION1817 FLUID	
3	ICJFUSION1818	INNER CVR JFUSION1818 FLUID	
3	ICJFUSION1819	INNER CVR JFUSION1819 FLUID	
3	ICJFUSION1820	INNER CVR JFUSION1820 FLUID	
3	ICJFUSION1821	INNER CVR JFUSION1821 FLUID	
3	ICJFUSION1822	INNER CVR JFUSION1822 FLUID	
3	ICJFUSION1916	INNER CVR JFUSION1916 FLUID	
3	ICJFUSION1917	INNER CVR JFUSION1917 FLUID	
3	ICJFUSION1918	INNER CVR JFUSION1918 FLUID	
3	ICJFUSION1919	INNER CVR JFUSION1919 FLUID	
3	ICJFUSION1920	INNER CVR JFUSION1920 FLUID	
3	ICJFUSION1921	INNER CVR JFUSION1921 FLUID	
3	ICJFUSION1922	INNER CVR JFUSION1922 FLUID	
3	ICJFUSION2016	INNER CVR JFUSION2016 FLUID	
3	ICJFUSION2017	INNER CVR JFUSION2017 FLUID	
3	ICJFUSION2018	INNER CVR JFUSION2018 FLUID	
3	ICJFUSION2019	INNER CVR JFUSION2019 FLUID	
3	ICJFUSION2020	INNER CVR JFUSION2020 FLUID	
3	ICJFUSION2021	INNER CVR JFUSION2021 FLUID	
3	ICJFUSION2022	INNER CVR JFUSION2022 FLUID	

Pos.	Item Number	Description	Remarks
3	ICJFUSION2118	INNER CVR JFUSION2118 FLUID	
3	ICJFUSION2119	INNER CVR JFUSION2119 FLUID	
3	ICJFUSION2120	INNER CVR JFUSION2120 FLUID	
3	ICJFUSION2121	INNER CVR JFUSION2121 FLUID	
3	ICJFUSION2122	INNER CVR JFUSION2122 FLUID	
3	ICJFUSION2218	INNER CVR JFUSION2218 FLUID	
3	ICJFUSION2219	INNER CVR JFUSION2219 FLUID	
3	ICJFUSION2220	INNER CVR JFUSION2220 FLUID	
3	ICJFUSION2221	INNER CVR JFUSION2221 FLUID	
3	ICJFUSION2222	INNER CVR JFUSION2222 FLUID	
4	BJFUSION1616	BASE JFUSION1616	
4	BJFUSION1617	BASE JFUSION1617	
4	BJFUSION1618	BASE JFUSION1618	
4	BJFUSION1619	BASE JFUSION1619	
4	BJFUSION1620	BASE JFUSION1620	
4	BJFUSION1716	BASE JFUSION1716	
4	BJFUSION1717	BASE JFUSION1717	
4	BJFUSION1718	BASE JFUSION1718	
4	BJFUSION1719	BASE JFUSION1719	
4	BJFUSION1720	BASE JFUSION1720	
4	BJFUSION1721	BASE JFUSION1721	
4	BJFUSION1816	BASE JFUSION1816	
4	BJFUSION1817	BASE JFUSION1817	
4	BJFUSION1818	BASE JFUSION1818	
4	BJFUSION1819	BASE JFUSION1819	
4	BJFUSION1820	BASE JFUSION1820	
4	BJFUSION1821	BASE JFUSION1821	
4	BJFUSION1822	BASE JFUSION1822	
4	BJFUSION1916	BASE JFUSION1916	
4	BJFUSION1917	BASE JFUSION1917	
4	BJFUSION1918	BASE JFUSION1918	
4	BJFUSION1919	BASE JFUSION1919	
4	BJFUSION1920	BASE JFUSION1920	
4	BJFUSION1921	BASE JFUSION1921	
4	BJFUSION1922	BASE JFUSION1922	
4	BJFUSION2016	BASE JFUSION2016	
4	BJFUSION2017	BASE JFUSION2017	
4	BJFUSION2018	BASE JFUSION2018	
4	BJFUSION2019	BASE JFUSION2019	
4	BJFUSION2020	BASE JFUSION2020	
4	BJFUSION2021	BASE JFUSION2021	
4	BJFUSION2022	BASE JFUSION2022	
4	BJFUSION2118	BASE JFUSION2118	
4	BJFUSION2119	BASE JFUSION2119	
4	BJFUSION2120	BASE JFUSION2120	
4	BJFUSION2121	BASE JFUSION2121	
4	BJFUSION2122	BASE JFUSION2122	
4	BJFUSION2218	BASE JFUSION2218	
4	BJFUSION2219	BASE JFUSION2219	
4	BJFUSION2220	BASE JFUSION2220	
4	BJFUSION2221	BASE JFUSION2221	
4	BJFUSION2222	BASE JFUSION2222	
5A	JBALCOP-BLH	OBLIQUITY PAD LH PLA B	
5A	JBALCOP-BRH	OBLIQUITY PAD RH PLA B	
5B	JBALCOP-CLH	OBLIQUITY PAD LH PLA C	
5B	JBALCOP-CRH	OLBIQUITY PAD RH PLA C	
5C	JBALCOP-DLH	OBLIQUITY PAD LH PLA D	

Pos.	Item Number	Description	Remarks
5C	JBALCOP-DRH	OBLIQUITY PAD RH PLA D	

Note: for part number information, please see attachment in informations panel.

Note: Cushion widths 16" - 20" use B size bladders and 21"-22" use C size bladders.

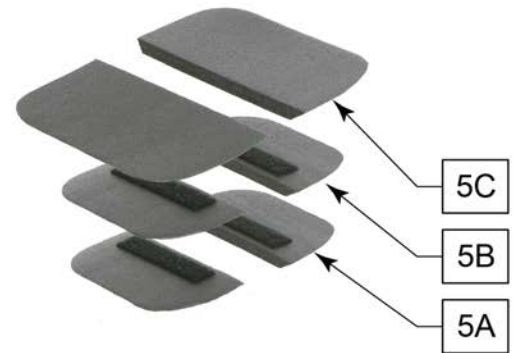

Obliquity Pad Ordering Guide

Cushion Width	1st Layer	2nd Layer
16"-20"	JBALCOP-BLH	JBALCOP-CLH
	JBALCOP-BRH	JBALCOP-CRH
21"-22"	JBALCOP-CLH	JBALCOP-DLH
	JBALCOP-CRH	JBALCOP-DRH

Note: Obliquity Pad is 1/2":
 One layer of pads = 1/2" height
 Two layers of pads = 1" height

Pos.	Item Number	Description	Remarks
1	CJFUSION1616CFHRP	COV FUSION RP CRYO HYB 16X16	
1	CJFUSION1617CFHRP	COV FUSION RP CRYO HYB 16X17	
1	CJFUSION1618CFHRP	COV FUSION RP CRYO HYB 16X18	
1	CJFUSION1619CFHRP	COV FUSION RP CRYO HYB 16X19	
1	CJFUSION1620CFHRP	COV FUSION RP CRYO HYB 16X20	
1	CJFUSION1716CFHRP	COV FUSION RP CRYO HYB 17X16	
1	CJFUSION1717CFHRP	COV FUSION RP CRYO HYB 17X17	
1	CJFUSION1718CFHRP	COV FUSION RP CRYO HYB 17X18	
1	CJFUSION1719CFHRP	COV FUSION RP CRYO HYB 17X19	
1	CJFUSION1720CFHRP	COV FUSION RP CRYO HYB 17X20	
1	CJFUSION1721CFHRP	COV FUSION RP CRYO HYB 17X21	
1	CJFUSION1816CFHRP	COV FUSION RP CRYO HYB 18X16	
1	CJFUSION1817CFHRP	COV FUSION RP CRYO HYB 18X17	
1	CJFUSION1818CFHRP	COV FUSION RP CRYO HYB 18X18	
1	CJFUSION1819CFHRP	COV FUSION RP CRYO HYB 18X19	

Pos.	Item Number	Description	Remarks
1	CJFUSION1820CFHRP	COV FUSION RP CRYO HYB 18X20	
1	CJFUSION1821CFHRP	COV FUSION RP CRYO HYB 18X21	
1	CJFUSION1822CFHRP	COV FUSION RP CRYO HYB 18X22	
1	CJFUSION1916CFHRP	COV FUSION RP CRYO HYB 19X16	
1	CJFUSION1917CFHRP	COV FUSION RP CRYO HYB 19X17	
1	CJFUSION1918CFHRP	COV FUSION RP CRYO HYB 19X18	
1	CJFUSION1919CFHRP	COV FUSION RP CRYO HYB 19X19	
1	CJFUSION1920CFHRP	COV FUSION RP CRYO HYB 19X20	
1	CJFUSION1921CFHRP	COV FUSION RP CRYO HYB 19X21	
1	CJFUSION1922CFHRP	COV FUSION RP CRYO HYB 19X22	
1	CJFUSION1924CFHRP	COV FUSION RP CRYO HYB 19X24	
1	CJFUSION2016CFHRP	COV FUSION RP CRYO HYB 20X16	
1	CJFUSION2017CFHRP	COV FUSION RP CRYO HYB 20X17	
1	CJFUSION2018CFHRP	COV FUSION RP CRYO HYB 20X18	
1	CJFUSION2019CFHRP	COV FUSION RP CRYO HYB 20X19	
1	CJFUSION2020CFHRP	COV FUSION RP CRYO HYB 20X20	
1	CJFUSION2021CFHRP	COV FUSION RP CRYO HYB 20X21	
1	CJFUSION2022CFHRP	COV FUSION RP CRYO HYB 20X22	
1	CJFUSION2118CFHRP	COV FUSION RP CRYO HYB 21X18	
1	CJFUSION2119CFHRP	COV FUSION RP CRYO HYB 21X19	
1	CJFUSION2120CFHRP	COV FUSION RP CRYO HYB 21X20	
1	CJFUSION2121CFHRP	COV FUSION RP CRYO HYB 21X21	
1	CJFUSION2122CFHRP	COV FUSION RP CRYO HYB 21X22	
1	CJFUSION2218CFHRP	COV FUSION RP CRYO HYB 22X18	
1	CJFUSION2219CFHRP	COV FUSION RP CRYO HYB 22X19	
1	CJFUSION2220CFHRP	COV FUSION RP CRYO HYB 22X20	
1	CJFUSION2221CFHRP	COV FUSION RP CRYO HYB 22X21	
1	CJFUSION2222CFHRP	COV FUSION RP CRYO HYB 22X22	
2	CFHB	CRYO FLUID HYBRID BLADDER B	Used in 16"-20" wide
2	CFHC	CRYO FLUID HYBRID BLADDER C	Used in 21"-22" wide
3	ICJFUSION1616RP	INN COV FUSION RP FLD 16X16	
3	ICJFUSION1617RP	INN COV FUSION RP FLD 16X17	
3	ICJFUSION1618RP	INN COV FUSION RP FLD 16X18	
3	ICJFUSION1619RP	INN COV FUSION RP FLD 16X19	
3	ICJFUSION1620RP	INN COV FUSION RP FLD 16X20	
3	ICJFUSION1716RP	INN COV FUSION RP FLD 17X16	
3	ICJFUSION1717RP	INN COV FUSION RP FLD 17X17	
3	ICJFUSION1718RP	INN COV FUSION RP FLD 17X18	
3	ICJFUSION1719RP	INN COV FUSION RP FLD 17X19	
3	ICJFUSION1720RP	INN COV FUSION RP FLD 17X20	
3	ICJFUSION1721RP	INN COV FUSION RP FLD 17X21	
3	ICJFUSION1816RP	INN COV FUSION RP FLD 18X16	
3	ICJFUSION1817RP	INN COV FUSION RP FLD 18X17	
3	ICJFUSION1818RP	INN COV FUSION RP FLD 18X18	
3	ICJFUSION1819RP	INN COV FUSION RP FLD 18X19	
3	ICJFUSION1820RP	INN COV FUSION RP FLD 18X20	
3	ICJFUSION1821RP	INN COV FUSION RP FLD 18X21	
3	ICJFUSION1822RP	INN COV FUSION RP FLD 18X22	
3	ICJFUSION1916RP	INN COV FUSION RP FLD 19X16	
3	ICJFUSION1917RP	INN COV FUSION RP FLD 19X17	
3	ICJFUSION1918RP	INN COV FUSION RP FLD 19X18	
3	ICJFUSION1919RP	INN COV FUSION RP FLD 19X19	
3	ICJFUSION1920RP	INN COV FUSION RP FLD 19X20	
3	ICJFUSION1921RP	INN COV FUSION RP FLD 19X21	
3	ICJFUSION1922RP	INN COV FUSION RP FLD 19X22	
3	ICJFUSION2016RP	INN COV FUSION RP FLD 20X16	
3	ICJFUSION2017RP	INN COV FUSION RP FLD 20X17	

Pos.	Item Number	Description	Remarks
3	ICJFUSION2018RP	INN COV FUSION RP FLD 20X18	
3	ICJFUSION2019RP	INN COV FUSION RP FLD 20X19	
3	ICJFUSION2020RP	INN COV FUSION RP FLD 20X20	
3	ICJFUSION2021RP	INN COV FUSION RP FLD 20X21	
3	ICJFUSION2022RP	INN COV FUSION RP FLD 20X22	
3	ICJFUSION2024RP	INN COV FUSION RP FLD 20X24	
3	ICJFUSION2118RP	INN COV FUSION RP FLD 21X18	
3	ICJFUSION2119RP	INN COV FUSION RP FLD 21X19	
3	ICJFUSION2120RP	INN COV FUSION RP FLD 21X20	
3	ICJFUSION2121RP	INN COV FUSION RP FLD 21X21	
3	ICJFUSION2122RP	INN COV FUSION RP FLD 21X22	
3	ICJFUSION2218RP	INN COV FUSION RP FLD 22X18	
3	ICJFUSION2219RP	INN COV FUSION RP FLD 22X19	
3	ICJFUSION2220RP	INN COV FUSION RP FLD 22X20	
3	ICJFUSION2221RP	INN COV FUSION RP FLD 22X21	
3	ICJFUSION2222RP	INN COV FUSION RP FLD 22X22	
3	ICJFUSION2224RP	INN COV FUSION RP FLD 22X24	
4	BJFUSION1616RP	BASE JAY FUSION RPROF 16X16	
4	BJFUSION1617RP	BASE JAY FUSION RPROF 16X17	
4	BJFUSION1618RP	BASE JAY FUSION RPROF 16X18	
4	BJFUSION1619RP	BASE JAY FUSION RPROF 16X19	
4	BJFUSION1620RP	BASE JAY FUSION RPROF 16X20	
4	BJFUSION1716RP	BASE JAY FUSION RPROF 17X16	
4	BJFUSION1717RP	BASE JAY FUSION RPROF 17X17	
4	BJFUSION1718RP	BASE JAY FUSION RPROF 17X18	
4	BJFUSION1719RP	BASE JAY FUSION RPROF 17X19	
4	BJFUSION1720RP	BASE JAY FUSION RPROF 17X20	
4	BJFUSION1721RP	BASE JAY FUSION RPROF 17X21	
4	BJFUSION1816RP	BASE JAY FUSION RPROF 18X16	
4	BJFUSION1817RP	BASE JAY FUSION RPROF 18X17	
4	BJFUSION1818RP	BASE JAY FUSION RPROF 18X18	
4	BJFUSION1819RP	BASE JAY FUSION RPROF 18X19	
4	BJFUSION1820RP	BASE JAY FUSION RPROF 18X20	
4	BJFUSION1821RP	BASE JAY FUSION RPROF 18X21	
4	BJFUSION1822RP	BASE JAY FUSION RPROF 18X22	
4	BJFUSION1916RP	BASE JAY FUSION RPROF 19X16	
4	BJFUSION1917RP	BASE JAY FUSION RPROF 19X17	
4	BJFUSION1918RP	BASE JAY FUSION RPROF 19X18	
4	BJFUSION1919RP	BASE JAY FUSION RPROF 19X19	
4	BJFUSION1920RP	BASE JAY FUSION RPROF 19X20	
4	BJFUSION1921RP	BASE JAY FUSION RPROF 19X21	
4	BJFUSION1922RP	BASE JAY FUSION RPROF 19X22	
4	BJFUSION2016RP	BASE JAY FUSION RPROF 20X16	
4	BJFUSION2017RP	BASE JAY FUSION RPROF 20X17	
4	BJFUSION2018RP	BASE JAY FUSION RPROF 20X18	
4	BJFUSION2019RP	BASE JAY FUSION RPROF 20X19	
4	BJFUSION2020RP	BASE JAY FUSION RPROF 20X20	
4	BJFUSION2021RP	BASE JAY FUSION RPROF 20X21	
4	BJFUSION2022RP	BASE JAY FUSION RPROF 20X22	
4	BJFUSION2024RP	BASE JAY FUSION RPROF 20X24	
4	BJFUSION2118RP	BASE JAY FUSION RPROF 21X18	
4	BJFUSION2119RP	BASE JAY FUSION RPROF 21X19	
4	BJFUSION2120RP	BASE JAY FUSION RPROF 21X20	
4	BJFUSION2121RP	BASE JAY FUSION RPROF 21X21	
4	BJFUSION2122RP	BASE JAY FUSION RPROF 21X22	
4	BJFUSION2218RP	BASE JAY FUSION RPROF 22X18	
4	BJFUSION2219RP	BASE JAY FUSION RPROF 22X19	

Pos.	Item Number	Description	Remarks
4	BJFUSION2220RP	BASE JAY FUSION RPROF 22X20	
4	BJFUSION2221RP	BASE JAY FUSION RPROF 22X21	
4	BJFUSION2222RP	BASE JAY FUSION RPROF 22X22	
4	BJFUSION2424	BASE JFUSION2424	
5A	JBALCOP-BLH	OBLIQUITY PAD LH PLA B	Left- 16 thru 20 wide
5A	JBALCOP-BRH	OBLIQUITY PAD RH PLA B	Right- 16 thru 20 wide
5B	JBALCOP-CLH	OBLIQUITY PAD LH PLA C	Left- 21 thru 22 wide
5B	JBALCOP-CRH	OBLIQUITY PAD RH PLA C	Right- 21 thru 22 wide
5C	JBALCOP-DLH	OBLIQUITY PAD LH PLA D	Left
5C	JBALCOP-DRH	OBLIQUITY PAD RH PLA D	Right

Note: for part number information, please see attachment in informations panel.

Note: Cushion widths , 16" - 20" use B Fluid Bladders and 21"-22" use C size bladders.




Pos.	Item Number	Description	Remarks
1	CJFUSION1616CFHP	COV FUSION POS CRYO HYB 16X16	
1	CJFUSION1617CFHP	COV FUSION POS CRYO HYB 16X17	
1	CJFUSION1618CFHP	COV FUSION POS CRYO HYB 16X18	
1	CJFUSION1619CFHP	COV FUSION POS CRYO HYB 16X19	
1	CJFUSION1620CFHP	COV FUSION POS CRYO HYB 16X20	
1	CJFUSION1716CFHP	COV FUSION POS CRYO HYB 17X16	
1	CJFUSION1717CFHP	COV FUSION POS CRYO HYB 17X17	
1	CJFUSION1718CFHP	COV FUSION POS CRYO HYB 17X18	
1	CJFUSION1719CFHP	COV FUSION POS CRYO HYB 17X19	
1	CJFUSION1720CFHP	COV FUSION POS CRYO HYB 17X20	
1	CJFUSION1721CFHP	COV FUSION POS CRYO HYB 17X21	
1	CJFUSION1816CFHP	COV FUSION POS CRYO HYB 18X16	
1	CJFUSION1817CFHP	COV FUSION POS CRYO HYB 18X17	
1	CJFUSION1818CFHP	COV FUSION POS CRYO HYB 18X18	

Pos.	Item Number	Description	Remarks
1	CJFUSION1819CFHP	COV FUSION POS CRYO HYB 18X19	
1	CJFUSION1820CFHP	COV FUSION POS CRYO HYB 18X20	
1	CJFUSION1821CFHP	COV FUSION POS CRYO HYB 18X21	
1	CJFUSION1822CFHP	COV FUSION POS CRYO HYB 18X22	
1	CJFUSION1916CFHP	COV FUSION POS CRYO HYB 19X16	
1	CJFUSION1917CFHP	COV FUSION POS CRYO HYB 19X17	
1	CJFUSION1918CFHP	COV FUSION POS CRYO HYB 19X18	
1	CJFUSION1919CFHP	COV FUSION POS CRYO HYB 19X19	
1	CJFUSION1920CFHP	COV FUSION POS CRYO HYB 19X20	
1	CJFUSION1921CFHP	COV FUSION POS CRYO HYB 19X21	
1	CJFUSION1922CFHP	COV FUSION POS CRYO HYB 19X22	
1	CJFUSION2016CFHP	COV FUSION POS CRYO HYB 20X16	
1	CJFUSION2017CFHP	COV FUSION POS CRYO HYB 20X17	
1	CJFUSION2018CFHP	COV FUSION POS CRYO HYB 20X18	
1	CJFUSION2019CFHP	COV FUSION POS CRYO HYB 20X19	
1	CJFUSION2020CFHP	COV FUSION POS CRYO HYB 20X20	
1	CJFUSION2021CFHP	COV FUSION POS CRYO HYB 20X21	
1	CJFUSION2022CFHP	COV FUSION POS CRYO HYB 20X22	
1	CJFUSION2024CFHP	COV FUSION POS CRYO HYB 20X24	
1	CJFUSION2118CFHP	COV FUSION POS CRYO HYB 21X18	
1	CJFUSION2119CFHP	COV FUSION POS CRYO HYB 21X19	
1	CJFUSION2120CFHP	COV FUSION POS CRYO HYB 21X20	
1	CJFUSION2121CFHP	COV FUSION POS CRYO HYB 21X21	
1	CJFUSION2122CFHP	COV FUSION POS CRYO HYB 21X22	
1	CJFUSION2218CFHP	COV FUSION POS CRYO HYB 22X18	
1	CJFUSION2219CFHP	COV FUSION POS CRYO HYB 22X19	
1	CJFUSION2220CFHP	COV FUSION POS CRYO HYB 22X20	
1	CJFUSION2221CFHP	COV FUSION POS CRYO HYB 22X21	
1	CJFUSION2222CFHP	COV FUSION POS CRYO HYB 22X22	
1	CJFUSION2224CFHP	COV FUSION POS CRYO HYB 22X24	
2	CFHB	CRYO FLUID HYBRID BLADDER B	Used in 16"-20" wide
2	CFHC	CRYO FLUID HYBRID BLADDER C	Used in 21"-22" wide
3	JBALCPKIT	JAY BALANCE CUSH POS KIT	5 pieces
4	ICJFUSION1616P	INN COV FUSION POS FLD 16X16	
4	ICJFUSION1617P	INN COV FUSION POS FLD 16X17	
4	ICJFUSION1618P	INN COV FUSION POS FLD 16X18	
4	ICJFUSION1619P	INN COV FUSION POS FLD 16X19	
4	ICJFUSION1620P	INN COV FUSION POS FLD 16X20	
4	ICJFUSION1716P	INN COV FUSION POS FLD 17X16	
4	ICJFUSION1717P	INN COV FUSION POS FLD 17X17	
4	ICJFUSION1718P	INN COV FUSION POS FLD 17X18	
4	ICJFUSION1719P	INN COV FUSION POS FLD 17X19	
4	ICJFUSION1720P	INN COV FUSION POS FLD 17X20	
4	ICJFUSION1721P	INN COV FUSION POS FLD 17X21	
4	ICJFUSION1816P	INN COV FUSION POS FLD 18X16	
4	ICJFUSION1817P	INN COV FUSION POS FLD 18X17	
4	ICJFUSION1818P	INN COV FUSION POS FLD 18X18	
4	ICJFUSION1819P	INN COV FUSION POS FLD 18X19	
4	ICJFUSION1820P	INN COV FUSION POS FLD 18X20	
4	ICJFUSION1821P	INN COV FUSION POS FLD 18X21	
4	ICJFUSION1822P	INN COV FUSION POS FLD 18X22	
4	ICJFUSION1916P	INN COV FUSION POS FLD 19X16	
4	ICJFUSION1917P	INN COV FUSION POS FLD 19X17	
4	ICJFUSION1918P	INN COV FUSION POS FLD 19X18	
4	ICJFUSION1919P	INN COV FUSION POS FLD 19X19	
4	ICJFUSION1920P	INN COV FUSION POS FLD 19X20	
4	ICJFUSION1921P	INN COV FUSION POS FLD 19X21	

Pos.	Item Number	Description	Remarks
4	ICJFUSION1922P	INN COV FUSION POS FLD 19X22	
4	ICJFUSION2016P	INN COV FUSION POS FLD 20X16	
4	ICJFUSION2017P	INN COV FUSION POS FLD 20X17	
4	ICJFUSION2018P	INN COV FUSION POS FLD 20X18	
4	ICJFUSION2019P	INN COV FUSION POS FLD 20X19	
4	ICJFUSION2020P	INN COV FUSION POS FLD 20X20	
4	ICJFUSION2021P	INN COV FUSION POS FLD 20X21	
4	ICJFUSION2022P	INN COV FUSION POS FLD 20X22	
4	ICJFUSION2118P	INN COV FUSION POS FLD 21X18	
4	ICJFUSION2119P	INN COV FUSION POS FLD 21X19	
4	ICJFUSION2120P	INN COV FUSION POS FLD 21X20	
4	ICJFUSION2121P	INN COV FUSION POS FLD 21X21	
4	ICJFUSION2122P	INN COV FUSION POS FLD 21X22	
4	ICJFUSION2218P	INN COV FUSION POS FLD 22X18	
4	ICJFUSION2219P	INN COV FUSION POS FLD 22X19	
4	ICJFUSION2220P	INN COV FUSION POS FLD 22X20	
4	ICJFUSION2221P	INN COV FUSION POS FLD 22X21	
4	ICJFUSION2222P	INN COV FUSION POS FLD 22X22	
5	BJFUSION1616	BASE JFUSION1616	
5	BJFUSION1617	BASE JFUSION1617	
5	BJFUSION1618	BASE JFUSION1618	
5	BJFUSION1619	BASE JFUSION1619	
5	BJFUSION1620	BASE JFUSION1620	
5	BJFUSION1716	BASE JFUSION1716	
5	BJFUSION1717	BASE JFUSION1717	
5	BJFUSION1718	BASE JFUSION1718	
5	BJFUSION1719	BASE JFUSION1719	
5	BJFUSION1720	BASE JFUSION1720	
5	BJFUSION1721	BASE JFUSION1721	
5	BJFUSION1816	BASE JFUSION1816	
5	BJFUSION1817	BASE JFUSION1817	
5	BJFUSION1818	BASE JFUSION1818	
5	BJFUSION1819	BASE JFUSION1819	
5	BJFUSION1820	BASE JFUSION1820	
5	BJFUSION1821	BASE JFUSION1821	
5	BJFUSION1822	BASE JFUSION1822	
5	BJFUSION1824	BASE JFUSION1824	
5	BJFUSION1916	BASE JFUSION1916	
5	BJFUSION1917	BASE JFUSION1917	
5	BJFUSION1918	BASE JFUSION1918	
5	BJFUSION1919	BASE JFUSION1919	
5	BJFUSION1920	BASE JFUSION1920	
5	BJFUSION1921	BASE JFUSION1921	
5	BJFUSION1922	BASE JFUSION1922	
5	BJFUSION2016	BASE JFUSION2016	
5	BJFUSION2017	BASE JFUSION2017	
5	BJFUSION2018	BASE JFUSION2018	
5	BJFUSION2019	BASE JFUSION2019	
5	BJFUSION2020	BASE JFUSION2020	
5	BJFUSION2021	BASE JFUSION2021	
5	BJFUSION2022	BASE JFUSION2022	
5	BJFUSION2118	BASE JFUSION2118	
5	BJFUSION2119	BASE JFUSION2119	
5	BJFUSION2120	BASE JFUSION2120	
5	BJFUSION2121	BASE JFUSION2121	
5	BJFUSION2122	BASE JFUSION2122	
5	BJFUSION2218	BASE JFUSION2218	

Pos.	Item Number	Description	Remarks
5	BJFUSION2219	BASE JFUSION2219	
5	BJFUSION2220	BASE JFUSION2220	
5	BJFUSION2221	BASE JFUSION2221	
5	BJFUSION2222	BASE JFUSION2222	
6A	JBALCOP-BLH	OBLIQUITY PAD LH PLA B	Left- 16 thru 20 wide
6A	JBALCOP-BRH	OBLIQUITY PAD RH PLA B	Right- 16 thru 20 wide
6B	JBALCOP-CLH	OBLIQUITY PAD LH PLA C	Left- 21 thru 22 wide
6B	JBALCOP-CRH	OBLIQUITY PAD RH PLA C	Right- 21 thru 22 wide
6C	JBALCOP-DLH	OBLIQUITY PAD LH PLA D	Left
6C	JBALCOP-DRH	OBLIQUITY PAD RH PLA D	Right

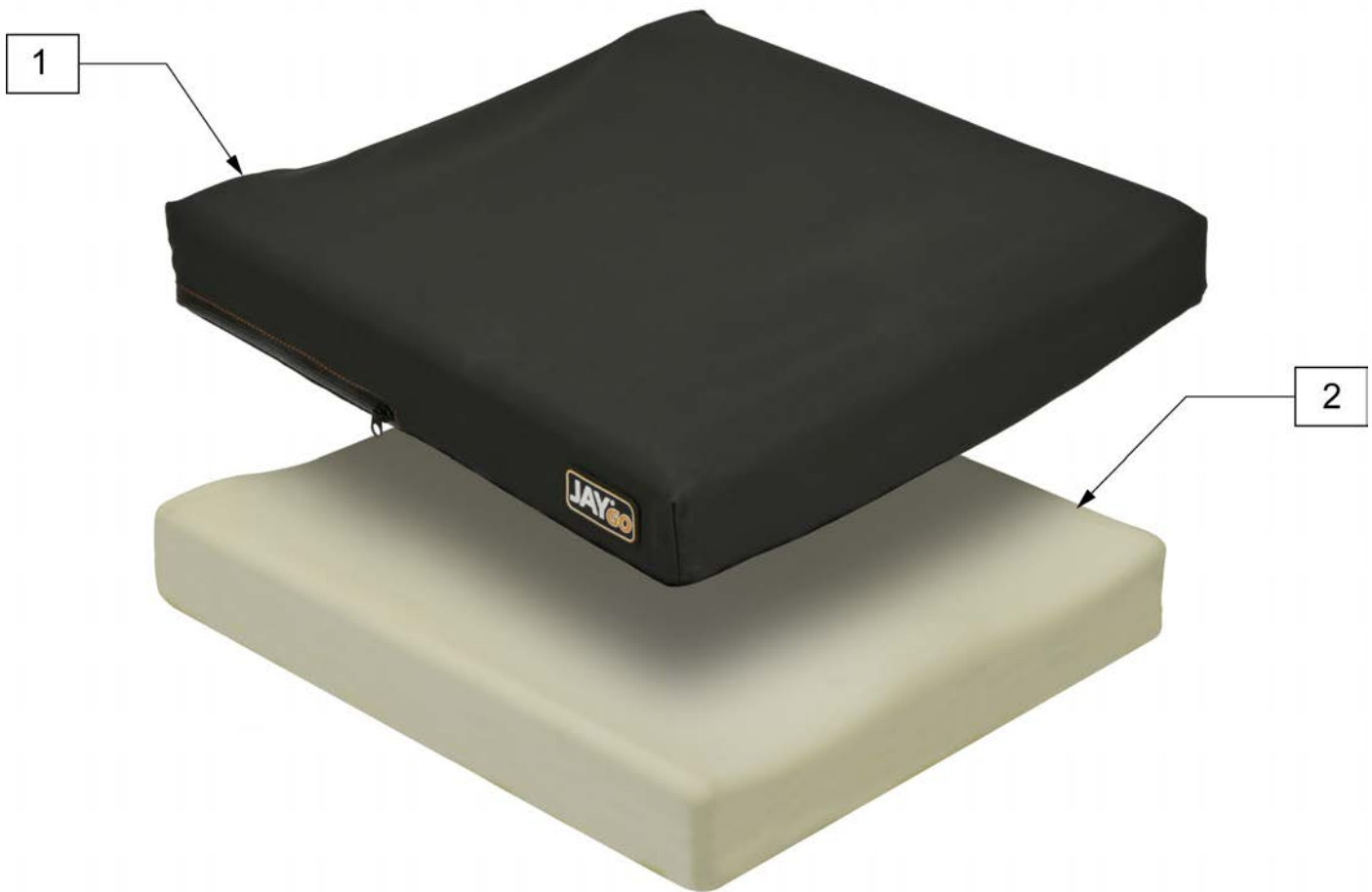

Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJGO1414	COVER JAY GO 14X14	1	CJGO1620	COVER JAY GO 16X20
1	CJGO1415	COVER JAY GO 14X15	1	CJGO1621	COVER JAY GO 16X21
1	CJGO1416	COVER JAY GO 14X16	1	CJGO1622	COVER JAY GO 16X22
1	CJGO1417	COVER JAY GO 14X17	1	CJGO1714	COVER JAY GO 17X14
1	CJGO1418	COVER JAY GO 14X18	1	CJGO1715	COVER JAY GO 17X15
1	CJGO1419	COVER JAY GO 14X19	1	CJGO1716	COVER JAY GO 17X16
1	CJGO1420	COVER JAY GO 14X20	1	CJGO1717	COVER JAY GO 17X17
1	CJGO1514	COVER JAY GO 15X14	1	CJGO1718	COVER JAY GO 17X18
1	CJGO1515	COVER JAY GO 15X15	1	CJGO1719	COVER JAY GO 17X19
1	CJGO1516	COVER JAY GO 15X16	1	CJGO1720	COVER JAY GO 17X20
1	CJGO1517	COVER JAY GO 15X17	1	CJGO1721	COVER JAY GO 17X21
1	CJGO1518	COVER JAY GO 15X18	1	CJGO1722	COVER JAY GO 17X22
1	CJGO1519	COVER JAY GO 15X19	1	CJGO1723	COVER JAY GO 17X23
1	CJGO1520	COVER JAY GO 15X20	1	CJGO1814	COVER JAY GO 18X14
1	CJGO1521	COVER JAY GO 15X21	1	CJGO1815	COVER JAY GO 18X15
1	CJGO1614	COVER JAY GO 16X14	1	CJGO1816	COVER JAY GO 18X16
1	CJGO1615	COVER JAY GO 16X15	1	CJGO1817	COVER JAY GO 18X17
1	CJGO1616	COVER JAY GO 16X16	1	CJGO1818	COVER JAY GO 18X18
1	CJGO1617	COVER JAY GO 16X17	1	CJGO1819	COVER JAY GO 18X19
1	CJGO1618	COVER JAY GO 16X18	1	CJGO1820	COVER JAY GO 18X20
1	CJGO1619	COVER JAY GO 16X19	1	CJGO1821	COVER JAY GO 18X21


Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJGO1822	COVER JAY GO 18X22	2	BJGO1420	BASE FOAM JAY GO 14X20
1	CJGO1823	COVER JAY GO 18X23	2	BJGO1514	BASE FOAM JAY GO 15X14
1	CJGO1824	COVER JAY GO 18X24	2	BJGO1515	BASE FOAM JAY GO 15X15
1	CJGO1915	COVER JAY GO 19X15	2	BJGO1516	BASE FOAM JAY GO 15X16
1	CJGO1916	COVER JAY GO 19X16	2	BJGO1517	BASE FOAM JAY GO 15X17
1	CJGO1917	COVER JAY GO 19X17	2	BJGO1518	BASE FOAM JAY GO 15X18
1	CJGO1918	COVER JAY GO 19X18	2	BJGO1519	BASE FOAM JAY GO 15X19
1	CJGO1919	COVER JAY GO 19X19	2	BJGO1520	BASE FOAM JAY GO 15X20
1	CJGO1920	COVER JAY GO 19X20	2	BJGO1521	BASE FOAM JAY GO 15X21
1	CJGO1921	COVER JAY GO 19X21	2	BJGO1614	BASE FOAM JAY GO 16X14
1	CJGO1922	COVER JAY GO 19X22	2	BJGO1615	BASE FOAM JAY GO 16X15
1	CJGO1923	COVER JAY GO 19X23	2	BJGO1616	BASE FOAM JAY GO 16X16
1	CJGO1924	COVER JAY GO 19X24	2	BJGO1617	BASE FOAM JAY GO 16X17
1	CJGO2015	COVER JAY GO 20X15	2	BJGO1618	BASE FOAM JAY GO 16X18
1	CJGO2016	COVER JAY GO 20X16	2	BJGO1619	BASE FOAM JAY GO 16X19
1	CJGO2017	COVER JAY GO 20X17	2	BJGO1620	BASE FOAM JAY GO 16X20
1	CJGO2018	COVER JAY GO 20X18	2	BJGO1621	BASE FOAM JAY GO 16X21
1	CJGO2019	COVER JAY GO 20X19	2	BJGO1622	BASE FOAM JAY GO 16X22
1	CJGO2020	COVER JAY GO 20X20	2	BJGO1714	BASE FOAM JAY GO 17X14
1	CJGO2021	COVER JAY GO 20X21	2	BJGO1715	BASE FOAM JAY GO 17X15
1	CJGO2022	COVER JAY GO 20X22	2	BJGO1716	BASE FOAM JAY GO 17X16
1	CJGO2023	COVER JAY GO 20X23	2	BJGO1717	BASE FOAM JAY GO 17X17
1	CJGO2024	COVER JAY GO 20X24	2	BJGO1718	BASE FOAM JAY GO 17X18
1	CJGO2117	COVER JAY GO 21X17	2	BJGO1719	BASE FOAM JAY GO 17X19
1	CJGO2118	COVER JAY GO 21X18	2	BJGO1720	BASE FOAM JAY GO 17X20
1	CJGO2119	COVER JAY GO 21X19	2	BJGO1721	BASE FOAM JAY GO 17X21
1	CJGO2120	COVER JAY GO 21X20	2	BJGO1722	BASE FOAM JAY GO 17X22
1	CJGO2121	COVER JAY GO 21X21	2	BJGO1723	BASE FOAM JAY GO 17X23
1	CJGO2122	COVER JAY GO 21X22	2	BJGO1814	BASE FOAM JAY GO 18X14
1	CJGO2123	COVER JAY GO 21X23	2	BJGO1815	BASE FOAM JAY GO 18X15
1	CJGO2124	COVER JAY GO 21X24	2	BJGO1816	BASE FOAM JAY GO 18X16
1	CJGO2217	COVER JAY GO 22X17	2	BJGO1817	BASE FOAM JAY GO 18X17
1	CJGO2218	COVER JAY GO 22X18	2	BJGO1818	BASE FOAM JAY GO 18X18
1	CJGO2219	COVER JAY GO 22X19	2	BJGO1819	BASE FOAM JAY GO 18X19
1	CJGO2220	COVER JAY GO 22X20	2	BJGO1820	BASE FOAM JAY GO 18X20
1	CJGO2221	COVER JAY GO 22X21	2	BJGO1821	BASE FOAM JAY GO 18X21
1	CJGO2222	COVER JAY GO 22X22	2	BJGO1822	BASE FOAM JAY GO 18X22
1	CJGO2223	COVER JAY GO 22X23	2	BJGO1823	BASE FOAM JAY GO 18X23
1	CJGO2224	COVER JAY GO 22X24	2	BJGO1824	BASE FOAM JAY GO 18X24
1	CJGO2319	COVER JAY GO 23X19	2	BJGO1915	BASE FOAM JAY GO 19X15
1	CJGO2320	COVER JAY GO 23X20	2	BJGO1916	BASE FOAM JAY GO 19X16
1	CJGO2321	COVER JAY GO 23X21	2	BJGO1917	BASE FOAM JAY GO 19X17
1	CJGO2322	COVER JAY GO 23X22	2	BJGO1918	BASE FOAM JAY GO 19X18
1	CJGO2323	COVER JAY GO 23X23	2	BJGO1919	BASE FOAM JAY GO 19X19
1	CJGO2324	COVER JAY GO 23X24	2	BJGO1920	BASE FOAM JAY GO 19X20
1	CJGO2419	COVER JAY GO 24X19	2	BJGO1921	BASE FOAM JAY GO 19X21
1	CJGO2420	COVER JAY GO 24X20	2	BJGO1922	BASE FOAM JAY GO 19X22
1	CJGO2421	COVER JAY GO 24X21	2	BJGO1923	BASE FOAM JAY GO 19X23
1	CJGO2422	COVER JAY GO 24X22	2	BJGO1924	BASE FOAM JAY GO 19X24
1	CJGO2423	COVER JAY GO 24X23	2	BJGO2015	BASE FOAM JAY GO 20X15
1	CJGO2424	COVER JAY GO 24X24	2	BJGO2016	BASE FOAM JAY GO 20X16
2	BJGO1414	BASE FOAM JAY GO 14X14	2	BJGO2017	BASE FOAM JAY GO 20X17
2	BJGO1415	BASE FOAM JAY GO 14X15	2	BJGO2018	BASE FOAM JAY GO 20X18
2	BJGO1416	BASE FOAM JAY GO 14X16	2	BJGO2019	BASE FOAM JAY GO 20X19
2	BJGO1417	BASE FOAM JAY GO 14X17	2	BJGO2020	BASE FOAM JAY GO 20X20
2	BJGO1418	BASE FOAM JAY GO 14X18	2	BJGO2021	BASE FOAM JAY GO 20X21
2	BJGO1419	BASE FOAM JAY GO 14X19	2	BJGO2022	BASE FOAM JAY GO 20X22

Pos.	Item Number	Description	Pos.	Item Number	Description
2	BJGO2023	BASE FOAM JAY GO 20X23	2	BJGO2222	BASE FOAM JAY GO 22X22
2	BJGO2024	BASE FOAM JAY GO 20X24	2	BJGO2223	BASE FOAM JAY GO 22X23
2	BJGO2117	BASE FOAM JAY GO 21X17	2	BJGO2224	BASE FOAM JAY GO 22X24
2	BJGO2118	BASE FOAM JAY GO 21X18	2	BJGO2319	BASE FOAM JAY GO 23X19
2	BJGO2119	BASE FOAM JAY GO 21X19	2	BJGO2320	BASE FOAM JAY GO 23X20
2	BJGO2120	BASE FOAM JAY GO 21X20	2	BJGO2321	BASE FOAM JAY GO 23X21
2	BJGO2121	BASE FOAM JAY GO 21X21	2	BJGO2322	BASE FOAM JAY GO 23X22
2	BJGO2122	BASE FOAM JAY GO 21X22	2	BJGO2323	BASE FOAM JAY GO 23X23
2	BJGO2123	BASE FOAM JAY GO 21X23	2	BJGO2324	BASE FOAM JAY GO 23X24
2	BJGO2124	BASE FOAM JAY GO 21X24	2	BJGO2419	BASE FOAM JAY GO 24X19
2	BJGO2217	BASE FOAM JAY GO 22X17	2	BJGO2420	BASE FOAM JAY GO 24X20
2	BJGO2218	BASE FOAM JAY GO 22X18	2	BJGO2421	BASE FOAM JAY GO 24X21
2	BJGO2219	BASE FOAM JAY GO 22X19	2	BJGO2422	BASE FOAM JAY GO 24X22
2	BJGO2220	BASE FOAM JAY GO 22X20	2	BJGO2423	BASE FOAM JAY GO 24X23
2	BJGO2221	BASE FOAM JAY GO 22X21	2	BJGO2424	BASE FOAM JAY GO 24X24

Note: for cushions purchased after 3/7/2016 that need a replacement fluid bladder, please order the fluid pad marked "2B"; for those cushions ordered before 3/7/2016, please order the fluid pad marked "2A".


Note: for cushions purchased after 3/7/2016 adapter Velcro will be shipped to allow the replacement parts to attach to existing base, fluid bladder, or positioning pieces.

Note: for cushions purchased after 3/7/2016 that are needing a replacement cover a new style replacement cover will need to be ordered; if positioning pieces are utilized in the existing cushion a P2 or P3 cushion is required, and if no positioning a non P cover will be adequate.

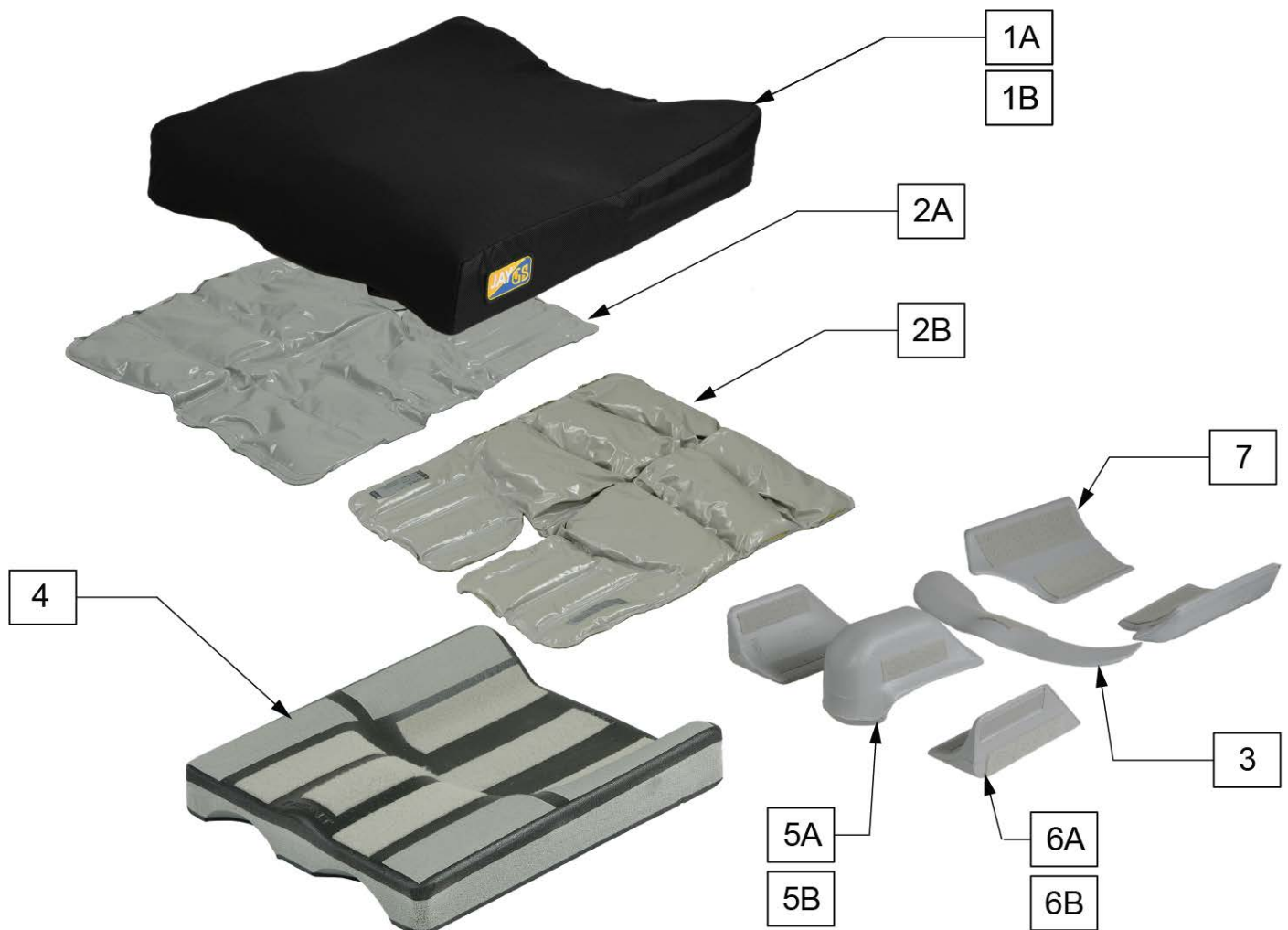

Pos.	Item Number	Description	Remarks
1A	CJGS10	CVR, JAY GS, 10IN X 10-13IN	
1A	CJGS12	CVR, JAY GS, 12IN X 12-15IN	
1A	CJGS14	CVR, JAY GS, 14IN X 14-17IN	
1A	CJGS16	CVR, JAY GS, 16IN X 16-19IN	
1A	CJGS18	CVR, JAY GS, 18IN X 17-20IN	

Pos.	Item Number	Description	Remarks
1B	CJGS10P2	CVR, JAY GS,2POS,10INX10-13IN	
1B	CJGS12P2	CVR, JAY GS,2POS,12INX12-15IN	
1B	CJGS14P2	CVR, JAY GS,2POS,14INX14-17IN	
1B	CJGS16P2	CVR, JAY GS,2POS,16INX16-19IN	
1B	CJGS18P2	CVR, JAY GS,2POS,18INX17-20IN	
1C	CJGS14P3	CVR,JAY GS,3POS,14INx14-17IN	
1C	CJGS16P3	CVR,JAY GS,3POS,16INx16-19IN	
1C	CJGS18P3	CVR,JAY GS,3POS,18INx17-20IN	
2A	F1100	FLUID PAD JAY GS JUNIOR 18 X 20	
2A	F1105	FLUID PAD JAY GS JUNIOR 16 X 19	
2A	F1110	FLUID PAD JAY GS JUNIOR 14 X 17	
2A	F1111	FLUID PAD JAY GS JUNIOR 10 X 13	
2A	F1115	FLUID PAD JAY GS JUNIOR 12 X 15	
2B	F1100.1	PRESSURE RELIEVING PAD 18 X 20	
2B	F1105.1	PRESSURE RELIEVING PAD 16 X 19	
2B	F1110.1	PRESSURE RELIEVING PAD 14 X 17	
2B	F1111.1	PRESSURE RELIEVING PAD 10 X 13	
2B	F1115.1	PRESSURE RELIEVING PAD 12 X 15	
3	B1171	PRE ISCHIAL SHELF EXTENSION GS (10" x 13")	
3	B1173	PRE ISCHIAL SHELF EXTENSION GS (12" x 15")	
3	B1174	PRE ISCHIAL SHELF EXTENSION GS (14" x 17")	
3	B1175	PRE ISCHIAL SHELF EXTENSION GS (16" x 19")	
3	B1176	PRE ISCHIAL SHELF EXTENSION GS (18" x 20")	
4	B1100	JAY GS BASE 18"	
4	B1105	JAY GS BASE 16"	
4	B1110	JAY GS BASE JUNIOR 14"	
4	B1111	JAY GS BASE SMALL CHILD 10"	
4	B1115	JAY GS BASE CHILD 12"	
5A	B1118	3" ABDUCTOR BUILD-UP	*
5B	B168	BUILD-UP 2" JUNIOR ABDUCTOR	*
6A	B163	2" JUNIOR ADDUCTOR WEDGES	Pair*
6B	B1113	WEDGES,3" ADDUCTOR	Pair*
7	B1167	HIP GUIDES,2",JAY & GS	Pair*

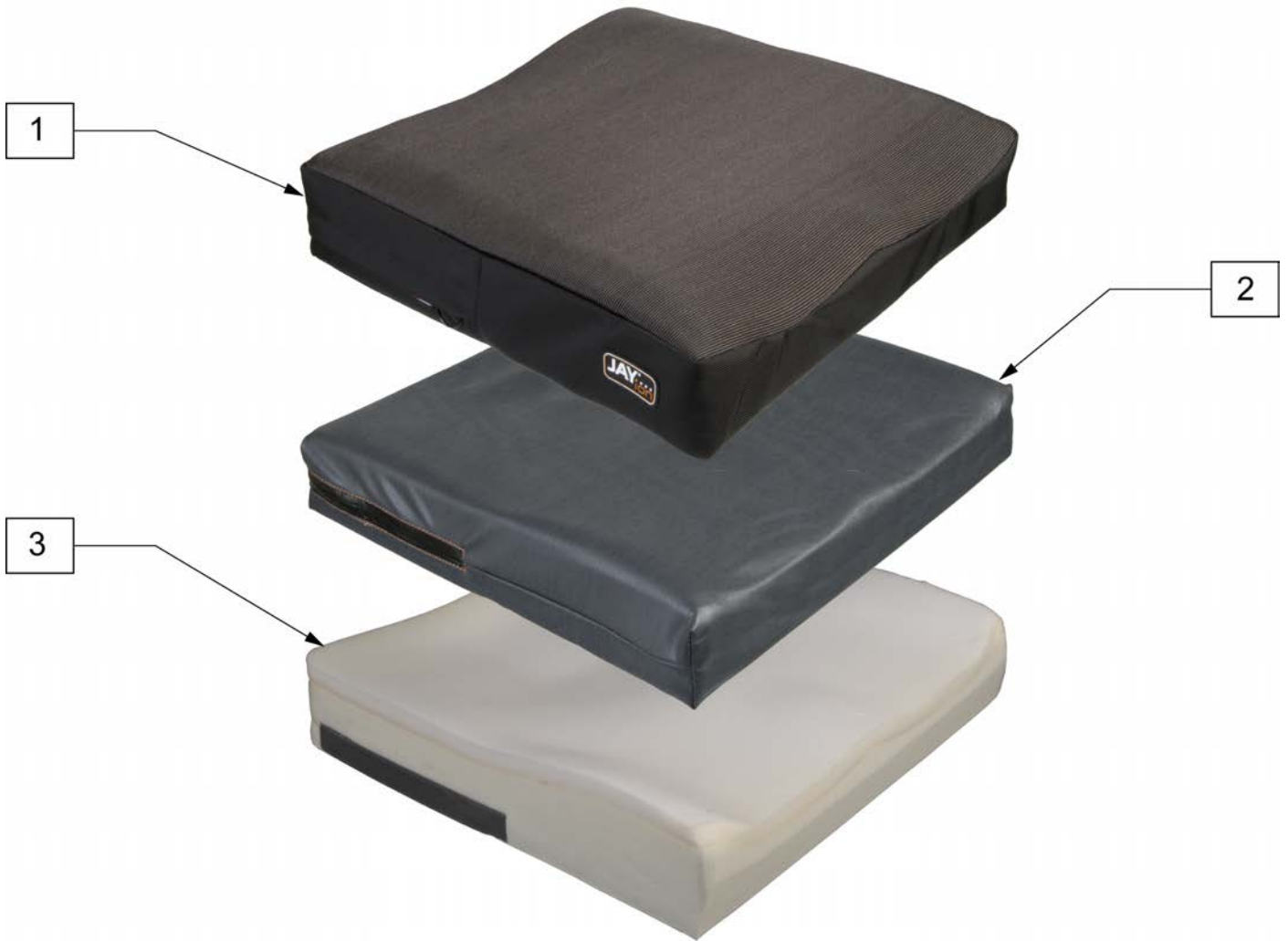


Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJI1414	COVER OUTER ION 14X14	1	CJI1620	COVER OUTER ION 16X20
1	CJI1415	COVER OUTER ION 14X15	1	CJI1621	COVER OUTER ION 16X21
1	CJI1416	COVER OUTER ION 14X16	1	CJI1622	COVER OUTER ION 16X22
1	CJI1417	COVER OUTER ION 14X17	1	CJI1714	COVER OUTER ION 17X14
1	CJI1418	COVER OUTER ION 14X18	1	CJI1715	COVER OUTER ION 17X15
1	CJI1514	COVER OUTER ION 15X14	1	CJI1716	COVER OUTER ION 17X16
1	CJI1515	COVER OUTER ION 15X15	1	CJI1717	COVER OUTER ION 17X17
1	CJI1516	COVER OUTER ION 15X16	1	CJI1718	COVER OUTER ION 17X18
1	CJI1517	COVER OUTER ION 15X17	1	CJI1719	COVER OUTER ION 17X19
1	CJI1518	COVER OUTER ION 15X18	1	CJI1720	COVER OUTER ION 17X20
1	CJI1519	COVER OUTER ION 15X19	1	CJI1721	COVER OUTER ION 17X21
1	CJI1520	COVER OUTER ION 15X20	1	CJI1722	COVER OUTER ION 17X22
1	CJI1521	COVER OUTER ION 15X21	1	CJI1723	COVER OUTER ION 17X23
1	CJI1614	COVER OUTER ION 16X14	1	CJI1814	COVER OUTER ION 18X14
1	CJI1615	COVER OUTER ION 16X15	1	CJI1815	COVER OUTER ION 18X15
1	CJI1616	COVER OUTER ION 16X16	1	CJI1816	COVER OUTER ION 18X16
1	CJI1617	COVER OUTER ION 16X17	1	CJI1817	COVER OUTER ION 18X17
1	CJI1618	COVER OUTER ION 16X18	1	CJI1818	COVER OUTER ION 18X18
1	CJI1619	COVER OUTER ION 16X19	1	CJI1819	COVER OUTER ION 18X19

Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJI1820	COVER OUTER ION 18X20	2	ICJI1418	COVER INNER ION 14X18
1	CJI1821	COVER OUTER ION 18X21	2	ICJI1514	COVER INNER ION 15X14
1	CJI1822	COVER OUTER ION 18X22	2	ICJI1515	COVER INNER ION 15X15
1	CJI1823	COVER OUTER ION 18X23	2	ICJI1516	COVER INNER ION 15X16
1	CJI1824	COVER OUTER ION 18X24	2	ICJI1517	COVER INNER ION 15X17
1	CJI1915	COVER OUTER ION 19X15	2	ICJI1518	COVER INNER ION 15X18
1	CJI1916	COVER OUTER ION 19X16	2	ICJI1519	COVER INNER ION 15X19
1	CJI1917	COVER OUTER ION 19X17	2	ICJI1520	COVER INNER ION 15X20
1	CJI1918	COVER OUTER ION 19X18	2	ICJI1521	COVER INNER ION 15X21
1	CJI1919	COVER OUTER ION 19X19	2	ICJI1614	COVER INNER ION 16X14
1	CJI1920	COVER OUTER ION 19X20	2	ICJI1615	COVER INNER ION 16X15
1	CJI1921	COVER OUTER ION 19X21	2	ICJI1616	COVER INNER ION 16X16
1	CJI1922	COVER OUTER ION 19X22	2	ICJI1617	COVER INNER ION 16X17
1	CJI1923	COVER OUTER ION 19X23	2	ICJI1618	COVER INNER ION 16X18
1	CJI1924	COVER OUTER ION 19X24	2	ICJI1619	COVER INNER ION 16X19
1	CJI2015	COVER OUTER ION 20X15	2	ICJI1620	COVER INNER ION 16X20
1	CJI2016	COVER OUTER ION 20X16	2	ICJI1621	COVER INNER ION 16X21
1	CJI2017	COVER OUTER ION 20X17	2	ICJI1622	COVER INNER ION 16X22
1	CJI2018	COVER OUTER ION 20X18	2	ICJI1714	COVER INNER ION 17X14
1	CJI2019	COVER OUTER ION 20X19	2	ICJI1715	COVER INNER ION 17X15
1	CJI2020	COVER OUTER ION 20X20	2	ICJI1716	COVER INNER ION 17X16
1	CJI2021	COVER OUTER ION 20X21	2	ICJI1717	COVER INNER ION 17X17
1	CJI2022	COVER OUTER ION 20X22	2	ICJI1718	COVER INNER ION 17X18
1	CJI2023	COVER OUTER ION 20X23	2	ICJI1719	COVER INNER ION 17X19
1	CJI2024	COVER OUTER ION 20X24	2	ICJI1720	COVER INNER ION 17X20
1	CJI2117	COVER OUTER ION 21X17	2	ICJI1721	COVER INNER ION 17X21
1	CJI2118	COVER OUTER ION 21X18	2	ICJI1722	COVER INNER ION 17X22
1	CJI2119	COVER OUTER ION 21X19	2	ICJI1723	COVER INNER ION 17X23
1	CJI2120	COVER OUTER ION 21X20	2	ICJI1814	COVER INNER ION 18X14
1	CJI2121	COVER OUTER ION 21X21	2	ICJI1815	COVER INNER ION 18X15
1	CJI2122	COVER OUTER ION 21X22	2	ICJI1816	COVER INNER ION 18X16
1	CJI2123	COVER OUTER ION 21X23	2	ICJI1817	COVER INNER ION 18X17
1	CJI2124	COVER OUTER ION 21X24	2	ICJI1818	COVER INNER ION 18X18
1	CJI2217	COVER OUTER ION 22X17	2	ICJI1819	COVER INNER ION 18X19
1	CJI2218	COVER OUTER ION 22X18	2	ICJI1820	COVER INNER ION 18X20
1	CJI2219	COVER OUTER ION 22X19	2	ICJI1821	COVER INNER ION 18X21
1	CJI2220	COVER OUTER ION 22X20	2	ICJI1822	COVER INNER ION 18X22
1	CJI2221	COVER OUTER ION 22X21	2	ICJI1823	COVER INNER ION 18X23
1	CJI2222	COVER OUTER ION 22X22	2	ICJI1824	COVER INNER ION 18X24
1	CJI2223	COVER OUTER ION 22X23	2	ICJI1915	COVER INNER ION 19X15
1	CJI2224	COVER OUTER ION 22X24	2	ICJI1916	COVER INNER ION 19X16
1	CJI2319	COVER OUTER ION 23X19	2	ICJI1917	COVER INNER ION 19X17
1	CJI2320	COVER OUTER ION 23X20	2	ICJI1918	COVER INNER ION 19X18
1	CJI2321	COVER OUTER ION 23X21	2	ICJI1919	COVER INNER ION 19X19
1	CJI2322	COVER OUTER ION 23X22	2	ICJI1920	COVER INNER ION 19X20
1	CJI2323	COVER OUTER ION 23X23	2	ICJI1921	COVER INNER ION 19X21
1	CJI2324	COVER OUTER ION 23X24	2	ICJI1922	COVER INNER ION 19X22
1	CJI2419	COVER OUTER ION 24X19	2	ICJI1923	COVER INNER ION 19X23
1	CJI2420	COVER OUTER ION 24X20	2	ICJI1924	COVER INNER ION 19X24
1	CJI2421	COVER OUTER ION 24X21	2	ICJI2015	COVER INNER ION 20X15
1	CJI2422	COVER OUTER ION 24X22	2	ICJI2016	COVER INNER ION 20X16
1	CJI2423	COVER OUTER ION 24X23	2	ICJI2017	COVER INNER ION 20X17
1	CJI2424	COVER OUTER ION 24X24	2	ICJI2018	COVER INNER ION 20X18
2	ICJI1414	COVER INNER ION 14X14	2	ICJI2019	COVER INNER ION 20X19
2	ICJI1415	COVER INNER ION 14X15	2	ICJI2020	COVER INNER ION 20X20
2	ICJI1416	COVER INNER ION 14X16	2	ICJI2021	COVER INNER ION 20X21
2	ICJI1417	COVER INNER ION 14X17	2	ICJI2022	COVER INNER ION 20X22

Pos.	Item Number	Description	Pos.	Item Number	Description
2	ICJI2023	COVER INNER ION 20X23	3	BJI1719	BASE FOAM JAY ION 17X19
2	ICJI2024	COVER INNER ION 20X24	3	BJI1720	BASE FOAM JAY ION 17X20
2	ICJI2117	COVER INNER ION 21X17	3	BJI1721	BASE FOAM JAY ION 17X21
2	ICJI2118	COVER INNER ION 21X18	3	BJI1722	BASE FOAM JAY ION 17X22
2	ICJI2119	COVER INNER ION 21X19	3	BJI1723	BASE FOAM JAY ION 17X23
2	ICJI2120	COVER INNER ION 21X20	3	BJI1814	BASE FOAM JAY ION 18X14
2	ICJI2121	COVER INNER ION 21X21	3	BJI1815	BASE FOAM JAY ION 18X15
2	ICJI2122	COVER INNER ION 21X22	3	BJI1816	BASE FOAM JAY ION 18X16
2	ICJI2123	COVER INNER ION 21X23	3	BJI1817	BASE FOAM JAY ION 18X17
2	ICJI2124	COVER INNER ION 21X24	3	BJI1818	BASE FOAM JAY ION 18X18
2	ICJI2217	COVER INNER ION 22X17	3	BJI1819	BASE FOAM JAY ION 18X19
2	ICJI2218	COVER INNER ION 22X18	3	BJI1820	BASE FOAM JAY ION 18X20
2	ICJI2219	COVER INNER ION 22X19	3	BJI1821	BASE FOAM JAY ION 18X21
2	ICJI2220	COVER INNER ION 22X20	3	BJI1822	BASE FOAM JAY ION 18X22
2	ICJI2221	COVER INNER ION 22X21	3	BJI1823	BASE FOAM JAY ION 18X23
2	ICJI2222	COVER INNER ION 22X22	3	BJI1824	BASE FOAM JAY ION 18X24
2	ICJI2223	COVER INNER ION 22X23	3	BJI1915	BASE FOAM JAY ION 19X15
2	ICJI2224	COVER INNER ION 22X24	3	BJI1916	BASE FOAM JAY ION 19X16
2	ICJI2319	COVER INNER ION 23X19	3	BJI1917	BASE FOAM JAY ION 19X17
2	ICJI2320	COVER INNER ION 23X20	3	BJI1918	BASE FOAM JAY ION 19X18
2	ICJI2321	COVER INNER ION 23X21	3	BJI1919	BASE FOAM JAY ION 19X19
2	ICJI2322	COVER INNER ION 23X22	3	BJI1920	BASE FOAM JAY ION 19X20
2	ICJI2323	COVER INNER ION 23X23	3	BJI1921	BASE FOAM JAY ION 19X21
2	ICJI2324	COVER INNER ION 23X24	3	BJI1922	BASE FOAM JAY ION 19X22
2	ICJI2419	COVER INNER ION 24X19	3	BJI1923	BASE FOAM JAY ION 19X23
2	ICJI2420	COVER INNER ION 24X20	3	BJI1924	BASE FOAM JAY ION 19X24
2	ICJI2421	COVER INNER ION 24X21	3	BJI2015	BASE FOAM JAY ION 20X15
2	ICJI2422	COVER INNER ION 24X22	3	BJI2016	BASE FOAM JAY ION 20X16
2	ICJI2423	COVER INNER ION 24X23	3	BJI2017	BASE FOAM JAY ION 20X17
2	ICJI2424	COVER INNER ION 24X24	3	BJI2018	BASE FOAM JAY ION 20X18
3	BJI1414	BASE FOAM JAY ION 14X14	3	BJI2019	BASE FOAM JAY ION 20X19
3	BJI1415	BASE FOAM JAY ION 14X15	3	BJI2020	BASE FOAM JAY ION 20X20
3	BJI1416	BASE FOAM JAY ION 14X16	3	BJI2021	BASE FOAM JAY ION 20X21
3	BJI1417	BASE FOAM JAY ION 14X17	3	BJI2022	BASE FOAM JAY ION 20X22
3	BJI1418	BASE FOAM JAY ION 14X18	3	BJI2023	BASE FOAM JAY ION 20X23
3	BJI1514	BASE FOAM JAY ION 15X14	3	BJI2024	BASE FOAM JAY ION 20X24
3	BJI1515	BASE FOAM JAY ION 15X15	3	BJI2117	BASE FOAM JAY ION 21X17
3	BJI1516	BASE FOAM JAY ION 15X16	3	BJI2118	BASE FOAM JAY ION 21X18
3	BJI1517	BASE FOAM JAY ION 15X17	3	BJI2119	BASE FOAM JAY ION 21X19
3	BJI1518	BASE FOAM JAY ION 15X18	3	BJI2120	BASE FOAM JAY ION 21X20
3	BJI1519	BASE FOAM JAY ION 15X19	3	BJI2121	BASE FOAM JAY ION 21X21
3	BJI1520	BASE FOAM JAY ION 15X20	3	BJI2122	BASE FOAM JAY ION 21X22
3	BJI1521	BASE FOAM JAY ION 15X21	3	BJI2123	BASE FOAM JAY ION 21X23
3	BJI1614	BASE FOAM JAY ION 16X14	3	BJI2124	BASE FOAM JAY ION 21X24
3	BJI1615	BASE FOAM JAY ION 16X15	3	BJI2217	BASE FOAM JAY ION 22X17
3	BJI1616	BASE FOAM JAY ION 16X16	3	BJI2218	BASE FOAM JAY ION 22X18
3	BJI1617	BASE FOAM JAY ION 16X17	3	BJI2219	BASE FOAM JAY ION 22X19
3	BJI1618	BASE FOAM JAY ION 16X18	3	BJI2220	BASE FOAM JAY ION 22X20
3	BJI1619	BASE FOAM JAY ION 16X19	3	BJI2221	BASE FOAM JAY ION 22X21
3	BJI1620	BASE FOAM JAY ION 16X20	3	BJI2222	BASE FOAM JAY ION 22X22
3	BJI1621	BASE FOAM JAY ION 16X21	3	BJI2223	BASE FOAM JAY ION 22X23
3	BJI1622	BASE FOAM JAY ION 16X22	3	BJI2224	BASE FOAM JAY ION 22X24
3	BJI1714	BASE FOAM JAY ION 17X14	3	BJI2319	BASE FOAM JAY ION 23X19
3	BJI1715	BASE FOAM JAY ION 17X15	3	BJI2320	BASE FOAM JAY ION 23X20
3	BJI1716	BASE FOAM JAY ION 17X16	3	BJI2321	BASE FOAM JAY ION 23X21
3	BJI1717	BASE FOAM JAY ION 17X17	3	BJI2322	BASE FOAM JAY ION 23X22
3	BJI1718	BASE FOAM JAY ION 17X18	3	BJI2323	BASE FOAM JAY ION 23X23



Pos.	Item Number	Description	Pos.	Item Number	Description
3	BJI2324	BASE FOAM JAY ION 23X24	3	BJI2422	BASE FOAM JAY ION 24X22
3	BJI2419	BASE FOAM JAY ION 24X19	3	BJI2423	BASE FOAM JAY ION 24X23
3	BJI2420	BASE FOAM JAY ION 24X20	3	BJI2424	BASE FOAM JAY ION 24X24
3	BJI2421	BASE FOAM JAY ION 24X21			

NOTE: SACRAL SEAL HELPS PREVENT FLUID FROM MIGRATING TO THE REAR OF THE FLUID PAD WHEN INDIVIDUALS ARE IN RECLINED OR POSTERIOR PELVIC TILT POSITION.

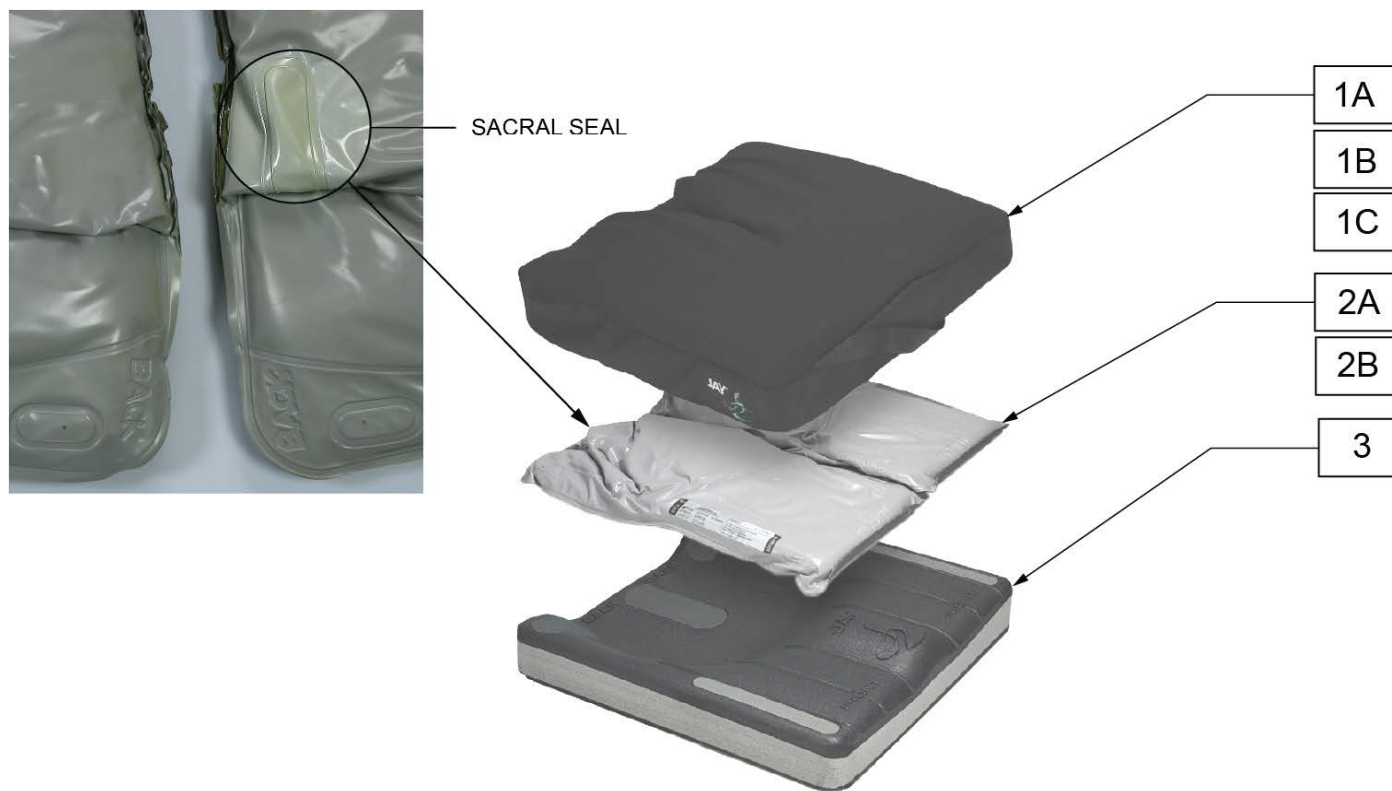

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	C2100-MJ	COVER J2 18 x 16		1A	C2118A	COVER J2 AIR EXCHANGE 20 x 18	
1	C2101	COVER J2 18 x 18		1B	C2100N	COVER J2 INCONTINENT 18 x 16	
1	C2102	COVER J2 20 x20		1B	C2101N	COVER J2 INCONTINENT 18 x 18	
1	C2103	COVER J2 18 x 20		1B	C2102N	COVER J2 INCONTINENT 20 x20	
1	C2104	COVER J2 24 x 20		1B	C2103N	COVER J2 INCONTINENT 18 x 20	
1	C2105	COVER J2 15.5 x 16		1B	C2104N	COVER J2 INCONTINENT 24 x 20	
1	C2106	COVER J2 15.5 x 18		1B	C2105N	COVER J2 INCONTINENT 15.5 x 16	
1	C2107	COVER J2 15.5 x 20		1B	C2106N	COVER J2 INCONTINENT 15.5 x 18	
1	C2108	COVER J2 14 x 16		1B	C2107N	COVER J2 INCONTINENT 15.5 x 20	
1	C2116	COVER J2 20 x 16		1B	C2108N	COVER J2 INCONTINEN T14 x 16	
1	C2117	COVER J2 17 x 17		1B	C2116N	COVER J2 INCONTINENT 20 x 16	
1	C2118	COVER J2 20 x 18		1B	C2117N	COVER J2 INCONTINENT 17 x 17	
1A	C2100A	COVER J2 AIR EXCHANGE 18 x 16		1B	C2118N	COVER J2 INCONTINENT 20 x 18	
1A	C2101A	COVER J2 AIR EXCHANGE 18 x 18		2	F2100	FLUID PAD J2 18 x 16	
1A	C2102A	COVER J2 AIR EXCHANGE 20 x20		2	F2101	FLUID PAD J2 18 x 18	
1A	C2103A	COVER J2 AIR EXCHANGE 18 x 20		2	F2102	FLUID PAD J2 20 x20	
1A	C2104A	COVER J2 AIR EXCHANGE 24 x 20		2	F2103	FLUID PAD J2 18 x 20	
1A	C2105A	COVER J2 AIR EXCHANGE 15.5 x 16		2	F2104	FLUID PAD J2 24 x 20	
1A	C2106A	COVER J2 AIR EXCHANGE 15.5 x 18		2	F2105	FLUID PAD J2 15.5 x 16	
1A	C2107A	COVER J2 AIR EXCHANGE 15.5 x 20		2	F2106	FLUID PAD J2 15.5 x 18	
1A	C2108A	COVER J2 AIR EXCHANGE 14 x 16		2	F2107	FLUID PAD J2 15.5 x 20	
1A	C2116A	COVER J2 AIR EXCHANGE 20 x 16		2	F2108	FLUID PAD J2 14 x 16	
1A	C2117A	COVER J2 AIR EXCHANGE 17 x 17		2	F2116	FLUID PAD J2 20 x 16	

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
2	F2117	FLUID PAD J2 17 x 17		2A	F2118.1	FLUID PAD J2 RECLINE 20 x 18	*
2	F2118	FLUID PAD J2 20 x 18		3	B2100	BASE J2 18 x 16	
2A	F2100.1	FLUID PAD J2 RECLINE 18 x 16	*	3	B2101	BASE J2 18 x 18	
2A	F2101.1	FLUID PAD J2 RECLINE 18 x 18	*	3	B2102	BASE J2 20 x20	
2A	F2102.1	FLUID PAD J2 RECLINE 20 x20	*	3	B2103	BASE J2 18 x 20	
2A	F2103.1	FLUID PAD J2 RECLINE 18 x 20	*	3	B2104	BASE J2 24 x 20	
2A	F2104.1	FLUID PAD J2 RECLINE 24 x 20	*	3	B2105	BASE J2 15.5 x 16	
2A	F2105.1	FLUID PAD J2 RECLINE 15.5 x 16	*	3	B2106	BASE J2 15.5 x 18	
2A	F2106.1	FLUID PAD J2 RECLINE 15.5 x 18	*	3	B2107	BASE J2 15.5 x 20	
2A	F2107.1	FLUID PAD J2 RECLINE 15.5 x 20	*	3	B2108	BASE J2 14 x 16	
2A	F2108.1	FLUID PAD J2 RECLINE 14 x 16	*	3	B2116	BASE J2 20 x 16	
2A	F2116.1	FLUID PAD J2 RECLINE 20 x 16	*	3	B2117	BASE J2 17 x 17	
2A	F2117.1	FLUID PAD J2 RECLINE 17 x 17	*	3	B2118	BASE J2 20 x 18	

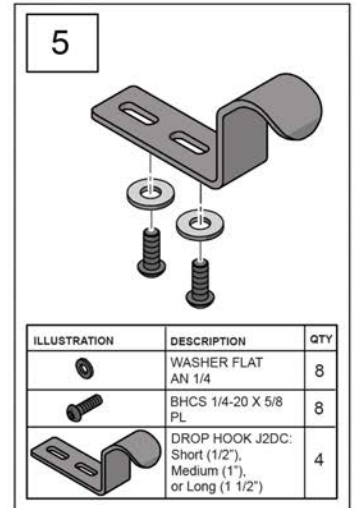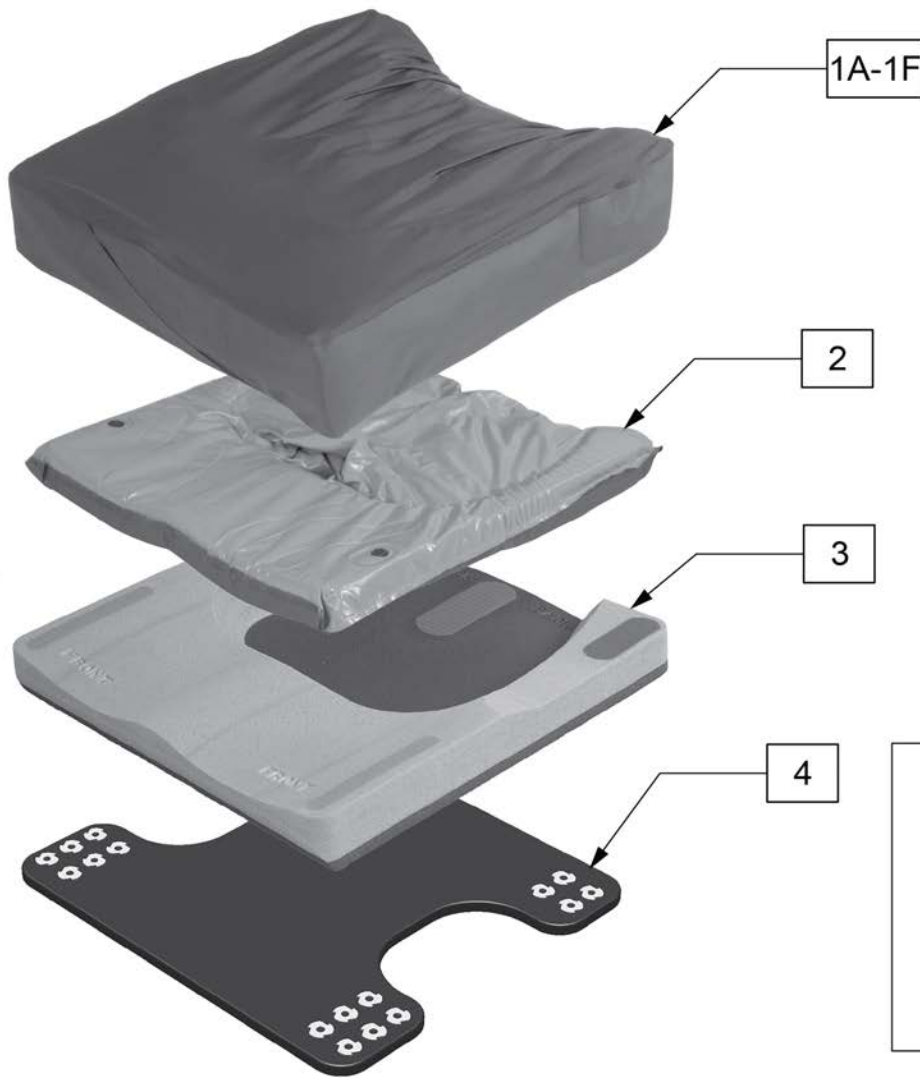

Pos.	Item Number	Description	Remarks
1A	C2444	COVER J2 DEEP CONTUR 14 x 14	Zippered Cover
1A	C2446	COVER J2 DEEP CONTUR 14 x 16	Zippered Cover
1A	C2455	COVER J2 DEEP CONTUR 15 x 15	Zippered Cover
1A	C2457	COVER J2 DEEP CONTUR 15 x 17	Zippered Cover
1A	C2466	COVER J2 DEEP CONTUR 16 x 16	Zippered Cover
1A	C2468	COVER J2 DEEP CONTUR 16 x 18	Zippered Cover
1A	C2475	COVER J2 DEEP CONTUR 17 x 15	Zippered Cover
1A	C2477	COVER J2 DEEP CONTUR 17 x 17	Zippered Cover
1A	C2486	COVER J2 DEEP CONTUR 18 x 16	Zippered Cover
1A	C2488	COVER J2 DEEP CONTUR 18 x 18	Zippered Cover
1A	C2406	COVER J2 DEEP CONTUR 20 x 16	Zippered Cover
1A	C2408	COVER J2 DEEP CONTUR 20 x 18	Zippered Cover
1A	C2426	COVER J2 DEEP CONTUR 22 x 16	Zippered Cover
1A	C2428	COVER J2 DEEP CONTUR 22 x 18	Zippered Cover
1A	C2460	COVER J2 DEEP CONTUR 16 x 20	Zippered Cover
1A	C2480	COVER J2 DEEP CONTUR 18 x 20	Zippered Cover
1A	C2400	COVER J2 DEEP CONTUR 20 x 20	Zippered Cover
1A	C2448	COVER J2 DEEP CONTUR 24 x 18	Zippered Cover

Pos.	Item Number	Description	Remarks
1B	C2444A	COVER J2 DEEP CONTOUR AIR EXCHANGE 14 x 14	Zippered Cover
1B	C2446A	COVER J2 DEEP CONTOUR AIR EXCHANGE 14 x 16	Zippered Cover
1B	C2455A	COVER J2 DEEP CONTOUR AIR EXCHANGE 15 x 15	Zippered Cover
1B	C2457A	COVER J2 DEEP CONTOUR AIR EXCHANGE 15 x 17	Zippered Cover
1B	C2466A	COVER J2 DEEP CONTOUR AIR EXCHANGE 16 x 16	Zippered Cover
1B	C2468A	COVER J2 DEEP CONTOUR AIR EXCHANGE 16 x 18	Zippered Cover
1B	C2475A	COVER J2 DEEP CONTOUR AIR EXCHANGE 17 x 15	Zippered Cover
1B	C2477A	COVER J2 DEEP CONTOUR AIR EXCHANGE 17 x 17	Zippered Cover
1B	C2486A	COVER J2 DEEP CONTOUR AIR EXCHANGE 18 x 16	Zippered Cover
1B	C2488A	COVER J2 DEEP CONTOUR AIR EXCHANGE 18 x 18	Zippered Cover
1B	C2406A	COVER J2 DEEP CONTOUR AIR EXCHANGE 20 x 16	Zippered Cover
1B	C2408A	COVER J2 DEEP CONTOUR AIR EXCHANGE 20 x 18	Zippered Cover
1B	C2426A	COVER J2 DEEP CONTOUR AIR EXCHANGE 22 x 16	Zippered Cover
1B	C2428A	COVER J2 DEEP CONTOUR AIR EXCHANGE 22 x 18	Zippered Cover
1B	C2460A	COVER J2 DEEP CONTOUR AIR EXCHANGE 16 x 20	Zippered Cover
1B	C2480A	COVER J2 DEEP CONTOUR AIR EXCHANGE 18 x 20	Zippered Cover
1B	C2400A	COVER J2 DEEP CONTOUR AIR EXCHANGE 20 x 20	Zippered Cover
1B	C2448A	COVER J2 DEEP CONTOUR AIR EXCHANGE 24 x 18	Zippered Cover
1C	C2444N	COVER J2 DEEP CONTOUR INCONTINENT 14 x 14	Zippered Cover
1C	C2446N	COVER J2 DEEP CONTOUR INCONTINENT 14 x 16	Zippered Cover
1C	C2455N	COVER J2 DEEP CONTOUR INCONTINENT 15 x 15	Zippered Cover
1C	C2457N	COVER J2 DEEP CONTOUR INCONTINENT 15 x 17	Zippered Cover
1C	C2466N	COVER J2 DEEP CONTOUR INCONTINENT 16 x 16	Zippered Cover
1C	C2468N	COVER J2 DEEP CONTOUR INCONTINENT 16 x 18	Zippered Cover
1C	C2475N	COVER J2 DEEP CONTOUR INCONTINENT 17 x 15	Zippered Cover
1C	C2477N	COVER J2 DEEP CONTOUR INCONTINENT 17 x 17	Zippered Cover
1C	C2486N	COVER J2 DEEP CONTOUR INCONTINENT 18 x 16	Zippered Cover
1C	C2488N	COVER J2 DEEP CONTOUR INCONTINENT 18 x 18	Zippered Cover
1C	C2406N	COVER J2 DEEP CONTOUR INCONTINENT 20 x 16	Zippered Cover
1C	C2408N	COVER J2 DEEP CONTOUR INCONTINENT 20 x 18	Zippered Cover
1C	C2426N	COVER J2 DEEP CONTOUR INCONTINENT 22 x 16	Zippered Cover
1C	C2428N	COVER J2 DEEP CONTOUR INCONTINENT 22 x 18	Zippered Cover
1C	C2460N	COVER J2 DEEP CONTOUR INCONTINENT 16 x 20	Zippered Cover
1C	C2480N	COVER J2 DEEP CONTOUR INCONTINENT 18 x 20	Zippered Cover
1C	C2400N	COVER J2 DEEP CONTOUR INCONTINENT 20 x 20	Zippered Cover
1C	C2448N	COVER J2 DEEP CONTOUR INCONTINENT 24 x 18	Zippered Cover
1D	C2444D	COVER J2 DEEP CONTOUR WITH DROP SEAT 14 x 14	Non-zippered Cover - Std Stretch
1D	C2446D	COVER J2 DEEP CONTOUR WITH DROP SEAT 14 x 16	Non-zippered Cover - Std Stretch
1D	C2455D	COVER J2 DEEP CONTOUR WITH DROP SEAT 15 x 15	Non-zippered Cover - Std Stretch
1D	C2457D	COVER J2 DEEP CONTOUR WITH DROP SEAT 15 x 17	Non-zippered Cover - Std Stretch
1D	C2466D	COVER J2 DEEP CONTOUR WITH DROP SEAT 16 x 16	Non-zippered Cover - Std Stretch
1D	C2468D	COVER J2 DEEP CONTOUR WITH DROP SEAT 16 x 18	Non-zippered Cover - Std Stretch
1D	C2475D	COVER J2 DEEP CONTOUR WITH DROP SEAT 17 x 15	Non-zippered Cover - Std Stretch
1D	C2477D	COVER J2 DEEP CONTOUR WITH DROP SEAT 17 x 17	Non-zippered Cover - Std Stretch
1D	C2486D	COVER J2 DEEP CONTOUR WITH DROP SEAT 18 x 16	Non-zippered Cover - Std Stretch
1D	C2488D	COVER J2 DEEP CONTOUR WITH DROP SEAT 18 x 18	Non-zippered Cover - Std Stretch
1D	C2406D	COVER J2 DEEP CONTOUR WITH DROP SEAT 20 x 16	Non-zippered Cover - Std Stretch
1D	C2408D	COVER J2 DEEP CONTOUR WITH DROP SEAT 20 x 18	Non-zippered Cover - Std Stretch
1D	C2426D	COVER J2 DEEP CONTOUR WITH DROP SEAT 22 x 16	Non-zippered Cover - Std Stretch
1D	C2428D	COVER J2 DEEP CONTOUR WITH DROP SEAT 22 x 18	Non-zippered Cover - Std Stretch
1D	C2460D	COVER J2 DEEP CONTOUR WITH DROP SEAT 16 x 20	Non-zippered Cover - Std Stretch
1D	C2480D	COVER J2 DEEP CONTOUR WITH DROP SEAT 18 x 20	Non-zippered Cover - Std Stretch
1D	C2400D	COVER J2 DEEP CONTOUR WITH DROP SEAT 20 x 20	Non-zippered Cover - Std Stretch
1D	C2448D	COVER J2 DEEP CONTOUR WITH DROP SEAT 24 x 18	Non-zippered Cover - Std Stretch
1E	C2444AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 14 x 14	Non-Zippered Cover
1E	C2446AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 14 x 16	Non-Zippered Cover
1E	C2455AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 15 x 15	Non-Zippered Cover


Pos.	Item Number	Description	Remarks
1E	C2457AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 15 x 17	Non-Zippered Cover
1E	C2466AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 16 x 16	Non-Zippered Cover
1E	C2468AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 16 x 18	Non-Zippered Cover
1E	C2475AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 17 x 15	Non-Zippered Cover
1E	C2477AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 17 x 17	Non-Zippered Cover
1E	C2486AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 18 x 16	Non-Zippered Cover
1E	C2488AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 18 x 18	Non-Zippered Cover
1E	C2406AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 20 x 16	Non-Zippered Cover
1E	C2408AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 20 x 18	Non-Zippered Cover
1E	C2426AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 22 x 16	Non-Zippered Cover
1E	C2428AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 22 x 18	Non-Zippered Cover
1E	C2460AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 16 x 20	Non-Zippered Cover
1E	C2480AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 18 x 20	Non-Zippered Cover
1E	C2400AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 20 x 20	Non-Zippered Cover
1E	C2448AD	COVER J2 DEEP CONTOUR AIR EXCHANGE WITH DROP SEAT 24 x 18	Non-Zippered Cover
1F	C2444ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 14 x 14	Non-Zippered Cover
1F	C2446ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 14 x 16	Non-Zippered Cover
1F	C2455ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 15 x 15	Non-Zippered Cover
1F	C2457ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 15 x 17	Non-Zippered Cover
1F	C2466ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 16 x 16	Non-Zippered Cover
1F	C2468ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 16 x 18	Non-Zippered Cover
1F	C2475ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 17 x 15	Non-Zippered Cover
1F	C2477ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 17 x 17	Non-Zippered Cover
1F	C2486ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 18 x 16	Non-Zippered Cover
1F	C2488ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 18 x 18	Non-Zippered Cover
1F	C2406ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 20 x 16	Non-Zippered Cover
1F	C2408ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 20 x 18	Non-Zippered Cover
1F	C2426ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 22 x 16	Non-Zippered Cover
1F	C2428ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 22 x 18	Non-Zippered Cover
1F	C2460ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 16 x 20	Non-Zippered Cover
1F	C2480ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 18 x 20	Non-Zippered Cover
1F	C2400ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 20 x 20	Non-Zippered Cover
1F	C2448ND	COVER J2 DEEP CONTOUR INCONTINENT WITH DROP SEAT 24 x 18	Non-Zippered Cover
2	F2444	FLUID PAD J2 DEEP CONTOUR 14 x 14	
2	F2446	FLUID PAD J2 DEEP CONTOUR 14 x 16	
2	F2455	FLUID PAD J2 DEEP CONTOUR 15 x 15	
2	F2457	FLUID PAD J2 DEEP CONTOUR 15 x 17	
2	F2466	FLUID PAD J2 DEEP CONTOUR 16 x 16	
2	F2468	FLUID PAD J2 DEEP CONTOUR 16 x 18	
2	F2475	FLUID PAD J2 DEEP CONTOUR 17 x 15	
2	F2477	FLUID PAD J2 DEEP CONTOUR 17 x 17	
2	F2486	FLUID PAD J2 DEEP CONTOUR 18 x 16	
2	F2488	FLUID PAD J2 DEEP CONTOUR 18 x 18	
2	F2406	FLUID PAD J2 DEEP CONTOUR 20 x 16	
2	F2408	FLUID PAD J2 DEEP CONTOUR 20 x 18	
2	F2426	FLUID PAD J2 DEEP CONTOUR 22 x 16	
2	F2428	FLUID PAD J2 DEEP CONTOUR 22 x 18	
2	F2460	FLUID PAD J2 DEEP CONTOUR 16 x 20	
2	F2480	FLUID PAD J2 DEEP CONTOUR 18 x 20	
2	F2400	FLUID PAD J2 DEEP CONTOUR 20 x 20	
2	F2448	FLUID PAD J2 DEEP CONTOUR 24 x 18	
3	B2444	BASE J2 DEEP CONTOUR 14 x 14	
3	B2446	BASE J2 DEEP CONTOUR 14 x 16	
3	B2455	BASE J2 DEEP CONTOUR 15 x 15	
3	B2457	BASE J2 DEEP CONTOUR 15 x 17	
3	B2466	BASE J2 DEEP CONTOUR 16 x 16	
3	B2468	BASE J2 DEEP CONTOUR 16 x 18	

Pos.	Item Number	Description	Remarks
3	B2475	BASE J2 DEEP CONTOUR 17 x 15	
3	B2477	BASE J2 DEEP CONTOUR 17 x 17	
3	B2486	BASE J2 DEEP CONTOUR 18 x 16	
3	B2488	BASE J2 DEEP CONTOUR 18 x 18	
3	B2406	BASE J2 DEEP CONTOUR 20 x 16	
3	B2408	BASE J2 DEEP CONTOUR 20 x 18	
3	B2426	BASE J2 DEEP CONTOUR 22 x 16	
3	B2428	BASE J2 DEEP CONTOUR 22 x 18	
3	B2460	BASE J2 DEEP CONTOUR 16 x 20	
3	B2480	BASE J2 DEEP CONTOUR 18 x 20	
3	B2400	BASE J2 DEEP CONTOUR 20 x 20	
3	B2448	BASE J2 DEEP CONTOUR 24 x 18	
5	M-24S	SHORT 1/2" DROP HOOKS	Pack of 4
5	M-24M	MEDIUM 1" DROP HOOKS	Pack of 4
5	M-24L	LONG 1 1/2" DROP HOOKS	Pack of 4
4-6	SD2444	SOLID DROP SEAT WITH HARDWARE 14 x 14	Ships standard w/ 1 1/2' drop hooks
4-6	SD2446	SOLID DROP SEAT WITH HARDWARE 14 x 16	Ships standard w/ 1 1/2' drop hooks
4-6	SD2455	SOLID DROP SEAT WITH HARDWARE 15 x 15	Ships standard w/ 1 1/2' drop hooks
4-6	SD2457	SOLID DROP SEAT WITH HARDWARE 15 x 17	Ships standard w/ 1 1/2' drop hooks
4-6	SD2466	SOLID DROP SEAT WITH HARDWARE 16 x 16	Ships standard w/ 1 1/2' drop hooks
4-6	SD2468	SOLID DROP SEAT WITH HARDWARE 16 x 18	Ships standard w/ 1 1/2' drop hooks
4-6	SD2475	SOLID DROP SEAT WITH HARDWARE 17 x 15	Ships standard w/ 1 1/2' drop hooks
4-6	SD2477	SOLID DROP SEAT WITH HARDWARE 17 x 17	Ships standard w/ 1 1/2' drop hooks
4-6	SD2486	SOLID DROP SEAT WITH HARDWARE 18 x 16	Ships standard w/ 1 1/2' drop hooks
4-6	SD2488	SOLID DROP SEAT WITH HARDWARE 18 x 18	Ships standard w/ 1 1/2' drop hooks
4-6	SD2406	SOLID DROP SEAT WITH HARDWARE 20 x 16	Ships standard w/ 1 1/2' drop hooks
4-6	SD2408	SOLID DROP SEAT WITH HARDWARE 20 x 18	Ships standard w/ 1 1/2' drop hooks
4-6	SD2426	SOLID DROP SEAT WITH HARDWARE 22 x 16	Ships standard w/ 1 1/2' drop hooks
4-6	SD2428	SOLID DROP SEAT WITH HARDWARE 22 x 18	Ships standard w/ 1 1/2' drop hooks
4-6	SD2460	SOLID DROP SEAT WITH HARDWARE 16 x 20	Ships standard w/ 1 1/2' drop hooks
4-6	SD2480	SOLID DROP SEAT WITH HARDWARE 18 x 20	Ships standard w/ 1 1/2' drop hooks
4-6	SD2400	SOLID DROP SEAT WITH HARDWARE 20 x 20	Ships standard w/ 1 1/2' drop hooks
4-6	SD2448	SOLID DROP SEAT WITH HARDWARE 24 x 18	Ships standard w/ 1 1/2' drop hooks
6	M-2404H	HANDLE KIT J2 DEEP CONTOUR CUSHION 14"	
6	M-2405H	HANDLE KIT J2 DEEP CONTOUR CUSHION 15"	
6	M-2406H	HANDLE KIT J2 DEEP CONTOUR CUSHION 16"	
6	M-2407H	HANDLE KIT J2 DEEP CONTOUR CUSHION 17"	
6	M-2408H	HANDLE KIT J2 DEEP CONTOUR CUSHION 18"	
6	M-2409H	HANDLE KIT J2 DEEP CONTOUR CUSHION 20"	
6	M-2410H	HANDLE KIT J2 DEEP CONTOUR CUSHION 22"	
6	M-2411H	HANDLE KIT J2 DEEP CONTOUR CUSHION 24"	
3,4,5, 6	B2444	BASE J2 DEEP CONTOUR WITH DROP SEAT 14 x 14	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2446	BASE J2 DEEP CONTOUR WITH DROP SEAT 14 x 16	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2455	BASE J2 DEEP CONTOUR WITH DROP SEAT 15 x 15	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2457	BASE J2 DEEP CONTOUR WITH DROP SEAT 15 x 17	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2466	BASE J2 DEEP CONTOUR WITH DROP SEAT 16 x 16	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2468	BASE J2 DEEP CONTOUR WITH DROP SEAT 16 x 18	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2475	BASE J2 DEEP CONTOUR WITH DROP SEAT 17 x 15	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2477	BASE J2 DEEP CONTOUR WITH DROP SEAT 17 x 17	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2486	BASE J2 DEEP CONTOUR WITH DROP SEAT 18 x 16	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2488	BASE J2 DEEP CONTOUR WITH DROP SEAT 18 x 18	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2406	BASE J2 DEEP CONTOUR WITH DROP SEAT 20 x 16	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2408	BASE J2 DEEP CONTOUR WITH DROP SEAT 20 x 18	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2426	BASE J2 DEEP CONTOUR WITH DROP SEAT 22 x 16	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2428	BASE J2 DEEP CONTOUR WITH DROP SEAT 22 x 18	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2460	BASE J2 DEEP CONTOUR WITH DROP SEAT 16 x 20	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2480	BASE J2 DEEP CONTOUR WITH DROP SEAT 18 x 20	Ships standard w/ 1 1/2' drop hooks

Pos.	Item Number	Description	Remarks
3,4,5, 6	B2400	BASE J2 DEEP CONTOUR WITH DROP SEAT 20 x 20	Ships standard w/ 1 1/2' drop hooks
3,4,5, 6	B2448	BASE J2 DEEP CONTOUR WITH DROP SEAT 24 x 18	Ships standard w/ 1 1/2' drop hooks

NOTE: ships with full set of 6 foam pieces.

NOTE: effective May 1, 2016, odd sizes and custom foam kit complete cushions will no longer be available; parts will continue to be offered.

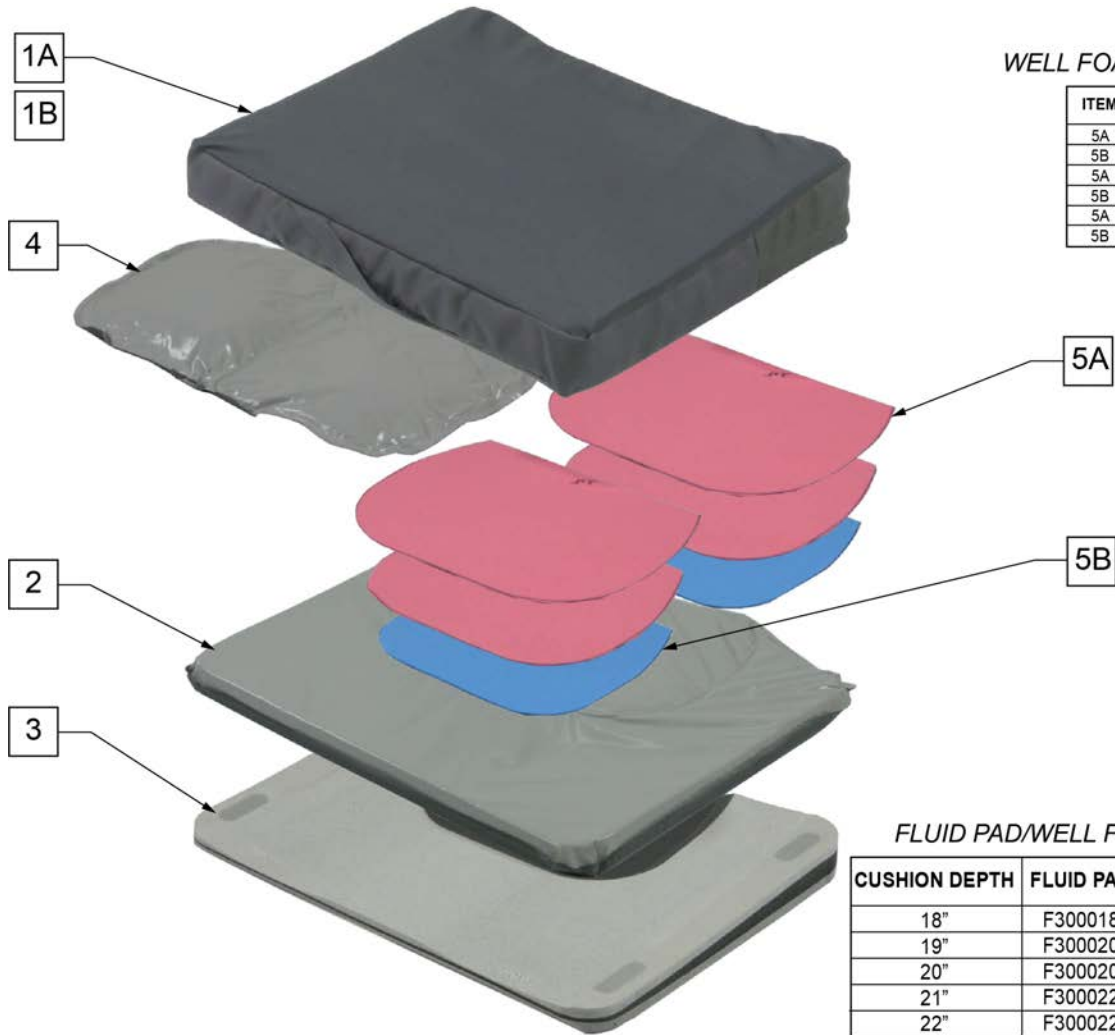

WELL FOAM KIT DENSITY GUIDE

ITEM	LAYER	FOAM DENSITY
5A	BOTTOM	MEDIUM
5B	BOTTOM	FIRM
5A	MIDDLE	MED/SOFT
5B	MIDDLE	MEDIUM
5A	TOP	MED/SOFT
5B	TOP	SOFT

FLUID PAD/WELL FOAM INSERT GUIDE

CUSHION DEPTH	FLUID PAD	STANDARD WELL FOAM	CUSTOM WELL FOAM
18"	F300018	300018-MJ	300018-MJ
19"	F300020	300020-MJ	300020-MJ
20"	F300020	300020-MJ	300020-MJ
21"	F300022	300022-MJ	300022-MJ
22"	F300022	300022-MJ	300022-MJ

Pos.	Item Number	Description
1A	C302018A	J2 PLUS CUSHION COVER AIR EXCHANGE 20 x 18
1A	C302019A	J2 PLUS CUSHION COVER AIR EXCHANGE 20 x 19
1A	C302020A	J2 PLUS CUSHION COVER AIR EXCHANGE 20 x 20
1A	C302021A	J2 PLUS CUSHION COVER AIR EXCHANGE 20 x 21
1A	C302022A	J2 PLUS CUSHION COVER AIR EXCHANGE 20 x 22
1A	C302118A	J2 PLUS CUSHION COVER AIR EXCHANGE 21 x 18
1A	C302119A	J2 PLUS CUSHION COVER AIR EXCHANGE 21 x 19
1A	C302120A	J2 PLUS CUSHION COVER AIR EXCHANGE 21 x 20
1A	C302121A	J2 PLUS CUSHION COVER AIR EXCHANGE 21 x 21
1A	C302122A	J2 PLUS CUSHION COVER AIR EXCHANGE 21 x 22
1A	C302218A	J2 PLUS CUSHION COVER AIR EXCHANGE 22 x 18
1A	C302219A	J2 PLUS CUSHION COVER AIR EXCHANGE 22 x 19

Pos.	Item Number	Description
1A	C302220A	J2 PLUS CUSHION COVER AIR EXCHANGE 22 x 20
1A	C302221A	J2 PLUS CUSHION COVER AIR EXCHANGE 22 x 21
1A	C302222A	J2 PLUS CUSHION COVER AIR EXCHANGE 22 x 22
1A	C302318A	J2 PLUS CUSHION COVER AIR EXCHANGE 23 x 18
1A	C302319A	J2 PLUS CUSHION COVER AIR EXCHANGE 23 x 19
1A	C302320A	J2 PLUS CUSHION COVER AIR EXCHANGE 23 x 20
1A	C302321A	J2 PLUS CUSHION COVER AIR EXCHANGE 23 x 21
1A	C302322A	J2 PLUS CUSHION COVER AIR EXCHANGE 23 x 22
1A	C302418A	J2 PLUS CUSHION COVER AIR EXCHANGE 24 x 18
1A	C302419A	J2 PLUS CUSHION COVER AIR EXCHANGE 24 x 19
1A	C302420A	J2 PLUS CUSHION COVER AIR EXCHANGE 24 x 20
1A	C302421A	J2 PLUS CUSHION COVER AIR EXCHANGE 24 x 21
1A	C302422A	J2 PLUS CUSHION COVER AIR EXCHANGE 24 x 22
1A	C302518A	J2 PLUS CUSHION COVER AIR EXCHANGE 25 x 18
1A	C302519A	J2 PLUS CUSHION COVER AIR EXCHANGE 25 x 19
1A	C302520A	J2 PLUS CUSHION COVER AIR EXCHANGE 25 x 20
1A	C302521A	J2 PLUS CUSHION COVER AIR EXCHANGE 25 x 21
1A	C302522A	J2 PLUS CUSHION COVER AIR EXCHANGE 25 x 22
1A	C302618A	J2 PLUS CUSHION COVER AIR EXCHANGE 26 x 18
1A	C302619A	J2 PLUS CUSHION COVER AIR EXCHANGE 26 x 19
1A	C302620A	J2 PLUS CUSHION COVER AIR EXCHANGE 26 x 20
1A	C302621A	J2 PLUS CUSHION COVER AIR EXCHANGE 26 x 21
1A	C302622A	J2 PLUS CUSHION COVER AIR EXCHANGE 26 x 22
1B	C302018LS	J2 PLUS CUSHION COVER LOW SHEAR 20 x 18
1B	C302019LS	J2 PLUS CUSHION COVER LOW SHEAR 20 x 19
1B	C302020LS	J2 PLUS CUSHION COVER LOW SHEAR 20 x 20
1B	C302021LS	J2 PLUS CUSHION COVER LOW SHEAR 20 x 21
1B	C302022LS	J2 PLUS CUSHION COVER LOW SHEAR 20 x 22
1B	C302118LS	J2 PLUS CUSHION COVER LOW SHEAR 21 x 18
1B	C302119LS	J2 PLUS CUSHION COVER LOW SHEAR 21 x 19
1B	C302120LS	J2 PLUS CUSHION COVER LOW SHEAR 21 x 20
1B	C302121LS	J2 PLUS CUSHION COVER LOW SHEAR 21 x 21
1B	C302122LS	J2 PLUS CUSHION COVER LOW SHEAR 21 x 22
1B	C302218LS	J2 PLUS CUSHION COVER LOW SHEAR 22 x 18
1B	C302219LS	J2 PLUS CUSHION COVER LOW SHEAR 22 x 19
1B	C302220LS	J2 PLUS CUSHION COVER LOW SHEAR 22 x 20
1B	C302221LS	J2 PLUS CUSHION COVER LOW SHEAR 22 x 21
1B	C302222LS	J2 PLUS CUSHION COVER LOW SHEAR 22 x 22
1B	C302318LS	J2 PLUS CUSHION COVER LOW SHEAR 23 x 18
1B	C302319LS	J2 PLUS CUSHION COVER LOW SHEAR 23 x 19
1B	C302320LS	J2 PLUS CUSHION COVER LOW SHEAR 23 x 20
1B	C302321LS	J2 PLUS CUSHION COVER LOW SHEAR 23 x 21
1B	C302322LS	J2 PLUS CUSHION COVER LOW SHEAR 23 x 22
1B	C302418LS	J2 PLUS CUSHION COVER LOW SHEAR 24 x 18
1B	C302419LS	J2 PLUS CUSHION COVER LOW SHEAR 24 x 19
1B	C302420LS	J2 PLUS CUSHION COVER LOW SHEAR 24 x 20
1B	C302421LS	J2 PLUS CUSHION COVER LOW SHEAR 24 x 21
1B	C302422LS	J2 PLUS CUSHION COVER LOW SHEAR 24 x 22
1B	C302518LS	J2 PLUS CUSHION COVER LOW SHEAR 25 x 18
1B	C302519LS	J2 PLUS CUSHION COVER LOW SHEAR 25 x 19
1B	C302520LS	J2 PLUS CUSHION COVER LOW SHEAR 25 x 20
1B	C302521LS	J2 PLUS CUSHION COVER LOW SHEAR 25 x 21
1B	C302522LS	J2 PLUS CUSHION COVER LOW SHEAR 25 x 22
1B	C302618LS	J2 PLUS CUSHION COVER LOW SHEAR 26 x 18
1B	C302619LS	J2 PLUS CUSHION COVER LOW SHEAR 26 x 19
1B	C302620LS	J2 PLUS CUSHION COVER LOW SHEAR 26 x 20
1B	C302621LS	J2 PLUS CUSHION COVER LOW SHEAR 26 x 21

Pos.	Item Number	Description
1B	C302622LS	J2 PLUS CUSHION COVER LOW SHEAR 26 x 22
2	E302018	ENVELOPE J2 PLUS 20 x 18
2	E302019	ENVELOPE J2 PLUS 20 x 19
2	E302020	ENVELOPE J2 PLUS 20 x 20
2	E302021	ENVELOPE J2 PLUS 20 x 21
2	E302022	ENVELOPE J2 PLUS 20 x 22
2	E302118	ENVELOPE J2 PLUS 21 x 18
2	E302119	ENVELOPE J2 PLUS 21 x 19
2	E302120	ENVELOPE J2 PLUS 21 x 20
2	E302121	ENVELOPE J2 PLUS 21 x 21
2	E302122	ENVELOPE J2 PLUS 21 x 22
2	E302218	ENVELOPE J2 PLUS 22 x 18
2	E302219	ENVELOPE J2 PLUS 22 x 19
2	E302220	ENVELOPE J2 PLUS 22 x 20
2	E302221	ENVELOPE J2 PLUS 22 x 21
2	E302222	ENVELOPE J2 PLUS 22 x 22
2	E302318	ENVELOPE J2 PLUS 23 x 18
2	E302319	ENVELOPE J2 PLUS 23 x 19
2	E302320	ENVELOPE J2 PLUS 23 x 20
2	E302321	ENVELOPE J2 PLUS 23 x 21
2	E302322	ENVELOPE J2 PLUS 23 x 22
2	E302418	ENVELOPE J2 PLUS 24 x 18
2	E302419	ENVELOPE J2 PLUS 24 x 19
2	E302420	ENVELOPE J2 PLUS 24 x 20
2	E302421	ENVELOPE J2 PLUS 24 x 21
2	E302422	ENVELOPE J2 PLUS 24 x 22
2	E302518	ENVELOPE J2 PLUS 25 x 18
2	E302519	ENVELOPE J2 PLUS 25 x 19
2	E302520	ENVELOPE J2 PLUS 25 x 20
2	E302521	ENVELOPE J2 PLUS 25 x 21
2	E302522	ENVELOPE J2 PLUS 25 x 22
2	E302618	ENVELOPE J2 PLUS 26 x 18
2	E302619	ENVELOPE J2 PLUS 26 x 19
2	E302620	ENVELOPE J2 PLUS 26 x 20
2	E302621	ENVELOPE J2 PLUS 26 x 21
2	E302622	ENVELOPE J2 PLUS 26 x 22
3	B302018	BASE FOAM J2 PLUS 20 x 18
3	B302019	BASE FOAM J2 PLUS 20 x 19
3	B302020	BASE FOAM J2 PLUS 20 x 20
3	B302021	BASE FOAM J2 PLUS 20 x 21
3	B302022	BASE FOAM J2 PLUS 20 x 22
3	B302118	BASE FOAM J2 PLUS 21 x 18
3	B302119	BASE FOAM J2 PLUS 21 x 19
3	B302120	BASE FOAM J2 PLUS 21 x 20
3	B302121	BASE FOAM J2 PLUS 21 x 21
3	B302122	BASE FOAM J2 PLUS 21 x 22
3	B302218	BASE FOAM J2 PLUS 22 x 18
3	B302219	BASE FOAM J2 PLUS 22 x 19
3	B302220	BASE FOAM J2 PLUS 22 x 20
3	B302221	BASE FOAM J2 PLUS 22 x 21
3	B302222	BASE FOAM J2 PLUS 22 x 22
3	B302318	BASE FOAM J2 PLUS 23 x 18
3	B302319	BASE FOAM J2 PLUS 23 x 19
3	B302320	BASE FOAM J2 PLUS 23 x 20
3	B302321	BASE FOAM J2 PLUS 23 x 21
3	B302322	BASE FOAM J2 PLUS 23 x 22
3	B302418	BASE FOAM J2 PLUS 24 x 18

Pos.	Item Number	Description
3	B302419	BASE FOAM J2 PLUS 24 x 19
3	B302420	BASE FOAM J2 PLUS 24 x 20
3	B302421	BASE FOAM J2 PLUS 24 x 21
3	B302422	BASE FOAM J2 PLUS 24 x 22
3	B302518	BASE FOAM J2 PLUS 25 x 18
3	B302519	BASE FOAM J2 PLUS 25 x 19
3	B302520	BASE FOAM J2 PLUS 25 x 20
3	B302521	BASE FOAM J2 PLUS 25 x 21
3	B302522	BASE FOAM J2 PLUS 25 x 22
3	B302618	BASE FOAM J2 PLUS 26 x 18
3	B302619	BASE FOAM J2 PLUS 26 x 19
3	B302620	BASE FOAM J2 PLUS 26 x 20
3	B302621	BASE FOAM J2 PLUS 26 x 21
3	B302622	BASE FOAM J2 PLUS 26 x 22
4	F300018	FLUID PAD INSERT J2 PLUS 18"
4	F300020	FLUID PAD INSERT J2 PLUS 20"
4	F300022	FLUID PAD INSERT J2 PLUS 22"
5	300018-MJ	WELL FOAM KIT J2 PLUS 18"
5	300020-MJ	WELL FOAM KIT J2 PLUS 20"
5	300022-MJ	WELL FOAM KIT J2 PLUS 22"
5A	300118-MJ	WELL FOAM KIT J2 PLUS 18" C-KIT
5A	300120-MJ	WELL FOAM KIT J2 PLUS 20" C-KIT
5A	300122-MJ	WELL FOAM KIT J2 PLUS 22" C-KIT

NOTE: 1 WHEN REDUCING THE P.L.A. SIZE, A NEW P.L.A. INSERT MUST BE ORDERED SEPARATELY IN THE MATCHING P.L.A. SIZE.

NOTE: 2. 12 THRU 15 WIDTH CUSHIONS ARE STANDARD WITH SIZE "A" P.L.A. RELIEF AND CAN NOT BE REDUCED.

NOTE: 3. 16 THRU 19 WIDTH CUSHIONS ARE STANDARD WITH SIZE "B" P.L.A. RELIEF AND CAN BE REDUCED TO SIZE "A" ONLY.

NOTE: 4. 20 THRU 24 WIDTH CUSHIONS ARE STANDARD WITH SIZE "C" P.L.A. RELIEF AND CAN BE REDUCED TO SIZE "B" OR SIZE "A".

NOTE: 5. WHEN CHANGING THE P.L.A. SIZE FROM "A" TO "C" OR FROM "C" TO "A" A NEW COVER MUST BE ORDERED SEPERATELY.

NOTE: 6. WHEN CHANGING THE P.L.A. SIZE FROM "A" TO "B", FROM "B" TO "A", FROM "B" TO "C", OR FROM "C" TO "B" A NEW COVER IS NOT REQUIRED.

NOTE: 7. SIZE "A" FIELD VARIABLE P.L.A. INSERT COMES WITH QTY (2) MEDIUM SUPPLEMENT FLUID PADS.

NOTE: 8. SIZE "B" FIELD VARIABLE P.L.A. INSERT COMES WITH QTY (2) LARGE SUPPLEMENT FLUID PADS.

NOTE: 9. SIZE "C" FIELD VARIABLE P.L.A. INSERT COMES WITH QTY (2) XLARGE SUPPLEMENT FLUID PADS.

NOTE: 10. 12 THRU 15 WIDTH REPLACEMENT BASE'S WILL ALWAYS SHIP AS SIZE "A" P.L.A. RELIEF.

NOTE: 11. 16 THRU 19 WIDTH REPLACEMENT BASE'S WILL SHIP STANDARD WITH SIZE "B" P.L.A. RELIEF AND WILL INCLUDE THE SIZE "A" P.L.A. REDUCER RING IF SIZE "A" IS ORDERED.

NOTE: 12. 20 THRU 24 WIDTH REPLACEMENT BASE'S WILL SHIP STANDARD WITH SIZE "C" P.L.A. RELIEF AND WILL INCLUDE THE SIZE "B" OR "A" P.L.A. REDUCER RING IF SIZE "B" OR "A" IS ORDERED.



Pos.	Item Number	Description	Remarks
1A	CJ3C	J3 CUSHION COVER	You will be prompted for Height, Width, Depth, P.L.A. Size & Cover Type
1B	MCJ3C	MODIFIED J3 CUSHION COVER	Order Modified Cushion Cover's Thru Customer Service ONLY.
2A	BJ3C	J3 CUSHION BASE	SEE NOTES 10-12 ABOVE - You will be prompted for Height, Width, Depth, & P.L.A. Size
2B	MBJ3C	MODIFIED J3 CUSHION BASE	Order Modified Cushion Base's Thru Customer Service ONLY.
3	J3CARSCA	STD P.L.A REDUCER C-A	SEE NOTES 1-6,10-12 ABOVE
3	J3CARSCB	STD P.L.A REDUCER C-B	SEE NOTES 1-6,10-12 ABOVE
3	J3CARSB A	STD P.L.A REDUCER B-A	SEE NOTES 1-6,10-12 ABOVE
3	J3CARDCA	DEEP P.L.A REDUCER C-A	SEE NOTES 1-6,10-12 ABOVE
3	J3CARDCB	DEEP P.L.A REDUCER C-B	SEE NOTES 1-6,10-12 ABOVE
3	J3CARDBA	DEEP P.L.A REDUCER B-A	SEE NOTES 1-6,10-12 ABOVE
4A	FJ3C	J3 P.L.A. INSERT BLADDER	SEE NOTES 7-9 ABOVE - You will be prompted for Height, P.L.A. size, and P.L.A. Fill Type.
4B	AJ3C	J3 AIR P.L.A. INSERT	You will be prompted for height,PLA size and single or dual valve
5	J3CAF SM	FIELD ADJ SUPPL SM	Small Supplement Fluid Pad; stamped w/ "S" on bottom of pad
5	J3CAF MD	FIELD ADJ SUPPL MED	Medium Supplement Fluid Pad; stamped w/ "M" on bottom of pad
5	J3CAF LG	FIELD ADJ SUPPL LG	Large Supplement Fluid Pad; stamped w/ "L" on bottom of pad
5	J3CAF XL	FIELD ADJ SUPPL XL	X-Large Supplement Fluid Pad; stamped w/ "X" on bottom of pad
6	PATCHKIT	AIR PATCH KIT	
7	AIRPUMP	AIR PUMP	

NOTE: 1 WHEN REDUCING THE P.L.A. SIZE, A NEW P.L.A. INSERT MUST BE ORDERED SEPARATELY IN THE MATCHING P.L.A. SIZE.

NOTE: 2. 12 THRU 15 WIDTH CUSHIONS ARE STANDARD WITH SIZE "A" P.L.A. RELIEF AND CAN NOT BE REDUCED.

NOTE: 3. 16 THRU 19 WIDTH CUSHIONS ARE STANDARD WITH SIZE "B" P.L.A. RELIEF AND CAN BE REDUCED TO SIZE "A" ONLY.

NOTE: 4. 20 THRU 24 WIDTH CUSHIONS ARE STANDARD WITH SIZE "C" P.L.A. RELIEF AND CAN BE REDUCED TO SIZE "B" OR SIZE "A".

NOTE: 5. WHEN CHANGING THE P.L.A. SIZE FROM "A" TO "C" OR FROM "C" TO "A" A NEW COVER MUST BE ORDERED SEPERATELY.

NOTE: 6. WHEN CHANGING THE P.L.A. SIZE FROM "A" TO "B", FROM "B" TO "A", FROM "B" TO "C", OR FROM "C" TO "B" A NEW COVER IS NOT REQUIRED.

NOTE: 7. SIZE "A" FIELD VARIABLE P.L.A. INSERT COMES WITH QTY (2) MEDIUM SUPPLEMENT FLUID PADS.

NOTE: 8. SIZE "B" FIELD VARIABLE P.L.A. INSERT COMES WITH QTY (2) LARGE SUPPLEMENT FLUID PADS.

NOTE: 9. SIZE "C" FIELD VARIABLE P.L.A. INSERT COMES WITH QTY (2) XLARGE SUPPLEMENT FLUID PADS.

NOTE: 10. 12 THRU 15 WIDTH REPLACEMENT BASE'S WILL ALWAYS SHIP AS SIZE "A" P.L.A. RELIEF.

NOTE: 11. 16 THRU 19 WIDTH REPLACEMENT BASE'S WILL SHIP STANDARD WITH SIZE "B" P.L.A. RELIEF AND WILL INCLUDE THE SIZE "A" P.L.A. REDUCER RING IF SIZE "A" IS ORDERED.

NOTE: 12. 20 THRU 24 WIDTH REPLACEMENT BASE'S WILL SHIP STANDARD WITH SIZE "C" P.L.A. RELIEF AND WILL INCLUDE THE SIZE "B" OR "A" P.L.A. REDUCER RING IF SIZE "B" OR "A" IS ORDERED.

[08/22/2018]









Pos.	Item Number	Description	Remarks
1A	CJ3CP	J3 CUSHION COVER	You will be prompted for Height, Width, Depth, P.L.A. Size & Cover Type
1B	MCJ3CP	MODIFIED J3 CUSHION COVER	Order Modified Cushion Cover's Thru Customer Service ONLY.
2A	BJ3CP	J3 CUSHION BASE	SEE NOTES 10-12 ABOVE - You will be prompted for Height, Width, Depth, & P.L.A. Size
2B	MBJ3CP	MODIFIED J3 CUSHION BASE	Order Modified Cushion Base's Thru Customer Service ONLY.
3	J3CARSCA	STD P.L.A REDUCER C-A	SEE NOTES 1-6,10-12 ABOVE
3	J3CARSCB	STD P.L.A REDUCER C-B	SEE NOTES 1-6,10-12 ABOVE
3	J3CARSCBA	STD P.L.A REDUCER B-A	SEE NOTES 1-6,10-12 ABOVE
3	J3CARDCA	DEEP P.L.A REDUCER C-A	SEE NOTES 1-6,10-12 ABOVE
3	J3CARDCB	DEEP P.L.A REDUCER C-B	SEE NOTES 1-6,10-12 ABOVE
3	J3CARDBA	DEEP P.L.A REDUCER B-A	SEE NOTES 1-6,10-12 ABOVE

Pos.	Item Number	Description	Remarks
4A	FJ3C	J3 P.L.A. INSERT BLADDER	SEE NOTES 7-9 ABOVE - You will be prompted for Height, P.L.A. size, and P.L.A. Fill Type.
4B	AJ3C	J3 AIR P.L.A. INSERT	You will be prompted for height, PLA size and single or dual valve
5	J3CAF5M	FIELD ADJ SUPPL SM	Small Supplement Fluid Pad; stamped w/ "S" on bottom of pad
5	J3CAF5M	FIELD ADJ SUPPL MED	Medium Supplement Fluid Pad; stamped w/ "M" on bottom of pad
5	J3CAF5L	FIELD ADJ SUPPL LG	Large Supplement Fluid Pad; stamped w/ "L" on bottom of pad
5	J3CAF5XL	FIELD ADJ SUPPL XL	X-Large Supplement Fluid Pad; stamped w/ "X" on bottom of pad
6	J3CAPS2	LATPEL 2"H (SMALL)	Pair- Used on 12 deep to 18 deep
6	J3CAPM2	LATPEL 2"H (MEDIUM)	Pair - Used on 19 deep to 24 deep
7	J3CALS2	LATTHI 2"H (CUSHIONS12W-14W)	Pair- Used on 12 deep to 16 deep
7	J3CALM2	LATTHI 2"H (CUSHIONS16W-19W)	Pair - Used on 17 deep to 19 deep
7	J3CALL2	LATTHI 2"H (CUSHIONS20W-24W)	Pair - Used on 20 deep to 24 deep
8	J3CAMS2	MEDTHI 2"H (CUSHIONS12W-14W)	Used on 12 deep to 16 deep
8	J3CAMP2	MEDTHI 2"H (CUSHIONS16W-19W)	Used on 17 deep to 19 deep
8	J3CAML2	MEDTHI 2"H (CUSHIONS20W-24W)	Used on 20 deep to 24 deep
9	PATCHKIT	AIR PATCH KIT	
10	AIRPUMP	AIR PUMP	
11	JSS0808	JAY SOLID SEAT 8" X 8"	
11	JSS0809	JAY SOLID SEAT 8" X 9"	
11	JSS0810	JAY SOLID SEAT 8" X 10"	
11	JSS0811	JAY SOLID SEAT 8" X 11"	
11	JSS0908	JAY SOLID SEAT 9" X 8"	
11	JSS0909	JAY SOLID SEAT 9" X 9"	
11	JSS0910	JAY SOLID SEAT 9" X 10"	
11	JSS0911	JAY SOLID SEAT 9" X 11"	
11	JSS0912	JAY SOLID SEAT 9" X 12"	
11	JSS1008	JAY SOLID SEAT 10" X 8"	
11	JSS1009	JAY SOLID SEAT 10" X 9"	
11	JSS1010	JAY SOLID SEAT 10" X 10"	
11	JSS1011	JAY SOLID SEAT 10" X 11"	
11	JSS1012	JAY SOLID SEAT 10" X 12"	
11	JSS1013	JAY SOLID SEAT 10" X 13"	
11	JSS1108	JAY SOLID SEAT 11" X 8"	
11	JSS1109	JAY SOLID SEAT 11" X 9"	
11	JSS1110	JAY SOLID SEAT 11" X 10"	
11	JSS1111	JAY SOLID SEAT 11" X 11"	
11	JSS1112	JAY SOLID SEAT 11" X 12"	
11	JSS1113	JAY SOLID SEAT 11" X 13"	
11	JSS1114	JAY SOLID SEAT 11" X 14"	
11	JSS1209	JAY SOLID SEAT 12" X 9"	
11	JSS1210	JAY SOLID SEAT 12" X 10"	
11	JSS1211	JAY SOLID SEAT 12" X 11"	
11	JSS1212	JAY SOLID SEAT 12" X 12"	
11	JSS1213	JAY SOLID SEAT 12" X 13"	
11	JSS1214	JAY SOLID SEAT 12" X 14"	
11	JSS1215	JAY SOLID SEAT 12" X 15"	
11	JSS1216	JAY SOLID SEAT 12" X 16"	
11	JSS1310	JAY SOLID SEAT 13" X 10"	
11	JSS1311	JAY SOLID SEAT 13" X 11"	
11	JSS1312	JAY SOLID SEAT 13" X 12"	
11	JSS1313	JAY SOLID SEAT 13" X 13"	
11	JSS1314	JAY SOLID SEAT 13" X 14"	

Pos.	Item Number	Description	Remarks
11	JSS1315	JAY SOLID SEAT 13" X 15"	
11	JSS1316	JAY SOLID SEAT 13" X 16"	
11	JSS1317	JAY SOLID SEAT 13" X 17"	
11	JSS1411	JAY SOLID SEAT 14" X 11"	
11	JSS1412	JAY SOLID SEAT 14" X 12"	
11	JSS1413	JAY SOLID SEAT 14" X 13"	
11	JSS1414	JAY SOLID SEAT 14" X 14"	
11	JSS1415	JAY SOLID SEAT 14" X 15"	
11	JSS1416	JAY SOLID SEAT 14" X 16"	
11	JSS1417	JAY SOLID SEAT 14" X 17"	
11	JSS1418	JAY SOLID SEAT 14" X 18"	
11	JSS1512	JAY SOLID SEAT 15" X 12"	
11	JSS1513	JAY SOLID SEAT 15" X 13"	
11	JSS1514	JAY SOLID SEAT 15" X 14"	
11	JSS1515	JAY SOLID SEAT 15" X 15"	
11	JSS1516	JAY SOLID SEAT 15" X 16"	
11	JSS1517	JAY SOLID SEAT 15" X 17"	
11	JSS1518	JAY SOLID SEAT 15" X 18"	
11	JSS1519	JAY SOLID SEAT 15" X 19"	
11	JSS1520	JAY SOLID SEAT 15" X 20"	
11	JSS1612	JAY SOLID SEAT 16" X 12"	
11	JSS1613	JAY SOLID SEAT 16" X 13"	
11	JSS1614	JAY SOLID SEAT 16" X 14"	
11	JSS1615	JAY SOLID SEAT 16" X 15"	
11	JSS1616	JAY SOLID SEAT 16" X 16"	
11	JSS1617	JAY SOLID SEAT 16" X 17"	
11	JSS1618	JAY SOLID SEAT 16" X 18"	
11	JSS1619	JAY SOLID SEAT 16" X 19"	
11	JSS1620	JAY SOLID SEAT 16" X 20"	
11	JSS1713	JAY SOLID SEAT 17" X 13"	
11	JSS1714	JAY SOLID SEAT 17" X 14"	
11	JSS1715	JAY SOLID SEAT 17" X 15"	
11	JSS1716	JAY SOLID SEAT 17" X 16"	
11	JSS1717	JAY SOLID SEAT 17" X 17"	
11	JSS1718	JAY SOLID SEAT 17" X 18"	
11	JSS1719	JAY SOLID SEAT 17" X 19"	
11	JSS1720	JAY SOLID SEAT 17" X 20"	
11	JSS1814	JAY SOLID SEAT 18" X 14"	
11	JSS1815	JAY SOLID SEAT 18" X 15"	
11	JSS1816	JAY SOLID SEAT 18" X 16"	
11	JSS1817	JAY SOLID SEAT 18" X 17"	
11	JSS1818	JAY SOLID SEAT 18" X 18"	
11	JSS1819	JAY SOLID SEAT 18" X 19"	
11	JSS1820	JAY SOLID SEAT 18" X 20"	
11	JSS1822	JAY SOLID SEAT 18" X 22"	
11	JSS1824	JAY SOLID SEAT 18" X 24"	
11	JSS1826	JAY SOLID SEAT 18" X 26"	
11	JSS1915	JAY SOLID SEAT 19" X 15"	
11	JSS1916	JAY SOLID SEAT 19" X 16"	
11	JSS1917	JAY SOLID SEAT 19" X 17"	
11	JSS1918	JAY SOLID SEAT 19" X 18"	
11	JSS1919	JAY SOLID SEAT 19" X 19"	
11	JSS1920	JAY SOLID SEAT 19" X 20"	
11	JSS2015	JAY SOLID SEAT 20" X 15"	
11	JSS2016	JAY SOLID SEAT 20" X 16"	
11	JSS2017	JAY SOLID SEAT 20" X 17"	
11	JSS2018	JAY SOLID SEAT 20" X 18"	

Pos.	Item Number	Description	Remarks
11	JSS2019	JAY SOLID SEAT 20" X 19"	
11	JSS2020	JAY SOLID SEAT 20" X 20"	
11	JSS2022	JAY SOLID SEAT 20" X 22"	
11	JSS2024	JAY SOLID SEAT 20" X 24"	
11	JSS2026	JAY SOLID SEAT 20" X 26"	
11	JSS2216	JAY SOLID SEAT 22" X 16"	
11	JSS2218	JAY SOLID SEAT 22" X 18"	
11	JSS2220	JAY SOLID SEAT 22" X 20"	
11	JSS2222	JAY SOLID SEAT 22" X 22"	
11	JSS2224	JAY SOLID SEAT 22" X 24"	
11	JSS2226	JAY SOLID SEAT 22" X 26"	
11	JSS2418	JAY SOLID SEAT 24" X 18"	
11	JSS2420	JAY SOLID SEAT 24" X 20"	
11	JSS2422	JAY SOLID SEAT 24" X 22"	
11	JSS2424	JAY SOLID SEAT 24" X 24"	
11	JSS2426	JAY SOLID SEAT 24" X 26"	
11	JSS2618	JAY SOLID SEAT 26" X 18"	
11	JSS2620	JAY SOLID SEAT 26" X 20"	
11	JSS2622	JAY SOLID SEAT 26" X 22"	
11	JSS2624	JAY SOLID SEAT 26" X 24"	

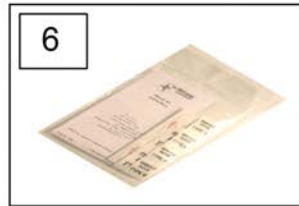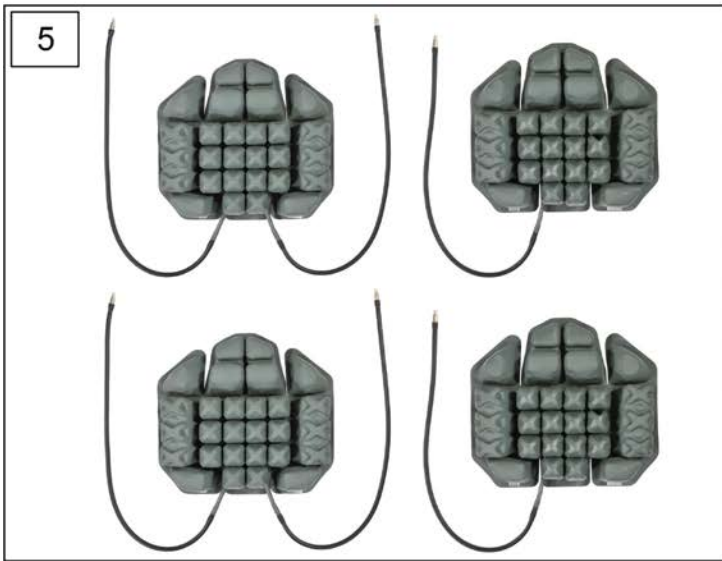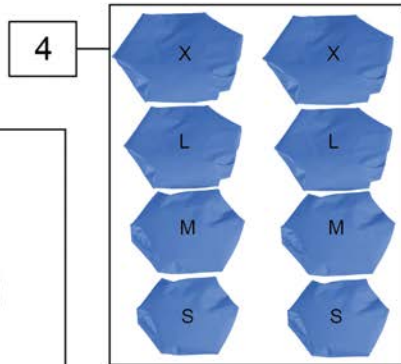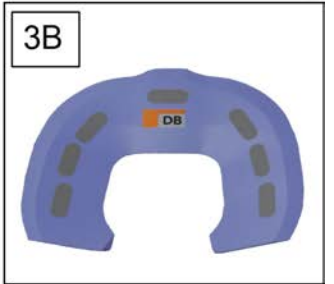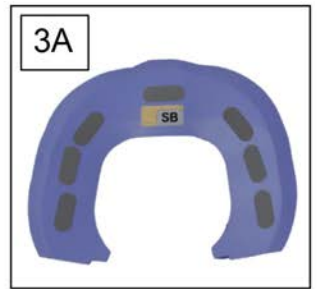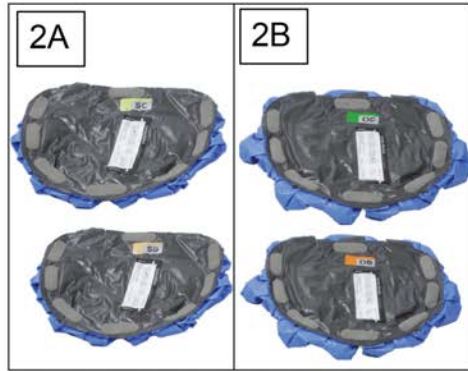

Pos.	Item Number	Description
1,2A,3A,4-7	J3CEKS	J3 CUSH STD PLA EVAL KIT
1,2B,3B,4-7	J3CEKD	J3 CUSH DP PLA EVAL KIT




Pos.	Item Number	Description	Remarks
2	J3CALM1	LATERAL THIGH 1" HEIGHT MEDIUM	8.25" LONG
2	J3CALL1	LATERAL THIGH 1" HEIGHT LARGE	11" LONG
2	J3CALS2	LATERAL THIGH 2" HEIGHT SMALL	5.5" LONG
2	J3CALM2	LATERAL THIGH 2" HEIGHT MEDIUM	8.25" LONG
2	J3CALL2	LATERAL THIGH 2" HEIGHT LARGE	11" LONG
3	J3CAMP1	MED THIGH 1" HEIGHT MEDIUM	6" LONG
3	J3CAML1	MED THIGH 1" HEIGHT LARGE	8.5" LONG
3	J3CAMS2	MED THIGH 2" HEIGHT SMALL	3.5" LONG
3	J3CAMP2	MED THIGH 2" HEIGHT MEDIUM	6" LONG
3	J3CAML2	MED THIGH 2" HEIGHT LARGE	8.5" LONG
4	J3CAPS2	LAT PEL 2" HEIGHT SMALL	4.75" LONG
4	J3CAPM2	LAT PEL 2" HEIGHT MEDIUM	6.25" LONG
5	J3CSAOR	OBLIQUITY PAD STD A RIGHT	1/2" in height
5	J3CSAOL	OBLIQUITY PAD STD A LEFT	1/2" in height
5	J3CSBOR	OBLIQUITY PAD STD B RIGHT	1/2" in height
5	J3CSBOL	OBLIQUITY PAD STD B LEFT	1/2" in height
5	J3CSCOR	OBLIQUITY PAD STD C RIGHT	1/2" in height
5	J3CSCOL	OBLIQUITY PAD STD C LEFT	1/2" in height
5	J3CDAOR	OBLIQUITY PAD DP A RIGHT	1/2" in height
5	J3CDAOL	OBLIQUITY PAD DP A LEFT	1/2" in height
5	J3CDBOR	OBLIQUITY PAD DP B RIGHT	1/2" in height
5	J3CDBOL	OBLIQUITY PAD DP B LEFT	1/2" in height
5	J3CDCOR	OBLIQUITY PAD DP C RIGHT	1/2" in height
5	J3CDCOL	OBLIQUITY PAD DP C LEFT	1/2" in height
6	J3CPRT	REPLACEMENT TAPE 30' ROLL	
1-6	J3CPEK	J3 CUSHION POSITIONING EVAL KIT	



Pos.	Item Number	Description	Remarks
>>>>		CUSHION COVERS	
N/A	109422-xxx	J3 CUSHION COVER	Please sell "CJ3C" or "CJ3CP"
N/A	109423-xxx	J3 CUSHION COVER	Please sell "CJ3C" or "CJ3CP"
N/A	109424-xxx	J3 CUSHION COVER	Please sell "CJ3C" or "CJ3CP"
N/A	109425-xxx	J3 CUSHION COVER	Please sell "CJ3C" or "CJ3CP"
N/A	109426-xxx	J3 CUSHION COVER	Please sell "CJ3C" or "CJ3CP"
N/A	109427-xxx	J3 CUSHION COVER	Please sell "CJ3C" or "CJ3CP"
>>>>		FLUID P.L.A. INSERT	
N/A	J3CSAFF	J3 P.L.A. INSERT FLUID BLADDER	Please sell "FJ3C"
N/A	J3CSBFF	J3 P.L.A. INSERT FLUID BLADDER	Please sell "FJ3C"
N/A	J3CSCFF	J3 P.L.A. INSERT FLUID BLADDER	Please sell "FJ3C"
N/A	J3CSAFV	J3 P.L.A. INSERT FLUID BLADDER	Please sell "FJ3C"
N/A	J3CSBFV	J3 P.L.A. INSERT FLUID BLADDER	Please sell "FJ3C"
N/A	J3CSCFV	J3 P.L.A. INSERT FLUID BLADDER	Please sell "FJ3C"
N/A	J3CDAFF	J3 P.L.A. INSERT FLUID BLADDER	Please sell "FJ3C"
N/A	J3CDBFF	J3 P.L.A. INSERT FLUID BLADDER	Please sell "FJ3C"
N/A	J3CDCFF	J3 P.L.A. INSERT FLUID BLADDER	Please sell "FJ3C"
N/A	J3CDAFV	J3 P.L.A. INSERT FLUID BLADDER	Please sell "FJ3C"
N/A	J3CDBFV	J3 P.L.A. INSERT FLUID BLADDER	Please sell "FJ3C"
>>>>		AIR P.L.A. INSERT	
N/A	J3CDCFV	J3 P.L.A. INSERT AIR BLADDER	Please sell "FJ3C"
N/A	J3CSAAS	J3 P.L.A. INSERT AIR BLADDER	Please sell "AJ3C"
N/A	J3CSBAS	J3 P.L.A. INSERT AIR BLADDER	Please sell "AJ3C"
N/A	J3CSCAS	J3 P.L.A. INSERT AIR BLADDER	Please sell "AJ3C"
N/A	J3CSAAD	J3 P.L.A. INSERT AIR BLADDER	Please sell "AJ3C"
N/A	J3CSBAD	J3 P.L.A. INSERT AIR BLADDER	Please sell "AJ3C"
N/A	J3CSCAD	J3 P.L.A. INSERT AIR BLADDER	Please sell "AJ3C"
N/A	J3CDAAS	J3 P.L.A. INSERT AIR BLADDER	Please sell "AJ3C"
N/A	J3CDBAS	J3 P.L.A. INSERT AIR BLADDER	Please sell "AJ3C"
N/A	J3CDCAS	J3 P.L.A. INSERT AIR BLADDER	Please sell "AJ3C"
N/A	J3CDAAD	J3 P.L.A. INSERT AIR BLADDER	Please sell "AJ3C"
N/A	J3CDBAD	J3 P.L.A. INSERT AIR BLADDER	Please sell "AJ3C"
N/A	J3CDCAD	J3 P.L.A. INSERT AIR BLADDER	Please sell "AJ3C"
>>>>		SUPPLEMENTAL FLUID PADS	
N/A	090509	SUPPLEMENTAL FLUID PAD SMALL	Please sell "J3CAF5M"
N/A	012010	SUPPLEMENTAL FLUID PAD MEDIUM	Please sell "J3CAFMD"
N/A	010510	SUPPLEMENTAL FLUID PAD LARGE	Please sell "J3CAFLG"
N/A	091409	SUPPLEMENTAL FLUID PAD X-LARGE	Please sell "J3CAFXL"
>>>>		LATERAL THIGH SUPPORTS (PAIR)	
N/A	108821-001	LATERAL THIGH SUPPORTS 1" HIGH (CUSHIONS12W-14W) LEFT	Please sell "J3CAL51" (includes Left & Right)
N/A	108822-001	LATERAL THIGH SUPPORTS 1" HIGH (CUSHIONS12W-14W) RIGHT	Please sell "J3CAL51" (includes Left & Right)
N/A	108821-002	LATERAL THIGH SUPPORTS 1" HIGH (CUSHIONS16W-19W) LEFT	Please sell "J3CALM1" (includes Left & Right)
N/A	108822-002	LATERAL THIGH SUPPORTS 1" HIGH (CUSHIONS16W-19W) RIGHT	Please sell "J3CALM1" (includes Left & Right)
N/A	108821-003	LATERAL THIGH SUPPORTS 1" HIGH (CUSHIONS20W-24W) LEFT	Please sell "J3CALL1" (includes Left & Right)
N/A	108822-003	LATERAL THIGH SUPPORTS 1" HIGH (CUSHIONS20W-24W) RIGHT	Please sell "J3CALL1" (includes Left & Right)
N/A	108823-001	LATERAL THIGH SUPPORTS 2" HIGH (CUSHIONS12W-14W) LEFT	Please sell "J3CAL52" (includes Left & Right)

Pos.	Item Number	Description	Remarks
N/A	108824-001	LATERAL THIGH SUPPORTS 2" HIGH (CUSHIONS12W-14W) RIGHT	Please sell "J3CAL2" (includes Left & Right)
N/A	108823-002	LATERAL THIGH SUPPORTS 2" HIGH (CUSHIONS16W-19W) LEFT	Please sell "J3CALM2" (includes Left & Right)
N/A	108824-002	LATERAL THIGH SUPPORTS 2" HIGH (CUSHIONS16W-19W) RIGHT	Please sell "J3CALM2" (includes Left & Right)
N/A	108823-003	LATERAL THIGH SUPPORTS 2" HIGH (CUSHIONS20W-24W) LEFT	Please sell "J3CALL2" (includes Left & Right)
N/A	108824-003	LATERAL THIGH SUPPORTS 2" HIGH (CUSHIONS20W-24W) RIGHT	Please sell "J3CALL2" (includes Left & Right)
>>>>		MEDIAL THIGH SUPPORTS (EACH)	
N/A	108825-001	MEDIAL THIGH SUPPORT 1" HIGH (CUSHIONS 12W-14W)	Please sell "J3CAMS1" (each)
N/A	108825-002	MEDIAL THIGH SUPPORT 1" HIGH (CUSHIONS 16W-19W)	Please sell "J3CMM1" (each)
N/A	108825-003	MEDIAL THIGH SUPPORT 1" HIGH (CUSHIONS 20W-24W)	Please sell "J3CAML1" (each)
N/A	108826-001	MEDIAL THIGH SUPPORT 2" HIGH (CUSHIONS 12W-14W)	Please sell "J3CAMS2" (each)
N/A	108826-002	MEDIAL THIGH SUPPORT 2" HIGH (CUSHIONS 16W-19W)	Please sell "J3CMM2" (each)
N/A	108826-003	MEDIAL THIGH SUPPORT 2" HIGH (CUSHIONS 20W-24W)	Please sell "J3CAML2" (each)
>>>>		LATERAL PELVIC SUPPORTS (PAIR)	
N/A	108227-001	LATERAL PELVIC SUPPORTS 2" HIGH (CUSHIONS 12W-14W)	Please sell "J3CAPS2" (includes Left & Right)
N/A	108827-002	LATERAL PELVIC SUPPORTS 2" HIGH (CUSHIONS 16W-19W)	Please sell "J3CAPM2" (includes Left & Right)
>>>>		P.L.A. REDUCER	
N/A	109368	STANDARD P.L.A REDUCER - SIZE B TO A	Please sell "J3CAR5BA"
N/A	109440	STANDARD P.L.A REDUCER - SIZE C TO A	Please sell "J3CAR5CA"
N/A	109369	STANDARD P.L.A REDUCER - SIZE C TO B	Please sell "J3CAR5CB"
N/A	109370	DEEP P.L.A REDUCER - SIZE B TO A	Please sell "J3CAR5DBA"
N/A	109441	DEEP P.L.A REDUCER - SIZE C TO A	Please sell "J3CAR5DCA"
N/A	109371	DEEP P.L.A REDUCER - SIZE C TO B	Please sell "J3CAR5DCB"
>>>>		OBLIQUITY PADS	
N/A	108830-001	OBLIQUITY PAD STANDARD A RIGHT	Please sell "J3CSAOR"
N/A	108831-001	OBLIQUITY PAD STANDARD A LEFT	Please sell "J3CSAOL"
N/A	108830-002	OBLIQUITY PAD STANDARD B RIGHT	Please sell "J3CSBOR"
N/A	108831-002	OBLIQUITY PAD STANDARD B LEFT	Please sell "J3CSBOL"
N/A	108830-003	OBLIQUITY PAD STANDARD C RIGHT	Please sell "J3CSCOR"
N/A	108831-003	OBLIQUITY PAD STANDARD C LEFT	Please sell "J3CSCOL"
N/A	108846-001	OBLIQUITY PAD DEEP A RIGHT	Please sell "J3CDAOR"
N/A	108845-001	OBLIQUITY PAD DEEP A LEFT	Please sell "J3CDAOL"
N/A	108846-002	OBLIQUITY PAD DEEP B RIGHT	Please sell "J3CDBOR"
N/A	108845-002	OBLIQUITY PAD DEEP B LEFT	Please sell "J3CDBOL"
N/A	108846-003	OBLIQUITY PAD DEEP C RIGHT	Please sell "J3CDCOR"
N/A	108845-003	OBLIQUITY PAD DEEP C LEFT	Please sell "J3CDCOL"

Note: Positioning attachments are 1 inch tall and are not replaceable.

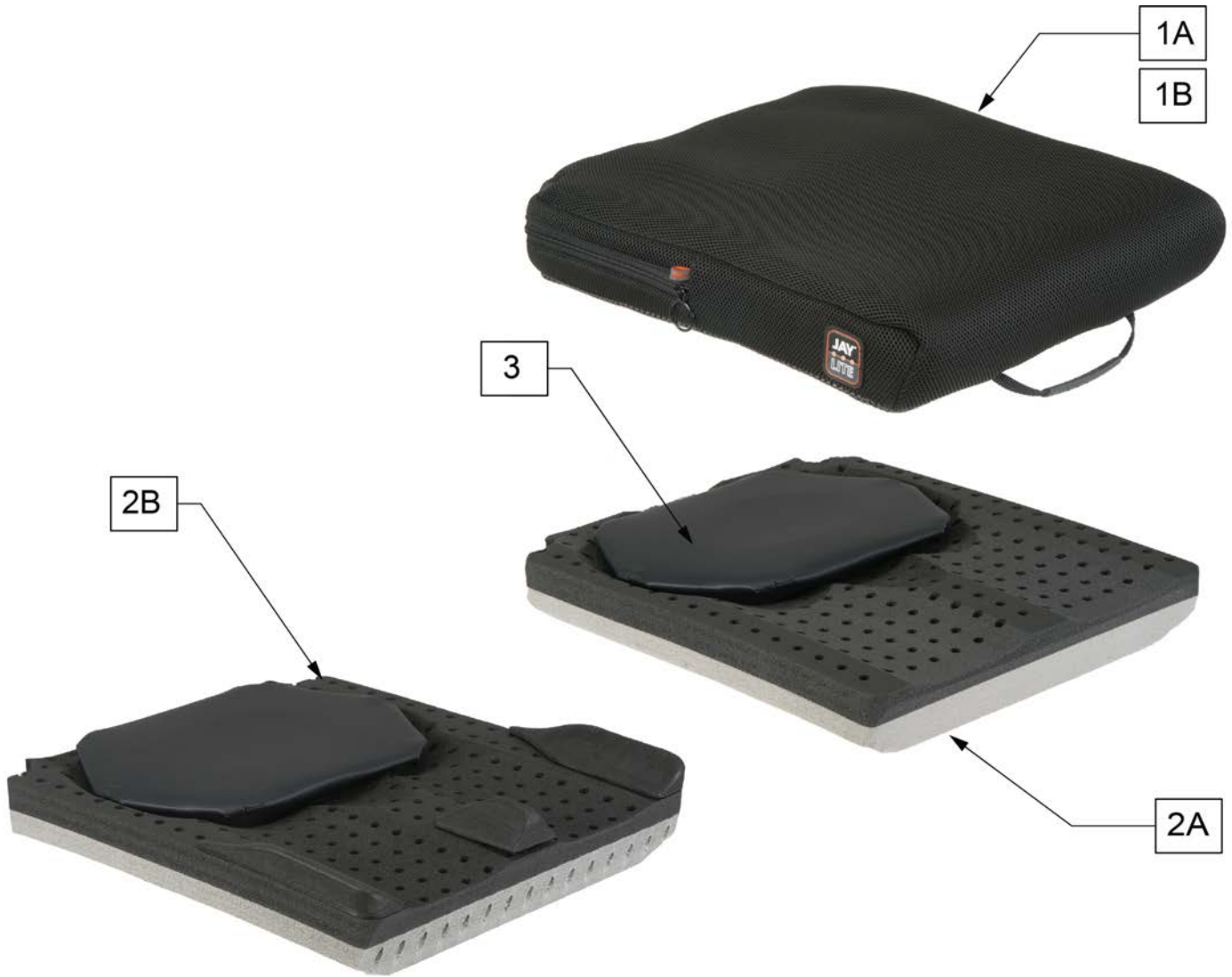


Pos.	Item Number	Description	Remarks
1A	CJL1414	COVER JAY LITE 1414IN 3535CM	
1A	CJL1416	COVER JAY LITE 1416IN 3535CM	
1A	CJL1418	COVER JAY LITE 1418IN 3535CM	
1A	CJL1515	COVER JAY LITE 1515IN	
1A	CJL1516	COVER JAY LITE 1516IN 3840CM	
1A	CJL1517	COVER JAY LITE 1517IN 3844CM	
1A	CJL1518	COVER JAY LITE 1518IN	
1A	CJL1615	COVER JAY LITE 1615IN 4038CM	
1A	CJL1616	COVER JAY LITE 1616IN 4040CM	
1A	CJL1617	COVER JAY LITE 1617IN 4043CM	
1A	CJL1618	COVER JAY LITE 1618IN 4045CM	
1A	CJL1620	COVER JAY LITE 1620IN 4050CM	
1A	CJL1715	COVER JAY LITE 1715IN 4338CM	
1A	CJL1716	COVER JAY LITE 1716IN 4340	


Pos.	Item Number	Description	Remarks
1A	CJL1717	COVER JAY LITE 1717IN 4343CM	
1A	CJL1718	COVER JAY LITE 1718IN 4345CM	
1A	CJL1816	COVER JAY LITE 1816IN 4540CM	
1A	CJL1817	COVER JAY LITE 1817IN 4543CM	
1A	CJL1818	COVER JAY LITE 1818IN 4545CM	
1A	CJL1820	COVER JAY LITE 1820IN 4550CM	
1A	CJL1916	COVER JAY LITE 1916IN 4840CM	
1A	CJL1917	COVER JAY LITE 1917IN 4843CM	
1A	CJL1918	COVER JAY LITE 1918IN 4845CM	
1A	CJL1920	COVER JAY LITE 1920IN 4850CM	
1A	CJL2016	COVER JAY LITE 2016IN 5040CM	
1A	CJL2018	COVER JAY LITE 2018IN 5045CM	
1A	CJL2020	COVER JAY LITE 2020IN 5050CM	
1A	CJL2218	COVER JAY LITE 2218IN 5545CM	
1A	CJL2220	COVER JAY LITE 2220IN 5550CM	
1A	CJL2222	COVER JAY LITE 2222IN 5555CM	
1A	CJL2418	COVER JAY LITE 2418IN 6045CM	
1A	CJL2420	COVER JAY LITE 2420IN 6050CM	
1A	CJL2422	COVER JAY LITE 2422IN 6055CM	
1A	CJL2424	COVER JAY LITE 2424IN 6060CM	
1B	CJL1414P	COVER, POS, JAY LITE 1414IN 3535CM	
1B	CJL1416P	COVER, POS, JAY LITE 1416IN 3535CM	
1B	CJL1418P	COVER, POS, JAY LITE 1418IN 3535CM	
1B	CJL1515P	COVER, POS, JAY LITE 1515IN	
1B	CJL1516P	COVER, POS, JAY LITE 1516IN 3840CM	
1B	CJL1517P	COVER, POS, JAY LITE 1517IN 3844CM	
1B	CJL1518P	COVER, POS, JAY LITE 1518IN	
1B	CJL1615P	COVER, POS, JAY LITE 1615IN 4038CM	
1B	CJL1616P	COVER, POS, JAY LITE 1616IN 4040CM	
1B	CJL1617P	COVER, POS, JAY LITE 1617IN 4043CM	
1B	CJL1618P	COVER, POS, JAY LITE 1618IN 4045CM	
1B	CJL1620P	COVER, POS, JAY LITE 1620IN 4050CM	
1B	CJL1715P	COVER, POS, JAY LITE 1715IN 4338CM	
1B	CJL1716P	COVER, POS, JAY LITE 1716IN 4340	
1B	CJL1717P	COVER, POS, JAY LITE 1717IN 4343CM	
1B	CJL1718P	COVER, POS, JAY LITE 1718IN 4345CM	
1B	CJL1816P	COVER, POS, JAY LITE 1816IN 4540CM	
1B	CJL1817P	COVER, POS, JAY LITE 1817IN 4543CM	
1B	CJL1818P	COVER, POS, JAY LITE 1818IN 4545CM	
1B	CJL1820P	COVER, POS, JAY LITE 1820IN 4550CM	
1B	CJL1916P	COVER, POS, JAY LITE 1916IN 4840CM	
1B	CJL1917P	COVER, POS, JAY LITE 1917IN 4843CM	
1B	CJL1918P	COVER, POS, JAY LITE 1918IN 4845CM	
1B	CJL1920P	COVER, POS, JAY LITE 1920IN 4850CM	
1B	CJL2016P	COVER, POS, JAY LITE 2016IN 5040CM	
1B	CJL2018P	COVER, POS, JAY LITE 2018IN 5045CM	
1B	CJL2020P	COVER, POS, JAY LITE 2020IN 5050CM	
1B	CJL2218P	COVER, POS, JAY LITE 2218IN 5545CM	
1B	CJL2220P	COVER, POS, JAY LITE 2220IN 5550CM	
1B	CJL2222P	COVER, POS, JAY LITE 2222IN 5555CM	
1B	CJL2418P	COVER, POS, JAY LITE 2418IN 6045CM	
1B	CJL2420P	COVER, POS, JAY LITE 2420IN 6050CM	
1B	CJL2422P	COVER, POS, JAY LITE 2422IN 6055CM	
1B	CJL2424P	COVER, POS, JAY LITE 2424IN 6060CM	
2A	BJL1414	BASE JAY LITE 1414IN 3535CM	
2A	BJL1416	BASE JAY LITE 1416IN 3535CM	
2A	BJL1418	BASE JAY LITE 1418IN 3535CM	

Pos.	Item Number	Description	Remarks
2A	BJL1515	BASE JAY LITE 1515IN	
2A	BJL1516	BASE JAY LITE 1516IN 3840CM	
2A	BJL1517	BASE JAY LITE 1517IN 3844CM	
2A	BJL1518	BASE JAY LITE 1518IN	
2A	BJL1615	BASE JAY LITE 1615IN 4038CM	
2A	BJL1616	BASE JAY LITE 1616IN 4040CM	
2A	BJL1617	BASE JAY LITE 1617IN 4043CM	
2A	BJL1618	BASE JAY LITE 1618IN 4045CM	
2A	BJL1620	BASE JAY LITE 1620IN 4050CM	
2A	BJL1715	BASE JAY LITE 1715IN 4338CM	
2A	BJL1716	BASE JAY LITE 1716IN 4340	
2A	BJL1717	BASE JAY LITE 1717IN 4343CM	
2A	BJL1718	BASE JAY LITE 1718IN 4345CM	
2A	BJL1816	BASE JAY LITE 1816IN 4540CM	
2A	BJL1817	BASE JAY LITE 1817IN 4543CM	
2A	BJL1818	BASE JAY LITE 1818IN 4545CM	
2A	BJL1820	BASE JAY LITE 1820IN 4550CM	
2A	BJL1916	BASE JAY LITE 1916IN 4840CM	
2A	BJL1917	BASE JAY LITE 1917IN 4843CM	
2A	BJL1918	BASE JAY LITE 1918IN 4845CM	
2A	BJL1920	BASE JAY LITE 1920IN 4850CM	
2A	BJL2016	BASE JAY LITE 2016IN 5040CM	
2A	BJL2018	BASE JAY LITE 2018IN 5045CM	
2A	BJL2020	BASE JAY LITE 2020IN 5050CM	
2A	BJL2218	BASE JAY LITE 2218IN 5545CM	
2A	BJL2220	BASE JAY LITE 2220IN 5550CM	
2A	BJL2222	BASE JAY LITE 2222IN 5555CM	
2A	BJL2418	BASE JAY LITE 2418IN 6045CM	
2A	BJL2420	BASE JAY LITE 2420IN 6050CM	
2A	BJL2422	BASE JAY LITE 2422IN 6055CM	
2A	BJL2424	BASE JAY LITE 2424IN 6060CM	
2B	BJL1414P	BASE, POS, JAY LITE 1414IN 3535CM	
2B	BJL1416P	BASE, POS, JAY LITE 1416IN 3535CM	
2B	BJL1418P	BASE, POS, JAY LITE 1418IN 3535CM	
2B	BJL1515P	BASE, POS, JAY LITE 1515IN	
2B	BJL1516P	BASE, POS, JAY LITE 1516IN 3840CM	
2B	BJL1517P	BASE, POS, JAY LITE 1517IN 3844CM	
2B	BJL1518P	BASE, POS, JAY LITE 1518IN	
2B	BJL1615P	BASE, POS, JAY LITE 1615IN 4038CM	
2B	BJL1616P	BASE, POS, JAY LITE 1616IN 4040CM	
2B	BJL1617P	BASE, POS, JAY LITE 1617IN 4043CM	
2B	BJL1618P	BASE, POS, JAY LITE 1618IN 4045CM	
2B	BJL1620P	BASE, POS, JAY LITE 1620IN 4050CM	
2B	BJL1715P	BASE, POS, JAY LITE 1715IN 4338CM	
2B	BJL1716P	BASE, POS, JAY LITE 1716IN 4340	
2B	BJL1717P	BASE, POS, JAY LITE 1717IN 4343CM	
2B	BJL1718P	BASE, POS, JAY LITE 1718IN 4345CM	
2B	BJL1816P	BASE, POS, JAY LITE 1816IN 4540CM	
2B	BJL1817P	BASE, POS, JAY LITE 1817IN 4543CM	
2B	BJL1818P	BASE, POS, JAY LITE 1818IN 4545CM	
2B	BJL1820P	BASE, POS, JAY LITE 1820IN 4550CM	
2B	BJL1916P	BASE, POS, JAY LITE 1916IN 4840CM	
2B	BJL1917P	BASE, POS, JAY LITE 1917IN 4843CM	
2B	BJL1918P	BASE, POS, JAY LITE 1918IN 4845CM	
2B	BJL1920P	BASE, POS, JAY LITE 1920IN 4850CM	
2B	BJL2016P	BASE, POS, JAY LITE 2016IN 5040CM	
2B	BJL2018P	BASE, POS, JAY LITE 2018IN 5045CM	

Pos.	Item Number	Description	Remarks
2B	BJL2020P	BASE, POS, JAY LITE 2020IN 5050CM	
2B	BJL2218P	BASE, POS, JAY LITE 2218IN 5545CM	
2B	BJL2220P	BASE, POS, JAY LITE 2220IN 5550CM	
2B	BJL2222P	BASE, POS, JAY LITE 2222IN 5555CM	
2B	BJL2418P	BASE, POS, JAY LITE 2418IN 6045CM	
2B	BJL2420P	BASE, POS, JAY LITE 2420IN 6050CM	
2B	BJL2422P	BASE, POS, JAY LITE 2422IN 6055CM	
2B	BJL2424P	BASE, POS, JAY LITE 2424IN 6060CM	
3	PUIJLA	PU FOAM INSERT JAY LITE PLA A	14" wide only
3	PUIJLB	PU FOAM INSERT JAY LITE PLA B	15"-17" wide
3	PUIJLC	PU FOAM INSERT JAY LITE PLA C	18"-20" wide

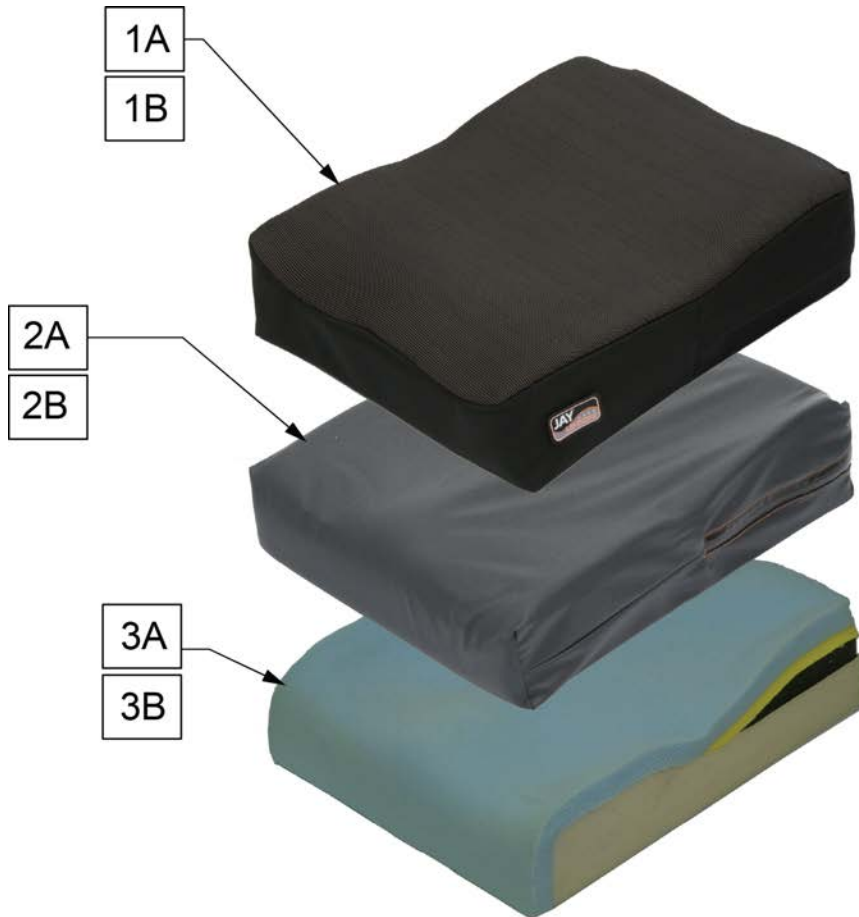

MEASURING GUIDE

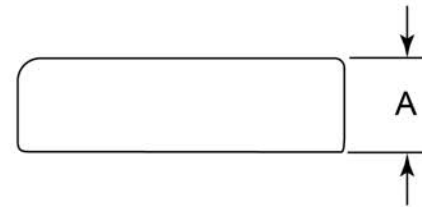

PROFILE	"A"
STANDARD	4 1/2
REDUCED PROFILE	3 1/2

Pos.	Item Number	Description	Pos.	Item Number	Description
		STANDARD OUTER COVER			
1A	CJU1414	OUTER CVR JAY UNION CUSH 14X14	1A	CJU1717	OUTER CVR JAY UNION CUSH 17X17
1A	CJU1415	OUTER CVR JAY UNION CUSH 14X15	1A	CJU1718	OUTER CVR JAY UNION CUSH 17X18
1A	CJU1416	OUTER CVR JAY UNION CUSH 14X16	1A	CJU1719	OUTER CVR JAY UNION CUSH 17X19
1A	CJU1417	OUTER CVR JAY UNION CUSH 14X17	1A	CJU1720	OUTER CVR JAY UNION CUSH 17X20
1A	CJU1418	OUTER CVR JAY UNION CUSH 14X18	1A	CJU1721	OUTER CVR JAY UNION CUSH 17X21
1A	CJU1514	OUTER CVR JAY UNION CUSH 15X14	1A	CJU1814	OUTER CVR JAY UNION CUSH 18X14
1A	CJU1515	OUTER CVR JAY UNION CUSH 15X15	1A	CJU1815	OUTER CVR JAY UNION CUSH 18X15
1A	CJU1516	OUTER CVR JAY UNION CUSH 15X16	1A	CJU1816	OUTER CVR JAY UNION CUSH 18X16
1A	CJU1517	OUTER CVR JAY UNION CUSH 15X17	1A	CJU1817	OUTER CVR JAY UNION CUSH 18X17
1A	CJU1518	OUTER CVR JAY UNION CUSH 15X18	1A	CJU1818	OUTER CVR JAY UNION CUSH 18X18
1A	CJU1519	OUTER CVR JAY UNION CUSH 15X19	1A	CJU1819	OUTER CVR JAY UNION CUSH 18X19
1A	CJU1520	OUTER CVR JAY UNION CUSH 15X20	1A	CJU1820	OUTER CVR JAY UNION CUSH 18X20
1A	CJU1614	OUTER CVR JAY UNION CUSH 16X14	1A	CJU1821	OUTER CVR JAY UNION CUSH 18X21
1A	CJU1615	OUTER CVR JAY UNION CUSH 16X15	1A	CJU1822	OUTER CVR JAY UNION CUSH 18X22
1A	CJU1616	OUTER CVR JAY UNION CUSH 16X16	1A	CJU1915	OUTER CVR JAY UNION CUSH 19X15
1A	CJU1617	OUTER CVR JAY UNION CUSH 16X17	1A	CJU1916	OUTER CVR JAY UNION CUSH 19X16
1A	CJU1618	OUTER CVR JAY UNION CUSH 16X18	1A	CJU1917	OUTER CVR JAY UNION CUSH 19X17
1A	CJU1619	OUTER CVR JAY UNION CUSH 16X19	1A	CJU1918	OUTER CVR JAY UNION CUSH 19X18
1A	CJU1620	OUTER CVR JAY UNION CUSH 16X20	1A	CJU1919	OUTER CVR JAY UNION CUSH 19X19
1A	CJU1714	OUTER CVR JAY UNION CUSH 17X14	1A	CJU1920	OUTER CVR JAY UNION CUSH 19X20
1A	CJU1715	OUTER CVR JAY UNION CUSH 17X15	1A	CJU1921	OUTER CVR JAY UNION CUSH 19X21
1A	CJU1716	OUTER CVR JAY UNION CUSH 17X16	1A	CJU1922	OUTER CVR JAY UNION CUSH 19X22
			1A	CJU1923	OUTER CVR JAY UNION CUSH 19X23

Pos.	Item Number	Description	Pos.	Item Number	Description
1A	CJU2015	OUTER CVR JAY UNION CUSH 20X15	2A	ICJU1619	INNER CVR JAY UNION CUSH 16X19
1A	CJU2016	OUTER CVR JAY UNION CUSH 20X16	2A	ICJU1620	INNER CVR JAY UNION CUSH 16X20
1A	CJU2017	OUTER CVR JAY UNION CUSH 20X17	2A	ICJU1714	INNER CVR JAY UNION CUSH 17X14
1A	CJU2018	OUTER CVR JAY UNION CUSH 20X18	2A	ICJU1715	INNER CVR JAY UNION CUSH 17X15
1A	CJU2019	OUTER CVR JAY UNION CUSH 20X19	2A	ICJU1716	INNER CVR JAY UNION CUSH 17X16
1A	CJU2020	OUTER CVR JAY UNION CUSH 20X20	2A	ICJU1717	INNER CVR JAY UNION CUSH 17X17
1A	CJU2021	OUTER CVR JAY UNION CUSH 20X21	2A	ICJU1718	INNER CVR JAY UNION CUSH 17X18
1A	CJU2022	OUTER CVR JAY UNION CUSH 20X22	2A	ICJU1719	INNER CVR JAY UNION CUSH 17X19
1A	CJU2023	OUTER CVR JAY UNION CUSH 20X23	2A	ICJU1720	INNER CVR JAY UNION CUSH 17X20
1A	CJU2024	OUTER CVR JAY UNION CUSH 20X24	2A	ICJU1721	INNER CVR JAY UNION CUSH 17X21
1A	CJU2117	OUTER CVR JAY UNION CUSH 21X17	2A	ICJU1814	INNER CVR JAY UNION CUSH 18X14
1A	CJU2118	OUTER CVR JAY UNION CUSH 21X18	2A	ICJU1815	INNER CVR JAY UNION CUSH 18X15
1A	CJU2119	OUTER CVR JAY UNION CUSH 21X19	2A	ICJU1816	INNER CVR JAY UNION CUSH 18X16
1A	CJU2120	OUTER CVR JAY UNION CUSH 21X20	2A	ICJU1817	INNER CVR JAY UNION CUSH 18X17
1A	CJU2121	OUTER CVR JAY UNION CUSH 21X21	2A	ICJU1818	INNER CVR JAY UNION CUSH 18X18
1A	CJU2122	OUTER CVR JAY UNION CUSH 21X22	2A	ICJU1819	INNER CVR JAY UNION CUSH 18X19
1A	CJU2123	OUTER CVR JAY UNION CUSH 21X23	2A	ICJU1820	INNER CVR JAY UNION CUSH 18X20
1A	CJU2124	OUTER CVR JAY UNION CUSH 21X24	2A	ICJU1821	INNER CVR JAY UNION CUSH 18X21
1A	CJU2217	OUTER CVR JAY UNION CUSH 22X17	2A	ICJU1822	INNER CVR JAY UNION CUSH 18X22
1A	CJU2218	OUTER CVR JAY UNION CUSH 22X18	2A	ICJU1915	INNER CVR JAY UNION CUSH 19X15
1A	CJU2219	OUTER CVR JAY UNION CUSH 22X19	2A	ICJU1916	INNER CVR JAY UNION CUSH 19X16
1A	CJU2220	OUTER CVR JAY UNION CUSH 22X20	2A	ICJU1917	INNER CVR JAY UNION CUSH 19X17
1A	CJU2221	OUTER CVR JAY UNION CUSH 22X21	2A	ICJU1918	INNER CVR JAY UNION CUSH 19X18
1A	CJU2222	OUTER CVR JAY UNION CUSH 22X22	2A	ICJU1919	INNER CVR JAY UNION CUSH 19X19
1A	CJU2223	OUTER CVR JAY UNION CUSH 22X23	2A	ICJU1920	INNER CVR JAY UNION CUSH 19X20
1A	CJU2224	OUTER CVR JAY UNION CUSH 22X24	2A	ICJU1921	INNER CVR JAY UNION CUSH 19X21
1A	CJU2319	OUTER CVR JAY UNION CUSH 23X19	2A	ICJU1922	INNER CVR JAY UNION CUSH 19X22
1A	CJU2320	OUTER CVR JAY UNION CUSH 23X20	2A	ICJU1923	INNER CVR JAY UNION CUSH 19X23
1A	CJU2321	OUTER CVR JAY UNION CUSH 23X21	2A	ICJU2015	INNER CVR JAY UNION CUSH 20X15
1A	CJU2322	OUTER CVR JAY UNION CUSH 23X22	2A	ICJU2016	INNER CVR JAY UNION CUSH 20X16
1A	CJU2323	OUTER CVR JAY UNION CUSH 23X23	2A	ICJU2017	INNER CVR JAY UNION CUSH 20X17
1A	CJU2324	OUTER CVR JAY UNION CUSH 23X24	2A	ICJU2018	INNER CVR JAY UNION CUSH 20X18
1A	CJU2418	OUTER CVR JAY UNION CUSH 24X18	2A	ICJU2019	INNER CVR JAY UNION CUSH 20X19
1A	CJU2419	OUTER CVR JAY UNION CUSH 24X19	2A	ICJU2020	INNER CVR JAY UNION CUSH 20X20
1A	CJU2420	OUTER CVR JAY UNION CUSH 24X20	2A	ICJU2021	INNER CVR JAY UNION CUSH 20X21
1A	CJU2421	OUTER CVR JAY UNION CUSH 24X21	2A	ICJU2022	INNER CVR JAY UNION CUSH 20X22
1A	CJU2422	OUTER CVR JAY UNION CUSH 24X22	2A	ICJU2023	INNER CVR JAY UNION CUSH 20X23
1A	CJU2423	OUTER CVR JAY UNION CUSH 24X23	2A	ICJU2024	INNER CVR JAY UNION CUSH 20X24
1A	CJU2424	OUTER CVR JAY UNION CUSH 24X24	2A	ICJU2117	INNER CVR JAY UNION CUSH 21X17
		STANDARD INNER COVER	2A	ICJU2118	INNER CVR JAY UNION CUSH 21X18
2A	ICJU1414	INNER CVR JAY UNION CUSH 14X14	2A	ICJU2119	INNER CVR JAY UNION CUSH 21X19
2A	ICJU1415	INNER CVR JAY UNION CUSH 14X15	2A	ICJU2120	INNER CVR JAY UNION CUSH 21X20
2A	ICJU1416	INNER CVR JAY UNION CUSH 14X16	2A	ICJU2121	INNER CVR JAY UNION CUSH 21X21
2A	ICJU1417	INNER CVR JAY UNION CUSH 14X17	2A	ICJU2122	INNER CVR JAY UNION CUSH 21X22
2A	ICJU1418	INNER CVR JAY UNION CUSH 14X18	2A	ICJU2123	INNER CVR JAY UNION CUSH 21X23
2A	ICJU1514	INNER CVR JAY UNION CUSH 15X14	2A	ICJU2124	INNER CVR JAY UNION CUSH 21X24
2A	ICJU1515	INNER CVR JAY UNION CUSH 15X15	2A	ICJU2217	INNER CVR JAY UNION CUSH 22X17
2A	ICJU1516	INNER CVR JAY UNION CUSH 15X16	2A	ICJU2218	INNER CVR JAY UNION CUSH 22X18
2A	ICJU1517	INNER CVR JAY UNION CUSH 15X17	2A	ICJU2219	INNER CVR JAY UNION CUSH 22X19
2A	ICJU1518	INNER CVR JAY UNION CUSH 15X18	2A	ICJU2220	INNER CVR JAY UNION CUSH 22X20
2A	ICJU1519	INNER CVR JAY UNION CUSH 15X19	2A	ICJU2221	INNER CVR JAY UNION CUSH 22X21
2A	ICJU1520	INNER CVR JAY UNION CUSH 15X20	2A	ICJU2222	INNER CVR JAY UNION CUSH 22X22
2A	ICJU1614	INNER CVR JAY UNION CUSH 16X14	2A	ICJU2223	INNER CVR JAY UNION CUSH 22X23
2A	ICJU1615	INNER CVR JAY UNION CUSH 16X15	2A	ICJU2224	INNER CVR JAY UNION CUSH 22X24
2A	ICJU1616	INNER CVR JAY UNION CUSH 16X16	2A	ICJU2319	INNER CVR JAY UNION CUSH 23X19
2A	ICJU1617	INNER CVR JAY UNION CUSH 16X17	2A	ICJU2320	INNER CVR JAY UNION CUSH 23X20
2A	ICJU1618	INNER CVR JAY UNION CUSH 16X18	2A	ICJU2321	INNER CVR JAY UNION CUSH 23X21

Pos.	Item Number	Description	Pos.	Item Number	Description
2A	ICJU2322	INNER CVR JAY UNION CUSH 23X22	3A	BJU2016	BASE FOAM JAY UNION CUSH 20X16
2A	ICJU2323	INNER CVR JAY UNION CUSH 23X23	3A	BJU2017	BASE FOAM JAY UNION CUSH 20X17
2A	ICJU2324	INNER CVR JAY UNION CUSH 23X24	3A	BJU2018	BASE FOAM JAY UNION CUSH 20X18
2A	ICJU2418	INNER CVR JAY UNION CUSH 24X18	3A	BJU2019	BASE FOAM JAY UNION CUSH 20X19
2A	ICJU2419	INNER CVR JAY UNION CUSH 24X19	3A	BJU2020	BASE FOAM JAY UNION CUSH 20X20
2A	ICJU2420	INNER CVR JAY UNION CUSH 24X20	3A	BJU2021	BASE FOAM JAY UNION CUSH 20X21
2A	ICJU2421	INNER CVR JAY UNION CUSH 24X21	3A	BJU2022	BASE FOAM JAY UNION CUSH 20X22
2A	ICJU2422	INNER CVR JAY UNION CUSH 24X22	3A	BJU2023	BASE FOAM JAY UNION CUSH 20X23
2A	ICJU2423	INNER CVR JAY UNION CUSH 24X23	3A	BJU2024	BASE FOAM JAY UNION CUSH 20X24
2A	ICJU2424	INNER CVR JAY UNION CUSH 24X24	3A	BJU2117	BASE FOAM JAY UNION CUSH 21X17
		STANDARD FOAM BASE	3A	BJU2118	BASE FOAM JAY UNION CUSH 21X18
3A	BJU1414	BASE FOAM JAY UNION CUSH 14X14	3A	BJU2119	BASE FOAM JAY UNION CUSH 21X19
3A	BJU1415	BASE FOAM JAY UNION CUSH 14X15	3A	BJU2120	BASE FOAM JAY UNION CUSH 21X20
3A	BJU1416	BASE FOAM JAY UNION CUSH 14X16	3A	BJU2121	BASE FOAM JAY UNION CUSH 21X21
3A	BJU1417	BASE FOAM JAY UNION CUSH 14X17	3A	BJU2122	BASE FOAM JAY UNION CUSH 21X22
3A	BJU1418	BASE FOAM JAY UNION CUSH 14X18	3A	BJU2123	BASE FOAM JAY UNION CUSH 21X23
3A	BJU1514	BASE FOAM JAY UNION CUSH 15X14	3A	BJU2124	BASE FOAM JAY UNION CUSH 21X24
3A	BJU1515	BASE FOAM JAY UNION CUSH 15X15	3A	BJU2217	BASE FOAM JAY UNION CUSH 22X17
3A	BJU1516	BASE FOAM JAY UNION CUSH 15X16	3A	BJU2218	BASE FOAM JAY UNION CUSH 22X18
3A	BJU1517	BASE FOAM JAY UNION CUSH 15X17	3A	BJU2219	BASE FOAM JAY UNION CUSH 22X19
3A	BJU1518	BASE FOAM JAY UNION CUSH 15X18	3A	BJU2220	BASE FOAM JAY UNION CUSH 22X20
3A	BJU1519	BASE FOAM JAY UNION CUSH 15X19	3A	BJU2221	BASE FOAM JAY UNION CUSH 22X21
3A	BJU1520	BASE FOAM JAY UNION CUSH 15X20	3A	BJU2222	BASE FOAM JAY UNION CUSH 22X22
3A	BJU1614	BASE FOAM JAY UNION CUSH 16X14	3A	BJU2223	BASE FOAM JAY UNION CUSH 22X23
3A	BJU1615	BASE FOAM JAY UNION CUSH 16X15	3A	BJU2224	BASE FOAM JAY UNION CUSH 22X24
3A	BJU1616	BASE FOAM JAY UNION CUSH 16X16	3A	BJU2319	BASE FOAM JAY UNION CUSH 23X19
3A	BJU1617	BASE FOAM JAY UNION CUSH 16X17	3A	BJU2320	BASE FOAM JAY UNION CUSH 23X20
3A	BJU1618	BASE FOAM JAY UNION CUSH 16X18	3A	BJU2321	BASE FOAM JAY UNION CUSH 23X21
3A	BJU1619	BASE FOAM JAY UNION CUSH 16X19	3A	BJU2322	BASE FOAM JAY UNION CUSH 23X22
3A	BJU1620	BASE FOAM JAY UNION CUSH 16X20	3A	BJU2323	BASE FOAM JAY UNION CUSH 23X23
3A	BJU1714	BASE FOAM JAY UNION CUSH 17X14	3A	BJU2324	BASE FOAM JAY UNION CUSH 23X24
3A	BJU1715	BASE FOAM JAY UNION CUSH 17X15	3A	BJU2418	BASE FOAM JAY UNION CUSH 24X18
3A	BJU1716	BASE FOAM JAY UNION CUSH 17X16	3A	BJU2419	BASE FOAM JAY UNION CUSH 24X19
3A	BJU1717	BASE FOAM JAY UNION CUSH 17X17	3A	BJU2420	BASE FOAM JAY UNION CUSH 24X20
3A	BJU1718	BASE FOAM JAY UNION CUSH 17X18	3A	BJU2421	BASE FOAM JAY UNION CUSH 24X21
3A	BJU1719	BASE FOAM JAY UNION CUSH 17X19	3A	BJU2422	BASE FOAM JAY UNION CUSH 24X22
3A	BJU1720	BASE FOAM JAY UNION CUSH 17X20	3A	BJU2423	BASE FOAM JAY UNION CUSH 24X23
3A	BJU1721	BASE FOAM JAY UNION CUSH 17X21	3A	BJU2424	BASE FOAM JAY UNION CUSH 24X24
3A	BJU1814	BASE FOAM JAY UNION CUSH 18X14			REDUCED PROFILE OUTER COVER
3A	BJU1815	BASE FOAM JAY UNION CUSH 18X15	1B	CJU1414RP	OUTER CVR UNION RP CUSH 14X14
3A	BJU1816	BASE FOAM JAY UNION CUSH 18X16	1B	CJU1415RP	OUTER CVR UNION RP CUSH 14X15
3A	BJU1817	BASE FOAM JAY UNION CUSH 18X17	1B	CJU1416RP	OUTER CVR UNION RP CUSH 14X16
3A	BJU1818	BASE FOAM JAY UNION CUSH 18X18	1B	CJU1417RP	OUTER CVR UNION RP CUSH 14X17
3A	BJU1819	BASE FOAM JAY UNION CUSH 18X19	1B	CJU1418RP	OUTER CVR UNION RP CUSH 14X18
3A	BJU1820	BASE FOAM JAY UNION CUSH 18X20	1B	CJU1514RP	OUTER CVR UNION RP CUSH 15X14
3A	BJU1821	BASE FOAM JAY UNION CUSH 18X21	1B	CJU1515RP	OUTER CVR UNION RP CUSH 15X15
3A	BJU1822	BASE FOAM JAY UNION CUSH 18X22	1B	CJU1516RP	OUTER CVR UNION RP CUSH 15X16
3A	BJU1915	BASE FOAM JAY UNION CUSH 19X15	1B	CJU1517RP	OUTER CVR UNION RP CUSH 15X17
3A	BJU1916	BASE FOAM JAY UNION CUSH 19X16	1B	CJU1518RP	OUTER CVR UNION RP CUSH 15X18
3A	BJU1917	BASE FOAM JAY UNION CUSH 19X17	1B	CJU1519RP	OUTER CVR UNION RP CUSH 15X19
3A	BJU1918	BASE FOAM JAY UNION CUSH 19X18	1B	CJU1520RP	OUTER CVR UNION RP CUSH 15X20
3A	BJU1919	BASE FOAM JAY UNION CUSH 19X19	1B	CJU1614RP	OUTER CVR UNION RP CUSH 16X14
3A	BJU1920	BASE FOAM JAY UNION CUSH 19X20	1B	CJU1615RP	OUTER CVR UNION RP CUSH 16X15
3A	BJU1921	BASE FOAM JAY UNION CUSH 19X21	1B	CJU1616RP	OUTER CVR UNION RP CUSH 16X16
3A	BJU1922	BASE FOAM JAY UNION CUSH 19X22	1B	CJU1617RP	OUTER CVR UNION RP CUSH 16X17
3A	BJU1923	BASE FOAM JAY UNION CUSH 19X23	1B	CJU1618RP	OUTER CVR UNION RP CUSH 16X18
3A	BJU2015	BASE FOAM JAY UNION CUSH 20X15	1B	CJU1619RP	OUTER CVR UNION RP CUSH 16X19


Pos.	Item Number	Description	Pos.	Item Number	Description
1B	CJU1620RP	OUTER CVR UNION RP CUSH 16X20	2B	ICJU1817RP	INNER CVR UNION RP CUSH 18X17
1B	CJU1714RP	OUTER CVR UNION RP CUSH 17X14	2B	ICJU1818RP	INNER CVR UNION RP CUSH 18X18
1B	CJU1715RP	OUTER CVR UNION RP CUSH 17X15	2B	ICJU1819RP	INNER CVR UNION RP CUSH 18X19
1B	CJU1716RP	OUTER CVR UNION RP CUSH 17X16	2B	ICJU1820RP	INNER CVR UNION RP CUSH 18X20
1B	CJU1717RP	OUTER CVR UNION RP CUSH 17X17	2B	ICJU1915RP	INNER CVR UNION RP CUSH 19X15
1B	CJU1718RP	OUTER CVR UNION RP CUSH 17X18	2B	ICJU1916RP	INNER CVR UNION RP CUSH 19X16
1B	CJU1719RP	OUTER CVR UNION RP CUSH 17X19	2B	ICJU1917RP	INNER CVR UNION RP CUSH 19X17
1B	CJU1720RP	OUTER CVR UNION RP CUSH 17X20	2B	ICJU1918RP	INNER CVR UNION RP CUSH 19X18
1B	CJU1814RP	OUTER CVR UNION RP CUSH 18X14	2B	ICJU1919RP	INNER CVR UNION RP CUSH 19X19
1B	CJU1815RP	OUTER CVR UNION RP CUSH 18X15	2B	ICJU1920RP	INNER CVR UNION RP CUSH 19X20
1B	CJU1816RP	OUTER CVR UNION RP CUSH 18X16	2B	ICJU2015RP	INNER CVR UNION RP CUSH 20X15
1B	CJU1817RP	OUTER CVR UNION RP CUSH 18X17	2B	ICJU2016RP	INNER CVR UNION RP CUSH 20X16
1B	CJU1818RP	OUTER CVR UNION RP CUSH 18X18	2B	ICJU2017RP	INNER CVR UNION RP CUSH 20X17
1B	CJU1819RP	OUTER CVR UNION RP CUSH 18X19	2B	ICJU2018RP	INNER CVR UNION RP CUSH 20X18
1B	CJU1820RP	OUTER CVR UNION RP CUSH 18X20	2B	ICJU2019RP	INNER CVR UNION RP CUSH 20X19
1B	CJU1915RP	OUTER CVR UNION RP CUSH 19X15	2B	ICJU2020RP	INNER CVR UNION RP CUSH 20X20
1B	CJU1916RP	OUTER CVR UNION RP CUSH 19X16			REDUCED PROFILE FOAM BASE COVER
1B	CJU1917RP	OUTER CVR UNION RP CUSH 19X17	3B	BJU1414RP	BASE FOAM UNION RP CUSH 14X14
1B	CJU1918RP	OUTER CVR UNION RP CUSH 19X18	3B	BJU1415RP	BASE FOAM UNION RP CUSH 14X15
1B	CJU1919RP	OUTER CVR UNION RP CUSH 19X19	3B	BJU1416RP	BASE FOAM UNION RP CUSH 14X16
1B	CJU1920RP	OUTER CVR UNION RP CUSH 19X20	3B	BJU1417RP	BASE FOAM UNION RP CUSH 14X17
1B	CJU2015RP	OUTER CVR UNION RP CUSH 20X15	3B	BJU1418RP	BASE FOAM UNION RP CUSH 14X18
1B	CJU2016RP	OUTER CVR UNION RP CUSH 20X16	3B	BJU1514RP	BASE FOAM UNION RP CUSH 15X14
1B	CJU2017RP	OUTER CVR UNION RP CUSH 20X17	3B	BJU1515RP	BASE FOAM UNION RP CUSH 15X15
1B	CJU2018RP	OUTER CVR UNION RP CUSH 20X18	3B	BJU1516RP	BASE FOAM UNION RP CUSH 15X16
1B	CJU2019RP	OUTER CVR UNION RP CUSH 20X19	3B	BJU1517RP	BASE FOAM UNION RP CUSH 15X17
1B	CJU2020RP	OUTER CVR UNION RP CUSH 20X20	3B	BJU1518RP	BASE FOAM UNION RP CUSH 15X18
		REDUCED PROFILE INNER COVER	3B	BJU1519RP	BASE FOAM UNION RP CUSH 15X19
2B	ICJU1414RP	INNER CVR UNION RP CUSH 14X14	3B	BJU1520RP	BASE FOAM UNION RP CUSH 15X20
2B	ICJU1415RP	INNER CVR UNION RP CUSH 14X15	3B	BJU1614RP	BASE FOAM UNION RP CUSH 16X14
2B	ICJU1416RP	INNER CVR UNION RP CUSH 14X16	3B	BJU1615RP	BASE FOAM UNION RP CUSH 16X15
2B	ICJU1417RP	INNER CVR UNION RP CUSH 14X17	3B	BJU1616RP	BASE FOAM UNION RP CUSH 16X16
2B	ICJU1418RP	INNER CVR UNION RP CUSH 14X18	3B	BJU1617RP	BASE FOAM UNION RP CUSH 16X17
2B	ICJU1514RP	INNER CVR UNION RP CUSH 15X14	3B	BJU1618RP	BASE FOAM UNION RP CUSH 16X18
2B	ICJU1515RP	INNER CVR UNION RP CUSH 15X15	3B	BJU1619RP	BASE FOAM UNION RP CUSH 16X19
2B	ICJU1516RP	INNER CVR UNION RP CUSH 15X16	3B	BJU1620RP	BASE FOAM UNION RP CUSH 16X20
2B	ICJU1517RP	INNER CVR UNION RP CUSH 15X17	3B	BJU1714RP	BASE FOAM UNION RP CUSH 17X14
2B	ICJU1518RP	INNER CVR UNION RP CUSH 15X18	3B	BJU1715RP	BASE FOAM UNION RP CUSH 17X15
2B	ICJU1519RP	INNER CVR UNION RP CUSH 15X19	3B	BJU1716RP	BASE FOAM UNION RP CUSH 17X16
2B	ICJU1520RP	INNER CVR UNION RP CUSH 15X20	3B	BJU1717RP	BASE FOAM UNION RP CUSH 17X17
2B	ICJU1614RP	INNER CVR UNION RP CUSH 16X14	3B	BJU1718RP	BASE FOAM UNION RP CUSH 17X18
2B	ICJU1615RP	INNER CVR UNION RP CUSH 16X15	3B	BJU1719RP	BASE FOAM UNION RP CUSH 17X19
2B	ICJU1616RP	INNER CVR UNION RP CUSH 16X16	3B	BJU1720RP	BASE FOAM UNION RP CUSH 17X20
2B	ICJU1617RP	INNER CVR UNION RP CUSH 16X17	3B	BJU1814RP	BASE FOAM UNION RP CUSH 18X14
2B	ICJU1618RP	INNER CVR UNION RP CUSH 16X18	3B	BJU1815RP	BASE FOAM UNION RP CUSH 18X15
2B	ICJU1619RP	INNER CVR UNION RP CUSH 16X19	3B	BJU1816RP	BASE FOAM UNION RP CUSH 18X16
2B	ICJU1620RP	INNER CVR UNION RP CUSH 16X20	3B	BJU1817RP	BASE FOAM UNION RP CUSH 18X17
2B	ICJU1714RP	INNER CVR UNION RP CUSH 17X14	3B	BJU1818RP	BASE FOAM UNION RP CUSH 18X18
2B	ICJU1715RP	INNER CVR UNION RP CUSH 17X15	3B	BJU1819RP	BASE FOAM UNION RP CUSH 18X19
2B	ICJU1716RP	INNER CVR UNION RP CUSH 17X16	3B	BJU1820RP	BASE FOAM UNION RP CUSH 18X20
2B	ICJU1717RP	INNER CVR UNION RP CUSH 17X17	3B	BJU1915RP	BASE FOAM UNION RP CUSH 19X15
2B	ICJU1718RP	INNER CVR UNION RP CUSH 17X18	3B	BJU1916RP	BASE FOAM UNION RP CUSH 19X16
2B	ICJU1719RP	INNER CVR UNION RP CUSH 17X19	3B	BJU1917RP	BASE FOAM UNION RP CUSH 19X17
2B	ICJU1720RP	INNER CVR UNION RP CUSH 17X20	3B	BJU1918RP	BASE FOAM UNION RP CUSH 19X18
2B	ICJU1814RP	INNER CVR UNION RP CUSH 18X14	3B	BJU1919RP	BASE FOAM UNION RP CUSH 19X19
2B	ICJU1815RP	INNER CVR UNION RP CUSH 18X15	3B	BJU1920RP	BASE FOAM UNION RP CUSH 19X20
2B	ICJU1816RP	INNER CVR UNION RP CUSH 18X16	3B	BJU2015RP	BASE FOAM UNION RP CUSH 20X15

Pos.	Item Number	Description	Pos.	Item Number	Description
3B	BJU2016RP	BASE FOAM UNION RP CUSH 20X16	3B	BJU2019RP	BASE FOAM UNION RP CUSH 20X19
3B	BJU2017RP	BASE FOAM UNION RP CUSH 20X17	3B	BJU2020RP	BASE FOAM UNION RP CUSH 20X20
3B	BJU2018RP	BASE FOAM UNION RP CUSH 20X18			



Pos.	Item Number	Description	Remarks
1	CJX21414	COVER, JAY X2, OUTER, 14X14	
1	CJX21416	COVER, JAY X2, OUTER, 14X16	
1	CJX21515	COVER, JAY X2, OUTER, 15X15	
1	CJX21516	COVER, JAY X2, OUTER, 15X16	
1	CJX21616	COVER, JAY X2, OUTER, 16X16	
1	CJX21617	COVER, JAY X2, OUTER, 16X17	
1	CJX21618	COVER, JAY X2, OUTER, 16X18	
1	CJX21620	COVER, JAY X2, OUTER, 16X20	
1	CJX21716	COVER, JAY X2, OUTER, 17X16	
1	CJX21717	COVER, JAY X2, OUTER, 17X17	
1	CJX21816	COVER, JAY X2, OUTER, 18X16	
1	CJX21818	COVER, JAY X2, OUTER, 18X18	
1	CJX21820	COVER, JAY X2, OUTER, 18X20	
1	CJX22020	COVER, JAY X2, OUTER, 20X20	
2A	FJX2-1	FL PD JAY X2 14X14 14X16	

Pos.	Item Number	Description	Remarks
2A	FJX2-2	FL PD JAY X2 15X15 15X16 16X16	
2A	FJX2-3	FL PD JAY X2 16X17 16X18 16X20	
2A	FJX2-4	FL PD JAY X2 17X16 18X16	
2A	FJX2-5	FL PD JAY X2 17X17 18X18 18X20	
2A	FJX2-6	FL PD JAY X2 20X20	
2B	F128	FLUID PAD JA 14X16	ACTIVE FLUID PAD FOR JAY YOUR WAY MODIFICATION
2B	F125	FLUID PAD JA 16X16	ACTIVE FLUID PAD FOR JAY YOUR WAY MODIFICATION
2B	F126	FLUID PAD JA 16X18	ACTIVE FLUID PAD FOR JAY YOUR WAY MODIFICATION
2B	F120	FLUID PAD JA 18X16	ACTIVE FLUID PAD FOR JAY YOUR WAY MODIFICATION
2B	F121	FLUID PAD JA 18X18	ACTIVE FLUID PAD FOR JAY YOUR WAY MODIFICATION
2B	F122	FLUID PAD JA 20X18	ACTIVE FLUID PAD FOR JAY YOUR WAY MODIFICATION
3	B119JA	1" ADDUCTOR WEDGES	PAIR
3	B113JA	2" ADDUCTOR WEDGES (PR)	PAIR
4	ICJX21414	COVER, JAY X2, INNER, 14X14	
4	ICJX21416	COVER, JAY X2, INNER, 14X16	
4	ICJX21515	COVER, JAY X2, INNER, 15X15	
4	ICJX21516	COVER, JAY X2, INNER, 15X16	
4	ICJX21616	COVER, JAY X2, INNER, 16X16	
4	ICJX21617	COVER, JAY X2, INNER, 16X17	
4	ICJX21618	COVER, JAY X2, INNER, 16X18	
4	ICJX21620	COVER, JAY X2, INNER, 16X20	
4	ICJX21716	COVER, JAY X2, INNER, 17X16	
4	ICJX21717	COVER, JAY X2, INNER, 17X17	
4	ICJX21816	COVER, JAY X2, INNER, 18X16	
4	ICJX21818	COVER, JAY X2, INNER, 18X18	
4	ICJX21820	COVER, JAY X2, INNER, 18X20	
4	ICJX22020	COVER, JAY X2, INNER, 20X20	
5	BJX21414	BASE, JAY X2, 14X14	
5	BJX21416	BASE, JAY X2, 14X16	
5	BJX21515	BASE, JAY X2, 15X15	
5	BJX21516	BASE, JAY X2, 15X16	
5	BJX21616	BASE, JAY X2, 16X16	
5	BJX21617	BASE, JAY X2, 16X17	
5	BJX21618	BASE, JAY X2, 16X18	
5	BJX21620	BASE, JAY X2, 16X20	
5	BJX21716	BASE, JAY X2, 17X16	
5	BJX21717	BASE, JAY X2, 17X17	
5	BJX21816	BASE, JAY X2, 18X16	
5	BJX21818	BASE, JAY X2, 18X18	
5	BJX21820	BASE, JAY X2, 18X20	
5	BJX22020	BASE, JAY X2, 20X20	

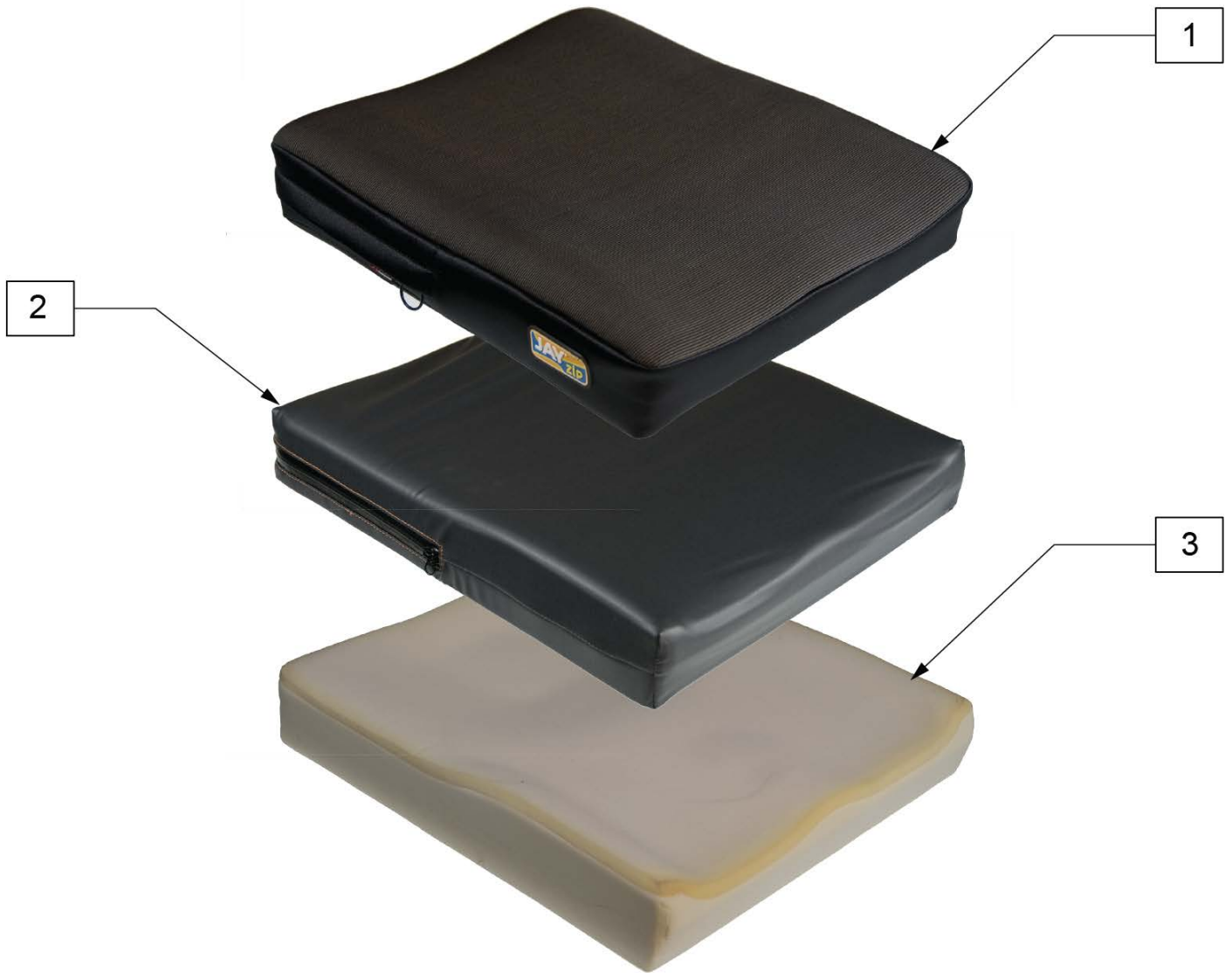


Pos.	Item Number	Description	Pos.	Item Number	Description
		OUTER COVER	1	CJZ1008	JAY ZIP OUTER COVER 10X08
1	CJZ0808	JAY ZIP OUTER COVER 08X08	1	CJZ1009	JAY ZIP OUTER COVER 10X09
1	CJZ0809	JAY ZIP OUTER COVER 08X09	1	CJZ1010	JAY ZIP OUTER COVER 10X10
1	CJZ0810	JAY ZIP OUTER COVER 08X10	1	CJZ1011	JAY ZIP OUTER COVER 10X11
1	CJZ0811	JAY ZIP OUTER COVER 08X11	1	CJZ1012	JAY ZIP OUTER COVER 10X12
1	CJZ0812	JAY ZIP OUTER COVER 08X12	1	CJZ1013	JAY ZIP OUTER COVER 10X13
1	CJZ0813	JAY ZIP OUTER COVER 08X13	1	CJZ1014	JAY ZIP OUTER COVER 10X14
1	CJZ0814	JAY ZIP OUTER COVER 08X14	1	CJZ1015	JAY ZIP OUTER COVER 10X15
1	CJZ0908	JAY ZIP OUTER COVER 09X08	1	CJZ1016	JAY ZIP OUTER COVER 10X16
1	CJZ0909	JAY ZIP OUTER COVER 09X09	1	CJZ1108	JAY ZIP OUTER COVER 11X08
1	CJZ0910	JAY ZIP OUTER COVER 09X10	1	CJZ1109	JAY ZIP OUTER COVER 11X09
1	CJZ0911	JAY ZIP OUTER COVER 09X11	1	CJZ1110	JAY ZIP OUTER COVER 11X10
1	CJZ0912	JAY ZIP OUTER COVER 09X12	1	CJZ1111	JAY ZIP OUTER COVER 11X11
1	CJZ0913	JAY ZIP OUTER COVER 09X13	1	CJZ1112	JAY ZIP OUTER COVER 11X12
1	CJZ0914	JAY ZIP OUTER COVER 09X14	1	CJZ1113	JAY ZIP OUTER COVER 11X13
1	CJZ0915	JAY ZIP OUTER COVER 09X15	1	CJZ1114	JAY ZIP OUTER COVER 11X14

Pos.	Item Number	Description	Pos.	Item Number	Description
1	CJZ1115	JAY ZIP OUTER COVER 11X15	2	ICJZ0914	JAY ZIP INNER COVER 09X14
1	CJZ1116	JAY ZIP OUTER COVER 11X16	2	ICJZ0915	JAY ZIP INNER COVER 09X15
1	CJZ1117	JAY ZIP OUTER COVER 11X17	2	ICJZ1008	JAY ZIP INNER COVER 10X08
1	CJZ1209	JAY ZIP OUTER COVER 12X09	2	ICJZ1009	JAY ZIP INNER COVER 10X09
1	CJZ1210	JAY ZIP OUTER COVER 12X10	2	ICJZ1010	JAY ZIP INNER COVER 10X10
1	CJZ1211	JAY ZIP OUTER COVER 12X11	2	ICJZ1011	JAY ZIP INNER COVER 10X11
1	CJZ1212	JAY ZIP OUTER COVER 12X12	2	ICJZ1012	JAY ZIP INNER COVER 10X12
1	CJZ1213	JAY ZIP OUTER COVER 12X13	2	ICJZ1013	JAY ZIP INNER COVER 10X13
1	CJZ1214	JAY ZIP OUTER COVER 12X14	2	ICJZ1014	JAY ZIP INNER COVER 10X14
1	CJZ1215	JAY ZIP OUTER COVER 12X15	2	ICJZ1015	JAY ZIP INNER COVER 10X15
1	CJZ1216	JAY ZIP OUTER COVER 12X16	2	ICJZ1016	JAY ZIP INNER COVER 10X16
1	CJZ1217	JAY ZIP OUTER COVER 12X17	2	ICJZ1108	JAY ZIP INNER COVER 11X08
1	CJZ1218	JAY ZIP OUTER COVER 12X18	2	ICJZ1109	JAY ZIP INNER COVER 11X09
1	CJZ1310	JAY ZIP OUTER COVER 13X10	2	ICJZ1110	JAY ZIP INNER COVER 11X10
1	CJZ1311	JAY ZIP OUTER COVER 13X11	2	ICJZ1111	JAY ZIP INNER COVER 11X11
1	CJZ1312	JAY ZIP OUTER COVER 13X12	2	ICJZ1112	JAY ZIP INNER COVER 11X12
1	CJZ1313	JAY ZIP OUTER COVER 13X13	2	ICJZ1113	JAY ZIP INNER COVER 11X13
1	CJZ1314	JAY ZIP OUTER COVER 13X14	2	ICJZ1114	JAY ZIP INNER COVER 11X14
1	CJZ1315	JAY ZIP OUTER COVER 13X15	2	ICJZ1115	JAY ZIP INNER COVER 11X15
1	CJZ1316	JAY ZIP OUTER COVER 13X16	2	ICJZ1116	JAY ZIP INNER COVER 11X16
1	CJZ1317	JAY ZIP OUTER COVER 13X17	2	ICJZ1117	JAY ZIP INNER COVER 11X17
1	CJZ1318	JAY ZIP OUTER COVER 13X18	2	ICJZ1209	JAY ZIP INNER COVER 12X09
1	CJZ1411	JAY ZIP OUTER COVER 14X11	2	ICJZ1210	JAY ZIP INNER COVER 12X10
1	CJZ1412	JAY ZIP OUTER COVER 14X12	2	ICJZ1211	JAY ZIP INNER COVER 12X11
1	CJZ1413	JAY ZIP OUTER COVER 14X13	2	ICJZ1212	JAY ZIP INNER COVER 12X12
1	CJZ1414	JAY ZIP OUTER COVER 14X14	2	ICJZ1213	JAY ZIP INNER COVER 12X13
1	CJZ1415	JAY ZIP OUTER COVER 14X15	2	ICJZ1214	JAY ZIP INNER COVER 12X14
1	CJZ1416	JAY ZIP OUTER COVER 14X16	2	ICJZ1215	JAY ZIP INNER COVER 12X15
1	CJZ1417	JAY ZIP OUTER COVER 14X17	2	ICJZ1216	JAY ZIP INNER COVER 12X16
1	CJZ1418	JAY ZIP OUTER COVER 14X18	2	ICJZ1217	JAY ZIP INNER COVER 12X17
1	CJZ1512	JAY ZIP OUTER COVER 15X12	2	ICJZ1218	JAY ZIP INNER COVER 12X18
1	CJZ1513	JAY ZIP OUTER COVER 15X13	2	ICJZ1310	JAY ZIP INNER COVER 13X10
1	CJZ1514	JAY ZIP OUTER COVER 15X14	2	ICJZ1311	JAY ZIP INNER COVER 13X11
1	CJZ1515	JAY ZIP OUTER COVER 15X15	2	ICJZ1312	JAY ZIP INNER COVER 13X12
1	CJZ1516	JAY ZIP OUTER COVER 15X16	2	ICJZ1313	JAY ZIP INNER COVER 13X13
1	CJZ1517	JAY ZIP OUTER COVER 15X17	2	ICJZ1314	JAY ZIP INNER COVER 13X14
1	CJZ1518	JAY ZIP OUTER COVER 15X18	2	ICJZ1315	JAY ZIP INNER COVER 13X15
1	CJZ1613	JAY ZIP OUTER COVER 16X13	2	ICJZ1316	JAY ZIP INNER COVER 13X16
1	CJZ1614	JAY ZIP OUTER COVER 16X14	2	ICJZ1317	JAY ZIP INNER COVER 13X17
1	CJZ1615	JAY ZIP OUTER COVER 16X15	2	ICJZ1318	JAY ZIP INNER COVER 13X18
1	CJZ1616	JAY ZIP OUTER COVER 16X16	2	ICJZ1411	JAY ZIP INNER COVER 14X11
1	CJZ1617	JAY ZIP OUTER COVER 16X17	2	ICJZ1412	JAY ZIP INNER COVER 14X12
1	CJZ1618	JAY ZIP OUTER COVER 16X18	2	ICJZ1413	JAY ZIP INNER COVER 14X13
		INNER COVER	2	ICJZ1414	JAY ZIP INNER COVER 14X14
2	ICJZ0808	JAY ZIP INNER COVER 08X08	2	ICJZ1415	JAY ZIP INNER COVER 14X15
2	ICJZ0809	JAY ZIP INNER COVER 08X09	2	ICJZ1416	JAY ZIP INNER COVER 14X16
2	ICJZ0810	JAY ZIP INNER COVER 08X10	2	ICJZ1417	JAY ZIP INNER COVER 14X17
2	ICJZ0811	JAY ZIP INNER COVER 08X11	2	ICJZ1418	JAY ZIP INNER COVER 14X18
2	ICJZ0812	JAY ZIP INNER COVER 08X12	2	ICJZ1512	JAY ZIP INNER COVER 15X12
2	ICJZ0813	JAY ZIP INNER COVER 08X13	2	ICJZ1513	JAY ZIP INNER COVER 15X13
2	ICJZ0814	JAY ZIP INNER COVER 08X14	2	ICJZ1514	JAY ZIP INNER COVER 15X14
2	ICJZ0908	JAY ZIP INNER COVER 09X08	2	ICJZ1515	JAY ZIP INNER COVER 15X15
2	ICJZ0909	JAY ZIP INNER COVER 09X09	2	ICJZ1516	JAY ZIP INNER COVER 15X16
2	ICJZ0910	JAY ZIP INNER COVER 09X10	2	ICJZ1517	JAY ZIP INNER COVER 15X17
2	ICJZ0911	JAY ZIP INNER COVER 09X11	2	ICJZ1518	JAY ZIP INNER COVER 15X18
2	ICJZ0912	JAY ZIP INNER COVER 09X12	2	ICJZ1613	JAY ZIP INNER COVER 16X13
2	ICJZ0913	JAY ZIP INNER COVER 09X13	2	ICJZ1614	JAY ZIP INNER COVER 16X14

Pos.	Item Number	Description	Pos.	Item Number	Description
2	ICJZ1615	JAY ZIP INNER COVER 16X15	3	BJZ1210	JAY ZIP BASE 12X10
2	ICJZ1616	JAY ZIP INNER COVER 16X16	3	BJZ1211	JAY ZIP BASE 12X11
2	ICJZ1617	JAY ZIP INNER COVER 16X17	3	BJZ1212	JAY ZIP BASE 12X12
2	ICJZ1618	JAY ZIP INNER COVER 16X18	3	BJZ1213	JAY ZIP BASE 12X13
		BASE	3	BJZ1214	JAY ZIP BASE 12X14
3	BJZ0808	JAY ZIP BASE 08X08	3	BJZ1215	JAY ZIP BASE 12X15
3	BJZ0809	JAY ZIP BASE 08X09	3	BJZ1216	JAY ZIP BASE 12X16
3	BJZ0810	JAY ZIP BASE 08X10	3	BJZ1217	JAY ZIP BASE 12X17
3	BJZ0811	JAY ZIP BASE 08X11	3	BJZ1218	JAY ZIP BASE 12X18
3	BJZ0812	JAY ZIP BASE 08X12	3	BJZ1310	JAY ZIP BASE 13X10
3	BJZ0813	JAY ZIP BASE 08X13	3	BJZ1311	JAY ZIP BASE 13X11
3	BJZ0814	JAY ZIP BASE 08X14	3	BJZ1312	JAY ZIP BASE 13X12
3	BJZ0908	JAY ZIP BASE 09X08	3	BJZ1313	JAY ZIP BASE 13X13
3	BJZ0909	JAY ZIP BASE 09X09	3	BJZ1314	JAY ZIP BASE 13X14
3	BJZ0910	JAY ZIP BASE 09X10	3	BJZ1315	JAY ZIP BASE 13X15
3	BJZ0911	JAY ZIP BASE 09X11	3	BJZ1316	JAY ZIP BASE 13X16
3	BJZ0912	JAY ZIP BASE 09X12	3	BJZ1317	JAY ZIP BASE 13X17
3	BJZ0913	JAY ZIP BASE 09X13	3	BJZ1318	JAY ZIP BASE 13X18
3	BJZ0914	JAY ZIP BASE 09X14	3	BJZ1411	JAY ZIP BASE 14X11
3	BJZ0915	JAY ZIP BASE 09X15	3	BJZ1412	JAY ZIP BASE 14X12
3	BJZ1008	JAY ZIP BASE 10X08	3	BJZ1413	JAY ZIP BASE 14X13
3	BJZ1009	JAY ZIP BASE 10X09	3	BJZ1414	JAY ZIP BASE 14X14
3	BJZ1010	JAY ZIP BASE 10X10	3	BJZ1415	JAY ZIP BASE 14X15
3	BJZ1011	JAY ZIP BASE 10X11	3	BJZ1416	JAY ZIP BASE 14X16
3	BJZ1012	JAY ZIP BASE 10X12	3	BJZ1417	JAY ZIP BASE 14X17
3	BJZ1013	JAY ZIP BASE 10X13	3	BJZ1418	JAY ZIP BASE 14X18
3	BJZ1014	JAY ZIP BASE 10X14	3	BJZ1512	JAY ZIP BASE 15X12
3	BJZ1015	JAY ZIP BASE 10X15	3	BJZ1513	JAY ZIP BASE 15X13
3	BJZ1016	JAY ZIP BASE 10X16	3	BJZ1514	JAY ZIP BASE 15X14
3	BJZ1108	JAY ZIP BASE 11X08	3	BJZ1515	JAY ZIP BASE 15X15
3	BJZ1109	JAY ZIP BASE 11X09	3	BJZ1516	JAY ZIP BASE 15X16
3	BJZ1110	JAY ZIP BASE 11X10	3	BJZ1517	JAY ZIP BASE 15X17
3	BJZ1111	JAY ZIP BASE 11X11	3	BJZ1518	JAY ZIP BASE 15X18
3	BJZ1112	JAY ZIP BASE 11X12	3	BJZ1613	JAY ZIP BASE 16X13
3	BJZ1113	JAY ZIP BASE 11X13	3	BJZ1614	JAY ZIP BASE 16X14
3	BJZ1114	JAY ZIP BASE 11X14	3	BJZ1615	JAY ZIP BASE 16X15
3	BJZ1115	JAY ZIP BASE 11X15	3	BJZ1616	JAY ZIP BASE 16X16
3	BJZ1116	JAY ZIP BASE 11X16	3	BJZ1617	JAY ZIP BASE 16X17
3	BJZ1117	JAY ZIP BASE 11X17	3	BJZ1618	JAY ZIP BASE 16X18
3	BJZ1209	JAY ZIP BASE 12X09			

Note: Custom Drop Seat is only available for the Jay Soft Combi & Soft Combi P, Jay Gel, Jay Easy, Jay 2 & J2 DC, Jay GS and Jay Triad Cushions.

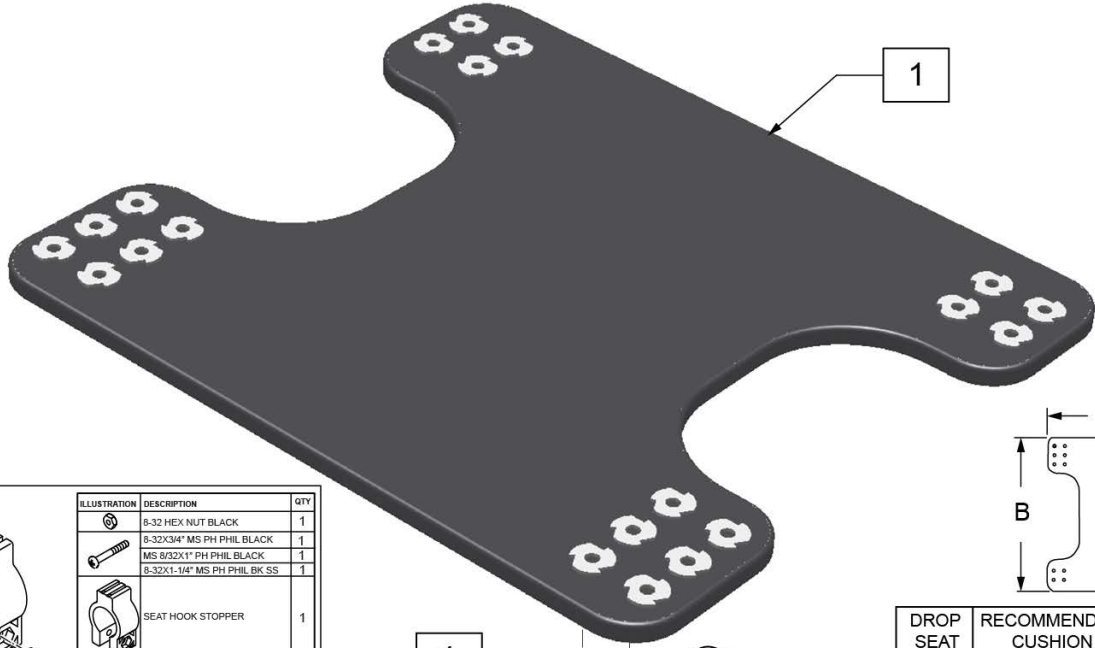

6

ILLUSTRATION	DESCRIPTION	QTY
	8-32 HEX NUT BLACK	1
	8-32X3/4" MS PH PHIL BLACK	1
	MS 8/32X1" PH PHIL BLACK	1
	8-32X1-1/4" MS PH PHIL BK SS	1
	SEAT HOOK STOPPER	1

7

ILLUSTRATION	DESCRIPTION	QTY
	8-32 HEX NUT BLACK	3
	8-32X3/4" MS PH PHIL BLACK	1
	MS 8/32X1" PH PHIL BLACK	3
	8-32X1-1/4" MS PH PHIL BK SS	3
	SEAT HOOK STOPPER	1
	SEAT HOOK RETAINER	2

4

2

3

5 DROP HOOK KITS

ITEM #	ILLUSTRATION	DESCRIPTION	QTY
2		WASHER FLAT AN 1/4"	8
3		BHCS 1/4-20 X 5/8 PL	8
4		DROP HOOK J2DC: Short (1 1/2"), Medium (1"), or Long (1 1/2")	4

DROP SEAT P/N	RECOMMENDED CUSHION SIZE	"A" (WIDTH)	"B" (DEPTH)
SD2444	14 x 14	10.75"	11"
SD2446	14 x 16	10.75"	13"
SD2455	15 x 15	11.75"	12"
SD2457	15 x 17	11.75"	14"
SD2460	16 x 20	12.75"	17"
SD2466	16 x 16	12.75"	13"
SD2468	16 x 18	12.75"	15"
SD2475	17 x 15	13.75"	12"
SD2477	17 x 17	13.75"	14"
SD2486	18 x 16	14.75"	13"
SD2488	18 x 18	14.75"	15"
SD2480	18 x 20	14.75"	17"
SD2406	20 x 16	16.75"	13"
SD2408	20 x 18	16.75"	15"
SD2400	20 x 20	16.75"	17"
SD2426	22 x 16	18.75"	13"
SD2428	22 x 18	18.75"	15"
SD2448	24 x 18	20.75"	15"

Pos.	Item Number	Description	Pos.	Item Number	Description
1	SD2444	SOLID DROP SEAT 14X14 W/HRDWR	1	SD2406	SOLID DROP SEAT 20X16 W/HRDWR
1	SD2446	SOLID DROP SEAT 14X16 W/HRDWR	1	SD2408	SOLID DROP SEAT 20X18 W/HRDWR
1	SD2455	SOLID DROP SEAT 15X15 W/HRDWR	1	SD2400	SOLID DROP SEAT 20X20 W/HRDWR
1	SD2457	SOLID DROP SEAT 15X17 W/HRDWR	1	SD2426	SOLID DROP SEAT 22X16 W/HRDWR
1	SD2460	SOLID DROP SEAT 16X20 W/HRDWR	1	SD2428	SOLID DROP SEAT 22X18 W/HRDWR
1	SD2466	SOLID DROP SEAT 16X16 W/HRDWR	1	SD2448	SOLID DROP SEAT 24X18 W/HRDWR
1	SD2468	SOLID DROP SEAT 16X18 W/HRDWR	5	M-24S	SHORT 1/2" DROP HOOKS
1	SD2475	SOLID DROP SEAT 17X15 W/HRDWR	5	M-24M	MEDIUM 1" DROP HOOKS
1	SD2477	SOLID DROP SEAT 17X17 W/HRDWR	5	M-24L	LONG 1 1/2" DROP HOOKS
1	SD2480	SOLID DROP SEAT 18X20 W/HRDWR	6	283	HOOK STOPPER W/HARDWARE
1	SD2488	SOLID DROP SEAT 18X18 W/HRDWR	7	282	HOOK RETAINER/STOP WITH HARDWARE
1	SD2486	SOLID DROP SEAT 18X16 W/HRDWR			

Note: made of Apple Wood and can also be referred to as a Rigidizer by some dealers.

Note: these Solid Seat Inserts REQUIRE the cushion to have a Zippered Style Outer Cover because they are inserted under the base and held in place by that outer cover; these are not intended to be used for Ballistic (Shower Cap) Edged Style outer covers that are standard on the J2 cushions.

Note: the ONLY MODIFICATIONS available on this Solid Seat Insert are Reducing Width and Reducing Depth/Length.




Pos.	Item Number	Description	Pos.	Item Number	Description
1	JSS0808	JAY SOLID SEAT 8" X 8"	1	JSS1212	JAY SOLID SEAT 12" X 12"
1	JSS0809	JAY SOLID SEAT 8" X 9"	1	JSS1213	JAY SOLID SEAT 12" X 13"
1	JSS0810	JAY SOLID SEAT 8" X 10"	1	JSS1214	JAY SOLID SEAT 12" X 14"
1	JSS0811	JAY SOLID SEAT 8" X 11"	1	JSS1215	JAY SOLID SEAT 12" X 15"
1	JSS0908	JAY SOLID SEAT 9" X 8"	1	JSS1216	JAY SOLID SEAT 12" X 16"
1	JSS0909	JAY SOLID SEAT 9" X 9"	1	JSS1310	JAY SOLID SEAT 13" X 10"
1	JSS0910	JAY SOLID SEAT 9" X 10"	1	JSS1311	JAY SOLID SEAT 13" X 11"
1	JSS0911	JAY SOLID SEAT 9" X 11"	1	JSS1312	JAY SOLID SEAT 13" X 12"
1	JSS0912	JAY SOLID SEAT 9" X 12"	1	JSS1313	JAY SOLID SEAT 13" X 13"
1	JSS1008	JAY SOLID SEAT 10" X 8"	1	JSS1314	JAY SOLID SEAT 13" X 14"
1	JSS1009	JAY SOLID SEAT 10" X 9"	1	JSS1315	JAY SOLID SEAT 13" X 15"
1	JSS1010	JAY SOLID SEAT 10" X 10"	1	JSS1316	JAY SOLID SEAT 13" X 16"
1	JSS1011	JAY SOLID SEAT 10" X 11"	1	JSS1317	JAY SOLID SEAT 13" X 17"
1	JSS1012	JAY SOLID SEAT 10" X 12"	1	JSS1411	JAY SOLID SEAT 14" X 11"
1	JSS1013	JAY SOLID SEAT 10" X 13"	1	JSS1412	JAY SOLID SEAT 14" X 12"
1	JSS1108	JAY SOLID SEAT 11" X 8"	1	JSS1413	JAY SOLID SEAT 14" X 13"
1	JSS1109	JAY SOLID SEAT 11" X 9"	1	JSS1414	JAY SOLID SEAT 14" X 14"
1	JSS1110	JAY SOLID SEAT 11" X 10"	1	JSS1415	JAY SOLID SEAT 14" X 15"
1	JSS 1111	JAY SOLID SEAT 11" X 11"	1	JSS1416	JAY SOLID SEAT 14" X 16"
1	JSS1112	JAY SOLID SEAT 11" X 12"	1	JSS1417	JAY SOLID SEAT 14" X 17"
1	JSS1113	JAY SOLID SEAT 11" X 13"	1	JSS1418	JAY SOLID SEAT 14" X 18"
1	JSS1114	JAY SOLID SEAT 11" X 14'	1	JSS1512	JAY SOLID SEAT 15" X 12"
1	JSS1209	JAY SOLID SEAT 12" X 9"	1	JSS1513	JAY SOLID SEAT 15" X 13"
1	JSS1210	JAY SOLID SEAT 12 X 10"	1	JSS1514	JAY SOLID SEAT 15" X 14"
1	JSS1211	JAY SOLID SEAT 12" X 11"	1	JSS1515	JAY SOLID SEAT 15" X 15"

Pos.	Item Number	Description	Pos.	Item Number	Description
1	JSS1516	JAY SOLID SEAT 15" X 16"	1	JSS1826	JAY SOLID SEAT 18" X 26"
1	JSS1517	JAY SOLID SEAT 15" X 17"	1	JSS1915	JAY SOLID SEAT 19" X 15"
1	JSS1518	JAY SOLID SEAT 15" X 18"	1	JSS1916	JAY SOLID SEAT 19" X 16"
1	JSS1519	JAY SOLID SEAT 15" X 19"	1	JSS1917	JAY SOLID SEAT 19" X 17"
1	JSS1520	JAY SOLID SEAT 15" X 20"	1	JSS1918	JAY SOLID SEAT 19" X 18"
1	JSS1612	JAY SOLID SEAT 16" X 12"	1	JSS1919	JAY SOLID SEAT 19" X 19"
1	JSS1613	JAY SOLID SEAT 16" X 13"	1	JSS1920	JAY SOLID SEAT 19" X 20"
1	JSS1614	JAY SOLID SEAT 16" X 14"	1	JSS2015	JAY SOLID SEAT 20" X 15"
1	JSS1615	JAY SOLID SEAT 16" X 15"	1	JSS2016	JAY SOLID SEAT 20" X 16"
1	JSS1616	JAY SOLID SEAT 16" X 16"	1	JSS2017	JAY SOLID SEAT 20" X 17"
1	JSS1617	JAY SOLID SEAT 16" X 17"	1	JSS2018	JAY SOLID SEAT 20" X 18"
1	JSS1618	JAY SOLID SEAT 16" X 18"	1	JSS2019	JAY SOLID SEAT 20" X 19"
1	JSS1619	JAY SOLID SEAT 16" X 19"	1	JSS2020	JAY SOLID SEAT 20" X 20"
1	JSS1620	JAY SOLID SEAT 16" X 20"	1	JSS2022	JAY SOLID SEAT 20" X 22"
1	JSS1622	JAY SOLID SEAT 16" X 22"	1	JSS2024	JAY SOLID SEAT 20" X 24"
1	JSS1713	JAY SOLID SEAT 17" X 13"	1	JSS2026	JAY SOLID SEAT 20" X 26"
1	JSS1714	JAY SOLID SEAT 17" X 14"	1	JSS2216	JAY SOLID SEAT 22" X 16"
1	JSS1715	JAY SOLID SEAT 17" X 15"	1	JSS2218	JAY SOLID SEAT 22" X 18"
1	JSS1716	JAY SOLID SEAT 17" X 16"	1	JSS2220	JAY SOLID SEAT 22" X 20"
1	JSS1717	JAY SOLID SEAT 17" X 17"	1	JSS2222	JAY SOLID SEAT 22" X 22"
1	JSS1718	JAY SOLID SEAT 17" X 18"	1	JSS2224	JAY SOLID SEAT 22" X 24"
1	JSS1719	JAY SOLID SEAT 17" X 19"	1	JSS2226	JAY SOLID SEAT 22" X 26"
1	JSS1720	JAY SOLID SEAT 17" X 20"	1	JSS2418	JAY SOLID SEAT 24" X 18"
1	JSS1814	JAY SOLID SEAT 18" X 14"	1	JSS2420	JAY SOLID SEAT 24" X 20"
1	JSS1815	JAY SOLID SEAT 18" X 15"	1	JSS2422	JAY SOLID SEAT 24" X 22"
1	JSS1816	JAY SOLID SEAT 18" X 16"	1	JSS2424	JAY SOLID SEAT 24" X 24"
1	JSS1817	JAY SOLID SEAT 18" X 17"	1	JSS2426	JAY SOLID SEAT 24" X 26"
1	JSS1818	JAY SOLID SEAT 18" X 18"	1	JSS2618	JAY SOLID SEAT 26" X 18"
1	JSS1819	JAY SOLID SEAT 18" X 19"	1	JSS2620	JAY SOLID SEAT 26" X 20"
1	JSS1820	JAY SOLID SEAT 18" X 20"	1	JSS2622	JAY SOLID SEAT 26" X 22"
1	JSS1822	JAY SOLID SEAT 18" X 22"	1	JSS2624	JAY SOLID SEAT 26" X 24"
1	JSS1824	JAY SOLID SEAT 18" X 24"			

NOTE: MAY ALSO BE USED WITH JAY AND JAY J2 CUSHIONS. ITEMS 1 & 5 MAY BE USED WITH JAY J2 DEEP CONTOUR CUSHION.

NOTE: ITEMS 1" OR GREATER COMBINED WITH J2 DEEP CONTOUR CUSHION ARE RECOGNIZED BY THE SADMERC AS A K0108 ADJUSTABLE SKIN PROTECTION AND POSITIONING CUSHION.


NOTE: JAY GS ACCESSORY BOX REPLACED BY JAY BOX OF ACCESSORIES (P/N JBOXACCS).

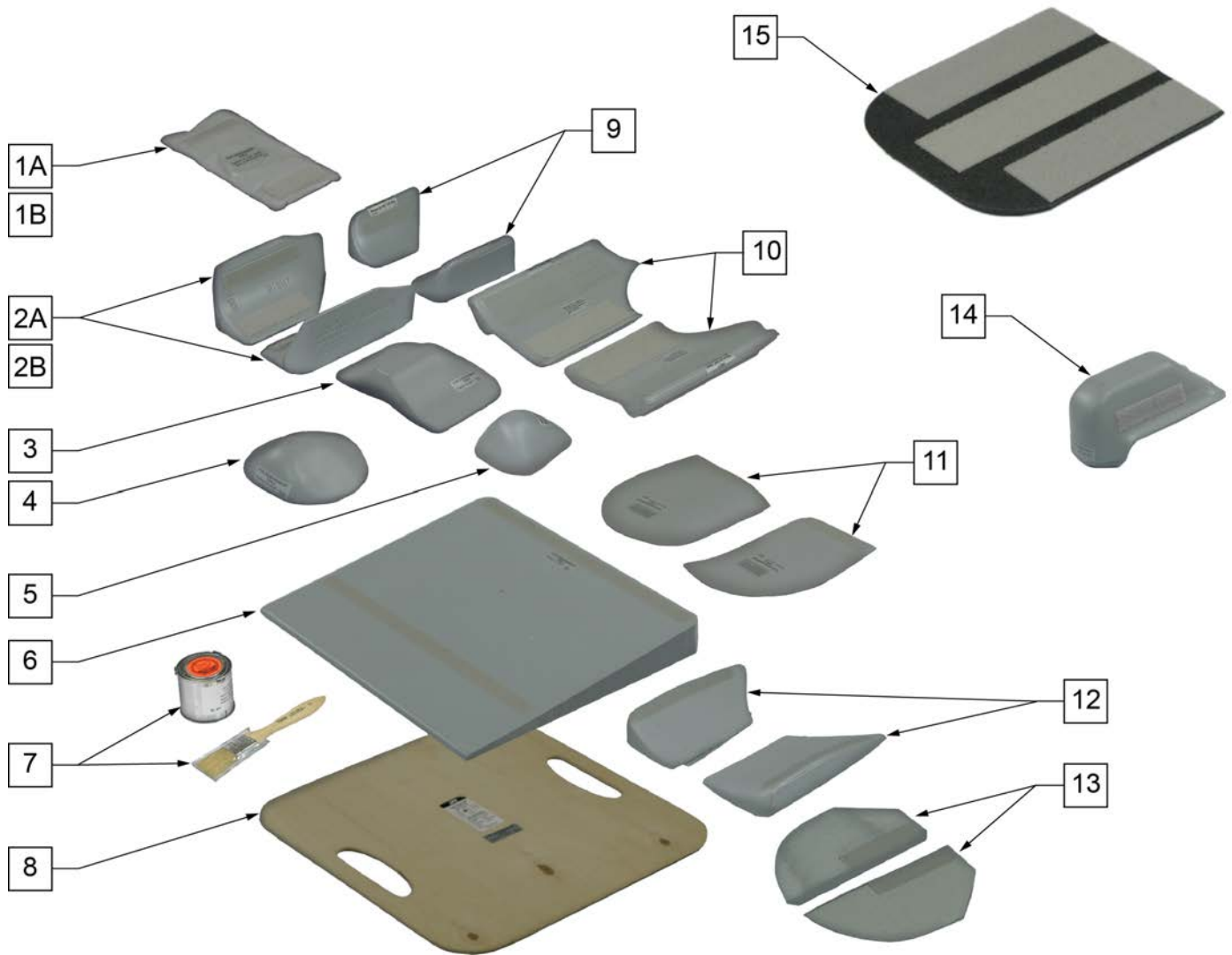

Pos.	Item Number	Description	Remarks
1A	F119	FLUID SUPPLEMENT (8 OZ)	Size 5" x 10.5"
1B	F1119	MINI FLUID SUPPLEMENT (2 OZ)	Size 3.5" x 7.5"
2A	B113J	2" ADDUCTOR WEDGES	Pair, Height 2"
2B	B1113	WEDGES, 3" ADDUCTOR (PAIR)	Pair, Height 3"
3	B118W	2" ABDUCTOR WEDGE	
4	B118	BLD UP,2" ABDUCTOR,JAY	






Pos.	Item Number	Description	Remarks
5	B168	BUILD-UP 2" JUNIOR ABDUCTOR	
6	B114	BASE WEDGE 1 1/2"	
7	086	FRONT PAINT 4 OZ + PAINT BRUSH	
8	S105	SOLID SEAT JAY 15.5X16	
8	S101	SOLID SEAT JAY 18 x 18	
8	S106	SOLID SEAT JAY 15.5 x 18	
8	S108	SOLID SEAT JAY 14 x 16	
8	S111	SOLID SEAT JAY 10 x 10	
8	S115	SOLID SEAT JAY 12 x 11	
9	B163	2" JUNIOR ADDUCTOR WEDGES	Pair
10	B1167	HIP GUIDES,2",JAY & GS	Pair
11	B116R	1" PELVIC OBLIQUITY BLD UP	Right*
11	B116L	1" PELVIC OBLIQUITY BLD UP	Left*
12	B119J	1" ADDUCTOR WEDGES -JAY	Pair
13	B2160L	1/2" PELVIC OBLIQUITY BUILD UP 2100	Left**
13	B2160R	1/2" PELVIC OBLIQUITY BUILD UP 2100	Right**
13	B2161L	1/2" PELVIC OBLIQUITY BUILD UP 2101	Left**
13	B2161R	1/2" PELVIC OBLIQUITY BUILD UP 2101	Right**
13	B2162L	1/2" PELVIC OBLIQUITY BUILD UP 2102	Left**
13	B2162R	1/2" PELVIC OBLIQUITY BUILD UP 2102	Right**
13	B2163L	1/2" PELVIC OBLIQUITY BUILD UP 2103	Left**
13	B2163R	1/2" PELVIC OBLIQUITY BUILD UP 2103	Right**
13	B2164L	1/2" PELVIC OBLIQUITY BUILD UP 2104	Left**
13	B2164R	1/2" PELVIC OBLIQUITY BUILD UP 2104	Right**
13	B2165L	1/2" PELVIC OBLIQUITY BUILD UP 2105	Left**
13	B2165R	1/2" PELVIC OBLIQUITY BUILD UP 2105	Right**
13	B2166L	1/2" PELVIC OBLIQUITY BUILD UP 2106	Left**
13	B2166R	1/2" PELVIC OBLIQUITY BUILD UP 2106	Right**
13	B2167L	1/2" PELVIC OBLIQUITY BUILD UP 2107	Left**
13	B2167R	1/2" PELVIC OBLIQUITY BUILD UP 2107	Right**
13	B2168L	1/2" PELVIC OBLIQUITY BUILD UP 2108	Left**
13	B2168R	1/2" PELVIC OBLIQUITY BUILD UP 2108	Right**
13	B2169L	1/2" PELVIC OBLIQUITY BUILD UP 2116	Left**
13	B2169R	1/2" PELVIC OBLIQUITY BUILD UP 2116	Right**
13	B2177L	1/2" PELVIC OBLIQUITY BUILD UP 2117	Left**
13	B2177R	1/2" PELVIC OBLIQUITY BUILD UP 2117	Right**
13	B2170L	1/2" PELVIC OBLIQUITY BUILD UP 2118	Left**
13	B2170R	1/2" PELVIC OBLIQUITY BUILD UP 2118	Right**
14	B1118	3" ABDUCTOR BUILD-UP	
15	K1100	WELL BUILD-UP GS 18"	
15	K1105	WELL BUILD-UP GS 16" 1	
15	K1110	WELL BUILD-UP GS 14"	
15	K1115	WELL BUILD-UP GS (12")	
1-12,14,15	JBOXACCS	JAY BOX OF ACCESSORIES	
6,10,11	156	CHILDREN'S ACCESSORIES KIT	

1

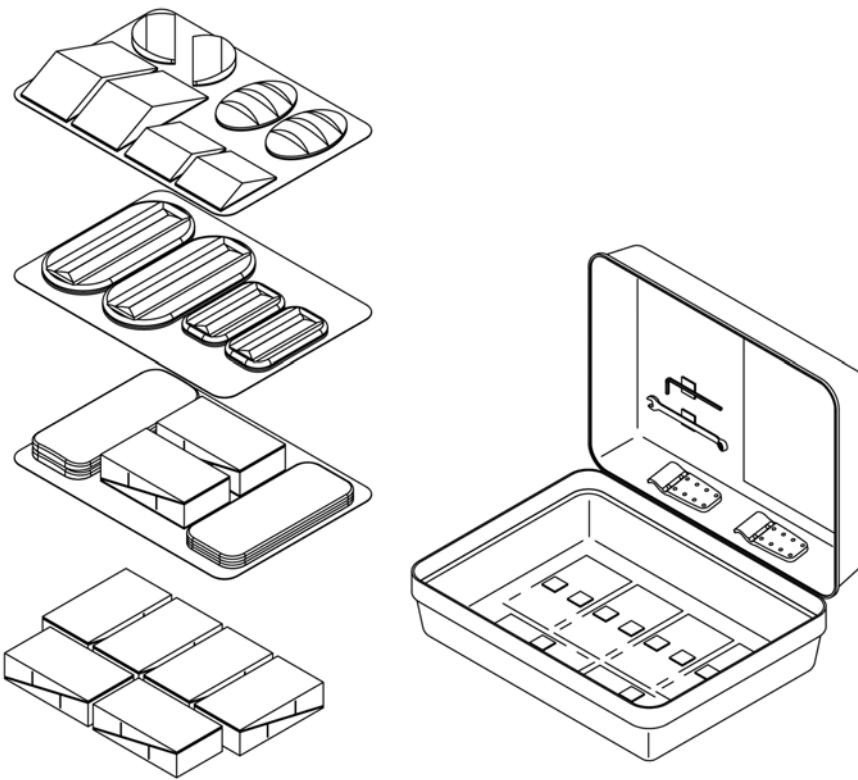

DESCRIPTION	QTY
BACK WEDGE BLK 3.5W W/VEL	16
LATERAL LUMBAR SUPP BLK W/VEL	2
LATERAL SUPP WEDGE BLK W/VEL	2
LUMBAR SUPP LG BLK 3.5W W/VEL	2
FLAT SHIM	2
CURVED SHIM	2
LUMBAR SUPP SM BLK 3.5W W/VEL	2
LARGE BEAN BAG	2
SMALL BEAN BAG	2
VARI DIA CLAMP INNER LONG	2
COMBO WRENCH 10MM	1
HEX KEY 4MM LONG	1
CASE INSERT 1 W/VEL J3 ASSES	1
CASE INSERT 2 W/VEL J3 ASSES	1
CASE INSERT 3 W/VEL J3 ASSES	1
CASE J3 SPINE ALIGN ASSES KIT	1

Pos.	Item Number	Description
1	DJ3SPNKA	ASSESSMENT KIT, SPINE ALIGN

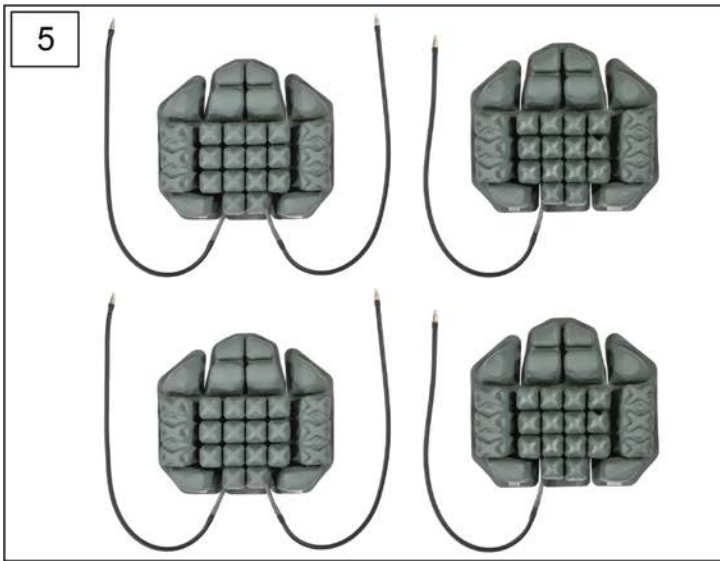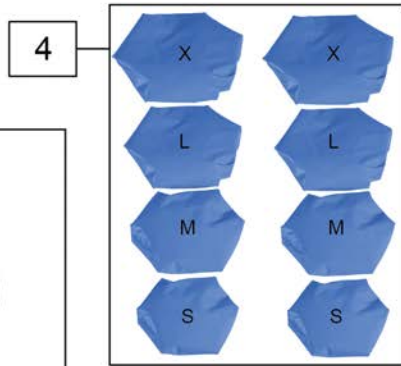
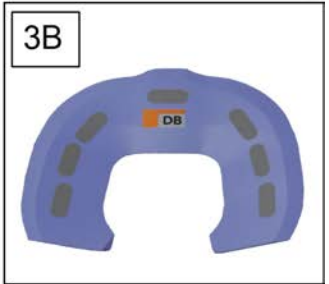
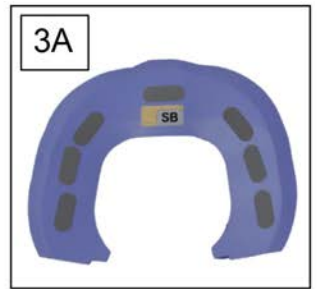
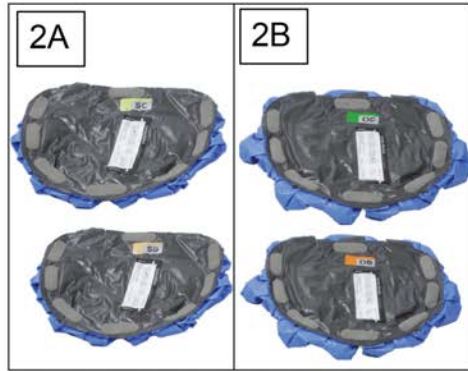


Pos.	Item Number	Description
1,2A,3A,4-7	J3CEKS	J3 CUSH STD PLA EVAL KIT
1,2B,3B,4-7	J3CEKD	J3 CUSH DP PLA EVAL KIT



Pos.	Item Number	Description	Remarks
2	J3CALM1	LATERAL THIGH 1" HEIGHT MEDIUM	8.25" LONG
2	J3CALL1	LATERAL THIGH 1" HEIGHT LARGE	11" LONG
2	J3CALM2	LATERAL THIGH 2" HEIGHT MEDIUM	8.25" LONG
2	J3CALL2	LATERAL THIGH 2" HEIGHT LARGE	11" LONG
3	J3CMM1	MED THIGH 1" HEIGHT MEDIUM	6" LONG
3	J3CAML1	MED THIGH 1" HEIGHT LARGE	8.5" LONG
3	J3CAMS2	MED THIGH 2" HEIGHT SMALL	3.5" LONG
3	J3CMM2	MED THIGH 2" HEIGHT MEDIUM	6" LONG
3	J3CAML2	MED THIGH 2" HEIGHT LARGE	8.5" LONG
4	J3CAPS2	LAT PEL 2" HEIGHT SMALL	4.75" LONG
4	J3CAPM2	LAT PEL 2" HEIGHT MEDIUM	6.25" LONG
5	J3CSAOR	OBLIQUITY PAD STD A RIGHT	1/2" in height
5	J3CSAOL	OBLIQUITY PAD STD A LEFT	1/2" in height
5	J3CSBOR	OBLIQUITY PAD STD B RIGHT	1/2" in height
5	J3CSBOL	OBLIQUITY PAD STD B LEFT	1/2" in height
5	J3CSCOR	OBLIQUITY PAD STD C RIGHT	1/2" in height
5	J3CSCOL	OBLIQUITY PAD STD C LEFT	1/2" in height
5	J3CDAOR	OBLIQUITY PAD DP A RIGHT	1/2" in height
5	J3CDAOL	OBLIQUITY PAD DP A LEFT	1/2" in height
5	J3CDBOR	OBLIQUITY PAD DP B RIGHT	1/2" in height
5	J3CDBOL	OBLIQUITY PAD DP B LEFT	1/2" in height
5	J3CDCOR	OBLIQUITY PAD DP C RIGHT	1/2" in height
5	J3CDCOL	OBLIQUITY PAD DP C LEFT	1/2" in height
6	J3CPRT	REPLACEMENT TAPE 30' ROLL	
1-6	J3CPEK	J3 CUSHION POSITIONING EVAL KIT	

Note: Custom Drop Seat is only available for the Jay Soft Combi & Soft Combi P, Jay Gel, Jay Easy, Jay 2 & J2 DC, Jay GS and Jay Triad Cushions.

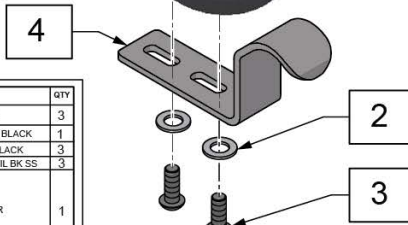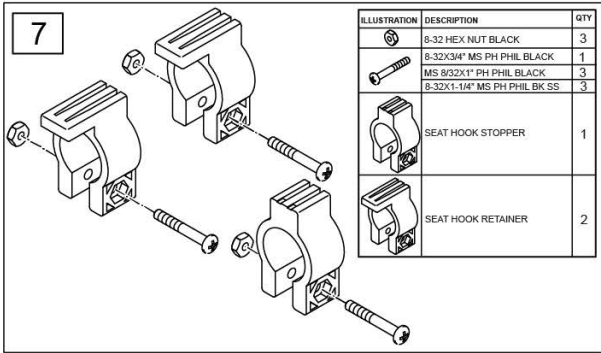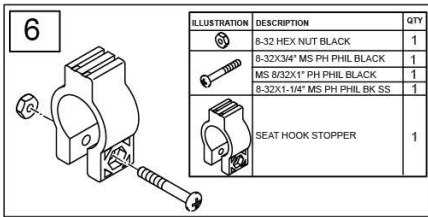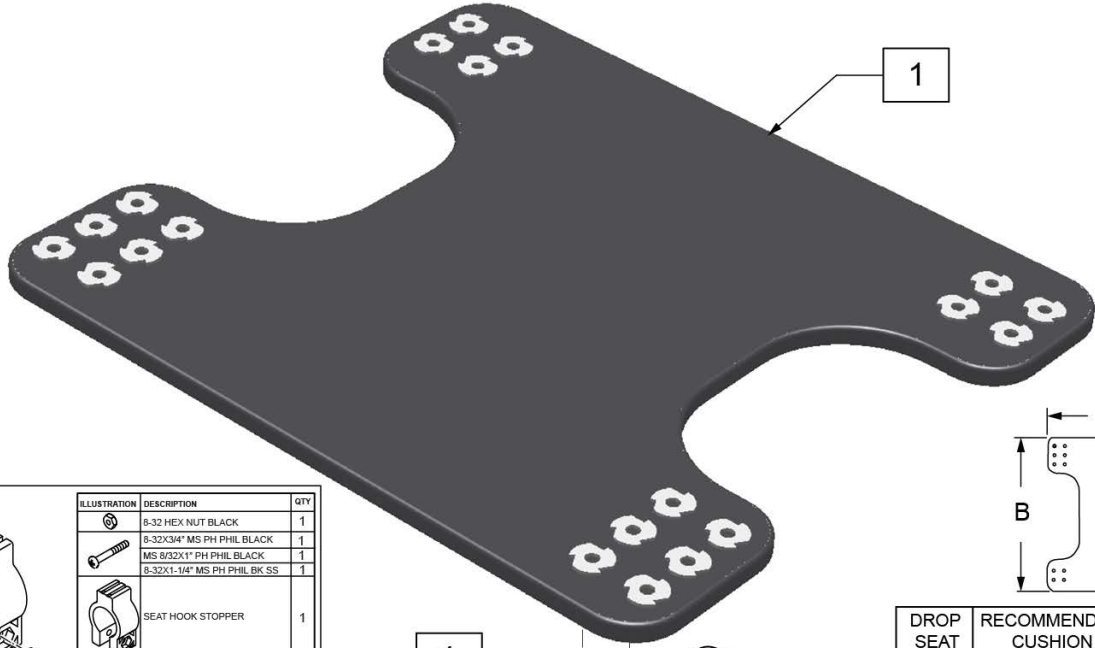


5] DROP HOOK KITS			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
2		WASHER FLAT AN 1/4"	8
3		BHCS 1/4-20 X 5/8 PL	8
4		DROP HOOK J2DC: Short (1 1/2"), Medium (1"), or Long (1 1/2")	4

DROP SEAT P/N	RECOMMENDED CUSHION SIZE	"A" (WIDTH)	"B" (DEPTH)
SD2444	14 x 14	10.75"	11"
SD2446	14 x 16	10.75"	13"
SD2455	15 x 15	11.75"	12"
SD2457	15 x 17	11.75"	14"
SD2460	16 x 20	12.75"	17"
SD2466	16 x 16	12.75"	13"
SD2468	16 x 18	12.75"	15"
SD2475	17 x 15	13.75"	12"
SD2477	17 x 17	13.75"	14"
SD2486	18 x 16	14.75"	13"
SD2488	18 x 18	14.75"	15"
SD2480	18 x 20	14.75"	17"
SD2406	20 x 16	16.75"	13"
SD2408	20 x 18	16.75"	15"
SD2400	20 x 20	16.75"	17"
SD2426	22 x 16	18.75"	13"
SD2428	22 x 18	18.75"	15"
SD2448	24 x 18	20.75"	15"

Pos.	Item Number	Description	Pos.	Item Number	Description
1	SD2444	SOLID DROP SEAT 14X14 W/HRDWR	1	SD2406	SOLID DROP SEAT 20X16 W/HRDWR
1	SD2446	SOLID DROP SEAT 14X16 W/HRDWR	1	SD2408	SOLID DROP SEAT 20X18 W/HRDWR
1	SD2455	SOLID DROP SEAT 15X15 W/HRDWR	1	SD2400	SOLID DROP SEAT 20X20 W/HRDWR
1	SD2457	SOLID DROP SEAT 15X17 W/HRDWR	1	SD2426	SOLID DROP SEAT 22X16 W/HRDWR
1	SD2460	SOLID DROP SEAT 16X20 W/HRDWR	1	SD2428	SOLID DROP SEAT 22X18 W/HRDWR
1	SD2466	SOLID DROP SEAT 16X16 W/HRDWR	1	SD2448	SOLID DROP SEAT 24X18 W/HRDWR
1	SD2468	SOLID DROP SEAT 16X18 W/HRDWR	5	M-24S	SHORT 1/2" DROP HOOKS
1	SD2475	SOLID DROP SEAT 17X15 W/HRDWR	5	M-24M	MEDIUM 1" DROP HOOKS
1	SD2477	SOLID DROP SEAT 17X17 W/HRDWR	5	M-24L	LONG 1 1/2" DROP HOOKS
1	SD2480	SOLID DROP SEAT 18X20 W/HRDWR	6	283	HOOK STOPPER W/HARDWARE
1	SD2488	SOLID DROP SEAT 18X18 W/HRDWR	7	282	HOOK RETAINER/STOP WITH HARDWARE
1	SD2486	SOLID DROP SEAT 18X16 W/HRDWR			

Note: item 11 requires drilling of (2) 7/32" holes for proper mounting and is used to keep the cushion from sliding off the seat.

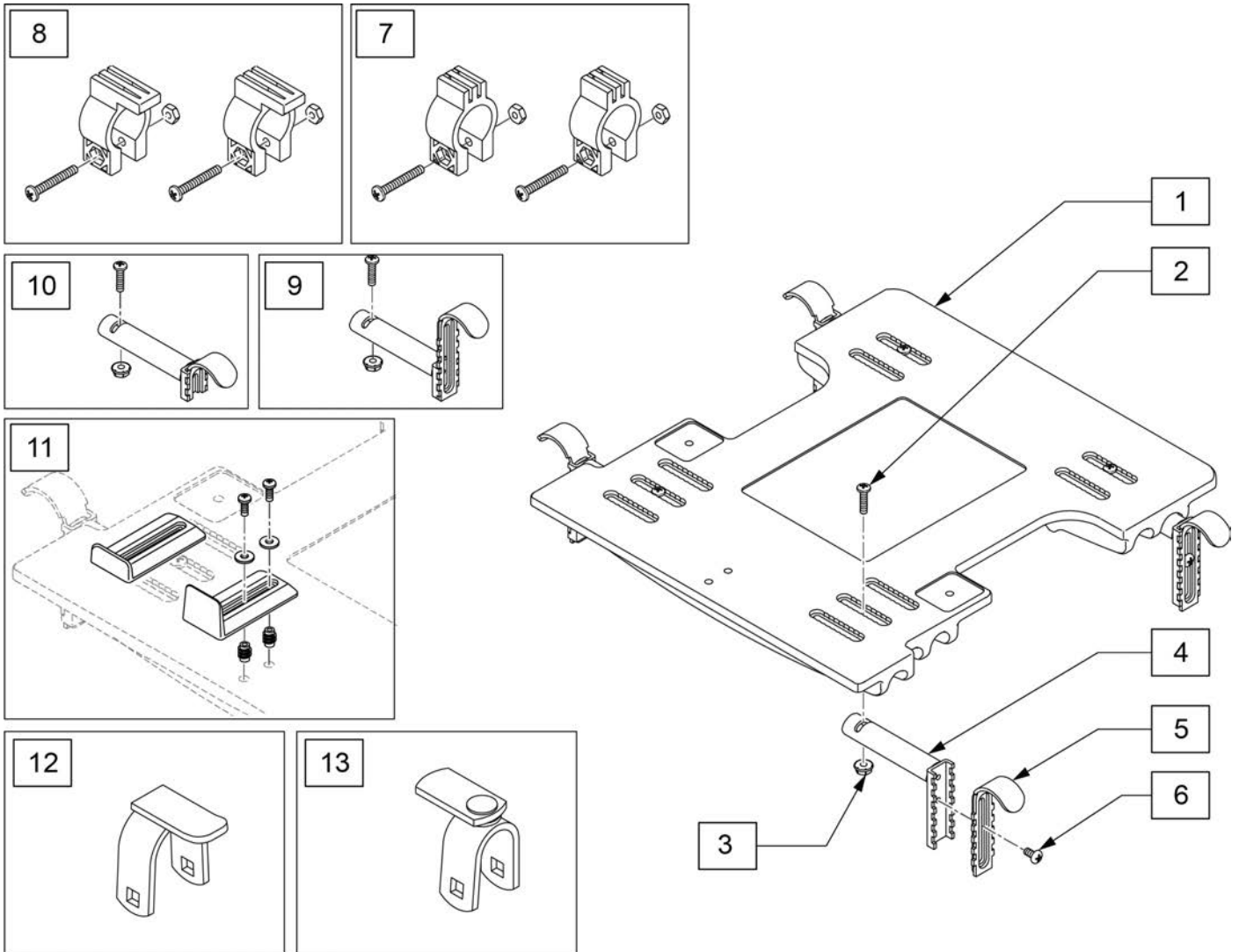

Pos.	Item Number	Description	Remarks
2	M-271	1/4"X20X1" BOLT	
3	M-272	WHIZ NUT	
7	283	HOOK STOPPER W/HARDWARE	for 1" round tubing (Q2, Quickie/Zippie IRIS) or 1" ovalized tubing (Q7, Zone & 7r)

Pos.	Item Number	Description	Remarks
8	284	HOOK RETAINER W/HARDWARE	for 1" round tubing (Q2, Quickie/Zippie IRIS) or 1" ovalized tubing (Q7, Zone & 7r)
9	287	HOOK&BRACKET MOD. 1 3/4"DROP	
9	298	HOOK & BRACKET MODFID MAX DROP	
10	286-MJ	HOOK & BRACKET MOD. FLUSH MOUNT	
10	296	HOOK & "L" ASSM FOR 20" FLUSH	
11	M-285	LIP KIT - FOR ADJUSTABLE SOLID SEAT	
2-6	2HL	HOOK & BRACKET - FASTENED	
2-6	3HL	LRG AD HOOK & BRACKET-FASTENED	
2-6	4HL	HOOK & "L"-FASTENED FOR 10"	
2-8	285	JAY SOLID SEAT HARDWARE PACKET	
1-8	270	SOLID SEAT ADJUSTABLE 15-18 X 12-18	INCLUDES HARDWARE P/N 2HL
1-8	271	SOLID SEAT ADJUSTABLE 10/11 X 10-12	INCLUDES HARDWARE P/N 4HL
1-8	272	SOLID SEAT ADJUSTABLE 11.5-14.5 X 10-16	INCLUDES HARDWARE P/N 2HL
1-8	270.1	SOLID SEAT ADJUSTABLE 15-18 W/2.5" DROP OP	INCLUDES HARDWARE P/N 3HL
1-8	271.1	SOLID SEAT ADJUSTABLE 10-11 W/2.5" DROP OP	INCLUDES HARDWARE P/N 6HL
1-8	272.1	SOLID SEAT ADJUSTABLE 11.5-14.5 W/2.5" DRP OP	INCLUDES HARDWARE P/N 3HL
1-8	273.1	SOLID SEAT ADJUSTABLE 18.5-22 X 12-20	INCLUDES HARDWARE P/N 3HL
1-8,11	270L	SOLID SEAT ADJUSTABLE W/LIP KIT	
1-8,11	271L	SOLID SEAT ADJUSTABLE 10" W/LIP KIT	
1-8,11	272L	SOLID SEAT ADJUSTABLE W/LIP KIT	
1-8,11	270.1L	SOLID SEAT ADJUSTABLE 15-18 W/2.5" DRP,LIP KT	
1-8,11	271.1L	SOLID SEAT ADJUSTABLE 10-11 W/2.5" DRP,LIP KT	
1-8,11	272.1L	SOLID SEAT ADJUSTABLE 11.5-14.5 /2.5 DRP,LIP	
1-8,11	273.1L	SOLID SEAT ADJUSTABLE LARGE ADULT W/LIP KIT	
12	HDW6461	DROP STOP 1-1/8 HLD DWN	For 1 1/8" Round tubing (QRi/Quickie 5r) 4 R'qd
13	129262	LONG DROP STOP W/ ROTATING	For the Zippie X'Cape "VersaRail" 4 R'qd

Note: made of Apple Wood and can also be referred to as a Rigidizer by some dealers.

Note: these Solid Seat Inserts REQUIRE the cushion to have a Zippered Style Outer Cover because they are inserted under the base and held in place by that outer cover; these are not intended to be used for Ballistic (Shower Cap) Edged Style outer covers that are standard on the J2 cushions.

Note: the ONLY MODIFICATIONS available on this Solid Seat Insert are Reducing Width and Reducing Depth/Length.




Pos.	Item Number	Description	Pos.	Item Number	Description
1	JSS0808	JAY SOLID SEAT 8" X 8"	1	JSS1212	JAY SOLID SEAT 12" X 12"
1	JSS0809	JAY SOLID SEAT 8" X 9"	1	JSS1213	JAY SOLID SEAT 12" X 13"
1	JSS0810	JAY SOLID SEAT 8" X 10"	1	JSS1214	JAY SOLID SEAT 12" X 14"
1	JSS0811	JAY SOLID SEAT 8" X 11"	1	JSS1215	JAY SOLID SEAT 12" X 15"
1	JSS0908	JAY SOLID SEAT 9" X 8"	1	JSS1216	JAY SOLID SEAT 12" X 16"
1	JSS0909	JAY SOLID SEAT 9" X 9"	1	JSS1310	JAY SOLID SEAT 13" X 10"
1	JSS0910	JAY SOLID SEAT 9" X 10"	1	JSS1311	JAY SOLID SEAT 13" X 11"
1	JSS0911	JAY SOLID SEAT 9" X 11"	1	JSS1312	JAY SOLID SEAT 13" X 12"
1	JSS0912	JAY SOLID SEAT 9" X 12"	1	JSS1313	JAY SOLID SEAT 13" X 13"
1	JSS1008	JAY SOLID SEAT 10" X 8"	1	JSS1314	JAY SOLID SEAT 13" X 14"
1	JSS1009	JAY SOLID SEAT 10" X 9"	1	JSS1315	JAY SOLID SEAT 13" X 15"
1	JSS1010	JAY SOLID SEAT 10" X 10"	1	JSS1316	JAY SOLID SEAT 13" X 16"
1	JSS1011	JAY SOLID SEAT 10" X 11"	1	JSS1317	JAY SOLID SEAT 13" X 17"
1	JSS1012	JAY SOLID SEAT 10" X 12"	1	JSS1411	JAY SOLID SEAT 14" X 11"
1	JSS1013	JAY SOLID SEAT 10" X 13"	1	JSS1412	JAY SOLID SEAT 14" X 12"
1	JSS1108	JAY SOLID SEAT 11" X 8"	1	JSS1413	JAY SOLID SEAT 14" X 13"
1	JSS1109	JAY SOLID SEAT 11" X 9"	1	JSS1414	JAY SOLID SEAT 14" X 14"
1	JSS1110	JAY SOLID SEAT 11" X 10"	1	JSS1415	JAY SOLID SEAT 14" X 15"
1	JSS 1111	JAY SOLID SEAT 11" X 11"	1	JSS1416	JAY SOLID SEAT 14" X 16"
1	JSS1112	JAY SOLID SEAT 11" X 12"	1	JSS1417	JAY SOLID SEAT 14" X 17"
1	JSS1113	JAY SOLID SEAT 11" X 13"	1	JSS1418	JAY SOLID SEAT 14" X 18"
1	JSS1114	JAY SOLID SEAT 11" X 14'	1	JSS1512	JAY SOLID SEAT 15" X 12"
1	JSS1209	JAY SOLID SEAT 12" X 9"	1	JSS1513	JAY SOLID SEAT 15" X 13"
1	JSS1210	JAY SOLID SEAT 12 X 10"	1	JSS1514	JAY SOLID SEAT 15" X 14"
1	JSS1211	JAY SOLID SEAT 12" X 11"	1	JSS1515	JAY SOLID SEAT 15" X 15"

Pos.	Item Number	Description	Pos.	Item Number	Description
1	JSS1516	JAY SOLID SEAT 15" X 16"	1	JSS1826	JAY SOLID SEAT 18" X 26"
1	JSS1517	JAY SOLID SEAT 15" X 17"	1	JSS1915	JAY SOLID SEAT 19" X 15"
1	JSS1518	JAY SOLID SEAT 15" X 18"	1	JSS1916	JAY SOLID SEAT 19" X 16"
1	JSS1519	JAY SOLID SEAT 15" X 19"	1	JSS1917	JAY SOLID SEAT 19" X 17"
1	JSS1520	JAY SOLID SEAT 15" X 20"	1	JSS1918	JAY SOLID SEAT 19" X 18"
1	JSS1612	JAY SOLID SEAT 16" X 12"	1	JSS1919	JAY SOLID SEAT 19" X 19"
1	JSS1613	JAY SOLID SEAT 16" X 13"	1	JSS1920	JAY SOLID SEAT 19" X 20"
1	JSS1614	JAY SOLID SEAT 16" X 14"	1	JSS2015	JAY SOLID SEAT 20" X 15"
1	JSS1615	JAY SOLID SEAT 16" X 15"	1	JSS2016	JAY SOLID SEAT 20" X 16"
1	JSS1616	JAY SOLID SEAT 16" X 16"	1	JSS2017	JAY SOLID SEAT 20" X 17"
1	JSS1617	JAY SOLID SEAT 16" X 17"	1	JSS2018	JAY SOLID SEAT 20" X 18"
1	JSS1618	JAY SOLID SEAT 16" X 18"	1	JSS2019	JAY SOLID SEAT 20" X 19"
1	JSS1619	JAY SOLID SEAT 16" X 19"	1	JSS2020	JAY SOLID SEAT 20" X 20"
1	JSS1620	JAY SOLID SEAT 16" X 20"	1	JSS2022	JAY SOLID SEAT 20" X 22"
1	JSS1622	JAY SOLID SEAT 16" X 22"	1	JSS2024	JAY SOLID SEAT 20" X 24"
1	JSS1713	JAY SOLID SEAT 17" X 13"	1	JSS2026	JAY SOLID SEAT 20" X 26"
1	JSS1714	JAY SOLID SEAT 17" X 14"	1	JSS2216	JAY SOLID SEAT 22" X 16"
1	JSS1715	JAY SOLID SEAT 17" X 15"	1	JSS2218	JAY SOLID SEAT 22" X 18"
1	JSS1716	JAY SOLID SEAT 17" X 16"	1	JSS2220	JAY SOLID SEAT 22" X 20"
1	JSS1717	JAY SOLID SEAT 17" X 17"	1	JSS2222	JAY SOLID SEAT 22" X 22"
1	JSS1718	JAY SOLID SEAT 17" X 18"	1	JSS2224	JAY SOLID SEAT 22" X 24"
1	JSS1719	JAY SOLID SEAT 17" X 19"	1	JSS2226	JAY SOLID SEAT 22" X 26"
1	JSS1720	JAY SOLID SEAT 17" X 20"	1	JSS2418	JAY SOLID SEAT 24" X 18"
1	JSS1814	JAY SOLID SEAT 18" X 14"	1	JSS2420	JAY SOLID SEAT 24" X 20"
1	JSS1815	JAY SOLID SEAT 18" X 15"	1	JSS2422	JAY SOLID SEAT 24" X 22"
1	JSS1816	JAY SOLID SEAT 18" X 16"	1	JSS2424	JAY SOLID SEAT 24" X 24"
1	JSS1817	JAY SOLID SEAT 18" X 17"	1	JSS2426	JAY SOLID SEAT 24" X 26"
1	JSS1818	JAY SOLID SEAT 18" X 18"	1	JSS2618	JAY SOLID SEAT 26" X 18"
1	JSS1819	JAY SOLID SEAT 18" X 19"	1	JSS2620	JAY SOLID SEAT 26" X 20"
1	JSS1820	JAY SOLID SEAT 18" X 20"	1	JSS2622	JAY SOLID SEAT 26" X 22"
1	JSS1822	JAY SOLID SEAT 18" X 22"	1	JSS2624	JAY SOLID SEAT 26" X 24"
1	JSS1824	JAY SOLID SEAT 18" X 24"			

Note: also known as the Honeycomb Solid Seat Insert, which is inside the non-skid cover material and is made of an aluminum core.

Note: use with Ballistic Edge (Show Cap) style outer cover options; traditionally used with J2, since the standard cover style is Ballistic for J2.

Note: only the outer non skid material can be cut down in width and depth; the Honeycomb insert is not modifiable.

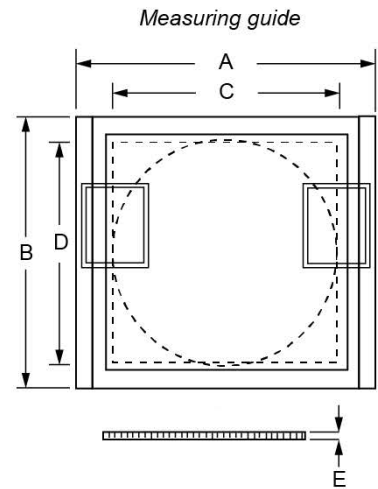

Note: the dotted square in the middle of the above diagram is the Honeycomb Insert and the dotted round in the middle of that is the foam between the non skid cover and the Honeycomb insert.

Part #	Complete Insert		Honeycomb Insert		
	Dim A	Dim B	Dim C	Dim D	Dim E
S2108	13.50"	16.00"	9.25"	13.00"	0.25"
S2105	15.00"	16.00"	11.25"	13.00"	0.25"
S2106	15.00"	18.00"	11.25"	15.00"	0.25"
S2107	15.00"	20.00"	11.25"	17.00"	0.25"
S2117	16.50"	17.00"	12.25"	14.00"	0.25"
S2100	17.50"	16.00"	13.25"	13.00"	0.25"
S2101	17.50"	18.00"	13.25"	15.00"	0.25"
S2103	17.50"	20.00"	13.25"	17.00"	0.25"
S2116	19.50"	16.00"	15.25"	13.00"	0.25"
S2118	19.50"	18.00"	15.25"	15.00"	0.25"
S2102	19.50"	20.00"	15.25"	17.00"	0.25"
S2104	23.50"	20.00"	19.25"	17.00"	0.25"

Pos.	Item Number	Description
1	S2108	SOLID SEAT J2 NON SKID 14 X 16
1	S2105	SOLID SEAT J2 NON SKID 15.5 X 16
1	S2106	SOLID SEAT J2 NON SKID 15.5 X 18
1	S2107	SOLID SEAT J2 NON SKID 15.5 X 20
1	S2117	SOLID SEAT J2 NON SKID 17 X 17
1	S2100	SOLID SEAT J2 NON SKID 18 X 16
1	S2101	SOLID SEAT J2 NON SKID 18 X 18
1	S2103	SOLID SEAT J2 NON SKID 18 X 20
1	S2116	SOLID SEAT J2 NON SKID 20 X 16
1	S2102	SOLID SEAT J2 NON SKID 20 X 20
1	S2104	SOLID SEAT J2 NON SKID 24 X 20




MEASURING GUIDE

	p/n 142 Standard	p/n 145 Junior
Width of Insert	14 3/4"	11"
Depth of Insert	9 1/2"	7"
Height (Front to Rear)	1/2" to 2"	1/2" to 1 1/2"
Waist Strap Length	21 1/4" to 42 3/4"	18" to 43 3/4"
Leg Strap Length	8" to 21"	3" to 21"

Pos.	Item Number	Description
1A	C141	COVER PROTECTOR CUSHION
1B	C144	JUNIOR COVER PROTECTOR CUSHION
2A	141	PROTECTOR CUSHION
2B	144	JUNIOR PROTECTOR CUSHION
3A	142	PROTECTOR SLING
3B	145	JUNIOR PROTECTOR SLING
1A,2A,3A	140	PROTECTOR CUSHION AND SLING
1B,2B,3B	143-MJ	JUNIOR PROTECTOR CUSHION & SLING

Note: Spinal Fluid Pads can be added to any Jay Back with a hard shell.

Note: The Spinal Fluid Pad measures 12" tall but the pad can be cut down to accommodate shorter backs by cutting in between the cells to remove cells that are not needed

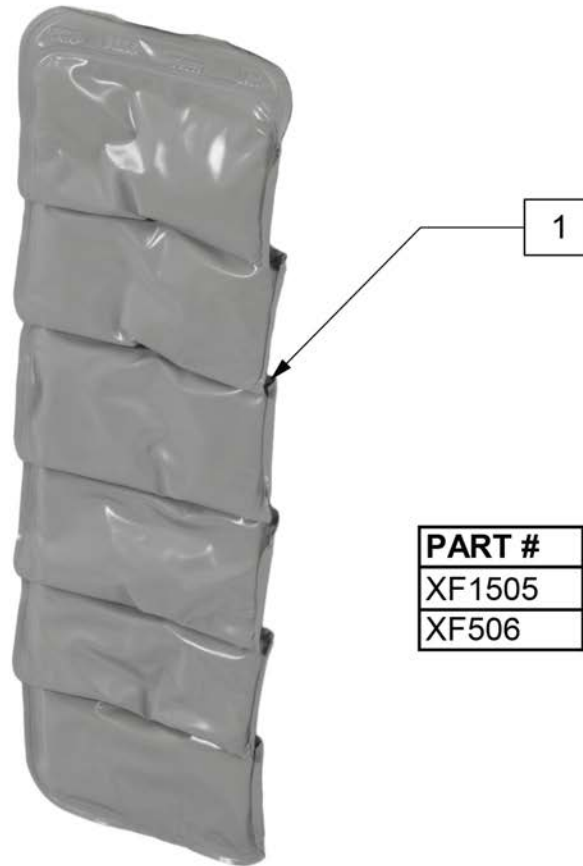

PART #	HEIGHT
XF1505	20"
XF506	23"

Pos.	Item Number	Description
1	MF506	FLUID PAD TALL JAY BACK



Pos.	Item Number	Description	Remarks
1	CCFOOTS22	FUL FOOTBX 22W SEP CALF CVR SD	Cover only
1	CCFOOTS20	FUL FOOTBX 20W SEP CALF CVR SD	Cover only
1	CCFOOTS18	FUL FOOTBX 18W SEP CALF CVR SD	Cover only
1	CCFOOTS16	FUL FOOTBX 16W SEP CALF CVR SD	Cover only
1	CCFOOTS22	FUL FOOTBX 22W SEP CALF CVR TL	Cover only
1	CCFOOTS20	FUL FOOTBX 20W SEP CALF CVR TL	Cover only
1	CCFOOTS18	FUL FOOTBX 18W SEP CALF CVR TL	Cover only
1	CCFOOTS16	FUL FOOTBX 16W SEP CALF CVR TL	Cover only
2	CCFOOTS22	FULL FOOTBOX 22W FOOT COVER	Cover only
2	CCFOOTS20	FULL FOOTBOX 20W FOOT COVER	Cover only
2	CCFOOTS18	FULL FOOTBOX 18W FOOT COVER	Cover only
2	CCFOOTS16	FULL FOOTBOX 16W FOOT COVER	Cover only
5	132826	JAY FOOTBOARD ASM 12W KIT	Cover, foam & board
5	132827	JAY FOOTBOARD ASM 14W KIT	Cover, foam & board
5	132828	JAY FOOTBOARD ASM 16W KIT	Cover, foam & board
5	132829	JAY FOOTBOARD ASM 18W KIT	Cover, foam & board
5	132830	JAY FOOTBOARD ASM 20W KIT	Cover, foam & board
5	132831	JAY FOOTBOARD ASM 22W KIT	Cover, foam & board

Pos.	Item Number	Description	Remarks
6	HDWFOOT	DUAL FOOTBOX HRDWR KIT	4 screws and washers



5 DUAL FOOTBOX HARDWARE KIT			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
3		BHCS 1/4-20 X 3/4	4
4		9/32ID 1OD .11-14INTHK WASHER	4

FOOT BOX WIDTH MATRIX	
Flared Frame Formula: (Seat Width) +2" = Foot Box Width	
Offset Hanger Receiver Formula: (Seat Width) +2" = Foot Box Width	
Inset Formulas:	
0" Inset (Seat Width)	= Foot Box Width
1" Inset (Seat Width) -2"	= Foot Box Width
2" Inset (Seat Width) -4"	= Foot Box Width


Pos.	Item Number	Description	Remarks
1	CCFOOT22	FUL FOOTBX 22W CALF CVR STD	Cover only - no pads included
1	CCFOOT20	FUL FOOTBX 20W CALF CVR STD	Cover only - no pads included
1	CCFOOT18	FUL FOOTBX 18W CALF CVR STD	Cover only - no pads included
1	CCFOOT16	FUL FOOTBX 16W CALF CVR STD	Cover only - no pads included
1	CCFOOT14	FUL FOOTBX 14W CALF CVR STD	Cover only - no pads included
1	CCFOOT12	FUL FOOTBX 12W CALF CVR STD	Cover only - no pads included
1	CCFOOTT22	FUL FOOTBX 22W CALF CVR TAL	Cover only - no pads included
1	CCFOOTT20	FUL FOOTBX 20W CALF CVR TAL	Cover only - no pads included
1	CCFOOTT18	FUL FOOTBX 18W CALF CVR TAL	Cover only - no pads included
1	CCFOOTT16	FUL FOOTBX 16W CALF CVR TAL	Cover only - no pads included
2	CCFOOTS22	FULL FOOTBOX 22W FOOT COVER	Cover only - no pads included
2	CCFOOTS20	FULL FOOTBOX 20W FOOT COVER	Cover only - no pads included
2	CCFOOTS18	FULL FOOTBOX 18W FOOT COVER	Cover only - no pads included
2	CCFOOTS16	FULL FOOTBOX 16W FOOT COVER	Cover only - no pads included
2	CCFOOTS14	FULL FOOTBOX 14W FOOT COVER	Cover only - no pads included
2	CCFOOTS12	FULL FOOTBOX 12W FOOT COVER	Cover only - no pads included
5	HDWFOOT	DUAL FOOTBOX HRDWR KIT	4 screws and washers



6 SINGLE FOOTBOX HARDWARE KIT			
ITEM #	ILLUSTRATION	DESCRIPTION	QTY
3		BHCS 1/4-20 X 3/4	2
4		9/32ID 1OD .11-.14INTHK WASHER	2

FOOT BOX WIDTH MATRIX	
Flared Frame Formula: (Seat Width) +2" = Foot Box Width	
Offset Hanger Receiver Formula: (Seat Width) +2" = Foot Box Width	
Inset Formulas:	
0" Inset (Seat Width)	= Foot Box Width
1" Inset (Seat Width) -2"	= Foot Box Width
2" Inset (Seat Width) -4"	= Foot Box Width


Pos.	Item Number	Description	Remarks
1	CCFOOTSS	SINGLE FOOTBOX CALF CVR STD	Cover only
1	CCFOOTST	SINGLE FOOTBOX CALF CVR TAL	Cover only
2	FCFOOTS	SINGLE FOOTBOX FOOT COVER	Cover only
5	132824	JAY FOOTBOARD ASM SINGLE STD	Cover, foam & board
6	HDWSFOOT	SINGLE FOOTBOX HRDWR KIT	2 screws and 2 washers

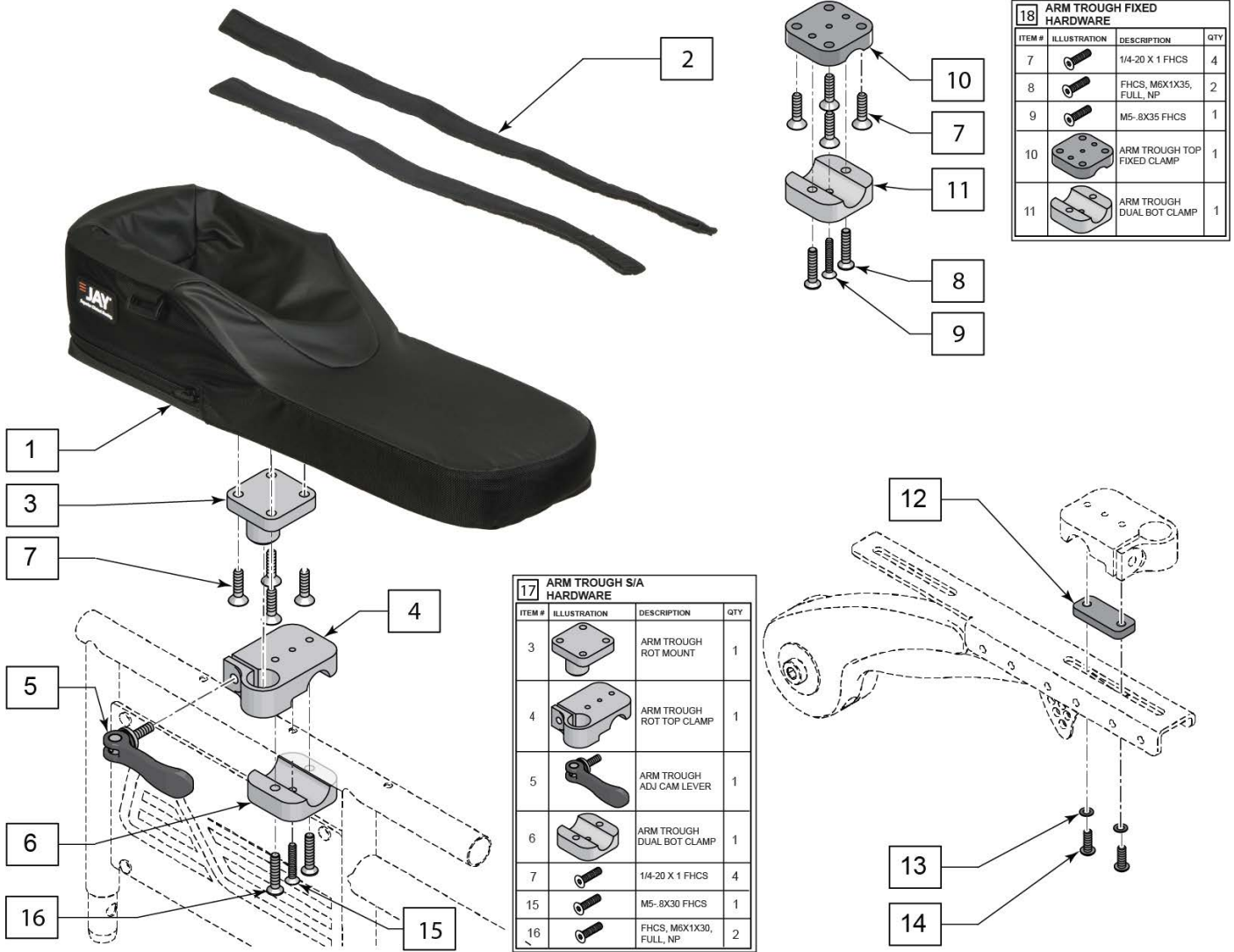

Pos.	Item Number	Description	Remarks
1	CJARM14	ARM TROUGH 14" COVER	
1	CJARM18	ARM TROUGH 18" COVER	
1	CJARM22	ARM TROUGH 22" COVER	
2	JARMSTRAP14	JAY ARMTROUGH 14IN CVR STRAP	
2	JARMSTRAP1822	JAY ARMTRGH 1822IN CVR STRAP	
5	JARMLEVER	ARM TROUGH ADJ CAM LEVER	
12	129821S	JAY ARMREST ADAPTER BRACKET	Used for Cantilever Armrests
17	SAARMHRDW	ARM TROUGH S/A HARDWARE	Dual Post Armrest, mounts to 3/4" round tubing
18	FXARMHRDW	ARM TROUGH FIXED HARDWARE	Dual Post Armrest, mounts to 3/4" round tubing
12-14	JARMADAP	ADAPTER JAY ARM TROUGH CANTI	Required to mount the S/A or Fixed Hardware to Cantilever Armrests



MEASURING GUIDE

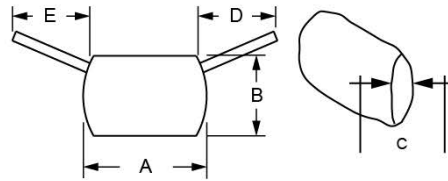

Part Number	Dim A (Width)	Dim B (Height)	Dim C (Thickness)	Dim D (Right Side Strap*)	Dim E (Left Side Strap*)
2250	15"	8.75"	2"	7"	7"
2251-MJ	11"	7"	2"	6.5"	15"

* Strap side is located as if you are resting against the back with the lumbar support on it.

Pos.	Item Number	Description	Remarks
1A	2250	LUMBAR SUPPORT SOFT COMBI	Adult
1B	2251-MJ	LUMBAR SUPPORT SFT COMBI JUNR	Junior

