


Fitting Guide Instructions: Print out CervAlign Fitting Guides on 8.5x11” paper at 100%. Cut out Fitting Guides and follow instructions on yellow-colored guide.



Top

Fitting Instructions:

1. Have the patient stand up straight.
2. Place the rounded edge of the sizing guide to the nape of the patient's neck.
3. Visualize a vertical line extending upward from the edge of the patient's shoulder blades. The flat edge of the sizing guide should land on this line.

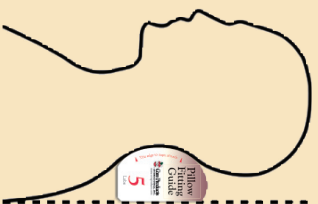

Patent # 6,471,661

- Curved design allows for shoulder comfort with mild neck extension.
- Curved cervical lobe follows the neckline, maintaining proper neck support as you roll from your side to your back.
- Large upper pillow section provides normal sleeping comfort.

Bottom