

XTRACOMFORT™



## INFLATABLE BED WEDGE PILLOW

Owner's Manual  
CSH1081GRY

## OVERVIEW

The Xtra-Comfort Inflatable Bed Wedge Pillow is designed to alleviate pressure on the body, reduce acid reflux and much more.

We are constantly answering questions and recording helpful videos to make using your Xtra-Comfort Inflatable Bed Wedge Pillow as easy as possible. Check out the included links and QR codes to help you through the process.

## WHAT'S INCLUDED


- 1x Inflation Assist Bag
- 1x Inflatable Large Wedge
- 1x Inflatable Small Wedge
- 1x Inflatable Lumbar Support
- 1x Large Wedge Cover
- 1x Small Wedge Cover
- 1x Lumbar Support Cover
- 1x Travel Carry Bag
- 1x Patch Kit

## APPLICATION

1. Connect the Inflation Assist Bag to the air valve on the Inflatable Large Wedge.



2. Open the Inflation Assist Bag wide to allow air in, move the bag up and down or lightly blow air into it. Then roll the bag down to push air into the pillow until it is fully inflated.

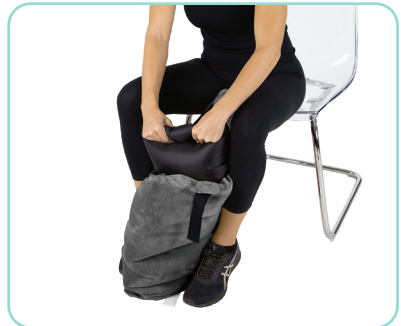

3. Once fully inflated you may adjust the level of support by pressing the deflation button on the air valve and controlling the amount of air inside the pillow.



4. Repeat steps 1 - 3 for the Inflatable Small Wedge and the Inflatable Lumbar Support.
5. Once all three pillows have been inflated, they can be assembled to make a full bed wedge or you may position them to best fit your needs. For a list of suggested positions check out our included "POSITIONS GUIDE" insert.

## REPAIR PROCEDURES

In case of leaks we've included a self patch kit that may be used to repair small holes and tears on your pillow. Use the soapy water test to determine the location of the leak and mark the area for reference.

1. Fully deflate the pillow and remove the zipper cover to expose the leak.

2. Lightly clean the surface around the puncture and make sure its dry before patching.

3. Remove the cap from the glue container, flip it over, and use the pointed end to break open the seal on the glue container.

4. Apply a thick line of glue in a circle about  $\frac{1}{4}$  to  $\frac{1}{2}$  inch away from the puncture then place the patch over the glue.



5. Use a relatively heavy item to rest on top of the glued area and let the glue dry for a minimum of 3 hours.

Note: Do not apply glue onto the puncture itself, only around, as this may cause the top and bottom layer of the pillow to glue together.

6. Re-inflate the pillow to ensure the leak has been patched before further use.

## CARE

Remove the covers, machine wash them with like-colors and mild detergent. Tumble dry on low. The inflatables can be spot cleaned with a damp cloth, dry immediately.

## WARNINGS

- Consult your doctor before use.
- Stop if pain worsens.
- If you experience any pain, swelling, sensation changes, or any unusual reaction while using this product please consult your doctor immediately.