



1. ATTACH FLAT-MOUNTS TO WHEELCHAIR BACK CANES USING EXISTING UPHOLSTERY SCREWS OR MOUNTING SCREWS WITH A MINIMUM PULLOUT STRENGTH OF 90kg (200LBS).
2. THREAD WEBBING THROUGH FLAT-MOUNT, AND THROUGH BOTH 3-BAR SLIDES.
3. THREAD WEBBING FROM OPPOSITE SIDE THROUGH FLAT-MOUNT AND BOTH 3-BAR SLIDES.
4. DOUBLE WEBBING BACK OVER AND THROUGH LAST BAR OF SLIDE.

note

description

MONOFLEX MOUNTING ON A SLING BACK WHEELCHAIR

drawing no.

D15-0930-30

date originated

9/30/2015

drawn by

D. HAGAN

rev.

A