

# Domus 2s

## Alternating Pressure Redistribution System

- Suitable for patients with low to medium risk of pressure ulcers
- One piece mattress with slope design to relieve pressure on the heel
- Medical grade PVC pad for better hygiene and safety
- An ideal transition mattress for homecare







## Domus 2s

## Alternating Pressure Redistribution System



Anti-Sliding
Extension Flap
Can be tucked
under the standard
mattress to prevent
pad from sliding



Heel Slope

Heel Slope
Allow the pad to fill in the space behind the Achilles tendon to distribute pressure over the entire lower legs, reducing the risk of heel breakdown



Adjustable Hangers Can secure control unit to fit most of beds



Weight Indicator Reference To facilitate optimized pressure setting



Nylon/PU Cover (Optional)
Vapor permeable and water resistant
with basic flame retardant level










Vapour Permeable

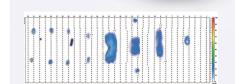

Pressure Relief During Alternating

Pressure mapping reveals that bladder A & B take turns to inflate and deflate can significantly relieve the body interface pressure

#### Specification

Pump	Dimension	9.8" x 4.9" x 3.3" / 25 x 11.2 x 9.5 cm
	Weight	2.65 lb / 1.2 Kg
	Case Material	Flame retardant ABS
	Supply Voltage	AC100-120V 60Hz, 0.1A
	Operating Cycle	10 min
Mattress		5" Overlay mattress
	Dimension	78" x 33" x 5" : 5" for 1-8 cells, 4.5" for 9-10 cells,
		4" for 11-14 cells, 3.5" for 15-16 cells
	Cell Height	16 x 5" ~ 3.5" cells
	Weight	6.83 lb / 3.1 Kg
	Material	Medical - grade PVC
	Maximum Patient Weight	260 lb / 120 Kg
	Flame Retardant Standards	CAL 117
HCPCS code		E0372

 $Specifications \ are \ subject \ to \ change \ for \ improvement \ without \ notice$ 

#### Ordering Information (For North America Only)

	Domus 2s	
System	PP02101	
Mattress	PM05301	
Pump	PP02001	

Distributed by

### Apex Medical USA

927 Mariner Street,
Brea, CA 92821, USA
T +1 714 671 3818
F +1 714 494 8173
sales.usa@apexmedicalcorp.com
www.apexmedicalcorp.com
Print-2019/05-500 All rights reserved